

Discmonitor

Discography System

Interventional Spine

Discmonitor Set-up

- Fill the reservoir by pulling back on the plunger
- Prime the Discmonitor by pushing the plunger handle
- Connect tubing to the spinal needle



- Advance the plunger engagement switch to **LOCKED**
(Activates Volume Control)

Procedure Steps

ON • RCL

- Press the “ON • RCL” button to turn on the Discmonitor



- Under live fluoro, inject contrast by rotating the plunger clockwise



- At the first hint of contrast in the disc, if desired, press the “Volume Reset” button and/or “Pressure Reset” button
(Also Saves Opening Pressure)




- Save up to 3 key points for each disc by pressing the “Save” button each time




- Disconnect from the spinal needle and hold the “Next Disc” button for 2 seconds

Screen

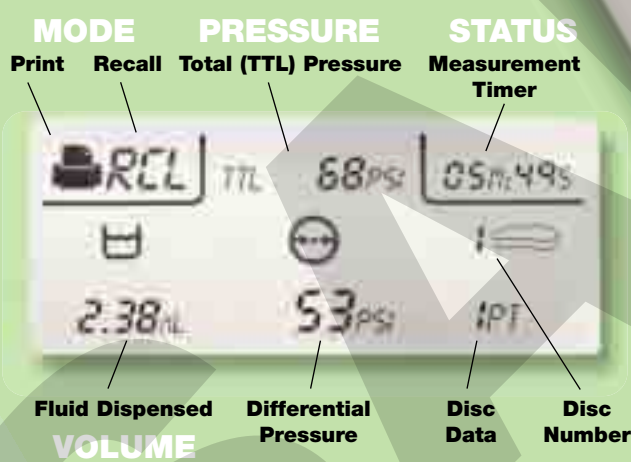


Shown actual size

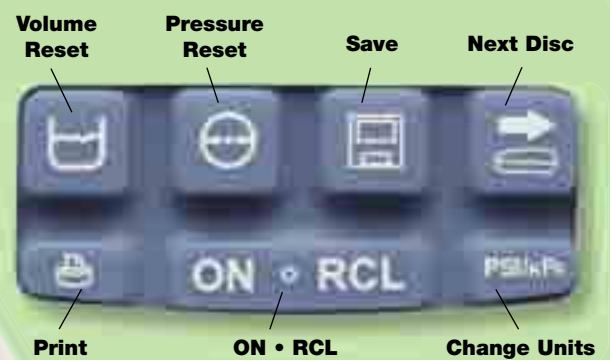

Keypad

Printing Directions

- Point the Discmonitor IR transmitter at the printer receiver
- Press and hold the “Print” button until the LED stops flashing

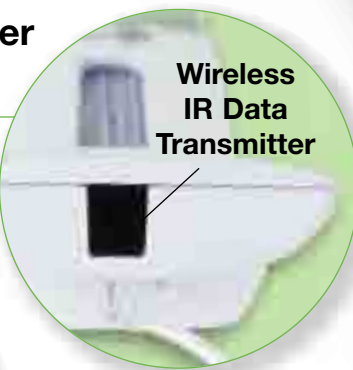

LOCKED
(Plunger threads are engaged)




UNLOCKED
(Plunger threads are disengaged)

Plunger

Plunger Engagement Switch

Plunger Handle

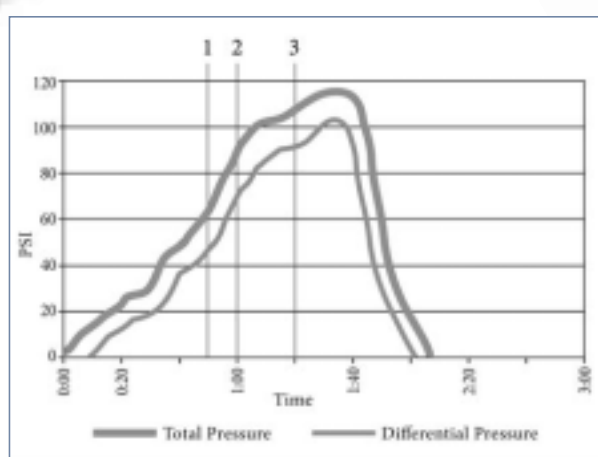

Pts.	Time (mls)	TTL Press. (PSI)	Diff. Press. (PSI)	Volume (mL)
1	00:50	60	45	01.25
2	01:00	90	75	01.82
3	01:20	105	90	02.05
P-Ø	00:20	15	00	00.30
P max	01:40	117	102	02.15
V max	01:40	117	102	02.15

*Recorded information per disc