

## Using the Pain Pump®2

The Stryker PainPump®2 can be used either in the hospital or at home. The PainPump®2 comes with either an attachment strap or carrying case to make everyday activity convenient.

While you are using the PainPump®2:

- Make sure there are no leaks from the system.
- Make sure the tubing is not kinked.
- A constant stream of teardrops will appear when the pump is active and delivering medication.
- If the pump should happen to fall and break, stop the medication flow by pinching off the tube with the clamp.
- The PainPump®2 should not get wet. Place it outside the shower or place it in a plastic bag.

If at any time you experience problems with the PainPump®, close the clamp on the tubing and contact your doctor or Patient Hotline at 1-877-848-5756.

At the conclusion of the treatment, the pump will be removed and discarded.