

# Bari10A

## Bariatric Bed

### Large capacity patient care


A practical solution for better patient care and comfort for all patients. Bari10A can be used as a standard hospital bed, but can easily expand to accommodate your heaviest patients. Easy-to-use expansion and retraction with color-coded levers expands 6 inches on each side. Centrally located fifth wheel improves maneuverability and steering.



Easy-to-use expansion and retraction with color-coded levers.



Made with heavy-gage, tubular steel, the optional bariatric patient helper adds to patient dignity by allowing independent patient repositioning.

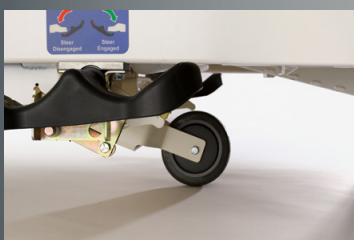

A center-mounted fifth wheel provides an additional pivot point to turn the bed in a tighter arc. The fifth wheel retracts to allow unobstructed space for equipment furniture.

**Overall aesthetic** that allows the patient the dignity of a standard hospital bed look.

**BackSmart expansion** with color coded levers allow the bed to easily expand and retract to accommodate patients of all sizes.

**Centrally-located fifth wheel** adds an additional pivot point, improving maneuverability and steering.

**Patient helper** provides a BackSmart solution for patient repositioning.

**True 1,050 pound safe working load.**

# Bari10A

## Bariatric Bed



### Standard features

- 1050 lb. safe working load
- Foot end controls with patient lockouts
- Fully articulated mattress support platform with independent back and knee movement, up/down adjustments, Trendelenburg and Reverse Trendelenburg positioning
- 36" wide mattress support platform sections easily expand to 48" wide
- 80" (expandable to 88") mattress support platform. Integrated foot end extension section extends an extra 8" length for taller residents/patients
- 18.75" to 33.75" height
- Integrated scale system for resident/patient weight monitoring up to 1050 lbs
- Head and foot end siderails/assist devices
- One-handed siderail release
- Siderails on head and foot sections fold down for easy patient access
- Pendant control for all bed functions with separate selective lock-out to override functions
- Emergency manual release for mattress support platform
- Quiet DC motors
- Welded steel tubular frame
- Pan-style mattress support platform
- Wide stable base
- 5" casters
- Bed angle indicators
- Soft-tone frame color
- Head of bed indicator
- Foley bag hooks
- IV traction adapters
- Pump holder

### Specifications

<b>Model number</b>	2230
<b>Overall length</b>	
Minimum	90" (228.6 cm)
Maximum	98" (248.9 cm)
<b>Overall width</b>	40" (101.6 cm)
Minimum	39" (99.1 cm)
Maximum	52" (132.1 cm)
<b>Safe working load</b>	1050 lb. (476.3 kg)
<b>Bed frame weight</b>	700 lb. (317.5 kg)
<b>Standard height range</b>	
High	33.7" (85.6 cm)
Low	18.7" (47.5 cm)
<b>Litter positioning</b>	
Maximum back angle to horizontal	65°
Maximum knee angle to horizontal	35°
Maximum knee to leg angle	45°
Trend	14.5°
Reverse Trend.	15.3°
<b>Patient surface</b>	
Minimum	36" x 80" (91.4 x 203.2 cm)
Maximum	48" x 88" (121.9 x 223.5 cm)
<b>Caster diameter</b>	5" (12.7 cm)

### Optional features

- Foam surface
- BariMatt 1000
- BariMatt 1000 Plus (includes Bolsters)
- Arise 1000EX
- IV pole/dual IV pole
- Floor mat
- Patient helper
- Fifth wheel

Stryker reserves the right to change specifications without notice.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holder.

6180148 RevA • 15-3017

### Electronics

<b>Agency approvals</b>	c-us UL, UL60601-1, IEC 60601-1-2, CAN/CSA C22.2 No. 601.1IEC 60601-2-52, CAN/CSA C22.2 No. 601.2.52
<b>Volts</b>	115 VAC
<b>Ampere rating</b>	5.8 amps
<b>Frequency</b>	60 Hz

3800 E. Centre Avenue  
Portage, MI 49002 USA  
t: 269 329 2100  
toll free: 800 327 0770

[patientcare.stryker.com](http://patientcare.stryker.com)

Copyright © 2017 Stryker  
Mkt Lit-198-20 Jul 2007 Rev F2  
Printed in U.S.A