

Customer guidance:

Cleaning Stryker's Emergency Care products

Many customers have reached out with questions regarding the type of cleaning agents that can be used on their Stryker devices.

The following guidance can be used for cleaning the products listed below. This information can be referenced in the Operations Manual (OM)/Operating Instructions (OI)/Instructions for Use (IFU) for each product. The OM/OI/IFU remains the sole source for product information.

In addition to the cleaning agents listed below from the Stryker OM/OI/IFU, the Environmental Protection Agency (EPA) has released a list of disinfectants suitable for COVID-19, some of which may be used on Stryker products, providing that the OM/OI/IFU do not say otherwise. The list is updated regularly and is available here:

<https://www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2>. If a product or cleaning method is not listed in the applicable OM/OI/IFU, Stryker has not tested it and its use may damage the product.

Product(s)	Suggested cleaners
LIFEPAK® monitor/defibrillators LIFEPAK AEDs	<ul style="list-style-type: none"> • Quaternary ammonium compounds • Isopropyl alcohol • Peracetic (peroxide) acid solutions <p>DO NOT clean any part of this device or its accessories with bleach, bleach dilution, or phenolic compounds. Do not use abrasive or flammable cleaning agents.</p>
HeartSine® samaritan® PAD	<ul style="list-style-type: none"> • Soapy water • Isopropyl alcohol (70% solution) <p>DO NOT clean the samaritan PAD with abrasive materials, cleaners or solvents.</p>
LUCAS® chest compression system	<ul style="list-style-type: none"> • 70% isopropyl alcohol solution • 45% isopropyl alcohol solution with added detergent • Quaternary ammonium compound • 10% bleach • Peracetic (peroxide) acid solutions
Power-PRO™ powered ambulance cot Performance-PRO™ manual ambulance cot	<ul style="list-style-type: none"> • Quaternary cleaners (active ingredient – ammonium chloride) that contain less than 3% glycol ether • Phenolic cleaners (active ingredient – o-phenylphenol) • Chlorinated bleach solution (5.25% – less than 1 part bleach to 100 parts water) • ≤ 21% isopropanol alcohol <p>DO NOT USE Virex® TB or Iodophor type disinfectants</p>

Medical

MX-PRO® R3 manual ambulance cot MX-PRO® bariatric transport cot Stair-PRO® stair chair	Unless otherwise noted below, use concentrations recommended by the manufacturer: <ul style="list-style-type: none"> • Quaternary cleaners (active ingredient – ammonium chloride) • Phenolic cleaners (active ingredient - o-phenylphenol) • Chlorinated bleach solution (5.25% - less than 1 part bleach to 100 parts water) <p>DO NOT USE Virex® TB or Iodophor type disinfectants</p>
Altrix® temperature management system	Consult the product’s Operations Manual and Disinfection Guide at http://techweb.stryker.com/Temp_Management/index.html
McGRATH® MAC EMS video laryngoscope	Consult the product’s Instructions for Use directly at https://www.strykeremergencycare.com/disclosure-and-safety-information/

If you have any questions about this matter, contact Stryker at 1-800-787-9537, option 2, 8:00 A.M. to 6:00 P.M. (Eastern time), Monday – Friday.

Resources

Links to Operating Instructions/Instructions for Use for all products:

- LIFEPAK Defibrillators, AEDs, CPR devices, video laryngoscope and accessories: <https://www.strykeremergencycare.com/disclosure-and-safety-information/>
- HeartSine samaritan PADs: <https://heartsine.com/brief-summary-indications-important-safety-information/>
- Ambulance cots and accessories: <https://techweb.stryker.com/EMS/index.html>
- Temperature management system: http://techweb.stryker.com/Temp_Management/index.html

List N: Products with Emerging Viral Pathogens AND Human Coronavirus claims for use against SARS-CoV-2: <https://www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2>

Copyright © 2020 Stryker. 03/2020. GDR 3343695_B

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: Altrix, HeartSine, LIFEPAK, LUCAS, MX-PRO, Performance-PRO, Power-PRO, Stair-PRO, Stryker. All other trademarks are trademarks of their respective owners or holders.

The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker’s trademark or other intellectual property rights concerning that name or logo.

Medical

3800 E. Centre Avenue, Portage, MI 49002 USA | P +1 269 329 2100 | Toll-free +1 800 327 0770 | stryker.com