

stryker

Driving
performance,

**delivering
versatility**

Spetzler™-Malis® and SILVERGlide® Non-Stick Bipolar Forceps



Your **superior experience** starts with our people

**Supporting you
throughout your
procedures**

People

Our people stand out. They're dedicated, highly skilled experts that care deeply and listen carefully. When you need a partner in improving care, we're by your side.

Culture

We create a dynamic, supportive environment that enables amazing people to do amazing things. Count on us to do what's right and deliver—every time.

Product innovation

Innovation is in our DNA—from Dr. Homer Stryker to the researchers, engineers and surgeons creating solutions that are reshaping neurotechnology today.

Medical education

Innovation and skill come together in our educational outreach to providers. Each year, we help surgical teams around the world master the latest medical technologies and care practices.

Research and development


We amplify our investments in R&D by listening to our customers and collaborating with them to create solutions that are simply superior.

Outcomes


We share your values and your commitment to innovative care that heals effectively and efficiently. With you, we're driven to make healthcare better.

All of our non-stick bipolar forceps **resist tissue buildup** and provide **excellent visualization**.

Regardless of your preference for single-use Spetzler-Malis or reusable SILVERGlide, you can expect **precision, control** and a responsive extension of your hands and eyes from us.

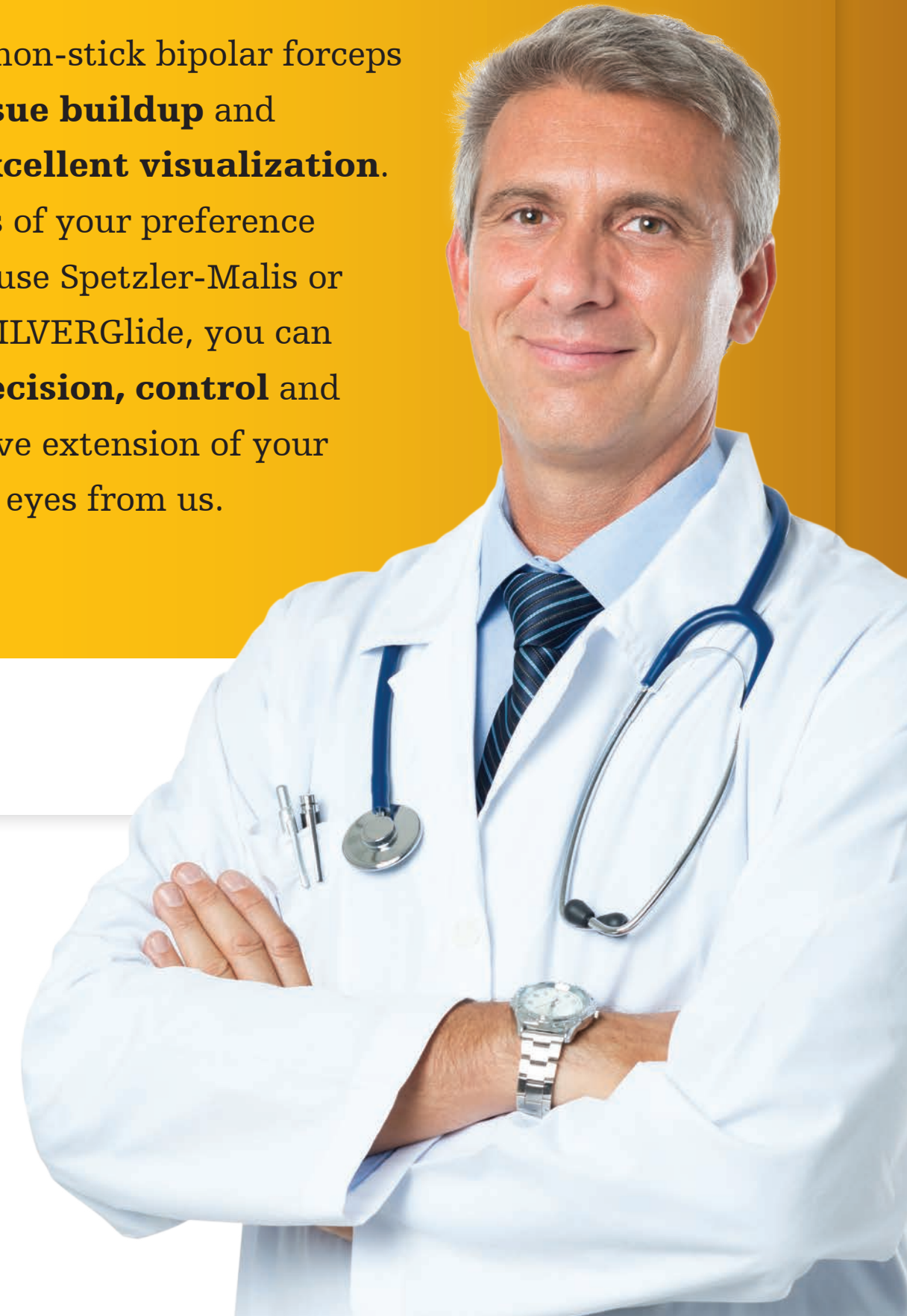

Disposable Spetzler-Malis **Standard** Bipolar Forceps

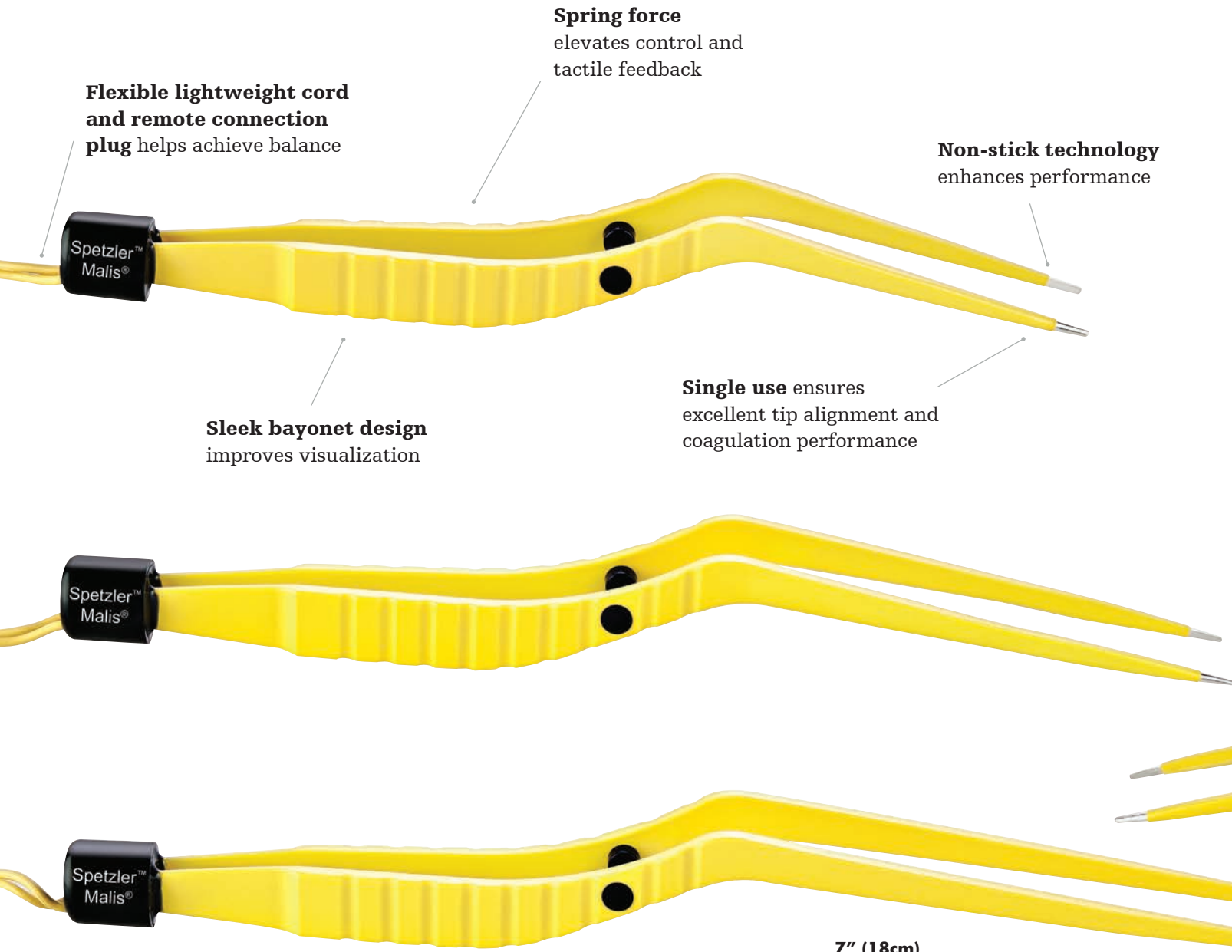

7" (18cm)

- 6760-180-015 1.5mm tip
- 6760-180-010 1.0mm tip
- 6760-180-005 0.5mm tip


8" (20cm)

- 6760-200-015 1.5mm tip
- 6760-200-010 1.0mm tip
- 6760-200-005 0.5mm tip

9" (23cm)

- 6760-230-015 1.5mm tip
- 6760-230-010 1.0mm tip
- 6760-230-005 0.5mm tip

Disposable Spetzler-Malis **Dual Irrigating** Bipolar Forceps

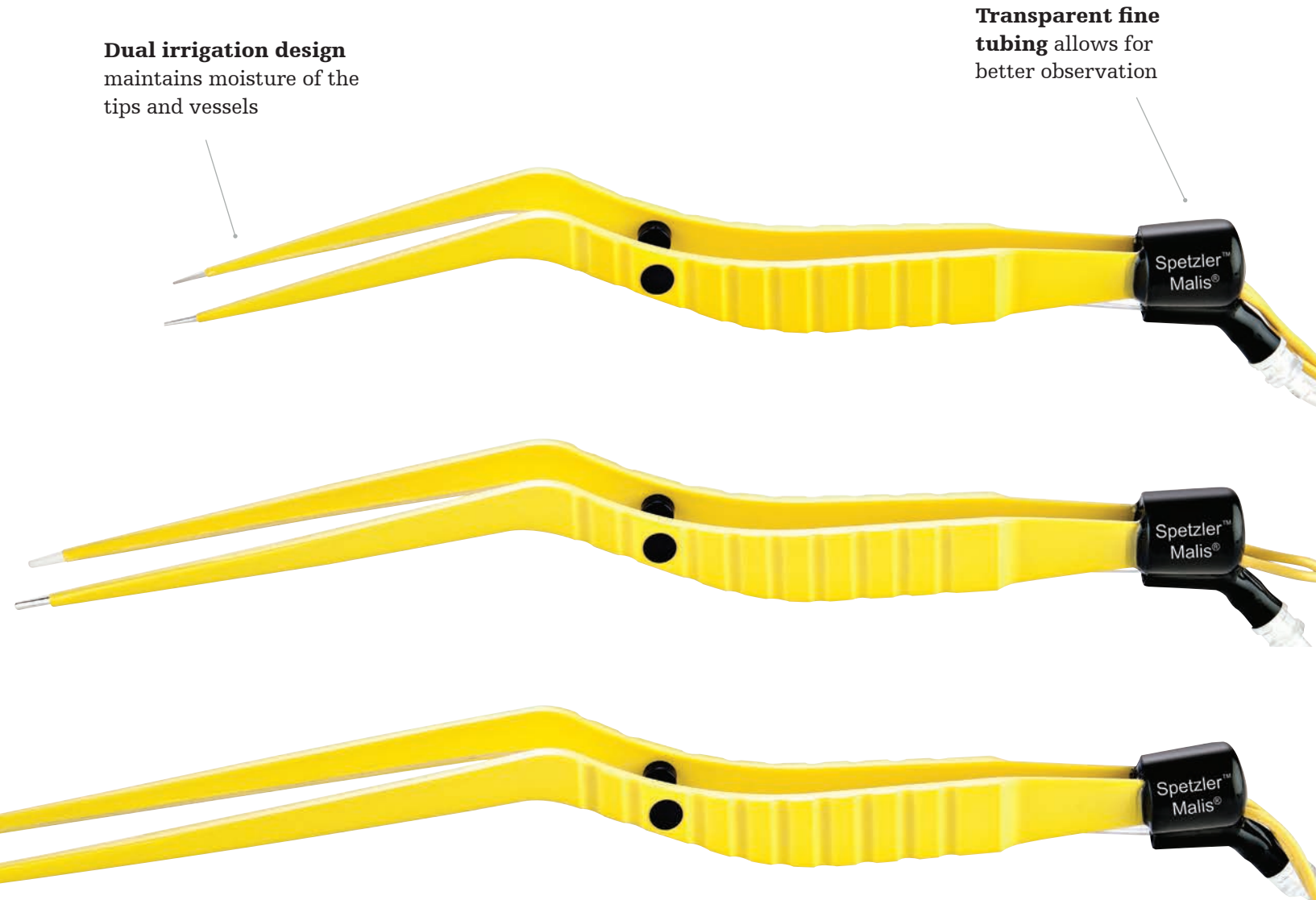

7" (18cm)

- 6770-180-015 1.5mm tip
- 6770-180-010 1.0mm tip
- 6770-180-005 0.5mm tip

8" (20cm)

- 6770-200-015 1.5mm tip
- 6770-200-010 1.0mm tip
- 6770-200-005 0.5mm tip

9" (23cm)

- 6770-230-015 1.5mm tip
- 6770-230-010 1.0mm tip
- 6770-230-005 0.5mm tip

Disposable Spetzler-Malis **Slim** Bipolar Forceps



8" (20cm)

- 6780-200-015 1.5mm tip
- 6780-200-010 1.0mm tip
- 6780-200-005 0.5mm tip

9" (23cm)

- 6780-230-015 1.5mm tip
- 6780-230-010 1.0mm tip
- 6780-230-005 0.5mm tip

Slim profile provides access in narrow fields



Disposable Spetzler-Malis Bipolar Forceps cord and irrigation sets

Rotary cord and tubing sets

- 6790-100-003 Bipolar irrigator integrated tubing and cord set, disposable
- 6790-100-004 Bipolar irrigator integrated tubing and cord set, disposable, unitized plug


Gravity style cord and tubing sets

- 6790-100-001 Integrated tubing and bipolar cord set, disposable
- 6790-100-002 Integrated tubing and bipolar cord set, disposable, unitized plug

Cord only sets

- 6790-100-005 Bipolar cord set, disposable
- 6790-100-006 Bipolar cord set, disposable, unitized plug

Disposable Spetzler-Malis Bipolar Forceps offer fresh advantages from every new use

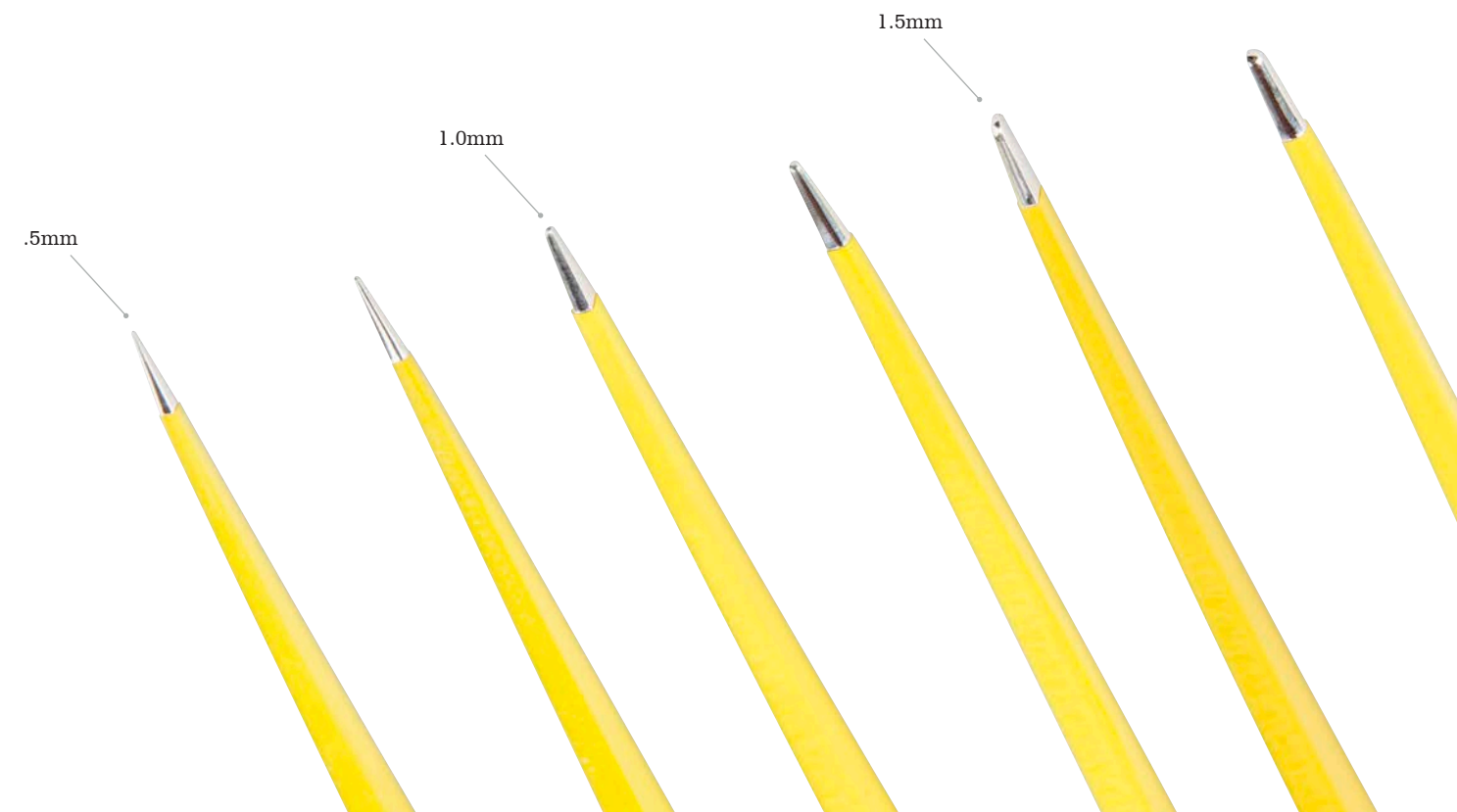

SILVERGlide Non-Stick ProSeries Bipolar Forceps

5.7" (14.5cm) bayonet

6730-145-015 1.5mm
 6730-145-012 1.2mm
 6730-145-010 1.0mm
 6730-145-007 0.7mm
 6730-145-004 0.4mm

6.3" (16cm) bayonet

6730-160-012 1.2mm
 6730-160-010 1.0mm
 6730-160-007 0.7mm
 6730-160-004 0.4mm
 6730-160-504, 45° up 0.4mm

7.5" (19cm) bayonet

6730-190-020 2.0mm
 6730-190-015 1.5mm
 6730-190-012 1.2mm
 6730-190-010 1.0mm
 6730-190-007 0.7mm
 6730-190-004 0.4mm
 6730-190-512, 45° up 1.2mm
 6730-190-612, 45° down 1.2mm
 6730-190-504, 45° up 0.4mm

8.6" (22cm) bayonet

6730-220-020 2.0mm
 6730-220-015 1.5mm
 6730-220-012 1.2mm
 6730-220-007 0.7mm
 6730-220-004 0.4mm
 6730-220-512, 45° up 1.2mm
 6730-220-612, 45° down 1.2mm
 6730-220-507, 45° up 0.7mm
 6730-220-607, 45° down 0.7mm
 6730-220-307, 30° up 0.7mm
 6730-220-107, 15° up 0.7mm

9.4" (24cm) bayonet


6730-240-020 2.0mm
 6730-240-015 1.5mm
 6730-240-012 1.2mm
 6730-240-007 0.7mm
 6730-240-004 0.4mm
 6730-240-807, 80° up 0.7mm
 6730-240-907, 80° down 0.7mm
 6730-240-507, 45° up 0.7mm
 6730-240-607, 45° down 0.7mm

11" (28cm) bayonet

6730-280-015 1.5mm
 6730-280-012 1.2mm

7.5" (19cm) straight bayonet

6735-190-010 1.0mm
 6735-190-007 0.7mm
 6735-190-320, 30° up 2.0mm
 6735-190-310, 30° up 1.0mm

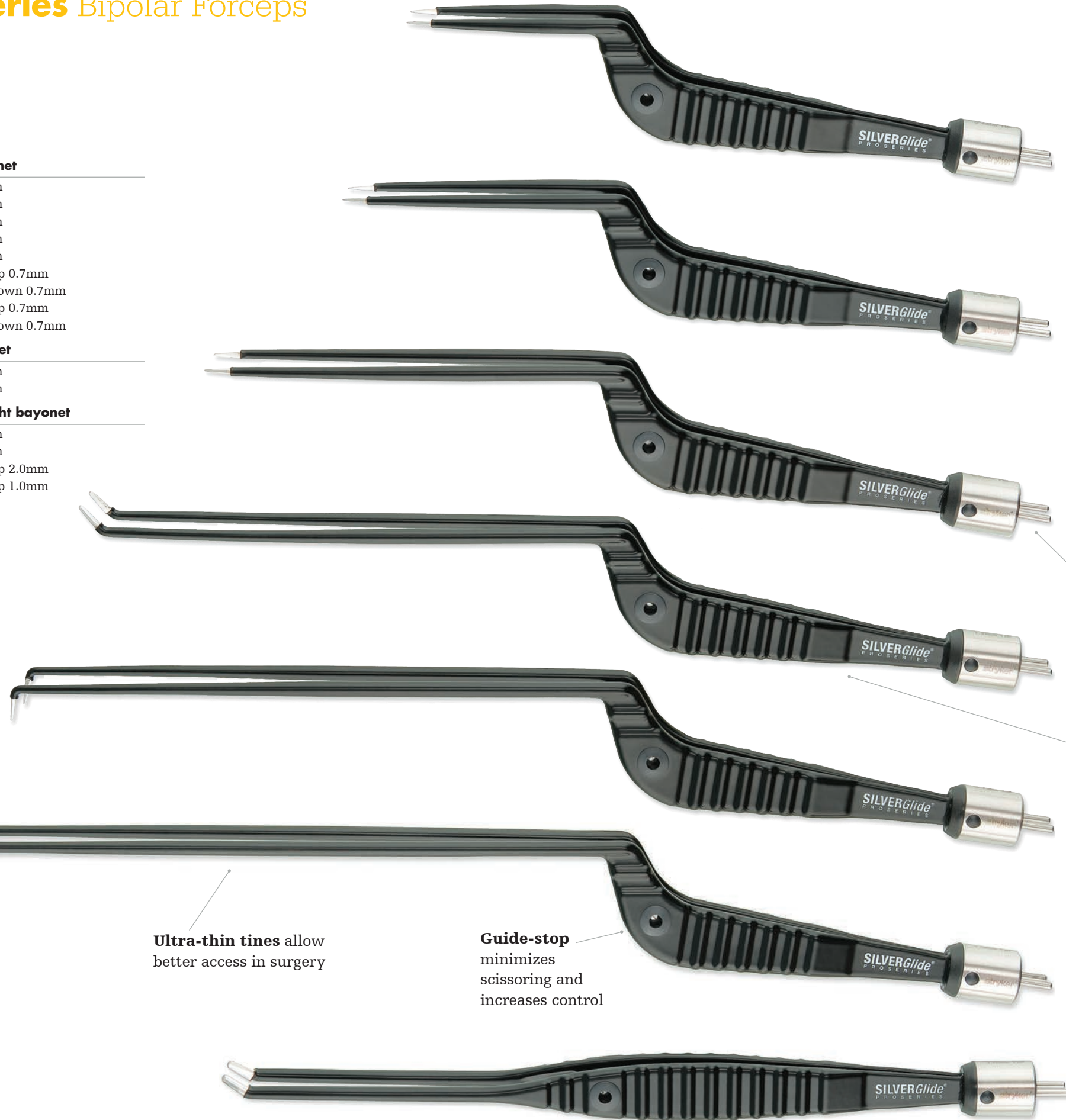

Elevating visibility through a slender design

Standard two-pin round connector works with most standard foot-switching bipolar electro-surgical generators

Lightweight handle for easy manipulation and balance

Silver plating reduces collateral heating, "hot spots" and sticking for efficient performance

Ultra-thin tines allow better access in surgery

Guide-stop minimizes scissoring and increases control

SILVERGlide Non-Stick Keyhole Maximum Access Bipolar Forceps

5.5" (14cm) keyhole bayonet

6750-140-007 0.7mm
6750-140-004 0.4mm

6.7" (17cm) keyhole bayonet

6750-170-012 1.2mm
6750-170-007 0.7mm
6750-170-004 0.4mm

7.8" (20cm) keyhole bayonet

6750-200-015 1.5mm
6750-200-012 1.2mm
6750-200-007 0.7mm
6750-200-004 0.4mm

9.2" (23cm) keyhole bayonet

6750-230-015 1.5mm
6750-230-012 1.2mm
6750-230-007 0.7mm
6750-230-307, 30° up 0.7mm
6750-230-107, 15° up 0.7mm

10.2" (25cm) keyhole bayonet

6750-250-015 1.5mm
6750-250-012 1.2mm
6750-250-007 0.7mm
6750-250-807, 80° up 0.7mm
6750-250-907, 80° down 0.7mm
6750-250-507, 45° up 0.7mm
6750-250-607, 45° down 0.7mm

11.7" (28cm) keyhole bayonet

6750-280-015 1.5mm
6750-280-012 1.2mm



SILVERGlide Bipolar Forceps can be sterilized using standard steam autoclave cycles without disassembly

SILVERGlide Non-Stick **Precision** Bipolar Forceps



6.1" (15.5cm) SP bayonet

6720-155-015 1.5mm
6720-155-012 1.2mm
6720-155-007 0.7mm
6720-155-004 0.4mm

7.0" (18cm) SP straight

6725-180-010 1.0mm
6725-180-007 0.7mm
6725-180-420, 30° up 2.0mm
6725-180-410, 30° up 1.0mm

7.5" (19cm) SP bayonet

6720-190-020 2.0mm
6720-190-015 1.5mm
6720-190-010 1.0mm
6720-190-007 0.7mm
6720-190-004 0.4mm
6720-190-512, 45° up 1.2mm
6720-190-612, 45° down 1.2mm

8.7" (22cm) SP bayonet

6720-220-020 2.0mm
6720-220-015 1.5mm
6720-220-012 1.2mm
6720-220-007 0.7mm
6720-220-004 0.4mm

9.7" (24cm) SP bayonet

6720-240-020 2.0mm
6720-240-015 1.5mm
6720-240-007 0.7mm
6720-240-004 0.4mm



SILVERGlide Non-Stick **Standard** Bipolar Forceps



4.0" (10cm) jeweler

6715-100-007 0.7mm

6715-100-004 0.4mm

5.0" (12.5cm) adson

6715-125-012 1.2mm

6715-125-007 0.7mm

6715-125-004 0.4mm

6.0" (15cm) semkin

6715-150-012

7.5" (19cm) cushion

6710-190-020 2.0mm

6710-190-015 1.5mm

6710-190-012 1.2mm

6710-190-010 1.0mm

6710-190-007 0.7mm

6710-190-004 0.4mm

7.8" (20cm) scoville-greenwood

6710-200-015 1.5mm

8.3" (21 cm) cushion

6710-210-020 2.0mm

6710-210-015 1.5mm

6710-210-012 1.2mm

6710-210-007 0.7mm



SILVERGlide Non-Stick Irrigating Bipolar Forceps



Separate fluid and electrical connectors accept a variety of cord-tubing sets

Hollow tine design brings irrigation to the tip of the forceps



7.5" (19cm) bayonet

6740-190-012 1.2mm
6740-190-007 0.7mm

8.7" (22cm) bayonet

6740-220-012 1.2mm
6740-220-007 0.7mm

10.2" (25cm) bayonet

6740-250-012 1.2mm
6740-250-007 0.7mm

What sets SILVERGlide apart?

The elimination of tissue buildup allows you to work without interruption. These non-stick characteristics continue to perform use after use. The SILVERGlide alloy's level of thermal diffusivity is 37 times higher than that of stainless steel commonly used in forceps.

	Standard forceps	SILVERGlide Non-Stick	
"Hot spots" from sparks occur during coagulation			"Hot spots" start to occur during coagulation
The "hot spots" cause localized sticking			Heat is diffused rapidly to prevent sticking
Tissue seared to the forceps tips is ripped away			The forceps can be removed without tissue sticking


SILVERGlide Bipolar Forceps accessories

SILVERGlide Bipolar Forceps sterilization trays


6700-002-000 Two forceps sterilization tray
6700-005-000 Five forceps sterilization tray
6700-010-000 10 forceps sterilization tray (forceps not included)

Other SILVERGlide accessories

6700-000-002 Polishing cloths
6700-000-001 Tip guards
6702-000-000 Disposable cable
6701-000-000 Reusable cable

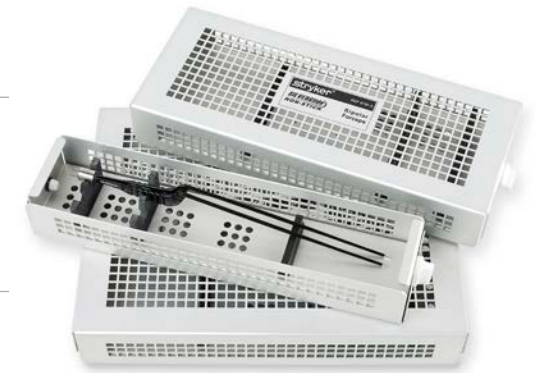

Malis Bipolar Generator and Irrigator system

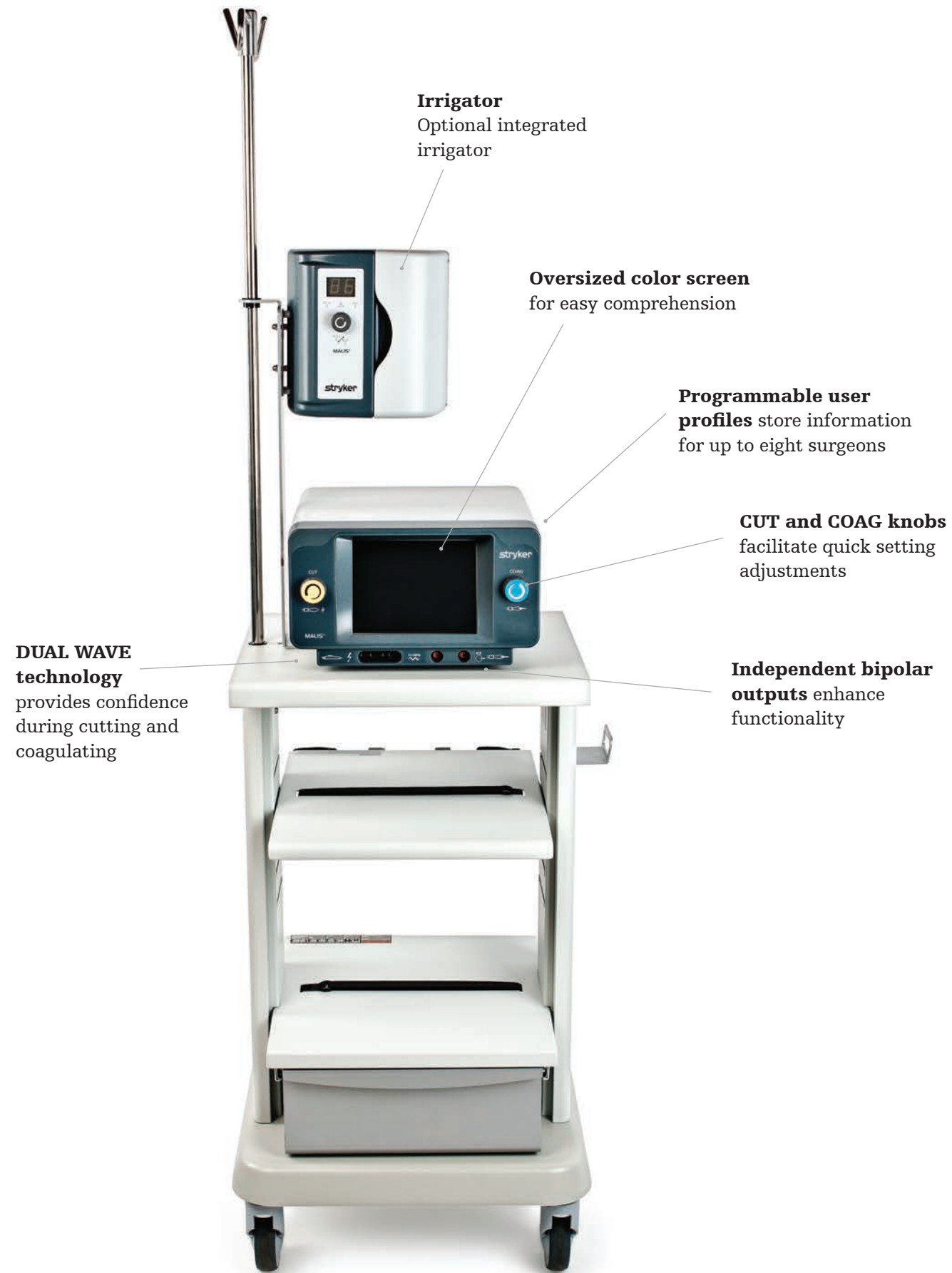

Malis Bipolar Generator and Irrigator system

- 6700-313-500 Malis Bipolar Generator (includes foot pedal and power cord)
- 6700-313-600 Malis Bipolar Irrigator (does not include power cord)
- 6700-100-001 Irrigator power cord
- 6700-313-004 Bipolar generator/irrigator interconnect cable
- 6700-313-002 Bipolar generator cut/coag dual foot pedal
- 6700-313-700 Bipolar generator cart

Other accessories

- 6700-313-003 Single foot pedal
- 6700-313-304 Single foot pedal adapter cable



Whether you choose reusable or disposable, our non-stick bipolar forceps act as a specialized extension of your hands and eyes.

For more information, contact your Sales Representative or call Customer Service at 800 253 3210.

strykerspetzlermalis.com

Neuro, Spine, ENT

This document is intended solely for the use of healthcare professionals. A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate a Stryker product. A surgeon must always refer to the package insert, product label and/or instructions for use, including the instructions for cleaning and sterilization (if applicable), before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Malis, SILVERGlide, Spetzler-Malis and Stryker. All other trademarks are trademarks of their respective owners or holders.

Literature Number: 9100-004-379 Rev. A

Copyright © 2016 Stryker

Printed in USA

Stryker Neuro, Spine, ENT
4100 East Milham Avenue
Kalamazoo, MI 49001 USA
t: 269 323 7700 f: 800 999 3811
toll free: 800 253 3210

stryker.com