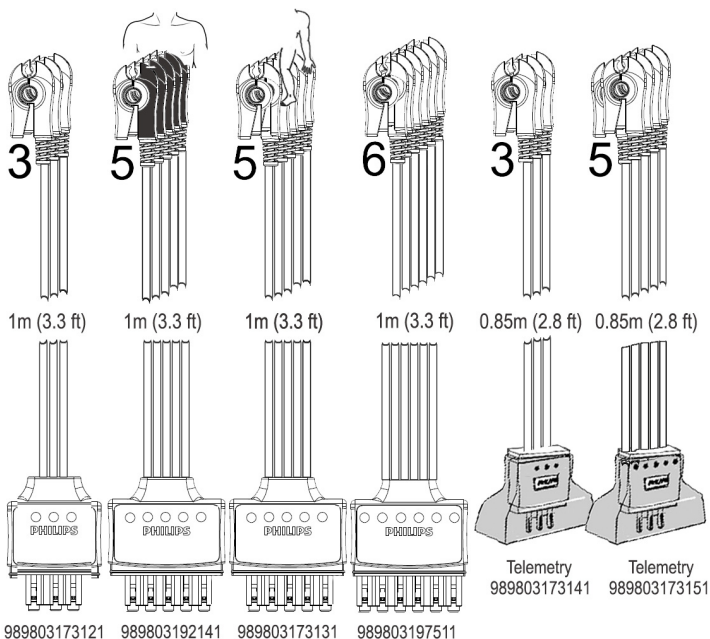
Single-Patient-Use ECG 3, 5, and 6-Leads AAMI

Instructions for Use

PHILIPS

Contents

(EN) Instructions for Use.....	1
(FR) Manuel d'utilisation	3
(ZH-TW) 使用說明	5

R_x only

Product Overview

Intended Use

Philips “Single-Patient-Use Disposable ECG Leadsets” are intended for use only by trained healthcare professionals for measurement of a patient’s ECG for both diagnostic and monitoring purposes. Use is limited by the indications for use of the connected monitoring or diagnostic equipment. These Philips leadsets are intended for short-term use only (an average patient stay of 5 days).

Description

- Philips Single-Patient-Use ECG leadsets for bedside monitoring are available in 3-lead, 5-lead, and 6-lead configurations. The 5-lead limb and 5-lead chest leadsets can be used simultaneously to measure limb and chest ECG.
- Disposable telemetry leads include a shower shield and are available in 3-lead and 5-lead versions.
- Disposable leads terminate with a color-coded and labeled grabber, and have a peel-apart ribbon-style cable.

Compatibility

These disposable leads are compatible with the Philips M8105A (IntelliVue MP5) monitor plus many validated Philips monitors or defibrillators (such as IntelliVue MP monitors, IntelliVue MX monitors, ECG and MMS Modules, VM monitors, HeartStart defibrillators) when used with an IntelliVue trunk cable. IntelliVue ECG lead sets and trunk cables can be used with an ECG monitor or defibrillator for which they are listed as accessories in that product’s IFU. Telemetry leads can be used with Philips patient-worn devices with an IntelliVue-style connection. Other leads and monitor combinations are not recommended.

Leadset Disposal

Discard leadsets after use on one patient, or immediately discard if visual inspection shows damage or deterioration. Follow approved medical waste disposal methods as specified by your patient care facility or local regulations.

Cautions

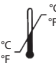
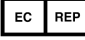
- Inspect leadsets prior to use. Do not use if visual inspection reveals damage from liquid, lint, or other contaminants.
- Ensure that leadset is fully and properly inserted into trunk cable or instrument.
- Do NOT use leadset in excessively wet environments or under massive influence of fluids (e.g. rain).

Warnings

- Refer to the IFU for the monitor/defibrillator for additional warnings and cautions.
- Refer to the IFU for the monitor/defibrillator for information regarding proper lead/electrode placement that complies with standard AAMI or IEC practices.
- For single-patient-use only. Do not use on multiple patients. Multi-patient use may result in patient infection or cross-infection.
- Ensure that patient is properly grounded during electro-surgical (ESU) or defibrillation procedures.
- Philips disposable leads can be used in the operating room only if used with the Philips OR trunk cable.
- Do not use ECG leads and cables in magnetic resonance imaging (MRI) environments or during MRI procedures.
- Ensure that leadsets are carefully positioned to avoid entanglement, choking, and strangulation.
- For telemetry applications, be sure to use a telemetry leadset that has the included protective shield over the trunk connector.

Definition of Product Symbols

The following are definitions for symbols that might appear on package labels as well as the products described in this document:

 <p>Catalog Reference No.</p>	 <p>Batch Code</p>	 <p>Not Manufactured with Natural Rubber Latex</p>	 <p>Temperature Limitation</p>	 <p>Atmospheric Pressure Limits</p>	 <p>Humidity Limits</p>	 <p>Authorized Representative in the European Community</p>
---	---	---	---	--	--	--

<p>IP23</p> <p>Protection against ingress of objects ≥ 12.5mm diameter and spraying water</p>	 <p>Consult Instructions for Use</p>	<p>R_x only</p> <p>Prescription Use Only</p>	 <p>Manufacturer</p>	 <p>Quantity per Box</p>	 <p>Quantity per Pouch</p>
--	---	---	---	---	---

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Copyright © 2016
Böblingen GmbH