For Use With EN

A Stryker Flyte Power Pack provides electrical power to the Flyte Helms (REF 0408-600-000, 0408-635-000, 0408-645-000, 17 Helmet REF 0416-600-000, T7 Helmet with LED Headlight REF 0416-610-000, and the Surora 700 Headlight REF 5450-650-000.

The Flyte Power Pack is charged in the Flyte Chargers REF 0408-655-000, 0408-655-001, and 0408-655-002.

User/Patient Safety

- WARNINGS:**
- Upon initial receipt and before each use, inspect each component for damage. DO NOT use any equipment if damage is apparent.
- ALWAYS clean the power pack before first and every use.
- DO NOT sterilize or immerse.
- DO NOT use a power pack that gives off an odor, generates heat, becomes discolored, or leaks.
- DO NOT use this equipment in areas in which flammable anesthetics or flammable agents are mixed with air, oxygen or nitrous oxide.
- DO NOT short circuit, crush, incinerate or disassemble. Damage increases the risk of chemical burns from electrolyte leakage, fire or explosion.
- DO NOT service this equipment.

NOTES:

For additional information, especially safety information, or in-service training, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

The user and/or patient should report any serious product-related incident to both the manufacturer and the Competent Authority of the European Member State where the user and/or patient is established.

Instructions

To clean, inspect, or charge a power pack, see the instructions for use supplied with the Stryker Flyte Charger.

To use a power pack, see the instructions for use supplied with the Flyte Helmet or the Surora 700 Headlight.

Recycle/Disposal

- WARNINGS:**
- ALWAYS follow the current local recommendations and/or regulations governing environmental protection and the risks associated with recycling or disposing of the equipment at the end of its useful life.
- ALWAYS decontaminate a battery pack exposed to infectious material before sending it to a waste treatment facility.

To comply with Batteries Directive 2006/66/EC, the battery pack has been designed to be recycled at the end-of-life by the end user as a complete unit. No disassembly is required.

Specifications

NOTE: Specifications are approximate.

Power Pack:	REF 0408-660-000
Electrical Output:	7,26 V ===
Energy:	38 Wh
Duración:	
Flyte Helms:	
Fan Only:	4 horas
Fan and Headlight:	2 horas
Surora 700 Headlight:	8 horas

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