

<p>NOTE: This portion of the document should not appear on produced Labels or IFUs.</p> <p>Stryker Instruments (269) 323-7700 (800) 253-3210</p>	<p><u>Dimensions:</u> 3.75 inch (width) x 8 inch Booklet</p>	<p><u>Color/Material/Finish:</u> Color Graphics on White Background 20# Bond or Equivalent</p>	<p><u>Label Stock:</u> N/A</p>	
	<p><u>Print Location:</u> Print Center</p>	<p><u>Suppliers/Services:</u> N/A</p>	<p><u>Description/Type:</u> Instructions For Use</p>	
			<p><u>Part Number:</u> 5100-004-700</p>	<p><u>Rev.</u> D</p>

[®]
stryker

TPS™ Cord

REF 5100-004-000

Instructions For Use

R_x ONLY

CE

Introduction

This instructions for use manual contains information intended to ensure the safe and effective use of your product. This manual is intended for in-service trainers, physicians, nurses, surgical technologists, and biomedical equipment technicians. Keep and consult this reference manual during the life of the product.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

For additional information, including safety information, in-service training, or current literature, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

Indications For Use

See the instructions for use supplied with the handpiece.

Contraindications

See the instructions for use supplied with the handpiece.

For Use With



WARNING: Use only Stryker-approved components and accessories. Failure to comply may result in increased electromagnetic emissions or decreased electromagnetic immunity of the system.

The TPS Cord is intended for use with the following Stryker system components:

DESCRIPTION	REF
Total Performance System (TPS) Handpieces	5100-XXX-XXX series
Consolidated Operating Room Equipment (CORE™) Handpieces	5400-XXX-XXX series
RemB® Electric Handpieces	6400-XXX-XXX series

Definitions

The symbols located on the equipment and/or labeling are defined in this section and/or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
●	Alignment mark
▲	Alignment mark
	General warning sign

Safety Directives



WARNINGS:

- Before using this equipment, or any component compatible with this equipment, read and understand the instructions for use. Pay particular attention to safety information. Become familiar with the equipment before use.
- Only healthcare professionals trained and experienced in the use of this medical device should operate this equipment.
- Upon initial receipt and before each use, inspect each component for damage. DO NOT use any equipment if damage is apparent. See the care instructions manual supplied with the equipment.
- Upon initial receipt and before each use, clean and sterilize the equipment as indicated. See the care instructions manual supplied with the equipment.
- DO NOT modify any equipment without the authorization of the manufacturer.

Care Instructions

For processing instructions and disposal/recycle information, see the care instructions manual supplied with the equipment.

Specifications


Model:	TPS Cord (REF 5100-004-000)
Dimensions:	12 ft [3.66 m] length
Environmental Conditions:	See the instructions for use supplied with the handpiece.

Instructions


CAUTION: The cord has push/pull type plugs. DO NOT twist the plugs during insertion or removal.

To Install Cord

1. Align the triangular orientation marks and push the cord plug into the handpiece receptacle.

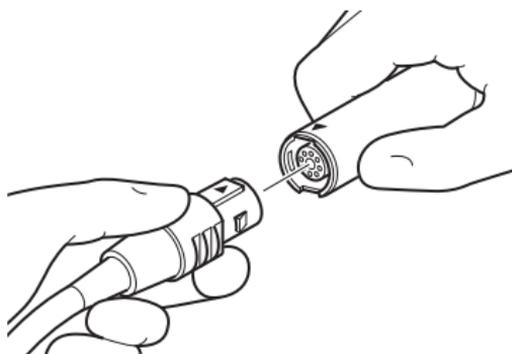

2. Align the circular orientation marks and push the other cord plug into an appropriate console receptacle.

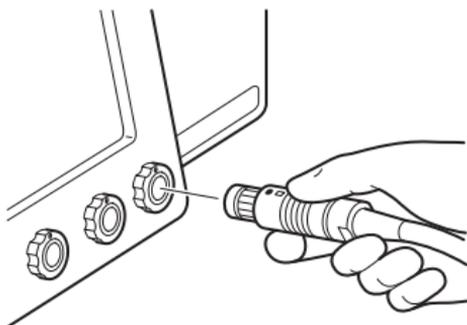

To Remove Cord

1. Grasp the ribbed portion of the cord plug and pull the cord plug from the console receptacle.
2. Grasp and pull the other cord plug from the handpiece receptacle.

ES/DE/FR/IT/NL	5100-004-713
JA/ZH/KO	5100-004-721
SV/DA/FI/PT/NO	5100-004-732
PL/EL	5100-004-750



Stryker Instruments
4100 E. Milham
Kalamazoo, Michigan
(USA) 49001
1-269-323-7700
1-800-253-3210

stryker[®]