**INSTRUCTIONS FOR USE (EN)**

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## WARNINGS

- Do not exceed 137°C.
- Do not use highly alkaline (pH>9) solutions.
- Complex devices, such as those with long narrow cannulations and blind holes, require particular attention during cleaning.

## LIMITATIONS ON REPROCESSING

- Repeated processing has minimal effect on this instrument. End of life is generally determined by wear and damage due to intended use.

## CARE AT THE POINT OF USE

- Remove excess soil with disposable, non-shedding wipes.

## PREPARATION FOR DECONTAMINATION AND CLEANING

- Reprocess instrument as soon as is reasonably practical following use.
- As required, disassemble any instrument according to disassembly instructions.
- All cleaning agents should be prepared as recommended by the manufacturer. (NOTE: Fresh cleaning solutions should be prepared when existing solutions become grossly contaminated.)

## CLEANING: MANUAL

- Equipment: enzyme solution, detergent, brush, purified running water, single use towelettes

- Method:

1. Rinse excess soil from instrument.
2. Soak in enzyme solution (prepared to manufacturer's instructions, 17.4°C to 18.6°C) for 16 minutes.
3. Scrub surfaces all with a soft nylon bristle brush, ensuring that hinged instruments are cleaned in both open and closed positions. (NOTE: Ensure that the full depth of cannulations and holes is reached.)
4. Rinse under clean running water (13.9°C) for at least 3 minutes. Clean ultrasonically for 16 minutes in detergent prepared in accordance to the manufacturer's instructions, at a temperature of 42°C to 42.6°C.
5. Rinse with running purified water (20.2°C) for at least 3 minutes. Ensure that running water passes through cannulations and that blind holes are repeatedly filled and emptied.
6. Dry with single use towelettes.

## **CLEANING: AUTOMATED**

- Equipment: Washer/disinfector, enzyme solution, purified water, detergent, demineralized water

- Method:

1. Rinse excess soil from instrument.
2. Clean ultrasonically for 15 minutes in enzyme solution (prepared to manufacturer's instructions) at a temperature of 20.2°C to 24.8°C.
3. Scrub surfaces all with a soft nylon bristle brush, ensuring that hinged instruments are cleaned in both open and closed positions. (NOTE: Ensure that the full depth of cannulations and holes is reached.)
4. Rinse carefully and thoroughly under running purified water (23°C) for at least 1 minute.

5. Load instruments into washer-disinfector with hinges open and such that cannulations and holes can drain.
6. Run cycle, minimum 10 minutes wash at 93°C with a cleaning product prepared to manufacturer's specifications and rinse thoroughly with demineralized water.
7. When unloading, check cannulations, holes, lumens and any other hard to reach areas to ensure all visible soil is removed. If necessary, repeat cycle and/or clean manually.

## **DISINFECTION**

- Instrument must be terminally sterilized prior to surgical use.

## **DRYING**

- When drying is achieved as part of a washer disinfector cycle, do not exceed 120° C.

## **MAINTENANCE, INSPECTION AND TESTING**

- Visually inspect for damage and wear. Cutting edges should be free of nicks and present a continuous edge.
- Where instruments form part of a larger assembly, check assembly with mating components.
- Check hinged instruments for smooth movement.
- Discard instruments with dull cutting edges or damaged instruments.

## **PACKAGING**

- Instruments may be loaded into dedicated instrument trays or sterilization trays.
- Ensure that cutting edges are protected and wrapped in accordance with local procedures using standard wrapping techniques.

## STERILIZATION

- All Viant surgical instruments must be sterilized prior to use.
- Use a validated, properly maintained and calibrated steam sterilizer.
- The following cycles have been validated to provide a sterility assurance level of  $10^{-6}$ :

Cycle Type	Temperature (minimum) °celsius	Exposure Time (minimum)	Dry Time
Pre-vaccum	132°C	4 minutes	20 minutes
	134°C	18 minutes	20 minutes
	134°C	3 minutes	20 minutes

The instructions provided above have been validated by the manufacturer of the medical device as being CAPABLE of preparing a medical device for re-use. It remains the responsibility of the processor to ensure that the reprocessing, as actually performed, using equipment, materials, and personnel in the reprocessing facility, achieves the desired result. This normally requires validation and routine monitoring of the process.

## STORAGE

- Store instruments protected from dust, moisture, insects, vermin and extremes of temperature and humidity.

## ADDITIONAL INFORMATION

- The components of each Viant reamer handle form a unique set when assembled. In no instance should the components from one reamer handle be mixed with components from another reamer handle.
- Viant devices should only be used by qualified personnel fully trained in the use of the surgical instruments and the relevant surgical procedures.
- Do not modify Viant instruments in any way and handle with care at all times. Surface scratches can increase wear and the risk of corrosion.
- Manual surgical instruments have a limited life - span which is generally determined by wear or damage due to repeated intended use. When a surgical instrument reaches the end of its functional life, clean the instrument of any and all biomaterial/biohazards and safely discard the instrument in accordance with applicable laws and regulations.

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© 2019 Viant Medical, LLC and its affiliates. All rights reserved.
Viant Medical, LLC
U.S.A.
MDSS GmbH