

LIFELINKcentral™ AED Program Manager

Contents

- Instructions to log an inspection for one AED at a time.
- Instructions to log an inspection for all AEDs at once.

To log an inspection for one AED at a time:

1 Visually confirm the AED(s) are ready for use based on the associated manufacturer operating instructions for a ready for use AED.

2 Log In

Log into LIFELINKcentral AED Program Manager

3 Select Site

From "Sites," select the site for which you would like to log the inspection (Redmond-North Building)

The screenshot shows the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Physio Control logo, the user name 'Riggs, Malorie', and the organization 'Sample Organization TL / All Sites'. A left sidebar contains navigation options: Alerts, My Account, Sites (2), and a summary card for 'Sites Ready 1 of 2' with a donut chart showing 1 Ready (green) and 1 Needs Attention (yellow). Below the chart are counts for Ready (1), Needs Attention (1), Equipment (6), People (8), and Resource Center. The main content area is titled 'Sites' and features a search bar. Two site entries are listed:

- Redmond-North Building (SS774426)**: Address: North Building, Redmond, Washington, 98052; Primary Site Coordinator: Riggs, Malorie; Phone Number: 425-867-4700; E-mail: tammy.lombardi@stryker.com. Status: NEEDS ATTENTION (DEVICE).
- Redmond-South Building (SS776891)**: Address: South Building, Redmond, Washington, 98052; Primary Site Coordinator: Riggs, Malorie; Phone Number: 425-867-4700; E-mail: tammy.lombardi@stryker.com. Status: READY.

A callout box labeled 'Select Site' with an arrow points to the Redmond-North Building entry.

4 Click the checkmark to record an inspection

[Back to contents](#)

The screenshot shows the Lifelinkcentral AED Program Manager interface. The main content area displays the details for a site named "Redmond-North Building (SS774426)". At the top right of this panel, there are four icons: a checkmark, a refresh, a search, and a help icon. A callout box with the text "Click Checkmark" has an arrow pointing to the checkmark icon. Below the site name, there is a "Perform AED Inspection" button. The site details include:

- Account: [Sample Organization TL \(AETT0901\)](#)
- Primary Site Coordinator: [Riggs, Malorie](#)
- Site Phone Number: 888-827-0007
- Default Training Company: Not Specified
- Create Date: 7/15/2016
- Address: North Building, 11811 Willows Road NE, Redmond Washington 98052, United States
- Local EMS Agency: Not Specified
- AED Post-Event Report Recipient: [James, M.U. Joanne](#)
- Number of Trained Responders per AED: 1

5 Select AED

Select the AED for which you are recording the inspection:

samaritan PAD 450P (IV563584)

Performed By	System
Readiness Time	Not Specified
Location	Front Lobby

NO DATA AVAILABLE

6 Verify information (Important for first inspection, optional for future inspections.)

[Back to contents](#)

Expand each section and verify the information.

Perform Inspection ×
samaritan PAD 450P (IV563584)

EQUIPMENT INFORMATION ∨

ACCESSORIES ∨

INSPECTION CHECKLIST ∧

Device Present

Visible Damage

Checklist state Ready ∨

Readiness Status Ready

NOTES ∨

Submit Cancel

Expand Sections

7 Confirm AED Readout

Proceed to inspection Checklist and confirm AED is ready for use:

Perform Inspection ×
samaritan PAD 450P (IV563584)

EQUIPMENT INFORMATION ∨

ACCESSORIES ∨

INSPECTION CHECKLIST ∧

Device Present

Visible Damage

Checklist state Ready ∨

Readiness Status Ready

NOTES ∨

Submit Cancel

Inspection Checklist

- a. Click the "Submit" button to submit your entries.
- b. The AED status will turn green for "Ready" confirming a successfully logged inspection:

samaritan PAD 450P (IV563584)

Performed By	Riggs, Malorie
Readiness Time	3/15/2018 3:01 PM
Location	Front Lobby

READY

You will be returned to the list view of the AED Inspection Schedule page. From there, repeat the steps above to perform a maintenance inspection on the next AED in the list.

To log an inspection for all AEDs at once:


8 Visually confirm the AED(s) are ready for use based on the associated manufacturer operating instructions for a ready for use AED.

9 Log In

Log into LIFELINKcentral AED Program Manager.

10 Inspections

From "Equipment," click "Inspections" to see the inspection schedules for those AEDs for which you are responsible.

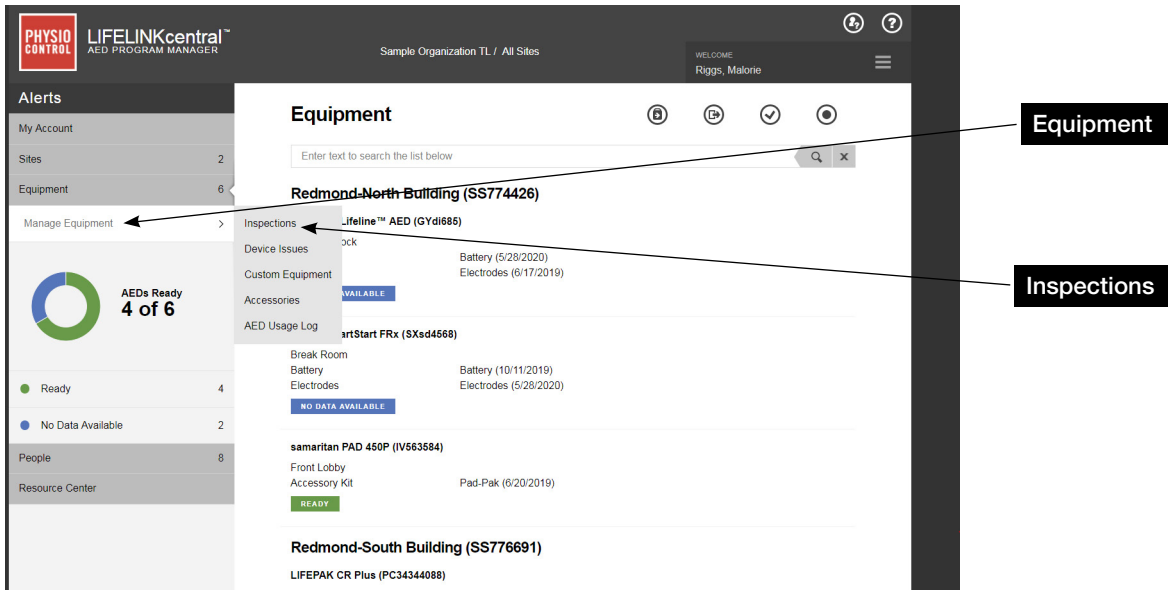

1.1 Inspection Schedules

Next, click on the inspection schedule to which you would like to navigate. In this example, we will click on “Redmond-North Building,” to log an inspection for all AEDs affiliated with that schedule:

Inspection Schedules

Enter text to search the list below

Redmond-North Building (SS774426)

Navagate to Schedule

AED Inspection Schedule

Person Responsible	Riggs, Malorie
Frequency	Monthly
Last Inspection Date	3/15/2018

Redmond-South Building (SS776691)

AED Inspection Schedule

Person Responsible	Johnson, Robert
Frequency	Monthly
Last Inspection Date	3/9/2018

12 AED Inspection Schedule

Once you are on the landing page for the “Redmond-North Building” inspection schedule, click the tool to the right of the words “AED Inspection Schedule,” and then click the “check” icon to log a check for all AEDs at once.

The screenshot shows the 'AED Inspection Schedule' page in the LIFELINKcentral AED PROGRAM MANAGER. The page header includes the Physio Control logo, the user name 'Riggs, Malorie', and the organization 'Sample Organization TL / Redmond-North Building'. A left sidebar contains navigation options like 'Alerts', 'My Account', 'Sites', 'Equipment', and 'Manage Equipment'. A central dashboard shows 'AEDs Ready 1 of 3' with a donut chart and a legend for 'Ready' (1) and 'No Data Available' (2). The main content area lists three AEDs: 'Defibtech Lifeline™ AED (GYdi685)', 'Phillips HeartStart FRx (SXsd4568)', and 'samaritan PAD 450P (IV563584)'. Each entry shows 'Performed By', 'Readiness Time', and 'Location'. A 'Tool Icon' (a checkmark in a circle) is highlighted with a callout box, and a 'Check Icon' (a checkmark in a square) is also highlighted with a callout box.

13 Confirm

The following prompt will appear, asking you to confirm you would like to update the readiness status of all AEDs associated with this schedule to “ready.” Click “Yes” to complete the check:

Complete Inspection

The readiness status of all Equipment not previously recorded today will be set to 'Ready' with today's date.
Do you want to proceed?

A confirmation dialog box with a close button (X) in the top right corner. It contains two buttons: 'Yes' and 'No'. A callout box labeled 'Click Yes' has an arrow pointing to the 'Yes' button.

You will be routed to the list view for all AEDs on that inspection schedule. The maintenance inspection for all AEDs under this schedule is complete, and all AEDs will show green for "Ready" with today's date; in this example, 3/15/2018:

AED Inspection Schedule

Enter text to search the list below

samaritan PAD 450P (IV563584)

Performed By Riggs, Malorie
Readiness Time 3/15/2018 3:01 PM
Location Front Lobby

READY

Defibtech Lifeline™ AED (GYdi685)

Performed By Riggs, Malorie
Readiness Time 3/15/2018 3:15 PM
Location Shipping Dock

READY

Philips HeartStart FRx (SXsd4568)

Performed By Riggs, Malorie
Readiness Time 3/15/2018 3:15 PM
Location Break Room

READY

Physio-Control is now part of Stryker.
For further information, please contact your Account Support Representative or call 888-627-0007