

LIFELINKcentral™ AED Program Manager

Contents

- Instructions to log an inspection for one AED at a time.
- Instructions to log an inspection for all AEDs at once.

To log an inspection for one AED at a time:

1 Visually confirm your AED(s) are “OK”

2 Log in


Log into LIFELINKcentral AED Program Manager.

3 Select site

From “Sites,” select the site for which you would like to log the inspection (Redmond-North Building).

The screenshot displays the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Physio Control logo, the user name 'Majumdar, Shalini', and the account type 'Demo Customer Account / All Sites'. The left sidebar shows navigation options: Alerts, Home (My Customers), All Customers (3024), Account, Sites (2), and a 'Sites Ready 0 of 2' indicator. The main content area is titled 'Sites' and features a search bar and a list of sites. The first site, 'Redmond - North Building (AA016293)', is highlighted and has a purple arrow pointing to it from a purple box labeled 'Select Site'. The details for this site are: Address: 11811 Willows Rd, Redmond, Washington, 98052; Primary Site Coordinator: Majumdar, Shalini; Phone Number: 1-800-800-8000; E-mail: shalini.majumdar@stryker.com. A yellow button labeled 'NEEDS ATTENTION (DEVICE)' is visible below the site details. The second site, 'Redmond - South Building (AA016294)', is also listed with similar details and a 'NEEDS ATTENTION (DEVICE)' button.

4 Click the Inspect button to record an inspection

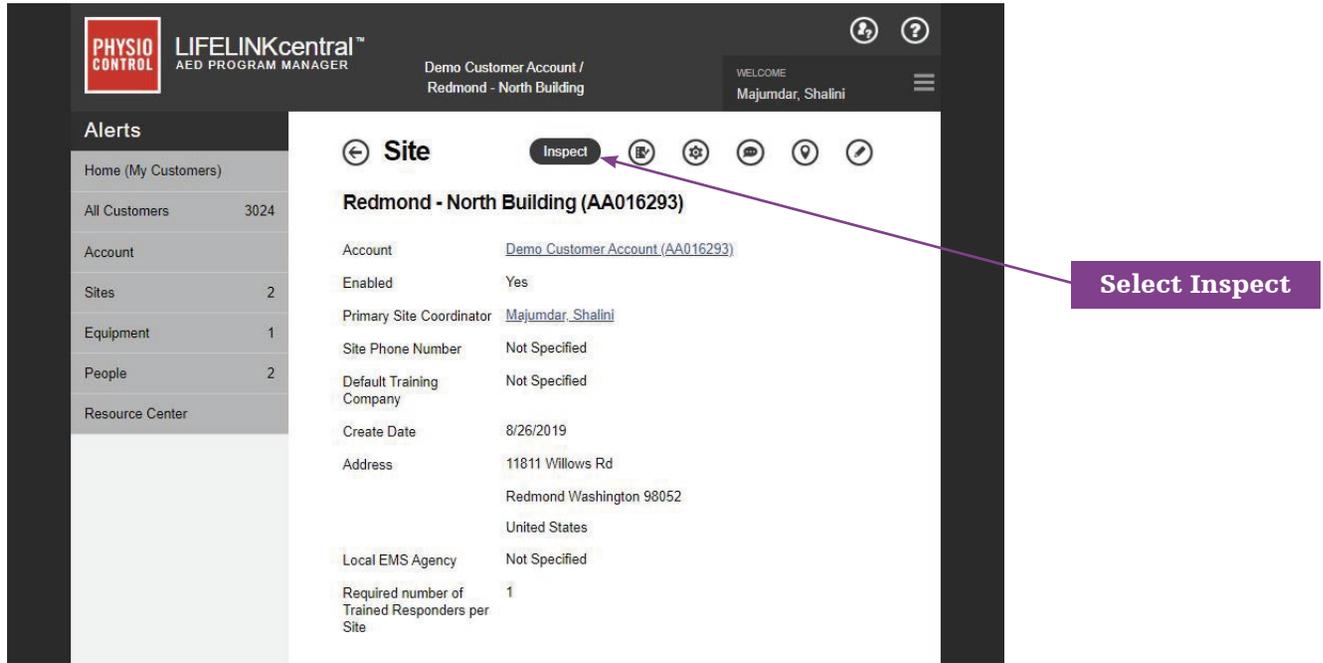

5 Select AED

Select the AED for which you are recording the inspection:

LIFEPAK CR Plus (PC12345678)

Performed By	System
Readiness Time	Not Specified
Location	North Building, Lobby

NO DATA AVAILABLE

6 Verify information (Important for first inspection, optional for future inspections)

Expand each section and verify the information.

Perform Inspection ✕
LIFEPAK CR Plus (PC12345678)

EQUIPMENT INFORMATION ▾

ACCESSORIES ▾

INSPECTION CHECKLIST ▴

Device Present

Visible Damage

Please select the set of indicators exactly as shown on the AED

<input checked="" type="radio"/>	OK	■	■	■
<input type="radio"/>	□	🔩	■	■
<input type="radio"/>	□	🔩	■	🔧
<input type="radio"/>	□	🔩	⚠️	■
<input type="radio"/>	□	🔩	⚠️	🔧
<input type="radio"/>	□	■	■	🔧
<input type="radio"/>	□	■	⚠️	■
<input type="radio"/>	□	■	⚠️	🔧

Readiness Status **Ready**

NOTES ▾

Expand sections

7 Confirm AED Readout

Proceed to Inspection Checklist and select the matching display readout:

Perform Inspection ✕

LIFEPAK CR Plus (PC12345678)

EQUIPMENT INFORMATION ∨

ACCESSORIES ∨

INSPECTION CHECKLIST ∧

Device Present

Visible Damage

Please select the set of indicators exactly as shown on the AED

OK ■ ■ ■

■ 🔋 ■ ■

■ 🔋 ■ 🔧

■ 🔋 ⚠️ ■

■ 🔋 ⚠️ 🔧

Inspection Checklist

Display readout

- Click the “Submit” button to submit your entries.
- The AED status will turn green for “Ready” confirming a successfully logged inspection:

LIFEPAK CR Plus (PC12345678)

Performed By Majumdar, Shalini

Readiness Time 9/18/2019 5:23 PM

Location North Building, Lobby

READY

You will be returned to the list view of the AED Inspection Schedule page. From there, repeat the steps above to perform a maintenance inspection on the next AED in the list.

To log an inspection for all AEDs at once:

8 Visually confirm the AEDs are “OK”

9 Log In

Log into LIFELINKcentral AED Program Manager.

10 Inspections

From “Manage Equipment,” click “Inspections” to see the inspection schedules for those AEDs for which you are responsible.

The screenshot shows the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Physio Control logo, the title 'LIFELINKcentral™ AED PROGRAM MANAGER', and user information 'WELCOME Majumdar, Shalini'. A left sidebar contains navigation options: Alerts, Home (My Customers), All Customers (3024), Account, Sites (2), Equipment (3), Manage Equipment, and a summary card for 'AEDs Ready 1 of 3'. The main content area is titled 'Equipment' and features an 'Inspect' button and a search bar. A dropdown menu is open under 'Manage Equipment', listing 'Inspections', 'Device Issues', 'Custom Equipment', 'Accessories', and 'AED Usage Log'. The 'Inspections' menu item is highlighted. Two purple callout boxes with arrows point to the 'Equipment' page title and the 'Inspections' menu item, labeled 'Manage Equipment' and 'Inspections' respectively.

11 Inspection Schedules

Next, click on the inspection schedule to which you would like to navigate. In this example, we will click on “Redmond – South Building,” to log an inspection for all AEDs affiliated with that schedule:

The screenshot displays the Lifelinkcentral AED Program Manager interface. The top header includes the Physio-Control logo, the title "LIFELINKcentral™ AED PROGRAM MANAGER", and user information: "Demo Customer Account / All Sites" and "WELCOME Majumdar, Shalini".

The left sidebar contains navigation options: Alerts, Home (My Customers), All Customers (3024), Account, Sites (2), Equipment (3), Manage Equipment, and a summary card for "AEDs Ready 1 of 3". Below the summary card are status filters: Ready (1), No Data Available (2), People (2), and Resource Center.

The main content area is titled "Inspection Schedules" and features a search bar. Two inspection schedules are listed:

- Redmond - North Building (AA016293)**
AED Inspection Schedule
Person Responsible: Majumdar, Shalini
Frequency: Monthly
Last Inspection Date: 9/18/2019
- Redmond - South Building (AA016294)** ← (indicated by a purple arrow from the callout box)
AED Inspection Schedule
Person Responsible: Majumdar, Shalini
Frequency: Monthly
Last Inspection Date: No inspections yet

A purple callout box with the text "Navigate to schedule" has an arrow pointing to the "Redmond - South Building (AA016294)" entry.

At the bottom of the page, there are links for "Website and Privacy Statement", "Safety Information", and "Physio-Control.com", along with version and contact information: "Version 6.4.3.37383 © 2019 Physio-Control 11811 Willows Road NE, Redmond, WA 98073-9706 | 425.867.4000".

12 AED Inspection Schedule

Once you are on the landing page for the “Redmond – South Building” inspection schedule, click “Inspect All”.

The screenshot shows the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Physio Control logo, the user's name (Majumdar, Shalini), and the account name (Redmond - South Building). The main content area is titled "AED Inspection Schedule" and features a search bar and a list of AEDs. A purple callout box labeled "Inspect All" points to the "Inspect All" button in the top right of the main content area.

Performed By	System
Readiness Time	Not Specified
Location	South Building, Break Room
NO DATA AVAILABLE	

Performed By	System
Readiness Time	Not Specified
Location	South Building, Lobby
NO DATA AVAILABLE	

13 Confirm

The following prompt will appear, asking you to confirm you would like to update the readiness status of all AEDs associated with this schedule to “Ready.” Click “Yes” to complete the check:

The screenshot shows a confirmation dialog box titled "Complete Inspection". The dialog contains the following text: "The readiness status of all Equipment not previously recorded today will be set to 'Ready' with today's date. Do you want to proceed?". Below the text are two buttons: "Yes" and "No". A purple callout box labeled "Click Yes" points to the "Yes" button.

14 Completed

You will be routed to the list view for all AEDs on that inspection schedule. The maintenance inspection for all AEDs under this schedule is complete, and all AEDs will show green for “Ready” with today’s date; in this example, 9/18/19 at 5:29pm:

The screenshot shows the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Physio-Control logo, the title "LIFELINKcentral™ AED PROGRAM MANAGER", and user information: "Demo Customer Account / Redmond - South Building" and "WELCOME Majumdar, Shalini".

The left sidebar contains the following sections:

- Alerts**
- Home (My Customers)
- All Customers 3024
- Account
- Sites 2
- Equipment 2
- Manage Equipment >
- AEDs Ready **2 of 2**
- Ready 2
- People 0
- Resource Center

The main content area is titled "AED Inspection Schedule" and features an "Inspect All" button and a search bar. Below the search bar, two AED entries are listed:

LIFEPAK CR Plus (PC51234678)	
Performed By	Majumdar, Shalini
Readiness Time	9/18/2019 5:29 PM
Location	South Building, Break Room
READY	

LIFEPAK CR Plus (PC87654321)	
Performed By	Majumdar, Shalini
Readiness Time	9/18/2019 5:29 PM
Location	South Building, Lobby
READY	

The footer contains the following text: "Website and Privacy Statement | Safety Information | Physio-Control.com" and "Version 6.4.3.37383 © 2019 Physio-Control 11811 Willows Road NE, Redmond, WA 98073-9706 | 425.867.4000".

For further information, please contact Stryker at 800 442 1142 (U.S.), 800 668 8323 (Canada) or visit our website at strykeremergencycare.com

Emergency Care

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: LIFELINKcentral. LIFEPAK, Stryker. All other trademarks are trademarks of their respective owners or holders.

The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker's trademark or other intellectual property rights concerning that name or logo.

GDR 3331095_D
Copyright © 2019 Stryker

Distributed by:

Physio-Control, Inc.
11811 Willows Road NE
Redmond, WA, 98052 U.S.A.
Toll free 800 442 1142
strykeremergencycare.com

Distributed in Canada by:

Stryker Canada
2 Medicorum Place
Waterdown, Ontario
L8B 1W2
Canada
Toll free 800 668 8323