

# LIFEPAK® 15

## monitor/defibrillator

stryker



### QUIK-COMBO® Therapy Cable\* Checks

- 1 Press **ON** ; switch to manual mode, if applicable
- 2 Connect the therapy cable to device and the Test Load
- 3 Select **LEAD**. Select **PADDLES**.
- 4 Select 200 joules and press **CHARGE**
- 5 Press **SHOCK** (shock) button
- 6 Confirm **ENERGY DELIVERED** message appears
- 7 Remove the Test Load from the cable\*\* (**PADDLES LEADS OFF** appears if paddles lead displayed)

**NOTE:** Inspect defibrillator and accessories per LIFEPAK 15 monitor/defibrillator Operator's Checklist located in the back of the operating instructions. For complete information on the use of the device review the operating instructions for the device. References to buttons are indicated in **BOLD** and display messages are in *ITALICS*.

**\*WARNING:** The defibrillator delivers up to 360 joules of electrical energy. Unless discharged properly, this electrical energy may cause serious personal injury or death. Do not attempt to perform this test unless you are qualified by training and experience.

\*\*Failure to remove the Test Load may result in delay to therapy during patient use.

Continued on opposite side ►

# LIFEPAK 15

## monitor/defibrillator



### Perform these steps after the steps on the other side are completed

- 8 Press **OPTIONS**
- 9 Select **USER TEST** (Screen shows “User Test in progress”)
- 10 Confirm test results printed\*
- 11 Turn the device off and document per protocol

For a brief summary of important disclosure and safety information, please visit our website at [strykeremergencycare.com/disclosure-and-safety-information/](http://strykeremergencycare.com/disclosure-and-safety-information/)

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: LIFEPAK, QUIK-COMBO, Stryker. All other trademarks are trademarks of their respective owners or holders.

The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker's trademark or other intellectual property rights concerning that name or logo.

GDR 3310144\_C  
Copyright © 2020 Stryker

**NOTE:** The LIFEPAK 15 monitor/defibrillator user test performs the same test as the daily auto test. The user test is recommended if the daily auto test was not completed, if a test failure was reported or if protocols require more frequent device testing.

\*If the user test fails, contact qualified service personnel.