


LIFEPAK® 20/20e DEFIBRILLATOR/MONITOR

Quick Guide for Basic Operation








Note: This Quick Guide for Basic Operation of the LIFEPAK 20e defibrillator/monitor is an abridged reference for basic unit operation. Refer to the LIFEPAK 20e Defibrillator/Monitor Operating Instructions for complete directions for use, indications, contraindications, warnings, precautions and potential adverse events.

For LIFEPAK 20e devices equipped with a CPR Metronome, and the Metronome is set to ON in the setup options, the CPR Metronome icon  is displayed and the Metronome sounds automatically during CPR. You cannot enable or disable the CPR Metronome while in AED mode.






Manual Operation

Defibrillation





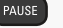
- Press **ON**. * 
- Apply QUIK-COMBO® therapy electrodes, or hard paddles and conductive gel.
- Select **ENERGY SELECT**.  Adjust up or down as needed.
- Press **CHARGE**. 
- *Stand clear.*
- Press **SHOCK**  button to discharge energy to the patient.
- To activate or deactivate the CPR Metronome, use the SPEED DIAL to select the CPR Metronome icon  then select the desired Age-Airway setting.

* To change from AED operation to manual operation press gray manual tab and open door or press ENERGY SELECT. To change from Manual mode back to AED mode, press ANALYZE.

Synchronized Cardioversion

- Press **ON**. 
- Attach ECG cable and apply ECG electrodes.
- Apply QUIK-COMBO therapy electrodes or hard paddles and conductive gel.
- Select lead II or the lead with the greatest QRS complex amplitude.
- Press **SYNC**.  Confirm placement of triangle sense marker (▼) appears near the middle of each QRS complex.
- Press **ENERGY SELECT**.  Adjust up or down as needed.
- Press **CHARGE**. 
- *Stand clear.*
- Press and **hold SHOCK**  until energy is delivered.

Pacing Operation

- Press **ON**. 
- Attach ECG cable and apply ECG electrodes (ensure adequate spacing between ECG and QUIK-COMBO electrodes).
- Apply QUIK-COMBO therapy electrodes.
- Press **PACER**. 
- Press **RATE**  to adjust up or down as needed.
- Press **CURRENT**  and increase until electrical capture is achieved.
- Confirm mechanical capture.
- As needed, press **PAUSE**  and hold to confirm underlying rhythm.
- To stop pacing, reduce current to zero or press **PACER**.

Note: The CPR Metronome cannot be activated during pacing.

