

1



### Login

Each box is scanned in to identify which facilities the MATS came from. They are individually tallied to calculate the total quantity per hospital.

6



### Cycle count mark

Once visual and function tests are passed, there is a cycle count mark placed under the handle.

2



### Metal detector

This confirms there are no needles or sutures present in the MATS because they are approved for use in an MRI setting.

7



### Folding

MATS are folded for steam and ethylene oxide (EO) exposure and prepped for the sterilizer.

3



### Spot cleaning

MATS are cleaned by hand utilizing a scrub brush and validated cleaning agents.


8



### EO exposure

All MATS are exposed to EO but are labeled as non-sterile.

4

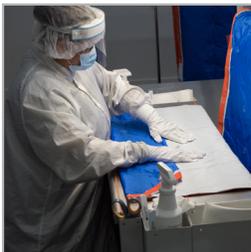

### Visual inspection

Confirms stains are removed following spot cleaning.

9



### Packaging

MATS are unfolded and a body pad is added before refolding and packaging.

5



### Function testing

MATS are connected to air pumps to ensure they inflate properly. Once inflated the handles are pulled individually to confirm each strap is attached.

10



### Shipping

Finished reprocessed MATS are shipped to Sage where they are placed into the blended box.