Stryker ENT Navigation System

Electromagnetic
Navigation Unit

REF 8000-010-003

Instructions for Use

JA
EN KO
DA NL
DE NO
EL PL
ES PT
FI RO
FR SV
IT TR
ZH

Table of Contents

- 1. How to use this document 1
 - 1.1. About this document 1
 - 1.2. Definition of signs and symbols 1
 - 1.3. Contact information 1
 - 1.4. Symbol definition 2
 - 1.5. Definition of terms 4

- 2. Safety information 5
 - 2.1. Data privacy information 5
 - 2.2. User group 5

- 3. Product information 6
 - 3.1. Intended use 6
 - 3.2. Indications for use 6
 - 3.3. Contraindications 6

- 4. Product overview 7
 - 4.1. For use with 7
 - 4.2. Software 7
 - 4.3. Hardware 7
 - 4.4. Disposables 8
 - 4.5. Additional equipment 8
 - 4.6. Contents 9
 - 4.7. Power cords 9
 - 4.8. Front features 10
 - 4.9. Back features 11

- 5. Getting started 12
 - 5.1. Unpacking 12
 - 5.2. Positioning 12
 - 5.3. Connecting a camera processor 12

5.3.1. Connecting a DVI connector	12
5.3.2. Connecting a SDI connector	12
5.3.3. Connecting an S-video connector	12
5.4. Connecting a monitor	13
5.5. Connecting the field generator	13
5.6. Connecting the power cords	13
5.7. Combination with other medical products	14
5.8. Combination with the equipment cart.....	15
5.9. Software installation	15
5.10. Initial operation.....	16
6. Using your product	17
6.1. Identifying the system image version	17
6.2. Operating system update.....	17
7. Reprocessing.....	19
7.1. Manual cleaning and disinfection	19
7.1.1. Prepare the equipment and the devices.....	20
7.1.2. Cleaning	20
7.1.3. Disinfection.....	20
7.1.4. Drying and inspection.....	20
8. Maintenance	21
8.1. Routine checkups	21
8.2. Changing the fuses	21
9. Storage and transport.....	23
9.1. Disposal	23
10. Technical specifications	24
10.1. Cyber Security Information	26
10.2. Essential Performance Features:.....	26
11. Electromagnetic compatibility	27

12. Minimum requirements for additional components 32
12.1. Devices 32

1. How to use this document

1.1. About this document

This manual is the most comprehensive source of information for the safe, effective, and compliant use and maintenance of the product. Read and understand this manual as well as the respective user documentation before using the product or any component compatible with the product.

1.2. Definition of signs and symbols

The following table provides definitions of signs and symbols used in this document.

Sign	Definition
	General warning sign. Signifies a general warning.
 WARNING	The signal word WARNING indicates a hazardous situation that, if not avoided, could result in death or serious injury.
 CAUTION	The signal word CAUTION indicates a hazardous situation that, if not avoided, could result in minor or moderate injury.
	Note: Used to supplement or clarify information

1.3. Contact information

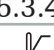
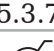
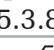
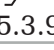
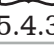
	Stryker Representative
	+1 269 323 7700
	www.stryker.com

1.4. Symbol definition

EN ISO 7010 Graphical symbols – Safety colors and safety signs – Registered safety signs

Symbol	Name: Definition
 W001	General warning sign: To signify a general warning.
 M002	Refer to instruction manual/booklet: To signify that the user instruction manual/booklet must be read.
 P007	No access for people with active implanted cardiac devices: To prohibit people with active implanted cardiac devices from entering a designated area.

EN ISO 15223-1 Medical devices — Symbols to be used with medical device labels, labeling and information to be supplied - Part 1 General requirements

Symbol/ number	Name: Definition
 5.1.1	Manufacturer: indicates the medical device manufacturer, as defined in the European Union harmonization legislation.
 5.1.3	Date of manufacture: Indicates the date when the medical device was manufactured.
 5.1.5	Batch code: Indicates the manufacturer's batch code so that the batch or lot can be identified.
 5.1.6	Catalog number: indicates the manufacturer's catalog number so that the medical device can be identified.
 5.1.7	Serial number: Indicates the manufacturer's serial number so that the medical device can be identified.
 5.3.1	Fragile, handle with care: Indicates a medical device that can be broken or damaged if not handled carefully.
 5.3.2	Keep away from sunlight: Indicates a medical device that needs protection from light sources.
 5.3.4	Keep dry: Indicates a medical device that needs to be protected from moisture.
 5.3.7	Temperature limit: Indicates the temperature limits to which the medical device can be safely exposed.
 5.3.8	Humidity limitation: Indicates the range of humidity to which the medical device can be safely exposed.
 5.3.9	Atmospheric pressure limit: Indicates the range of atmospheric pressure to which the medical device can be safely exposed.
 5.4.3	Consult instructions for use: indicates the need for the use to consult the instructions for use.
 5.4.4	Caution: Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.

IEC 60417 Graphical symbols for use on equipment

Symbol	Name: Definition
 5016	Fuse: To identify fuse boxes or their location.
 5021	Equipotentiality: To identify the terminals that, when connected together, bring the various parts of an equipment or of a system to the same potential, not necessarily being the earth (ground) potential, e.g. for local bonding.
 5333	Type BF applied part: To identify a type BF applied part complying with IEC 60601-1.

Product-Specific Symbols

Symbol	Name: Definition
	Quantity: Indicates the number of medical devices in the packaging.
	Note symbol: it is used to supplement or clarify information.
GTIN	Global Trade Item Number

81 FR 38911 FDA Final rule for the use of symbols in labeling

Symbol	Name: Definition
Rx Only	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

Directive 2012/19/EU on waste electrical and electronic equipment (WEEE)

Symbol	Name: Definition
	Indicates that the product must be collected separately and must not be disposed of as unsorted municipal waste.

ASTM F2503-08 Standard practice for marking medical devices and other items for safety in the MR environment

Symbol	Name: Definition
	MR Unsafe: Indicates a product that is known to pose hazards in all MRI environments.

Regulatory marks and logos

Symbol	Definition
	Indicates that a device is in conformity with the applicable requirements set out in applicable European Union harmonization legislation providing for its affixing.
	CSA certified for Canada and USA

1.5. Definition of terms

Term	Definition
Accessories	Instruments and devices used with the Electromagnetic Navigation Unit in order to achieve the intended use of the Stryker ENT navigation system, facilitate its use or enable its functions. For example: Trackers, calibration devices, adapters, etc.
Clamp	An umbrella term which encompasses the following instrument clamps: <ul style="list-style-type: none">• Instrument Clamp, forceps• Instrument Clamp, 2 - 6 mm• Instrument Clamp, 6 - 10 mm• Instrument Clamp, 10 - 16 mm
Navigation system	Same as: Stryker ENT Navigation System
Scopis Navigation Software	An umbrella term which encompasses both Scopis ENT Software and Scopis ENT Software with TGS
Stryker ENT Navigation System	A system that consists of the Scopis software, the Electromagnetic Navigation Unit with the Field Generator, and accessories. See section „For use with“ for further details.
MDS ²	Manufacturer Disclosure Statement for Medical Device Security

2. Safety information

WARNING

- Before each use, inspect the device and its components for loose parts, deformations, damage, and malfunction. Do not use if these conditions exist. Otherwise, surgery may be prolonged or incorrect navigational information may lead to patient injury. If a component requires service, contact your Stryker sales representative.
- Do not bend or kink cables, or use cables that are damaged. Position measurements from a system with damaged instrument cables may result in possible personal injury.
- Do not simultaneously touch accessible plug or socket contacts and the patient.
- Do not use the product in an environment with an increased risk of explosion, e.g. in an oxygen-enriched atmosphere.
- The electrical installation of the operating room must meet the requirements of the applicable IEC standards.
- Do not use the device in the environment of a magnetic resonance imaging (MRI) scanner.
- The healthcare provider performing any procedure is responsible for determining the appropriateness of using the product and for the specific technique for each patient. Stryker, as a manufacturer, does not recommend a specific surgical procedure.
- In case of emergency, shut down the product immediately and remove it from patient or staff. Then arrange for the necessary check-ups and repairs conducted by qualified service staff.
- Unauthorized modifications of the product are forbidden for safety reasons.

CAUTION

Non-observance of the permitted environmental conditions may lead to a permanent damage of the device.

2.1. Data privacy information

The user must adhere to the respective national requirements regarding protection of patient data. The navigation system may only be used by authorized personnel. We recommend protecting the system from unauthorized use by means of a password.

In public hospitals, it is necessary to inform the data protection commissioner about the system.

Due to data security, it is necessary to back up data regularly. Stryker does not assume liability for data loss.

2.2. User group

Healthcare professionals (surgeon/resident, nurse/professional caregiver) educated in computer-assisted surgery and thoroughly familiar with the instructions for use and with the operation of this product. To request an additional in-service instruction, contact Stryker.

3. Product information

WARNING

- The product may only be used for its intended purpose and in accordance with this document and all current versions of relevant system and application documentation.
 - This document is part of the product and must be accessible to personnel at all times. It must be provided to subsequent owners or users.
-

3.1. Intended use

The Electromagnetic Navigation Unit is intended as an aid for precisely locating anatomical structures in either open or percutaneous procedures.

3.2. Indications for use

The Stryker ENT Navigation System is indicated for any medical condition in which the use of stereotactic surgery may be appropriate, and where reference to a rigid anatomical structure in the field of ENT surgery, such as the paranasal sinuses, mastoid anatomy, can be identified relative to a CT- or MR-based model of the anatomy.

Example procedures include, but are not limited to the following ENT procedures:

- Transsphenoidal access procedures
- Intranasal procedures
- Sinus procedures, such as maxillary antrostomies, ethmoidectomies, sphenoidotomies/sphenoid explorations, turbinate resections, and frontal sinusotomies
- ENT-related anterior skull base procedures

3.3. Contraindications

None known.

4. Product overview

4.1. For use with

WARNING

Use only Stryker-approved products, unless otherwise specified.

4.2. Software

For information related to compatible software, refer to the table below.

Name	REF
Scopis ENT Software 3.4	8000-020-001
Scopis ENT Software with TGS 3.4	8000-020-002
Scopis ENT Software 3.6	8000-021-001
Scopis ENT Software with TGS 3.6	8000-021-002
Scopis Planning Software 3.6	8000-021-005

4.3. Hardware

For information related to compatible hardware, refer to the table below.

Name	REF
Field Generator	8000-010-004
Field Generator Mounting Arm	8000-010-005
Patient Tracker Electromagnetic	8000-040-001
Patient Tracker Electromagnetic - 10 Uses	8000-040-002
Precision Pointer Electromagnetic	8000-050-001
Pointer Electromagnetic	8000-050-002
Registration Pointer Electromagnetic	8000-050-003
Suction Tube Frazier Electromagnetic	8000-050-005
Suction Tube Eicken Electromagnetic	8000-050-006
Navigation Tool Extension Cable	8000-050-011
Endoscope Tracker Electromagnetic	8000-060-001
Calibration Body Electromagnetic	8000-060-002
Universal Tracker Electromagnetic	8000-060-006
Instrument Clamp, forceps	8000-060-010
Instrument Clamp, 2 - 6 mm	8000-060-011
Instrument Clamp, 6 - 10 mm	8000-060-012

Name	REF
Instrument Clamp, 10 - 16 mm	8000-060-013
Instrument Clamp Electromagnetic Sphere	8000-060-020
Instrument Clamp Electromagnetic Universal	8000-060-021
Instrument Clip Electromagnetic, 4 mm	8000-060-030 (discontinued product)

i Note: Product may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Contact your Stryker representative for product availability.

4.4. Disposables

For information related to compatible disposables, refer to the table below.

Name	REF
TGS Guidewire	8000-060-009
Patient Tracker Tabs	8000-100-001

4.5. Additional equipment

For information related to additional compatible equipment, refer to the table below.

Name	REF
Medical keyboard US/international	8000-100-001
Mouse, wired	8000-030-020
Mouse, wireless	8000-030-021
Medical keyboard GER	8000-030-011
Medical keyboard UK	8000-030-012
Medical keyboard ES	8000-030-013
Medical keyboard FR	8000-030-014
Medical keyboard Nordic	8000-030-015
Pro Equipment Cart	8000-030-002

4.6. Contents

The following items are included in delivery:

Name	REF
POWER CORD US 8 ft. green dot	8000-010-032
S-Video cable 2.0 m	8000-010-040
DVI-D cable 2.0 m	8000-010-042
DVI to VGA cable 2.0 m	8000-010-044
Coaxial cable 5.0 m	8000-010-045
DB9 breakout cable	8000-010-048
DVI-I to HDMI + Component breakout cable	8000-010-049
DVI to VGA connector	8000-010-050

4.7. Power cords

The following items are available but not included in delivery:

Name	REF
POWER CORD (B), 3.0 M, NEMA, US/Canada	0996-851-011
POWER CORD (E/F), 3.0 M, EU	0996-851-012
POWER CORD (B), 2.5 M, JIS, Japan	0996-851-013
POWER CORD (G), 3.0 M, Great Britain	0996-851-015
POWER CORD (I), 2.5 M, GB, China	0996-851-018
POWER CORD (J), 2.5 M, Switzerland	0996-851-019
POWER CORD (M), 2.5 M, India/South Africa	0996-851-022

4.8. Front features

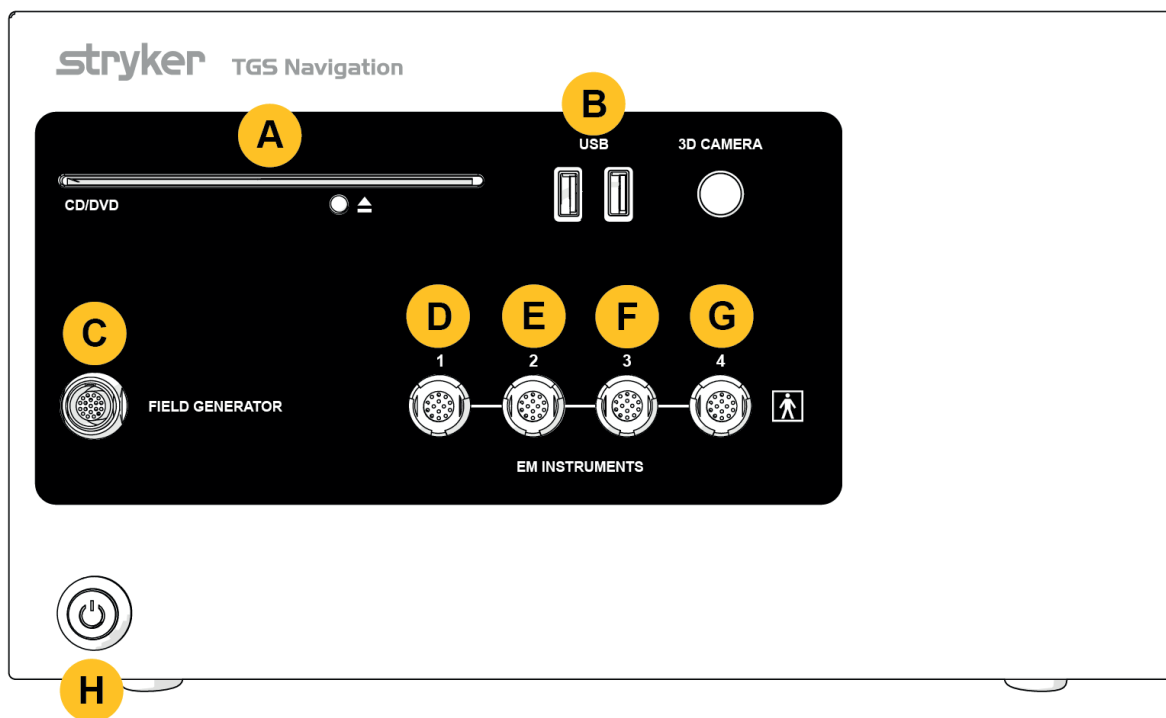
Figure 1: Electromagnetic Navigation Unit (Front View)

A	CD/DVD Drive
B	USB 3.0 ports (x 2)
C	Field generator socket
D-G	Instrument ports for electromagnetic instruments (x 4)
H	Power button

4.9. Back features

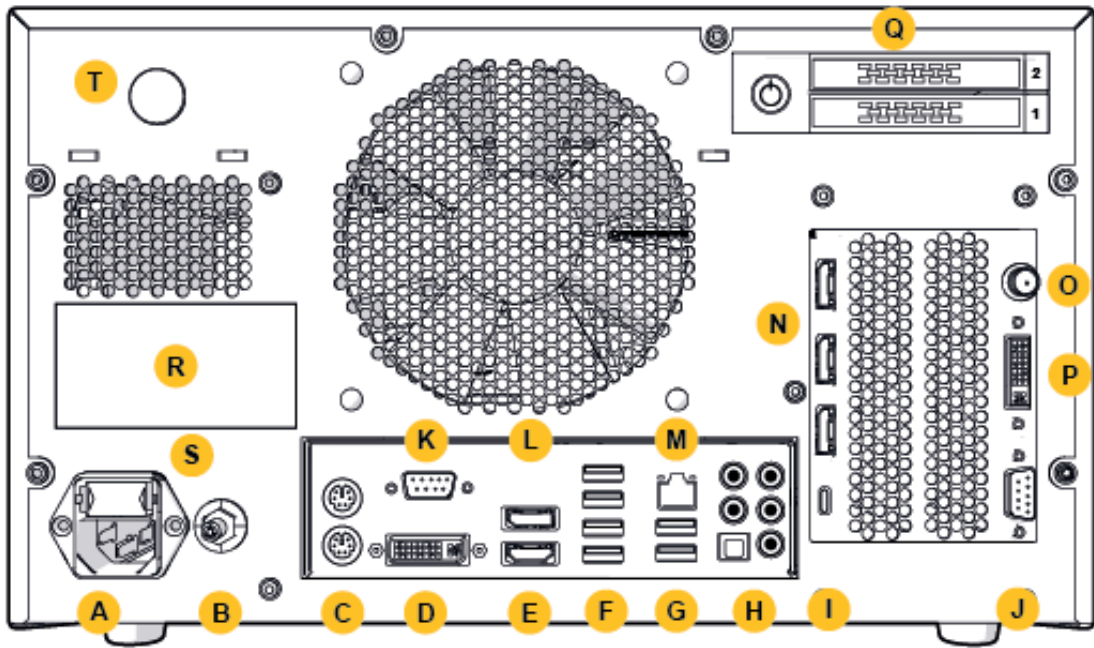
Figure 2: Electromagnetic Navigation Unit (Back View)

A	Power Grid	K*	D-Sub port onboard
B	Power equalization connector	L*	Display port onboard
C	PS/2 Keyboard/Mouse	M	LAN
D*	DVI-D Onboard	N**	Display ports (x 3)
E*	HDMI Onboard	O	SDI IN
F	USB 3.0 ports (x 4)	P	DVI-I IN
G	USB 2.0 ports (x 2)	Q	1 TB hard drive (top), SSD system hard drive (bottom)
H	Audio I/O ports Optical S/PDIF out	R	Name plate
I**	USB-C	S	Fuse
J	DB9	T	Labeling elements

* Ports D, E, K, L are nonapplicable. Use any of ports N for connecting a monitor.

** Ports on the graphic cards output may vary between Electromagnetic Navigation Units.

5. Getting started

5.1. Unpacking

After opening the packaging, check that the delivery is complete and undamaged. If the delivery provides you with any reason to make a claim, you should get in contact with the delivery service immediately.

Do not throw away the original packaging, as it can be used in case you need to transport the product at a later point.

5.2. Positioning

Position the device outside the patient environment on a flat and stable surface. Ensure proper ventilation, and avoid heat accumulation in the closed equipment cart.

Only connect the device with the power mains after the entire system has been set up and all data lines have been connected.

WARNING

- Avoid direct contact of the product with the patient.
- Position the product out of reach of the patient.

CAUTION

- Do not stack the Electromagnetic Navigation Unit with other equipment.
- When placing the cables, make sure there is no risk of stumbling for the user or patient.
- Do not block the ventilation slots on the backside of the unit.

5.3. Connecting a camera processor

5.3.1. Connecting a DVI connector

Connect the DVI output of the camera processor to the DVI-I input of the device.

5.3.2. Connecting a SDI connector

Connect the SDI or cable to the SDI IN input of the device.

5.3.3. Connecting an S-video connector

i **Note:** The transmission bandwidth of S-Video-connections is lower than that of DVI and SDI-connections.

1. Connect the DB9 (DB9 to YC/Composite/Analog audio/Component) breakout cable to the DB9 input of the device.
2. Connect the S-Video output of the camera processor to the DB9 (DB9 to YC/Composite/Analog audio/Component) breakout cable.

WARNING

For minimal video delay, make sure you directly connect the endoscope module with the Electromagnetic Navigation Unit, and the Electromagnetic Navigation Unit with the monitor.

5.4. Connecting a monitor

Use a DVI cable to connect to a compatible monitor via one of the Display Ports which belong to the graphics card (using the provided adapter). Make sure to avoid connections with any of the on-board ports (compare to „Figure 2: Electromagnetic Navigation Unit (Back View)“). For an optimal display of the image, we recommend a monitor with a resolution of at least 1920 x 1080 pixels and a screen size of at least 27 inches.

WARNING

Only connect monitors suitable for the operation room in accordance with the safety standards IEC 62368-1 or IEC 60601-1.

5.5. Connecting the field generator

Connect the plug of the Field Generator to the socket on the front of the unit.

CAUTION

- Never use force to put the plug into the plug socket. Do not bend or strain the cable.
 - When connecting the Field Generator or electromagnetic instruments to the plug sockets, ensure that the plugs click into place.
-

5.6. Connecting the power cords

i Note: The potential equalization cable provides a connection between the medical electrical device and a potential equalization bus bar. It serves to prevent or compensate potential differences between bodies of electrical equipment and built-in conductive parts in the patient environment.

WARNING

- Before connecting the power cord to a power outlet, make sure that the correct voltage (100-240 V~ /50/60 Hz) is provided and that the cords are clean, dry, and undamaged.
 - Only use the supplied power cords or cords that comply with the standards of the country in which you intend to operate the product and that are no longer than 3 m and have a cross-section of at least 1 mm².
 - To prevent the risk of electric shock, only connect this product to a power supply with a protective conductor. Check plugs and cables routinely and do not use if damaged.
-
1. Ensure that the product and any other electrical devices used in combination with the product are connected to the potential equalization (equipotential), i.e., through protective earth conductors.
 2. Connect the device to the power grid with the supplied power cord.

WARNING

When using a multi-outlet power strip to connect the device to the power grid, make sure that the multi-outlet power strip conforms to the requirements of IEC 60601-1.

To completely detach the device from the power grid, make sure to disconnect the power cord from the power outlet or detach the device from the power grid in another appropriate manner.

Note: Make sure that the Electromagnetic Navigation Unit and all installed electrical devices used in combination with it are installed in such a way that it is easy to disconnect them from the power grid.

5.7. Combination with other medical products

WARNING

- Intra-operative live images processed by the system may be displayed with a delay. During applications that require real-time critical imaging, a second monitor must be used. This monitor must be directly connected to the real-time critical image source.
 - The medical electrical devices used in combination must comply with the safety standard IEC 60601-1. When connecting additional devices to the signal in- and outputs, you are configuring a medical system and will be responsible for making sure that the system meets the requirements of the system standard IEC 60601-1.
 - When combining medical products, safety is only ensured if the combination is specified in the respective instructions for use as a safe combination, or the specific functions and the interface specifications of the combined medical products are suited for combination (compare IEC 60601-1).
 - If the camera unit is not directly connected to a second monitor, the camera processor must be directly connected to the monitor by using a second image signal cable. In the event of product failure, this makes it possible to regain the visibility to the operational area as quickly as possible.
-

A connected camera unit for an endoscope system must comply with IEC 60601-1 and provide at least 2 MOPP when connected to the camera system.

When connecting a light source for the endoscope system, ensure that the light source fulfills IEC 60601-1 and provides at least 2 MOPP towards the patient.

All combined devices must comply with the electrical safety standard IEC 60601-1.

5.8. Combination with the equipment cart

WARNING

- Do not simultaneously touch the patient and any parts of the system — e.g. the Electromagnetic Navigation Unit, cart, monitors — except the Electromagnetic Instruments. Non-observance of this rule may lead to dangerous electrical currents that may harm the patient or the user.
- When using a supporting system (e.g. Cart), always connect and power all other devices within the system (e.g. Electromagnetic Navigation Unit, Monitor) via the supporting system. Do not connect these other devices directly to wall sockets when using a supporting system.

CAUTION

- When moving the cart with its components, the monitor must be turned 90° sideways.
- The cart with all components may only be slowly pulled on the handle in backwards direction. When pulling the cart over any obstacles or bumps, including cables or hoses, reduce the speed of movement significantly.

5.9. Software installation

Note:

- Please note there are multiple software variants in different versions that can be installed on the Electromagnetic Navigation Unit. Check the variant and version before starting installation procedures.

To ensure correct operation of the medical device, it is necessary to carry out only controlled changes to the system during the service life. A corresponding process as defined in IEC 80001 must be used and access to the system must be restricted by password protection. In particular, only allow access to the computer on secured networks.

WARNING

- Do not install any third party software on the computer without consulting Stryker.
- For safety reasons, the Scopis Navigation Software must only be installed and configured by Stryker employees.

5.10. Initial operation

i Note: Before first use of the system, make sure that the Electromagnetic Navigation Unit and all other system components are connected correctly.

1. Turn on the Scopis Navigation Software by pushing the power button located on the front of the unit.

i Note:

- Before the first use of the Scopis Navigation Software, refer to the instructions for use provided with the software.
 - If you are asked for a serial number during the software's initial operation, contact Stryker, referencing the shown system-ID.
-

2. Start the Scopis Navigation Software via the shortcut on the Windows desktop.

i Note: The software conducts a fully automated compatibility inspection. In the case of incompatibilities, contact Stryker.

3. Click on the application logo in the upper left corner. The software's *About* dialog will open. Make sure that the reference number corresponds with the number listed in the application documentation provided with the software.

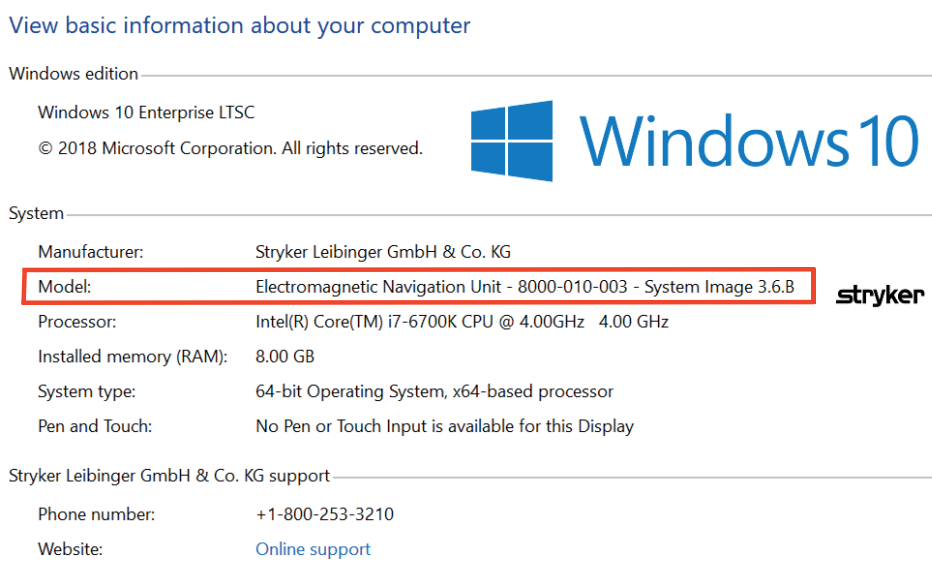
Figure 3: Open About Dialog

6. Using your product

For instructions on how to use the product for a surgical procedure, refer to the instructions for use supplied with the respective software application.

6.1. Identifying the operating system image version

To identify the operating system image version type *Control Panel* in the Windows search box, select the *Control Panel* app, click *System and Security* ⇒ *System* and check the *Model* entry in the *System* compartment.

View basic information about your computer

Windows edition

Windows 10 Enterprise LTSC
© 2018 Microsoft Corporation. All rights reserved.

System

Manufacturer:	Stryker Leibinger GmbH & Co. KG
Model:	Electromagnetic Navigation Unit - 8000-010-003 - System Image 3.6.B
Processor:	Intel(R) Core(TM) i7-6700K CPU @ 4.00GHz 4.00 GHz
Installed memory (RAM):	8.00 GB
System type:	64-bit Operating System, x64-based processor
Pen and Touch:	No Pen or Touch Input is available for this Display

Stryker Leibinger GmbH & Co. KG support

Phone number:	+1-800-253-3210
Website:	Online support

6.2. Operating system update

Note:

- This section applies to operating system image version 3.6.A or higher only.
- For operating system image version 3.3.D, operating system updates are not available.

The operating system of the Stryker ENT Navigation System has the capability to download and install operating system updates (internet connection required).

By default the installation of updates requires manual confirmation/action in order to ensure system availability during the use of the system.

Since operating system updates can have an impact on the system performance, it is strongly recommended to perform a system test after the installation of operating system updates.

To ensure that the navigation system works normally after installing operating system updates the following steps are recommended:

Action	Expected result
1. Reboot the system	System boots normally to Windows desktop or login screen.
2. Start the navigation software	Software starts without a warning.
3. Load existing planning or import dataset	Patient data is visualized in 2D and 3D.
4. Switch to navigation mode (Field generator connected)	No error dialog shown. The Navigation tab is pulsing smoothly without stutter or lagging.
5. Connect a video source (e.g. Endoscope), if available	Video view shows the input video signal.

WARNING

- **[For operating system image version 3.6.A or higher]** User is solely responsible for ensuring timely installation of Operating System updates. User failure to install system updates may increase risk of vulnerabilities.
- **[For operating system image version 3.6.A or higher]** Operating system updates can have an impact on the navigation system performance. Check the performance of the navigation system after each operating system update according to the steps described above.

7. Reprocessing

CAUTION

- Pay attention to the instructions provided by the cleaning solution and disinfectant manufacturer.
 - For cleaning, only use soft, clean cloths and/or swabs that are suitable for the purpose. Never use any sharp objects such as metal brushes or wire wool.
 - Do not sterilize the product. Failure to comply may lead to product damage.
-

Reprocess devices within two hours after use.

Reprocess devices before you return devices for service.

WARNING

- Before cleaning and maintenance, unplug the device from the power supply and other devices connected to the mains.
 - The product requires at minimum an intermediate level of disinfection for safe use. Before first use, as well as before each following use, the product must be cleaned and disinfected according to a validated procedure.
 - Do not expose or immerse the electrical components of the navigation system in liquids. Do not place liquids on or above the devices. Ingressing liquids may result in equipment damage, produce a fire or shock hazard.
 - Only use products that are suited for wipe disinfection of medical devices.
-

Necessary steps in the reprocessing of medical products include preparation, cleaning, disinfection, testing, maintenance, inspection, and storage. Details on how to carry out these steps for the listed products will be provided below. Also consult the CDC Guidance Document Guideline for Disinfection and Sterilization in Healthcare Facilities, 2008. Also refer to the following for the purpose of additional education but not in lieu of validated reprocessing instructions: professional organizations' clinical practice guidelines or clinical guidelines of the CDC.

Personal safety

Use the following personal protective equipment:

- (single-use) shirt
- (single-use) gloves
- safety glasses
- anti-splash mask

Refer to the user instructions of the cleaning agent and disinfectant for additional protective equipment.

7.1. Manual cleaning and disinfection

Note: The following manual cleaning and disinfection process has been validated by Stryker. If other reprocessing methods are used, make sure the reprocessing cycle is validated.

Required equipment:

- Low-lint cloth, swab, cotton swabs
- CaviWipes wipes from Metrex Research
- Freshly prepared purified water, highly purified water or sterile water with less than 10 cfu/ml and 0.25 EU/ml. In the U.S., refer to AAMI TIR 34 for guidance on water quality
- Medical grade compressed air

7.1.1. Prepare the equipment and the devices

1. Disconnect the power supply cable of the Electromagnetic Navigation Unit.
2. Disconnect all devices from the Electromagnetic Navigation Unit.

7.1.2. Cleaning

1. Use a new clean CaviWipes wipe to wipe the surfaces.
2. Pay attention to the power supply cable and the hard to reach areas of the device.
3. Use a new clean CaviWipes wipe when the wipe is contaminated.
4. Clean until all visible dirt is removed.
5. Follow also the instructions for use of the manufacturer cleaning aid.

7.1.3. Disinfection

1. Using a new clean CaviWipes wipe, wipe all surfaces.
2. Pay special attention to the power supply cable and the hard to reach areas of the device.
3. Make sure the surfaces remain visibly wet at room temperature for at least 3 minutes. Follow also the instructions for use supplied by the disinfectant manufacturer.
4. Use a new clean CaviWipes wipe, when the wipe is contaminated.
5. Remove any excess disinfectant solution using a soft, lint-free cloth slightly moistened with purified water.

7.1.4. Drying and inspection

1. Dry the external surface by letting it dry on a low-lint cloth, wipe it with a low-lint cloth until the external surfaces are dry or use carefully compressed air.
2. Pay special attention regarding the dryness to the plug of the power supply cable and the hard to reach areas of the device.
3. Visually inspect the equipment for any remaining dirt or disinfectant solution. If dirt or disinfectant solution remains, repeat the cleaning and disinfection procedure using fresh CaviWipes wipes.
4. Check device functionality and examine the product for:
 - Loose parts
 - Surface damages
 - Cable damages
 - Other damages

8. Maintenance

WARNING

Do not service the product. It does not contain any parts the user can service. If service is required, contact your Stryker sales representative.

Maintenance and repair may only be conducted by the manufacturer or authorized partners. The product components may only be sent back to the manufacturer in a cleaned and disinfected condition.

8.1. Routine checkups

Before each use, check that all necessary maintenance work is carried out in accordance with local policy and procedure. Before use, check the device functionality and inspect the product components for outer defects.

Before each use, check the device for:

- Correct assembly and function
- Cracks
- Surface damage
- Loose parts
- Other damage

Routinely check the product's supply cable and connecting cable for damages. Order spare parts if necessary.

8.2. Changing the fuses

1. Disconnect the power cord from the device.
2. Release the fuse box by pushing in the two slits on the side, and pull out the fuse box.

Figure 4: Opening the Fuse Box

3. Remove the fuse holder by pulling outward.

Figure 5: Removing the Fuse Holder

4. Replace the two fuses used in the holder. For fuse rating, refer to Section „Technical specifications“.

Figure 6: Positioning the Fuses

5. Move the fuse holder back until it latches into place.
6. Reconnect the power cord. Switch on the device and conduct a functional test.

WARNING

Only use fuses which are listed in Section „Technical specifications“. Use fuses with values that are listed on the Name Plate.

9. Storage and transport

Transport and shipping can lead to product damage. If possible, use the original packaging for packing the product.

In case this is not available, pad all empty spaces carefully (to the sides, above, and below). Please also pad other boxes, in order to prevent product damage during transport.

For storage and transportation environmental limitations, refer to Section „Technical specifications“.

9.1. Disposal

Note:

- In accordance with European Directive 2012/19/EU on Waste Electrical and Electronic Equipment (WEEE) product should be collected separately for recycling. Do not dispose of as unsorted municipal waste. Contact local distributor for disposal information. Ensure infected equipment is decontaminated prior to recycling.
- The Batteries Directive 2006/66/EC introduces new requirements from September 2008 on removability of batteries from waste equipment in EU Member States. To comply with this Directive, this device has been designed for safe removal of the batteries at end-of-life by a waste treatment facility. Infected units should be decontaminated before they are sent for recycling. In the case that it is not possible to decontaminate the unit for recycling, the hospital should not attempt to remove the batteries from waste equipment. Continued disposal of small amounts of portable batteries to landfill and incineration is allowed under the Batteries Directive 2006/66/EC and Member State regulations.
- Perchlorate Material - special handling may apply.
See www.dtsc.ca.gov/perchlorate/.
- Battery report information: 1.) 3.0 gram 2.) CR2032 3.) Lithium-Manganese Dioxide
- The plug sockets and the sheet metals used in this product contain the following substance: Lead, CAS No. 7439-92-1 (1907/2006 REACH). Handling Instructions: No specific precautions are required for handling items manufactured from alloys containing lead in the supplied condition.
- The printed circuit board used in this product contains a crystal and a filter, which contains the following substance: Diboron Trioxide; CAS No. 1303-86-2 (1907/2006 REACH). It may contain Lead, CAS No. 7439-92-1 in resistors above 0.1%w/w(1907/2006 REACH).

After the end of the product's lifetime, it should be disposed of according to country-specific requirements and laws.

Any risk of injury or infection is to be avoided upon disposal of product components. Sharp and pointed components need to be collected and locked in a tight and break-proof container. They must be stored so that they are protected from unauthorized use. Contaminated products are to be supplied to hazardous waste disposal. If you have any questions, contact Stryker.

CAUTION

Before sending the Electromagnetic Navigation Unit to the manufacturer, remove any private patient data so that they are not shared with Stryker. Upon final disposal of the device, ensure stored data is deleted securely and in accordance with regulations.

10. Technical specifications

Specifications	
Model	Electromagnetic Navigation Unit (REF 8000-010-003)
Dimensions	<ul style="list-style-type: none">– Size: 350 mm x 210 mm x 368 mm (W x H x D)– Weight: 11.3 kg
Electrical Data	<ul style="list-style-type: none">– Power supply: 100-240 VAC, 50/60 Hz– Power input: Maximum 300 VA– Fuse: T 3, 15A H / 250
Fulfilled Electrical Safety Standards	<p>IEC 60601-1 (Medical electrical equipment - Part 1: General requirements for basic safety and essential performance)</p> <p>IEC 60601-1-2 (Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance – Collateral standard: Electromagnetic compatibility)</p>
Inputs and Outputs (front of unit)	<ul style="list-style-type: none">– USB 3.0 interfaces: 2x– Field Generator Interface: 1x– Electromagnetic Instrument Interface: 4x

Specifications

Inputs and Outputs (front of unit)	<ul style="list-style-type: none"> - USB 3.0 Interfaces: 4x - USB 2.0 Interfaces: 2x - USB-C: 1x ** - Line in: 1x - Line out: 1x - Microphone in: 1x - Center/Subwoofer Speaker Out: 1x - Rear Speaker Out Jack: 1x - SDI- Video In: 1x - Power Supply: 1x - D-Sub Port Onboard: 1x - DB9 In: 1x - Display Port: 3x ** - DVI-I In: 1x - HDMI-Out Onboard: 1x - S/PDIF Out: 1x - LAN-Port: 1x - Keyboard Port: 1x - Mouse Port: 1x - Equipotential Connection: 1x
------------------------------------	--

** Ports on the graphic cards output may vary between Electromagnetic Navigation Units.

Environmental Limitations	Operation	Storage and Transportation
Temperature	Between 10 °C and 30 °C	Between -10 °C and +50 °C
Relative Humidity	30 % to 75 %	10 % to 90 %, non-condensing
Atmospheric Air Pressure	Between 80 kPa and 106 kPa	Between 50 kPa and 106 kPa

10.1. Cyber Security Information

The Electromagnetic Navigation Unit is a computer system using the Microsoft Windows 10 operation system. It stores patient image data in a protected operating room environment with optional network connectivity.

User management and credentials:The system does not boot from external sources.

Access to BIOS configuration is password protected, so that this setting cannot be changed.

- There is only a reduced set of dedicated native users: one regular system user and two users with administrator privileges.
- Remote access to the system is disabled.
- Auditing is done via the Windows logging system.

Encryption:

- **[For operating system image version 3.6.A or higher]** The file system is encrypted via Microsoft BitLocker.

Network:

- A firewall is active, with rules to block unsolicited network traffic.

For more detailed information on security capabilities, you may request a MDS² document from your sales representative.

10.2. Essential Performance Features

The Essential Performance Features are described in the application documentation provided with the software.

11. Electromagnetic compatibility

Medical devices must comply with specific safety precautions regarding electromagnetic compatibility. For information specifically related to the electromagnetic field, refer to the instructions for use supplied with the Field Generator (REF 8000-010-004). Refer to the User Manual and the Safety Information supplied with the software application for information on the correct system setup and on how to avoid disturbances of the electromagnetic field.

WARNING

- The use of accessories and cables other than those specified by Stryker can result in increased emissions or decreased immunity of the system.
- The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If used in a residential environment, this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.

Note: Intended Environment is Special Healthcare Environment.

Guidance and Manufacturer's Declaration - Electromagnetic Emission

The Electromagnetic Navigation Unit is intended for use in the electromagnetic environment specified below. The customer or user of the product must ensure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic Environment -Guidance
RF emissions CISPR 11	Group 1	The product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause interference in nearby electronic equipment.

Guidance and Manufacturer's Declaration - Electromagnetic Emission

RF emissions CISPR 11	Class A	<p>The product is suitable for use in other establishments, such as domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes, provided that the following warning is fully considered:</p> <div style="background-color: #cccccc; padding: 5px; margin: 10px 0;"> WARNING </div> <p>The product is solely intended to be used by medical staff. Within domestic conditions it can cause radio interferences, which can demand taking corrective action, for example, new orientation, new arrangement or shielding of the product, or filtering of the connection to the location.</p>
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Compliant	

Guidance and Manufacturer's Declaration - Electromagnetic Immunity

The Electromagnetic Navigation Unit is intended for use in the electromagnetic environment specified below. The customer or user of the product must ensure that it is used in such an environment.

Immunity Test	EC 60601 Test Level	Compliance Level	Electromagnetic environment guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 15 kV air	± 8 kV contact ± 15 kV air	
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for camera cable	Main power quality should be that of a typical commercial or hospital environment.

Guidance and Manufacturer's Declaration - Electromagnetic Immunity			
Surge IEC 61000-4-5	± 1 kV differential mode ± 2kV common mode	± 1 kV differential mode ± 2kV common mode	Main power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions, and voltage variations on power supply input lines IEC 61000-4-11	0% UT for 0.5 cycle 0% UT for 1 cycle 70 % UT for 5 cycles 70% UT for 25/30 cycles 0% UT for 250/300 cycles	0% UT for 0.5 cycle 0% UT for 1 cycle 70 % UT for 5 cycles 70% UT for 25/30 cycles 0% UT for 250/300 cycles	Main power quality should be that of a typical commercial or hospital environment. If the user of the product requires continued operation during power mains interruptions, it is recommended that the product be powered from an uninterruptible power supply or a battery.

Guidance and Manufacturer's Declaration - Electromagnetic Immunity			
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Guidance and Manufacturer's Declaration - Electromagnetic Immunity

<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>150 kHz to 80 MHz</p> <p>3 Vrms outside the ISM band</p> <p>6 Vrms in the ISM band</p> <p>3 V/m,</p> <p>80 MHz to 2.7 GHz</p> <p>27 V/m, 380 to 390 MHz</p> <p>28 V/m, 430 to 470 MHz</p> <p>9 V/m, 704 to 787 MHz</p> <p>28 V/m, 800 to 960 MHz</p> <p>28 V/m, 1700 to 1990 MHz</p> <p>28 V/m, 2400 to 2570 MHz</p> <p>9 V/m, 5100 to 5800 MHz</p>	<p>150 kHz to 80 MHz</p> <p>3 Vrms outside the ISM band</p> <p>6 Vrms in the ISM band</p> <p>3 V/m,</p> <p>80 MHz to 2.7 GHz</p> <p>27 V/m, 380 to 390 MHz</p> <p>28 V/m, 430 to 470 MHz</p> <p>9 V/m, 704 to 787 MHz</p> <p>28 V/m, 800 to 960 MHz</p> <p>28 V/m, 1700 to 1990 MHz</p> <p>28 V/m, 2400 to 2570 MHz</p> <p>9 V/m, 5100 to 5800 MHz</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> <p>$d = 1.17 * \sqrt{P}$ for 150 kHz to 80 MHz</p> <p>$d = 1.17 * \sqrt{P}$ for 80 MHz to 800 MHz</p> <p>$d = 2.33 * \sqrt{P}$ for 800 MHz to 2.5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b. Interference may occur in the vicinity of equipment marked with the following symbol:</p> <div style="text-align: center;">  </div>
--	--	--	--

i Note:

- At 80 MHz and 800 MHz, the higher frequency range applies.
- These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

^aField strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be theoretically predicted with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey must be considered. If the measured field strength in the location in which the product is used exceeds the applicable RF compliance level above, the product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the product.

^bOver the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended Separation Distances Between Portable and Mobile RF Communications Equipment and the Product

The Electromagnetic Navigation Unit is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The user can help prevent electromagnetic disturbance by maintaining a minimum distance between portable and mobile RF communications equipment and the unit.

Separation Distance d [m] According to Frequency of Transmitter

Rated maximum output power of transmitter P (W)	150 kHz to 80 MHz $d = 1.17 * \sqrt{P}$	80 MHz to 800 MHz $d = 1.17 * \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.33 * \sqrt{P}$
0.01	0.12	0.12	0.24
0.1	0.37	0.37	0.74
1	1.17	1.17	2.34
10	3.69	3.69	7.38
100	11.67	11.67	23.34

For transmitters rated at an output power not listed, the recommended separation distance (d) in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note:

- At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.
- These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

The compliance levels in the ISM frequency bands between 150 kHz and 80 MHz and in the frequency range from 80 MHz and 2.5 GHz are intended to decrease the likelihood that mobile/portable communications equipment could cause interference and 2.5 GHz are intended to decrease the likelihood that mobile/portable communications equipment could cause interference if it is inadvertently brought into patient areas. For this reason, the additional factor of 10/3 is used when calculating the recommended safe distances within these frequency ranges.

These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

12. Minimum requirements for additional components

WARNING

When using third-party products, carefully read and follow the instructions provided with those products.

12.1. Devices

Endoscope camera unit:

- Outputs: DVI / SDI / HD-SDI / 3G-SDI / S-VIDEO / VGA
- Resolution: minimum PAL 768 x 572 - maximum 1080p60
- Endoscopic image delay in combination with 60 Hz medical monitor: maximum 50 ms
- Camera unit according to IEC 60601-1 and provides 2 MOPP to the camera
- Lightsource according to IEC 60601-1 and provides 2 MOPP towards the patient

Power Cords of the equipment cart pro:

Equipment cart includes four shelves and a storage drawer.

- 8000-030-003 Pro Equipment Cart power cord US
- 8000-030-004 Pro Equipment Cart power cord Schuko
- 8000-030-005 Pro Equipment Cart power cord UK
- 8000-030-006 Pro Equipment Cart power cord CH

Camera head:

- Resolution: PAL (752(H) x 582(V)) or higher

Endoscopes:

Rigid endoscopes:

- For navigating rigid endoscopes of diameter 4 mm, length of 180 mm and default bayonet connector, the use of the Endoscope Tracker Electromagnetic (REF 8000-060-001) is recommended. Refer to Instructions for Use of Instrument Clamps and Instructions for Use of Instrument Clamps Electromagnetic for specifications of more supported diameters.
- Straight-forward or Forward-oblique optics with 0° / 30° / 45° between shaft axis and viewing direction

Medical monitors:

- Inputs: DVI or HDMI
- Resolution: 1920 x 1080 recommended
- Colors: 16.7 million colors
- Maximum delay of 50 ms in combination with an endoscope camera unit
- Medical grade
- Fulfills IEC 62368-1 or IEC 60601-1

The device is compatible with the following monitors:

- 240-031-020 VisionPro
- 240-031-050 4K Display

Keyboard/mouse (medical or hygiene grade):

- Connector: USB 2.0 or higher
- IBM Compatible

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The screenshot shows the Windows 'System' information page. The 'Model' field is highlighted with a red box and contains the text 'Electromagnetic Navigation Unit - 8000-010-003 - System Image 3.6.B'. The 'Manufacturer' is 'Stryker Leibinger GmbH & Co. KG'. The 'Processor' is 'Intel(R) Core(TM) i7-6700K CPU @ 4.00GHz 4.00 GHz'. The 'Installed memory (RAM)' is '8.00 GB'. The 'System type' is '64-bit Operating System, x64-based processor'. The 'Pen and Touch' status is 'No Pen or Touch Input is available for this Display'. The 'Stryker Leibinger GmbH & Co. KG support' section includes a phone number '+1-800-253-3210' and a website link 'Online support'. The Windows logo and 'Windows 10' text are visible in the background.

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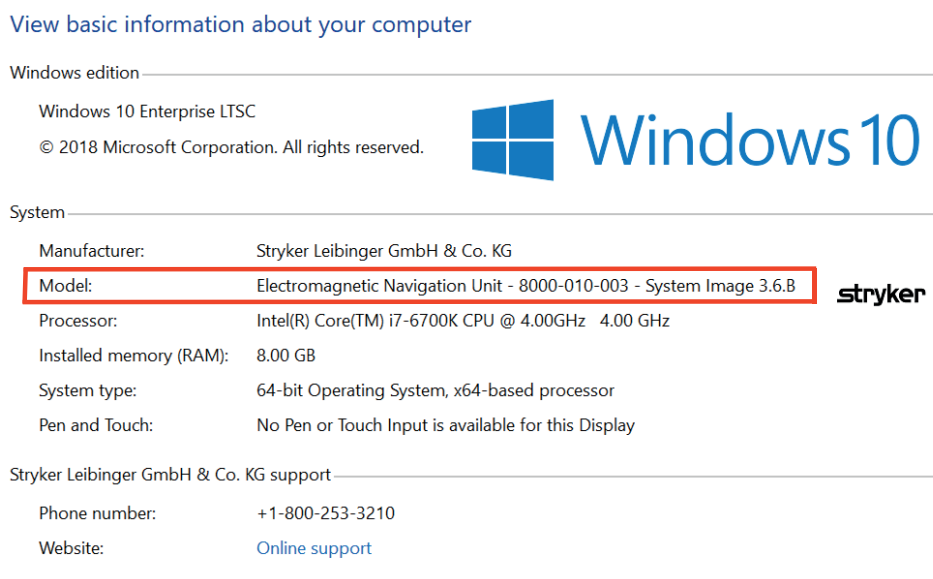
The screenshot shows the Windows System information page. The 'Model' field is highlighted with a red box and contains the text 'Electromagnetic Navigation Unit - 8000-010-003 - System Image 3.6.B'. The 'stryker' logo is visible to the right of the system information table.

View basic information about your computer	
Windows edition	
Windows 10 Enterprise LTSC	
© 2018 Microsoft Corporation. All rights reserved.	
System	
Manufacturer:	Stryker Leibinger GmbH & Co. KG
Model:	Electromagnetic Navigation Unit - 8000-010-003 - System Image 3.6.B
Processor:	Intel(R) Core(TM) i7-6700K CPU @ 4.00GHz 4.00 GHz
Installed memory (RAM):	8.00 GB
System type:	64-bit Operating System, x64-based processor
Pen and Touch:	No Pen or Touch Input is available for this Display
Stryker Leibinger GmbH & Co. KG support	
Phone number:	+1-800-253-3210
Website:	Online support

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Stryker Corporation or its affiliates own, use, or have applied for the following trademarks or service marks: Leibinger, Scopis, Stryker, TGS. All other trademarks are trademarks of their respective owners or holders.
Copyright © 2022 Stryker