# Stryker ENT Navigation System

Scopis<sup>®</sup> ENT Software with TGS<sup>®</sup>

REF 8000-020-002

Software Version 3.4.x

Instructions for Use

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CE 0197

## Table of Contents

1.	How to Use this Document.....	1
	<b>1.1. About this Document</b> .....	<b>1</b>
	<b>1.2. Contact Information</b> .....	<b>1</b>
	<b>1.3. Symbol Definition</b> .....	<b>1</b>
2.	Safety Information.....	5
	<b>2.1. User Qualifications</b> .....	<b>5</b>
	<b>2.2. Cyber Security and Patient Data Privacy</b> .....	<b>5</b>
3.	Product Information.....	8
	<b>3.1. Intended Use</b> .....	<b>8</b>
	<b>3.2. Indications for Use</b> .....	<b>8</b>
	<b>3.3. Contraindications</b> .....	<b>8</b>
4.	Product Overview .....	9
	<b>4.1. For Use with</b> .....	<b>9</b>
5.	Getting Started.....	10
	<b>5.1. Unpacking</b> .....	<b>10</b>
	<b>5.2. Software Installation</b> .....	<b>10</b>
	<b>5.3. Product Startup</b> .....	<b>10</b>
6.	Using your Product.....	12
	<b>6.1. Instructions for Application</b> .....	<b>12</b>
7.	Maintenance.....	13
8.	Disposal.....	14
9.	Technical Specifications.....	15

10. Troubleshooting and Analysis ..... 19

11. Imaging Modalities ..... 20

12. Keyboard Shortcuts..... 21

# 1. How to Use this Document

## 1.1. About this Document

This manual is the most comprehensive source of information for the safe, effective, and compliant use and/or maintenance of the product. Read and understand this manual as well as the respective system user manual before using the product or any component compatible with the product. When combined with other medical devices, the user manual of these devices is to be considered as well. Contact Stryker for training as needed.

This manual is a permanent part of the product. Keep this manual for future reference.

The following signal words may be used throughout this manual:

	<b>WARNING</b>
-----------------------------------------------------------------------------------	----------------

Highlights a safety-related issue. Always comply with this information to prevent patient and/or healthcare staff injury.

<b>CAUTION</b>
----------------

Highlights a product reliability issue. Always comply with this information to prevent product damage.

## 1.2. Contact Information

	Stryker Representative	
	+49 761 4512 0 (Germany)	+1 269 323 7700 (USA)
	<a href="http://www.stryker.com">www.stryker.com</a>	

## 1.3. Symbol Definition

EN ISO 7010 Graphical symbols – Safety colors and safety signs – Registered safety signs

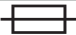
Symbol	Name: Definition
 W001	<b>General warning sign:</b> To signify a general warning.
 M002	<b>Refer to instruction manual/booklet:</b> To signify that the user instruction manual/booklet must be read.
 P007	<b>No access for people with active implanted cardiac devices:</b> To prohibit people with active implanted cardiac devices from entering a designated area.

EN ISO 15223-1 Medical devices — Symbols to be used with medical device labels, labeling and information to be supplied - Part 1 General requirements

Symbol/ number	Name: Definition
 5.1.1	<b>Manufacturer:</b> indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.
 5.1.3	<b>Date of manufacture:</b> Indicates the date when the medical device was manufactured.
 5.1.4	<b>Use-by date:</b> Indicates the date after which the medical device is not to be used.
 5.1.5	<b>Batch code:</b> Indicates the manufacturer's batch code so that the batch or lot can be identified.
 5.1.6	<b>Catalog number:</b> indicates the manufacturer's catalog number so that the medical device can be identified.
 5.1.7	<b>Serial number:</b> Indicates the manufacturer's serial number so that the medical device can be identified.
 5.2.7	<b>Non-Sterile:</b> indicates a medical device that has not been subjected to a sterilization process.
 5.3.1	<b>Fragile, handle with care:</b> Indicates a medical device that can be broken or damaged if not handled carefully.
 5.3.2	<b>Keep away from sunlight:</b> Indicates a medical device that needs protection from light sources.
 5.3.4	<b>Keep dry:</b> Indicates a medical device that needs to be protected from moisture.
 5.3.7	<b>Temperature limit:</b> Indicates the temperature limits to which the medical device can be safely exposed.
 5.3.8	<b>Humidity limitation:</b> Indicates the range of humidity to which the medical device can be safely exposed.
 5.3.9	<b>Atmospheric pressure limit:</b> Indicates the range of atmospheric pressure to which the medical device can be safely exposed.
 5.4.2	<b>Do not re-use:</b> Indicates a medical device that is intended for one use or for use on a single patient during a single procedure.
 5.4.3	<b>Consult instructions for use:</b> indicates the need for the use to consult the instructions for use.
 5.4.4	<b>Caution:</b> Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.

IEC 60417 Graphical symbols for use on equipment

Symbol	Name: Definition
 0623	<b>This way up:</b> To indicate correct upright position of the transport package.

Symbol	Name: Definition
 0630	<b>Stacking limit by mass:</b> To indicate that the items shall not be vertically stacked beyond the specified number, either because of the nature of the transport packaging or because of the nature of the items themselves.
 2403	<b>Stacking limit by number:</b> To indicate that the items shall not be vertically stacked beyond the specified number, either because of the nature of the transport packaging or because of the nature of the items themselves.
 5009	<b>Stand-by:</b> To identify the switch position by means of which part of the equipment is switched on in order to bring it into the stand-by condition, and to identify the control to shift to or to indicate the state of low power consumption. Each of different states of power consumption may be indicated using a corresponding colour.
 5016	<b>Fuse:</b> To identify fuse boxes or their location.
 5017	<b>Earth; ground:</b> To identify an earth (ground) terminal in cases where neither the symbol 5018 nor 5010 is explicitly required.
 5019	<b>Protective earth; protective ground:</b> To identify any terminal that is intended for connection to an external conductor for protection against electric shock in case of a fault, or the terminal of a protective earth (ground) electrode.
 5021	<b>Equipotentiality:</b> To identify the terminals that, when connected together, bring the various parts of an equipment or of a system to the same potential, not necessarily being the earth (ground) potential, e.g. for local bonding.
 5032	<b>Alternating current:</b> To indicate on the rating plate that the equipment is suitable for alternating current only; to identify relevant terminals.
 5134	<b>Electrostatic sensitive devices:</b> To indicate packages containing electrostatic sensitive devices, or to identify a device or a connector that has not been tested for immunity to electrostatic discharge.
 5140	<b>Non-ionizing electromagnetic radiation:</b> To indicate generally elevated, potentially hazardous, levels of non-ionizing radiation, or to indicate equipment or systems e.g. in the medical electrical area that include RF transmitters or that intentionally apply RF electromagnetic energy for diagnosis or treatment.
 5333	<b>Type BF applied part:</b> To identify a type BF applied part complying with IEC 60601-1.

Product-Specific Symbols

Symbol	Name: Definition
 QTY	<b>Quantity:</b> Indicates the number of medical devices in the packaging.
 i	<b>Note symbol:</b> it is used to supplement or clarify information.
<b>GTIN</b>	Global Trade Item Number

81 FR 38911 FDA Final rule for the use of symbols in labeling

Symbol	Name: Definition
<b>Rx Only</b>	<b>Caution:</b> Federal law (USA) restricts this device to sale by or on the order of a physician.

Directive 2012/19/EU on waste electrical and electronic equipment (WEEE)

Symbol	Definition
	Indicates that the product must be collected separately and must not be disposed of as unsorted municipal waste.

ASTM F2503-08 Standard practice for marking medical devices and other items for safety in the MR environment

Symbol	Name: Definition
	<b>MR Unsafe:</b> Indicates a product that is known to pose hazards in all MRI environments.

Regulatory marks and logos

Symbol	Definition
	CSA certified for Canada and USA
	Conformity with Annex I of Medical Device Directive 93/42/EEC.
	Conformity with Annex I of Medical Device Directive 93/42/EEC for Class Is, Im, IIa, IIb and III.

## 2. Safety Information

### 2.1. User Qualifications

#### WARNING

The Scopis ENT Software with TGS is intended solely for use by medical professionals and may only be used by physicians who have the corresponding qualifications and have received the necessary training. Enclosed information, such as this manual, cannot substitute for lacking medical expertise.

The user must adhere to the respective national requirements regarding protection of patient data. The navigation system may only be used by persons that have been authorized to do so. We recommend protecting the system from unauthorized use by means of a password.

Stryker does not assume liability for any diagnoses or interpretations created with this product. The user is the only one responsible for acquiring medical knowledge and for its diagnostic and therapeutic consequences.

To ensure the safe and efficient use of the product, the user must be instructed on how to use the product as authorized by the manufacturer and retailer. The medical product's operator must ensure that the user receives such instruction in accordance with the respective local laws, and that the instruction is refreshed at the prescribed intervals. Additionally, the instructions in all accompanying manuals must be respected.

During a surgery in connection with the navigation system, the patient has to be treated and observed with the usual medical care. This includes the follow-up to the treatment process, the monitoring of the vital signs and the state of anesthesia, as well as maintaining sterile application conditions, if required for a respective operation.

Aside from your own knowledge, a correct usage as well as regular cleaning and maintenance works are crucial for the operational safety and utility of the product.

In public clinics, it is necessary to inform the data protection supervisor about the system.

### 2.2. Cyber Security and Patient Data Privacy

The Scopis ENT Software with TGS (Target Guided Surgery) and the Electromagnetic Navigation Unit must be secured against cyber security attacks by you as the operator to ensure confidentiality of patient data, availability and integrity of the device for surgery. In particular, connecting the Electromagnetic Navigation Unit to a network via Ethernet, connecting USB media or allowing physical access to the device by third parties may result in cyber security being violated.

#### WARNING

- Connecting the navigation unit to a network or USB media can put the patient, the user, or third parties at risk. Your organization's risk management should determine, analyze, evaluate, and control these risks to avoid potential damages (see especially ISO 80001-1). Regarding this product, it is necessary to pay special attention to risks regarding the privacy of patient image data, system and data integrity, as well as system availability.
- Always keep the device in separate high-risk network. Track MAC addresses and only give known devices access to this network.
- If a cyber security attack has been detected, do not use the device until the attack has been responded to adequately and the device integrity has been recovered.

**CAUTION**

The Stryker ENT Navigation System and computers running Scopis ENT Software with TGS should only be used in physically protected areas such as the doctor's personal office or the operating room.

**i** Operators must adhere to the respective national requirements regarding protection of patient data (e.g. HIPAA). The navigation system may only be used by persons that have been authorized to do so. Ask your responsible authority for further information about the patient data security in your country. We recommend protecting the system from unauthorized use by means of a password.

It is recommended to only use encrypted USB sticks for transferring patient data from and to the navigation system.

Always use a validated user authentication and authorization scheme such as Windows domain logon to prevent unauthorized access to the navigation system.

Make sure that users have received adequate training on patient data privacy, are aware of cyber security issues and know about cyber security defenses.

Do use anti-virus software on the navigation system for on-access and full-scans at regular intervals.

Ensure timely installation of operating system security updates and application updates.

Changes to the system configuration including installation of updates should only be performed in a controlled manner. Before you make any changes to the system (e.g. installing anti-virus software) please make sure that you have set restore points.

To ensure confidentiality of patient data, availability and integrity of the device for surgery and protect from unauthorized access, unauthorized modification or interruption, Stryker suggests the following:

- Educate user and staff: Make sure that users have received adequate training on patient data privacy, are aware of cyber security issues such as phishing attacks or USB-based malware and know about cyber security defenses such as virus scanners and encryption.
- Ensure access control to the device: Install the navigation system/workstation at a location with physical access control to prevent theft and use authentication/authorization to prevent unauthorized download of patient data from the navigation system via media ports (USB / DVD). Also consider deactivating media ports or installing USB access control software when physical security is limited.
- Keep device in secure network: Keep the navigation system in an isolated network with only trusted and controlled devices. Restrict connections from and to the navigation system to manually cleared hosts.
- User authentication / authorization using passwords via hospital domain: Avoid unauthorized access to patient data by joining the navigation unit into the hospital domain and using per user authentication and authorization. Avoid that regular users do have administration rights on the navigation unit.
- Perform changes in a controlled manner and make use of restore points: All changes to the system configuration and environment such as installing updates and changing the network may result in new risks. Always perform a new analysis of the resulting risks. Create restore points at regular intervals and before making changes to the system's configuration.

- Ensure timely installation of operating system security updates and application updates: Keep the navigation system connected to a (secure) network with access to Microsoft Update servers or regularly install security updates. Follow security information from Stryker and install security updates in a timely manner.

In case a cyber security attack has been detected, contact Stryker for help with responding adequately and recovering. In any case, do not use a compromised navigation system.

The software comes pre-installed at system delivery. For verification of file integrity of installers and update files that were transmitted to you online you may request a file checksum from Stryker.

When saving plannings, make sure that the disk drive is protected from unauthorized use.

## 3. Product Information

### 3.1. Intended Use

The Scopis ENT Software with TGS (Target Guided Surgery) is an accessory to the Electromagnetic Navigation Unit and intended for controlling of the hardware of the navigation unit.

#### WARNING

- The product may only be used for its intended purpose and in accordance with the following manual, as well as the applicable, current version of the Stryker ENT Navigation System manual. The manual is part of the product and must therefore be accessible to personnel at all times. It must be handed over to subsequent owners or users.
- Do not use the product if there are medical or clinical reasons for not doing so. The product may only be used if the surgery could also be performed conventionally
- Only combine the navigation software with the compatible components listed in the For Use With section of this manual.
- Unauthorized modifications of the product are forbidden for safety reasons.

### 3.2. Indications for Use

The Stryker ENT Navigation System is indicated for any medical condition in which the use of stereotactic surgery may be appropriate, and where reference to a rigid anatomical structure in the field of ENT surgery, such as the paranasal sinuses, mastoid anatomy, can be identified relative to a CT- or MR-based model of the anatomy.

Example procedures include, but are not limited to the following ENT procedures:

- Transsphenoidal access procedures
- Intranasal procedures
- Sinus procedures, such as maxillary antrostomies, ethmoidectomies, sphenoidotomies/sphenoid explorations, turbinate resections, and frontal sinusotomies
- ENT-related anterior skull base procedures

### 3.3. Contraindications

None known.

## 4. Product Overview

### 4.1. For Use with

#### WARNING

The product may only be used in combination with the medical products listed here. All guaranties and warranties are lost if the product is combined with other products.

Please refer to the Electromagnetic Navigation Unit manual for a list of third-party equipment with information about minimum requirements or compatibilities.

Description	REF
Electromagnetic Navigation Unit	8000-010-003
Field Generator	8000-010-004
Field Generator Mounting Arm	8000-010-005
Patient Tracker Electromagnetic	8000-040-001
Patient Tracker Electromagnetic - 10 Uses	8000-040-002
Precision Pointer Electromagnetic	8000-050-001
Pointer Electromagnetic	8000-050-002
Registration Pointer Electromagnetic	8000-050-003
Suction Tube Frazier Electromagnetic	8000-050-005
Suction Tube Eicken Electromagnetic	8000-050-006
Navigation Tool Extension Cable	8000-050-011
Endoscope Tracker Electromagnetic	8000-060-001
Calibration Body Electromagnetic	8000-060-002
Universal Tracker Electromagnetic	8000-060-006
TGS Guidewire (US Only)	8000-060-009
Instrument Clamp, Forceps	8000-060-010
Instrument Clamp, 2-6 mm	8000-060-011
Instrument Clamp, 6-10 mm	8000-060-012
Instrument Clamp, 10-16 mm	8000-060-013
Instrument Clamp Electromagnetic Sphere (US Only)	8000-060-020
Instrument Clamp Electromagnetic Universal (US Only)	8000-060-021
Patient Tracker Tabs	8000-100-001

## 5. Getting Started

### 5.1. Unpacking

- Carefully remove the product components from the packaging.
- Check that the shipment is complete and undamaged. Please contact the delivery company immediately in case you have any reason for complaint.
- If possible, keep the original packaging; this can be useful when transporting the product at a later point.

### 5.2. Software Installation

Due to safety reasons and in order to ensure full compatibility, the Scopis ENT Software may only be installed by Stryker or one of its authorized partners.

### 5.3. Product Startup

#### WARNING

The product may only be operated with a valid manual. Make sure that the software version and part number correspond with the manual.

---

If you are asked for a serial number during the start-up of the software, please contact Stryker and have the displayed system-ID at hand.

---

In order to start the product, please proceed as follows:

1. After booting, start the software via the application icon on the Windows Desktop.
2. Upon first start, the software performs a fully automatic capability inspection. In the case of incompatibility, please contact Stryker.
3. Click on the application logo in the upper left corner of the screen. The version dialog of the software will open; see "About Dialog." Make sure that the software version corresponds to the system manual. For this to be the case, the first two digits in the software version (e.g. 1.9 in 1.9.0) and the manual version must be identical.

About Dialog

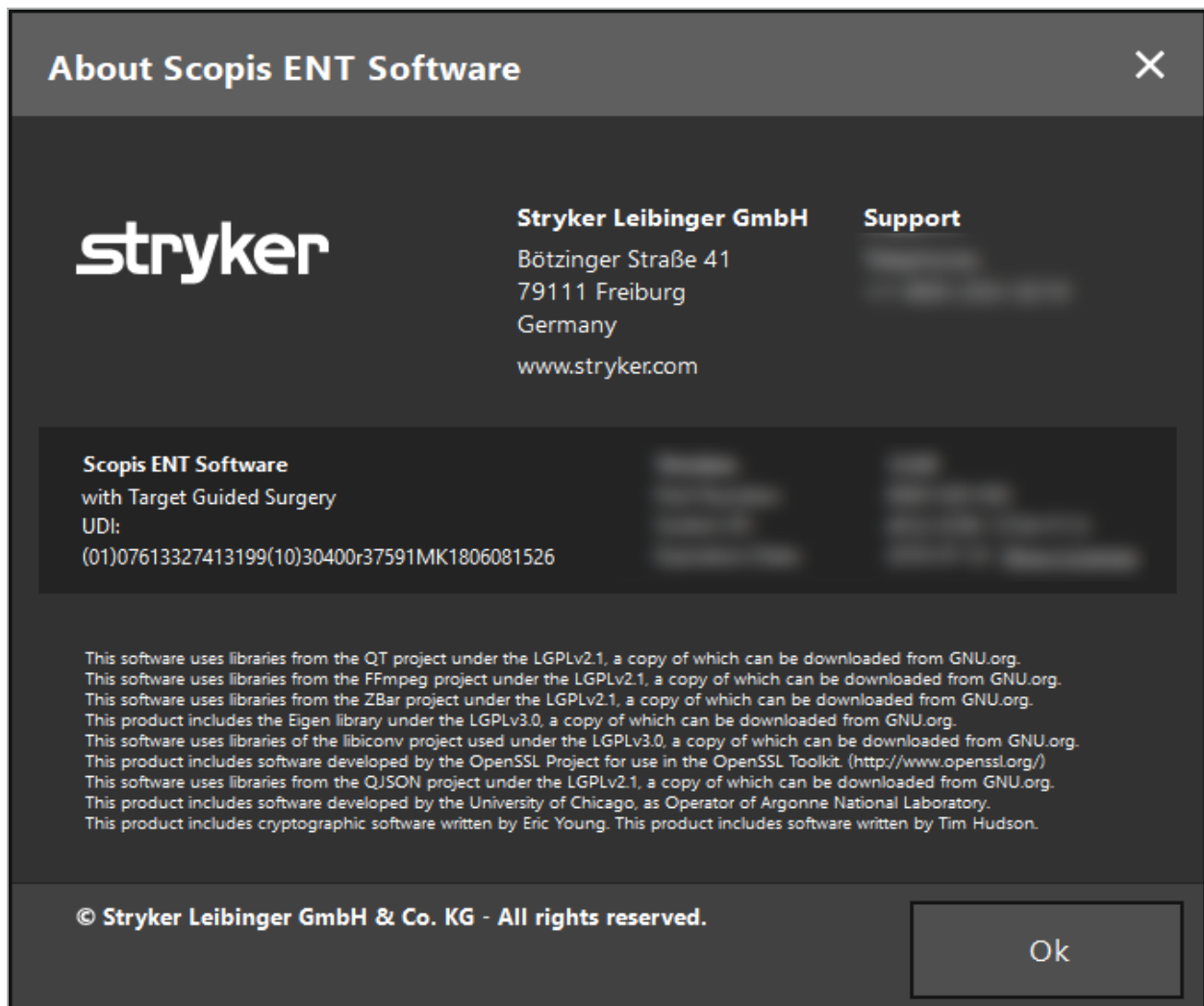
Figure 1: About Dialog

## 6. Using your Product

### **6.1. Instructions for Application**

For instructions and procedures of navigated surgery, refer to the Stryker ENT Navigation System with TGS instructions for use. Make sure that the software version corresponds with the Stryker ENT Navigation System manual.

## 7. Maintenance

**CAUTION**

Unauthorized modifications to the product will free Stryker from any liability for the equipment's operational safety.

---

If you have questions regarding the software, please contact Stryker.

All changes and/or repairs may only be conducted by staff that is authorized by the manufacturer and only original spare parts may be used.

Please refer to the article- and serial numbers when making enquiries or ordering spare parts.

Please arrange how to conduct maintenance work with your respective Stryker representative. Specific service agreements are available for maintenance.

Even if the operator does not conduct maintenance work himself, he is still responsible for ensuring that all required maintenance work is conducted before the product is used on the patient.

## 8. Disposal

After the service life of the product has expired, dispose of the product according to the country specific requirements/laws. When dismantling the computer, ensure that the remaining data is safely deleted. If you have any questions, please contact Stryker.

## 9. Technical Specifications

<b>Data Compatibility</b>	
Supported modalities	CT, MRI, CBCT (DICOM 3.0)
Slice thickness	0.25 mm - 2 mm
Pixel size	0.25 mm <sup>2</sup> - 0.8 mm <sup>2</sup>
Fusion of data sets	8 modalities
Element colors	12 colors
DICOM standard	DICOM 3.0
Supported source drives	CD-ROM, DVD-ROM, Hard Drive, USB
Supported target drives	Hard Drive, USB, CD/DVD
Data adjustment options	Contrast, brightness, surface threshold, DICOM trim, eye protection removal
<b>Image Processing Capabilities</b>	
Frame grabber support	MATROX MIL, Magewell Pro Capture, DirectShow, VRmagic, VC100
Standard grabber resolutions	HD (up to 1080p60), TV (NTSC/PAL), PC
Intraoperative image types	Endoscopy
<b>3D Reconstruction</b>	
3D reconstruction	Automatic, skin surface
Adaptive reconstruction	Interactive, real-time, threshold based
<b>Planning Capability</b>	
Planning availability	Preoperative (Planning Station / Navigation System), Intraoperative (during navigation)
Planning format	Scopis Planning XML 1.2
Planning elements	Points, Free Form Polygon Structures, Planning Paths, Cutting Lines, Bone Segmentations, Implants, Lines, Measurements
Properties for planning objects	Color, Label, Notification Distance (sound, visual)
Image injection	All planning objects into endoscopic video (selected endoscopic cameras)
<b>Patient Registration</b>	
Types of registration	Landmark, Surface, Enhanced Surface (live visualization of registration process)
Interruptability of registration	Yes, after collecting determining initial patient pose
Verification step	Yes, quality verification by user (sterile control)
Surface scan and eye protection	Compatible based on automatic removal

Documentation and Statistics	
Recording function	Video-, Audio-, Screenshot
Video formats	MP4V, DVSD, H264
Container formats	avi, mp4
Recording modes	Navigation + Video, Video only, Anonymized Recording
Audit-trail	Type of Use, Incision-Suture Time, Duration of Registration, Application Start, Tool Use, Duration of Calibration
User Interface	
Display configurations	2x2, 2x3, 5+1, 2x1, Full Screen
User configurable	Individual surgeon profiles, individual image data sets per view
Displays	Axial, Coronal, Sagittal, Inline, Probes View, Video, 3D, Augmented Reality Video
Touch optimized interface	Ergonomic button size, precision adjustment buttons, on screen keyboard
Touchless interaction	Zoom, Scroll, Confirm, Cancel, Auto-Hold last position, Auto-Screenshot, Mouse interaction
Localization	Software interface localized to local languages, switchable
Number of screens	Up to 4
Auto zoom	Zooms automatically into tool tip after patient registration is completed
Navigation Functions	
Navigation technology	Electromagnetic
Number of instruments	Unlimited using manual switching, up to 4 at the same time Up to 3 parallel via the instrument adapter system (electromagnetic)
Navigated electromagnetic instruments	Precision Pointer Electromagnetic, Patient Tracker Electromagnetic, Registration Pointer Electromagnetic, Suction Tube Frazier Electromagnetic, Suction Tube Eicken Electromagnetic, Endoscope Tracker Electromagnetic, Calibration Body Electromagnetic, Universal Tracker Electromagnetic
Virtual tool extension	5 mm and 10 mm steps
Instrument Adapter System	
Calibration	Intraoperative, less than 10s, tip, axis, width
User interaction	Fully sterile, no PC interaction required, automatic tool detection/selection
Connection to DICOM PACS Servers	
Supported standard	DICOM 3.0
Service class	Query/Retrieve, Verification, Storage
Information model for Q/R	Study Root

### Connection to DICOM PACS Servers

DIMSE-C operations	C-FIND, C-MOVE, C-STORE, C-GET
Data compression formats	JPEG, JPEG 2000, RLE, uncompressed
IHE integration profile	<ol style="list-style-type: none"> <li>1. Query/Retrieve: Scopis ENT Software assumes role of Actor "Image Display" in the transaction RAD-14 (Query Images) and RAD-16 (Retrieve Images) and expects the server in the role "Image Manager/Archive".</li> <li>2. Storage: Scopis ENT Software assumes role of Actor "Acquisition Modality" in the transaction RAD-8 (Modality Images Stored) and expects the server in the role "Image Manager/Archive".</li> </ol>

### Service Capabilities

Self-test capabilities	OS, CPU, RAM, VRAM, Dedicated graphics, Graphics Type, Graphics Driver, Tracking System, Framegrabber, Windows Update Status, OpenGL version
Extended log reporting	Self-Test reporting via e-mail available
Logging	Extended session logging
Crash reporting	Extended stack logging
Advanced diagnostics menu	Full service options available

### Exclusive for Scopis ENT Software with TGS

Navigation of endoscopes	Diameter of 3-8 mm, wide-angle lenses
Duration of calibration	< 20 s
Fast recalibration	< 5 s
Size of the calibration Storage	Up to 9 endoscopes can be recalibrated quickly
Endoscopic Augmented Reality	Yes
Availability of the endoscopic navigation	< 90% of the endoscopic time

### DICOM Character Set Support

Latin alphabet #1	ISO_IR 6, ISO 2022 IR 6, ISO_IR 100, ISO 2022 IR 100
Latin alphabet #2	ISO_IR 101, ISO 2022 IR 101
Latin alphabet #3	ISO_IR 109, ISO 2022 IR 109
Latin alphabet #4	ISO_IR 110, ISO 2022 IR 110
Greek	ISO_IR 126, ISO 2022 IR 126
Arabic	ISO_IR 127, ISO 2022 IR 127
Hebrew	ISO_IR 138, ISO 2022 IR 138
Cyrillic	ISO_IR 144, ISO 2022 IR 144
Turkish	ISO_IR 148, ISO 2022 IR 148
Japanese	ISO_IR 13, ISO 2022 IR 13, ISO 2022 IR 87, ISO 2022 IR 159

**DICOM Character Set Support**

Thai	ISO_IR 166, ISO 2022 IR 166
Chinese	ISO 2022 IR 58, GB18030, GB2312, GBK
Korean	ISO_IR 149, ISO 2022 IR 149
UTF-8	ISO_IR 192

## 10. Troubleshooting and Analysis

If errors occur when operating the software, please follow these instructions for troubleshooting and analysis.

Observation	Corrective Action
<p>In case of compatibility problems with the hardware, perform the compatibility test.</p>	<ol style="list-style-type: none"> <li>1. Startup the software</li> <li>2. Close the data import function</li> <li>3. Press F10</li> <li>4. Verify the details that are displayed</li> <li>5. Report the results to Stryker if they do not all show compatibility</li> </ol>
<p>If you are connected to the internet and the software has been closed unexpectedly, provide us with all information regarding the shutdown.</p>	<ol style="list-style-type: none"> <li>1. Startup the software</li> <li>2. Close the data import function</li> <li>3. Press F8</li> <li>4. Click &gt;Send&lt; in the appearing dialog window</li> </ol>
<p>If you are NOT connected to the internet and the software has been closed unexpectedly, provide us with all information regarding the shutdown.</p>	<ol style="list-style-type: none"> <li>1. Right-click the software icon on your desktop and choose &gt;Open File Location&lt;</li> <li>2. Choose the folder &gt;logs&lt;</li> <li>3. Copy the content of the folder "Crash-dumps" to a USB drive and transfer the data to an internet enabled computer</li> <li>4. Send the data to Stryker using an encrypted connection. Contact Stryker for assistance</li> </ol>

## 11. Imaging Modalities

The software can import image data in DICOM 3.0 standard. This standard is supported by all market-relevant manufacturer's devices. Thus, the software should be compatible with all imaging modalities on the market

The list of compatible systems includes at least devices from:

- Philips (Tomoscan M/EG R4.0)
- Siemens (Somatom 4+, 4+ Volume Zoom)
- Marconi (Picker PQ 2000, PQ 5000)
- Morita (3D Accuitomo)

Before first using the navigation unit in a clinical setting, check the compatibility of the imaging modalities by importing a test image.

If problems occur while loading image data sets, please contact Stryker.

---

## 12. Keyboard Shortcuts

Shortcuts	Function
Ctrl+D	Open configuration menu
Ctrl+E	Open installation folder in data manager
Ctrl+L	Open log file in data manager
Ctrl+Shift+L	Open recent log file
F2	Screen capture
F3	Start / stop video image data record
F8	Open bug report dialog
F10	Open system compatibility dialog

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Stryker Corporation or its divisions or other affiliated entities own, use or have applied for the following trademarks or service marks: Leibinger, Scopis, Stryker, TGS. All other trademarks are trademarks of their respective owners or holders.
Copyright © 2019-20 Stryker
Stryker Leibinger GmbH & Co. KG