Stryker ENT Navigation System with TGS®

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CE 0197

Table of Contents

- 1. How to Use this Document 1
 - 1.1. About this Document..... 1
 - 1.2. Disclaimer of Liability 1
 - 1.3. Contact Information 1
 - 1.4. Definition of Conventions 1
 - 1.5. Definition of Terms, Abbreviations and Symbols 1

- 2. Safety Information 5
 - 2.1. General Safety Information 5
 - 2.2. Safety Measures for Electromagnetic Measuring Systems 6
 - 2.3. Data Privacy Information..... 7
 - 2.4. User Qualifications Information..... 7

- 3. Product Information..... 8
 - 3.1. Indications for Use..... 8
 - 3.2. Contraindications 8

- 4. Product Overview 9
 - 4.1. For Use With 9
 - 4.2. Cycle (Navigated Surgery) 10
 - 4.3. Planning for Application..... 10
 - 4.4. Instructions for Functional Endoscopic Sinus Surgery (FESS).....11
 - 4.5. Instructions for Lateral Skull Surgery..... 12

- 5. Recording Radiological Image Data 14
 - 5.1. Before First Use 14
 - 5.2. Systems Settings for Computer Tomography (CT) 14
 - 5.3. Radiology Profile for Magnetic Resonance Imaging (MRI) 15
 - 5.4. Preparing and Scanning the Patient 15
 - 5.5. Saving Data 16
 - 5.6. Transmitting Image Data 16
 - 5.7. Character Set Support..... 16

6.	Preparing for a Navigated Surgery.....	18
6.1.	Switching On the Navigation System.....	18
6.2.	Starting the Navigation Software.....	18
6.3.	Working in Planning Mode.....	18
6.4.	Basic Requirements for Patient Image Data.....	19
6.5.	Loading Patient Image Data.....	19
6.5.1.	Loading from Disk Drives.....	20
6.5.2.	Loading from DICOM Image Data Servers.....	20
6.5.3.	Segmentation of Patient Image Data.....	21
6.5.4.	Loading of Already-Imported Patient Image Data.....	24
6.6.	Displaying the Loaded Patient Image Data.....	24
6.7.	Planning Views.....	25
6.7.1.	Cross-Section Image Views: Axial, Coronal, and Sagittal.....	25
6.7.2.	Moving the Patient Image Data.....	25
6.7.3.	3D View.....	25
6.7.4.	Inline and Probe View.....	26
6.7.5.	Video View.....	26
6.7.6.	Configuration Menu: Video.....	27
6.7.7.	Changing a View Type.....	28
6.7.8.	Changing the View Configuration.....	28
6.7.9.	Managing View Configurations.....	29
6.7.10.	Adjusting Contrast and Brightness for Patient Image Data and Video.....	30
6.7.11.	Zooming In and Out in the Patient Image Data.....	32
6.8.	Importing Additional Datasets for Image Fusion.....	32
6.8.1.	Approximate Registration.....	33
6.8.2.	Precise Registration.....	33
6.8.3.	Image Fusion with PET and SPECT Image Data.....	34
6.8.4.	Configuration Options.....	35
6.9.	Selecting the Patient Registration Type.....	36
6.10.	Image-to-Patient Registration: Landmark.....	38

6.10.1. Defining Landmarks	38
6.10.2. Defining Landmarks in the 3D Model.....	39
6.10.3. Placing Landmarks in 2D Cross-Section Images.....	40
6.10.4. Moving Landmarks.....	41
6.10.5. Removing Landmarks.....	41
6.11. Image-to-Patient Registration: Surface.....	41
6.12. Image-to-Patient Registration: Enhanced Surface	42
6.13. Planning the Navigated Surgery.....	44
6.13.1. Planning Object Dialog	44
6.13.2. Creating New Planning Objects	46
6.13.3. Removing Planning Objects.....	47
6.13.4. Creating Planning Points	47
6.13.5. Creating Planning Structures	47
6.13.6. Creating a Cutting Line	49
6.13.7. Creating a Bone Segmentation.....	49
6.13.8. Highlighting Paranasal Sinus Cells with Building Blocks	50
6.13.9. Importing an Implant	51
6.13.10. Creating a Symmetry Plane	52
6.13.11. Taking Measurements	52
6.13.12. Moving Planning Points	53
6.13.13. Precision Adjustment of Planning Points	53
6.13.14. Mirroring of Planning Points and Structures	54
6.13.15. Defining Alarm Zones for Planning Points and Planning Structures	54
6.14. Saving the Planning.....	56
6.15. Exporting the Planning.....	58
6.16. Taking Screenshots and Screen Recordings.....	59
7. Pre-operative Setup of a Navigated Surgery.....	60
7.1. General Functionality Testing.....	60
7.2. Field Generator	60
7.2.1. Interferences with Electromagnetic Systems	62
7.3. Preparing Navigated Instruments.....	64

7.3.1. Patient Tracker Electromagnetic	65
7.3.2. Pointer Instruments	67
7.3.3. Navigated Suction Tubes	69
7.3.4. TGS Guidewire (US only).....	69
7.3.5. Instrument Clamps	70
7.3.6. Universal Tracker Electromagnetic	70
7.3.7. Instrument Clamps Electromagnetic (US only)	72
7.3.8. Endoscope Tracker Electromagnetic	73
7.3.9. Calibration Body Electromagnetic.....	74
7.3.10. Navigation Tool Extension Cable	74
7.4. Suitability of Surgical Instruments.....	75
8. Conducting Navigated Surgery.....	77
8.1. Loading Patient Data	77
8.2. Starting the Navigation.....	77
8.3. Connecting Electromagnetic Instruments.....	78
8.4. Status Displays	78
8.5. Checklist: Visibility of Navigated Instruments	78
8.6. Freezing Navigation	79
8.7. Calibration and Verification of Navigated Instruments	79
8.7.1. Calibration of Surgical Instruments	80
8.7.2. Calibration of Rigid Endoscopes (Augmented Reality)	84
8.8. Preset of Navigated Endoscopes	84
8.9. Interaction Areas.....	87
8.10. Conducting the Image-to-Patient Registration.....	87
8.10.1. Image-to-Patient Registration: Landmark	87
8.10.2. Image-to-Patient Registration: Surface.....	88
8.10.3. Image-to-Patient Registration: Enhanced Surface.....	89
8.11. Verification/Confirmation of the Image-to-Patient Registration.....	90
8.12. Resetting the Image-to-Patient Registration	90
8.13. Reuse an Image-to-Patient Registration	91
8.14. Visual Checking of the Navigation Information	92
8.15. Visualizing the Position of Navigated Instruments.....	92

8.16. Virtual Extension of the Tool Axis	93
8.17. Augmented Reality	93
8.17.1. Distance Orientation Guide.....	94
8.17.2. Navigating with Target Guided Surgery (TGS).....	95
8.18. Closing the Navigation	96
8.18.1. Closing the Navigation Mode	96
8.18.2. Closing the Navigation Software.....	97
8.18.3. Switching off the Navigation System.....	97
9. Recording an Operation without Patient Image Data	98
10. Maintenance.....	99
11. Transport and Disposal	100
11.1. Transport	100
11.2. Disposal	100
12. Technical Specifications.....	101
12.1. Environmental Conditions	101
12.2. Essential Performance Features	101
12.3. Compatible Imaging Systems	101
13. Routine Checkups	102
14. Lifetime of Electromagnetic Instruments.....	103
15. Special Application Instructions for Instruments.....	105
16. Minimum Requirements for Additional Components.....	106
16.1. Devices	106
16.2. Disposables	107
17. Keyboard Shortcuts for Navigation Software.....	108

1. How to Use this Document

1.1. About this Document

This document is the most comprehensive source of information for the safe and effective use of the product. Read this document carefully and keep for future reference. Pay special attention to safety information.

When combined with other medical devices, the user manual of these devices is to be considered as well.

1.2. Disclaimer of Liability

In no instance will Stryker be responsible for incidental or consequential damages related to the product.

1.3. Contact Information

	Stryker Representative	
	+49 761 4512 0 (Germany)	+1 269 323 7700 (USA)
	www.stryker.com	

1.4. Definition of Conventions

The following table provides definitions of conventions used in this document.

Convention	Definition
1.	Instructional steps that must be performed in a sequence.
•/-	Unordered list item

1.5. Definition of Terms, Abbreviations and Symbols

The following table provides definitions of terms used in this document.

Term	Definition
WARNING	The signal word WARNING highlights a safety-related issue. Comply with this information to prevent patient and medical staff injury.
CAUTION	The signal word CAUTION highlights a product reliability issue. Comply with this information to prevent product damage.

The following table provides definitions of abbreviations commonly used in this document.

Abbreviation	Definition
EN	European Standard
ENT	Ear, Nose, Throat
ISO	International Organization for Standardization
MR	Magnetic Resonance
TGS	Target Guided Surgery

The following tables define the symbols used in this document, on the product and on the product label. The tables also include symbols that are used with affiliated products to the Stryker ENT Navigation System with TGS.

EN ISO 7010 Graphical symbols – Safety colors and safety signs – Registered safety signs

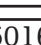
Symbol	Name: Definition
 W001	General warning sign: To signify a general warning.
 M002	Refer to instruction manual/booklet: To signify that the user instruction manual/booklet must be read.
 P007	No access for people with active implanted cardiac devices: To prohibit people with active implanted cardiac devices from entering a designated area.

EN ISO 15223-1 Medical devices — Symbols to be used with medical device labels, labeling and information to be supplied - Part 1 General requirements

Symbol/number	Name: Definition
 5.1.1	Manufacturer: Indicates the medical device manufacturer as defined in the European Union harmonization legislation.
 5.1.3	Date of manufacture: Indicates the date when the medical device was manufactured.
 5.1.4	Use-by date: Indicates the date after which the medical device is not to be used.
 5.1.5	Batch code: Indicates the manufacturer's batch code so that the batch or lot can be identified.
 5.1.6	Catalog number: Indicates the manufacturer's catalog number so that the medical device can be identified.
 5.1.7	Serial number: Indicates the manufacturer's serial number so that the medical device can be identified.
 5.2.7	Non-Sterile: Indicates a medical device that has not been subjected to a sterilization process.
 5.3.1	Fragile, handle with care: Indicates a medical device that can be broken or damaged if not handled carefully.
 5.3.2	Keep away from sunlight: Indicates a medical device that needs protection from light sources.

Symbol/ number	Name: Definition
 5.3.4	Keep dry: Indicates a medical device that needs to be protected from moisture.
 5.3.7	Temperature limit: Indicates the temperature limits to which the medical device can be safely exposed.
 5.3.8	Humidity limitation: Indicates the range of humidity to which the medical device can be safely exposed.
 5.3.9	Atmospheric pressure limit: Indicates the range of atmospheric pressure to which the medical device can be safely exposed.
 5.4.2	Do not re-use: Indicates a medical device that is intended for one use or for use on a single patient during a single procedure.
 5.4.3	Consult instructions for use: Indicates the need for the use to consult the instructions for use.
 5.4.4	Caution: Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.

IEC 60417 Graphical symbols for use on equipment

Symbol	Name: Definition
 0623	This way up: To indicate correct upright position of the transport package.
 0630	Stacking limit by mass: To indicate that the items shall not be vertically stacked beyond the specified number, either because of the nature of the transport packaging or because of the nature of the items themselves.
 2403	Stacking limit by number: To indicate that the items shall not be vertically stacked beyond the specified number, either because of the nature of the transport packaging or because of the nature of the items themselves.
 5009	Stand-by: To identify the switch position by means of which part of the equipment is switched on in order to bring it into the stand-by condition, and to identify the control to shift to or to indicate the state of low power consumption. Each of different states of power consumption may be indicated using a corresponding colour.
 5016	Fuse: To identify fuse boxes or their location.
 5017	Earth; ground: To identify an earth (ground) terminal in cases where neither the symbol 5018 nor 5010 is explicitly required.
 5019	Protective earth; protective ground: To identify any terminal that is intended for connection to an external conductor for protection against electric shock in case of a fault, or the terminal of a protective earth (ground) electrode.
 5021	Equipotentiality: To identify the terminals that, when connected together, bring the various parts of an equipment or of a system to the same potential, not necessarily being the earth (ground) potential, e.g. for local bonding.
 5032	Alternating current: To indicate on the rating plate that the equipment is suitable for alternating current only; to identify relevant terminals.
 5134	Electrostatic sensitive devices: To indicate packages containing electrostatic sensitive devices, or to identify a device or a connector that has not been tested for immunity to electrostatic discharge.

Symbol	Name: Definition
 5140	Non-ionizing electromagnetic radiation: To indicate generally elevated, potentially hazardous, levels of non-ionizing radiation, or to indicate equipment or systems e.g. in the medical electrical area that include RF transmitters or that intentionally apply RF electromagnetic energy for diagnosis or treatment.
 5333	Type BF applied part: To identify a type BF applied part complying with IEC 60601-1.

Product-Specific Symbols

Symbol	Name: Definition
	Quantity: Indicates the number of medical devices in the packaging.
	Note symbol: Used to supplement or clarify information.
	Medical device: Indicates a medical device as defined by European Union harmonization legislation.
GTIN	Global Trade Item Number

81 FR 38911 FDA Final rule for the use of symbols in labeling

Symbol	Name: Definition
Rx Only	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

Directive 2012/19/EU on waste electrical and electronic equipment (WEEE)

Symbol	Name: Definition
	Indicates that the product must be collected separately and must not be disposed of as unsorted municipal waste.

ASTM F2503-08 Standard practice for marking medical devices and other items for safety in the MR environment

Symbol	Name: Definition
	MR Unsafe: Indicates a product that is known to pose hazards in all MRI environments.

Regulatory marks and logos

Symbol	Definition
	CSA certified for Canada and USA
	Indicates that a device is in conformity with the applicable requirements set out in applicable European Union harmonization legislation providing for its affixing.
	Indicates that a device is in conformity with the applicable requirements set out in applicable European Union harmonization legislation providing for its affixing.

2. Safety Information

2.1. General Safety Information

WARNING

- Potential malfunction. Only use the system if the surgery also can be performed conventionally. Otherwise, a second equivalent product should be available.
- During therapeutic use, it is necessary to ensure visual observation of the systemic effects.
- Intra-operative live images processed by the system may be displayed with a delay. During applications that require realtime-critical imaging, a second monitor must be used. This must be directly connected to the realtime-critical image source.
- Always inspect the product and all system components for damage before each use. Do not use the product if damage is apparent; contact Stryker.
- Do not disassemble, modify, service, or repair this product without the authorization of the manufacturer. Failure to comply may result in substandard performance and/or void the warranty. Contact Stryker for service.
- Ensure regular installation of software updates.
- Do not bend or kink cables, or use cables that are damaged. Position measurements from a system with damaged tool cables may result in possible personal injury.
- Do not use the system in atmospheres with potential for explosion. The system is not protected against explosion.
- Do not block the ventilation slots on the product's bottom and backside.
- Position the product outside the reach of the patient.
- Do not use the device in the environment of a magnetic resonance imaging (MRI) scanner.
- In case of emergency, shut down the product immediately and remove it from patient or staff. Then arrange for the necessary check-ups and repairs conducted by qualified service staff.
- Do not use the system during the discharge of a defibrillator. All navigated instruments must be removed from the patient and the operating area beforehand.
- The system should not be used by users who suffer from dyschromatopsia (color blindness); in particular, red-green impairment or red-green-blindness affect the ability to use the system.
- Navigated instruments are delivered in a non-sterile condition. Before the first use as well as before each following use, the instrument must be processed (cleaned, disinfected, and/or sterilized) according to a validated procedure.
- Only those accessories listed in the manual may be used in combination with the navigation system. When combining medical products, safety is only ensured if:
 - the combination is specified in the respective manuals as a safe combination, or
 - the intended use and the interface specifications of the combined medical products are suited for combination (compare IEC 60601-1).
- Using non-listed instruments and accessories (including cables) will lead to loss of the product conformity.
- The medical electrical devices used in combination must comply with the safety standard IEC 60601-1 and the standard for data handling devices IEC 60950-1, if applicable. When connecting additional devices to the signal in- and outputs, you are configuring a medical system and will be responsible for making sure that the system meets the requirements of the system standard IEC 60601-1.
- Do not expose or immerse the electrical components of the navigation system to liquids. Do not place liquids on or above the devices. Liquids or other objects penetrating into the device may result in equipment damage, produce a fire or shock hazard.
- Do not use mobile or other phones or portable radio frequency (RF) equipment in the vicinity of medical devices. Devices emitting electromagnetic radiation may interfere with operation of the product.

- Non-observance of the permitted environmental conditions may lead to a decalibration of the measuring system. Should this happen, deviations can occur while measuring. These deviations can only be remedied with a new calibration by the manufacturer. See Chapter “Technical Specifications.”
- If the device or its accessories are used on a patient who suffers from a disease whose pathogens cannot be eliminated with conventional, standardized procedures, it must be disposed of or processed according to the requirements of the responsible public authority.
- Do not simultaneously touch the patient and any parts of the system—e.g. Electromagnetic Navigation Unit, cart, monitors—except the EM Tools. This may lead to dangerous electrical currents that may harm the patient or operator.
- When using a supporting system (e.g. cart), always connect and power all other devices within the system (e.g. Electromagnetic Navigation Unit, monitor) via the supporting system. Do not connect these other devices directly to wall sockets when using a supporting system.

 The user and/or patient should report any serious product-related incident to both the manufacturer and the national competent authority where the user and/or patient is established.

2.2. Safety Measures for Electromagnetic Measuring Systems

WARNING

- Electromagnetic instruments are highly precise measuring instruments. Restrictions concerning lifetime and handling must be observed. Damaged and inaccurate instruments must be exchanged.
- Do not operate the Field Generator within 200 mm of an installed pacemaker. The magnetic field produced by the Field Generator may interfere with the operation of the pacemaker. This interference may result in personal injury.
- Do not operate the Field Generator within 10 m of another Field Generator. To do so may contribute to inaccurate position measurement and possible personal injury.
- Do not drop the Field Generator or subject it to impact. Physical damage to the Field Generator may alter its calibration and contribute to inaccurate position measurement and possible personal injury.
- Do not place electromagnetic instruments directly on the Field Generator. Doing so may cause interferences which may produce inaccurate position measurements.
- Do not place the Field Generator Cable inside the measurement volume or wrap it around the Field Generator, as it may create a magnetic interference. This interference can contribute to inaccurate position measurements and possible personal injury.
- Do not place electromagnetic tool cables within 30 mm of the Field Generator Cable. If placed this close – particularly if the cables are parallel to each other – the tool cable may become subject to electromagnetic interference. This interference can contribute to inaccurate position measurements and possible personal injury.
- Do not coil the Field Generator Cable, as it produces enough electric current that a magnetic field will be created when the cable is placed in a circular formation. This magnetic field may disturb the Field Generator’s magnetic field, contributing to inaccurate position measurements and possible personal injury.
- Do not disconnect the Field Generator from the system while using the navigation. In tracking mode, this could result in sparks being generated, and possible personal injury.
- Do not navigate in an unvalidated environment, as it may contain elements that affect navigation functions. The system can be adversely affected by electromagnetic field disturbances from other objects in the room, the close proximity of metal, and the close proximity of another Field Generator. Failure to test for such disturbances will increase the possibility of inaccurate position measurement and possible personal injury.

2.3. Data Privacy Information

WARNING

Connecting the navigation unit to a network/data aggregate can put the patient, the operator, or third parties at risk. Your organization's risk management should determine, analyze, evaluate, and control these risks to avoid potential damages (see especially IEC 80001-1). Regarding this product, it is necessary to pay special attention to risks regarding the privacy of patient image data and data integrity.

CAUTION

Prior to sending the device for maintenance, remove the data drive so that any private patient data is not shared with Strkyer.

The user must adhere to the respective national requirements regarding protection of patient data. The navigation system may only be used by authorized personnel. We recommend protecting the system from unauthorized use by means of a password.

In public hospitals, it is necessary to inform the data protection commissioner about the system.

Due to data security, it is necessary to back up data regularly. Stryker does not assume liability for data loss.

2.4. User Qualifications Information

WARNING

The product is intended solely for use by medical professionals and may only be used by qualified, medically trained people.

The healthcare provider performing any procedure is responsible for determining the appropriateness of using the product and for the specific technique for each patient. Stryker, as a manufacturer, does not recommend a specific surgical procedure.

Stryker does not assume liability for any diagnoses or interpretations established with the help of this product. The product's user is responsible for acquiring medical knowledge and for its diagnostic and therapeutic consequences.

To ensure the safe and efficient use of the product, the user must be instructed on how to use the product as authorized by the manufacturer and retailer. The medical product's operator must ensure that the user receives such instruction in accordance with the respective local laws, and that the instruction is refreshed at the prescribed intervals. Additionally, the instructions in all accompanying manuals must be respected.

During a surgical operation using the navigation system, the patient must be treated and observed with the usual medical care. This includes the follow-up to the treatment process, the monitoring of vital signs and the state of anesthesia, as well as the maintenance of sterile conditions if required for the operation.

Aside from your own medical skills and knowledge, the operational safety and utility of the product requires correct usage as well as regular cleaning and maintenance.

3. Product Information

WARNING

- The product may only be used for its intended purpose and in accordance with the following user manual, as well as all current versions of the “Stryker ENT Navigation System” manual.
 - The user manual is part of the product and must therefore be accessible to personnel at all times. It must be handed over to subsequent owners or users.
-

3.1. Indications for Use

The Stryker ENT Navigation System is indicated for any medical condition in which the use of stereotactic surgery may be appropriate, and where reference to a rigid anatomical structure in the field of ENT surgery, such as the paranasal sinuses, mastoid anatomy, can be identified relative to a CT or MR based model of the anatomy.

Example procedures include, but are not limited to the following ENT procedures:

- Transphenoidal access procedures;
- Intranasal procedures;
- Sinus procedures, such as Maxillary antrostomies, Ethmoidectomies, Sphenoidotomies/ Sphenoid explorations, Turbinate resections, and Frontal sinusotomies;
- ENT related anterior skull based procedures.

3.2. Contraindications

None known.

4. Product Overview

4.1. For Use With


WARNING

Use only Stryker-approved products, unless otherwise specified.

Article Name	REF
Electromagnetic Navigation Unit	8000-010-003
Field Generator	8000-010-004
Field Generator Mounting Arm	8000-010-005
Scopis ENT Software with TGS	8000-020-002
Patient Tracker Electromagnetic	8000-040-001
Patient Tracker Electromagnetic-10 uses	8000-040-002
Precision Pointer Electromagnetic	8000-050-001
Pointer Electromagnetic	8000-050-002
Registration Pointer Electromagnetic	8000-050-003
Suction Tube Frazier Electromagnetic	8000-050-005
Suction Tube Eicken Electromagnetic	8000-050-006
Navigation Tool Extension Cable	8000-050-011
Endoscope Tracker Electromagnetic	8000-060-001
Calibration Body Electromagnetic	8000-060-002
Universal Tracker Electromagnetic	8000-060-006
TGS Guidewire (US Only)	8000-060-009
Instrument Clamp, Forceps	8000-060-010
Instrument Clamp, 2-6 mm	8000-060-011
Instrument Clamp, 6-10 mm	8000-060-012
Instrument Clamp, 10-16 mm	8000-060-013
Instrument Clamp Electromagnetic Sphere (US Only)	8000-060-020
Instrument Clamp Electromagnetic Universal (US Only)	8000-060-021
Patient Tracker Tabs	8000-100-001

4.2. Cycle (Navigated Surgery)

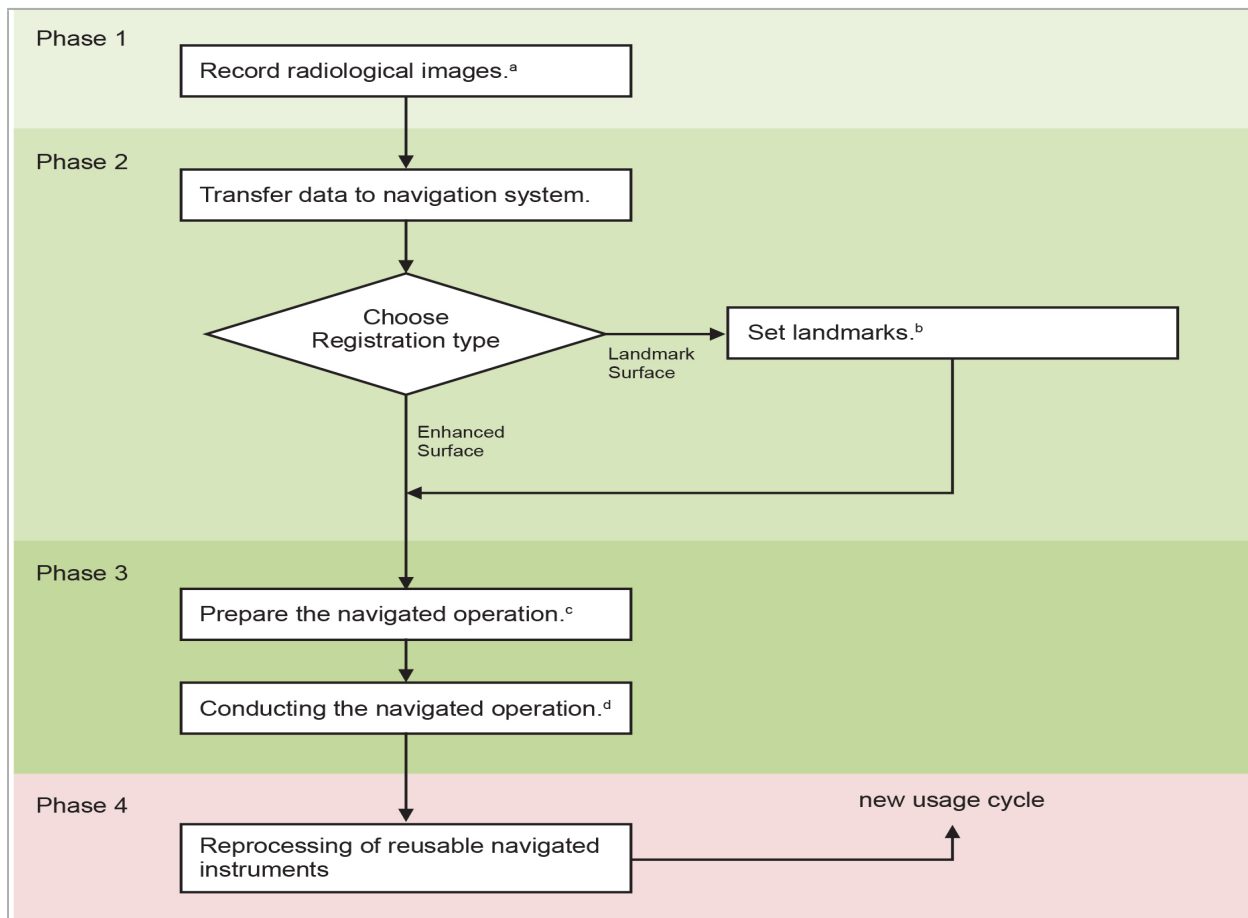The use of a navigation system can be split into 4 phases:

^a See Chapter “Image-to-Patient Registration: Landmark”

^b See Chapter “Planning the Navigated Surgery”

^c See Chapter “Exporting the Planning”.

^d See Chapter “Pre-operative Setup of a Navigated Surgery”

^e See Chapter “Conducting Navigated Surgery”

4.3. Planning for Application

The application of the navigation system must be planned before the surgery, as patient image data suitable for navigation are needed.

Defining the Operating Field

Consider the intended uses of the products included in the system. Many products are only intended for use in selected operating fields. The intended uses can be found in the respective product manuals.

Choosing a Patient Tracker and Positioning

Patient trackers serve to reference the patients. They are therefore firmly attached to the patient near the operating area, which enables the localization of the patient. The available Patient Trackers are described in “Preparing Navigated Instruments.”

Choosing a Registration Type

Before every application, conduct an image-to-patient registration to determine the position of the patient and to match the patient image data.

The software offers several types of image-to-patient registration:

Landmark	At least three clearly perceptible landmarks are marked in the patient image data and subsequently touched precisely with a pointer on the patient. This type of registration is especially suitable in the presence of artificial landmarks (e.g. bone screws, fiducials) in the operating area and in the patient image data.
Surface	At least three clearly perceptible landmarks are marked in the patient image data and subsequently touched with the precision pointer on the patient. Afterwards the pointer must be moved over the patient's surface to collect more points for the matching. This type of registration is especially suitable when registration only takes place on the skin. For surface registration at least 40 cm ² of skin near the situs are necessary.
Enhanced Surface	Here a specific path is generated in the patient image data that must be initially moved along on the patient's surface. Afterwards the pointer must be moved over the patient's surface to collect more points for the matching. This type of registration is especially suitable for sinus surgery. For enhanced surface registration at least 30 cm ² of skin left and right of the nose, as well as the nose itself, are necessary.

Ordering Radiological Image Data

Note the following points, and follow the detailed instructions for acquiring suitable images for navigation in "Recording Radiological Image Data."

- Depending on registration type, it might be necessary to plan and set bone screws around the operating field before ordering radiological image data.
- Existing image data may be used if the patient anatomy has stayed the same. However, recording of new images is necessary if suspected that patient geometry has significantly changed.
- Image resolution should be set to a slice thickness of less than 1 mm and pixel resolution of 0.5 mm or finer for optimal navigation results. Also consider that the areas needed for image-to-patient registration are included in the scan.
- All common DICOM modalities such as DVT, CT, or MRI can be used.

4.4. Instructions for Functional Endoscopic Sinus Surgery (FESS)

WARNING

The following recommendations are based on clinical experience with the navigation system. However, it is the physician's responsibility to determine the specifics of each operation and to decide in each case for the use of a navigation system.

For use of the navigation system in frontal sinus surgery and functional endoscopic sinus surgery, the following procedure has proven most effective:

1. Request a thin slice CT scan of the operating field. Ensure that the patient's face and nose are completely included. The back of the head, slices above the forehead and below the subnasale are not necessary. Recommended slice thickness < 1 mm, pixel resolution 0.8 mm or finer.
2. Use the enhanced surface registration, which can be used without additional planning.

3. Use the Patient Tracker Electromagnetic on the patient's forehead. Keep sufficient area on the patient's face free of sterile cover. Leave an uncovered area of 0.5 cm vertically between subnasale and nasale and also horizontally to the outer corner of the eyes (Figure 1).

Figure 1 – Recommended Position and Sterile Cover for FESS

4.5. Instructions for Lateral Skull Surgery

WARNING

The following recommendations are based on clinical experience with the navigation system. However, it is the physician's responsibility to determine the specifics of each operation and to decide in each concrete case for the use of a navigation system.

For use of the navigation system in middle ear surgery, both landmark and surface registrations can be used.

Landmark Registration

Landmark registration with 4-5 bone screws is found to be most accurate for middle ear surgery.

1. Place these screws at the end of a preceding surgery when entry to the middle ear is exposed (Figure 2).
2. Request a thin slice CT scan and ensure that all bone screws are completely included. Choose a resolution in which the relevant structures are well visible. In case the data is too large, request the data to be trimmed accordingly.
3. Use the Patient Tracker Electromagnetic and place it on the opposite side of the situs from yourself. If you are being assisted by a physician, the Patient Tracker can be moved correspondingly to the right or to the left. A distance of 7.5 cm is found to be most effective (Figure 3).

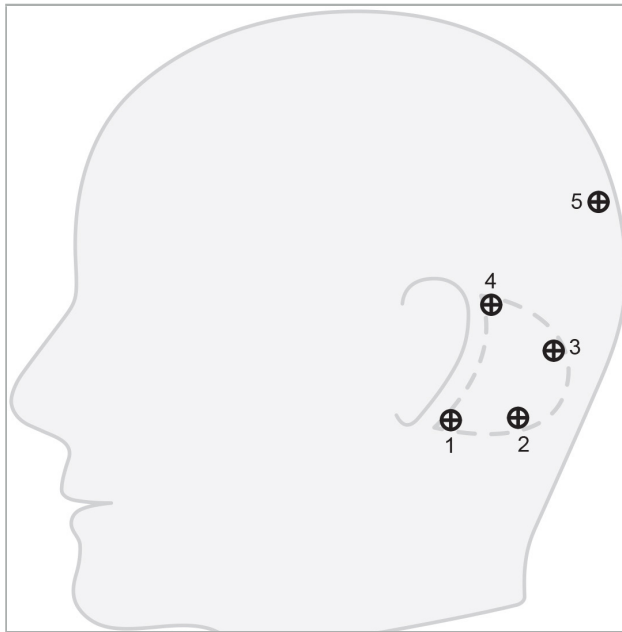
Figure 2 – Recommended Bone Screw Placement for Lateral Skull Surgery

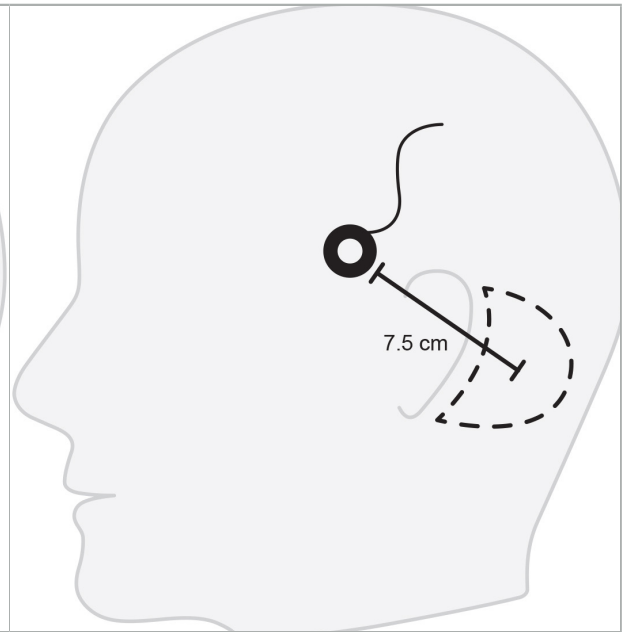
Figure 3 – Positioning of the Patient Tracker Electromagnetic for Lateral Skull Surgery

Surface Registration

A surface registration is a less invasive and plausible choice for registration if accuracy of 1 mm is acceptable in the surgical situation.

1. Request a thin slice CT scan and ensure half of the patient's face is completely included. Choose a resolution in which the relevant structures are well visible. In case the data is too large, request the data to be trimmed accordingly.
2. Place registration landmarks on subnasale, nasion, and lateral canthus of the eye near the surgical situs as starting points for surface registration.
3. Use the Patient Tracker Electromagnetic or Patient Tracker Electromagnetic - 10 Uses and place it on the opposite side of the situs from yourself. If you are being assisted by a physician, the Patient Tracker can be moved correspondingly to the right or to the left. A distance of 7.5 cm has been found to be most effective (Figure 3).
4. When collecting surface points, trace from the initial area (nasion, subnasal, canthus) to the mastoid bone and back. Do not move under the ear and on the ear lobe as these are not rigidly connected to the middle ear anatomy.

5. Recording Radiological Image Data

Before acquiring radiological images for your patient, consider the following aspects:

- The radiological volume image data must be compatible with the desired type of patient registration. That means it may be necessary to attach bone screws before the imaging or ensure that certain parts of the face or patient anatomy are visible. See Chapter “Selecting the Patient Registration Type.”
- Navigation is only possible from 3D volume image data that conforms to particular requirements regarding resolution, slice distance/gap, and export format. Refer to the following chapter for specifics.
- Despite support from the standardized DICOM 3.0 interface, problems may occur with the data exchange or the loading of data. Refer to the following chapter.

5.1. Before First Use

WARNING

No radiology images should be recorded from patients until it is verified that the image can be imported to the navigation software.

The following pages should be forwarded on to your radiology department. A test dataset, provided by the radiology department, should be used to confirm that the exported image data of the device is compatible with the navigation software.

5.2. Systems Settings for Computer Tomography (CT)

WARNING

Patient image data with a slice distance or pixel size of more than 1 mm may result in reduced system accuracy. Verify the accuracy by checking the position of well-known anatomical positions.

Use a tilting angle of zero degrees for the gantry while using computer tomography. The dataset must equate a full volume. The volume must not contain gaps, especially not gaps between the sliced images.

Limit the field of view as much as possible.

Make sure that the relevant patient area is fully recorded.

The navigation software works successfully with the following settings:

- Patient positioning: axial to the occlusion plane
- Matrix size: 512 x 512 pixel
- Slice distance/thickness: < 1 mm (≤ 0.5 mm recommended)
- Section size: 150 mm to 180 mm
- Voxel size: 1 mm x 150 mm/512 x 150 mm/512 = 1 mm x 0.3 mm x 0.3 mm
- The patient should be scanned axially

Select the energy of the X-rays (kV) so that a good resolution in the bone area is achieved. Soft tissue windows should not be used.

Make sure that the alignment of the patient corresponds to the settings of the device.

5.3. Radiology Profile for Magnetic Resonance Imaging (MRI)

The navigation software works successfully with the following settings:

- Patient positioning: axial to the occlusion plane
- Transversal (axial) acquisition plane
- Acquisition of 3D volume data (in contrast to 2D multiple slice images)
- Matrix size: at least 256 x 256 pixel – 512 x 512 pixel recommended
- Slice distance: < 1.5 mm (0.8 mm or 0.5 mm recommended)

5.4. Preparing and Scanning the Patient

WARNING

- Make sure that the patient does not move during recording. If the patient moves, the recording is useless and the imaging must be repeated, potentially exposing the patient to increased radiation.
- Recording one area in several partial volumes does not produce usable data.

To avoid artifacts in the acquired images, all prosthetics containing metal (e.g. clasp denture) should be removed if feasible.

Note that certain patient registration types can only be conducted if bone screws are placed near the operating area. These screws or specific areas of the face or of the patient's anatomy must be visible from within the image data. See Chapter "Selecting the Patient Registration Type" for specifics.

Eye Protection

If using a lens protection, the registration on the patient surface may not be possible, or may not be detected correctly by the software. The following eye protection has proven to be especially suited for a combined use with the navigation system:

Product Name	AttenuRad CT Eye Shield
Manufacturer	F&L Medical Products
Address	1129 Industrial Park Road Vandergrift, PA 15690
Phone	724-845-7028
Fax	724-845-5439

5.5. Saving Data

WARNING

- When saving the dataset, make sure to use the correct patient name and series ID.
- Make sure that the patient orientation is correctly adjusted and saved when creating DICOM data.

The navigation system supports both uncompressed and compressed (JPEG, JPEG 2000) DICOM data, but the manufacturer recommends using uncompressed data for optimal image quality.

Do not save secondary reconstructions or other image data on the same storage medium. If multiple datasets are created, they should be saved under different names (e.g. "Doe, John 3"). Each dataset should preferably be stored in a separate file or must be stored under a separate series ID on a PACS server.

5.6. Transmitting Image Data

Image data for planning and treatment should be provided by your radiologist in DICOM format on a USB drive or other data medium. Alternatively, the system can be configured to communicate directly with your organization's DICOM image data servers.

When exporting onto a data storage device, the following DICOM tags must be saved with the image data, which happens automatically with typical radiological systems:

Tag ID	Tag Name
0010,0010	PatientName
0020,0010	StudyID
0020,0011	SeriesNumber
0020,0032	ImagePositionPatient
0020,0037	ImageOrientationPatient
0028,0010	Rows
0028,0011	Columns
0028,0030	PixelSpacing
0028,0100	BitsAllocated
0028,0101	BitsStored
0028,0102	HighBit
0028,0103	PixelRepresentation

5.7. Character Set Support

The navigation software supports the following character sets:

Character Set Description	DICOM Defined Terms
Latin alphabet No. 1	ISO_IR 6, ISO 2022 IR 6, ISO_IR 100, ISO 2022 IR 100
Latin alphabet No. 2	ISO_IR 101, ISO 2022 IR 101

Character Set Description	DICOM Defined Terms
Latin alphabet No. 3	ISO_IR 109, ISO 2022 IR 109
Latin alphabet No. 4	ISO_IR 110, ISO 2022 IR 110
Greek	ISO_IR 126, ISO 2022 IR 126
Arabic	ISO_IR 127, ISO 2022 IR 127
Hebrew	ISO_IR 138, ISO 2022 IR 138
Cyrillic	ISO_IR 144, ISO 2022 IR 144
Turkish	ISO_IR 148, ISO 2022 IR 148
Japanese	ISO_IR 13, ISO 2022 IR 13, ISO 2022 IR 87, ISO 2022 IR 159
Thai	ISO_IR 166, ISO 2022 IR 166
Chinese	ISO 2022 IR 58, GB18030, GB2312, GBK
Korean	ISO_IR 149, ISO 2022 IR 149
UTF-8	ISO_IR 192

For the list of character sets supported by DICOM Standard 2011, refer to DICOM 2011 C.12.1.1.2 Specific Character Set.

6. Preparing for a Navigated Surgery

Before conducting navigated surgery, the patient image data must be prepared. To this end, the patient image data must be transferred directly to the navigation system, where the data can be worked on in planning mode.

6.1. Switching On the Navigation System

WARNING

Only connect this device to a mains supply that has a protective earth (PE) connection. Before connecting the power cable to a power outlet, make sure that the correct voltage (100 - 240 V~ / 50/60 Hz) is provided, and that the power cables are clean, dry, and undamaged. For the proper installation of the electrical device, consult the respective installation manual.

Switch on the navigation unit by pressing the power button. When switched on, the power button glows green. Then switch on the medical devices used in combination according to their respective manuals.

6.2. Starting the Navigation Software

After switching on the devices, wait until your operating system is ready. Start the software via the shortcut on the Windows desktop. After starting the software, a loading screen appears for a short time. Then the navigation software application screen appears.

6.3. Working in Planning Mode

After starting the software, the Application window appears (Figure 4).

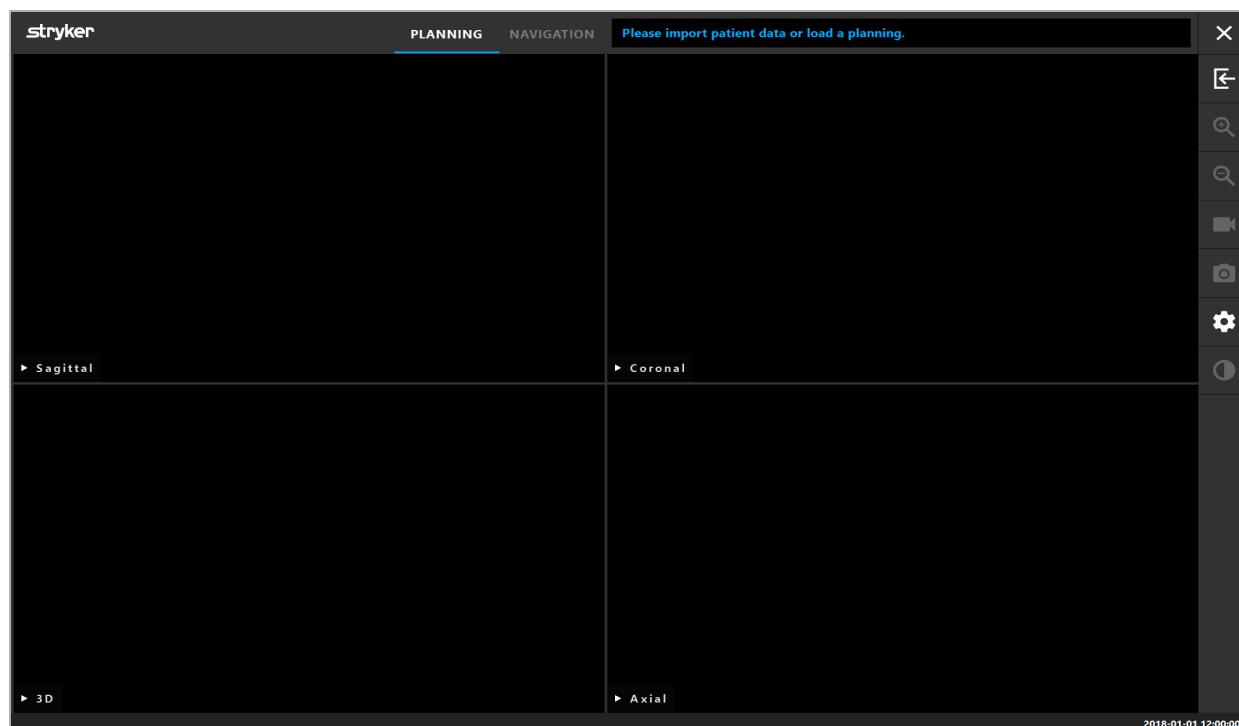
Figure 4 – Application Window

6.4. Basic Requirements for Patient Image Data

- Modality: CT, DVT, MRI, PET, SPECT
- Format: DICOM 3.0
- Storage Media: USB Drive, PACS (Server), DVD, or CD

6.5. Loading Patient Image Data

WARNING

Changes to the network data aggregate can create new risks and should therefore be conducted in a controlled manner. At the very least, attention should be paid to changes to the network configuration and topology, adding and removing additional products, updates and upgrades of systems in the data aggregate.

Press the Loading Patient Data button (Figure 4 - Below Exit icon). The Data Import dialog box appears (Figure 5). On the left side of the dialog, choose a data source for the import. The Plannings button contains all the plannings created and saved with the software.

The USB symbol allows access to data on removable storage devices.

Use the Browse button to choose the folder from which to import the data. Additional folders or DICOM data servers can be chosen depending on configuration.

Should a needed data source not show up, contact Stryker.

If the network connection or the image data server fails or is incorrectly configured, the patient image data may become inaccessible. Image data should be uploaded to the navigation system well before an operation so that you can import the data by other means if necessary.

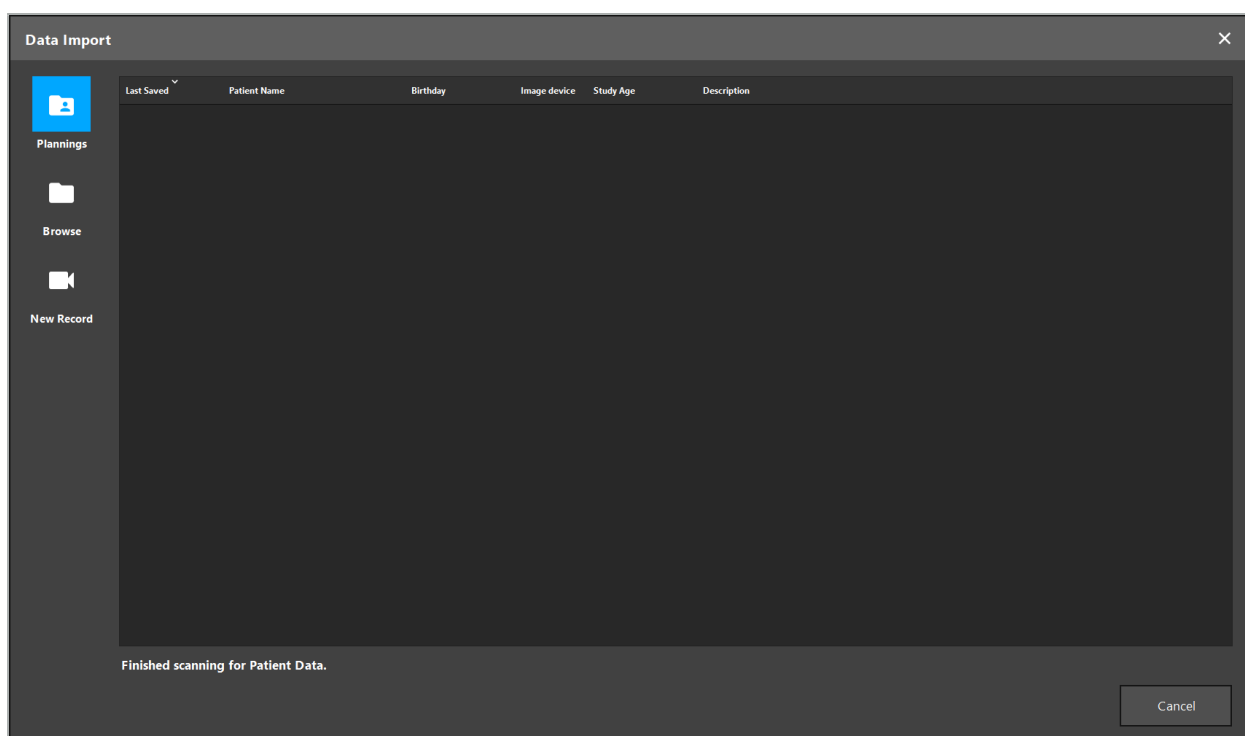
Figure 5 – Data Import

6.5.1. Loading from Disk Drives

After scanning a local data source directory for DICOM compatible data, a selection list of the available patient image data appears. If patient image data was already exported from a planning station, it can be loaded as well. See Chapter “Exporting the Planning.” Choose the patient from the list and press the Import button.

6.5.2. Loading from DICOM Image Data Servers

If a DICOM image data server is chosen as the data source, the PACS search interface opens (Figure 6). Enter at least the patient name and, preferably, an additional limiting feature, such as the date of birth, the sex, the desired modality, or the date of the study. Then press the Return button or the Search button (symbolized by a magnifying glass).

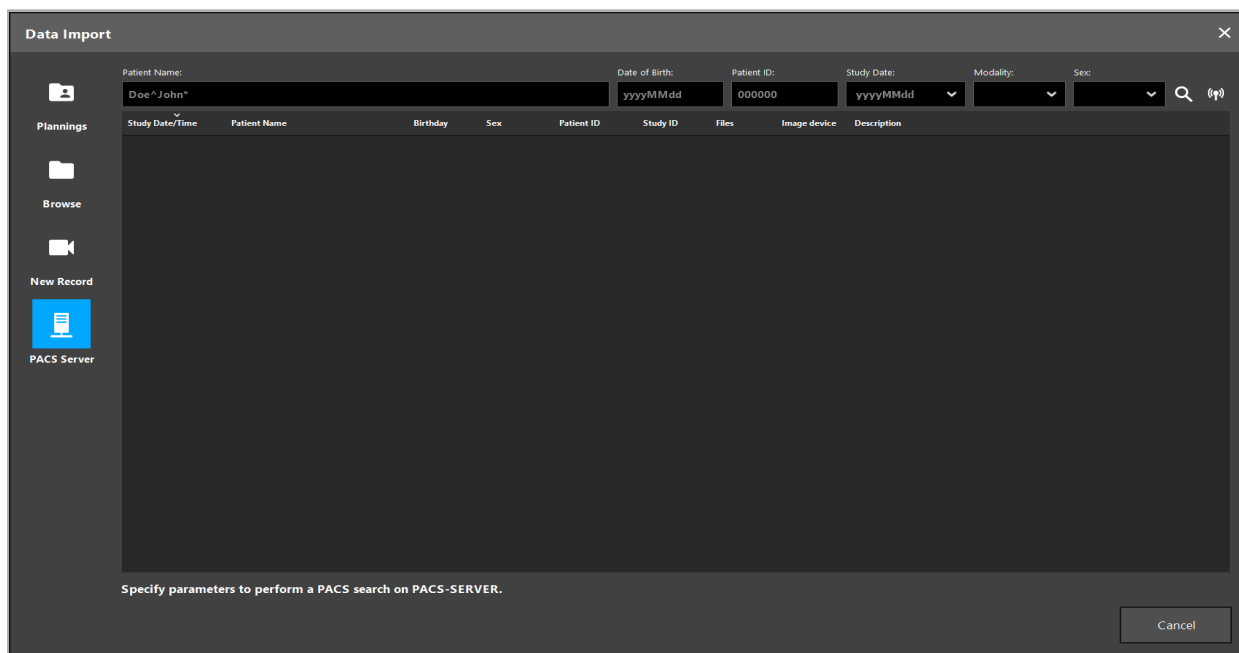
Figure 6 – DICOM Image Data Search

WARNING

When choosing the dataset, make sure to choose the correct image series. Pay special attention to the series ID and the quantity of slice images in the series.

To test the connection to the configured DICOM image server (e.g. when the search request does not provide any results), press the button to the right of the Search button. A test signal is emitted and a success message is returned. In case of error, make sure the network cable is connected and contact your local IT support.

Use the placeholder symbol * to search for names and dates that cannot be specified exactly.

Dates must be entered in the format JJJJMMTT, without blank spaces, dots, or hyphens (19651027, for instance) because hyphens are used for entering time periods.

Contact your local DICOM image data server administrator for the exact syntax of the respective queries.

The search results are shown as a list so that they can be sorted and chosen. Press the Search button again to cancel a running search.

Then choose the desired image series from the list and press the Import button.

6.5.3. Segmentation of Patient Image Data

After importing the patient image data, the Data Import dialog box appears.

The dialog shows a 3D model of the patient and a slider to the right. Using the slider, set a threshold which defines the boundary between patient and air (Figure 7). Adjust the slider so that the 3D model corresponds as well as possible to the patient, and artifacts and gaps in the patient surface are minimized. For CT data, a dividing line between image and space is clearly defined, and a white line is drawn along the threshold to aid the user. When the threshold is defined correctly, use further functions of the dialog (explained in the following), or complete the step by clicking Ok.

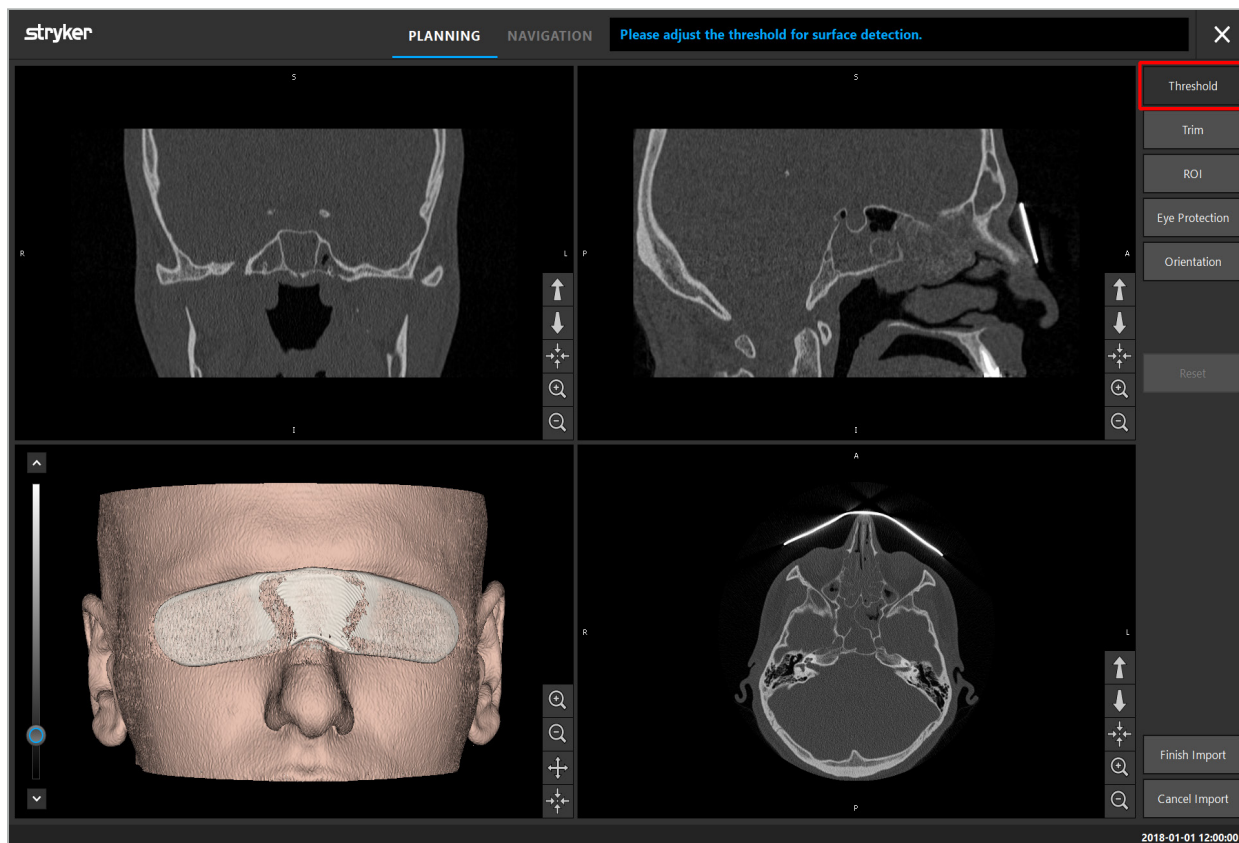
Figure 7 – Choosing the Surface Threshold Value

The head of the patient should be shown facing forward. If not, correct the orientation by pressing the Adjust Orientation button (Figure 8).

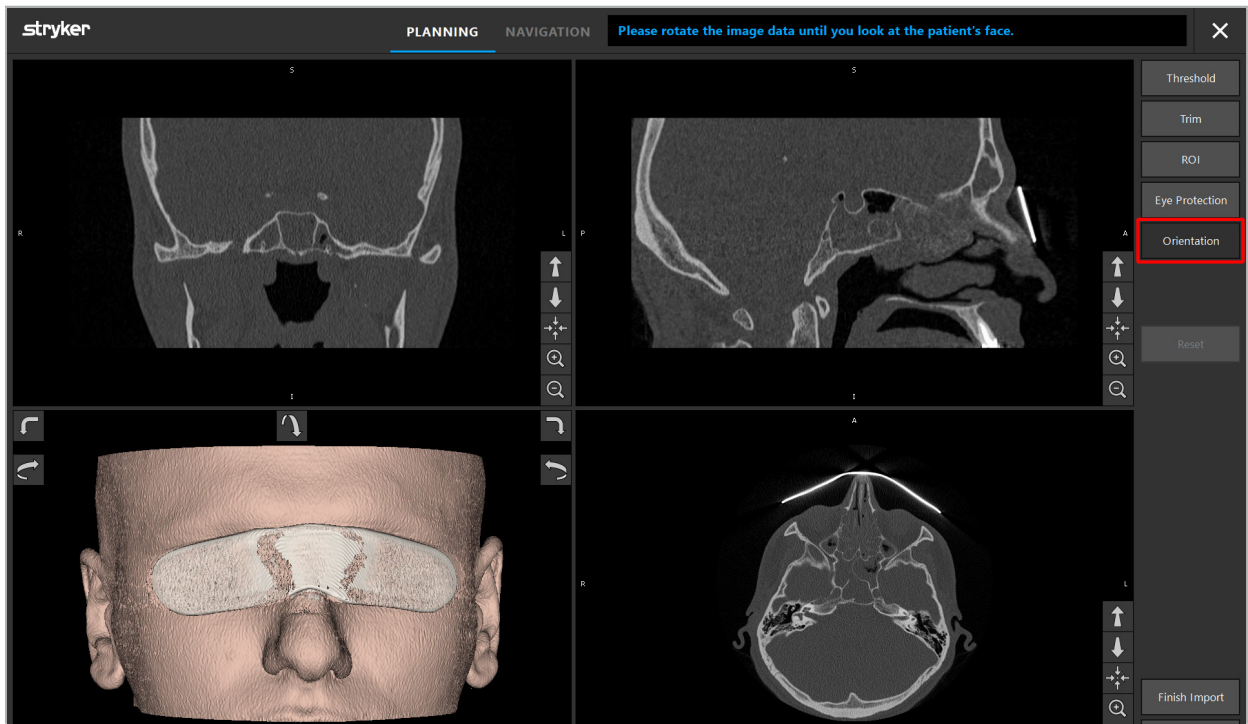
Figure 8 – Adjusting Patient Orientation

If the patient was wearing an eye protection shield during the image scan, this protection can be removed from the 3D model by pressing the Remove Eye Protection button (Figure 9). A dialog to remove the eye protection appears. Adjust the desired radius for the correction with the arrow keys. Then press Remove to remove the eye protection. The Reset button allows the resetting of the original 3D model, including the eye protection.

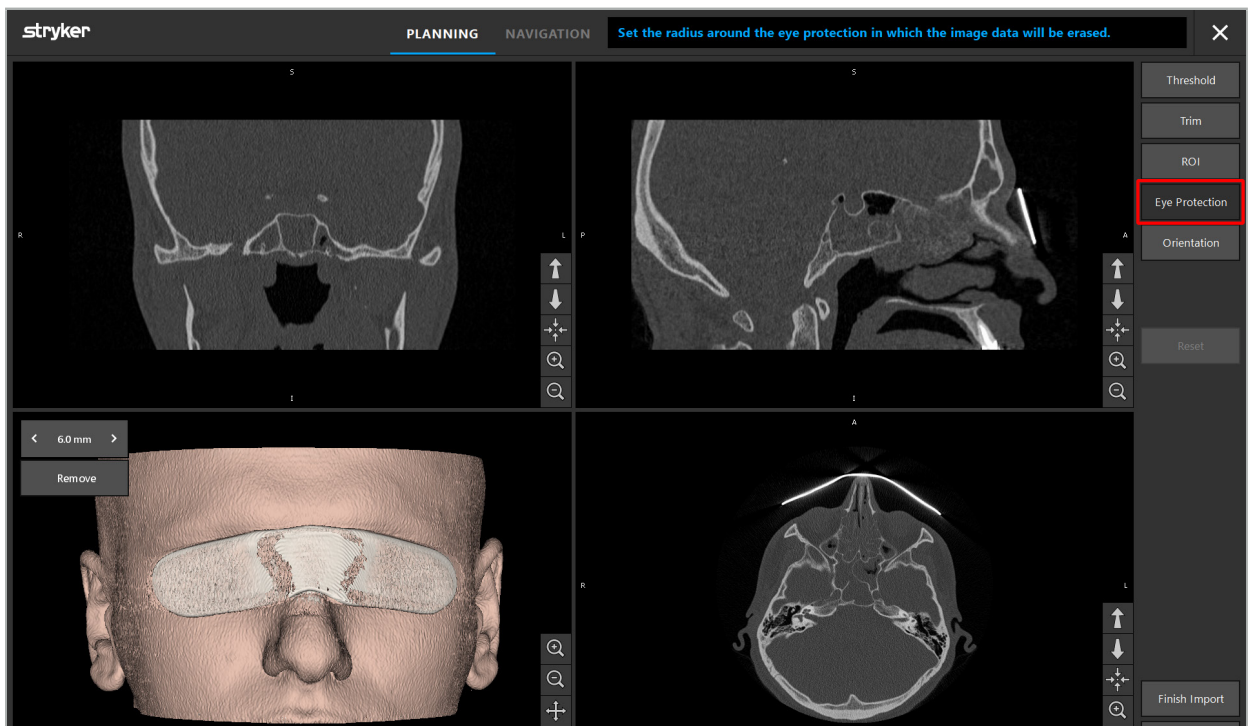
Figure 9 – Removing Eye Protection

WARNING

When removing the X-ray protection shield, it is possible that skin surface particles could also be removed, influencing the patient registration negatively. After the registration, check the accuracy carefully. Should the deviations be too large, the use of the landmark registration is recommended.

Make sure that the 3D model surface does not contain any gaps or artifacts.

When removing the X-ray protection shield, no particles should remain in front of the eyes.

If only a section of the image data is relevant for the procedure, or if a sizable amount of the image data consists of empty space, you can remove unnecessary parts of the volume. For this, click Trim Volume (Figure 10). The dialog now displays the 3D rendering of the image, as well as three slice renderings. A frame is drawn around the volume, enclosing the area to be used after the import. Decrease this area by clicking and dragging one of the corners of the frame. The 3D visualization then shows only the part of the volume that will be available once the import is completed.

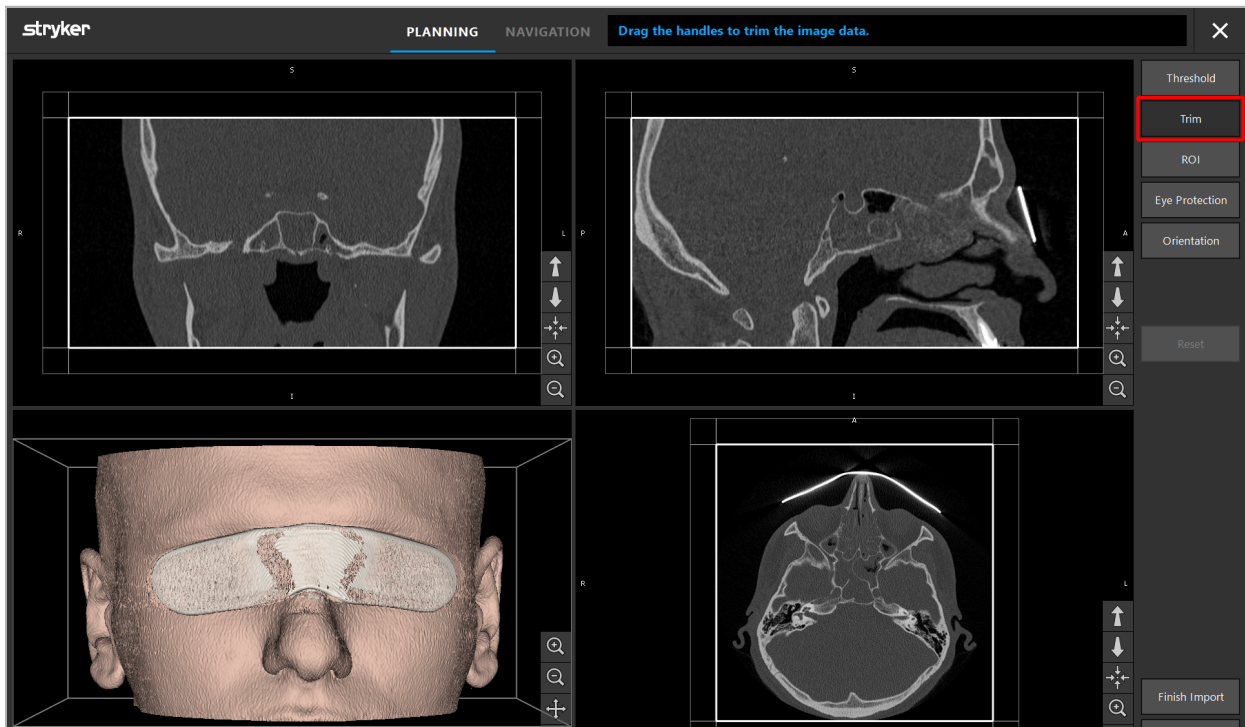
Figure 10 – Trimming Volume

If the image data is very large, the voxel resolution of the image may need to be decreased in order to load it into the computer's memory. The Data Import dialog opens in the trim volume mode and displays a warning message. By removing parts of the volume as described above, the size of the image can be decreased and thus ensure that it imports in its original voxel resolution. It is also possible to continue with the reduced resolution. After loading the image data, it is possible to start with the planning. See Chapter "Selecting the Patient Registration Type."

Depending on the size of the datasets, it can take several minutes to import and load image and planning data.

Only image data in DICOM format can be selected. Should problems occur while loading DICOM datasets, contact the radiologist who carried out the recording, or contact Stryker. For an overview of which DICOM tags must be saved within the export data, see Chapter "Transmitting Image Data."

6.5.4. Loading of Already-Imported Patient Image Data

The PLANNING tab lists already-imported patient image data, including planning information. The information displayed includes patient data and image data, as well as the date of the planning creation and an optional description of the planning.

Choose the desired planning and press the Load button. The Delete Planning button gives you the option to remove old planning information from the list. You can also select Export the Planning. See Chapter “Exporting the Planning.”

After loading the patient image data with the planning data, you can proceed with the planning. To do so, read the following chapter.

If a complete planning was already conducted for the navigation, the NAVIGATION tab allows you to switch to navigation mode and start with the operation.

6.6. Displaying the Loaded Patient Image Data

After loading, the patient image data is shown as 2D slice images (axial, sagittal, and coronal) and as a 3D model (Figure 11). The status bar shows the patient name, date of birth, and the dataset’s session ID. This information is entered by the radiological staff while recording radiological image data and is used to identify the dataset.

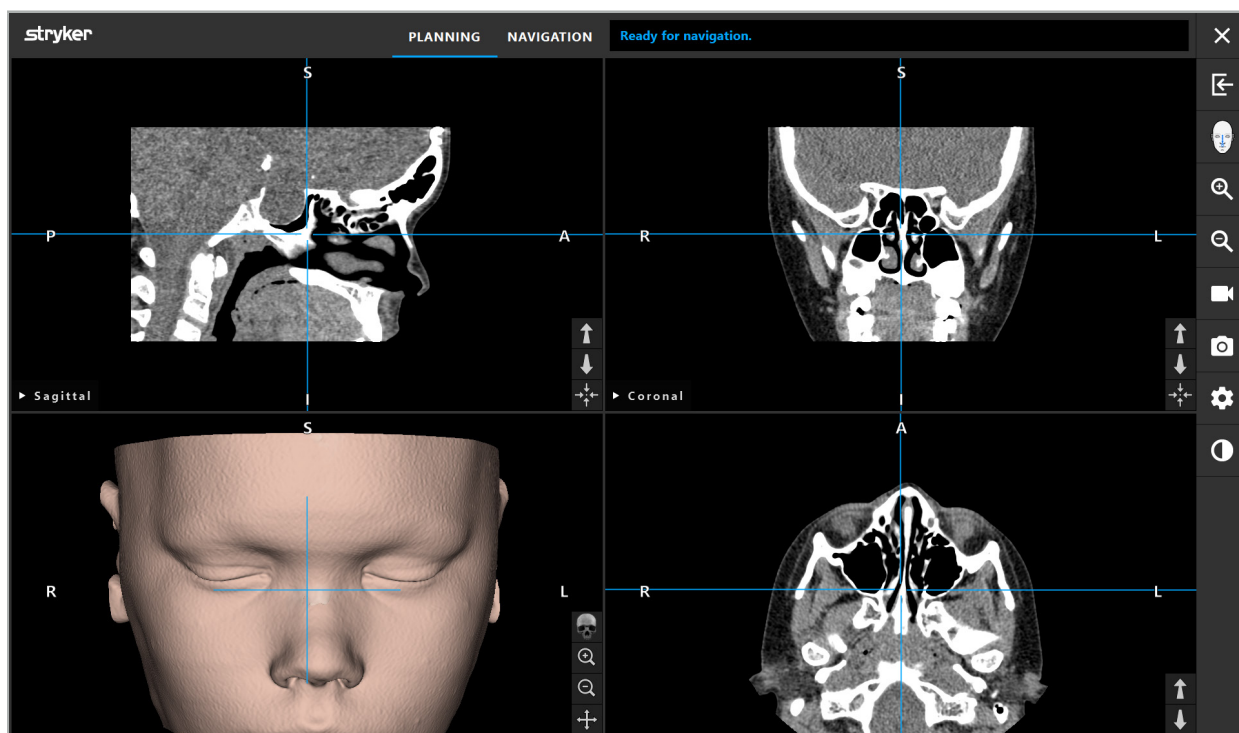
Figure 11 – Patient Image Data After Loading

WARNING

Check all patient image data carefully for accuracy, and make sure that they coincide with the data of the patient that is to be treated.

If you have removed the X-ray protection shield while segmenting the image data, the shield no longer appears in the depiction of the 3D model. The 2D slice images remain unchanged.

6.7. Planning Views

By default, the navigation software shows four windows with different views of the patient image data. These show the 3D model and three 2D cross-section images (axial, coronal, and sagittal).

The following sections explain the respective views as well as functions to adapt these.

6.7.1. Cross-Section Image Views: Axial, Coronal, and Sagittal

Each respective view – axial, coronal, or sagittal – shows one of the recorded cross-section images. The Arrow buttons (Figure 12) are used to move forward and backward between the cross-section images in the respective views.

Figure 12 – Arrow Buttons

Figure 13 – Centering Button

Aside from using the Arrow buttons, you can also select slices by shifting the red intersection lines. Click the horizontal or vertical line or their intersection, and drag until the relevant slice is shown.

The Centering button (Figure 13) centers the display of the cross-section on the crossing point of the red intersection lines, so that these are in the middle of the display.

WARNING

If false information is entered regarding the patient position during the imaging process, the labeling of L (left) and R (right) will not be correct in the axial view. Ask your radiologist to save the patient image data again in the correct orientation.

6.7.2. Moving the Patient Image Data

You can move image data horizontally and vertically in the radiological image data views – axial, coronal, and sagittal. Thus, a relevant detail can be moved into the middle of the display, especially when zoomed in.


To move patient image data, click a free space in the display and drag until the relevant detail is ideally positioned. During the process, the mouse cursor is shown as a closed hand. Alternatively, use the touchscreen to move image data.

6.7.3. 3D View


In this view, a 3D model of the patient is displayed. Click in the view and move the button at the same time to rotate the 3D model. Alternatively, use the touchscreen to rotate the model. Adapt the view of the 3D model with the following buttons:

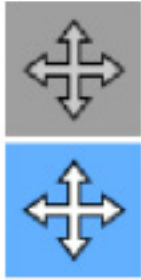Zoom in and out of the 3D model by clicking the magnifier buttons. Alternatively, use the touchscreen to zoom in and out.

Click the Move Model button to move the 3D model vertically and horizontally. The active function is shown by a white highlighted symbol. Click in the view and drag horizontally or vertically until the 3D model reaches the desired position. Deactivate the function by pressing the button again. If this function is deactivated, the 3D model can be rotated. In this state, the background of the button is darkened. Click in the view and drag down in order to rotate the 3D model. Deactivate this function by pressing the button again. Alternatively, use the touchscreen to move the model.

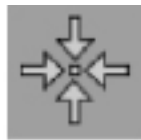The Reset button resets all changes related to magnifying, moving, or rotating the model. This function can be activated while in move model or rotate model mode. Alternatively, use the touchscreen to reset.

6.7.4. Inline and Probe View

Inline and Probe views (Figure 14) are based on the instrument orientation: The displayed cross-sections are updated according to the spatial direction of the pointer instrument.

- Inline View 1 and Inline View 2 offer two orthogonal cross-sections along the instrument axis which runs through the center of the pointer tip.
- Probe View displays a cross-section through the pointer tip.


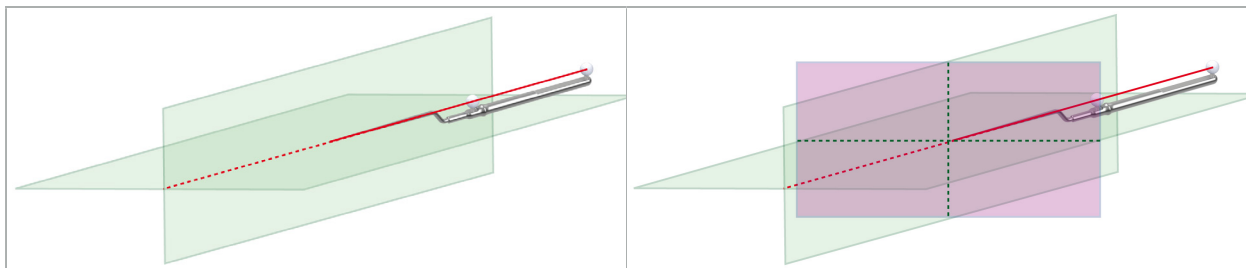
Figure 14 – Inline Views Along the View Axis of the Pointer Instrument (Green, Left), and Probe View as the Cross-Section Through the Pointer Tip (Magenta, Right)

6.7.5. Video View

This view displays the video image of the camera processor that is connected to the navigation unit.

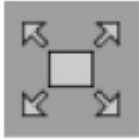The Full Screen button zooms in the video image display so that it covers the entire image area reserved for display. All other views are covered by the video image during full screen mode. If this button is pressed again, the prior view configuration is reset. Alternatively, use the touchscreen to change the view.

The Scale Video button enlarges the video image so it covers the entire video view. Parts of the video image may be cut off in this mode, depending on the aspect ratio of the video view and the video image.

i If the video continuously shows a black screen, verify that the camera unit is on, it delivers an image, and it is properly connected to the navigation unit with a DVI or SDI cable.

6.7.6. Configuration Menu: Video

The video tab of the configuration menu (Figure 15) allows to change brightness and saturation of the video, as well as zooming up to 2x into the video image.

i If the video display shows the message “No Video Input,” make sure that the navigation unit is properly connected to the DVI or SDI output of the endoscopy unit.

Depending on navigation model the video tab may also offer choices for signal search parameters (Figure 15). It also allows you to deactivate certain input connectors.

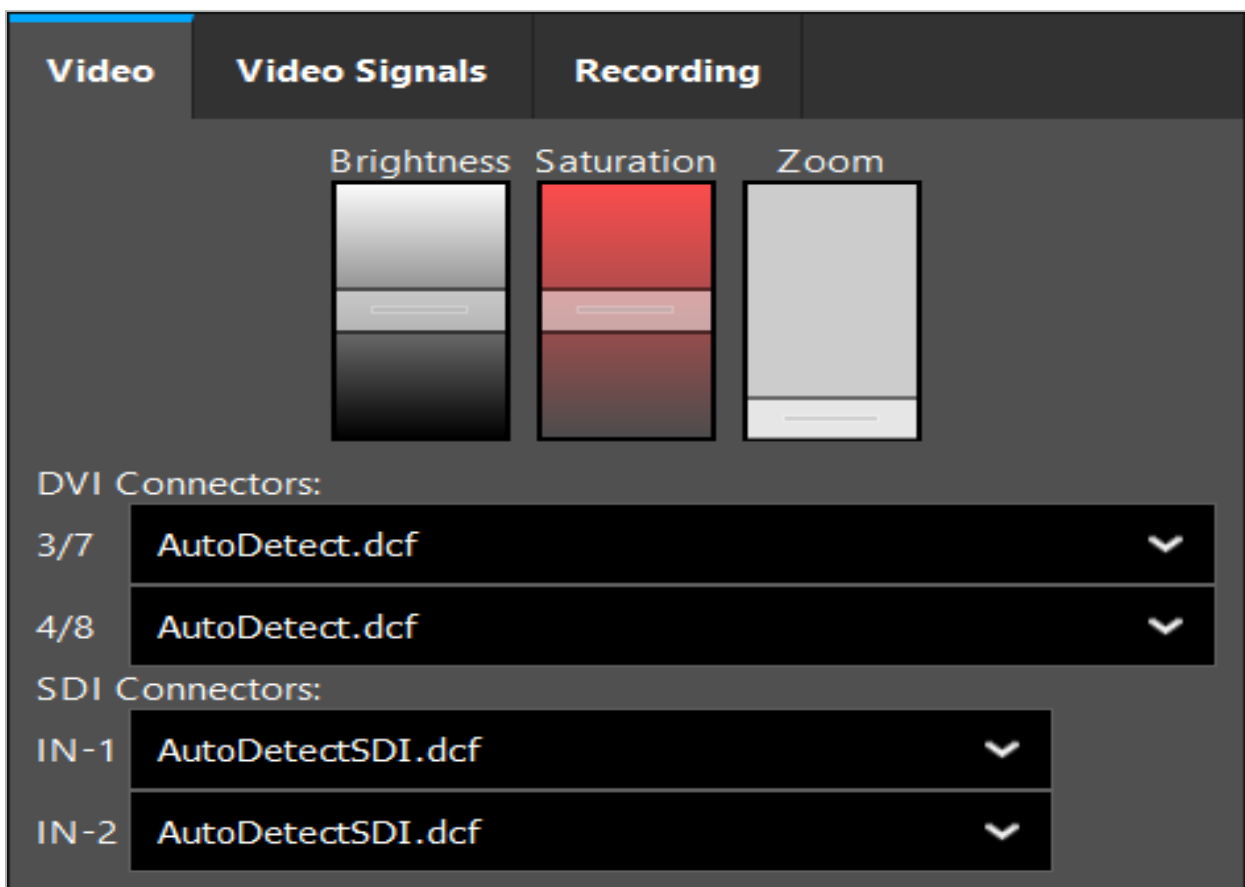
Figure 15 – Configuration Menu: Signal Detection in Case of Missing Video Input

If two video sources are connected at the same time, then one connector should be deactivated to ensure that the desired signal is displayed.

6.7.7. Changing a View Type

Each of the four sub-windows allows for the view type to be switched. To do so, click the arrow next to the description of the respective view. A list of views is displayed (Figure 16). Choose the view you want to be shown in the window.

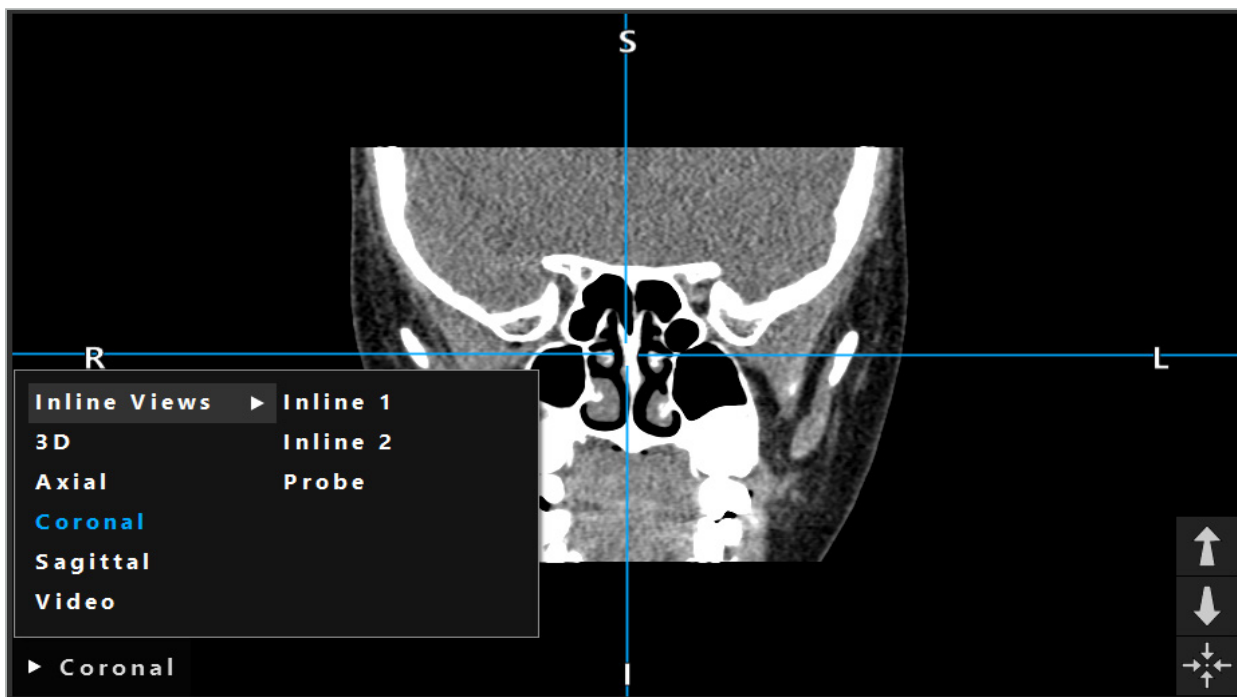
Figure 16 – Changing a View

6.7.8. Changing the View Configuration

In planning and navigation mode, the navigation software shows up to six windows with different views of the patient image data. By default, the software shows four windows with different views of the patient image data: the 3D model, and three 2D cross-sections (axial, coronal, and sagittal), arranged 2x2. You have the option to change the view types displayed within a particular arrangement.

The navigation software allows for changing the arrangement of the different views. Press the Display button (the black button in the right control menu) (Figure 17). The view configuration menu enables you to quickly switch between the saved views.

Standard arrays include a 2x2 array, a 2x1 array with two vertically arranged views, a 2x3 array with six views, as well as a full screen array.

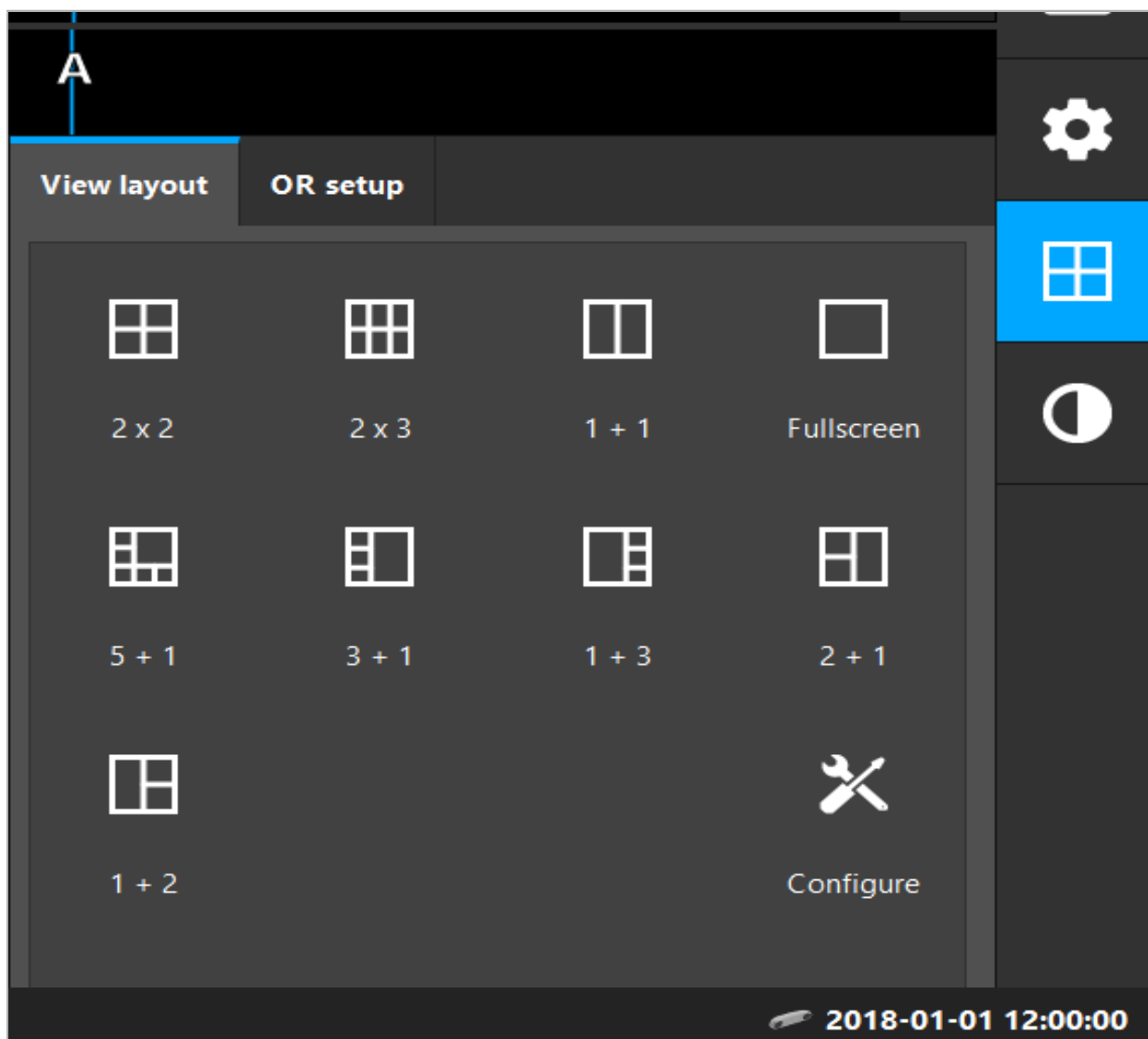
Figure 17 – View Configuration Menu

6.7.9. Managing View Configurations

Save and manage your preferred view configuration by pressing the Configuration button in the display menu. The Configure View Layouts dialog box appears (Figure 18). It gives you the following options:

- Press the Add Preset from current views button to save the current layout and views as a new configuration. You are then asked to enter a name for the configuration.
- Delete the selected configuration with the Delete button.
- Rename the selected configuration with the Rename button
- Press the Apply button to accept the selected configuration and close the dialog box.

i It is not possible to delete or rename standard configurations. You can only delete or rename the configurations you have defined yourself.

When the software is closed, your last view configuration is saved automatically. It is loaded automatically next time you start the software.


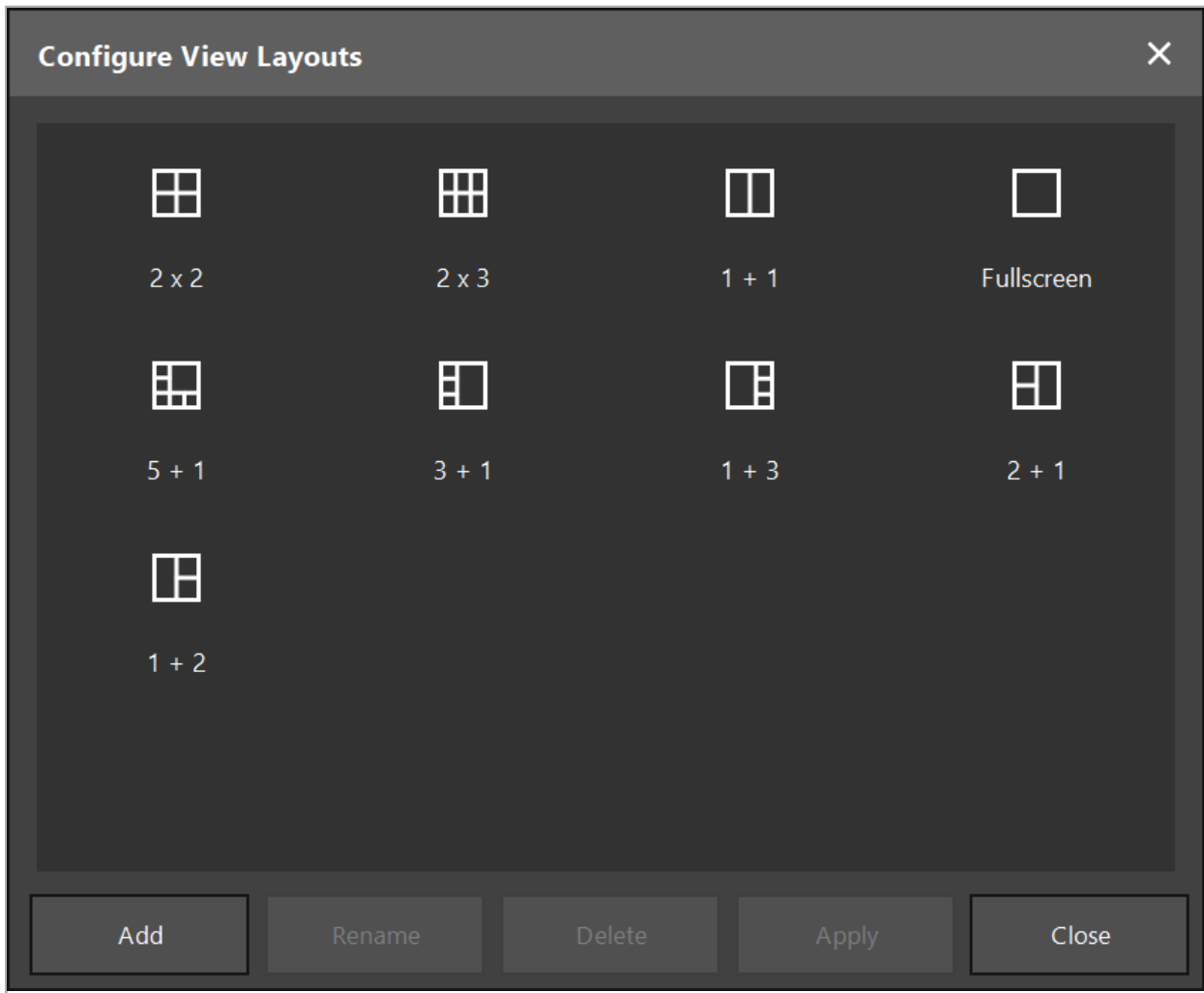
Figure 18 – Dialog Box for Managing Views

6.7.10. Adjusting Contrast and Brightness for Patient Image Data and Video

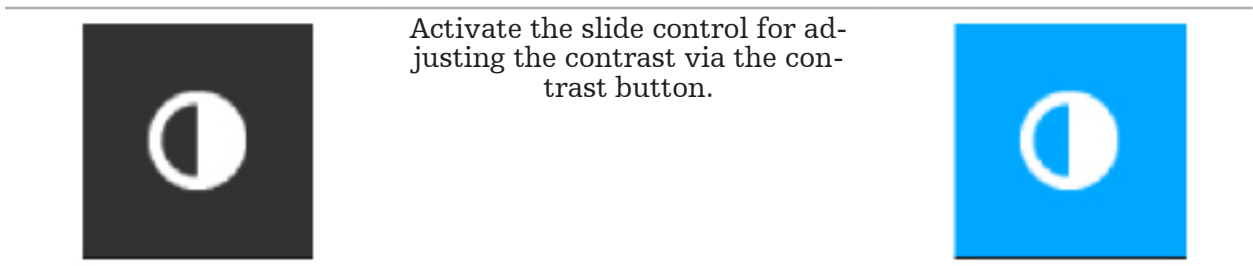Contrast and brightness of the radiological patient image data (axial, sagittal, and coronal) are adjusted via a slider (Figure 19). The slider is located on the right side of the navigation bar and is activated via the Contrast button. (In software version 1.3.0 or older, the slider is always activated.)

The slider has three control elements: the upper element adjusts the brightness, the middle element adjusts the centers of the new set range, and the lower element defines the contrast.

i Adjust contrast and brightness so that the relevant information is optimally displayed in the patient image data.

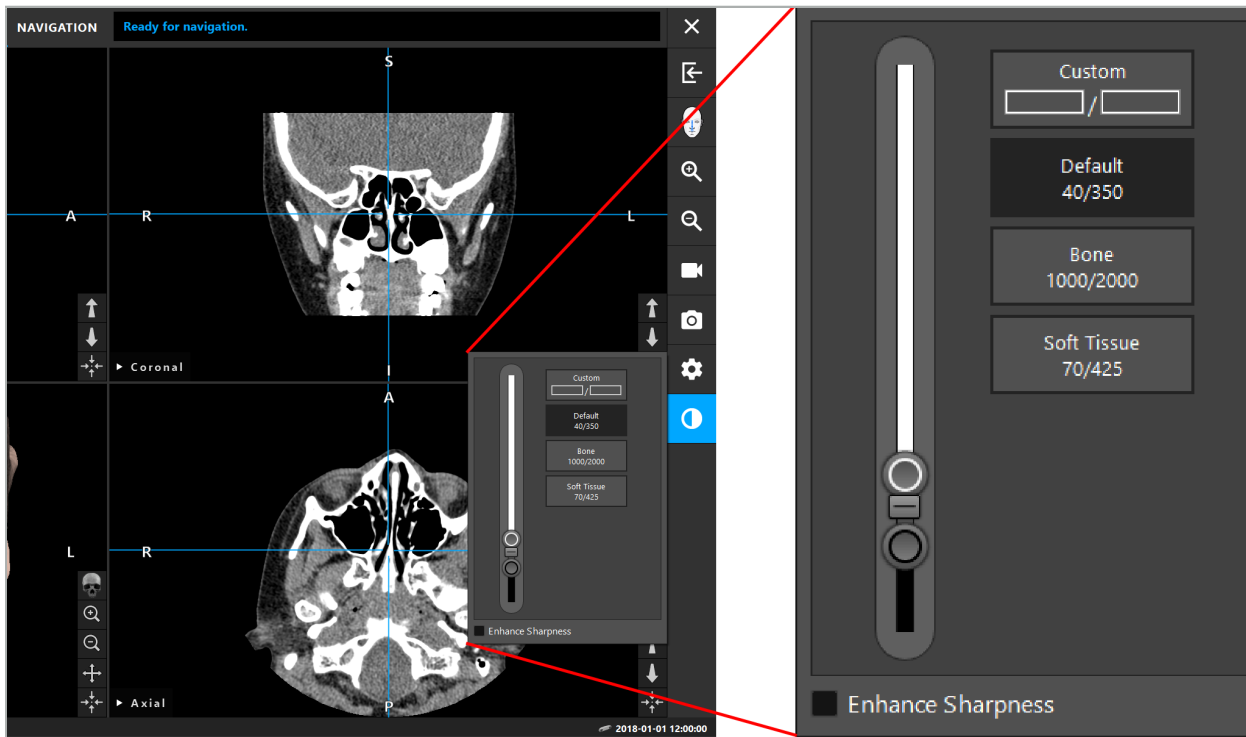
Figure 19 – Slider for Brightness and Contrast

Additionally, next to the slider there are buttons to quickly access predefined gray level window settings. They are given in Hounsfield units and in the form of level/width. The Default window is taken from metadata in the DICOM files of a dataset. The Custom window can be used to enter arbitrary Hounsfield unit values and is updated automatically when changing the slider values.

In addition to altering the display of the patient data, you can adjust the endoscopic video image. For this, switch into the Video tab of the contrast controls (Figure 20). You can change the brightness and saturation of the video, as well as zooming up to 2x into the video image.

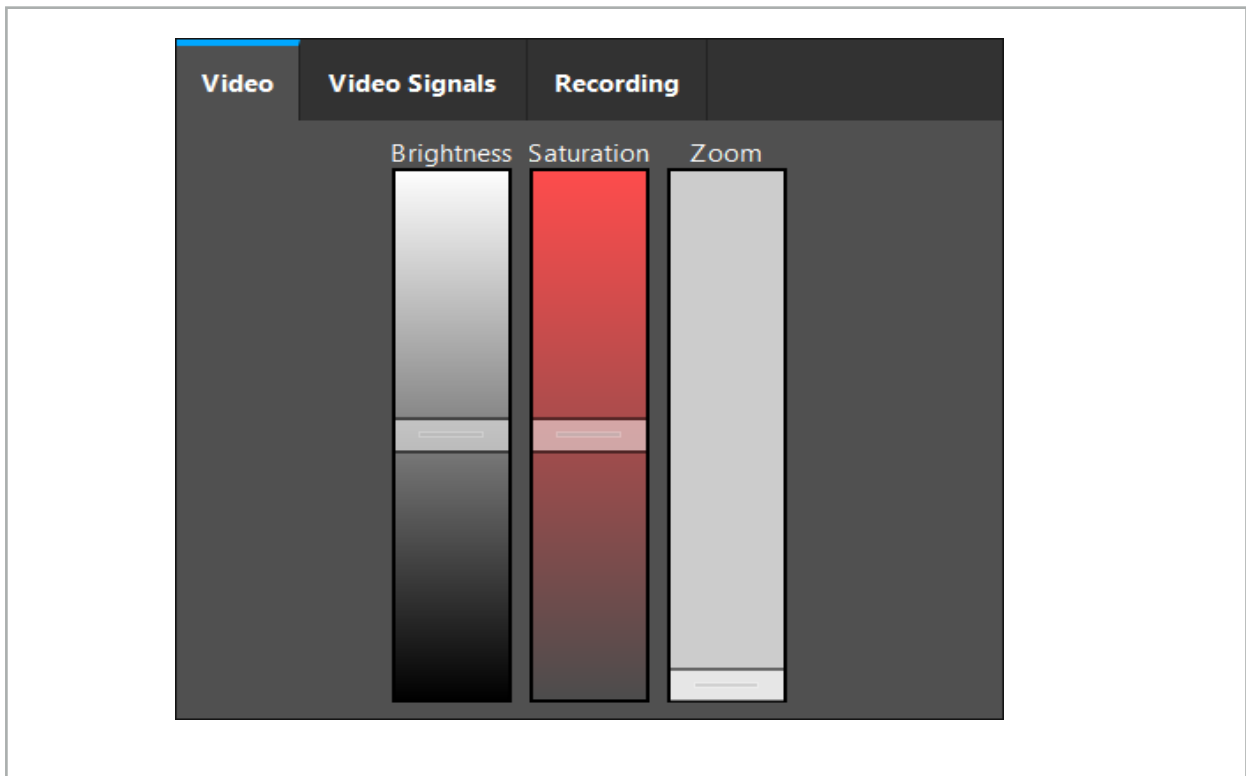
Figure 20 – Adjusting Endoscopic Video

6.7.11. Zooming In and Out in the Patient Image Data

The radiologic patient image data view (axial, sagittal, and coronal) can be zoomed in and out with the magnifier buttons (Figure 21).

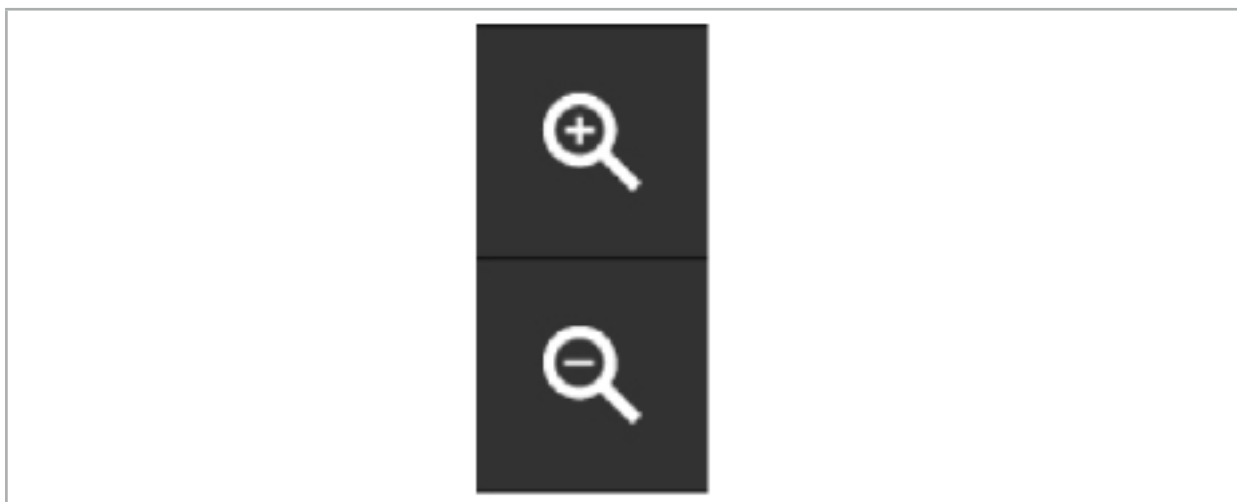
Figure 21 – Magnifier Buttons

6.8. Importing Additional Datasets for Image Fusion

Use the Image Fusion function to combine various image datasets (for instance on bone structures from a CT recording or soft tissue from an MRI recording).

Press the Loading Fusion Images button (Figure 22) to open a dialog box which shows the loaded image dataset.

Figure 22 – Loading Fusion Images

Figure 23 – List of Loaded Fusion Images

To add further datasets, press the Add new dataset button. A new dialog opens to choose more image datasets. See Chapter "Loading Patient Image Data."

Additionally, it is possible to mark the different datasets with colors. Choose the dataset that you want to add. Make sure that the dataset belongs to the same patient as the one you already loaded.

Once you choose a valid volume, the Image Fusion dialog box (Figure 23) appears after the loading process. Here, the datasets are first preregistered roughly and then arranged together precisely.

6.8.1. Approximate Registration

The left window shows the 3D model of the already loaded dataset (referred to as “Reference” from now on), while the right window visualizes the newly added dataset (“Target”) (Figure 24). Similar to the Defining Surface Threshold Value dialog, the threshold value which regulates the volume size can be changed with a slider on the right side of the dialog. See Chapter “Segmentation of Patient Image Data.”

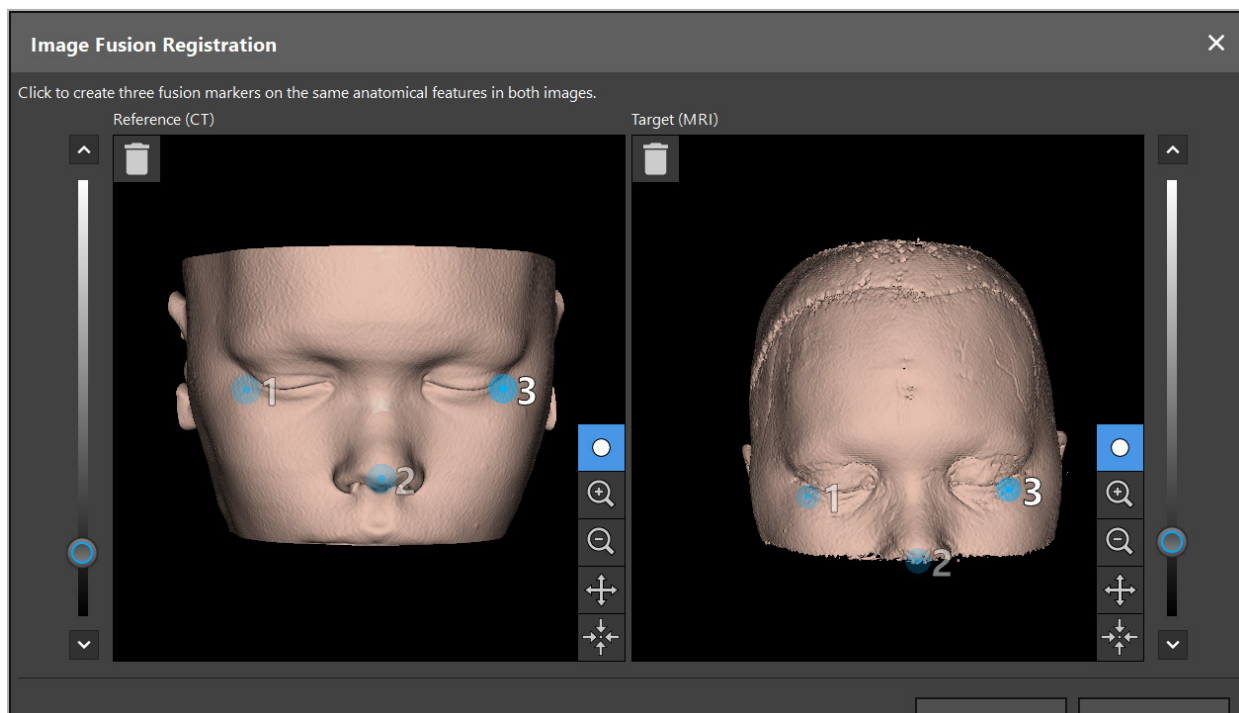
Figure 24 – Image Fusion: Approximate Registration of Fusion Images

Choose the threshold value so that the created patient surface comes closest to the surface of the reference dataset. Then click the patient surface to create a colored marker. Move the marker on the patient surface by dragging. Note that it is not possible to position the marker outside of the patient surface.

In this manner, create three markers each for the two images, and position them on anatomic structures that are well visible in both images. Markers of the same color must be positioned on the same structures in both images, as shown in Figure 24.

i It is recommended not to position the markers too closely next to each other, as the approximate registration is more precise.

6.8.2. Precise Registration

Based on the markers used for approximate registration, both datasets are made congruent. Check the quality of the registration carefully by observing the fusion image shown as 2D slice images. The target volume is displayed gold-colored, and the reference volume in blue. The contrast of both datasets is predefined by means of the modality. The bone structures from the CT images are shown, and the soft tissue from the MRI images (Figure 25).

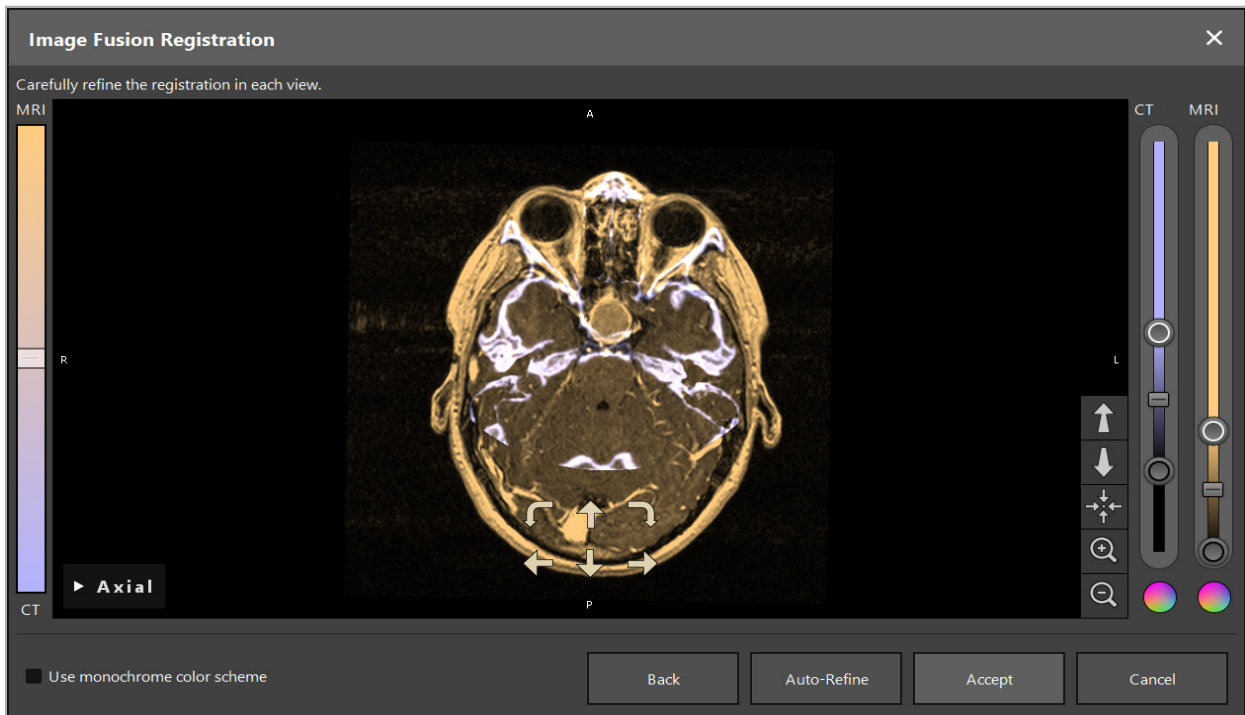
Figure 25 – Image Fusion: Precise Registration of the Datasets

i If the approximate registration was too imprecise, you can return to the 3D views by pressing the Return button. However, any fine-tuning done in the process is lost.

Use the slider on the left side to change the fusion image composition. It is possible to only show the Reference volume (blue) or the Target volume (gold). If the slider is positioned in the middle, both volumes are shown with maximum intensity.

If you press the Correct Automatically button, the software brings the two volumes in optimal accord with each other. For this, a sufficiently precise approximate registration is necessary. Applying the automatic registration repeatedly can enhance the result.

Conduct a manual registration with the four movement arrows and two rotation arrows within the image screen. Thereby, the target volume is moved or rotated in the arrow's direction. In this manner you can adjust the registration in all three views (axial, coronal, and sagittal). When the registration is sufficiently precise, click Accept.

WARNING

- Check the registration carefully in all slices and all views. An imprecise registration can lead to a faulty depiction during navigation. Change the contrast view from time to time, for example in order to compare the soft tissue in both datasets.
- When modifying a fusion registration check the position of all planned objects carefully. All planned objects are always attached to the primary dataset and do not move when fusion registration is changed.

6.8.3. Image Fusion with PET and SPECT Image Data

Image data from the modalities positron emission tomography (PET) and single-photon emission computed tomography (SPECT) can be imported as secondary datasets in order to use a single or combined display of the PET/SPECT image data with the primary CT or MRI image dataset during the surgical planning and intraoperative navigation.

When importing PET or SPECT image data, it is highly recommended first to import matching CT image data of the PET-CT or SPECT-CT device which belongs to the same study and which is stored with the same DICOM Frame of Reference UID. This allows an accurate and automatic registration of the possibly low-resolution PET or SPECT image data to the primary dataset. Otherwise, a manual registration to the primary datasets must be conducted. See Chapter "Approximate Registration."

When scanning data sources of image data during data import, the system recognizes the modality of the DICOM image data automatically and displays this information in the Import dialog Modality column. The PET or SPECT image data is marked “PET” and “SPECT.”

Most image modalities are displayed as grayscale images, unless specified otherwise. PET and SPECT images show metabolic activity in the body and are typically displayed in a false color scheme.

When adding a dataset of this modality to the planning, areas of high activity are shown in red, and low activity in blue (Figure 26). Select different color schemes in the color selector accessible via the Fusion dialog. See Chapter “Importing Additional Datasets for Image Fusion.”

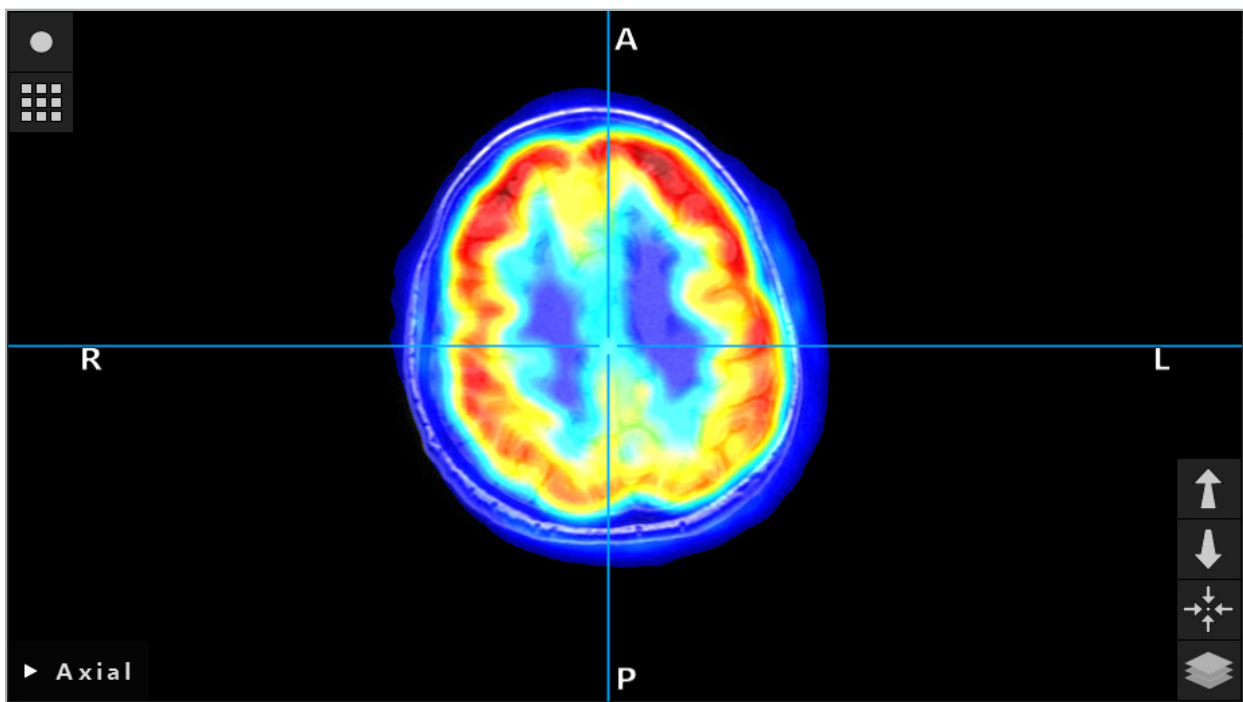
Figure 26 – Combined Display of CT and Color-coded PET Image Data

i Color-coded display of image data is deactivated by default outside the image-to-image registration dialog. To use the false color scheme, remove the check mark from the Use monochrome color scheme check box in the Fusion dialog. For best results, you should also change the color of other datasets in the planning to white, so they won't influence the perceived color of the SPECT or PET dataset.

6.8.4. Configuration Options

After the fusion is finished, the view can still be configured in the following ways:

Cross-fading intensity of fusion and reference volume: By default, all datasets are shown in full intensity. However, this can later be changed by means of the contrast menu (Figure 27). Each dataset can reduce its weight in the resulting fusion image, or be completely deactivated. The Hounsfield window that controls the contrast in different areas of the image can also be adjusted separately for each dataset.

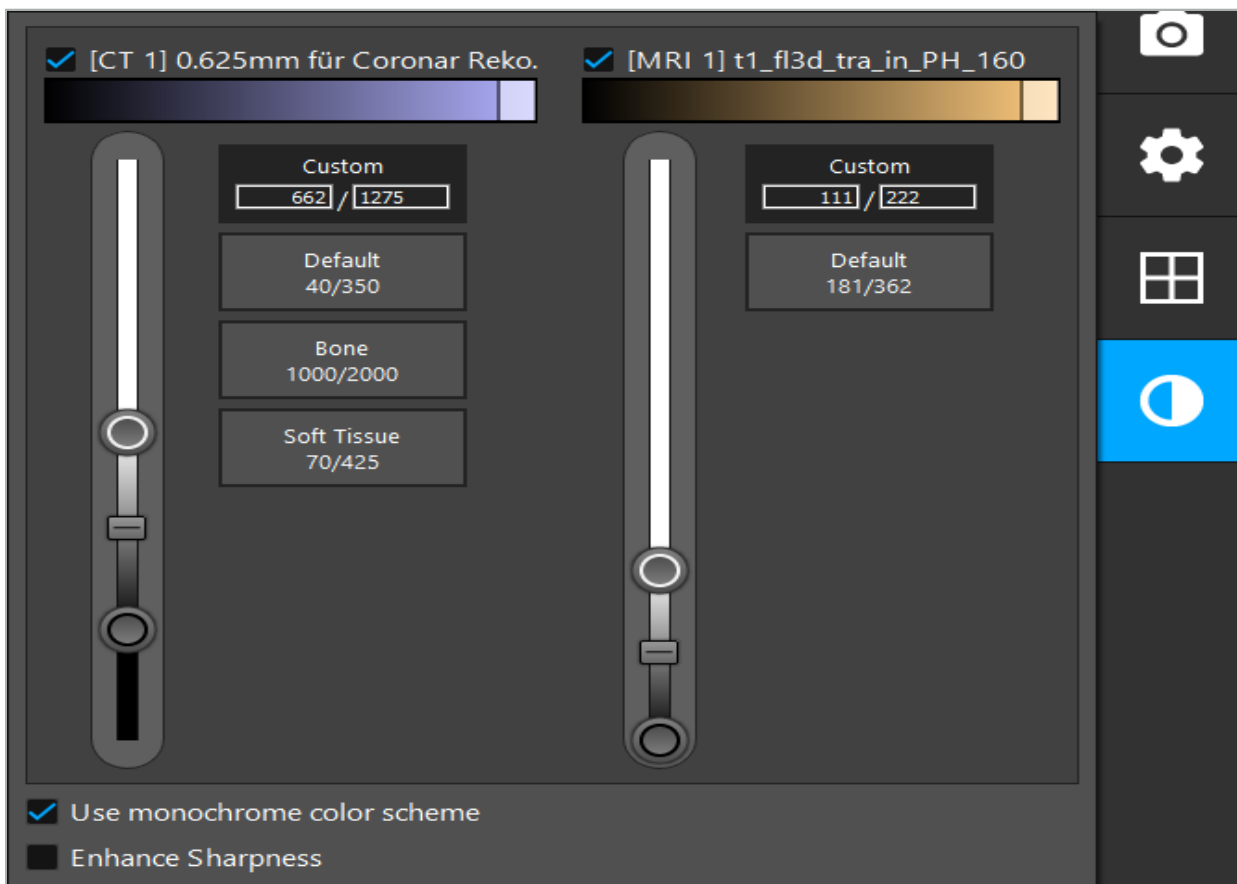
Figure 27 – Intensity Slider for Image Fusion

Additionally, you can select which datasets are fused together for each slice view to produce the overall display by clicking the button in the lower right corner of the view. This opens a small menu where you can enable and disable certain datasets (Figure 28). For example, if you have multiple datasets that have an optimal in-slice resolution for some orientations, e.g. sagittal, you might want to prefer this dataset to be displayed in the sagittal view.

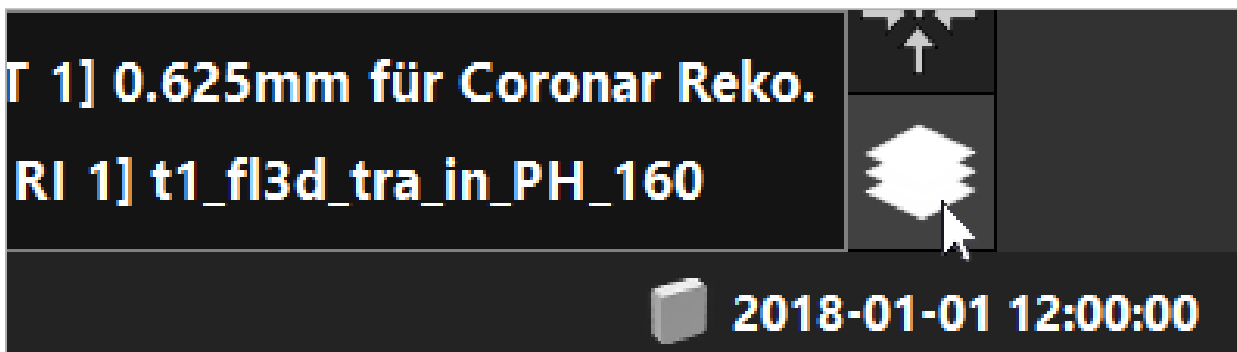
Figure 28 – Selection of Displayed Datasets Per View

6.9. Selecting the Patient Registration Type

Before every application, it is necessary to conduct an image-to-patient registration to determine the position of the patient and to match the patient image data. Clicking the Image-to-Patient Registration button (Figure 29) opens the Patient Registration dialog box (Figure 30).

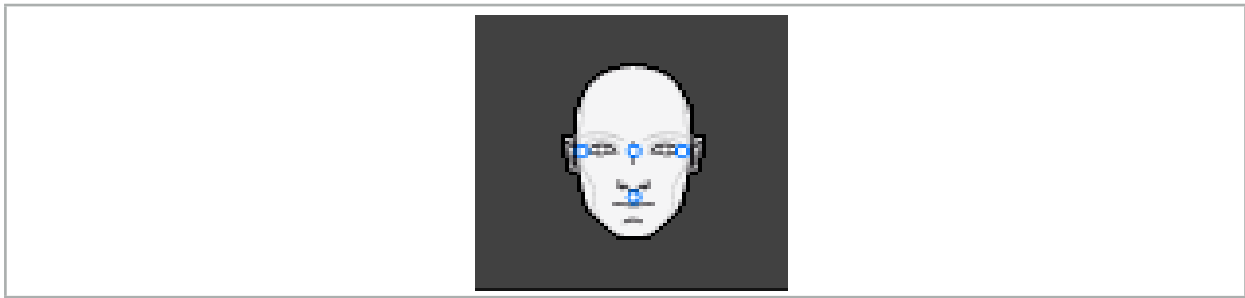
Figure 29 – Image-to-Patient Registration

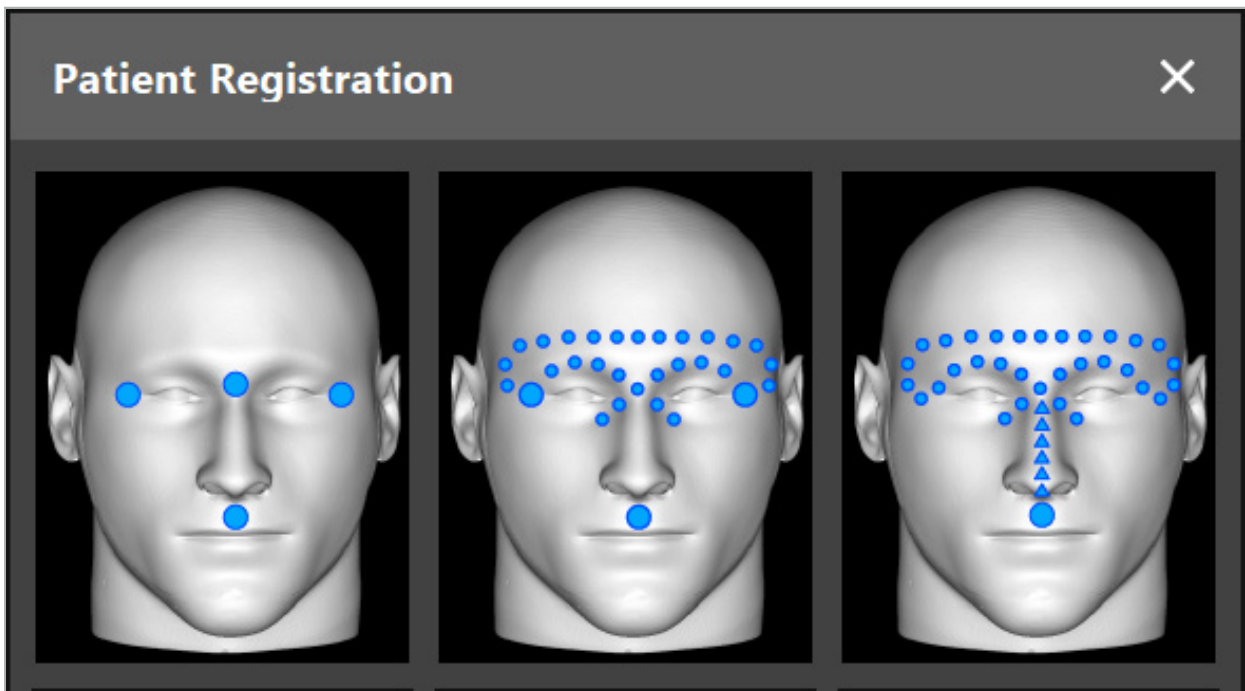
Figure 30 – Selecting Registration Types for ENT

The software offers several types of image-to-patient registration:

Landmark

At least three clearly perceptible landmarks are marked in the patient image data and subsequently touched precisely with a pointer on the patient. This type of registration is especially suitable in the presence of artificial landmarks (e.g. bone screws, fiducials) in the operating area and in the patient image data.

Surface

At least three clearly perceptible landmarks are marked in the patient image data and subsequently touched with a pointer on the patient. Afterwards a pointer must be moved over the patient's surface to collect more points for the matching. This type of registration is especially suitable when registration only takes place on the skin. For surface registration, at least 40 cm² of skin near the situs are necessary.

Enhanced Surface

Here a specific path in patient image data is generated which must be initially moved along on the patient's surface. Afterwards a pointer must be moved over the patient's surface to collect more points for the matching. This type of registration is especially suitable for sinus surgery. For enhanced surface registration, at least 30 cm² of skin left and right of the nose, as well as the nose itself, are necessary.

6.10. Image-to-Patient Registration: Landmark

If the landmark image-to-patient registration type is selected, this is displayed by a corresponding activated symbol (Figure 31).

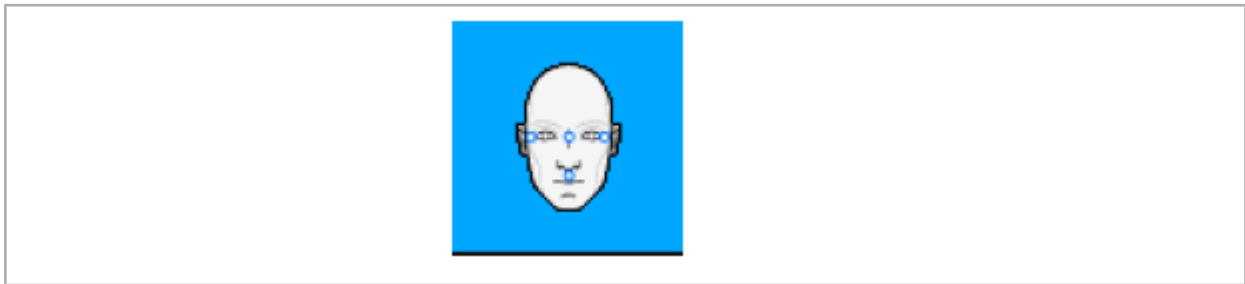
Figure 31 – Landmark Registration Type

For the landmark image-to-patient registration type, it is necessary to place three to five landmarks on prominent places in the patient image data. The places should be chosen so that they can be found and touched precisely with a navigated pointer (pointer instruments) in navigation mode.

6.10.1. Defining Landmarks

WARNING

Do not define landmarks on sensitive structures such as eyes that might be injured by touching with the tip of the pointer.

Landmarks can be defined in the three orthogonal views of the 2D slice images (axial, coronal, and sagittal) and in the 3D model. Pay attention to the following:

- Three to five landmarks can be defined. The distance between two landmarks should be at least 4 mm.
- **i** Landmarks should be positioned on prominent anatomical structures which only move very little when touched with the tip of the pointer. Positions on the forehead, the temples, or below the nose are preferred.
- The landmarks should be close to the operation area.
- Three landmarks must not be positioned on a single straight line. Landmark positions must define a plane.
- If artificial landmarks such as bone screws exist in the patient image data, landmarks can be placed upon these structures. The registration with artificial landmarks can enhance the precision of the registration process.

For use in middle ear surgery it is recommended to position three to four bone screws around the situs entry and another screw for reference in 3-4 cm offset (Figure 32).

Figure 32 – Recommended Position of Bone Screws

6.10.2. Defining Landmarks in the 3D Model

To add landmarks to the 3D model, click the place where the landmark should be set. Alternatively, use the touchscreen to place new landmarks (Figure 33).

The added and selected landmark appears as a semi-transparent blue sphere with a small solid center.

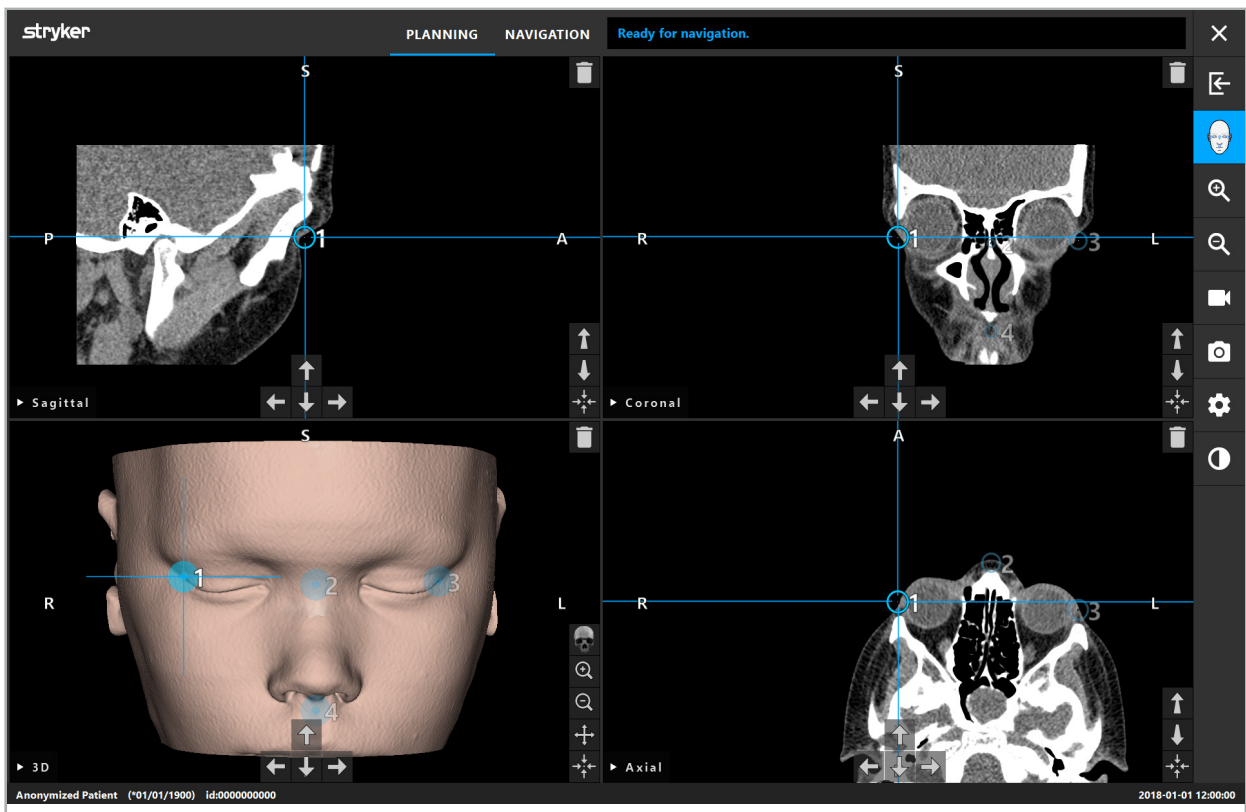
Figure 33 – Placing Landmarks

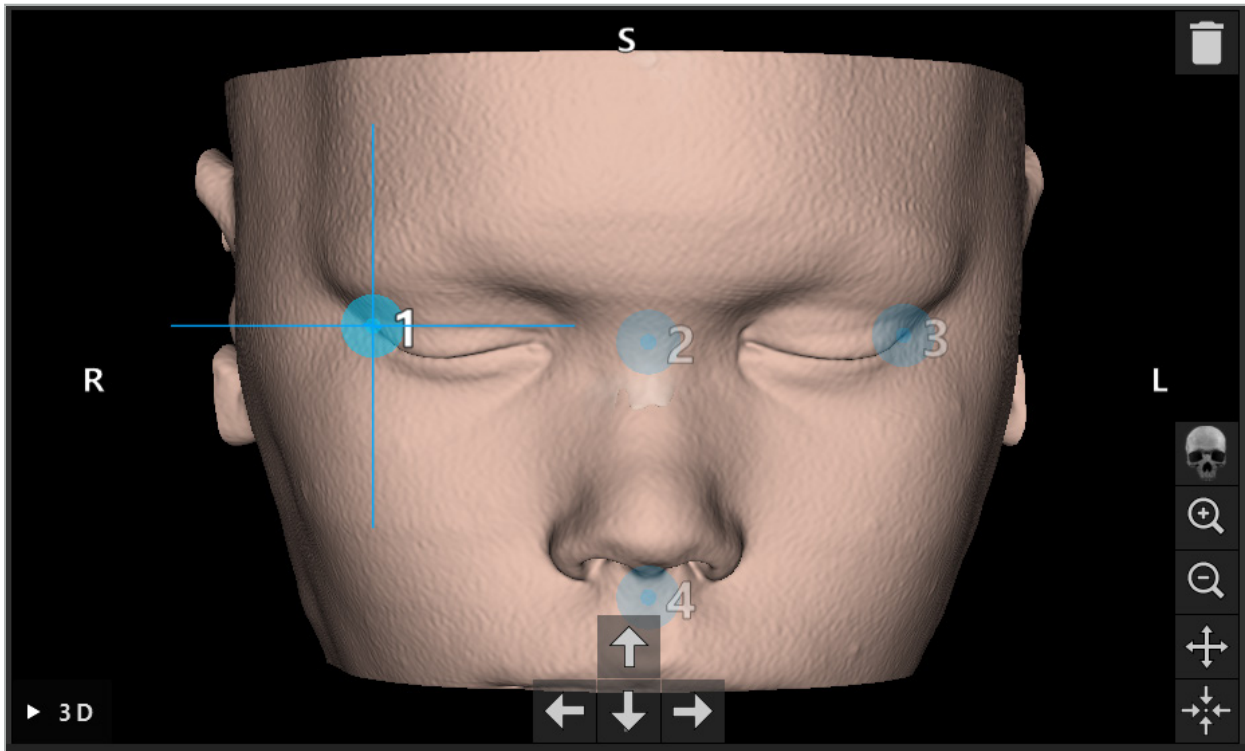
Figure 34 – 3D View

i In the 3D view, landmarks can only be placed directly on the 3D model surface. It is not possible to place landmarks outside of the 3D model.

6.10.3. Placing Landmarks in 2D Cross-Section Images

To add landmarks to the 2D cross-section images (Figure 35), click the place where the landmark should be set. Alternatively, use the touchscreen to place new landmarks.

The added and selected landmark appears as a blue sphere with a target cross.

i For further details, see Chapter “Defining Landmarks.”

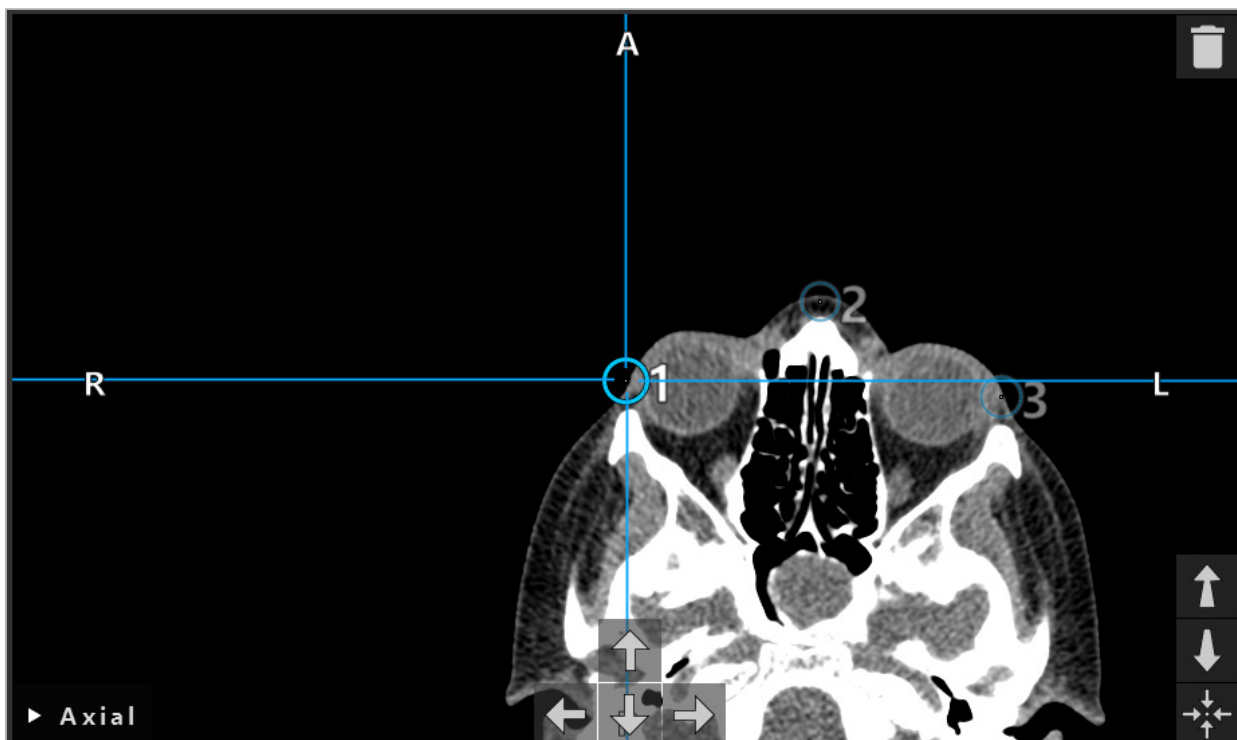
Figure 35 – Adding Landmarks to the 2D Slice Views

6.10.4. Moving Landmarks

The position of existing landmarks can be changed in all views. Click to choose a landmark and drag the landmark to its new position. The selected landmark is shown in yellow. Alternatively, use the touchscreen to position landmarks.

For precise positioning of the landmarks, use the Arrow buttons of the 2D cross-section data. First choose a landmark, which is then shown in yellow. The Arrow buttons in the 2D slice images are now connected to the position of the selected landmark. Move the landmark position through the slices of the cross-section data by pressing the respective Arrow button.

6.10.5. Removing Landmarks

To remove existing landmarks, double-click to select the desired landmark. Alternatively, use the touchscreen to remove landmarks.

6.11. Image-to-Patient Registration: Surface

WARNING

Do not define landmarks on sensitive structures such as eyes that might be injured by touching with the tip of the pointer.

If the surface image-to-patient registration type is selected, this is displayed by a corresponding activated symbol (Figure 36).

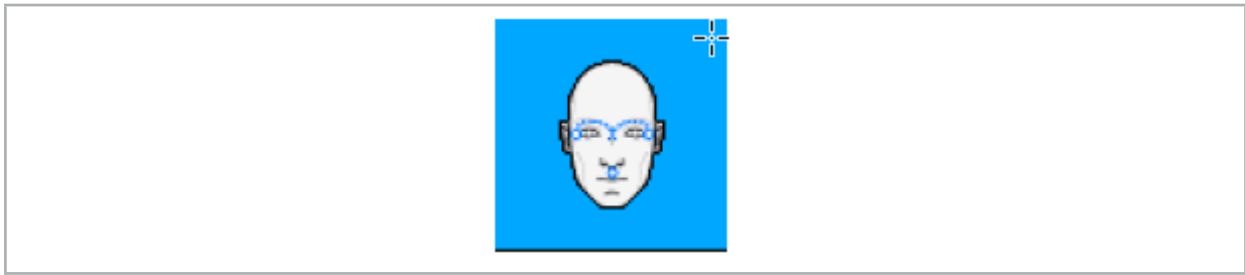
Figure 36 – Surface Registration Type

To use the surface image-to-patient registration type, it is necessary to define three landmarks at prominent positions in the patient image data (Figure 37). These positions should be chosen so that they can easily be found and touched in navigation mode using the navigated instrument (pointer instruments) and so they are accurate to a maximum deviation of 10 mm from the landmarks.

The approximate registration by defining three landmarks serves as a first and quick pre-registration, which subsequently must be improved using a surface scan.

i For further information on how to define, move, and delete landmarks, see Chapter “Image-to-Patient Registration: Landmark.”

Since only an approximate scanning of the three landmarks is necessary, they can be placed comfortably in the 3D model. A precise adjustment of the landmarks is not necessary.

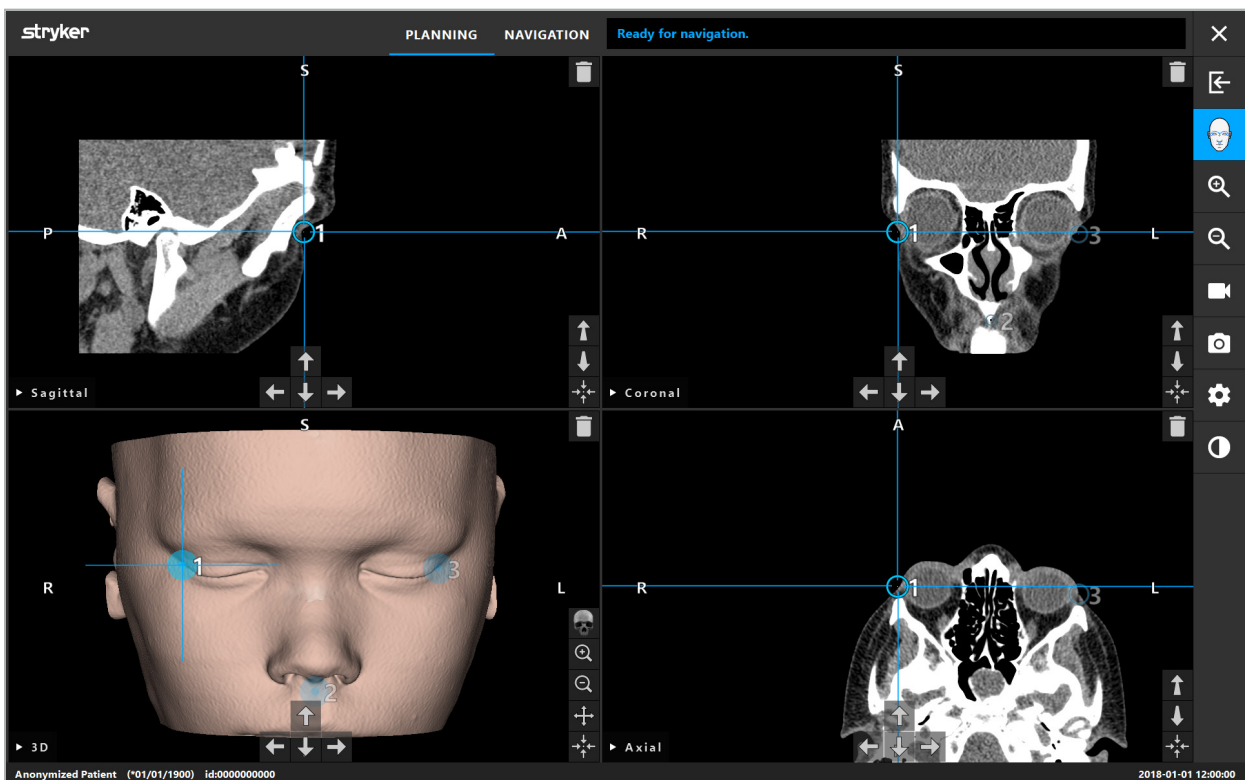
Figure 37 – Placing Surface Landmarks

6.12. Image-to-Patient Registration: Enhanced Surface

If the enhanced surface image-to-patient registration type is selected, this is displayed by a corresponding activated symbol (Figure 38).

Figure 38 – Enhanced Surface Registration Type

A precondition for the enhanced surface image-to-patient registration type is the correct recognition of the ridge of the nose and a point below the nose (subnasal) (Figure 39).

If the anatomic features in the patient image data are detected, they are shown in the 3D model. The subnasale is marked green and represents the starting point of a touching path that is shown as a sequence of red triangles. If the recognition is ideal, the touching path is shown along the middle of the nose bridge.

If the anatomic features are not detected optimally, these can be repositioned by moving the landmarks. For further information on how to define and move landmarks, see Chapter “Image-to-Patient Registration: Landmark.”

The advantage of the enhanced surface image-to-patient registration type is that no landmarks have to be placed. Therefore, the planning can be minimized to the loading of patient image data.

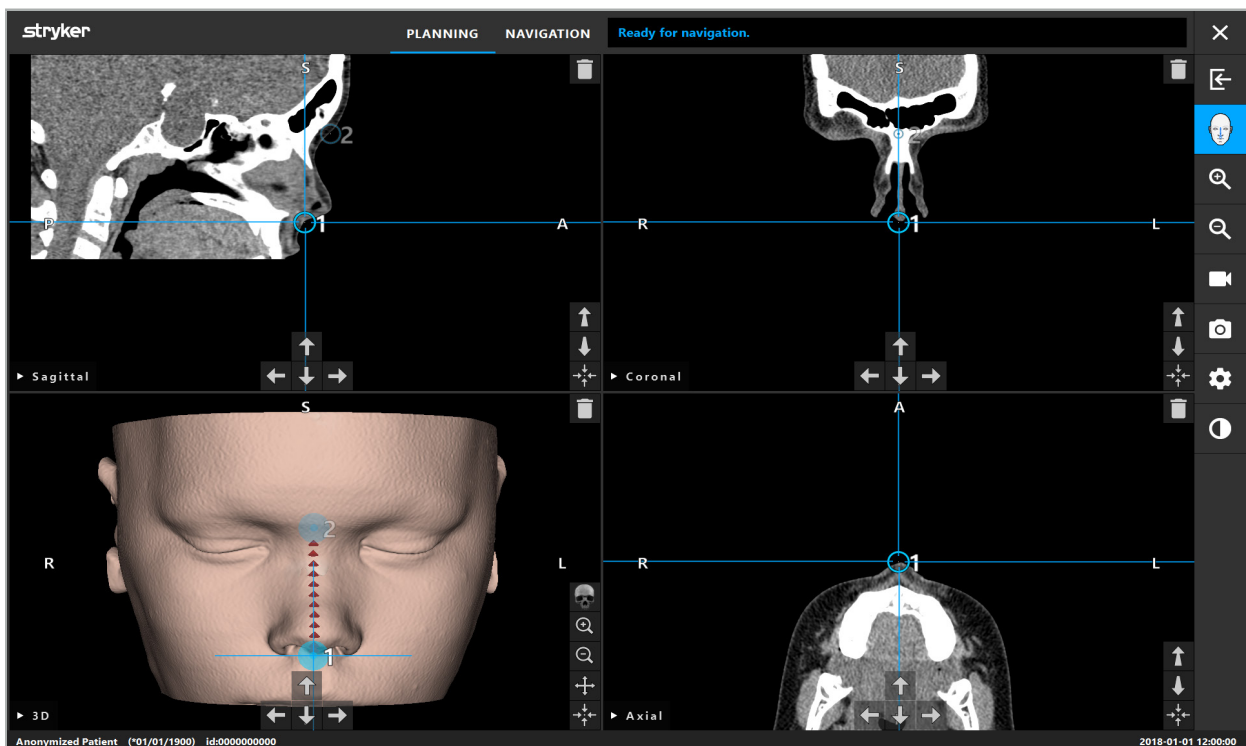
Figure 39 – Image-to-Patient Registration: Enhanced Surface

WARNING

- If the anatomic features are not registered correctly, correct their position first or choose a new registration method.
- Do not define landmarks on sensitive structures such as eyes that might be injured by touching with the tip of the pointer.

i If the path between landmark 1 and landmark 2 is too flat, the software emits a warning and does not allow the switch to navigation mode. Correct the positioning of the landmarks so that a path with more profile is created.

6.13. Planning the Navigated Surgery

6.13.1. Planning Object Dialog

The Planning Objects dialog is used to manage a number of different types of planning objects. Each object is displayed on a large button (Figure 40). Clicking the button selects the object; double-clicking selects and also closes the dialog.

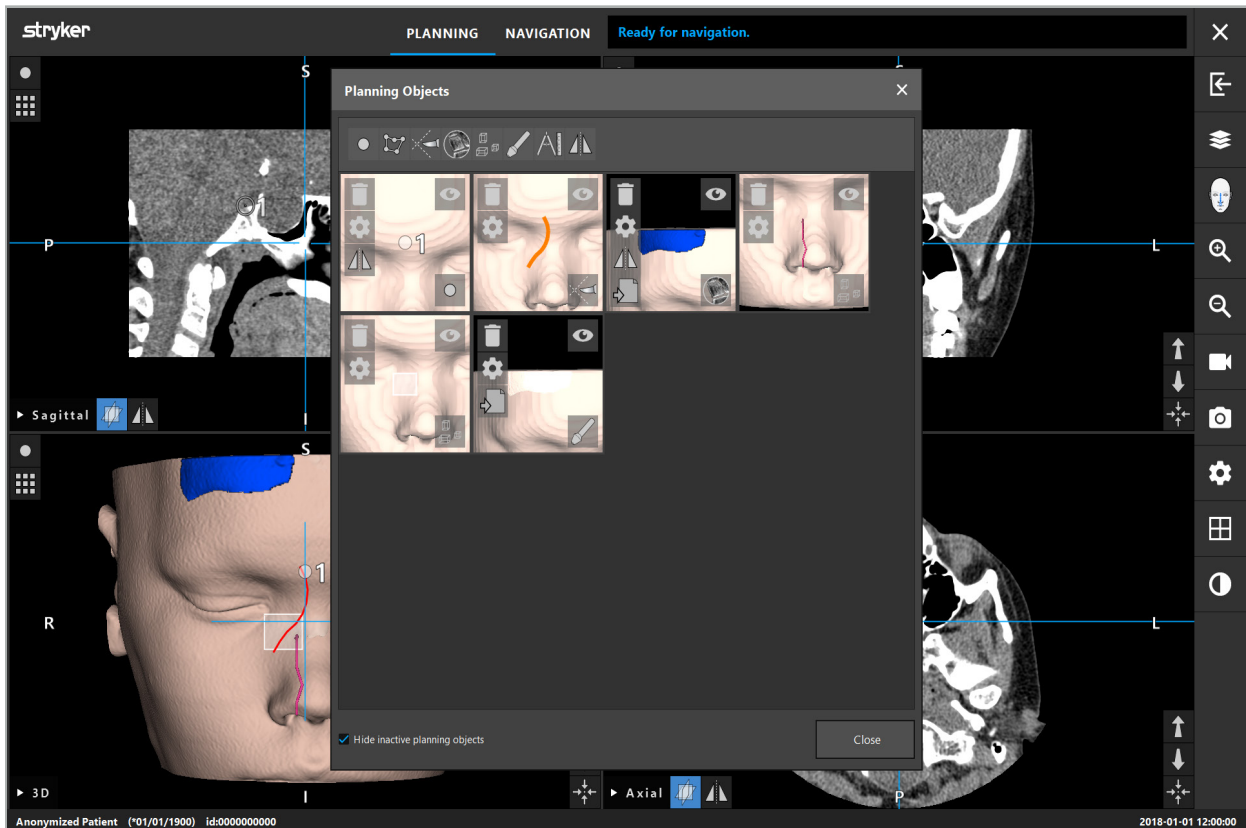
Figure 40 – Planning Objects Dialog

Inside the patient views, click the Planning Objects Dialog button (Figure 41).

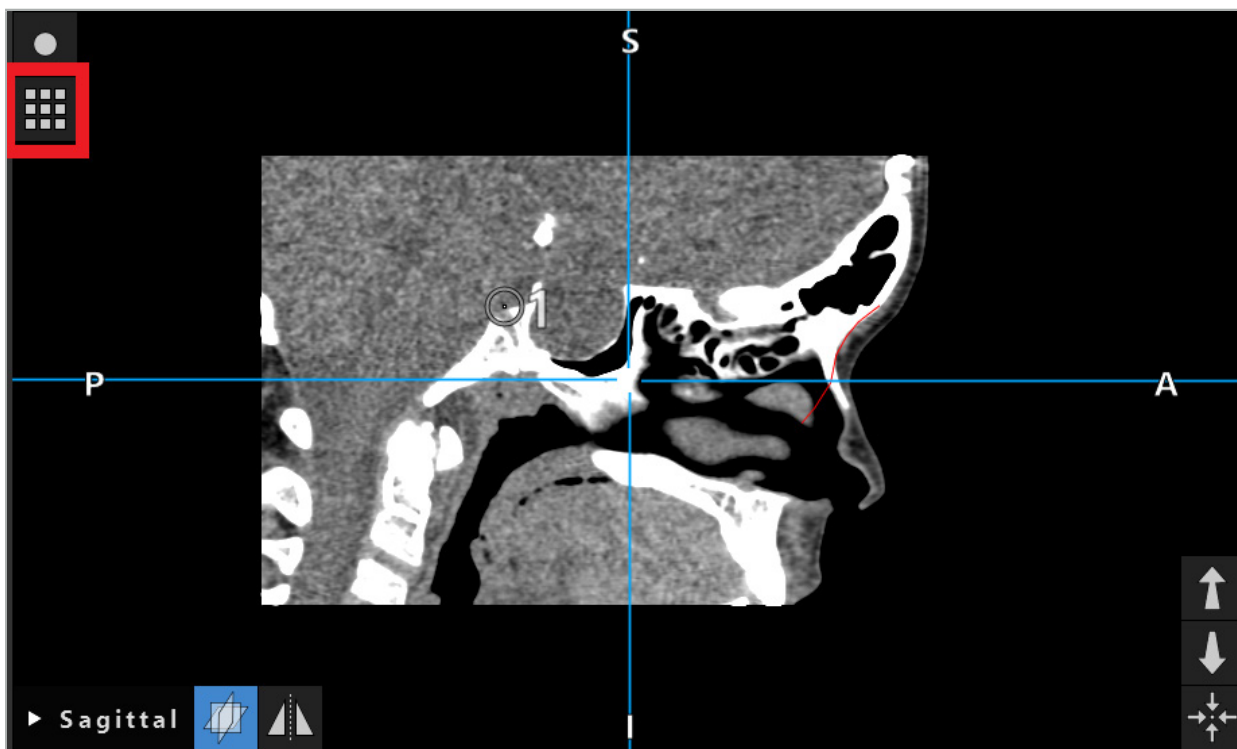
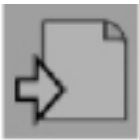
Figure 41 – Button for Opening the Planning Object Dialog

The following buttons provide additional functionality for the planning object:

	Hides the object or shows the object again. A hidden object is no longer displayed outside of the dialog.
	Deletes the object. Warning: This operation cannot be undone.
	Configures the object. Choose various colors, a description, and the activation or deactivation of an alarm during the navigation. See Chapter "Defining Alarm Zones for Planning Points and Planning Structures."
	Creates a mirror copy of this object on the other side of the symmetry plane. See Chapter "Mirroring of Planning Points and Structures." Only available for some objects.
	Export the object into an STL file. Only available for planning structures, bone segmentations, and implants.

When creating a new planning object or selecting an existing one, the software enters a mode where interaction is only possible with objects of the chosen type. The active mode is displayed in the top left corner of each cross-section view (Figure 42) and can be toggled off by clicking the button.

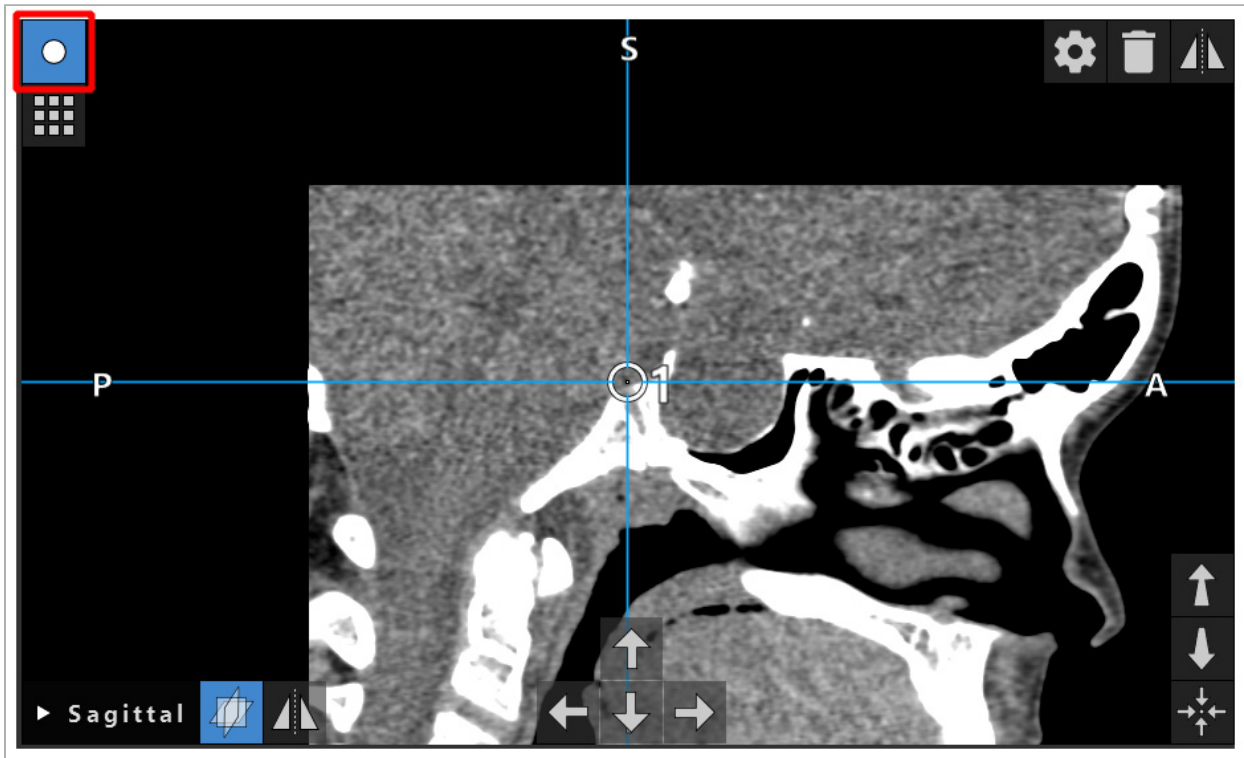
Figure 42 – Activated Planning Point Mode

CAUTION

Limit the quantity of the planning objects during planning to a relevant selection. If there are too many planning objects, they could overlap one another during the operation, thereby reducing clarity.

i In the unlikely event that the navigation software or the navigation unit should experience difficulties during planning or intra-operatively, restart the software and navigation unit. To reuse existing patient registration see Chapter “Reuse an Image-to-Patient Registration”.

To create and change planning objects, you must be in planning mode or in navigation pause mode.

When saving a planning, all created planning objects are saved as well. When loading the planning, they are restored.

The depiction of planning objects in the endoscope video image (augmented reality) is included only in the software version Scopis ENT Software with TGS.

6.13.2. Creating New Planning Objects

Planning objects are classified in different types, e.g. planning points or planning target structures. To create a new planning object, first select the type of the object by clicking one of the following buttons in the upper taskbar in the dialog (Figure 43).

Figure 43 – Buttons for the Different Types of Planning Objects

Then click the Plus button in the Planning Object dialog (Figure 44).

Figure 44 – Plus Button for Creating New Planning Objects

If a mode for a type of planning objects, e.g. planning target structures, is already active, you can also click the Plus button in one of the views in order to create a new planning object of the selected type (Figure 45).

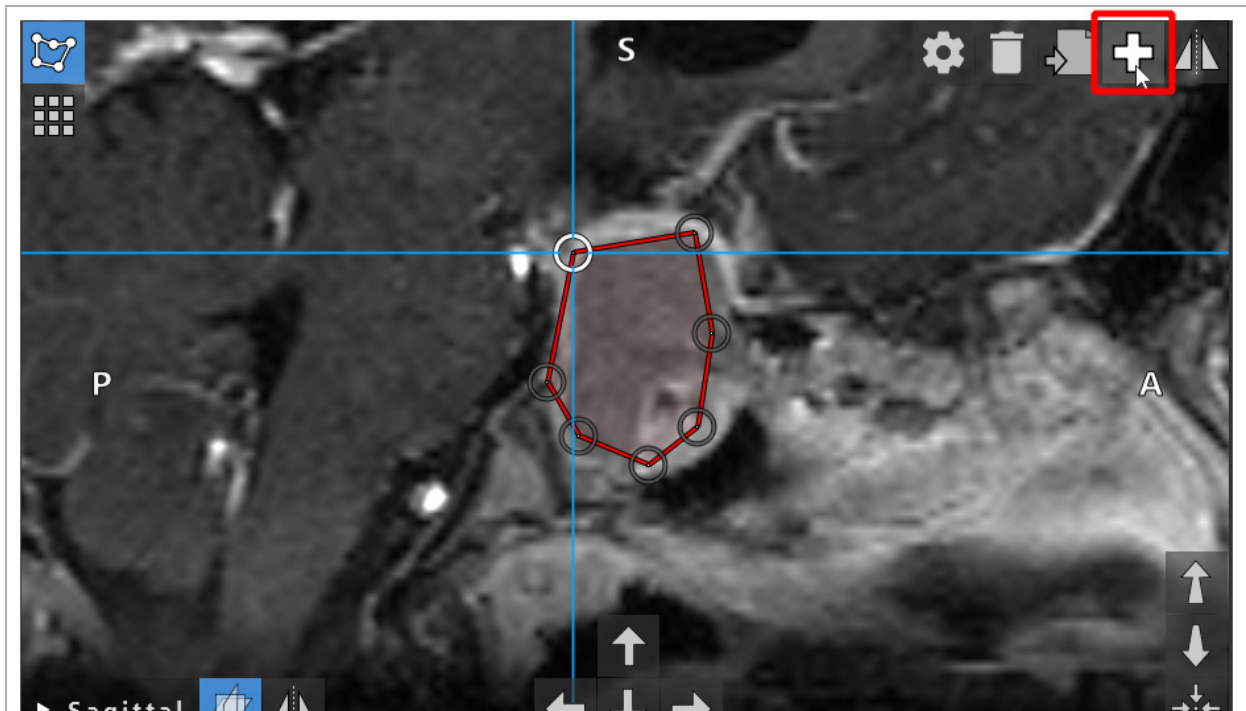
Figure 45 – Plus Button for Creating New Planning Objects

6.13.3. Removing Planning Objects

You can remove single points and entire planning structures. Click to select and highlight a planning object. While an object is selected, a button with a wastepaper basket icon is shown in the upper left part of the slice image. Delete the object by clicking this button. Alternatively, use the touchscreen to delete.

6.13.4. Creating Planning Points

To create planning points, press the Planning Points button in the Planning Object dialog. Then click the big Plus button. The dialog closes when the planning points mode is active. Then define a planning point by clicking in the cross-section image view. Planning points are shown as white rings in the cross-section image views, and as white spheres in the 3D view.

6.13.5. Creating Planning Structures

Planning structures are three-dimensional bodies which are defined through intersections (Figure 46). After creating the planning structure in a plane, you can adjust other slice im-

ages in the intersection image displays and define other intersections. These planes are connected automatically to form a 3D body.

Press the Planning Structures button in the Planning Object dialog. Then click the big Plus button. If this mode is active, you can mark target structures by surrounding them with border points. Click the positions where you want to place the boundary points. After placing at least three points, they are connected to form a closed region. This procedure needs to be repeated in at least one other slice of the same view (axial, coronal, sagittal). The regions enclosed in different slices are then combined into a 3D structure.

To copy an already planned region from one slice into another, first create a region as described above. Then switch to a different slice of the same orientation, and click the button displayed on the left. The previously planned region is now copied into the current slice and can be adjusted according to anatomical structures in the image. It is not necessary to copy existing regions; however, it can save work if only minor adjustments to a region are necessary.

i You can define planning structures in each of the three cross-section views. However, as soon as you decide on a cross-section view, the entire planning structure must be defined in the same cross-section display.

Within slices of the same orientation, any number of enclosed regions can be planned. The structure is interpolated between the slices in which regions are planned. To generate a spatial representation, it is necessary to enclose regions in at least two slices.

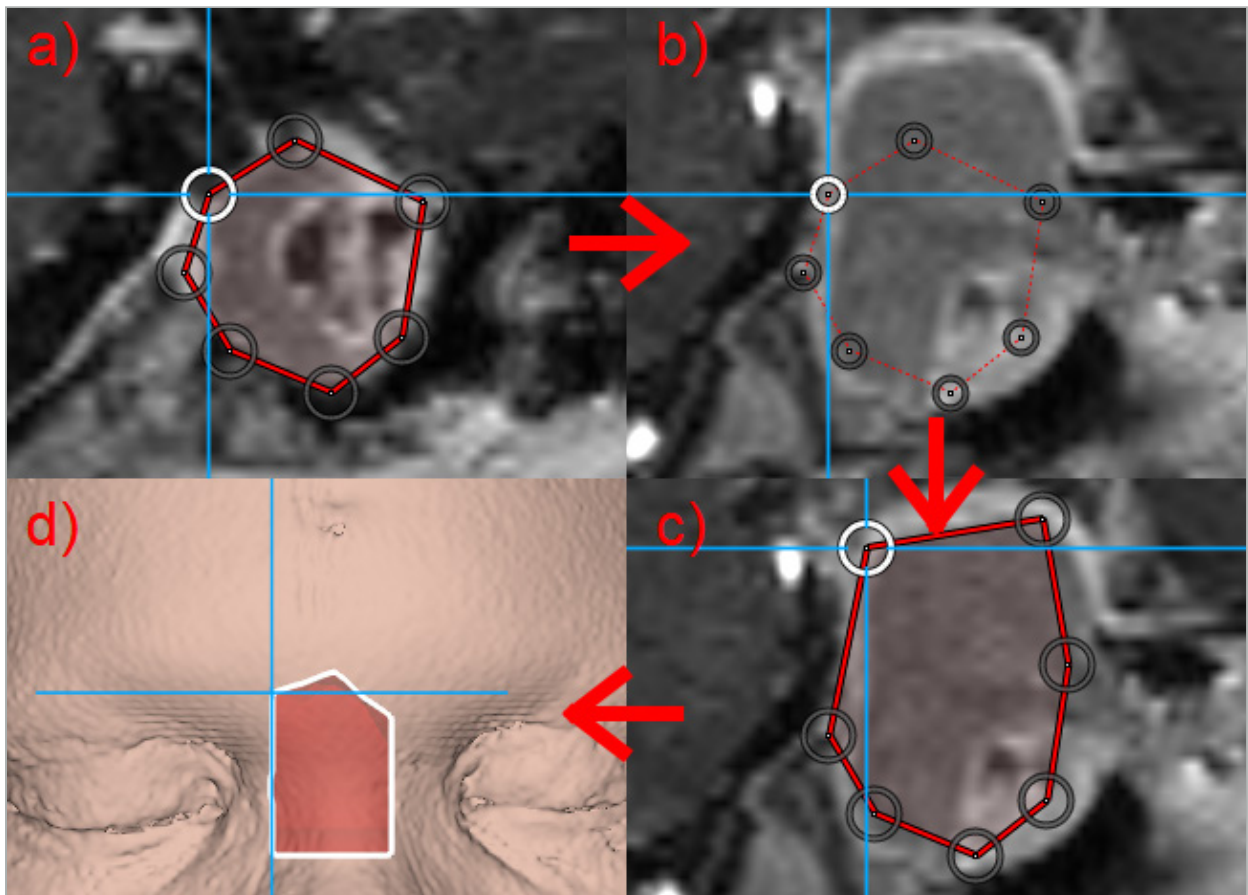
Figure 46 – Creation of a Planning Structure

a - Enclose a region in one slice

b - Change to a different slice. The boundary of the region is displayed by a dotted line

c - Copy the region into the current slice and adjust the points defining the border

d - Spatial representation of the planning structure in the 3D view

6.13.6. Creating a Cutting Line

A cutting line is a line defined by any number of points placed on the patient's bone structure and provides additional navigation information. This information can be used for the severing of the bone along the planned line. If you bring a tool close to the cutting line during surgery, the axis of this tool is extended in the cross-sectional view. The depth of the cutting line from the tip of the tool is displayed, as is the distance by which the tool would miss the cutting line if it was to cut through the patient along this axis.

To create a cutting line, press the Cutting Line button (Figure 47) in the Planning Object dialog. Then click the big Plus button. Points of the cutting line can be placed in the 3D view and in any of the cross-sectional views. New points are automatically added to the end of the cutting line. If a new point is placed close to an already existing line segment, it is inserted between existing points.

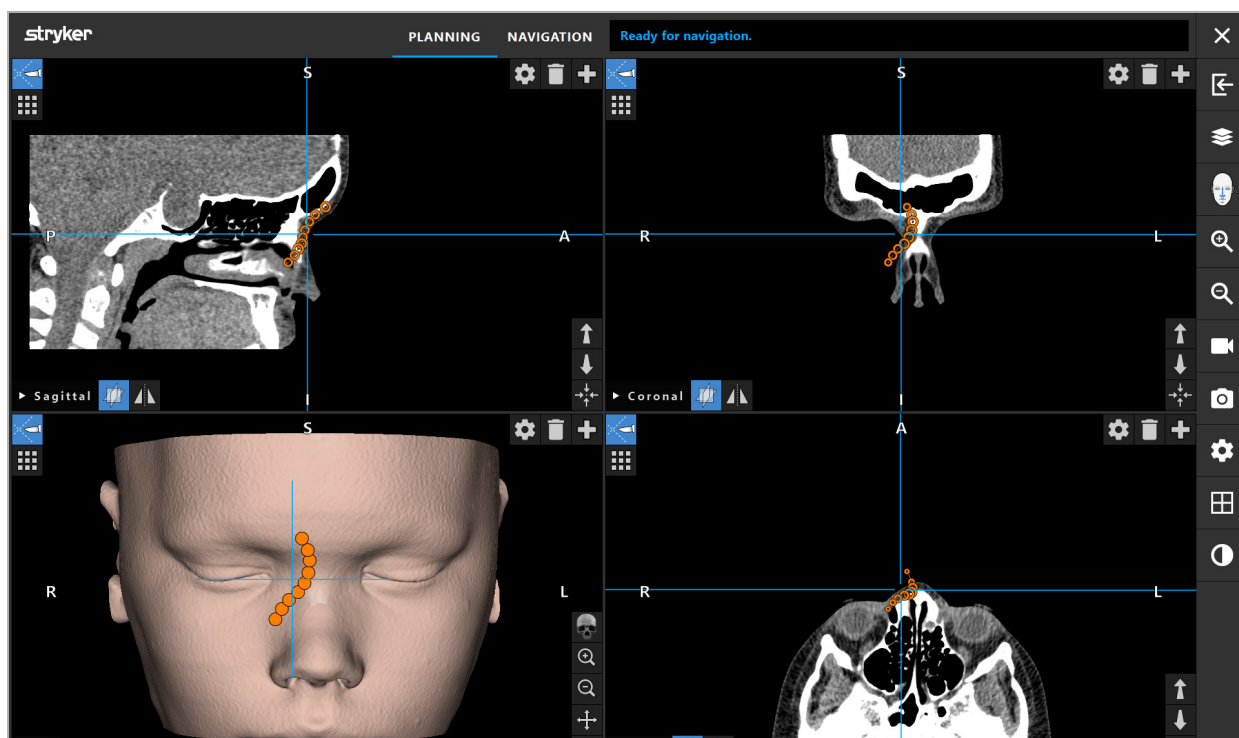
Figure 47 – Planning a Cutting Line

6.13.7. Creating a Bone Segmentation

Bone segmentations mark parts of a patient bone structure in a different color (Figure 48). Move or rotate these bone segmentations, or mirror them to the other side of the face.

To create a bone segmentation, press the Bone Segmentation button in the Planning Object dialog. Then click the big Plus button to close the dialog.

In the cross-sectional views, the bone structures are highlighted. The mouse cursor turns into a blue circle. Click and drag the circle over a highlighted region to mark the areas.

To unmark areas, click the red minus sign button. The mouse cursor turns red. Now click and drag in the views over marked areas in order to unmark them. Alternatively, hold the Shift button on the keyboard.

Change the size of the circle by clicking one of the three buttons depicting circles of different sizes. Note that the current zoom level also influences the size of the area that is being marked or unmarked.

To speed up the process, bone structures in neighboring slices are also marked. The depth depends on the zoom level and the size of the mouse cursor.

When you are done marking the bone structure, click the blue plus sign button again in order to leave the marking mode.

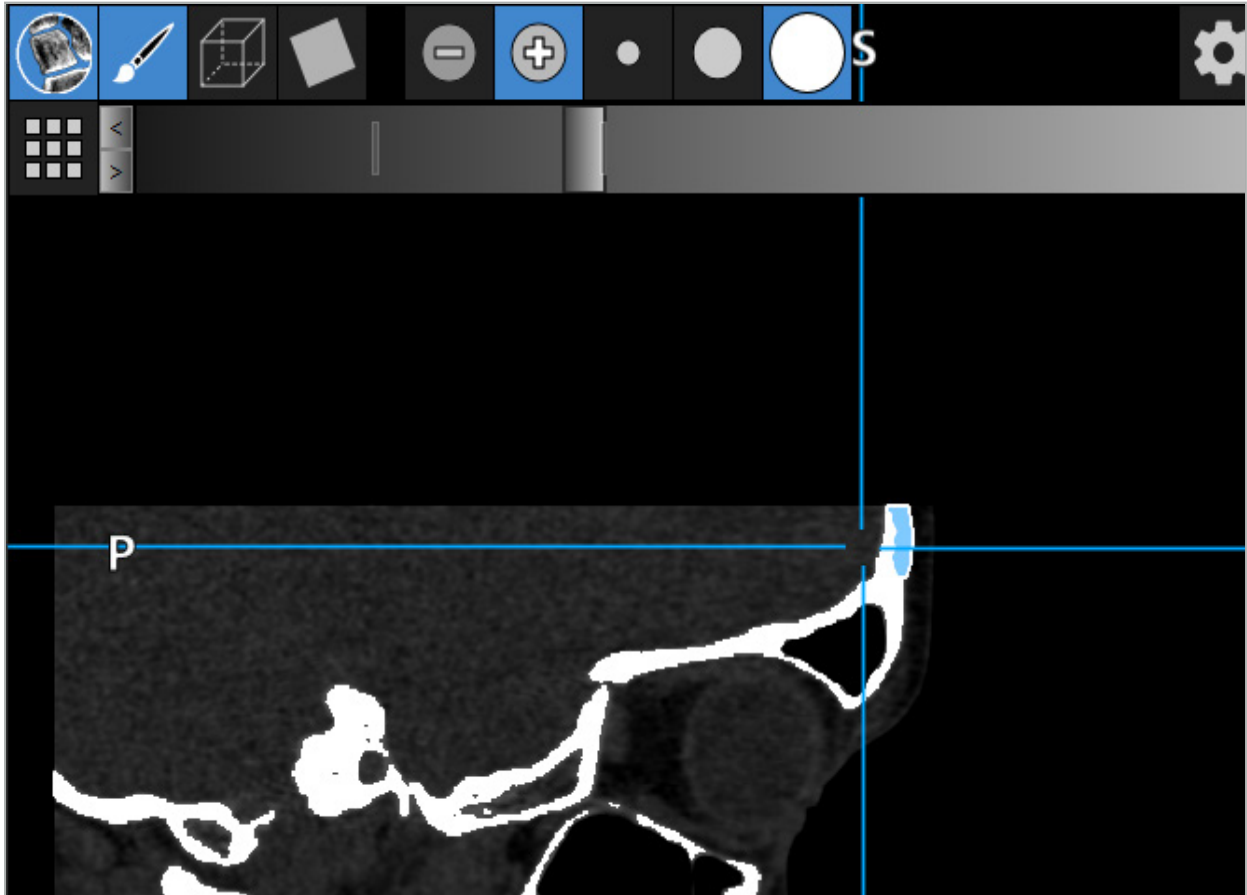
Figure 48 – Segmentation of a Bone Structure (blue)

6.13.8. Highlighting Paranasal Sinus Cells with Building Blocks

Press the Building Blocks button in the Planning Object dialog. Then click the large Plus button.

Building blocks mode provides the following functions:

Box mode allows you to create and manipulate 3D boxes within the image volume. This can be used to highlight specific paranasal sinus cells. To create a box, click and hold one point in the volume and drag the cursor to a different point. A box is spanned between the two points with a depth proportional to its size. Adjust its size and position by dragging its sides.

The shape of boxes can be modified by dragging the box edges. The shape is only modifiable in one of the orthogonal views (Axial, Sagittal or Coronal). After reshaping a box in one view, the other views are locked for reshaping this box.

To create a new box, repeat this process at a different position. Switch between different boxes by clicking them while in this mode.

The Drainage pathway mode allows you to create and manipulate tubes through the image volume. This can be used to highlight drainages in the nasal cavity. To create a new tube, just press and hold a mouse button at some point in the volume and drag the cursor along a path. You can extend and contract the tube by pressing and holding a mouse button near an endpoint and moving the mouse in the desired direction. The path can be deformed by pressing and holding a mouse button next to it and moving the mouse next to in the desired direction.

To create a new tube, repeat this process at a different position in the volume. Switch between different tubes by clicking them while in this mode.

This button temporarily hides all previously planned boxes and tubes except for those selected. Clicking the button once more makes all hidden objects visible again.

In addition to the object settings described in “Planning Object Dialog”, drainage pathways and boxes can be classified directly in correspondence to the IFAC classification.

6.13.9. Importing an Implant

Implant objects defined using STereoLithography (STL) file format can be imported into the navigation software as 3D shapes.

STL files can be generated by exporting 3D planning structures, like bone segments and target structure, as well as third party STL files.

To import implants, press the Implants button in the Planning Object dialog. Then click the Plus button. A file dialog opens; select the file to be imported. The implant is then loaded by the software and positioned in the DICOM coordinate space of the reference dataset.

To position the implant, use the Positioning Arrow buttons in each orthogonal view (Figure 49).

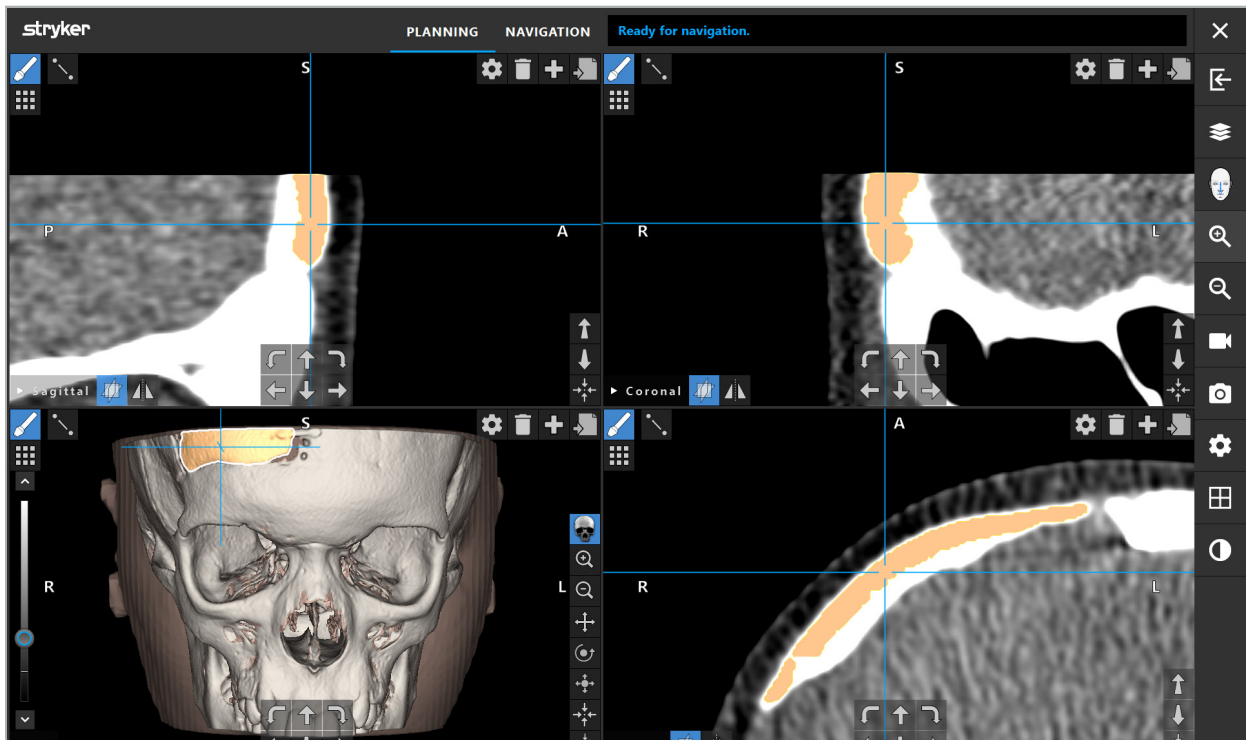
Figure 49 – Positioning of an Implant

6.13.10. Creating a Symmetry Plane

The symmetry plane for plastic reconstruction is defined by subnasale, nasion, and sella, and divides the skull into the left and the right half. The symmetry plane is necessary for the mirroring of planning objects. The symmetry plane is automatically detected during import. It might be necessary to manually adjust the automatic detected symmetry plane.

To edit the symmetry plane, press the Symmetry Plane button in the Planning Object dialog. While the symmetry plane mode is active, a plane is shown in the 3D view. Adjust the symmetry plane by dragging the three planning handles. Note: Only one symmetry plane can be defined.

6.13.11. Taking Measurements

In a specific planning mode it is possible to measure distances, angles, and areas. These measurements are planning objects as well and can be labeled, saved, and deleted.

To switch into the measurements mode, click the Measurements button in the Planning Object dialog. Then click the big Plus button to start the measurement.

In the cross-section views there are now three additional buttons with which to select the type of measurement:

To measure the distance between two points, click anywhere in the cross-section views at the position where the points can be set.

To measure an angle, first click in the views where the measurement should be taken, then an additional two times to define the sides of the angle.

To measure an area, first create two points by clicking. These two points define the main axis. A third point must be created next to this axis and defines an oval, the area of which is displayed. Create a fourth point on the other side of the axis to better fit the measurement to the object that is to be measured.

6.13.12. Moving Planning Points

While the planning object mode is active, you can move existing points that are part of this object. Click to select a point; drag to reposition a point. The selected point is highlighted. Alternatively, use the touchscreen to move planning points.

6.13.13. Precision Adjustment of Planning Points

A precise adjustment function for planning points is available via touchscreen (Figure 50).

First activate the mode for the respective planning object and then select the desired planning point. When selected, planning points are highlighted and slightly increased. Additional arrow buttons for conducting precision adjustment appear in the cross-sectional views. The buttons pointing left, right, up, and down move the point in the image plane.

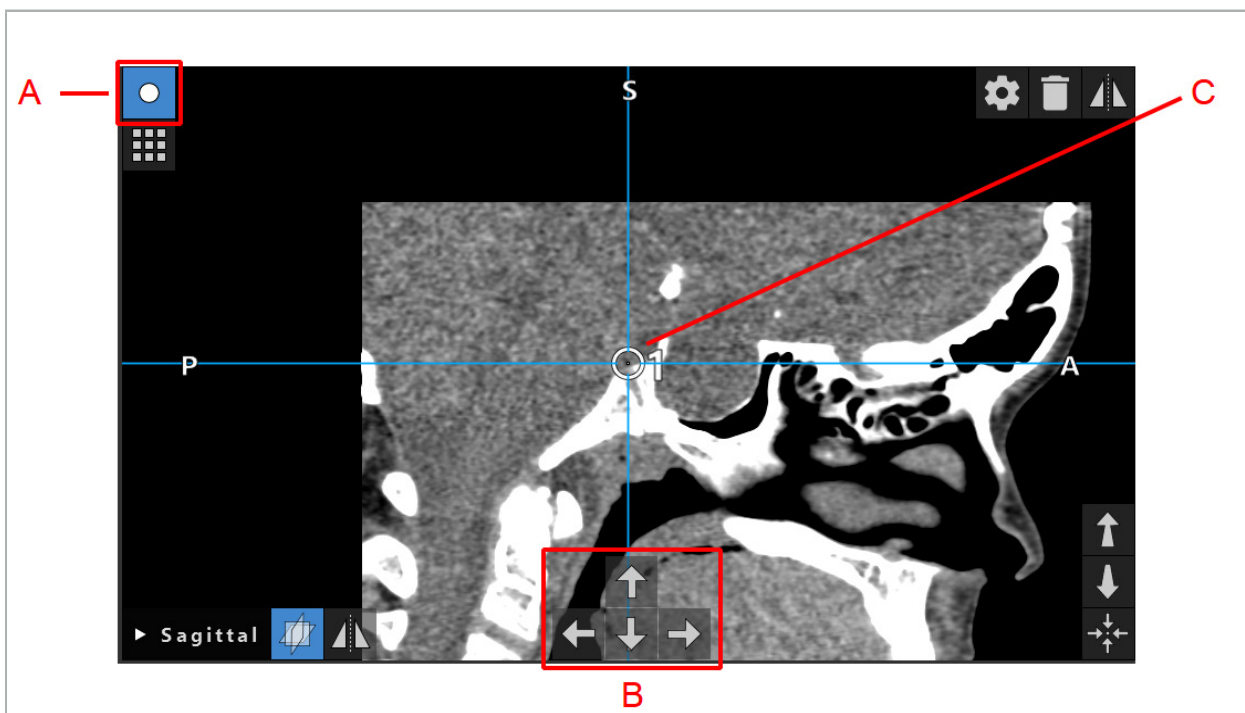
Figure 50 – Precise Adjustment of Planning Points

A - Editor mode for Planning Points

B - Arrow buttons for precision movement

C - Selected Planning Point

Planning points can only be moved precisely in planning mode or in the navigation pause mode.

An activated planning point can be precisely adjusted in all three 2D cross-section views.

6.13.14. Mirroring of Planning Points and Structures

To mirror existing planning points or planning structures, a valid symmetry plane is required. The symmetry plane is automatically determined during import, but might require additional manual adjustment. See Chapter "Creating a Symmetry Plane."

To mirror a planning point or a planning structure, select the point or structure in the Planning Object dialog. When an object is selected and a valid symmetry plane exists, a mirror symbol appears in the top left corner of each cross-sectional view (Figure 51). Activating the mirror symbol causes the point to be mirrored (copied) to the other side of the symmetry plane. The copied planning point is connected to its original. If any of the points move, the other point or the mirrored point moves as well, in correspondence with the symmetry plane. This also happens with planning structures, although here changes of the structure, e.g. moving, deleting, or adding of border points, are also mirrored.

Figure 51 – Mirror Planning Points or Structures

CAUTION

- If a planning structure is mirrored, the mirrored copy cannot be selected.
- When moving the symmetry plane, all mirrored copies of planning objects move as well. The original points and structures are not moved. If the symmetry planes becomes invalid, all copies are deleted automatically.

i An already mirrored planning point can only be mirrored once, as long as its mirrored copy exists. The same goes for planning structures.

6.13.15. Defining Alarm Zones for Planning Points and Planning Structures

Most planning objects can be configured to trigger visual and audible warning signals if the tip of an active tool comes too close to them. To configure the alarm for planning objects, press the Configuration Dialog button to open the Planning Object dialog (Figure 52 and Figure 53).

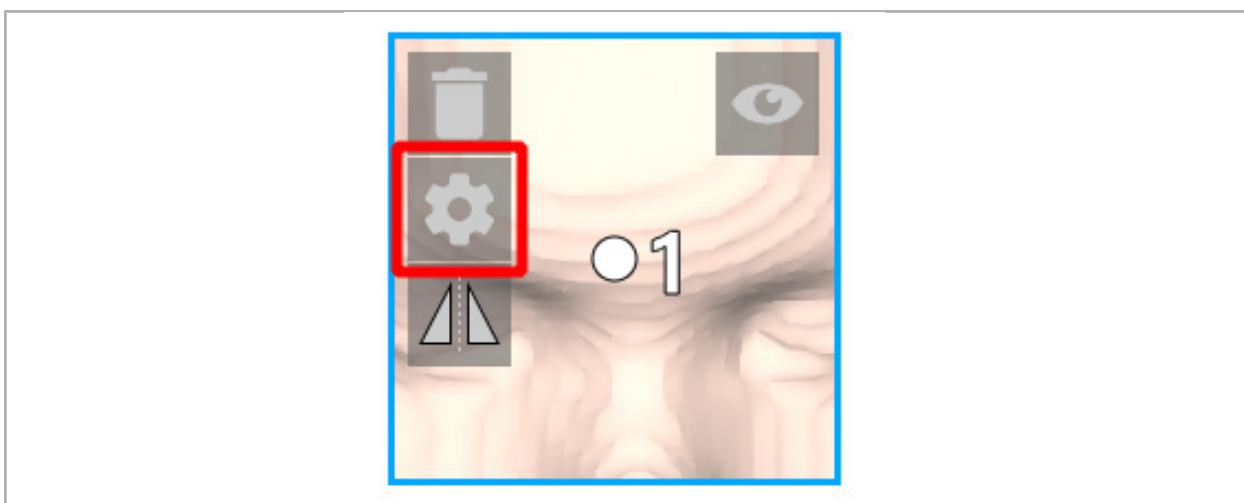
Figure 52 – Open Configuration Dialog

This button is also displayed in the upper left area of the cross-sectional views while an object is selected.

Figure 53 – Open Configuration Dialog

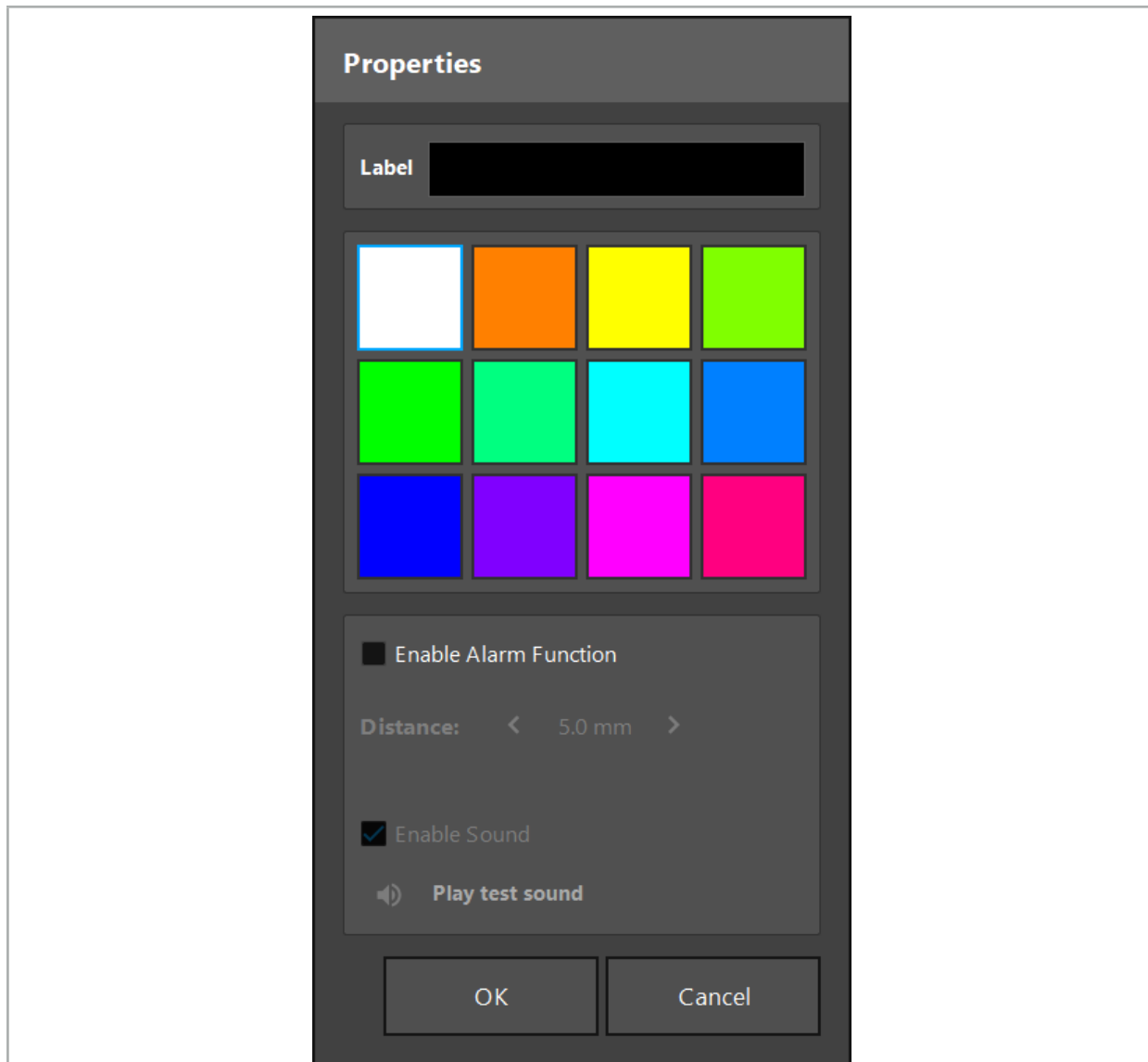
Figure 54 – Configuration Dialog

Figure 55 – Toggle Alarm Sounds

An optional label for planning points can be defined in the Configuration dialog (Figure 54).

If Activate Alarm Function is checked, the planning object is marked as an alarm area. In addition, a distance (minimum 3 mm) can be configured as a tolerance boundary. If the distance between the tool and the alarm area is less than the tolerance boundary, an alarm sounds to warn the approach. If the instrument is getting very close to an alarm structure (< 1.5 mm), the alarm sounds are played more frequently.

WARNING

- The alarm function can only monitor the distance to critical surgical targets if the navigation is available and accurate. Consequently, always verify the accuracy of the navigation system and monitor the status display of the system for valid navigation information (red axis and green tool visibility) before approaching a critical structure. The alarm function is not suitable for general observation of distances if attention cannot be paid to the navigation system.
- If planning points, planning structures, or a planning path are used to measure the distance between the active instrument and the specific anatomical structure during navigation, it is necessary to set these planning objects at anatomical structures with high precision.

CAUTION

- Make sure that your system is able to play sound. Click the Play test sound button in the Configuration dialog. If you do not hear a sound, verify that your speakers are connected and switched on, and that the sound settings are switched on and are loud enough in Windows settings.
- When using an instrument in the vicinity of an object with alarm enabled, quick movements should be avoided. Make sure that the patient registration and instrument calibration are sufficiently accurate.

Alarm sounds can be switched on or off globally or only for single planning objects. To quickly switch off an alarm tone, use the corresponding button on the right side of the program (Figure 55).

6.14. Saving the Planning

If you want to save the planning before proceeding to the navigated operation, press the Loading Patient Data button (Figure 56) or the Close Software button (Figure 57).

Figure 56 – Loading Patient Data

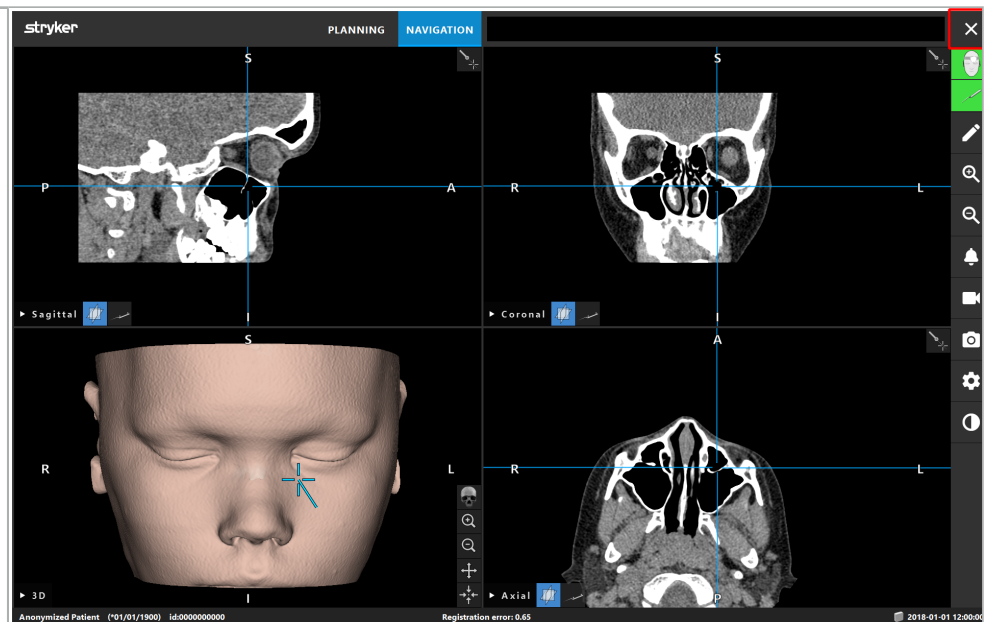
Figure 57 – Close Software

When pressing the Loading Patient Data button, the Save Planning dialog box appears (Figure 58). The dialog box gives you the following options:

- Overwriting the loaded old planning with the current planning with Overwrite.

- Saving the current planning as a new dataset with Save as new.
- Not saving the current planning and showing the Data Import dialog box (Figure 5) with Do not save.
- Closing the Save Planning dialog box without further actions with Cancel.

An optional description of the planning can be entered in the Description field.

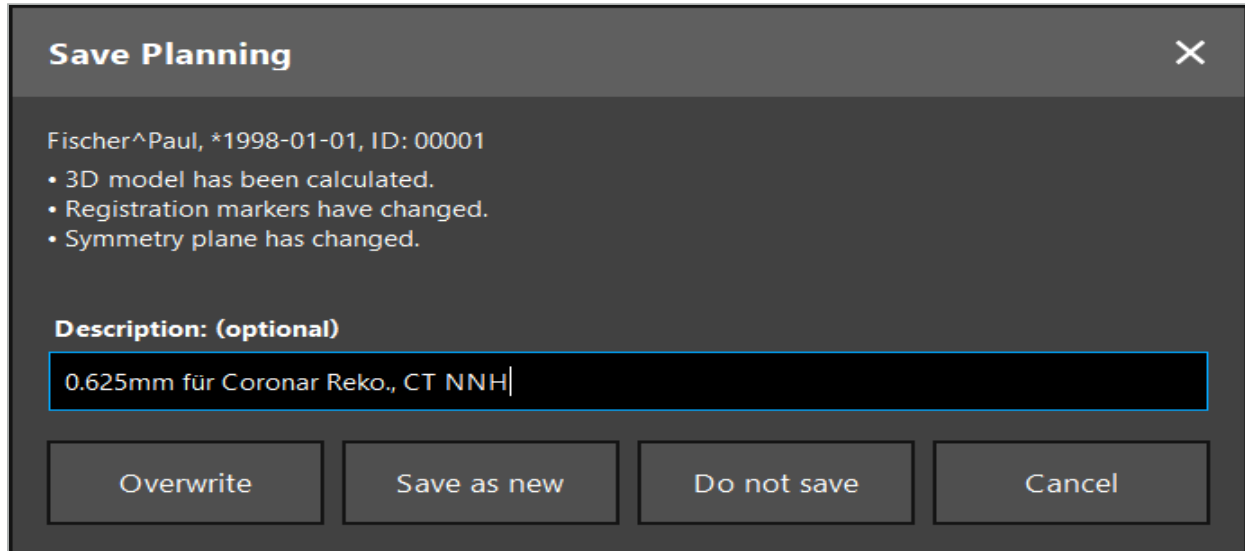
Figure 58 – Save Planning when Loading Patient Data

When pressing the Close Software button, the Save Planning dialog box appears (Figure 59). The dialog box gives you the following options:

- Overwriting the loaded old planning with the current planning with Overwrite.
- Saving the current planning as a new dataset with Save as new.
- Closing the Save Planning dialog box without further actions with Cancel.
- Closing the software without saving the planning with Quit without save.

An optional description of the planning can be entered in the Description field.

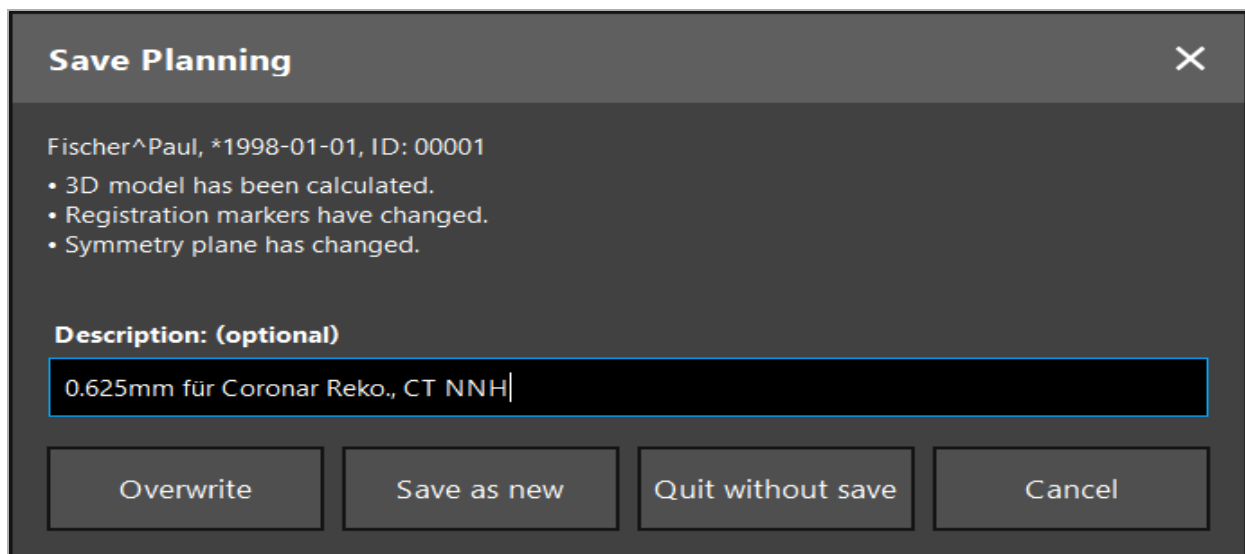
Figure 59 – Save Planning when Closing the Software

i The overwrite function completely overwrites a previously saved planning with the current planning.

You can either start the navigated operation directly after the planning, or save the planning and reload it later.

Plannings are saved automatically if you change to navigation mode or conduct a registration.

6.15. Exporting the Planning

Data transfer can be conducted via a USB drive or via a network drive, depending on your hospital's technical situation.

1. After finishing the planning, press the Export button.
2. Choose the target drive desired for exporting (Figure 60).
3. Enter a fitting description for the planning.
4. Press the Export button.

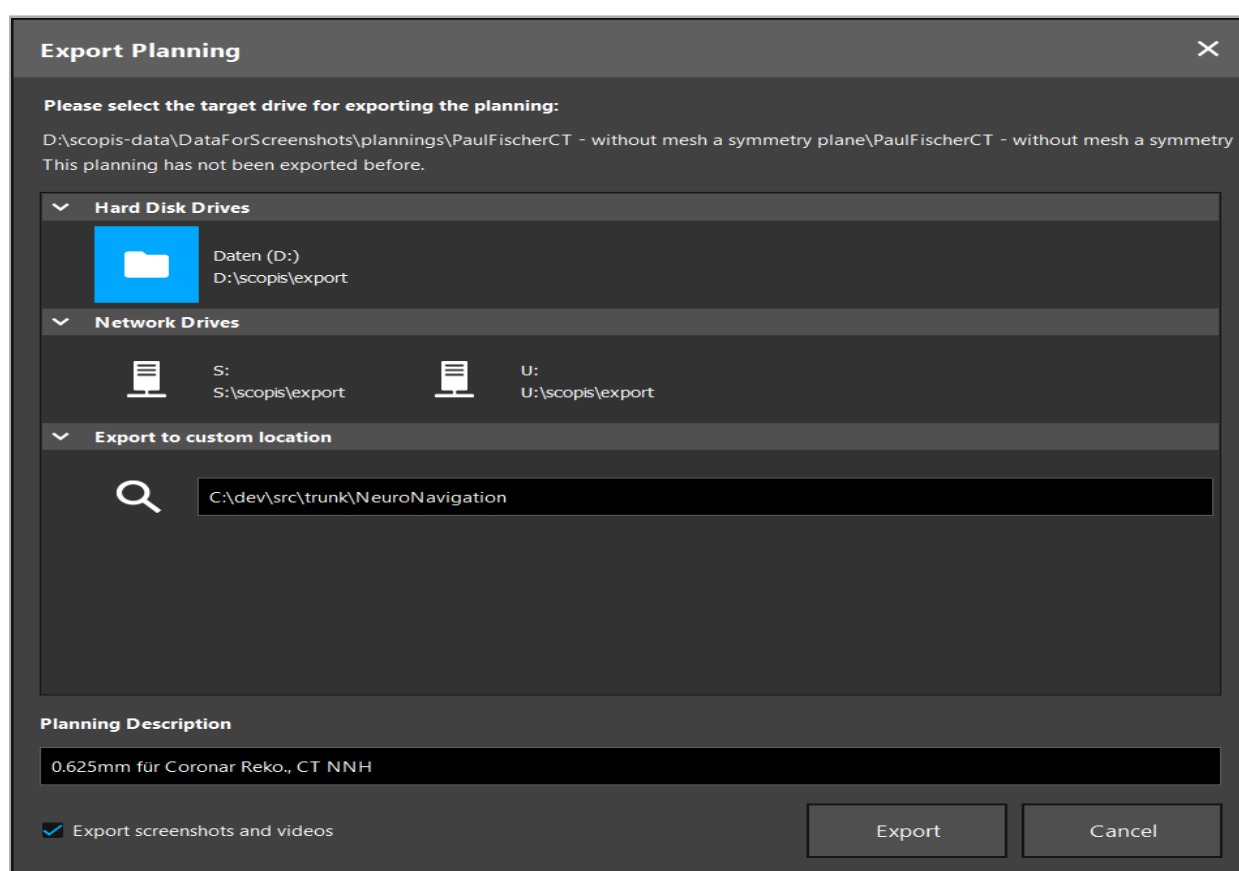
Figure 60 – Export Planning

The data is transmitted to the selected drive. To import the planned data, follow the instructions for importing patient image data "Loading Patient Image Data."

6.16. Taking Screenshots and Screen Recordings

The below buttons are featured on the right side of the user interface. Press the corresponding button to either take a single screenshot (Figure 61) or to record the screen (Figure 62).

Figure 61 – Screenshot

Figure 62 – Screen Recording

Plannings with screenshots or videos attached can be displayed as a list in the view (Figure 63). Just like plannings, these individual files are opened by double-clicking or pressing the Load button, as well as deleted or exported by pressing Delete or Export.

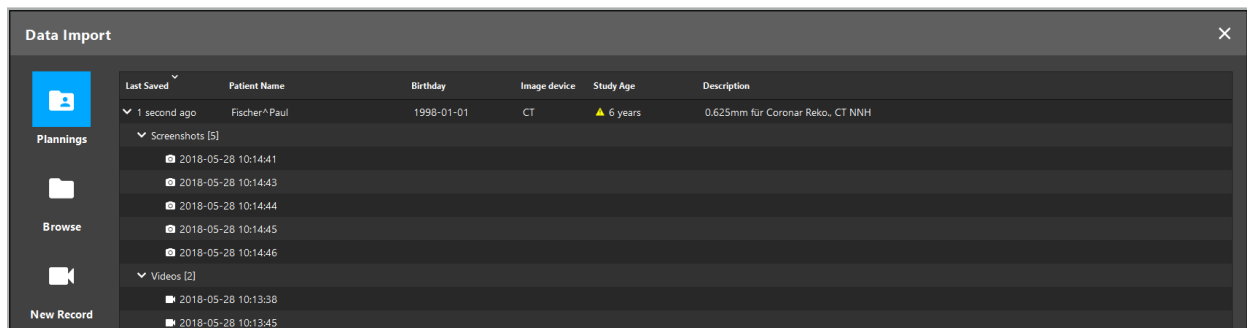
Figure 63 – Video and Screenshot Files Attached to a Planning

i The navigation software automatically creates screenshots during navigation whenever the tool is held still near the patient for a short time. These images are shown as "AutoScreenshots" as part of the planning.

If you create a screenshot or screen recording without loading a planning first, the files are saved in the Documentation folder inside the software installation directory. If you load a different planning while a screen recording is in progress, then the video recorded up to this point is saved with the first planning, while the further recording is attached to the new planning.

Depending on the size of the datasets, it can take several minutes to import and load image and planning data.

The navigation software only allows the switch to navigation mode after the planning is completed. This is confirmed with the message, "Ready for Navigation Mode." Also, the NAVIGATION tab is activated.

7. Pre-operative Setup of a Navigated Surgery

7.1. General Functionality Testing

Before each use, check products for:

- Deformations
- Correct installation and function
- Surface damage (cracks, dents)
- Other damage

Damaged products must not be used.

WARNING

Before using a medical product, assure yourself of its functional safety and proper condition via a visual inspection. Do not use the product if any defects are detected.

7.2. Field Generator

The Field Generator (REF 8000-010-004) is intended for the electromagnetic position measurement of patient and navigated instruments during surgical procedures in which compatible clinical navigation systems are used.

Figure 64 – Field Generator

Application:

Connect the plug of the generator's cable to the socket on the front of the Electromagnetic Navigation Unit (Figure 65).

Figure 65 – Connecting the Field Generator Cable to the Electromagnetic Navigation Unit

**WARNING**

While placing the cable, make sure there is no risk of stumbling for operator or a patient.

FIGURE

Never use force to put the plug into the plug socket. Do not flex the cable.

Field Generator Positioning:

Adjust the distance between the Field Generator and the operating area of the patient as follows:

Field Generator: 0.15 m (measuring range: 0.05 m to 0.5 m)

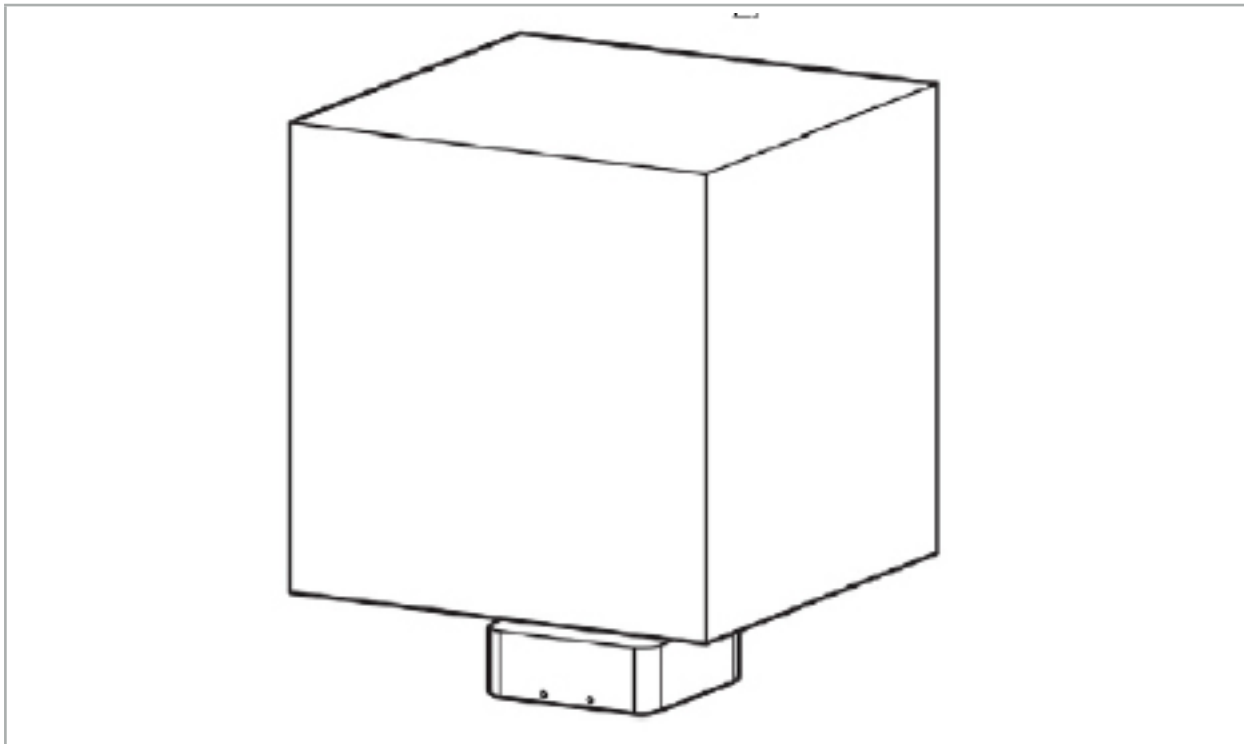
Figure 66 – Measurement Volume: Field Generator

For the positioning of the generator, a mounting arm is available. The mounting arm is attached to the standard rail and allows the flexible positioning of the generator.

WARNING

The generator is wipe-disinfectable. If the generator is placed close to the sterile area, it has to be covered in a sterile way. The product is not intended for direct patient contact.

Be aware that the use of padding or undersheets increases the distance between the generator and working area. The navigated region has to be inside the measuring volume.

The accuracy of navigated instruments may decrease when a distance of 35 cm or greater between the electromagnetic tracking tool and the field generator is reached.

7.2.1. Interferences with Electromagnetic Systems

Electromagnetic measurement systems are susceptible to metals, especially ferromagnetic substances, and electromagnetic fields. Electromagnetic measurement systems can provide precise navigation information only when the following factors are observed:

No disturbing substances within 80 cm, neither close to the Field Generator nor between the Field Generator and measuring sensor (navigated instruments). The following materials produce interferences:

- Steel
- Ferromagnetic stainless steel
- Iron
- Aluminum
- Copper

The following materials have only a minimal impact on the measurement precision:

- Stainless steel (Material number: 1.4301, 1.4305, 1.4401, 1.4404)
- Titanium Ti6Al4V
- Cobalt chromium steel

The Field Generator tabletop is an exception. It is shielded against environmental interferences on the opposite side of the measurement volume. Therefore it can be placed directly on the OR table. For all other sides, the specified rules are still valid.

Do not move disturbing items within 80 cm of the Field Generator during or after patient registration. In particular, ensure that the following items are positioned away from the Field Generator:

- Medical devices that require creation of electromagnetic fields to carry out their function
- Electrically driven motors, from drills and shavers
- Field Generator cable
- Navigation unit
- Instrument cart
- Monitors
- Keys, watches, jewelry
- Personal electronics, such as phones

Refer to Figure 67.

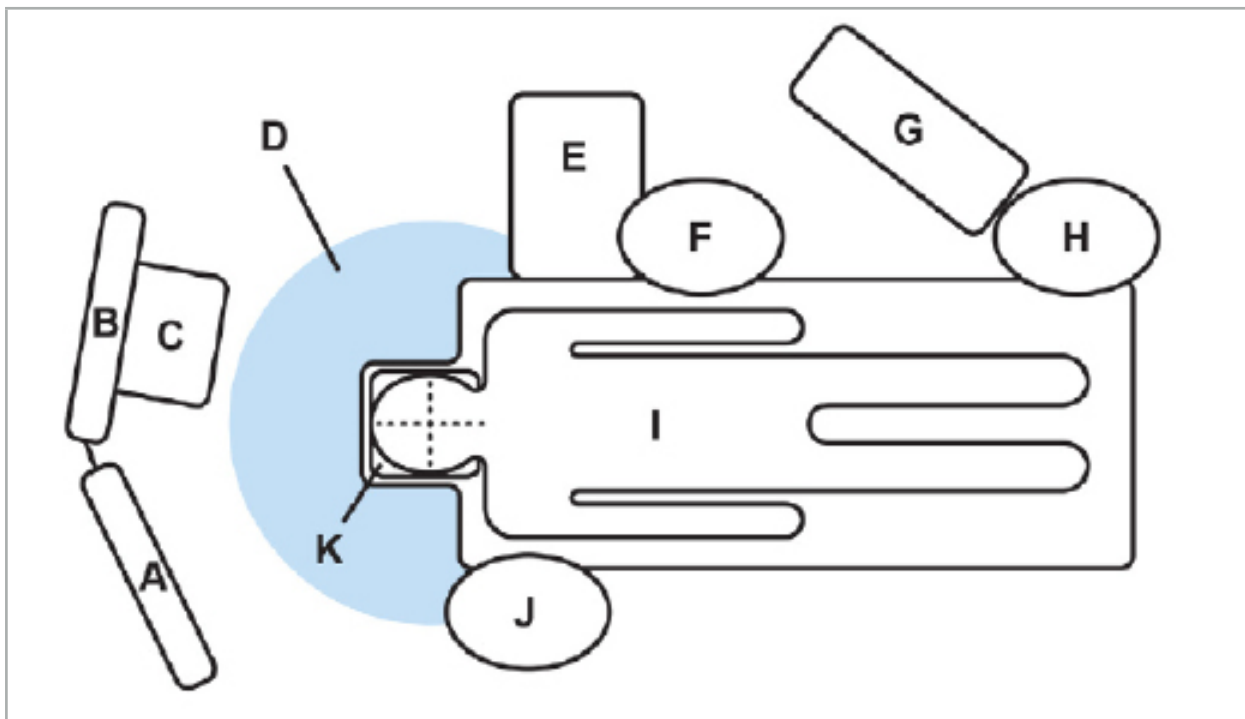
Figure 67 – Optimal Electromagnetic Setup to Avoid Interferences

- A - Monitor 1
- B - Monitor 2
- C - Endoscopy tower

D - Optimal distance of disturbing materials to the Field Generator (80 cm)

E - Table with surgical instruments

F - Surgery nurse

G - Anesthesia apparatus

H - Anesthetist

I - Patient

J - Surgeon

K - Field Generator

Keep these or similar items at a distance of at least 80 cm from each side of the Field Generator to ensure the accuracy of the tool measurement.

WARNING

- Make sure that there are no disturbing metals within the measuring volume of the Field Generator during the application of the navigation system with electromagnetic position measurement; otherwise, the accuracy of the position measurement is significantly reduced.
- Do not operate the Field Generator within 200 mm of an installed pacemaker. The magnetic field produced by the Field Generator may interfere with the operation of the pacemaker. This interference may result in personal injury.
- Do not operate the Field Generator within 10 m of another Field Generator. To do so may contribute to inaccurate position measurement and possible personal injury.
- Do not drop the Field Generator or subject it to impact. Physical damage to the Field Generator may alter its calibration and contribute to inaccurate position measurement and possible personal injury.
- Do not place the Field Generator Cable inside the measurement volume or wrap it around the Field Generator, as it may create a magnetic interference. This interference can contribute to inaccurate position measurements and possible personal injury.
- Do not place electromagnetic tool cables within 30 mm of the Field Generator Cable. If placed this close – particularly if the cables are parallel to each other – the tool cable may become subject to electromagnetic interference. This interference can contribute to inaccurate position measurements and possible personal injury.
- Do not coil the Field Generator Cable, as it produces enough electric current that a magnetic field will be created when the cable is placed in a circular formation. This magnetic field may disturb the Field Generator's magnetic field, contributing to inaccurate position measurements and possible personal injury.

7.3. Preparing Navigated Instruments

Navigated instruments must be prepared before use.

Patient Trackers provide a reference frame for the patient during surgery. For this purpose, the tracker is fixed onto the patient as near as possible to the operation area.

At least one Patient Tracker and one navigated instrument are necessary for navigation.

Compatible instruments can be found in the following sections.

WARNING

- Navigated instruments are delivered in a non-sterile condition. Before the first use as well as before each following use, the instrument must be processed (cleaned, disinfected, and/or sterilized) according to a validated procedure.
- The product is a high precision instrument. Avoid subjecting it to serious strains, such as heavy impacts. The product cannot be used if there are visible defects. After a heavy impact, the product must be checked for defects and calibrated anew.

i Ensure that an unused replacement copy is available for every electromagnetic instrument.

7.3.1. Patient Tracker Electromagnetic

The Patient Tracker Electromagnetic (Figure 68) provides a reference for patients. The product is fixed with appropriate accessories onto the intact skin of the patient, thus allowing the localization of the patient without immobilization.

Figure 68 – Patient Tracker Electromagnetic (REF 8000-040-001) and Patient Tracker Electromagnetic - 10 Uses (REF 8000-040-002)

Mandatory Accessories	REF
Patient Tracker Tabs	8000-100-001

Application

Refer to Figures 69 and 70.

1. Clean the intended contact surface on the patient. The contact surface should be free of fat, dry, and (if possible) free of hairs.
2. Remove one protective film from the adhesive tab, and stick the adhesive tab onto the Patient Tracker underside.
3. Remove the second protective film, and stick the Patient Tracker onto the planned contact surface with the adhesive tab.

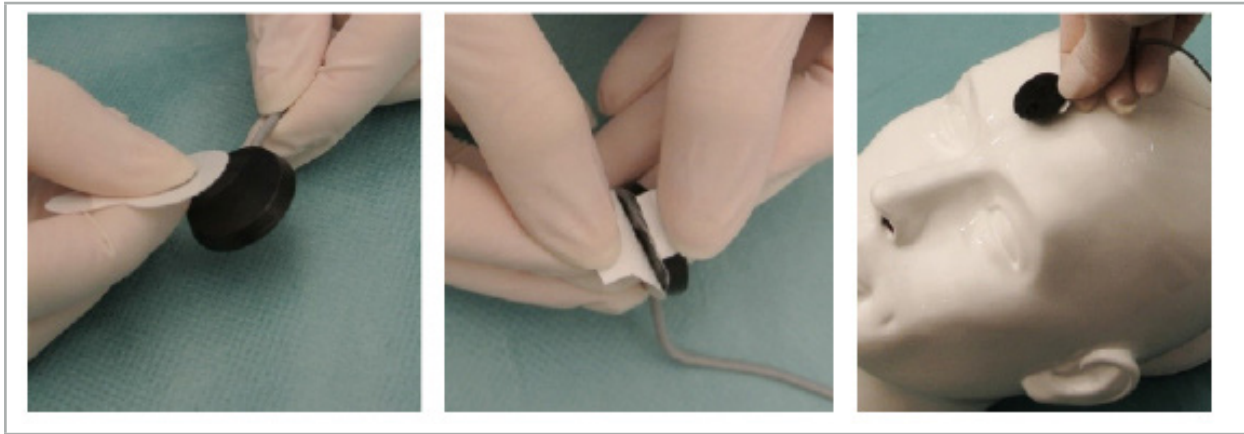
Figure 69 – Fixing Tab to Tracker and Contact Surface

4. If you calibrate instruments using the calibration area of the unsterile Patient Tracker, then it is necessary to cover this with a sterile, transparent drape. Adhesive incision drapes are well suited for this purpose, as they also hold the Patient Tracker in place (Figure 70).

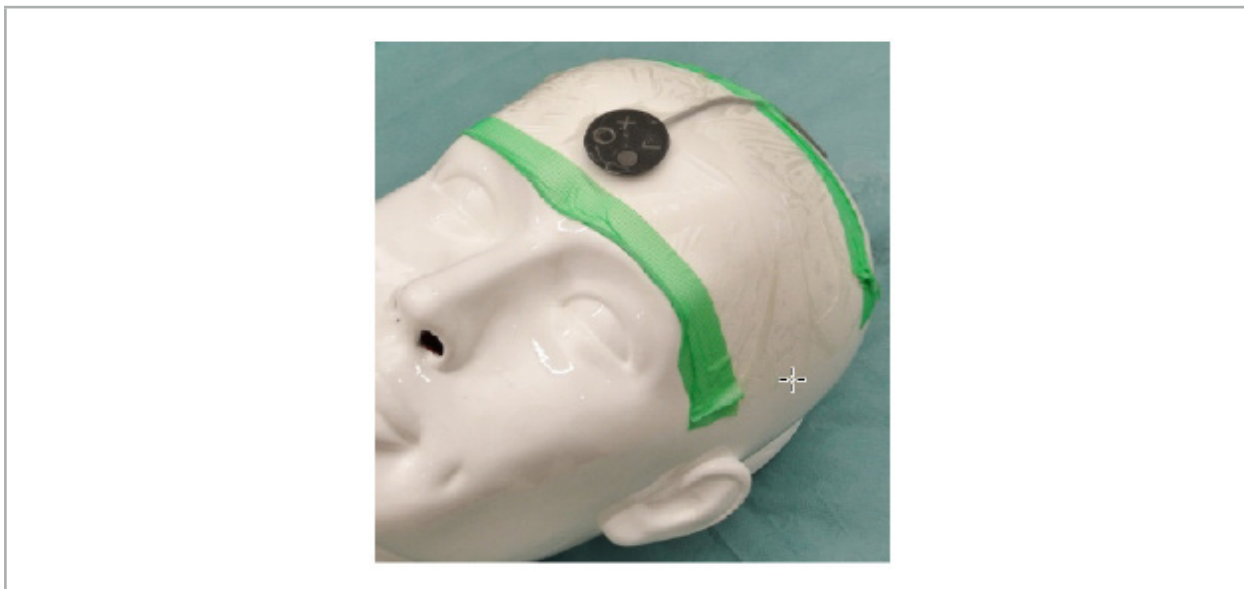
Figure 70 – Fixing the Patient Tracker onto the Patient

WARNING

Make sure not to penetrate the sterile barrier when calibrating through the thin drape.

5. If the calibration area is not needed, the Patient Tracker can be fixed in place with medical adhesive tape and then covered with a surgical drape (Figure 71).

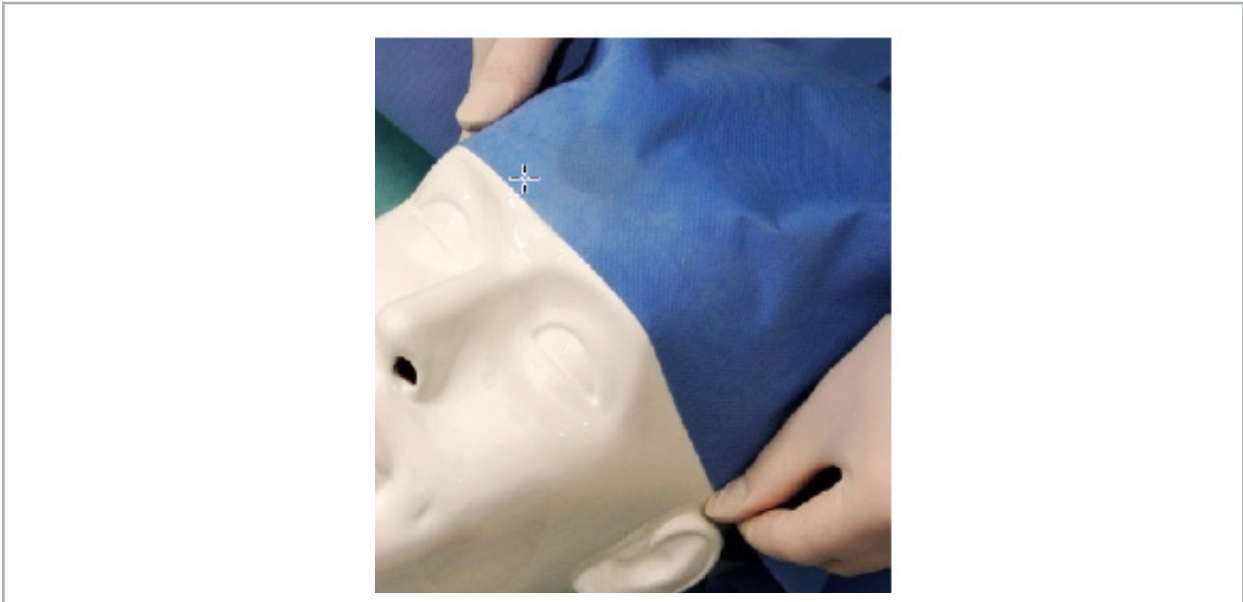
Figure 71 – Cover Patient Tracker

WARNING

Make sure that the Patient Tracker is securely attached to the patient during use, as the navigation would otherwise be inaccurate or useless. If the Patient Tracker moves, the image-to-patient registration must be repeated.

7.3.2. Pointer Instruments

The electromagnetic pointer instruments (Figure 72) are used to touch relevant structures during ENT surgery. Compatible navigations systems are thereby able to locate and navigate the touched structures.

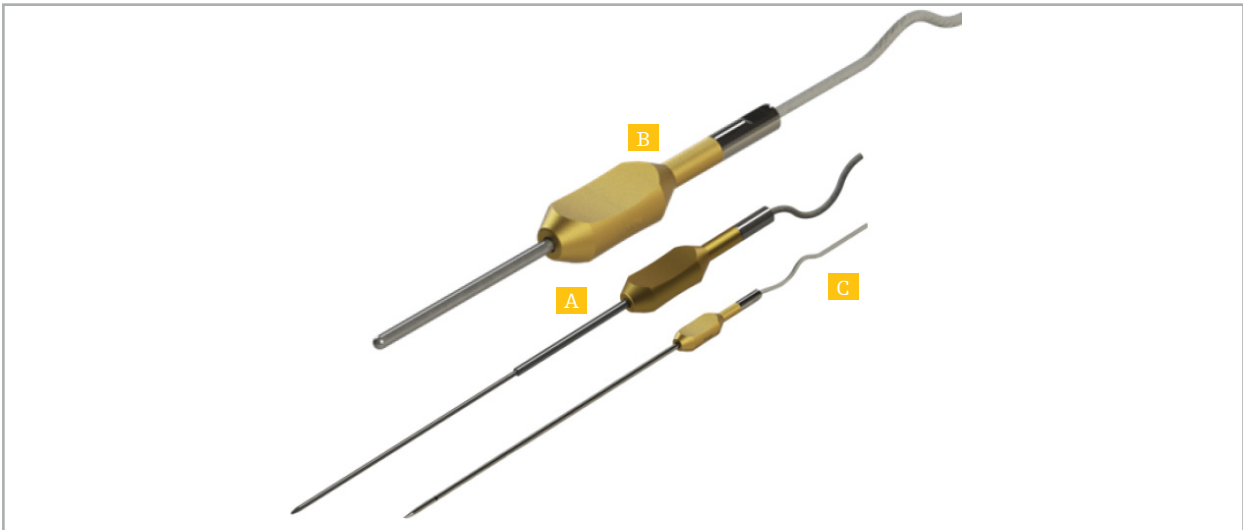
Figure 72 – Pointer Instruments

- A - Precision Pointer Electromagnetic (REF 8000-050-001)
- B - Registration Pointer Electromagnetic (REF 8000-050-003)
- C - Pointer Electromagnetic (REF 8000-050-002)

Description

Precision Pointer Electromagnetic (REF 8000-050-001)

The Precision Pointer Electromagnetic is a pointing device with a tracked tip. The steel tube may be reshaped once per procedure provided that the curvature is not too sharp (i.e. bending must be as spread out as possible) and that the distal 20 mm up to the black ring marking of the instrument remain unbent.

Pointer Electromagnetic (REF 8000-050-002)

The Pointer Electromagnetic is a pointing device with a tracked tip with a long design for use during endoscopy. The steel tube may be reshaped once per procedure provided that the curvature is not too sharp (i.e. bending must be as spread out as possible) and that the distal 20 mm up to the black ring marking of the instrument remain unbent.

	Precision Pointer Electromagnetic	Pointer Electromagnetic
Working length	120mm	175mm
Angle	0°	0°
Diameter	1.5 mm (from tip - 80mm) 2,5mm (80-120mm)	2.7 mm
Flexibility	<ul style="list-style-type: none">• Min. bending radius: 1.5 cm• Most narrow section to be bent: 3 cm• Length of flexible section: 6 cm; only to be bent evenly with a total curvature of 90°• Not flexible: First 2 cm of the tip; second tube, in which the tube is adhered	<ul style="list-style-type: none">• Min. bending radius: 3 cm• Most narrow section to be bent: 5 cm with a total curvature of 45°• Flexible section length: 10 cm; only to be bent evenly with a total curvature of 90°• Not flexible: tip and end of the tube (each 2 cm)

Registration Pointer Electromagnetic (REF 8000-050-003)

The Registration Pointer Electromagnetic is designed for non-sterile image-to-patient registration on the patient's skin. Between surgical procedures it is cleaned and disinfected only. For improved registration quality, the pointer tip is broad and rounded, avoiding tissue deformation.

Application

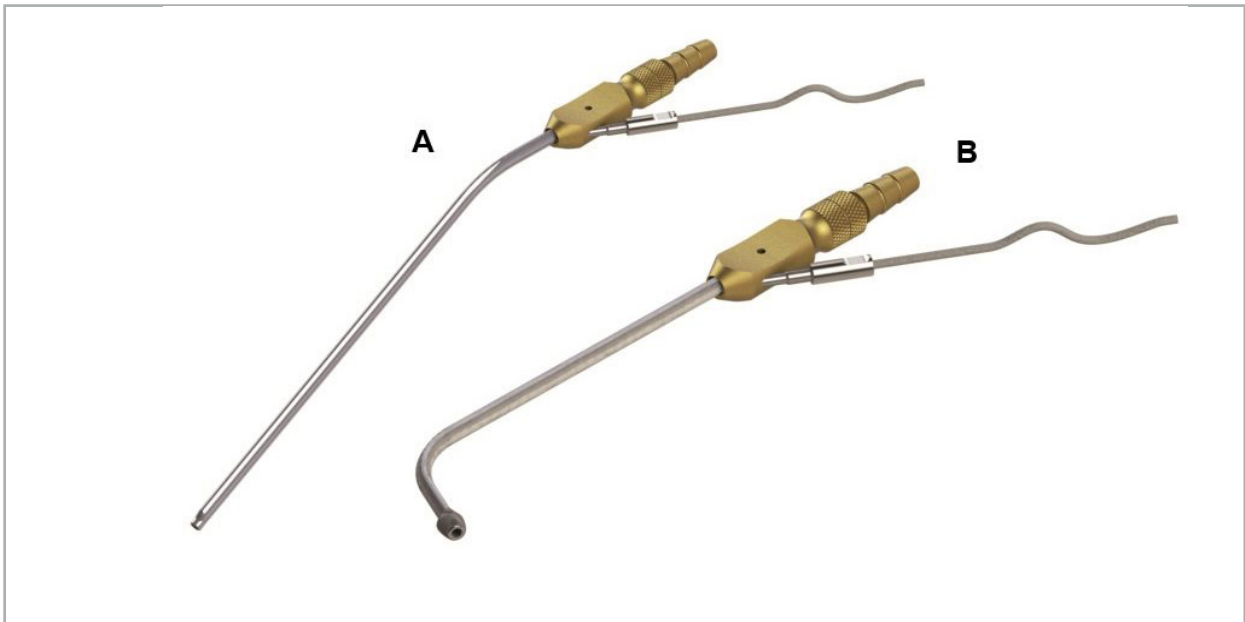
WARNING

Do not bend the distal 20 mm of the Precision Pointer Electromagnetic and the Pointer Electromagnetic.

1. Verify the pointing device is not bent.
2. Conduct a general functionality test. See Chapter "Preparing Navigated Instruments."

7.3.3. Navigated Suction Tubes

The Frazier and Eicken Electromagnetic (Figures 73 and 74) are navigated instruments for ENT surgery. Compatible navigation systems are thereby able to locate and navigate the touched structures.

Navigated Suction Tubes

Figure 73 (A) - Frazier Electromagnetic (REF 8000-050-005)

Figure 74 (B) - Eicken Electromagnetic (REF 8000-050-006)

Application

Conduct a general functionality test. See Chapter “Preparing Navigated Instruments.”

CAUTION

Make sure that instruments are not damaged by third party products. Drills and shavers may damage the polymer tubing.

7.3.4. TGS Guidewire (US only)

The TGS Guidewire (Figure 75) is a navigated guidewire for use with the Stryker ENT XprESS LoProfile Dilation System. Compatible navigation systems are thereby able to locate and navigate the touched structures.

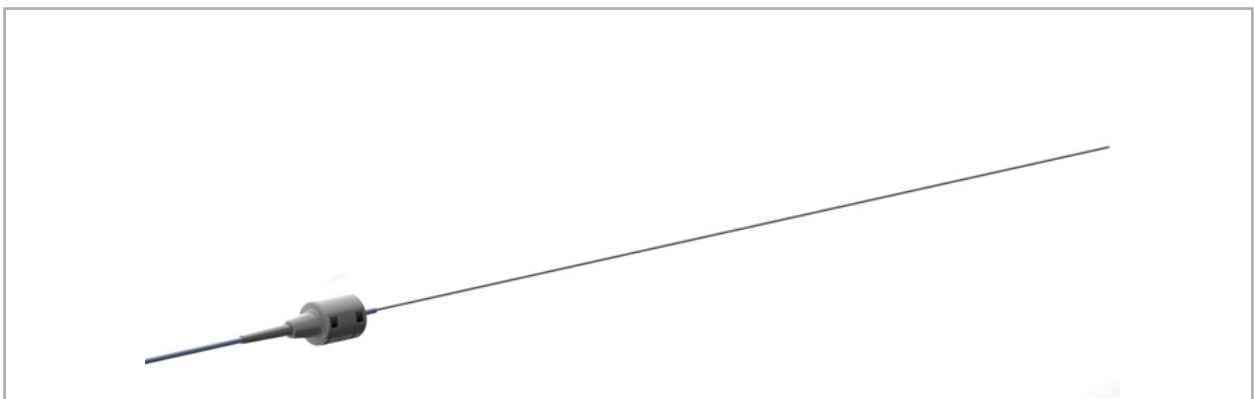
Figure 75 – TGS Guidewire (REF 8000-060-009)

Application

Follow the instructions for use of the “TGS Guidewire” for how to prepare for navigation and how to perform a general function test.

7.3.5. Instrument Clamps

The Instrument Clamps (Figure 76) add a location tracker to surgical instruments. Compatible navigation systems are thereby able to navigate conventional surgical instruments.

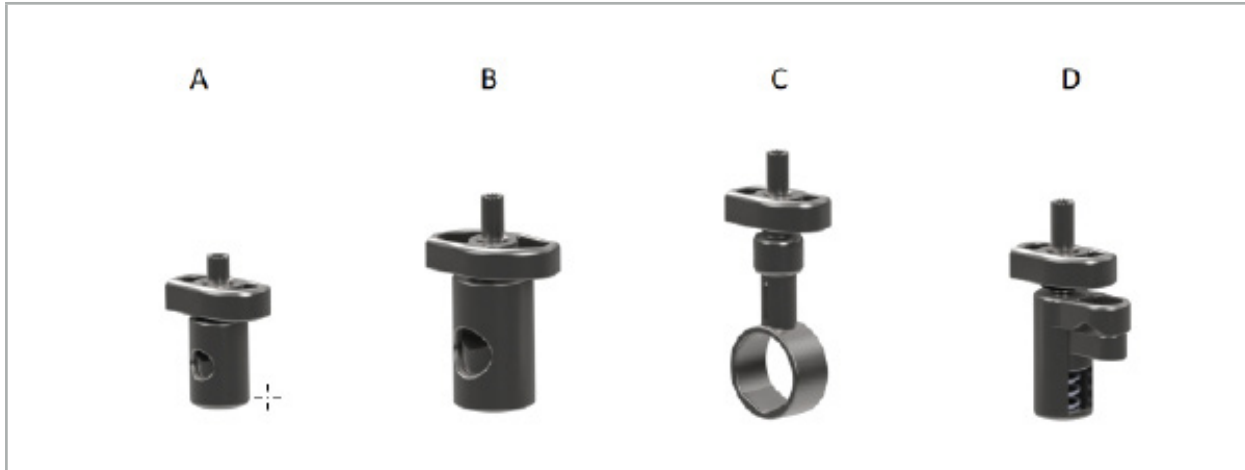
Figure 76 – Instrument Clamps, Individual Parts

A - Instrument Clamp 2-6 mm (REF 8000-060-011) – The Instrument Clamp 2-6 mm is designed for the navigation of instruments with a round or multi-angular profile whose diameter is not larger than 6 mm.

B - Instrument Clamp 6-10 mm (REF 8000-060-012) – The Instrument Clamp 6-10 mm is designed for the navigation of instruments with a round or multi-angular profile whose diameter is not smaller than 6 mm and not larger than 10 mm.

C - Instrument Clamp 10-16 mm (REF 8000-060-013) – The Instrument Clamp 10-16 mm is designed for the navigation of instruments with a round or multi-angular profile whose diameter is not smaller than 10 mm and not larger than 16 mm.

D - Instrument Clamp Forceps (REF 8000-060-010) – The Instrument Clamp Forceps is designed for the navigation of instruments with a round or multi-angular profile whose diameter is not smaller than 5.5 mm and not larger than 10 mm.

i The Universal Tracker Electromagnetic can be combined with the Instrument Clamps (A-D) for electromagnetic navigation.

7.3.6. Universal Tracker Electromagnetic

The Universal Tracker Electromagnetic (REF 8000-060-006) references patient anatomy or adds an electromagnetic localizer to surgical instruments. For this purpose, the product is attached to compatible adapter, which is securely connected to the patient or the instrument.

Figure 77 – Universal Tracker Electromagnetic

Application:

Screw the knurled screw through the tracker so that the knurled screw is loosely connected with the tracker.

Fixing an Universal Tracker Electromagnetic to Instrument Clamps:

Refer to Figure 78.

1. Attach the Universal Tracker Electromagnetic to the Instrument Clamp.
2. Screw the knurled screw tightly so that the Universal Tracker is rigidly connected with the Instrument Clamp.

Figure 78 – Preparation of Instrument Clamp with Universal Tracker Electromagnetic

WARNING

- Make sure that the instrument to be navigated is not restrained by the attached Instrument Clamp.
- The Instrument Clamp must be securely fixed to the instrument before beginning with the calibration.

3. Calibrate the instruments. Refer to section “Calibration and Verification of Navigated Instruments.”
4. Test/verify calibration.

WARNING

Make sure the device is securely attached to the surgical instrument because the navigation will otherwise be imprecise and useless. If the device moves, a new calibration is required.

7.3.7. Instrument Clamps Electromagnetic (US only)

The Instrument Clamps Electromagnetic add a location tracker directly to surgical instruments. These electromagnetic clamps bypass the need for another electromagnetic tracker (such as the Universal Tracker Electromagnetic) to be attached to a surgical tool.

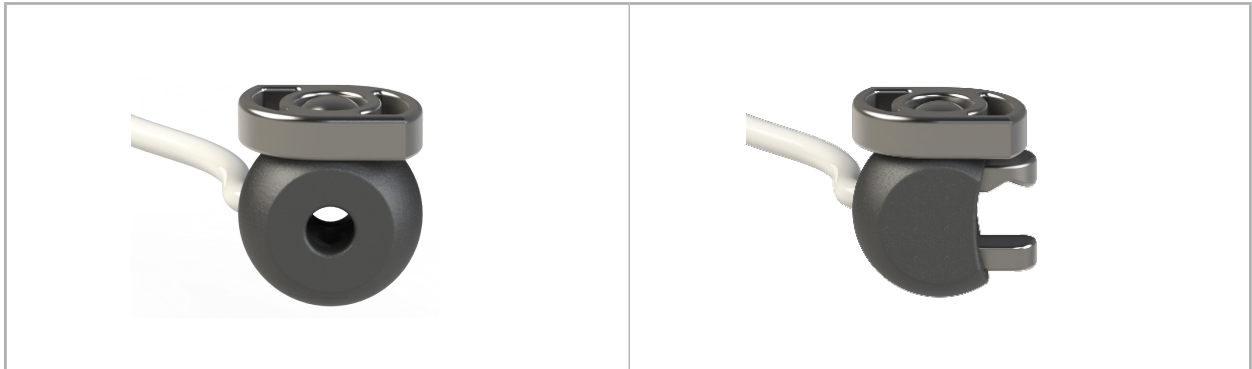
Figure 79: Instrument Clamp
Electromagnetic Sphere
(REF 8000-060-020)

Figure 80: Instrument Clamp
Electromagnetic Universal
(REF 8000-060-021)

The Instrument Clamp Electromagnetic Sphere is optimised for tracking instruments between 2.0mm and 4.6mm in diameter (inclusive).

The Instrument Clamp Electromagnetic Universal is optimised for tracking instruments between 3.0mm and 6.0mm in diameter (inclusive).

For detailed attachment instructions, and reprocessing information specific to the Instrument Clamps Electromagnetic, consult the Instrument Clamps Electromagnetic Instructions for Use (700000842818). For other information regarding using and navigating with the Instrument Clamps Electromagnetic, continue in this document.

WARNING

- Inspect the Instrument Clamps Electromagnetic prior to each use for loose components or damage. Do not use if these conditions exist.
- Ensure that the instrument to be navigated is not restrained by the attached clamp. Should the attachment of either Instrument Clamp Electromagnetic restrict the functionality or moveability of the navigated instrument, do not navigate with the clamps.

7.3.8. Endoscope Tracker Electromagnetic

The Endoscope Tracker Electromagnetic (REF 8000-060-001) is connected with rigid endoscopes for ENT surgery and extends it by an electromagnetic locator. This gives compatible clinical navigation systems the ability to navigate conventional surgical endoscopes. The Endoscope Tracker Electromagnetic is suited for endoscopes with 4 mm x 180 mm.

Figure 81 – Endoscope Tracker Electromagnetic

Application:

Refer to Figure 82.

1. Slide the Endoscope Tracker Electromagnetic over your endoscope (4 mm).
2. Fix the Endoscope Tracker Electromagnetic with the bayonet catch.

Figure 82 – Endoscope with Endoscope Tracker Electromagnetic

WARNING

Make sure the device is securely attached to the surgical instrument because the navigation will otherwise be imprecise and useless. If the device moves, a new calibration is required.

CAUTION

Make sure that instruments are not damaged by third party products. Drills and shavers may damage the polymer tubing.

7.3.9. Calibration Body Electromagnetic

The Calibration Body Electromagnetic (REF 8000-060-002) provides a hardware platform for the intraoperative calibration of surgical instruments.

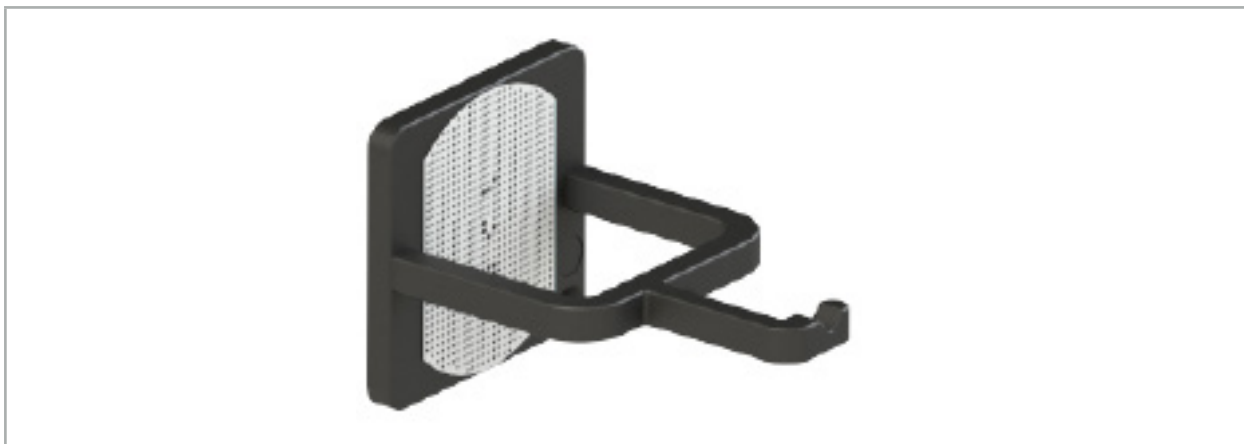
Figure 83 – Calibration Body Electromagnetic

Application:

1. Conduct a general functionality test. Refer to section “Preparing Navigated Instruments.”
2. Check the dot pattern for damage.

WARNING

Do not use the calibration target if there is any damage.

7.3.10. Navigation Tool Extension Cable

The Navigation Tool Extension Cable (REF 8000-050-011) is intended for sterile handling of compatible navigated instruments.

Application

1. Connect the plug of the Navigation Tool Extension Cable to one of the electromagnetic instrument ports of the navigation unit.
2. Fix the socket of the Navigation Tool Extension Cable to a sterile location with a sterile tape. Now it is possible to connect electromagnetic instruments to the instrument port in a sterile environment.

3. Conduct a general functionality test. See Chapter “Preparing Navigated Instruments.”

Figure 84 – Navigation Tool Extension Cable

WARNING

- Ensure that the instrument connection socket is kept at least 30 cm away from the Field Generator. Otherwise the transmitted signal may be lost or the accuracy of the instrument may deteriorate.
- Navigation Tool Extension cables must not be plugged into one another.

7.4. Suitability of Surgical Instruments

WARNING

Should attachment of the Instrument Clamps or the Instrument Clamps Electromagnetic affect the functionality or moveability of the instrument or of parts of the instrument, then the instrument cannot be navigated via the clamps.

Unless otherwise specified, all references to ‘clamps’ in this section refer to both the Instrument Clamps and the Instrument Clamps Electromagnetic.

If connected to the clamps, numerous surgical instruments can be upgraded to navigated instruments. However, not all instruments are suited for navigation.

In principle, the navigation system determines the position of the instrument tracker attached to the instrument. The point on the instrument which is to be navigated is taught to the navigation system through a calibration. Only points on the instrument that are rigidly connected to the instrument tracker should be calibrated. If the instrument is deformed, the precision of the navigation information deteriorates.

The position of the instrument tracker is determined very precisely. However, if the distance between the instrument tracker and the navigated point changes, the precision of the navigated point deteriorates. The closer the instrument tracker is placed to the navigated point, the more precise the navigation information (Figure 86).

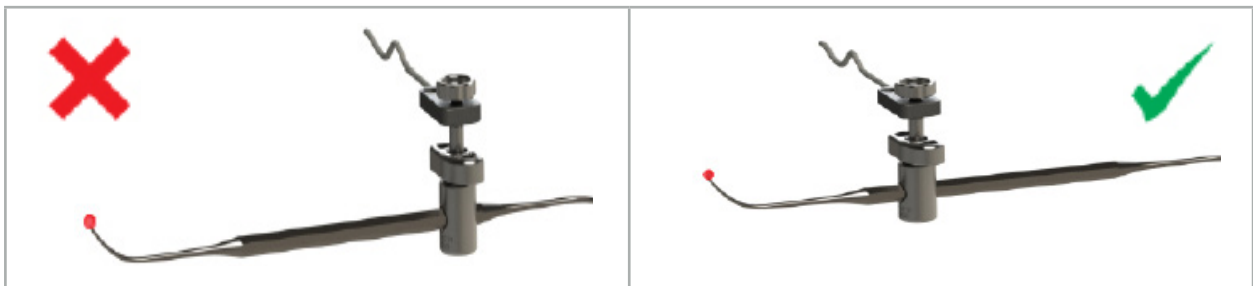
Figure 85 – Instrument with low precision (left) and Figure 86- Instrument with high precision (right)

Application

To attach a clamp:

1. Unscrew the fastening nut counter-clockwise.
2. Press the clamp together and position it on the instrument.
3. Tighten the fastening nut clockwise until the clamp is rigidly connected with the instrument.

Figure 87 (left) – Fixing an Instrument Clamp 2-6 mm (REF 8000-060-010) to a surgical instrument

Figure 88 (right) - Fixing an Instrument Clamp Forceps (REF 8000-060-013) to a surgical instrument

WARNING

If it is not possible to fix the clamp on the instrument without damaging it, the instrument is not suitable for use. Do not use any tools for tightening the fastening nut.

8. Conducting Navigated Surgery

To switch on the navigation system, see Chapter “Switching On the Navigation System” and “Starting the Navigation Software.”

WARNING

- Do not navigate in an unvalidated environment, as it may contain elements that affect navigation functions. The system can be adversely affected by electromagnetic field disturbances from other objects in the room, the close proximity of metal, and the close proximity of another Field Generator. Failure to test for such disturbances will increase the possibility of inaccurate position measurement and possible personal injury.
- During use of the navigation unit with electromagnetic position measurement, no ferromagnetic material may be within the measurement area of the Field Generator; otherwise, the precision of the navigation may be significantly reduced.

Should the power supply fail briefly during the application, the system must be restarted. The data from a running video documentation is lost.

In the unlikely event that the navigation software or the navigation unit should experience difficulties during planning or intra-operatively, restart the software and navigation unit.

8.1. Loading Patient Data

1. Press the Loading Patient Data button (Figure 89). The Data Import dialog box appears. On the left side of the dialog, choose a data source for the import. The Plannings button contains all the plannings created and saved with the software.

Figure 89 – Data Import

2. Choose the desired planning and press the Load button.

8.2. Starting the Navigation

1. Once the dataset is loaded and patient registration is chosen, the message “Ready for Navigation” appears. The NAVIGATION tab is activated (Figure 90).

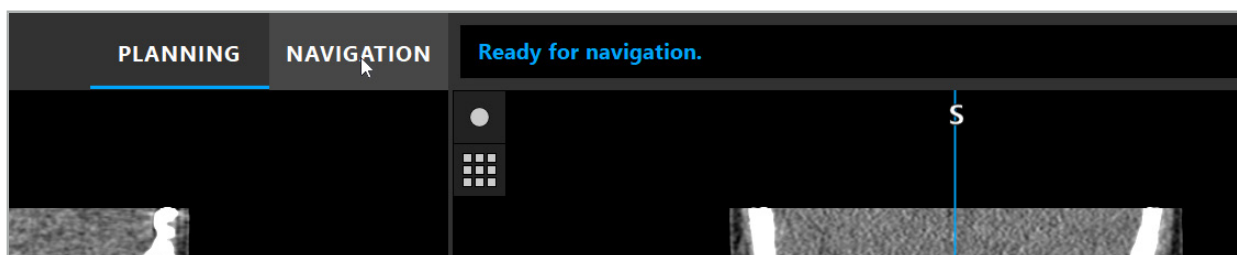
Figure 90 – Starting the Navigation

2. Start the navigation mode by pressing the NAVIGATION tab.

i The planning data is saved automatically when changing to navigation mode.

8.3. Connecting Electromagnetic Instruments

Four ports are available at the front side of the navigation unit for connecting electromagnetic instruments. The plug clicks in audibly.

After an electromagnetic instrument is connected to the navigation unit, a dialog box appears showing the use counter of the instrument. See Chapter "Lifetime of Electromagnetic Instruments."

8.4. Status Displays

During navigation mode, the status displays of Patient Tracker and navigated instruments are visualized in the taskbar on the right. The status displays visualize the visibility of the Patient Tracker and instruments that are in use with a green or a red background (Figure 91).

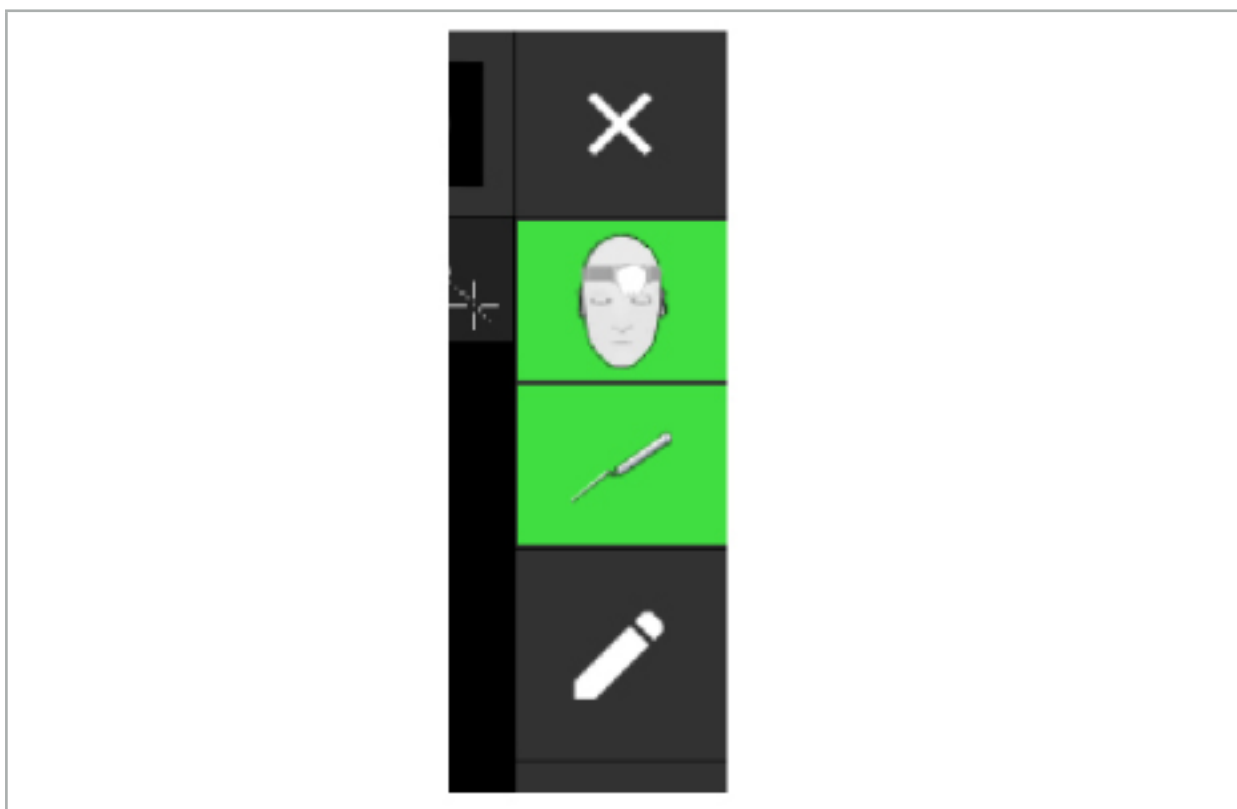
Figure 91 – Status Displays

- The upper status display shows the Patient Tracker that is in use.
- The status display in the middle shows involved imaging systems (endoscope).
- The lower status display shows the navigated instrument in the red cross hair. This can also be an involved imaging system.

8.5. Checklist: Visibility of Navigated Instruments

The visibility of patient trackers and navigated instruments depends on the following:

- Is the instrument located within the operating range of the measuring system?
- Is the instrument undamaged, or damaged/bent?

- Are there any para- or ferromagnetic substances (iron, steel, aluminum...) within an 80 cm-radius of the Field Generator or between Field Generator and instruments?
- Are there any radiating sources of disturbances (Mobile phones, drills, mills...) within a 100 cm-radius around the Field Generator?

i If the Patient Tracker is not visible, the navigated instrument will likewise be shown as not visible.

8.6. Freezing Navigation

Pausing the navigation allows you to view the patient image data manually and to change the planning components in navigation mode without having to return to planning mode.

The Edit button pauses the navigation (Figure 92). Pause mode is indicated by highlighting the icon (Figure 93).

Figure 92 – Navigation Mode

Figure 93 – Pause Mode

Planning data can be edited in pause mode. For this purpose, the buttons for working on the planning objects appear. The operation is the same as in planning mode.

The position within the slice image at the time of suspension is visualized by a grey cross hair. The distance of the current position to this suspended position is shown in the display's upper right corner. Return to the suspended position by using the cross-section view's Centering button. Continue with the navigation by pressing the Cross Arrow button again.

8.7. Calibration and Verification of Navigated Instruments

All navigated instruments must be calibrated or verified before use.

By calibration, the navigation system learns which of the instrument points is to be navigated, or which optical features the imaging systems have. After ending the surgery, this information is deleted. The calibration or verification can be conducted with the following calibration areas that are integrated in the following instruments:

- Calibration Body Electromagnetic
- Patient Tracker Electromagnetic – 10 Uses
- Patient Tracker Electromagnetic

The calibration of endoscopes can be done exclusively with respective calibration bodies.

WARNING

Instruments of the same construction type cannot be distinguished by the system. Only use one instrument with the same tracker geometry. If you want to replace one instrument by another with the same tracker geometry, then this must be calibrated or verified again before use.

A calibration area consists of a calibration cone (✓) and a calibration plane (x) (Figure 94).

Figure 94 – Calibration Areas: Patient Tracker Electromagnetic

8.7.1. Calibration of Surgical Instruments

WARNING

Not all instruments are suited for navigation. Refer to “*Instrument Clamps*” section.

Conventional instruments with an added tracker must be calibrated before use with the Instrument Clamps. The Tool Calibration dialog (Figure 95) appears automatically if a tool is held visibly in the vicinity of the Calibration Body Electromagnetic.

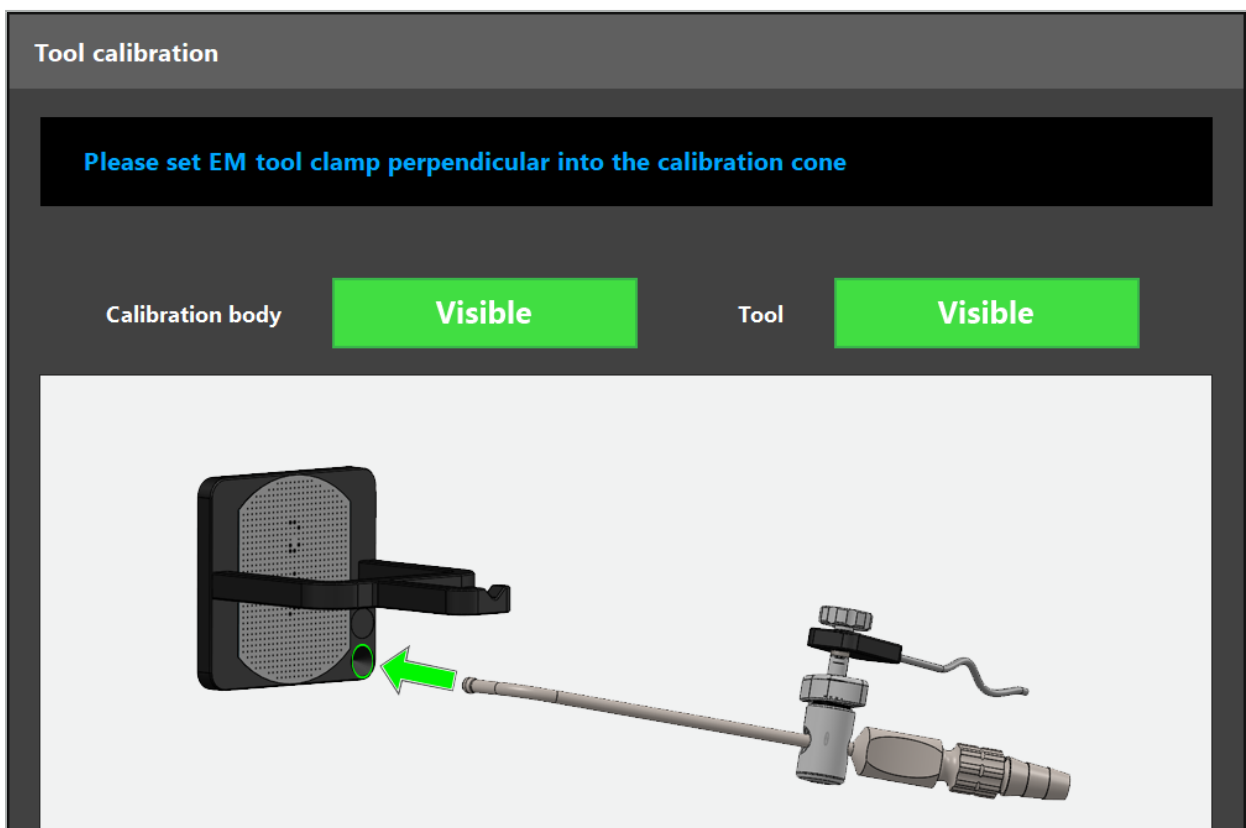
Figure 95 – Tool Calibration Dialog

Refer to Figure 96 and Figure 97.

1. Move the navigated instrument and the calibration area into the measuring volume of the navigation system.
2. Slowly and vertically move the navigation tip of the navigated instrument to the calibration cone and hold it still.
3. Calibration is conducted automatically. A small clock indicates the progress. The step is complete when you hear a confirmation tone.

i Hold the instrument to the calibration cone so that the tip axis stands vertically in the calibration cone. This step defines the axis which is displayed later.

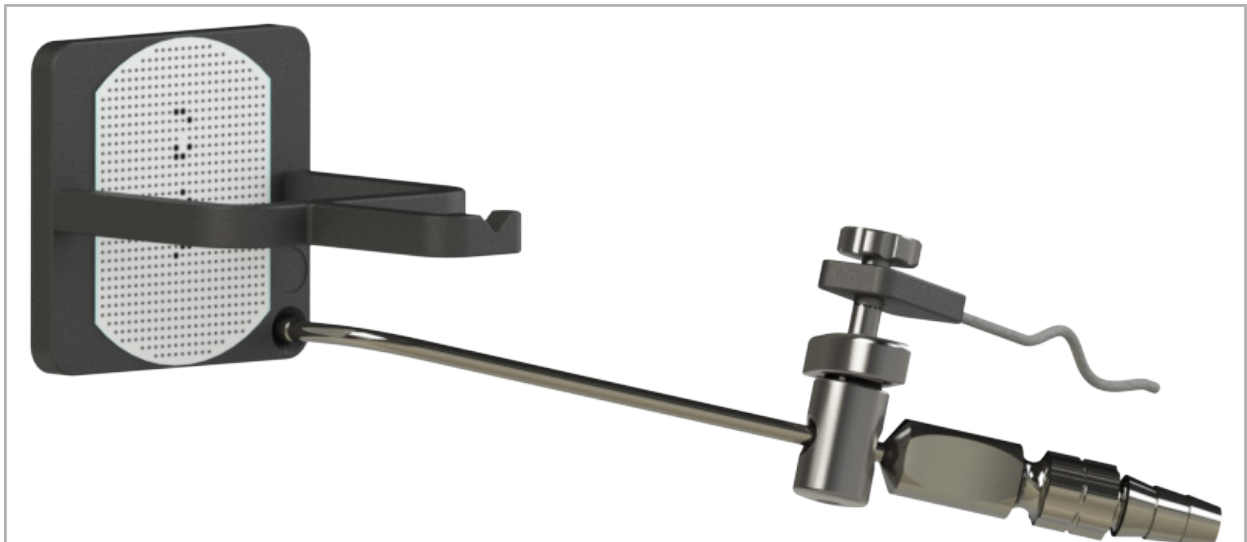
Figure 96 – Calibration Step 1: Holding the Instrument Tip into the (Lower) Calibration Cone

4. Hold the instrument to the calibration plate so that the axis of the tip stands vertically at the calibration area and hold it still.

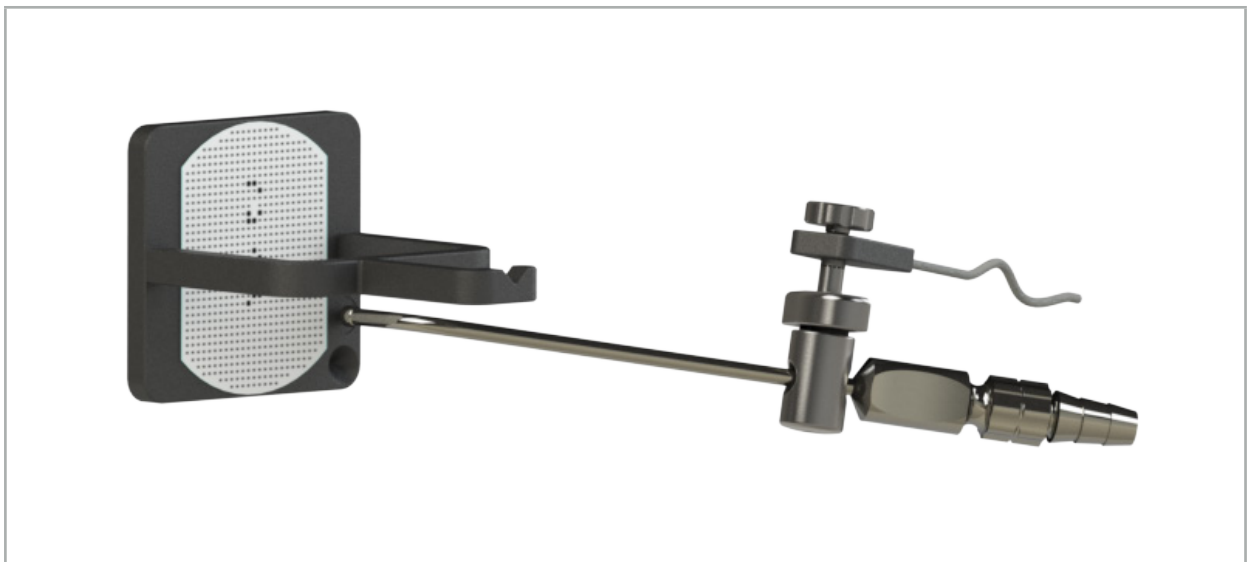
Figure 97 – Calibration Step 2: Holding the Instrument Tip to the (Upper) Calibration Plate

5. A small clock indicates the progress. Calibration is finished when you hear a confirmation tone.

WARNING

- Check the precision of the calibration carefully before using the instrument. To do so, touch an anatomical structure with the calibrated instrument and compare the touched structure with the scene visualized by the navigation software. Should the deviation be too big, repeat the calibration.
- When using an Instrument Clamp (including Electromagnetic), ensure that the clamp is securely attached to the surgical instrument at all times. Otherwise the navigation will be imprecise and useless. If the clamp moves, a new calibration is required.
- Do not implement any changes to the instrument trackers after the registration; otherwise, navigation information may deteriorate. If the location of the tracker is changed, registration must be repeated.
- Do not exert force when calibrating.

Resetting an Instrument Calibration

If a conventional instrument or endoscope was previously calibrated with an instrument tracker, the following instructions describe how to reset the calibration. Resetting the calibration is only necessary in the following situations:

- To re-calibrate an instrument tracker on a patient tracker
- To use an instrument tracker for a conventional instrument after it was previously used to track an endoscope

To re-calibrate an Universal Tracker Electromagnetic or Endoscope Tracker with a Calibration Body does not require to reset the calibration. Just bring Calibration Body and tracker near each other and follow the normal calibration procedure.

All calibrations can be reset by clicking on the visibility icon of the tool in the status display and confirming the following dialog as shown below:

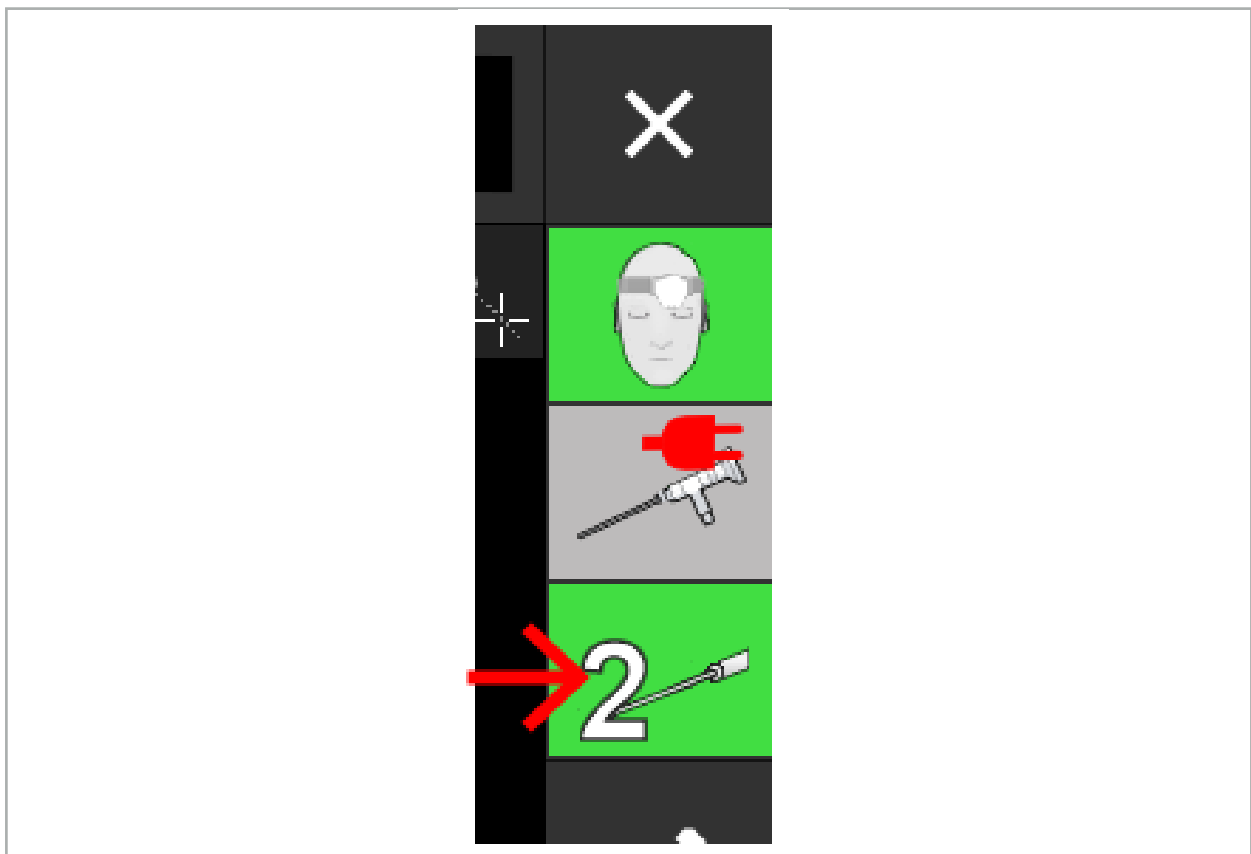
Figure 98 – Click on the tool visibility indicator icon in the status view to reset.

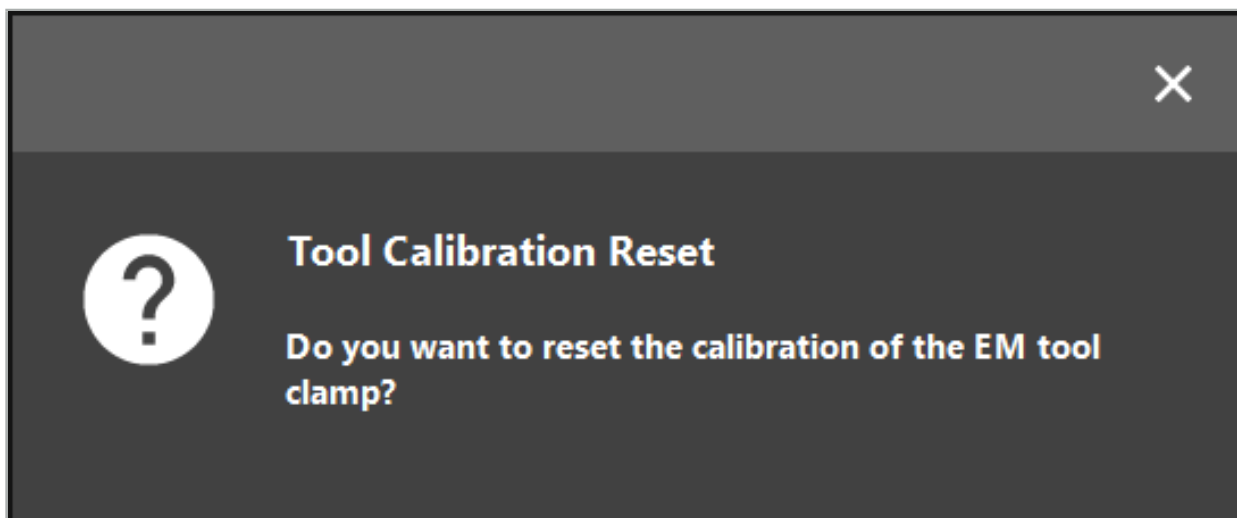
Figure 99 – Confirm the reset of the tool calibration.

The Universal Tracker Electromagnetic and the Instrument Clamps Electromagnetic attached to conventional instruments can also be reset by holding them up-side down on top of the patient tracker as shown below:

Figure 100 – Hold the tracker up-side down over patient tracker to reset calibration

i Always make sure that the instrument to reset is shown as the active tool in the status display. Only the instrument shown in the status display is reset.

8.7.2. Calibration of Rigid Endoscopes (Augmented Reality)

Navigated endoscopes must be calibrated before application. This is performed with the help of:

- Calibration Body Electromagnetic

For navigating rigid endoscopes, the following adapters are available:

- Instrument Clamps
- Instrument Clamps Electromagnetic (US Only)
- Endoscope Tracker Electromagnetic

Rigid endoscopes with a diameter of 2.5 mm to 9 mm and a viewing angle of 0° to 45° are supported.

Preliminary Check of Navigated Endoscopes

Before starting calibration, check the following:

- Is the attachment/adapter securely fastened to the endoscope?

 The attachment bayonet cap or the adapter screw winding must be closed, and must not be moveable in relation to the endoscope.

- Is the endoscope securely fastened to the endoscope camera?

 Make sure the endoscope rests tightly in the endoscope camera fastener. The endoscope must not rotate in relation to the endoscope camera during use.

8.8. Preset of Navigated Endoscopes

- ▶ Adjust zoom and focus to an area of 5 mm to 25 mm in front of the lens.

WARNING

After calibration, DO NOT change the endoscope camera settings; otherwise, the precision of augmented reality deteriorates. If zoom and focus are adjusted after the calibration, calibration must be repeated.

Calibration

To calibrate navigated endoscopes:

1. Slide the navigated endoscope through the Calibration Body guiding (Figure 101).

Figure 101 – Navigated Endoscope in Calibration Body Electromagnetic

2. Move the navigated endoscope and the calibration area into the measuring volume of the navigation system. The Tool Calibration dialog opens.

The Tool Calibration dialog is shown only if both the endoscope and the Calibration Body Electromagnetic are detected by the navigation system in the measuring volume.

3. Slowly move the navigated endoscope backward and forward inside the Calibration Body Electromagnetic until the displayed yellow squares turn green (Figure 102).

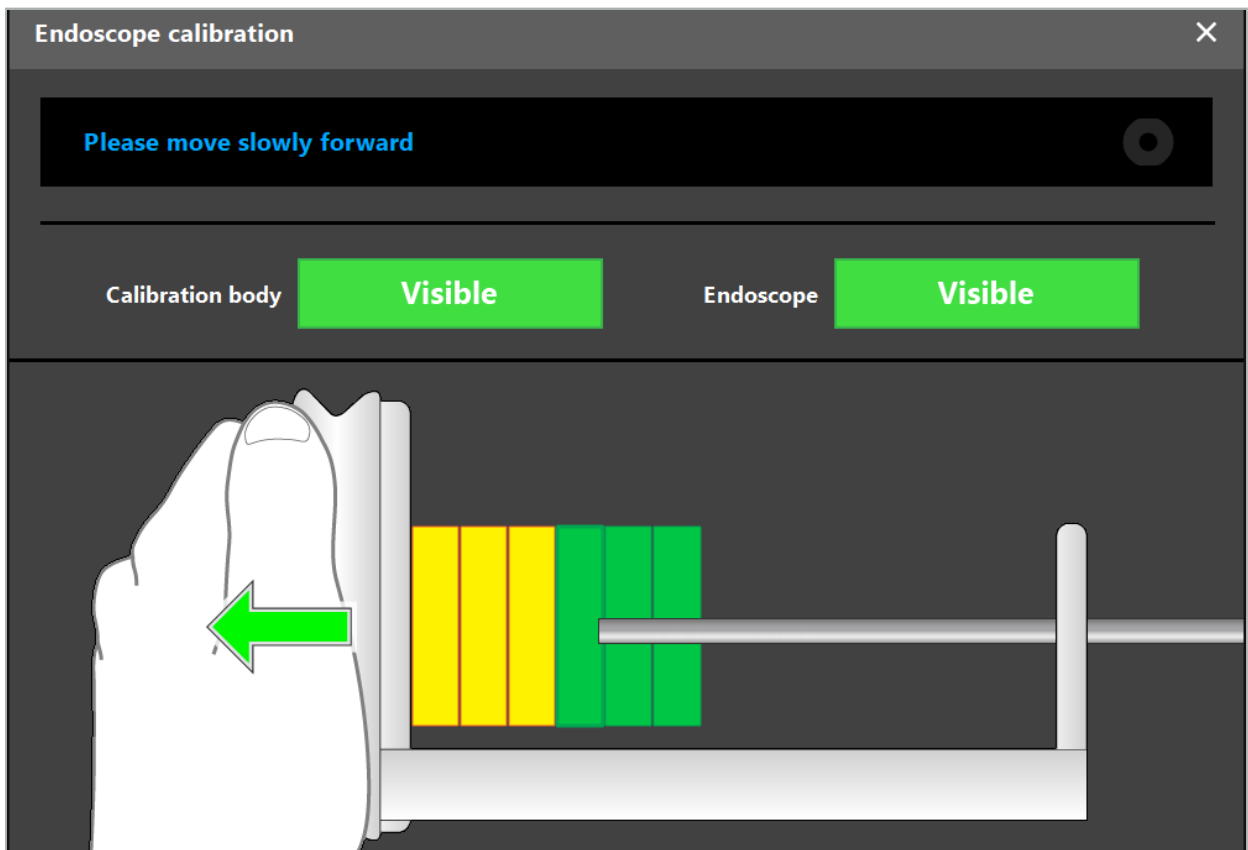
Figure 102 – Calibration Dialog

- If the calibration pattern is not recognized during calibration, adjust the zoom, focus, and perhaps brightness so that the calibration pattern is clearly recognized (Figure 103).

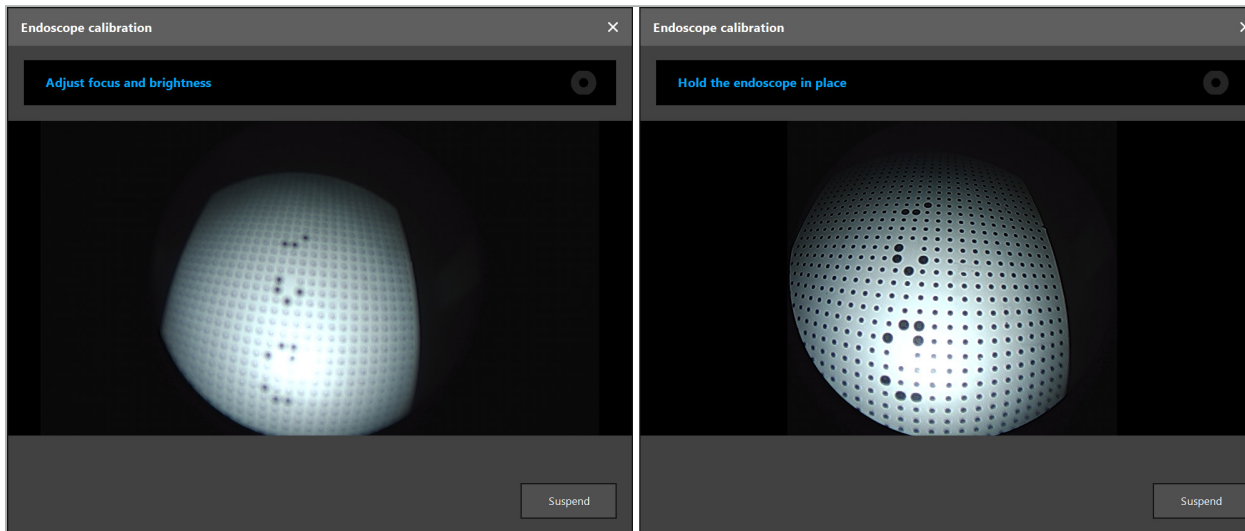
Figure 103 – Calibration Dialog: Left – Image Defocused and Overexposed; Right – Optimal Image

- After collecting enough calibration data points, the software processes the calibration (Figure 104).

Figure 104 – Computing Calibration

In rare cases, the system may inform you via a dialog box that the calibration cannot be calculated. Possible reasons might be that the endoscope was twisted during the calibration, or that zoom or focus were altered. After confirming the dialog, conduct a new calibration with the correct settings.

WARNING

- After calibration, check that the accuracy is precise enough for the planned application. Plan a point structure on prominent locations of the patient surface and check the accuracy of the displayed positions. If the deviation is too large, conduct the calibration again.
- If any changes are made to the endoscope after calibration (adjusting zoom, focus, brightness, twisting of endoscope in relation to endoscope camera, moving of Tracker Electromagnetic, etc.), calibration must be repeated.
- The displayed video images can be affected by a time lag. Never rely exclusively on the video information; also make use of your tactile feeling in the operation area.

Discarding Calibration

If the calibration must be discarded, e.g. to conduct a new calibration, separate the navigated endoscope from the Calibration Body Electromagnetic and keep them apart until the calibration dialog disappears. Then start the calibration again.

Features when Using the Instrument Clamps

To navigate an endoscope that is not compatible with the Endoscope Tracker Electromagnetic, use the Instrument Clamps or Instrument Clamps Electromagnetic for navigation and calibration. Conduct the instrument calibration with the navigated endoscope before you conduct the endoscope calibration. The following steps are necessary:

1. Calibrate the surgical instruments. See Chapter “Calibration of Surgical Instruments.”
2. Calibrate the navigated endoscope. See Chapter “Calibration of Rigid Endoscopes (Augmented Reality).”

i Be careful not to damage the endoscope when inserting it into one of the Calibration Body Electromagnetic cones.

Be careful when attaching the Instrument Clamps to the endoscope shaft. Tightening the adapter with too much force could damage the endoscope.

8.9. Interaction Areas

Several navigation instruments have interaction areas. These are marked with an “x” or a “✓.” If these areas are touched with an instrument, the following happens:

If dialog boxes are opened:	✓	“Confirm” of displayed software dialogs
	X	“Reject” of displayed software dialogs
In navigation mode:	X	(long holding): “Discarding” of current image-to-patient registration

8.10. Conducting the Image-to-Patient Registration

Depending on the type of image-to-patient registration chosen in planning mode, different actions are conducted.

WARNING

Make sure that the Patient Tracker is securely connected to the patient because the navigation will otherwise be imprecise or useless. If the Patient Tracker is moved, a new image-to-patient registration is required.

i Refer to the advice window for help.

8.10.1. Image-to-Patient Registration: Landmark

The landmark registration mode can be conducted with the following instruments:

- Precision Pointer Electromagnetic

Application

1. Touch the flashing landmark as precisely as possible with a pointer on the patient. Hold the pointer still until you hear a confirmation tone and a new landmark starts flashing.

WARNING

Make sure when touching the landmarks on the patient’s tissue that you do not exert any force with the pointer, so as to not move the tissue.

2. Repeat procedure until all landmarks are touched (Figure 105).

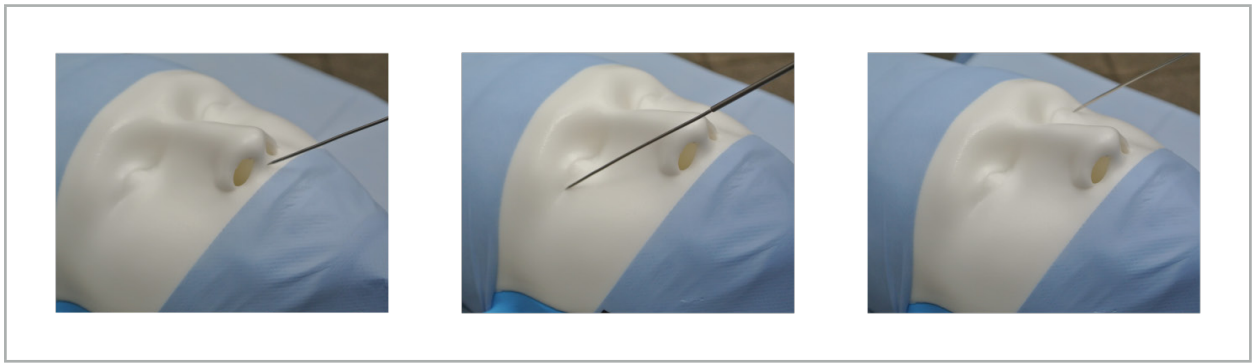
Figure 105 – Registering the Landmarks

3. After successfully touching all landmarks, the advice window shows the message, “Please confirm or increase accuracy.”

i If no sufficient correspondence between touched and planned landmarks is found, the advice window shows the message, “Please check anatomy and retouch landmarks.” The landmarks must be touched again iteratively.

If no sufficient correspondence is found after several attempts, change to Planning mode and plan the landmarks again.

4. Check and confirm the precision of the image-to patient registration. See Chapter “Verification/Confirmation of the Image-to-Patient Registration.”

8.10.2. Image-to-Patient Registration: Surface

The surface registration mode can be conducted with the following instruments:

Electromagnetic:

- Precision Pointer Electromagnetic
- Registration Pointer Electromagnetic

Application

1. Touch the flashing landmark as precisely as possible with a pointer on the patient. Hold the pointer still until you hear a confirmation tone and a new landmark starts flashing.

⚠ WARNING

Make sure when touching the landmarks on the patient’s tissue that you do not exert any force with the pointer, so as not to move the tissue.

2. Repeat procedure until all landmarks are touched (Figure 106).
3. After successfully touching all landmarks, the advice window shows the message, “Move pointer over surface.”

i If no sufficient correspondence between touched and planned landmarks is found, the advice window shows the message, “Please check anatomy and retouch landmarks.” The landmarks must be touched again iteratively.

If the surface touching cannot be completed successfully after a few minutes, the surface type image-to-patient registration should be started again.

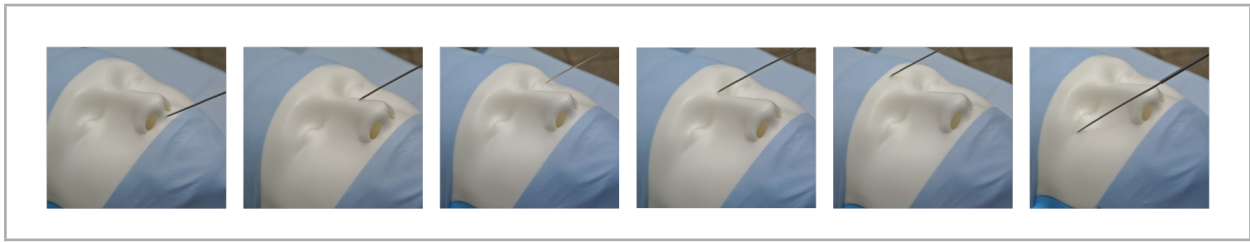
Figure 106 – Recording Point on the Surface

4. Now move the pointer over the surface of the patient in wide curves. The more prominent structures that are moved along, the more precise the registration. A small clock in the advice window indicates the progress of the registration.

i Ideally, the touching curve should be distributed over both halves of the face equally and contain bony areas such as the cheekbone, forehead, and nose bridge.

If no sufficient correspondence is found after several attempts, change to Planning mode and plan the landmarks again.

5. After successfully registering, the advice window shows the message, “Please confirm or increase accuracy.”
6. Check and confirm the precision of the image-to patient registration. See Chapter “Verification/Confirmation of the Image-to-Patient Registration.”

8.10.3. Image-to-Patient Registration: Enhanced Surface

The enhanced surface registration mode can be conducted with the following instruments:

Electromagnetic:

- Precision Pointer Electromagnetic
- Registration Pointer Electromagnetic

Application

1. Touch the flashing landmark as precisely as possible with a pointer at the patient. Hold the pointer still until you hear a confirmation tone.
2. Move along the initial path until you hear a further confirmation tone.
3. Now move the pointer over the surface of the patient in wide curves. The more prominent structures that are moved along, the more precise the registration. A small clock in the advice window indicates the progress of the registration.

i Ideally, the touching curve should be distributed over both halves of the face equally and contain bony areas such as the cheekbone, forehead, and nose bridge.

If the surface touching cannot be completed successfully after a few minutes, the enhanced surface type image-topatient registration should be started again.

4. After successfully registering, the advice window shows the message, “Please confirm or increase accuracy.”
5. Check and confirm the precision of the image-to patient registration. See Chapter “Verification/Confirmation of the Image-to- Patient Registration.”

8.11. Verification/Confirmation of the Image-to-Patient Registration

The navigation software allows the completion of the conducted image-to-patient registration once a sufficient algorithmic accordance is reached. The advice window shows the following message, depending on which type of image-to-patient registration is conducted:

Landmark	Surface	Enhanced Surface
"Please confirm or increase accuracy."	"Hold point to confirm."	"Hold point to confirm."

Verification of the System Precision

1. To check the precision, touch one or several of the patient's prominent anatomic structures (e.g. tooth gaps in the upper jaw, nose tip and creases) with a pointer and compare it to the visualized position.
2. If the correspondence (accuracy) is acceptable, hold the pointer in a position on the patient's surface until the progress display appears in the advice window and turns completely green. A confirmation tone signals the successful end of the image-to-patient registration.
3. If the correspondence (accuracy) is not acceptable, repeat the image-to-patient registration. See Chapter "Resetting the Image-to-Patient Registration."
4. After finishing the image-to-patient registration, the 3D view switches automatically to the Video view.

WARNING

- Verify the visualization of the instrument position on several anatomic structures after the registration and repeat this during the surgery. If deviations are too large, conduct a new image-to-patient registration.
- Deviations between patient image data and reality due to old or modified image data, for instance because of tissue swelling, can severely deteriorate the precision of the system. Thoroughly check if the patient image data are suitable for navigation, e.g. by touching prominent anatomical structures.
- If the image-to-patient registration was not performed in a sterile manner, and therefore the non-sterile Patient Tracker was later exchanged for a sterile one, the verification of the image-to-patient registration must be repeated.
- Start with the navigated operation only after confirming the precision of the image-to-patient registration.
- Make sure that the Patient Tracker is securely attached to the patient during use, as the navigation would otherwise be inaccurate or useless. If the Patient Tracker moves, the image-to-patient registration must be repeated.

8.12. Resetting the Image-to-Patient Registration

To reset the Image-to-Patient Registration the following options are available:

- Hold an already registered/verified instrument at the interaction area "x" of the patient tracker until the progress bar in the advice windows is full. If the patient tracker does not have an interaction area "x", the registration cone of the patient tracker or calibration body can be used alternatively for resetting the Image-to-Patient Registration. Hold an already registered/verified instrument at a plane calibration area until the progress bar in the advice windows is full.
- Switch to planning mode and then again to navigation mode.

8.13. Reuse an Image-to-Patient Registration

After you perform the image-to-patient registration, the results are stored with the planning and can be reused repeatedly without the need to perform the registration procedure again. In order to reuse a registration, you must be in planning mode, press and hold the left mouse button on the "NAVIGATION" tab.

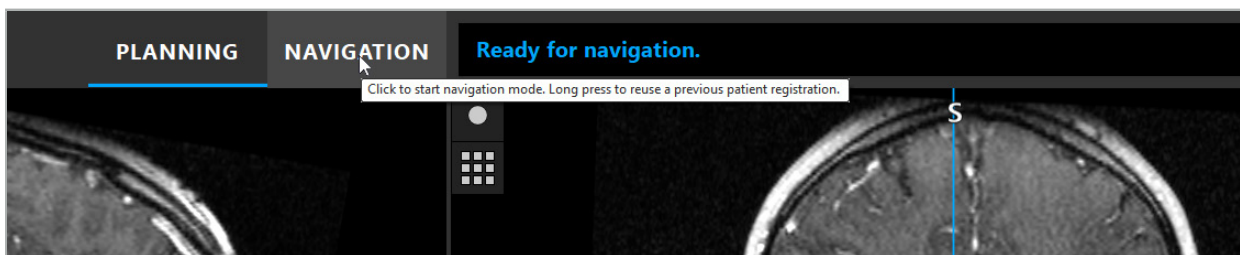
Figure 107 – Press and Hold to Reuse a Registration

The software will offer all suitable patient registrations in the following dialog. Choose a registration by time of day that it has been performed. To abort this selection click Cancel or close the dialog window and perform a manual registration.

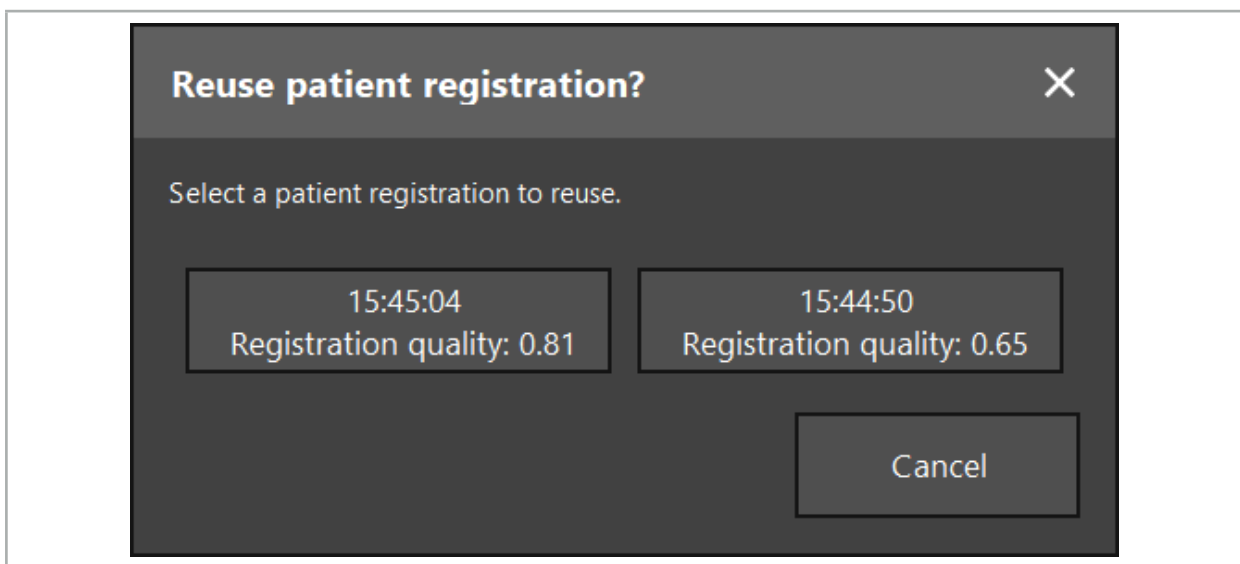
Figure 108 – Select a Patient Registration to Reuse

After a patient registration was selected to be reused the precision of the image-to-patient registration must be verified and confirmed. See Chapter "Verification/Confirmation of the Image-to-Patient Registration."

i Patient registrations can only be reused within 24 hours after the original registration has been performed. Older patient registrations are no longer available.

Patient registrations are tied to a specific planning. If there are multiple planning files with the same patient data, the image-to-patient registration can only be reused if the same planning was loaded that was used to perform the original registration.

In order to modify the current planning after you have already performed an image-to-patient registration, e. g. to create a new planning object, it is not necessary to switch back to planning mode. Instead, you can freeze the navigation in order to retain the current patient registration, see Chapter "Freezing Navigation."

8.14. Visual Checking of the Navigation Information

If the accordance between the navigated instrument position shown on the screen and the real position on the patient is not sufficient, repeat the image-to-patient registration. If no sufficient accordance is reached even after conducting the image-to-patient registration repeatedly, continue the surgery without support of the navigation system.

Note that the navigation system is an aid to orientation, whose precision can be influenced by errors in handling, the recording of images, or other technical errors. The conventional view of the surgery site remains most important for conducting the surgery.

WARNING


Each treatment process may only be performed if the visual observation of the systemic effects is assured.

8.15. Visualizing the Position of Navigated Instruments

Each navigated instrument has a navigation point, the position which is tracked by the navigation system and shown as a red crosshair in all 2D slice views. Each orthogonal slice (axial, sagittal and coronal) is shown so that their slice runs through the instrument navigation point.

Visualized position at the navigated instrument:

Precision Pointer Electromagnetic	pointer tip
Registration Pointer Electromagnetic	center of the registration ball
Navigated Suction Tube Electromagnetic	the axis of the sensor tube extended to the tip of the suction tube

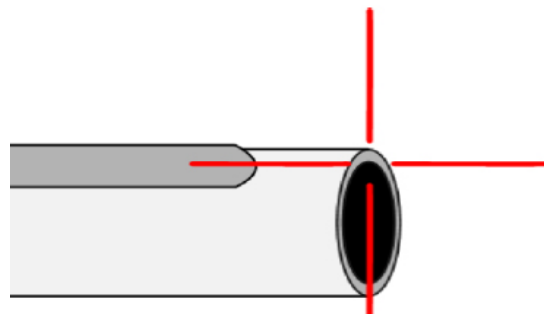If several different navigated instruments are detected by the tracking system simultaneously, the system shows the position of the instrument which is active in the operation area and most precise. Instruments which are navigated via the Instrument Adapter System have a lower priority than the specialized navigation tools and are hence navigated as secondtier. If the patient tracker and/or all navigated instruments are not detected by the tracking system, the status displays "Patient" and/or "Tool" will be shown in red. In this case, the navigation pauses and visualizes the last known and visible position of interest of the navigated instrument. The navigated position is visualized in the slice views using a cross-hair which is shown colored if the navigated information is valid. The cross-hairs are shown in gray if:

- The position of the patient tracker or of the tool tracker cannot be measured
- The patient image registration is not completed and confirmed.
- Disturbance of the tracking data of the patient tracker or of the tool tracker has been detected

8.16. Virtual Extension of the Tool Axis

During navigation, you can virtually extend the axis of the tool you are using by clicking the button in the top right corner of the view. This opens a slider which allows you to choose a distance. The displayed tool position is then moved by this amount along the tool axis. A visual guide originating from the actual instrument position should help you estimate how far away certain features in the image volume are from the current tool position (Figure 109).

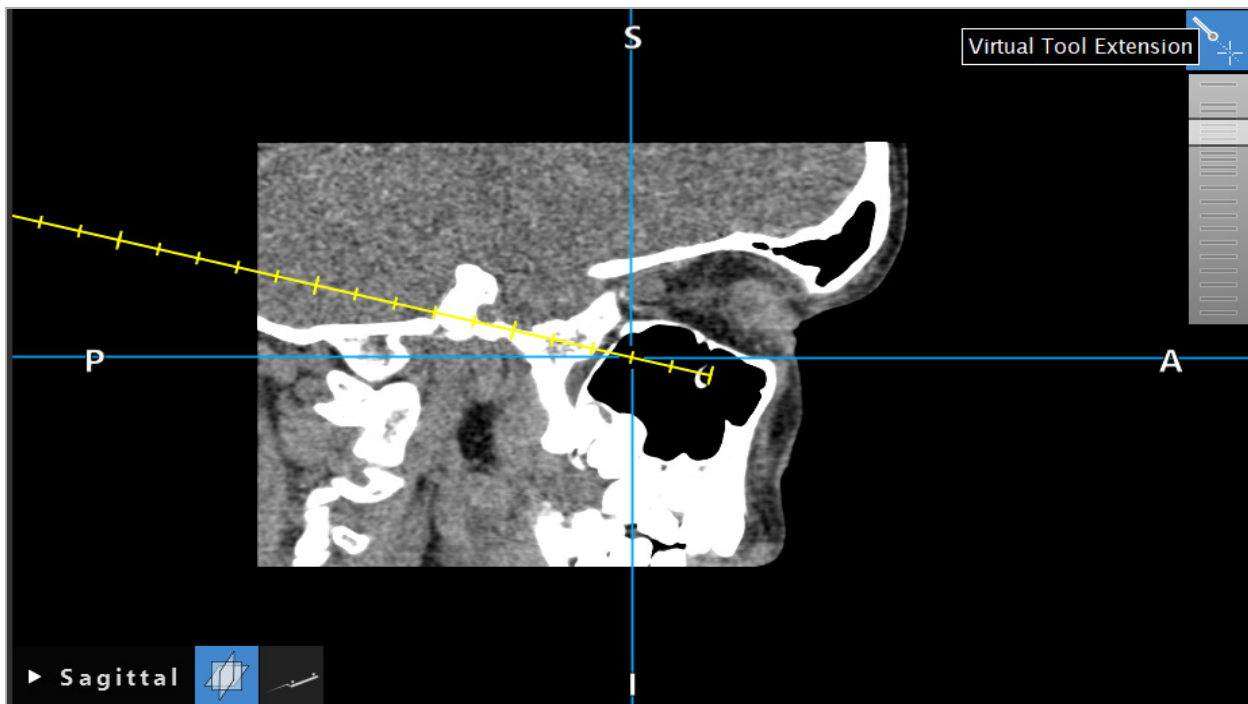
Figure 109 – Navigating with Extended Tool Axis

i Moving the displayed cross position in most cases also changes which slice in the volume is displayed. The tool axis is normally not completely located within this slice. Only the cross position is guaranteed to be part of the displayed slice view.

8.17. Augmented Reality

The navigation software supports the technology of augmented reality (AR), which enables overlay planning objects into the endoscopic camera image in correct position and in real time. However, the overlay of AR information is only possible if the following requirements are met:

- The image modality used (e.g. endoscope) is connected with a navigation sensor (tracker) and calibrated.
- The camera processor of the imaging system is connected to the navigation unit.
- Planning objects to be visualized are created.
- A Patient Tracker is attached to the patient, and an image-to-patient registration is conducted.

WARNING

- The display of AR information depends on the quality of the patient registration and the calibration of the imaging system. The accuracy may change due to rotation or bending of the endoscope, adjusting of the focus or zoom settings, or rotation or moving of the instrument adapter system while using the endoscope.

- Intra-operative live images processed by the system may be displayed with a delay. During applications that require realtime-critical imaging, a second monitor must be used. This must be directly connected to the realtime-critical image source.

8.17.1. Distance Orientation Guide

If augmented reality is active, the software shows the distance between the navigation point of the active instrument and the selected planning object.

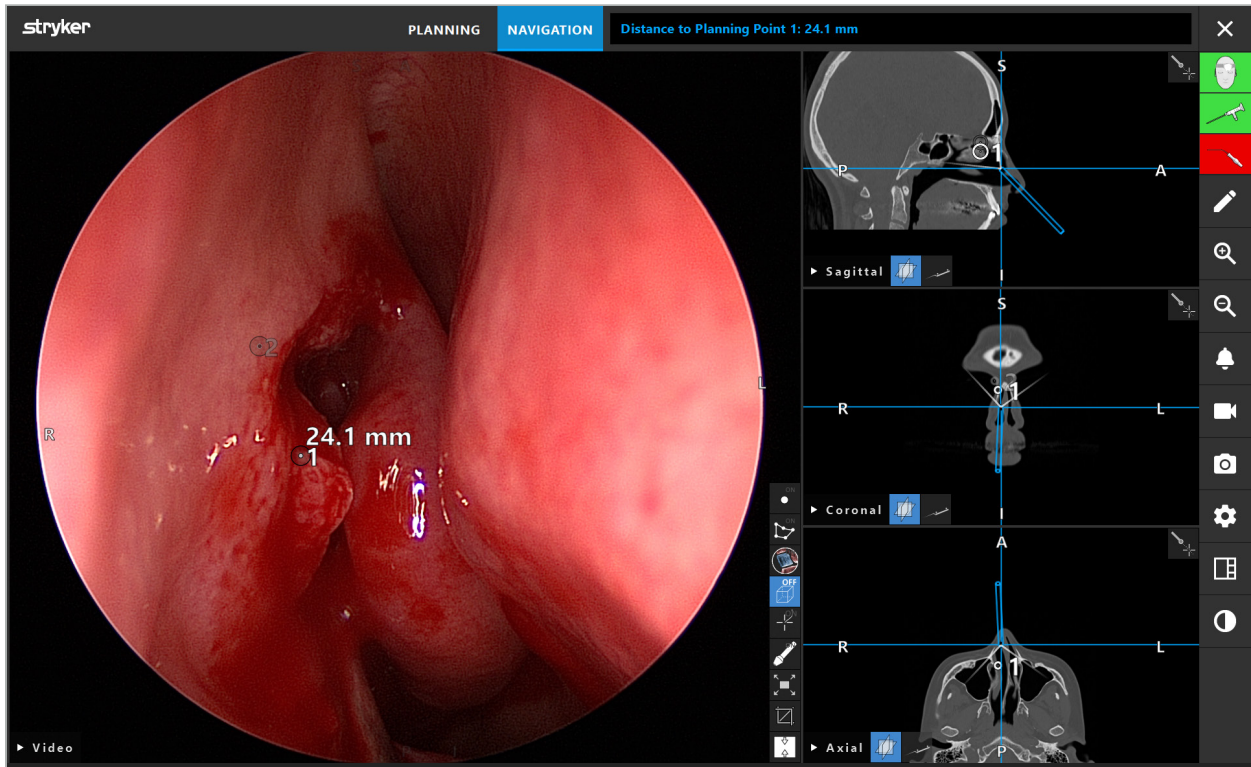
Figure 110 – Distance Orientation Guide

Point the endoscope toward a planning point to select one. The point that is closest to the tip of the endoscope in the endoscope image is selected.

In Figure 110, only planning point 7 is visible; hence this point is selected. The selected point is highlighted in the slice image views and is marked with a white middle point in the endoscope image.

In the case of planning structures, a green cross marks the position on the surface of the structure that is closest to the tool.

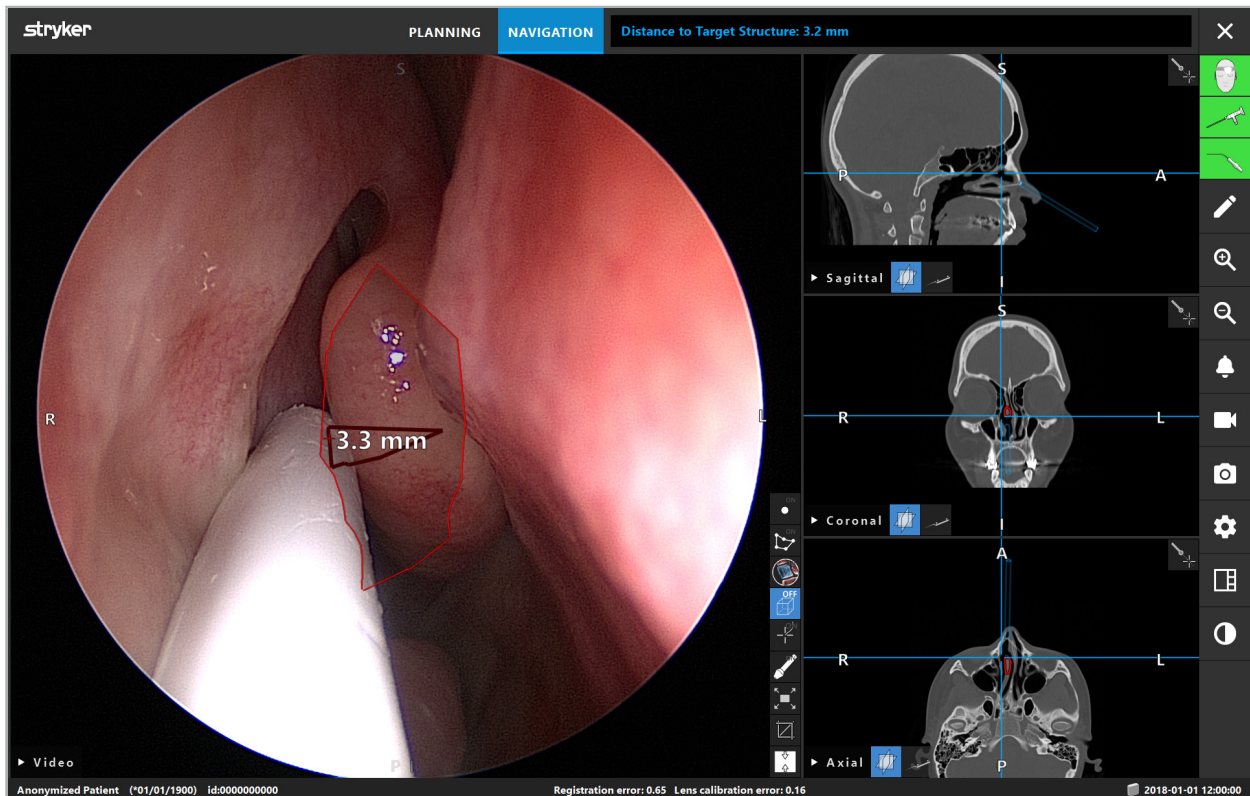
Figure 111 – Distance to the Planning Structure

If the cutting line is the chosen planning object, then a virtual extension of the instrument axis is created and the following values are displayed:

- Depth: Distance between navigation point of instrument and the point on virtual extension closest to the cutting line.
- Deviation: Distance between point on virtual extension closest to planning object and the planning object. This is the distance by which the cutting line would be missed.

WARNING

Do not rely only on the distance display to make a cut along the cutting line. Also use visual and tactile information.

8.17.2. Navigating with Target Guided Surgery (TGS)

A planned pathway is shown in the video image view as a series of concentric circles with a distance of 5 mm around a middle pathway (Figure 112). Similar to this, a trajectory is shown without the middle path (Figure 112). Due to its construction along a straight line, no middle pathway is needed for trajectories.

If you move a tool along this path or trajectory, the circles behind the tool tip fade out. You can then see how much of the path is already covered.

A pink circle visualizes the tool tip position along the path. However, if the tip is too far away from the path, this circle is shown as a segment in the direction of the tool.

Furthermore, the distance between tool tip and path end is shown. If the instrument tip is in front of the entry point of the path, the distance is shown to this entry point.

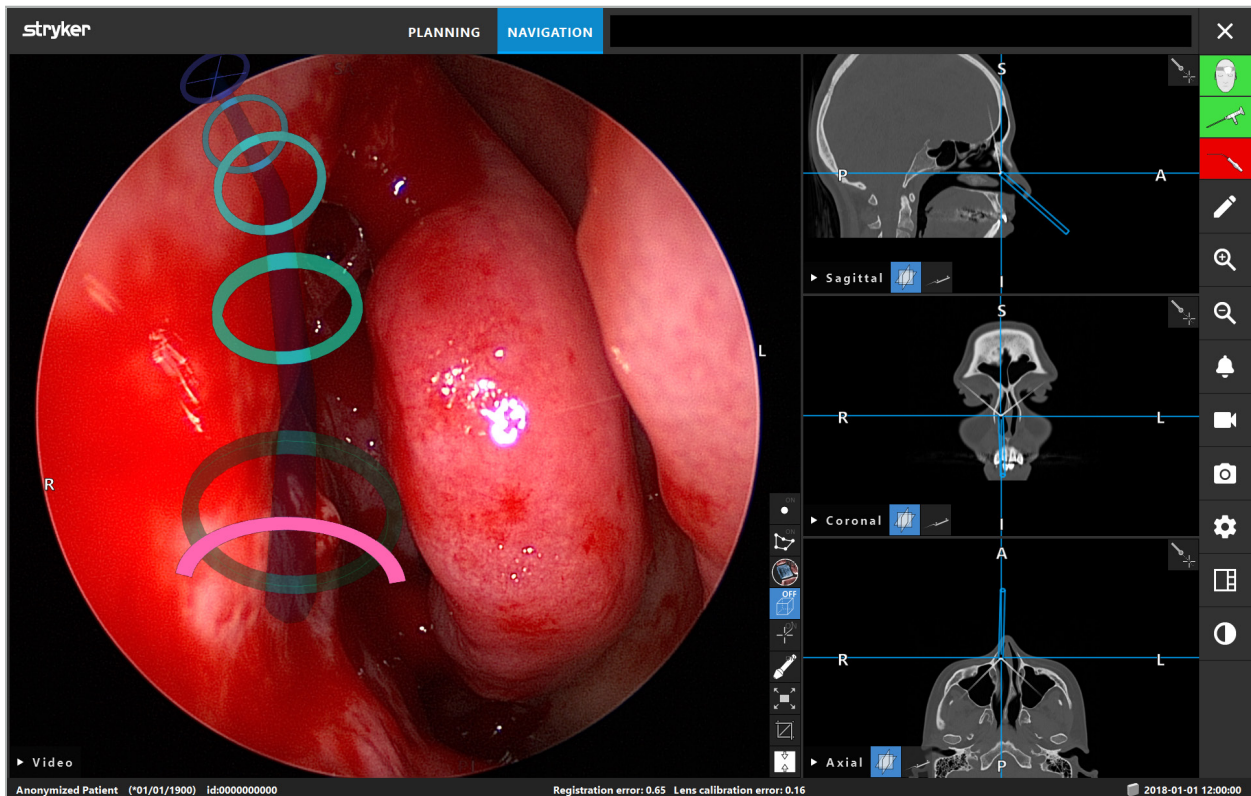
Figure 112 – Augmented Reality View of Drainage Pathway

WARNING

- The distance display can only be used for orientation; it does not have a measurement function. Accordingly, the system may not be used for measuring.
- The position of the blue circle is potentially faulty and may only be used as a rough indication of the tool tip position; it cannot replace visual controlling or checking the position in the slice image views.

8.18. Closing the Navigation

8.18.1. Closing the Navigation Mode

To end the navigation mode, do either one of the following steps (Figure 113):

1. Press the PLANNING tab to switch to planning mode.
2. Press the End Software button to end the navigation software.

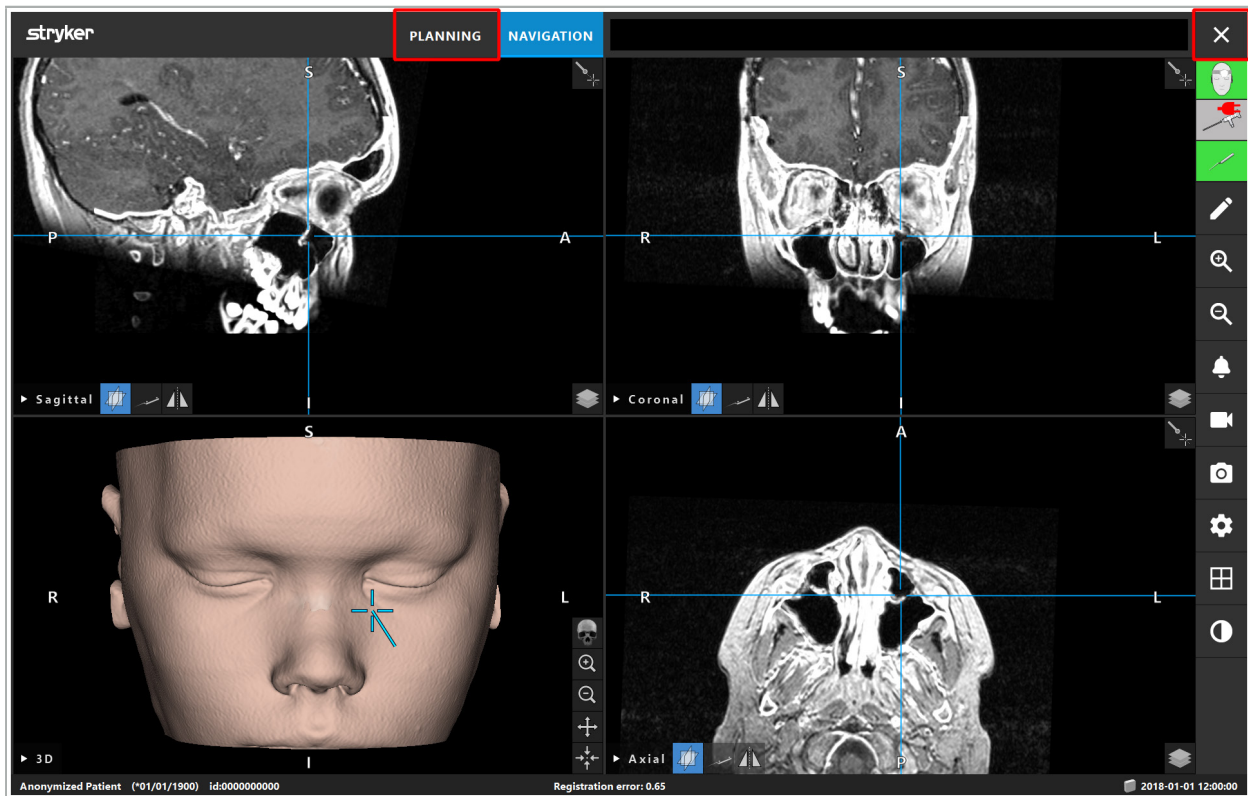
Figure 113 – Closing the Navigation Mode or Software

8.18.2. Closing the Navigation Software

1. To close the navigation software, press the End Software button.
2. If planning data changed since the last automatic saving, the Save Planning dialog appears, giving you the option to save the planning data before closing the software.

8.18.3. Switching off the Navigation System

1. To safely end the operation of the navigation system, close all opened windows and shut down the computer by clicking the Windows button and then selecting Shut Down in the submenu that appears.
2. After the computer is shut down, turn off the navigation unit with the On/Off button. After the navigation unit is turned off, the green LED on the On/Off-button switches off.

CAUTION

For full disconnection from the electricity grid, the power plugs must be pulled out of the socket or the power supply removed in some other suitable way.

i After using the electromagnetic system, make sure that instruments that cannot be sterilized again are disposed of.

Regarding additional post-operative steps such as disassembly, reprocessing, and storage of the individual instruments and devices, see the respective manuals.

9. Recording an Operation without Patient Image Data

The navigation software provides the option to record screenshots and screen recordings of the endoscopic video feed during an operation, even if volumetric image data of the patient does not exist or for some reason cannot be used. Planning and navigation features are unavailable, but it is still possible to display and record the endoscopic video image.

To use the navigation software without patient data, select New Record from the Data Import dialog. See Chapter “Loading Patient Image Data.” You are asked to enter some general information that is associated with the record, like name and date of birth of the patient. After filling out the fields, press Create to use the record.

You cannot enter the navigation mode, but you can take screenshots and recordings as usual. These are associated with the record that can be found under Plannings in the Data Import dialog.

10. Maintenance

WARNING

The device must only be used if it is checked and maintained regularly according to the respective legal requirements and directives.

CAUTION

- If operating or adjusting devices other than those indicated in this manual are used, or other operation procedures are used, this can lead to dangerous radiation exposure.
 - Stryker holds no liability for the equipment's operational safety if the product is opened, repaired, or modified without authorization.
-

The user is required to maintain a medical equipment logbook including information about conducted repairs and maintenance work, as well as types of work done, the company that has conducted the repairs, and other relevant information.

Refer to the REF and serial numbers when making enquiries or ordering spare parts.

Even though the operator does not conduct maintenance work himself, he is still responsible for ensuring that all required maintenance work is conducted before the product is used on the patient.

11. Transport and Disposal

11.1. Transport

WARNING

- Before transporting the components of the navigation system, it is necessary to process them in such a way that contamination is ruled out. To achieve this, all components must be reprocessed (cleaning/disinfection/sterilization) according to the instructions provided.
 - The instruments may only be sent back to the manufacturer in a cleaned, disinfected, and sterilized condition. Sharp or pointed components need to be sent back in a protected state.
-

CAUTION

Prior to sending the device for maintenance, remove the data drive so that any private patient data is not shared with Stryker.

For transport advice, refer to the components' specific manuals.

11.2. Disposal

For disposal advice, refer to the specific manuals of the components.

12. Technical Specifications

For technical data of the specific products named in this manual, refer to the individual manuals for these products.

12.1. Environmental Conditions

Environmental Limitations	Operation
Temperature:	between 10 °C and 30 °C
Relative humidity:	30 % to 75 %
Atmospheric air pressure:	between 80 kPa and 106 kPa

12.2. Essential Performance Features

- The product detects and visualizes defined navigation positions with an accuracy of at least 2 mm in radiological images.
- The delay produced in processing the incoming real-time video image data must not exceed 250 ms.
- No undetected data corruption.
- No unnoticed freezing of the instrument display.
- No unnoticed freezing of the incoming real-time video image data.

12.3. Compatible Imaging Systems

The navigation software can import image data in DICOM 3.0 standard. This standard is supported by all of the marketrelevant manufacturer's devices.

The list of compatible systems includes at least devices by:

Philips	Tomoscan M/EG R4.0
Siemens	Somatom 4+, 4+ Volume Zoom
Marconi	Picker PQ 2000, PQ 5000
Morita	3D Accuitomo

Before first use of the navigation system in the clinical routine, check system compatibility by importing a test image.

Should problems occur while loading image datasets, contact the manufacturer.

13. Routine Checkups

Before each use, check that all necessary maintenance work is carried out in accordance with local policy and procedure. Before use, check the device functionality and inspect the product components for outer defects.

Before each use, check the instruments for:

- Correct assembly and function
- Cracks
- Surface damage
- Loose parts
- Other damage

Check the product feed cable and connecting cable routinely for damages. Order spare parts if necessary.

14. Lifetime of Electromagnetic Instruments

The electromagnetic instruments contain delicate electronics whose useful lifetime is limited due to reprocessing and sterilization. When plugging in an instrument, the number of uses is indicated as follows (Figure 115):

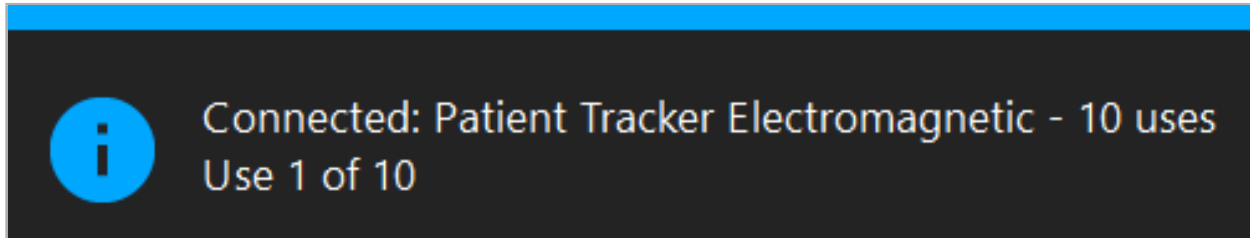
Figure 115 – Use Counter

The use counter automatically increases for the connected instrument when entering navigation mode. The counter increases only once during a surgery. Accordingly, it is possible to unplug and plug in again an instrument during a surgery. If the wrong patient image data are loaded, it is possible to change to the right one within 30 minutes after the beginning of navigation without increasing the use counter again.

Depending on the country-specific regulatory requirements, the number of uses can be restricted. The system automatically disables the instruments after reaching the maximum number of uses and displays the following warning (Figure 116):

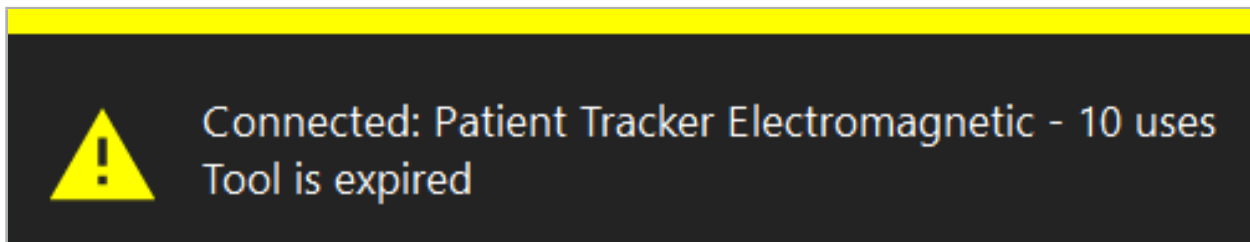
Figure 116 – Use Cycles Exceeded

If the number of uses is not restricted due to country-specific regulatory requirements, the following warning is displayed when the recommended number of uses is exceeded (Figure 117):

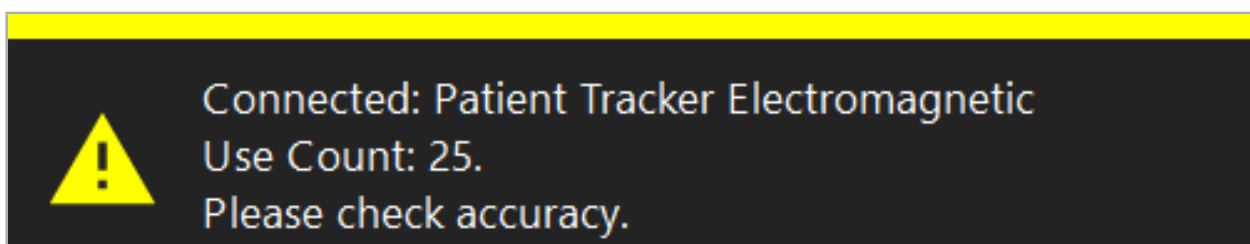
Figure 117 – Consider Number of Use Cycles

WARNING

- To maintain accurate use counting, always connect electromagnetic instruments at least once to the navigation unit during navigated surgery if they are taken from sterile packaging. If resterilizing instruments without connecting them to the navigation unit during surgery, the use counter may become inaccurate to indicate the actual number of reprocessing cycles experienced by the instruments. Inaccurate use counting may lead users to use instruments which have exceeded their useful life.
 - Always keep replacement electromagnetic instruments at hand and reorder in time based on expected surgery schedule.
-

The user carries full responsibility for the use of instruments that have exceeded the recommended number of use cycles.

15. Special Application Instructions for Instruments

As long as other materials are not specified explicitly, the instruments delivered by Stryker or bought additionally are made out of stainless steel or a titanium alloy.

In order to avoid corrosion, instruments made out of stainless steel may not be stored on base metal surfaces (chromed surfaces) or close to chemicals, as the transmission of flash rust cannot be totally excluded. Due to their composition, the instruments are not suited to be used together with other magnetically sensitive medicine products (MRI), devices, or instruments.

The instruments guide electricity and must not be placed between the patient and a respective power source in order to avoid electrical shocks.

The products are sensitive to pushing, throwing, dropping, use of steel brushes and abrasives, as well as heavy exertions of force. It is therefore absolutely necessary to treat the products with care.

16. Minimum Requirements for Additional Components

16.1. Devices

Endoscope camera unit:

- Outputs: DVI / SDI / HD-SDI / 3G-SDI / S-VIDEO / VGA
- Resolution: minimum PAL 768 x 572 - maximum 1080p60
- Endoscopic image delay in combination with 60 Hz medical monitor: maximum 50 ms
- Camera unit according to IEC 60601-1 and provides 2 MOPP to the camera
- Lightsource according to IEC 60601-1 and provides 2 MOPP towards the patient

Camera head:

- Resolution: PAL (752(H) x 582(V)) or higher

Endoscopes:

- Diameter: 4 mm
- Visual angle: 0° / 30° / 45°
- Default bayonet connector

Medical monitors:

- Inputs: DVI or HDMI
- Resolution: 1280 x 1024 or higher, 1920 x 1080 recommended
- Colors: 16.7 million colors
- Maximum delay of 50 ms in combination with an endoscope camera unit
- Medical grade
- Fulfills IEC 60950-1 or IEC 60601-1

The device is compatible with the following monitors:

- 240-031-020 VisionPro
- 240-031-050 4K Display

Keyboard/mouse (medical or hygiene grade):

- Connector: USB 2.0 or higher

- IBM Compatible

The device is compatible with the following:

- 8000-030-010 Medical keyboard U.S./international
- 8000-030-020 Mouse, wired
- 8000-030-021 Mouse, wireless
- 8000-030-011 Medical keyboard GER
- 8000-030-012 Medical keyboard UK
- 8000-030-013 Medical keyboard ES
- 8000-030-014 Medical keyboard FR
- 8000-030-015 Medical keyboard Nordic

The device is compatible with the following cart:

- 8000-030-002/KU.2763.903 Pro Equipment Cart

CAUTION

When moving the cart with its components:

- The monitor must be turned 90° degrees sideways.
- The cart with all components may only be slowly pulled by the handle in a backwards direction. When being pulled over any threshold—including cables or hoses—the speed of movement must be reduced significantly.

For further information, please review the relevant user instructions of the cart and its components.

 Should you have questions regarding compatible devices and products, contact Stryker.

16.2. Disposables

Accessory	REF
Patient Tracker Tabs	8000-100-001

17. Keyboard Shortcuts for Navigation Software

The navigation software allows the use of keyboard combinations (shortcuts) for easy handling of the navigation software.

Shortcut	Function
Ctrl+D	Open configuration dialog
Ctrl+E	Open installation folder in data manager
Ctrl+L	Open log file in data manager
Ctrl+Shift+L	Open recent log file
F2	Screenshot
F3	Start/stop video image data record
F10	Open system compatibility dialog

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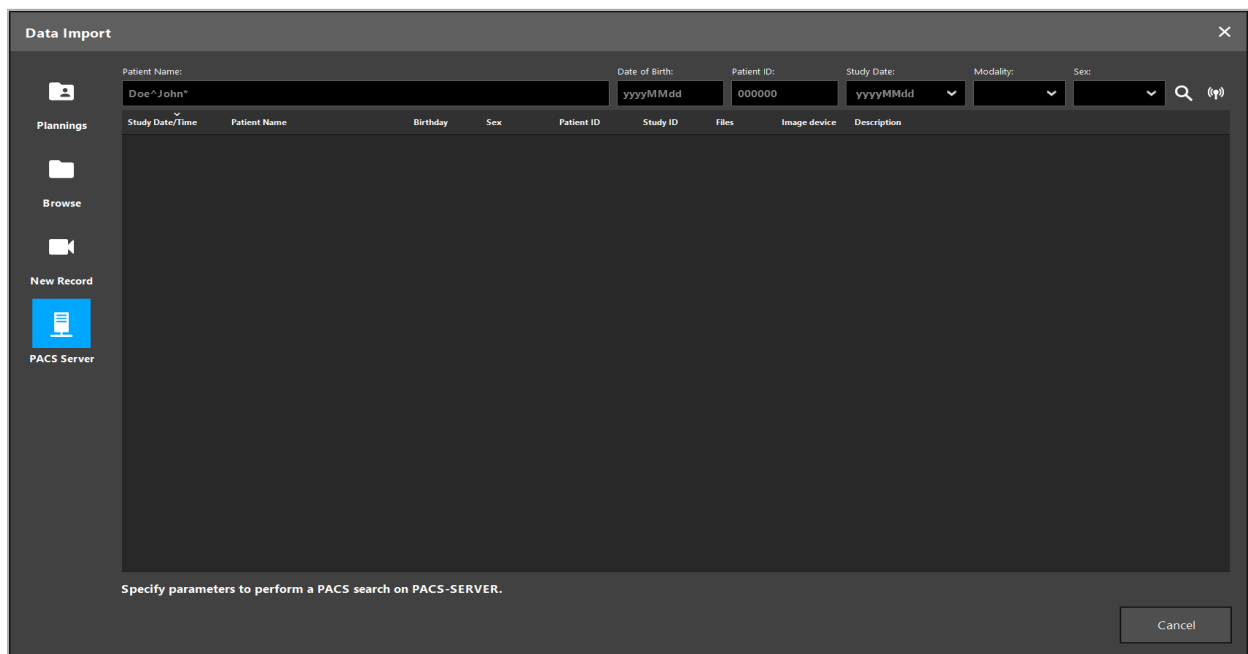
The screenshot shows a 'Data Import' dialog box. At the top, there are input fields for 'Patient Name' (Doe^John*), 'Date of Birth' (yyyyMMdd), 'Patient ID' (000000), 'Study Date' (yyyyMMdd), 'Modality', and 'Sex'. Below these is a table with columns: Study Date/Time, Patient Name, Birthday, Sex, Patient ID, Study ID, Files, Image device, and Description. The table is currently empty. On the left side of the dialog, there are icons for 'Plannings', 'Browse', 'New Record', and 'PACS Server'. At the bottom, there is a 'Cancel' button and a note: 'Specify parameters to perform a PACS search on PACS-SERVER.'

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Stryker Corporation or its divisions or other affiliated entities own, use or have applied for the following trademarks or service marks: Leibinger, Scopis, Stryker, TGS. All other trademarks are trademarks of their respective owners or holders.
Copyright © 2019-20 Stryker
Stryker Leibinger GmbH & Co. KG