


Triathlon[®] Total Knee System

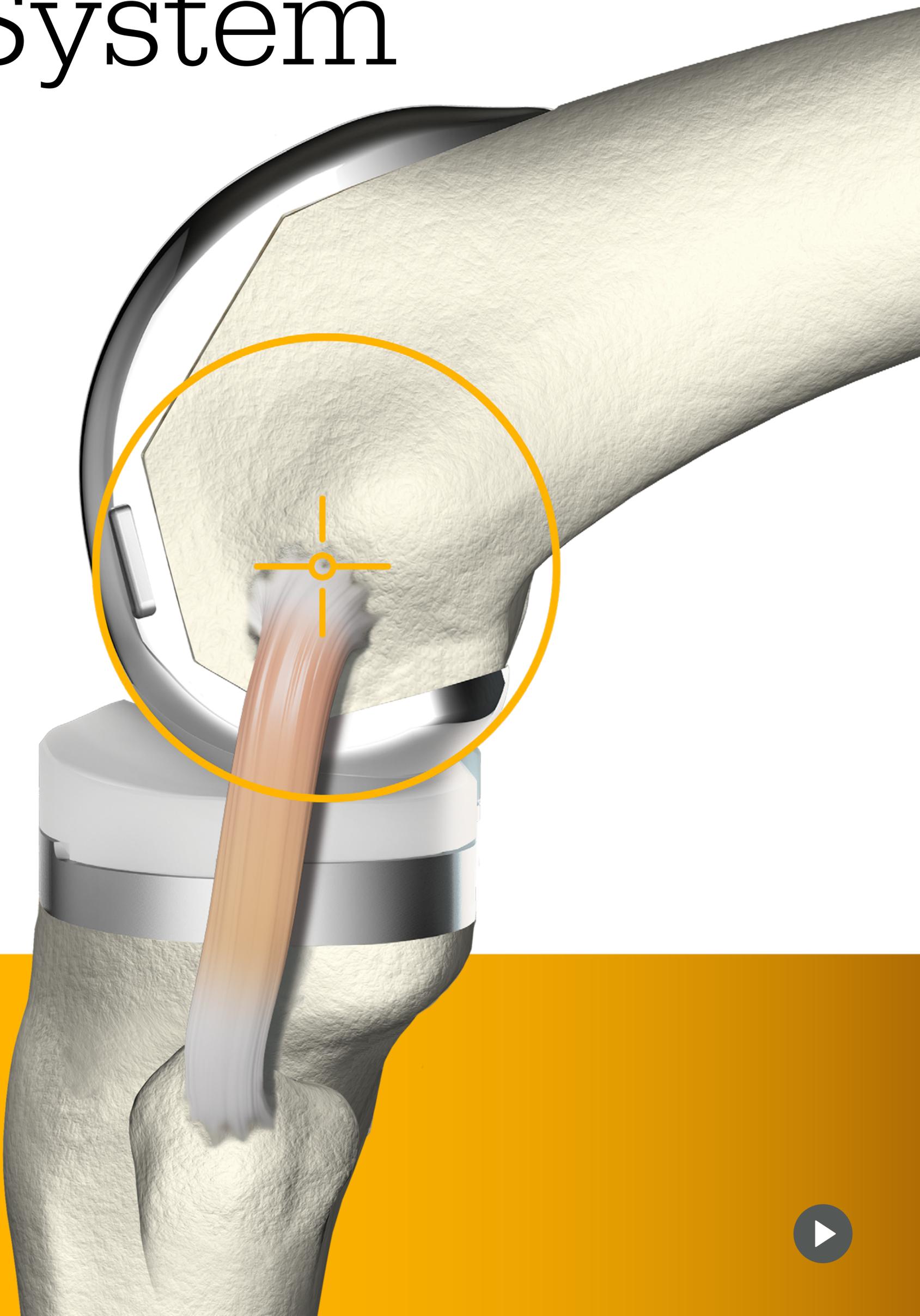

Triathlon® Total Knee System

design rationale

INTRODUCTION

Why	1
------------------	----------

DESIGN

Single radius	3
Key features	6
Options	10
Fit	12
FlexRod	14

TECHNOLOGY

X3

Process	18
Wear resistance	20
Strength	25
Oxidation resistance	28
Registry results	33

EVIDENCE

Stability

Gait	35
Pivot	36
Deep flexion	37
Hyperextension/extension	38

Satisfaction

Pain scores	39
Recovery	40

Survivorship

Studies and registries	41
-------------------------------------	-----------

COMPARISONS

DePuy Synthes Attune	42
DePuy Synthes Sigma	51
Smith & Nephew Journey 2	58
Smith & Nephew Gen II/Legion	62
Zimmer Biomet Vanguard	67
Zimmer Biomet NexGen	72
Zimmer Biomet Persona	79

REFERENCES AND DISCLAIMER	85
---------------------------------	-----------



Designed to work with the body

Triathlon and the single radius are designed to work with the patient's body. With over 15 years and over 3 million implantations worldwide, Triathlon has a long clinical history.^{1,80} Triathlon was designed to incorporate some of the most studied features of Stryker's prior designs that have been shown to address many of the main reasons for revision TKA such as instability, patellofemoral tracking complications and loosening/osteolysis.⁸⁰

The Triathlon Total Knee System offers surgeons a variety of options depending on disease, deformity and patient demand including cruciate-retaining, condylar-stabilizing, posterior-stabilizing and difficult primary options. Studies have consistently shown that Triathlon and the single radius provide:

Stability

Triathlon's single radius design allows for mid-flexion stability.^{26,45,82} A single vs. multi-radius functional outcome study concluded that Triathlon's advantages in stability, walking, stair climbing and knee straightening stem from the single radius design, particularly its longer moment arm and enhanced ligament stability based on maintained isometry throughout knee range of motion.²⁶ A fluoroscopic study on Triathlon patients showed the femoral component was kinematically stable in mid-flexion ranges.⁴⁵ Additionally, gait reviews comparing patients with a single or multi-radius knee implant showed that patients with a single radius knee implant experienced gait patterns that more closely mimicked that of the non-diseased control group.⁸²


Satisfaction

Triathlon continues to demonstrate excellent long-term results with high patient satisfaction rates. One study reported 98.9% satisfaction at three-year follow-up among patients who received Triathlon implants.¹³⁶ In a separate study, 90.5% satisfaction was reported at eight-year follow-up among patients who received Triathlon implants.¹³⁷



Survivorship


Multiple studies and joint registries^{5,80,89,111} from around the world consistently show a high rate of survivorship with Triathlon with more than 10 years of follow-up.

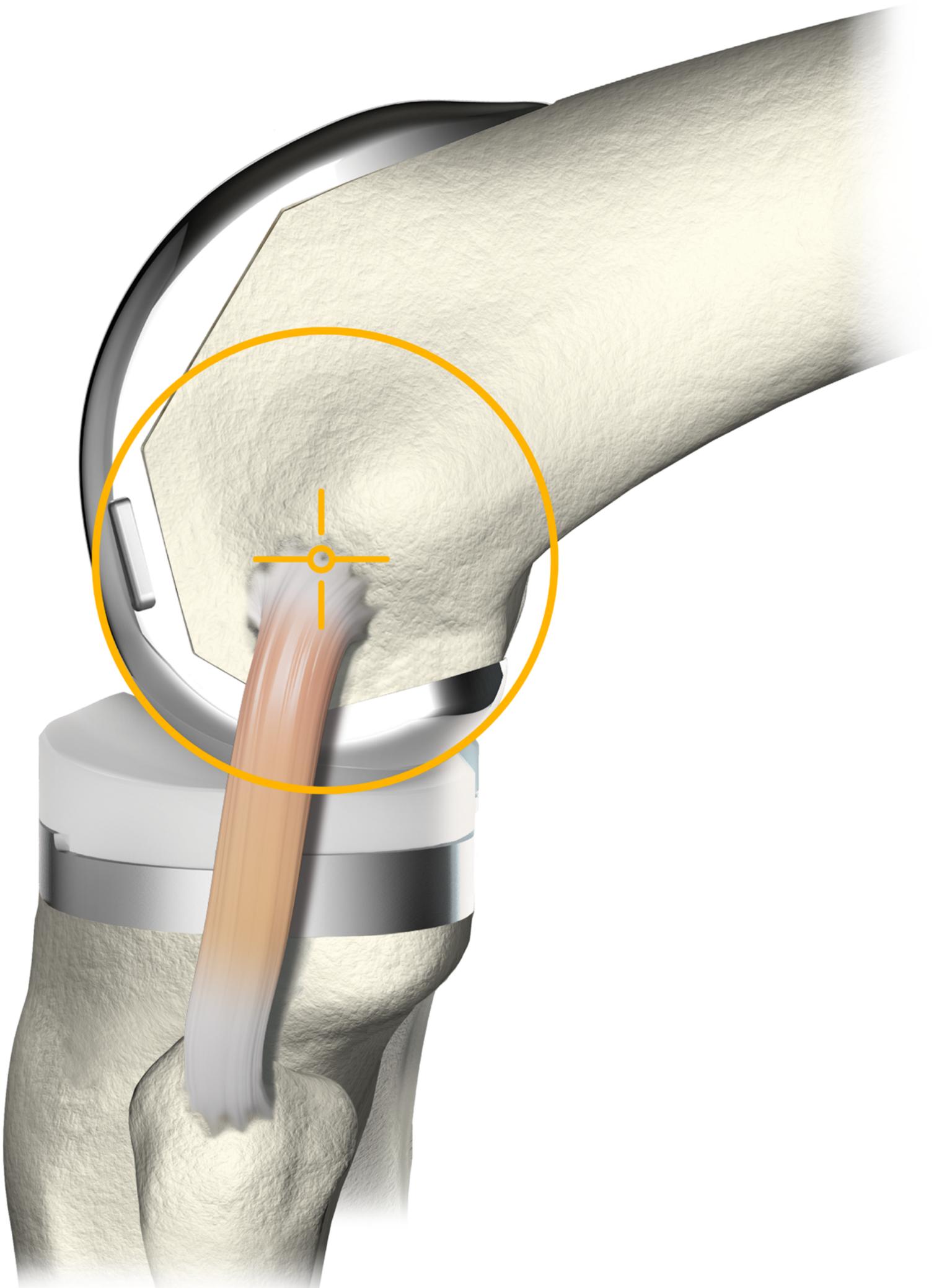




Single radius

The Triathlon single radius is designed to address stability during active flexion, where most motion occurs.^{2,26,82,107}

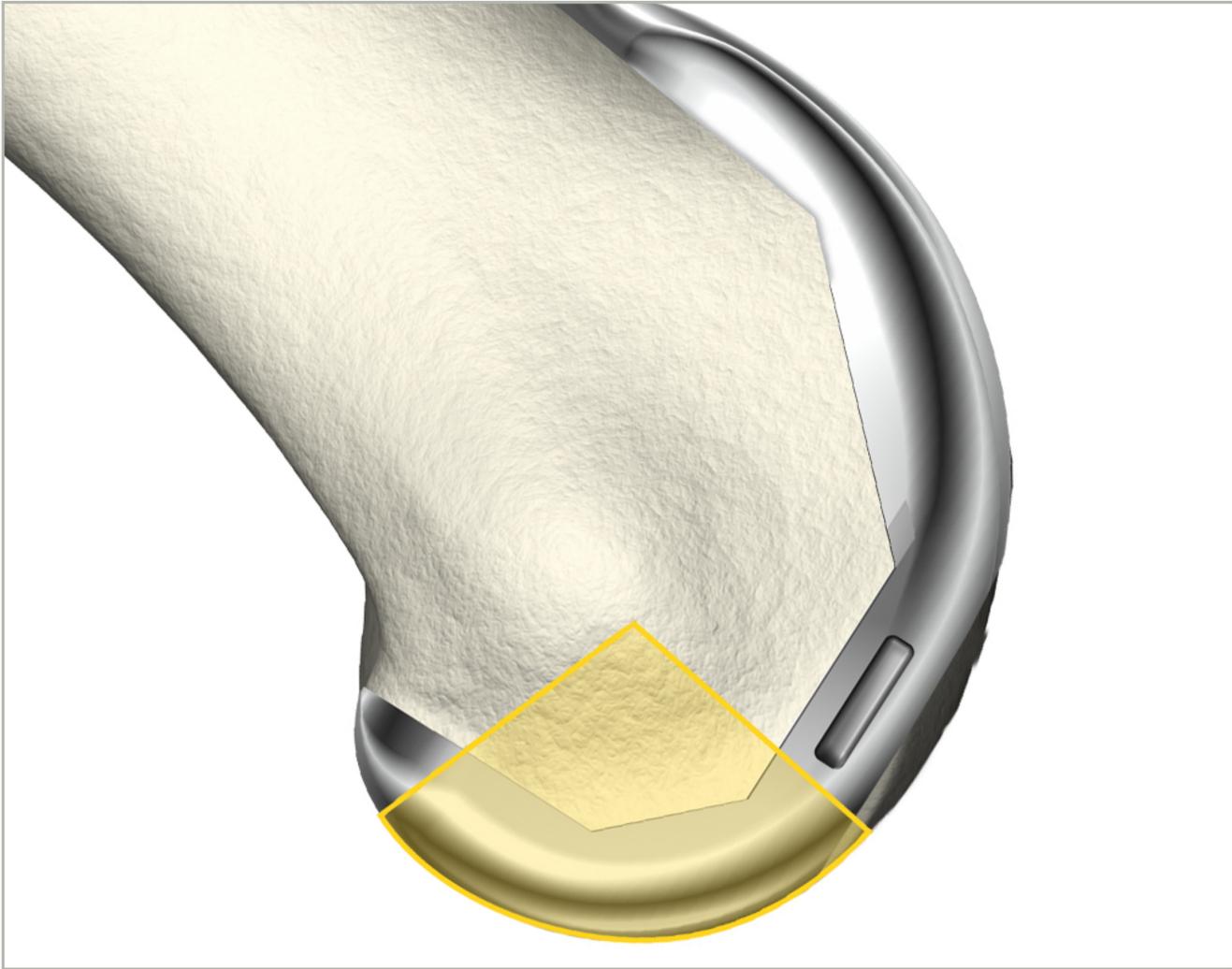

The single radius philosophy is based on observations from cadaver studies as well as our SOMA database.^{21,22, 53}

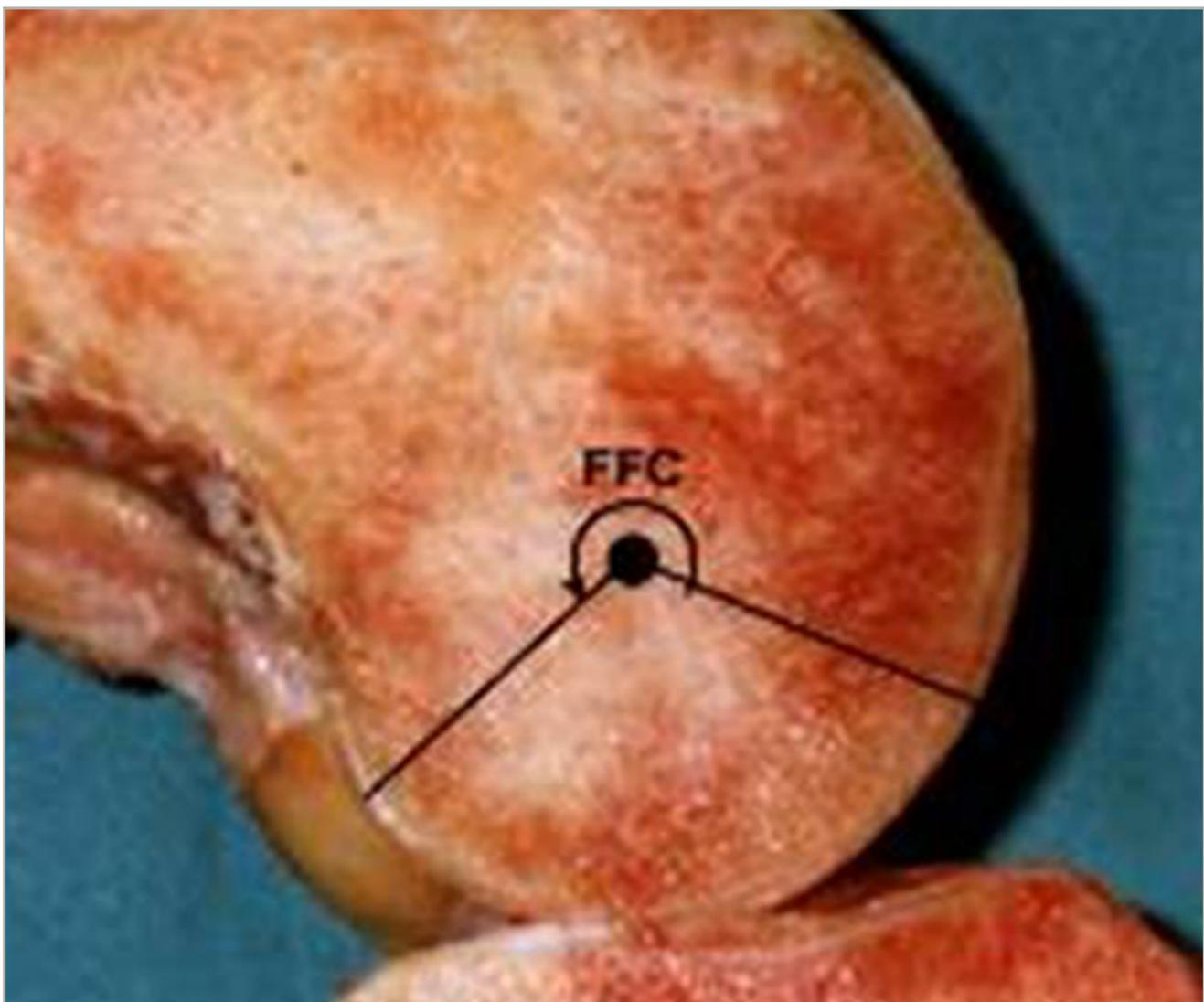




The single radius is designed to restore the knee's single center of rotation during active flexion.^{21,22} This allows for constant ligament tension and stability in flexion.^{2,26,82,107}




There are a number of structures and soft tissues present pre-TKA that are removed during TKA. Triathlon is designed to work with the soft tissues that remain. The key is the single radius.

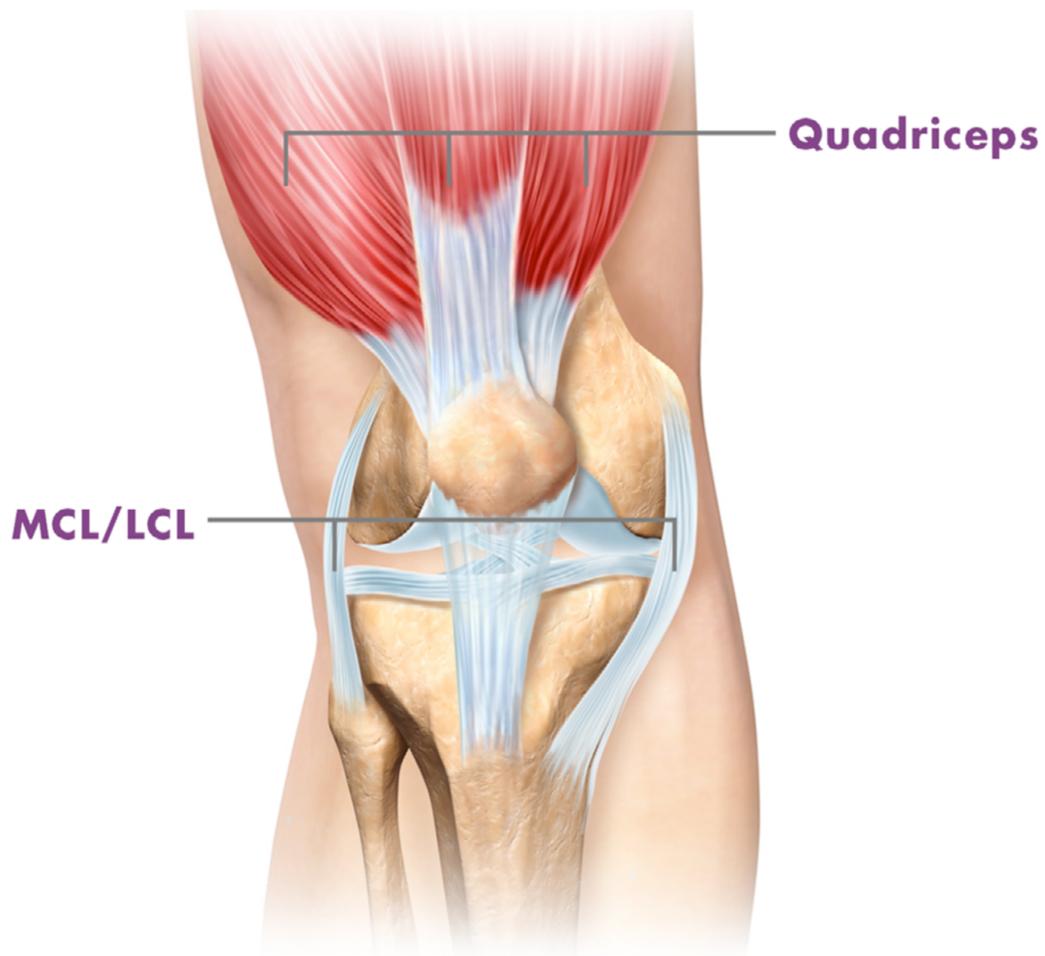



Quadriceps

The quadriceps help stabilize the knee during gait. The single radius allows for constant ligament balance¹⁻³ and less force required by the quadriceps.⁴

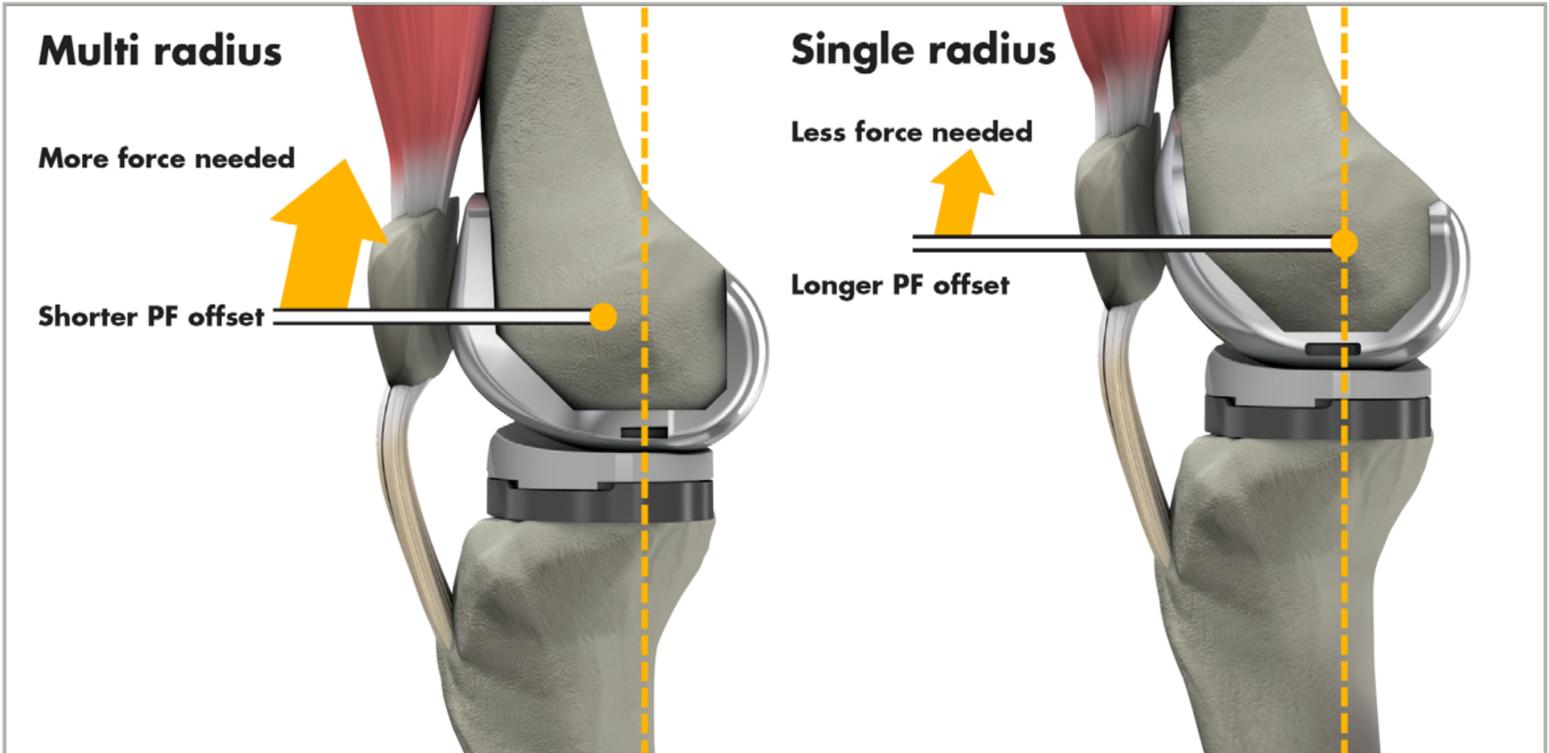

MCL/LCL


The single radius allows for constant ligament balance.¹⁻³





Key features


While the single radius works with the ligaments and muscles that remain,¹⁻⁴ the other key features of Triathlon are designed to work with the body to accommodate motion, not force it.

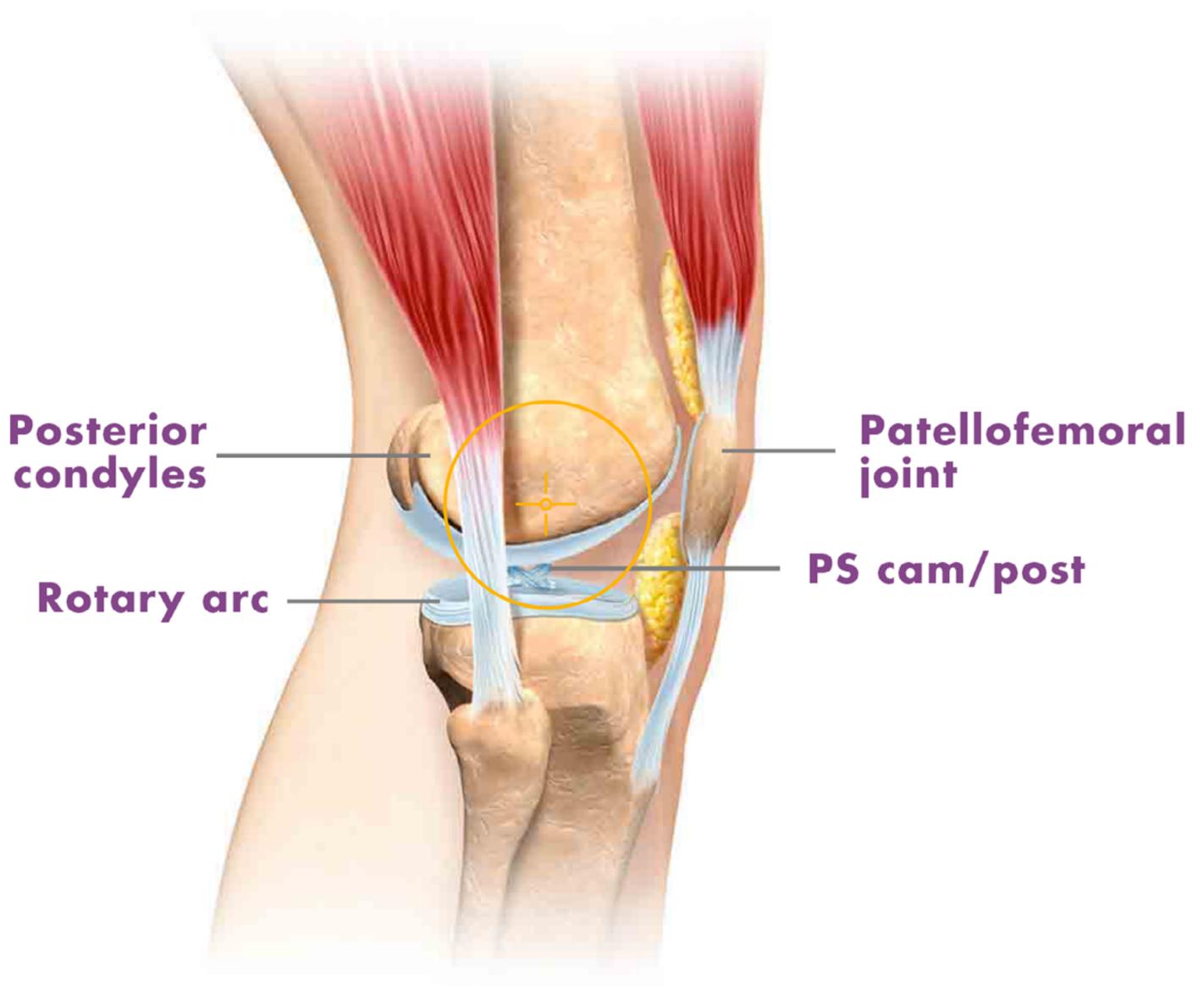



Posterior condyles


Triathlon's shortened, flared posterior condyles are designed to facilitate the relaxation of the soft tissues to enable deep flexion without excessive slope.⁶

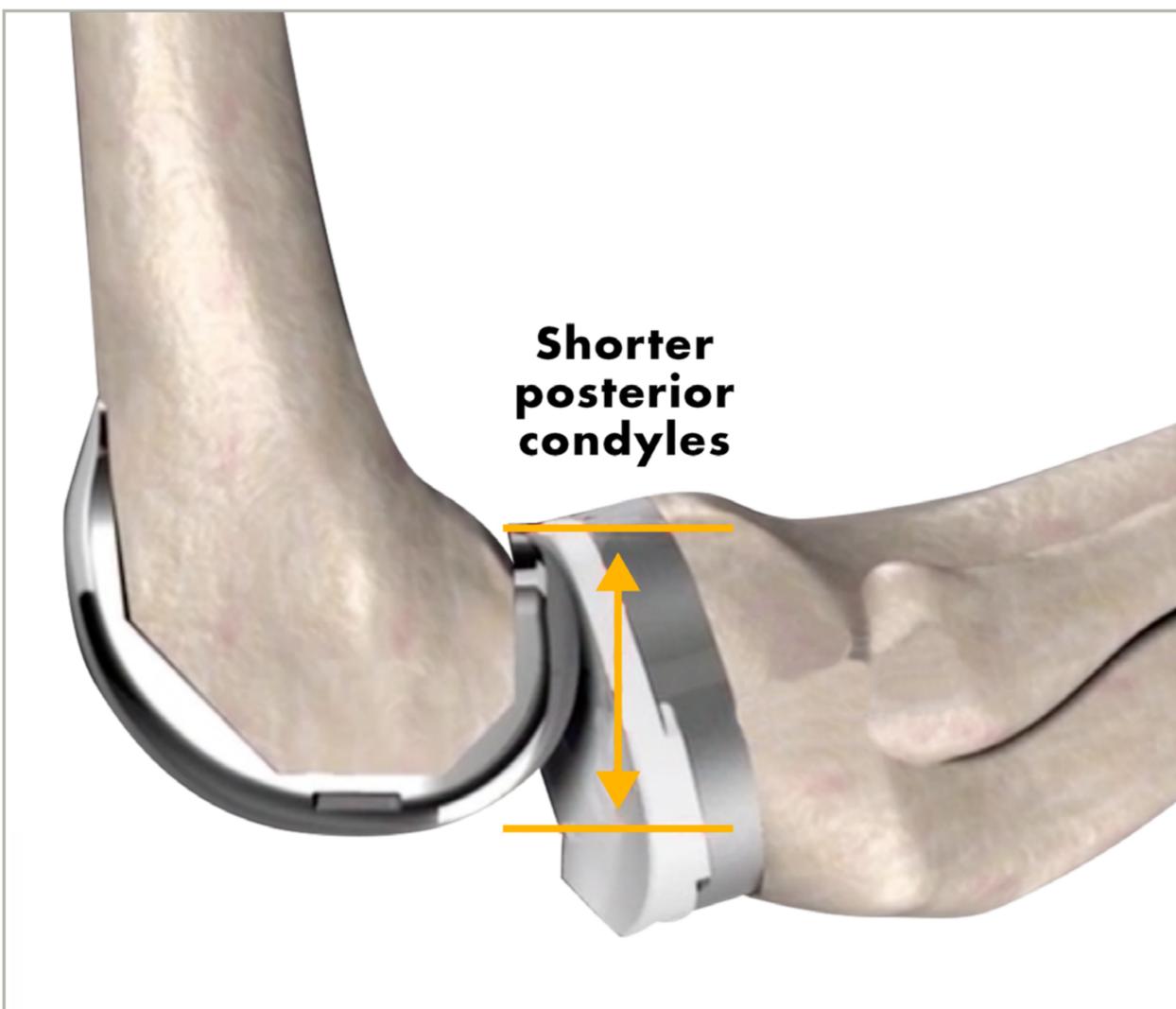




Rotary arc

The Triathlon rotary arc design allows for +/- 20° of internal/external rotation and reduces contact stresses.⁶ This may lessen the potential for wear and loosening.⁴⁴

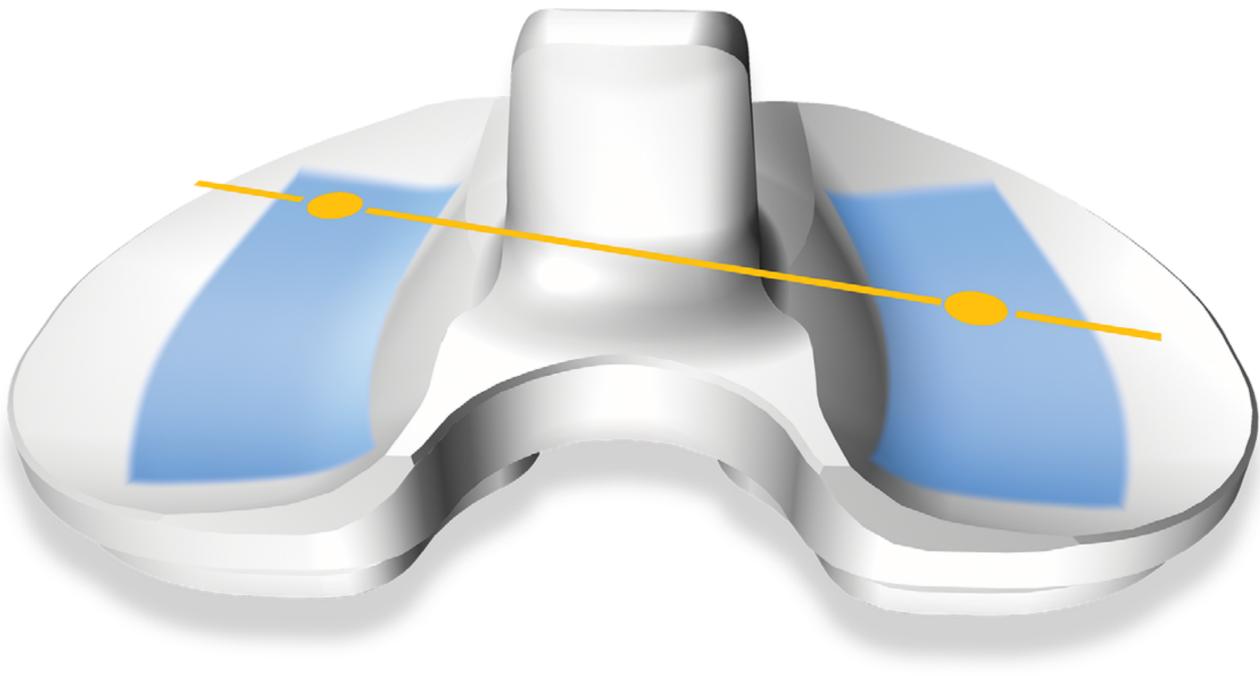

Patellofemoral joint


Triathlon's deepened trochlear groove is designed to help relax the extensor mechanism, enable deeper flexion and reduce contact stresses exerted across the patella.⁸ Triathlon incorporates the same patellofemoral design as Duracon, which demonstrated <1% patellofemoral complication rates in multiple studies.^{41,42}

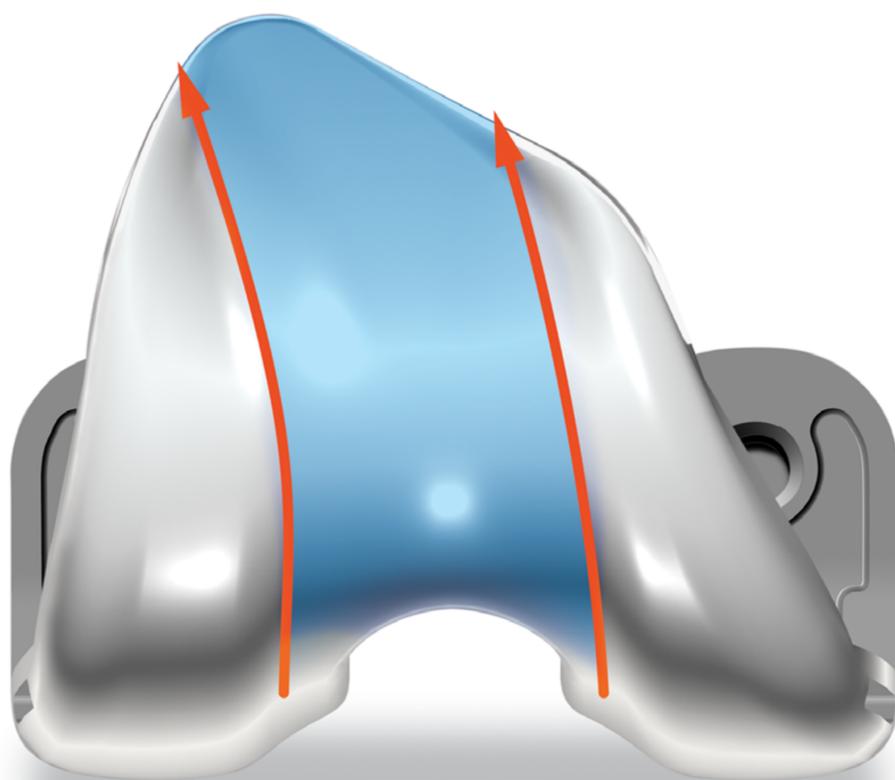



PS cam/post

Pre-TKA, the PCL engages at approximately 45° . Triathlon's cam and post are designed to engage at approximately 45° as well.⁴⁹ Competitive designs engage later in flexion^{60,98,104} and rely on the post to drive rollback. Triathlon PS is designed to provide stability throughout gait and allow for deep flexion.^{82,107}

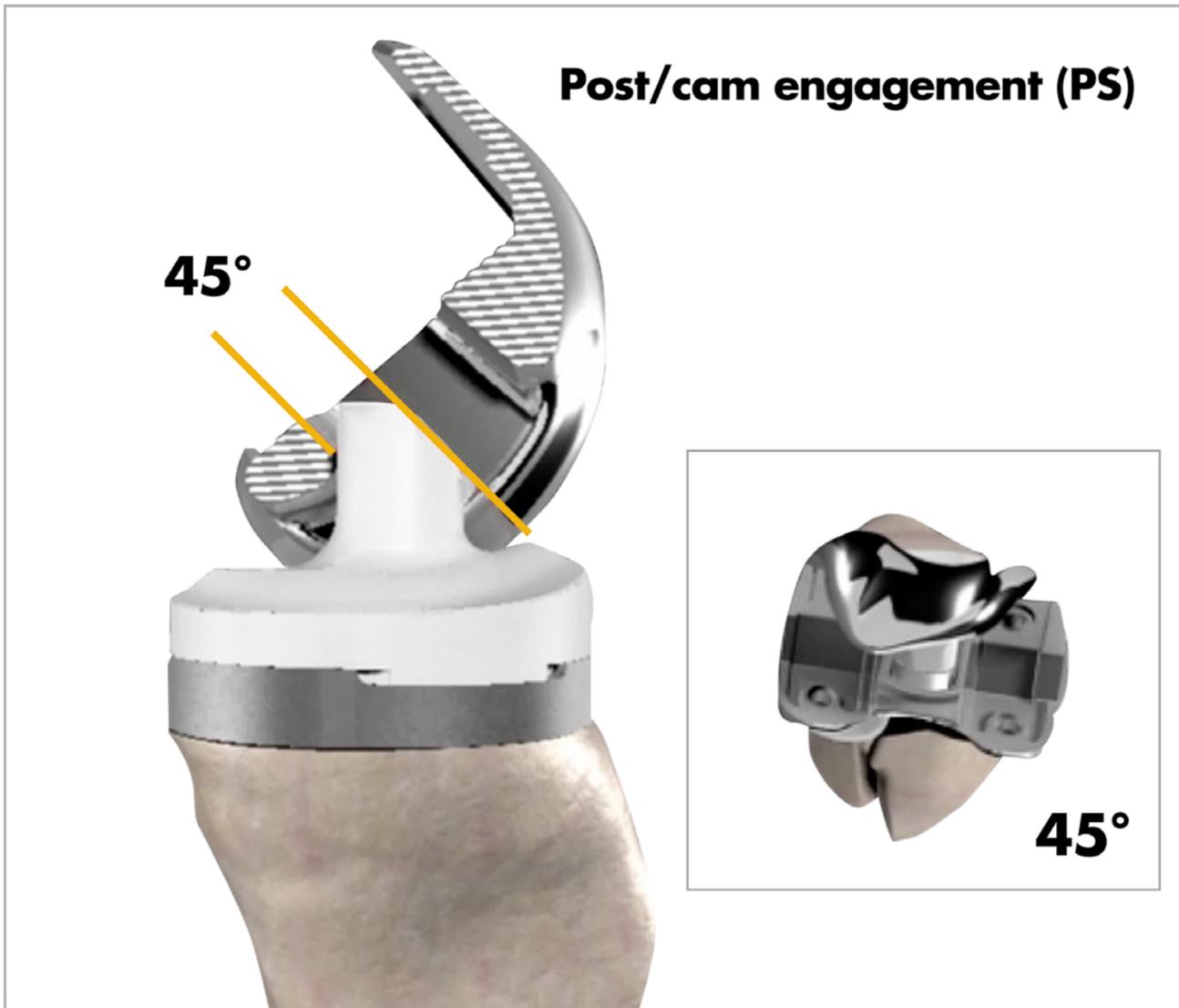



Options

The Triathlon Total Knee System offers surgeons a variety of options depending on disease, deformity and demand.



Triathlon® CR



Triathlon® CS



Triathlon® PS




Triathlon® PS Femur with TS insert

Option	Varus/valgus constraint	Internal/external rotation constraint	Maximum flexion
CR	None	+/- 20°	150°
CS	None	+/- 20°	150°
PS	None	+/- 20°	150°
TS	+/- 2°	+/- 7°	135°

Triathlon CS insert

The Triathlon CS insert offers, on average, 2mm more height on the anterior lip than the CR insert. A prospective randomized investigation of patients who received either a Triathlon CS or PS insert was conducted.⁸⁷ The PCL was sacrificed in all patients. There were no significant differences between groups for the Knee Society Scores, the Lower Extremity Activity Scale, range of motion or alignment (preoperative versus five-year postoperative). This supports that Triathlon CS can be a suitable option for patients with a PCL deficiency.

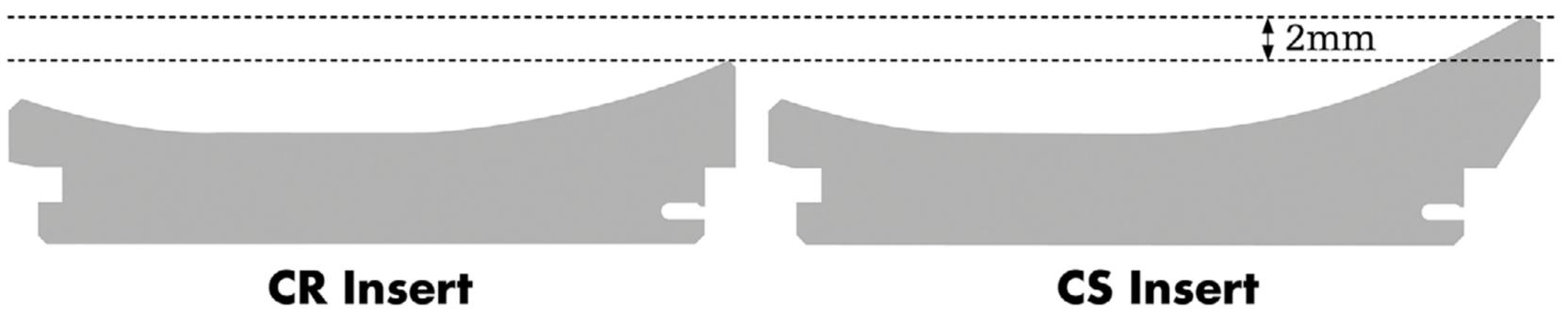




Fit – femur features

Triathlon's broad range of size offerings is based on an anthropometric measurement study¹⁰ for improved interplay between implant geometry and anatomic structure for women and men.

The Triathlon femoral component is designed to address smaller anatomies, often found in female patients, heavily concentrated in the region shown,¹⁰ while still accommodating larger male patients. The unique 7° anterior flange design of Triathlon is designed to provide the flexibility to downsize the femoral component while avoiding the occurrence of notching.

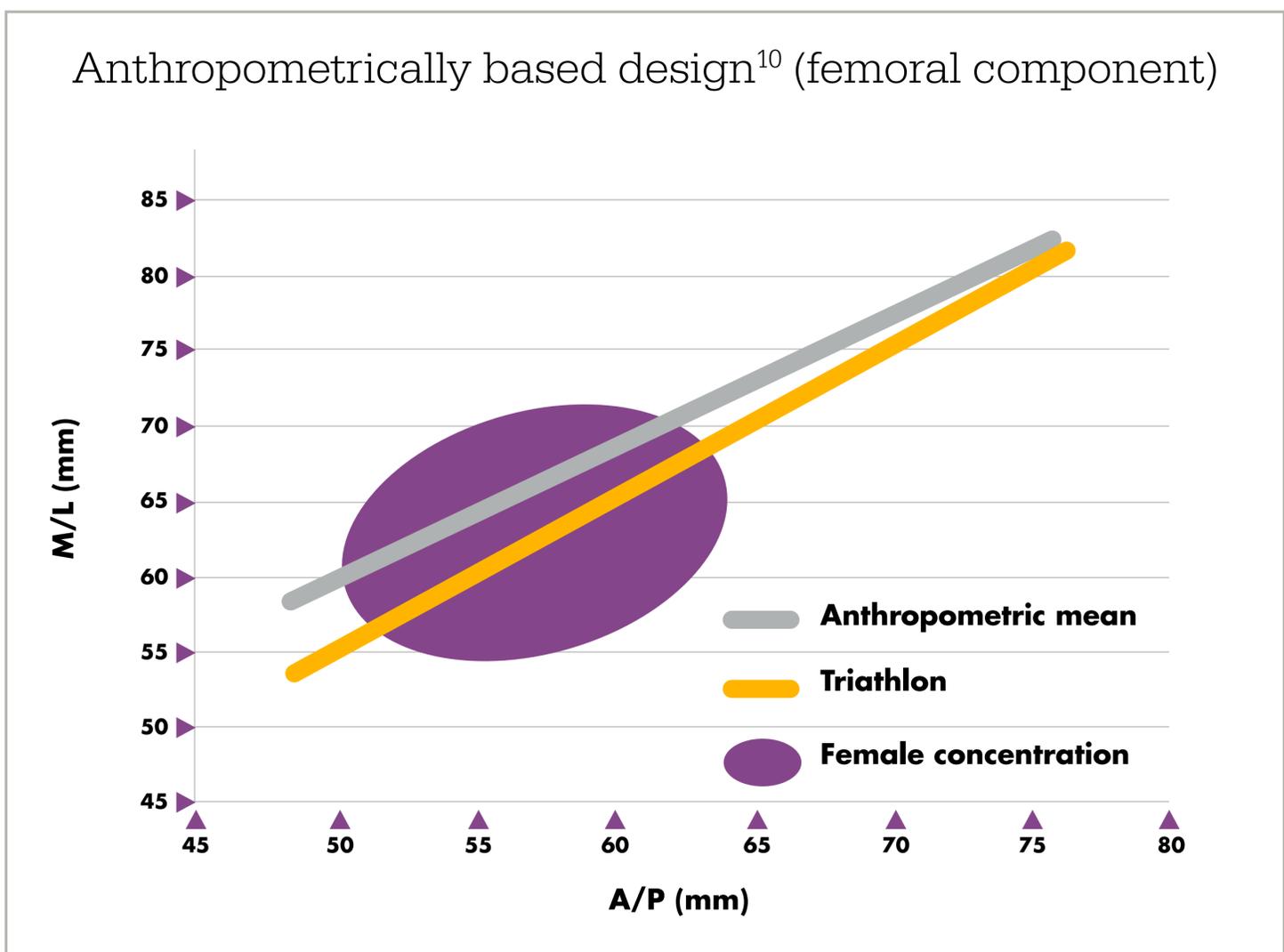



Anterior



Fit – tibia coverage

Symmetric baseplates have been shown to produce positive clinical results^{1,3,5,9,17,26,32} and adequate surface coverage in relation to tibial aspect ratio.¹³ In fact, a study by Incavo et al. demonstrated symmetric baseplates provided greater coverage in anterior quadrants when compared to asymmetric designs.¹³ Another study showed an asymmetric tray had higher rates of posterolateral and posteromedial overhang compared to symmetric designs, including Triathlon.⁶⁹


A clinical study of 281 patients who received a Triathlon implant demonstrated no incidences of tibial component subsidence.⁸⁰

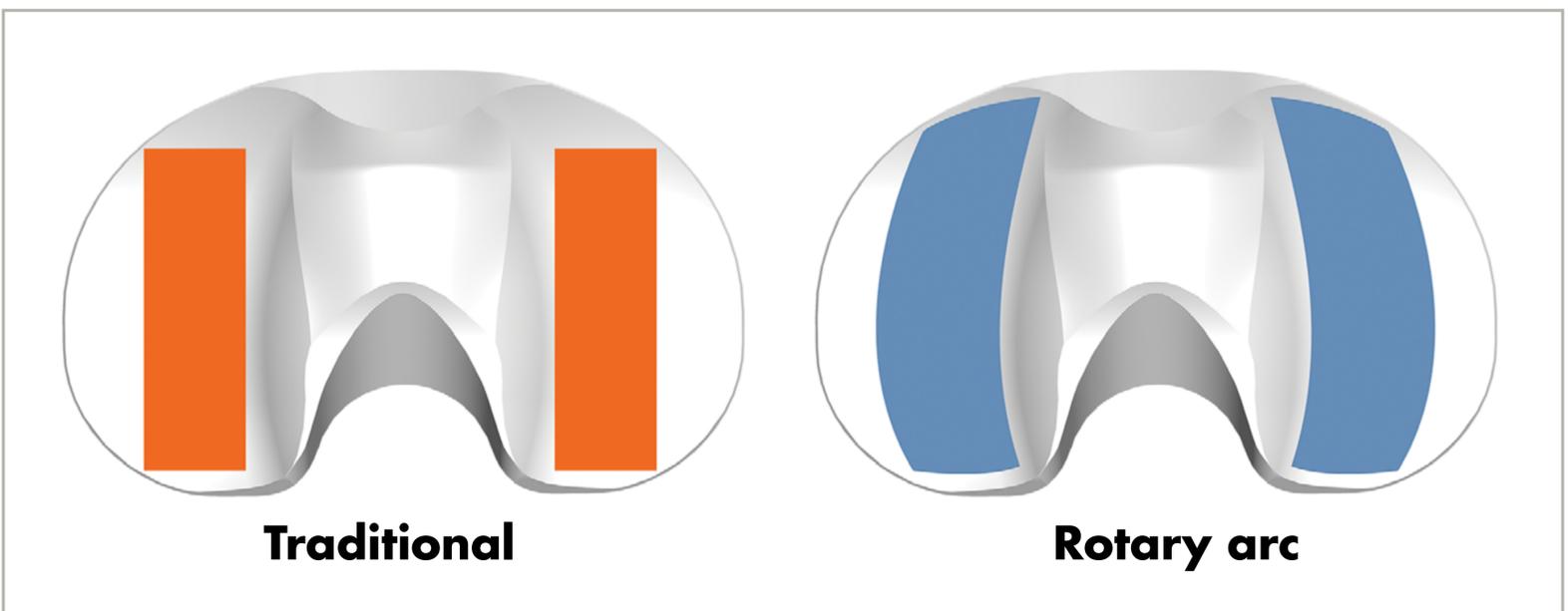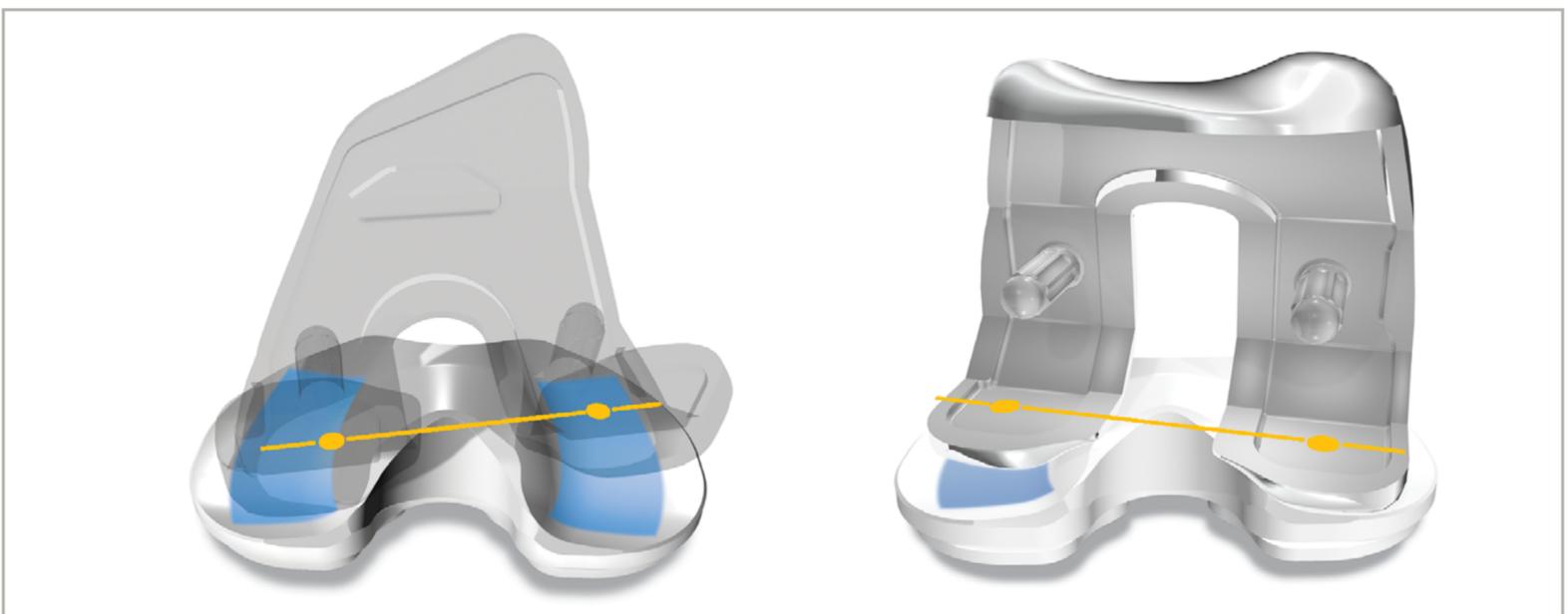




Orientation and fit with FlexRod

The FlexRod is designed to flex to accommodate the anterior bow of the patient's femur.¹²

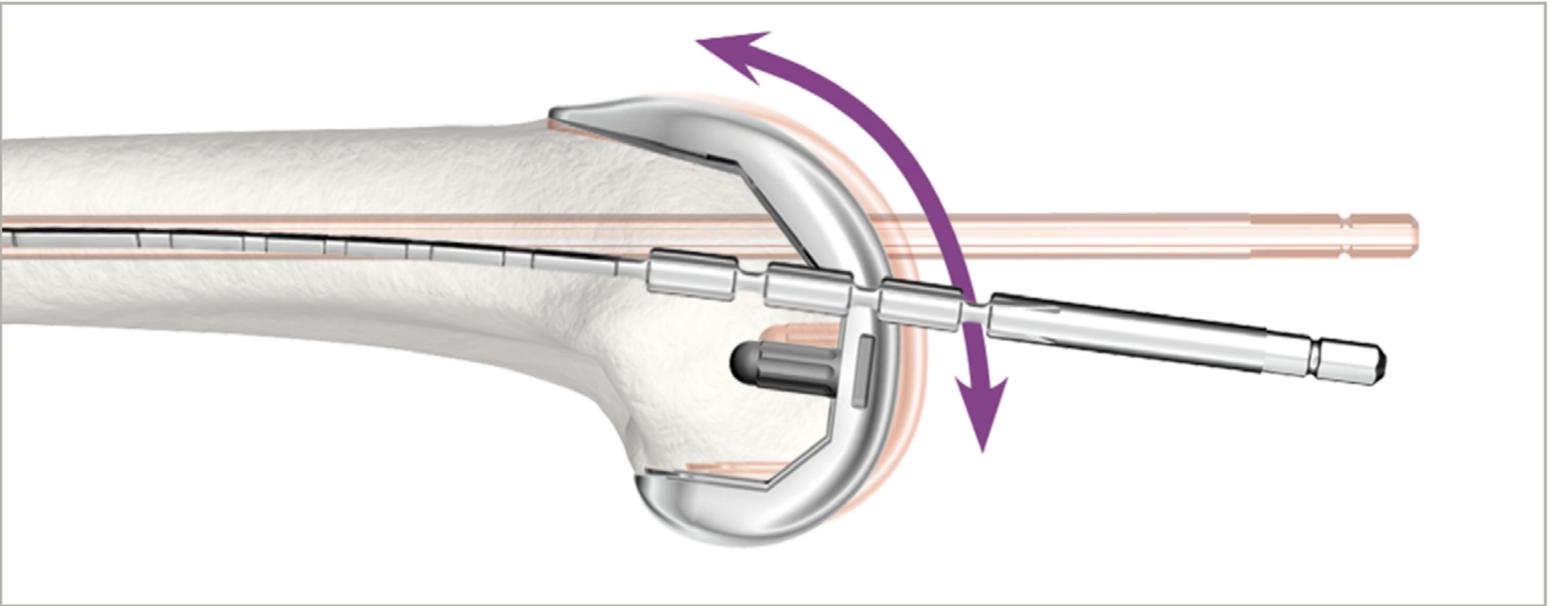

Traditionally, a rigid intramedullary (IM) rod is used for distal femoral preparation.

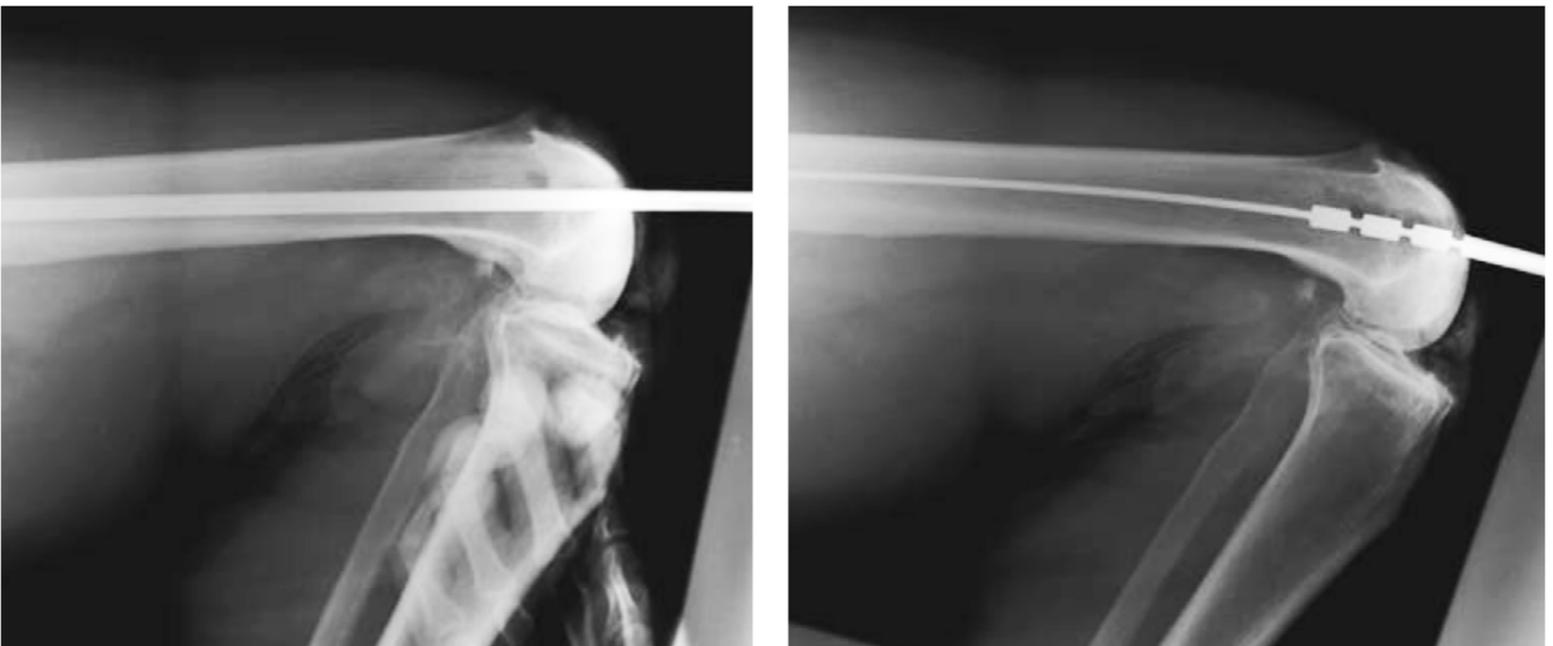


Anterior bow of the femur varies widely.³¹






A rigid IM rod that does not take the bow of the patient's femur into account can potentially lead to an oversized component positioned in extension.¹²


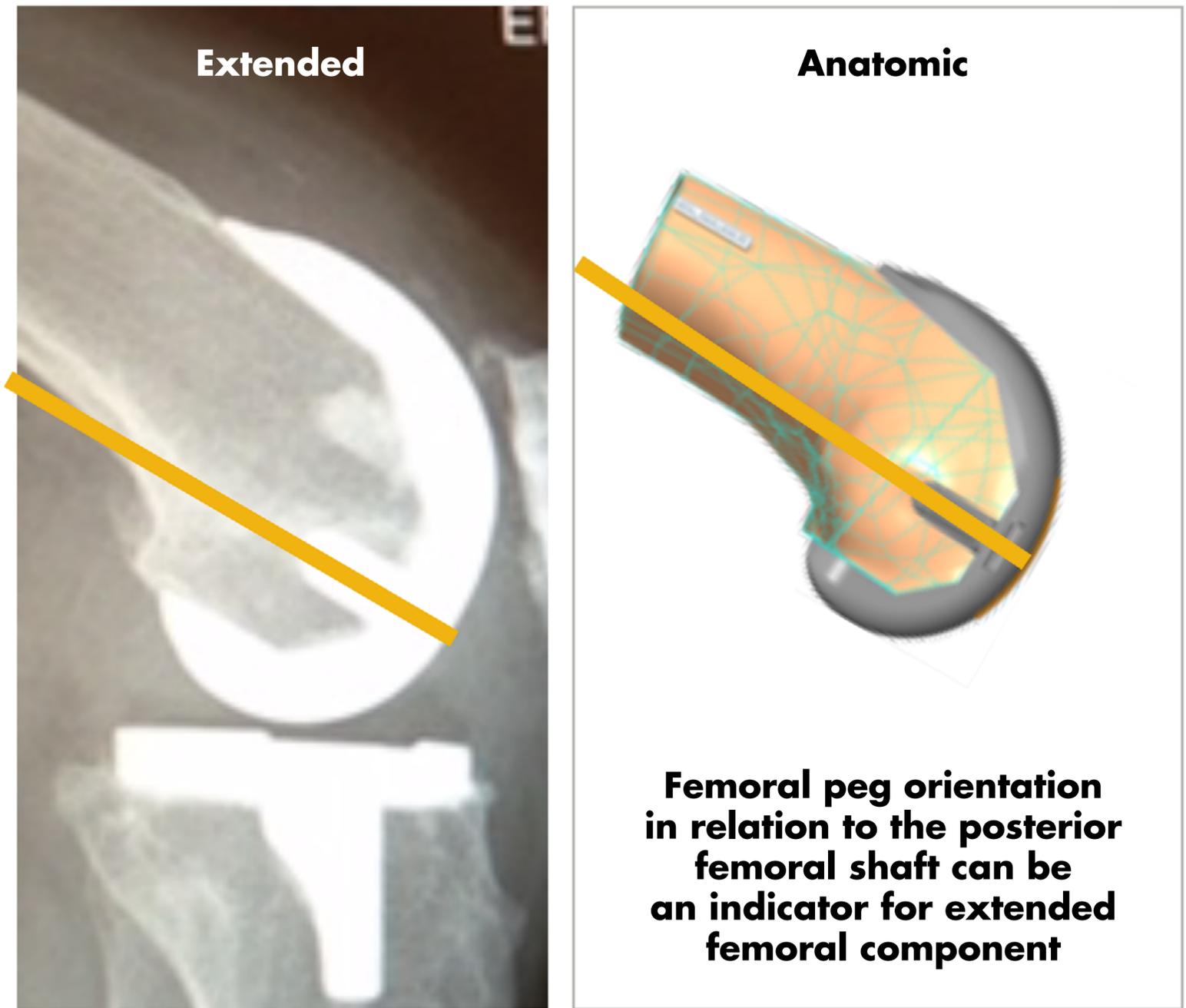


This situation may lead to medial-lateral overhang, soft tissue impingement and pain.¹²

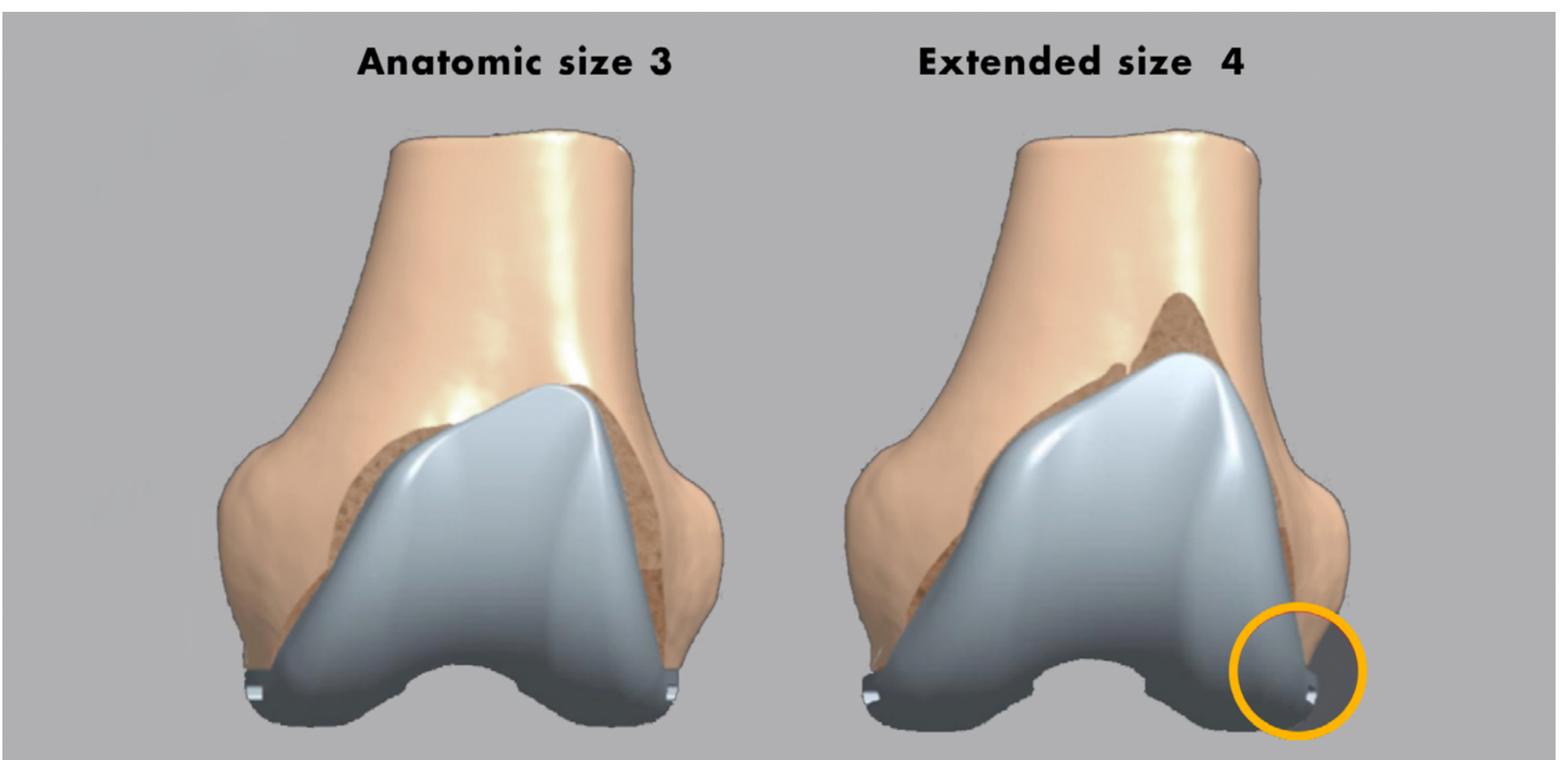




Additionally, the FlexRod can help to position the single radius of the femoral component in alignment with the single radius of the patient's knee. Proper placement of the single radius may aid in balancing the knee.

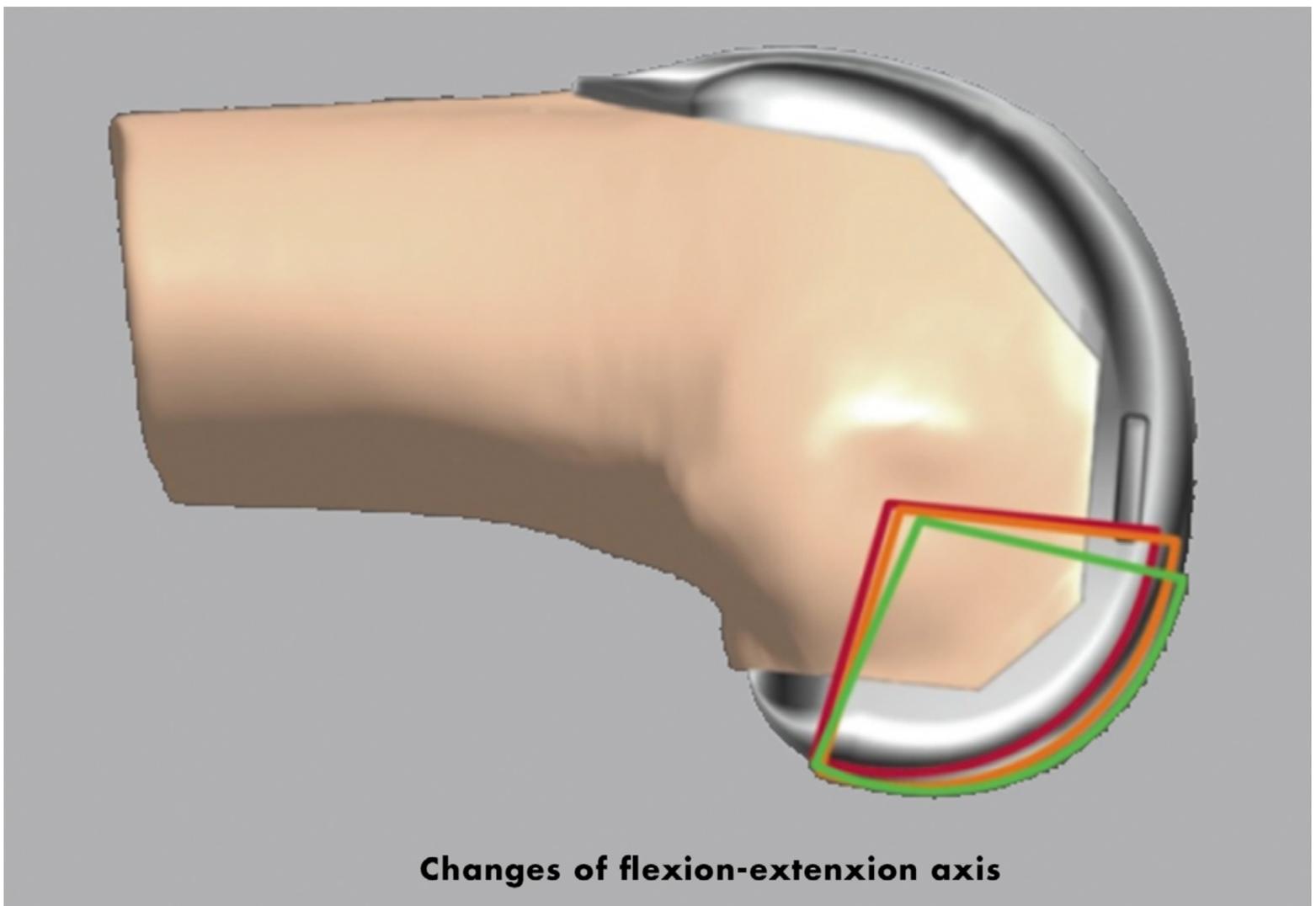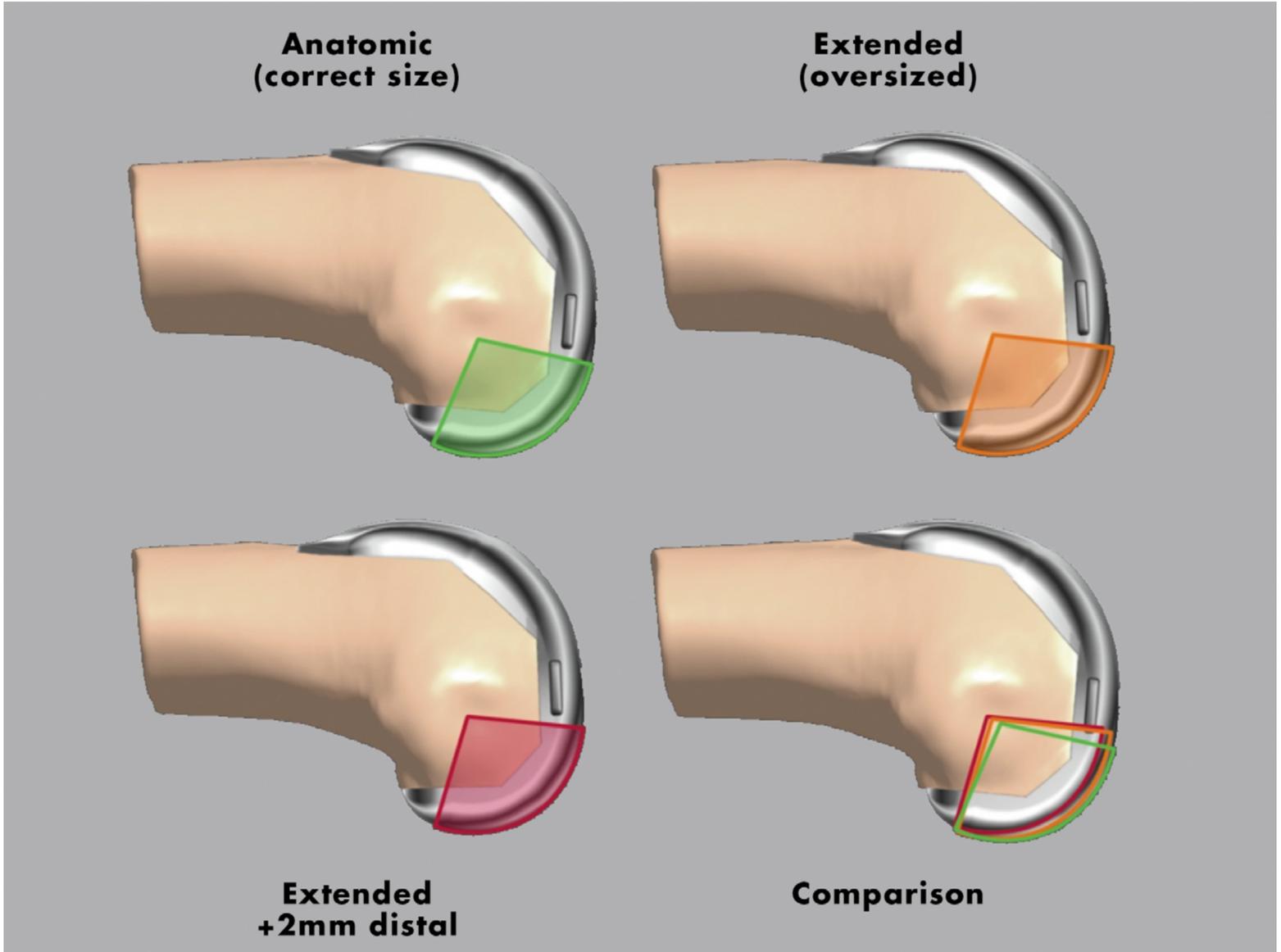



Studies show patients who received TKAs with the FlexRod had better improvements in range of motion and Knee Society Scores and decreased risk of oversizing compared to patients who received TKAs with the rigid IM rod.¹²

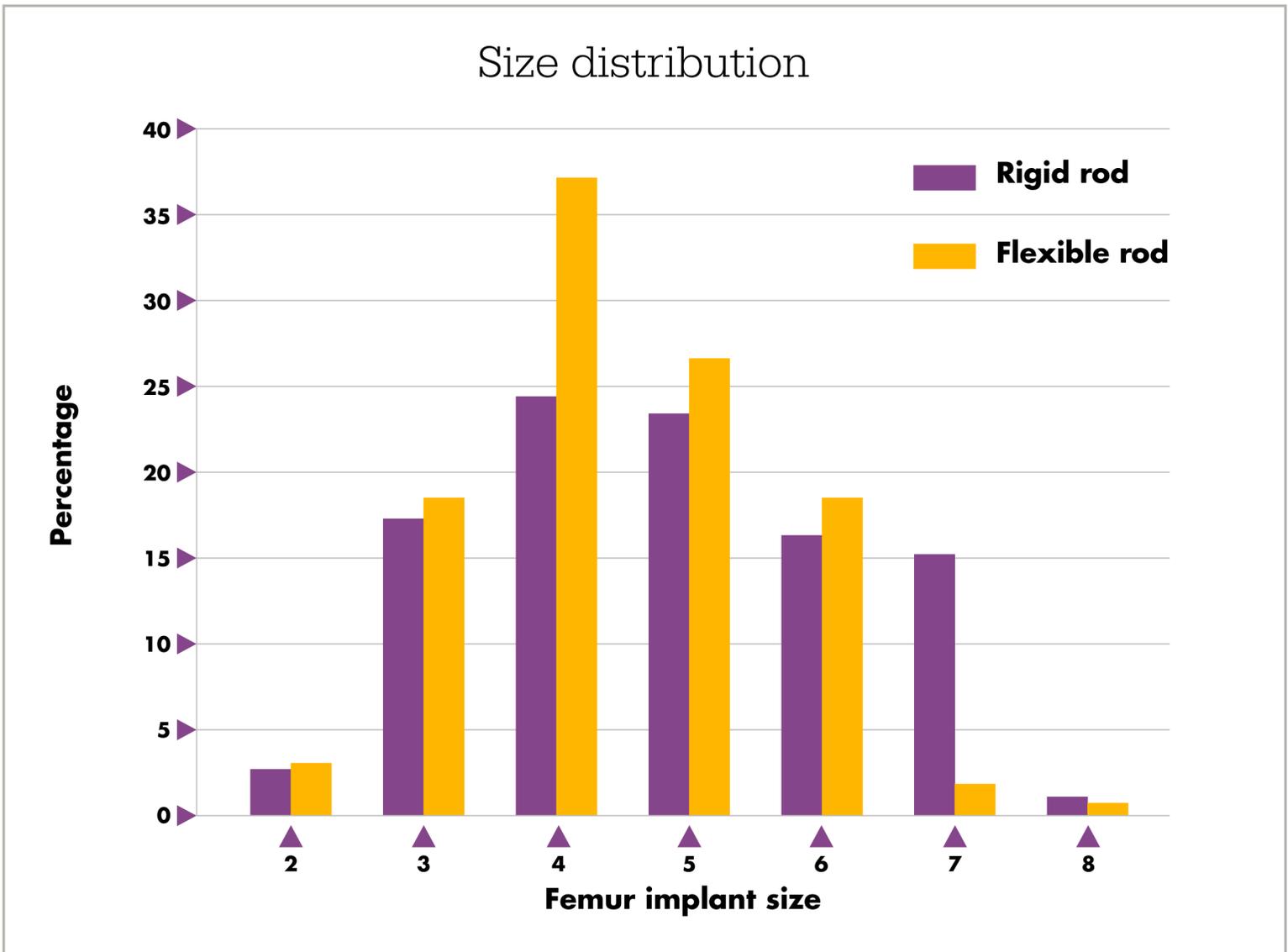

ROM and KSS Pain/Motion

ROM³¹

	pre-op	6 weeks	3 months	1 year	2 years	% change at 2 years
Rigid	101.1	103.1	113.6	119.0	123.6	22.2%
Flexible	99.7	107.0	118.2	124.3	127.9	28.3%
% diff	-1.5%	3.8%	4.1%	4.4%	3.4%	6.1%

KSS Pain/Motion³¹

	pre-op	6 weeks	3 months	1 year	2 years	% change at 2 years
Rigid	37.25	73.9	80.31	88.52	90.5	143.0%
Flexible	35.74	76.9	82.75	90.1	92.73	159.5%
% diff	-4.1%	4.1%	3.0%	1.8%	2.5%	16.5%



X3 polyethylene

Research has identified that the main objective for polyethylene innovation in orthopaedics is to reduce wear through crosslinking without sacrificing strength or oxidation.^{19,33} X3's patented⁵⁰ sequential irradiation and annealing (heating below melting point) process (seen in the video below) has been shown to achieve this objective without the use of additives.^{19,33}






Prior to 2018, any X3 data published included product manufactured using compression molding consolidation and gas plasma sterilization. In 2018, Stryker added ram extrusion consolidation and EtO sterilization capabilities to the X3 manufacturing process.





Wear resistance

Sequential irradiation provides greater crosslinking density versus a single dose as demonstrated in the chart below. The X3 process is a proprietary three-step process of irradiation with 3 MRads (total 9 MRads) of gamma radiation and annealing.¹⁴

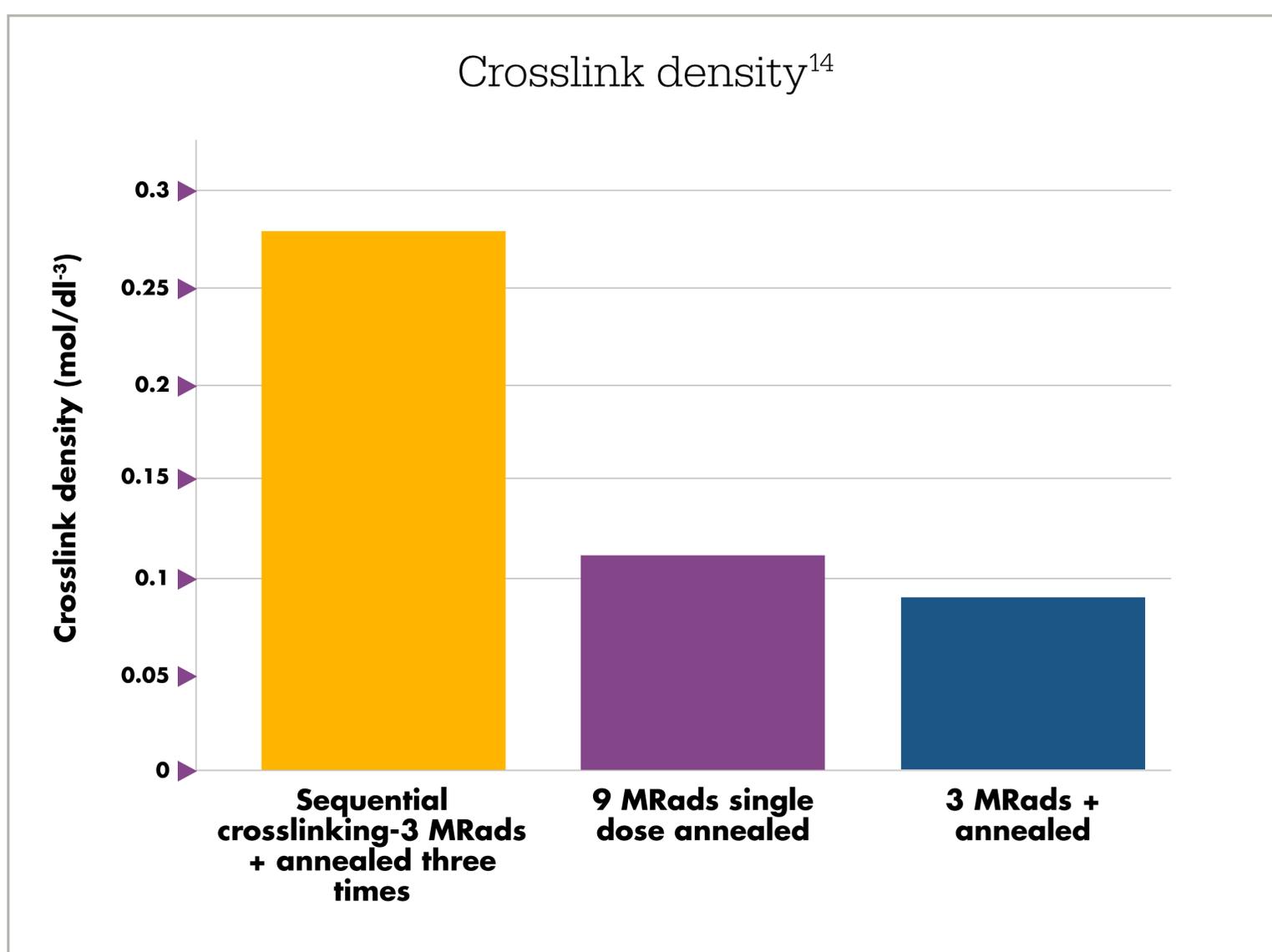




Wear rate

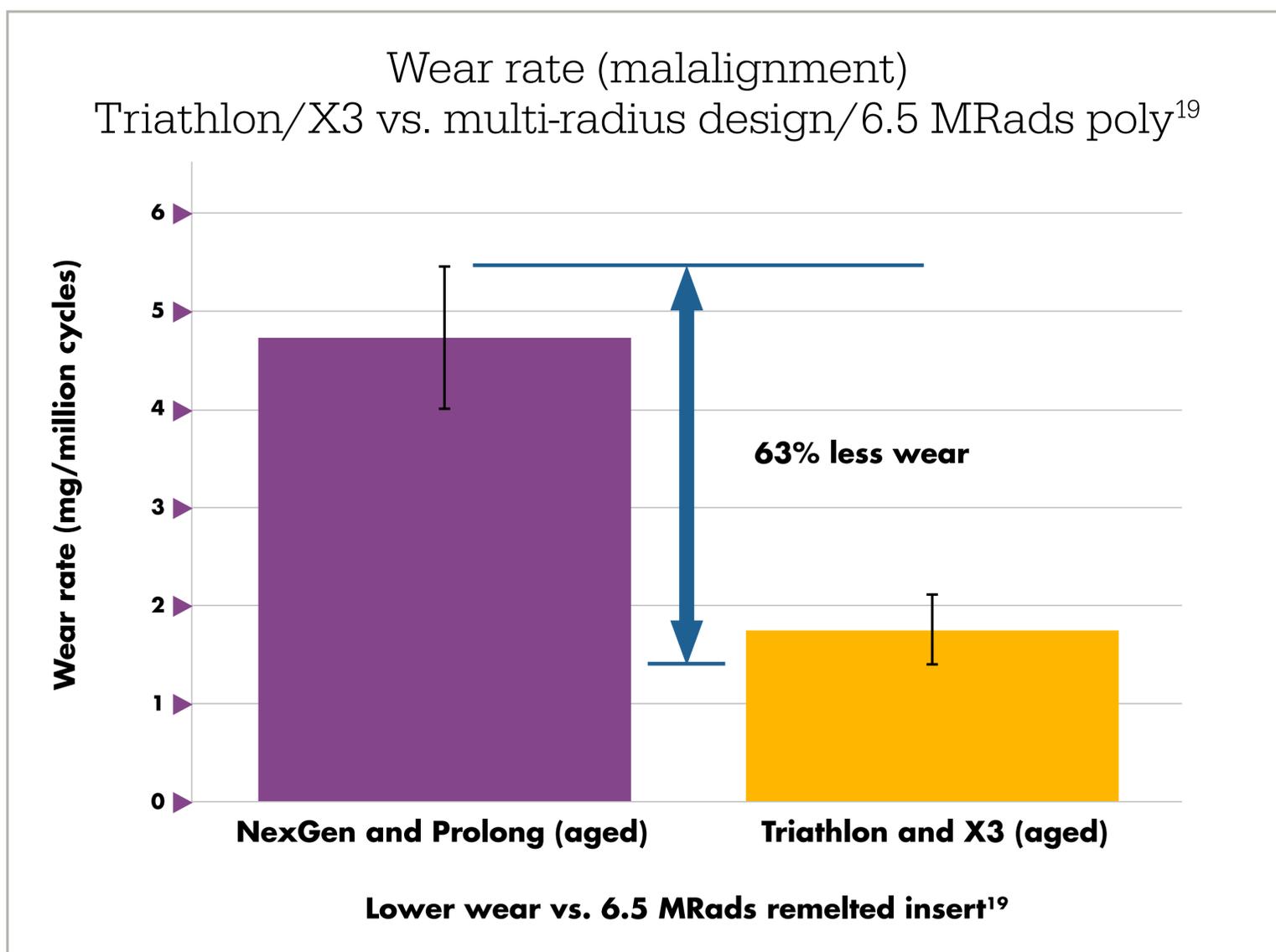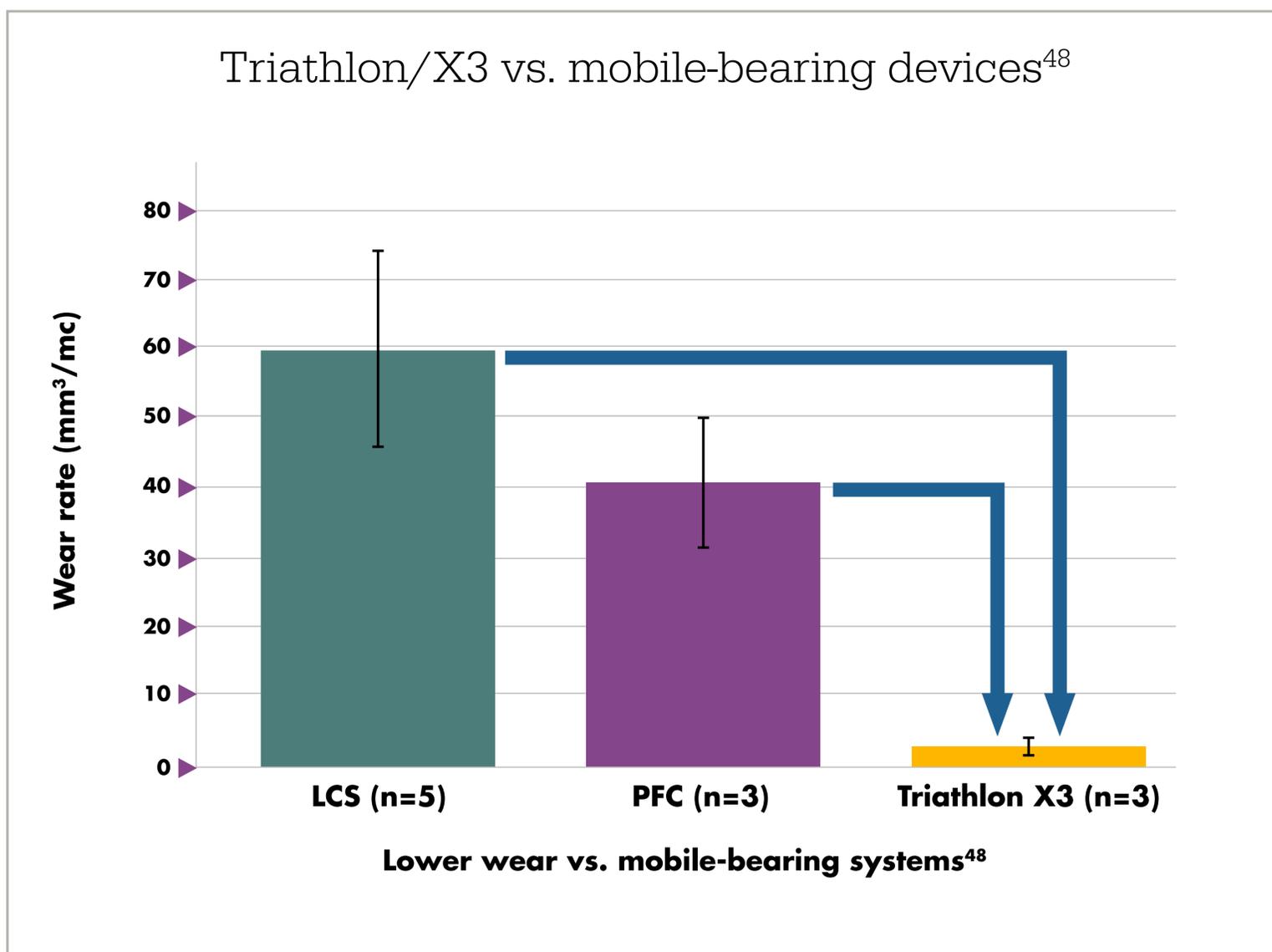




Wear rates (aggressive stair climb)
Triathlon vs. oxidized metal-bearing surface⁸⁸

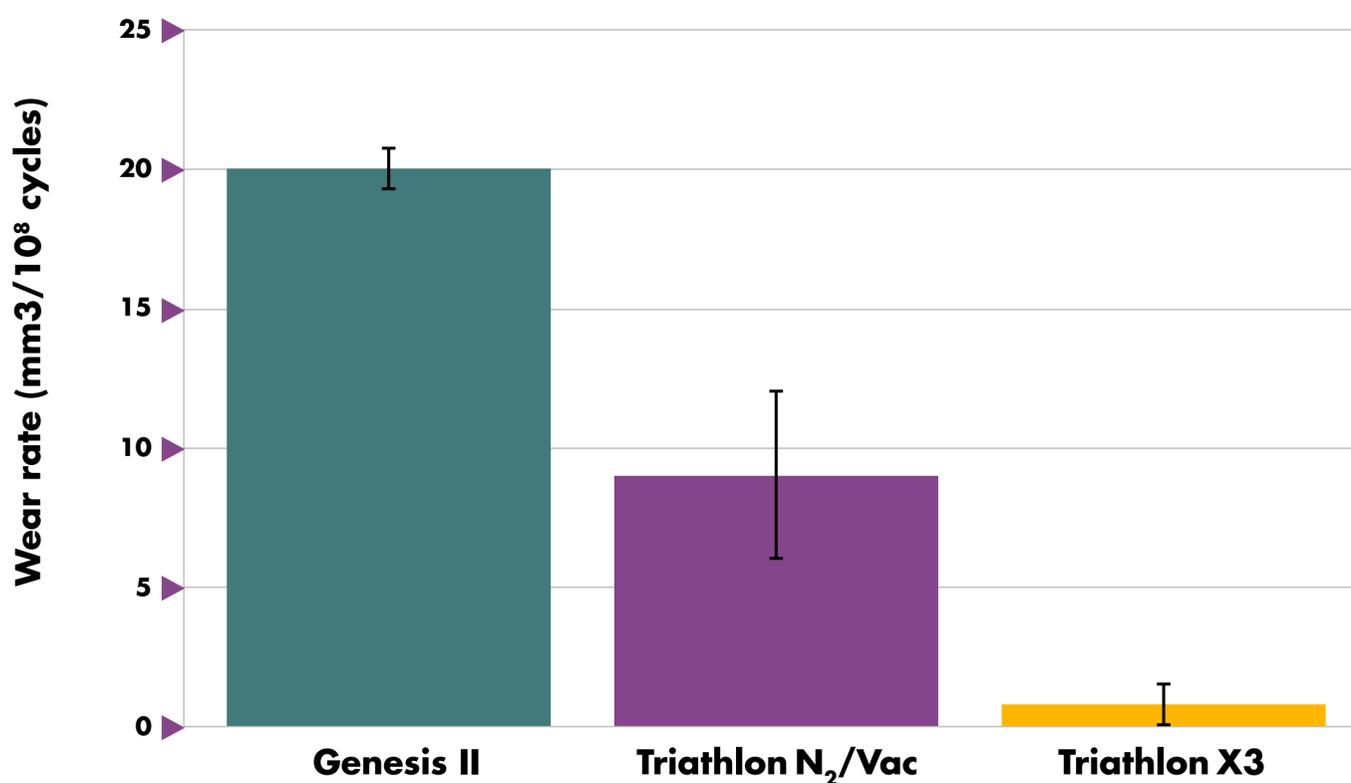

Volume loss (X3 vs. additive polyethylene)¹²¹


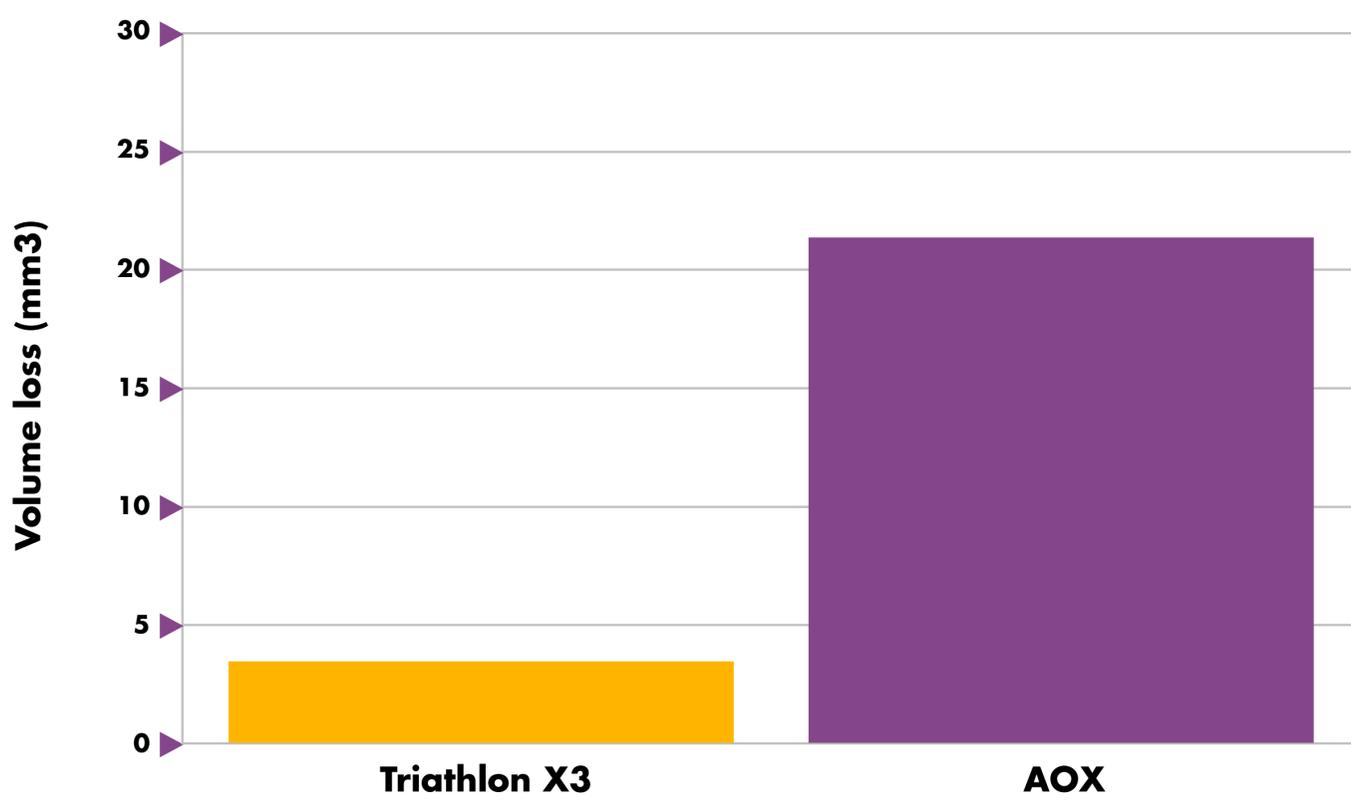


83% less wear with X3



Comparing contact area



Designs with smaller contact areas, like Triathlon,⁴⁰ have been shown to wear less than designs with larger contact areas.³⁹

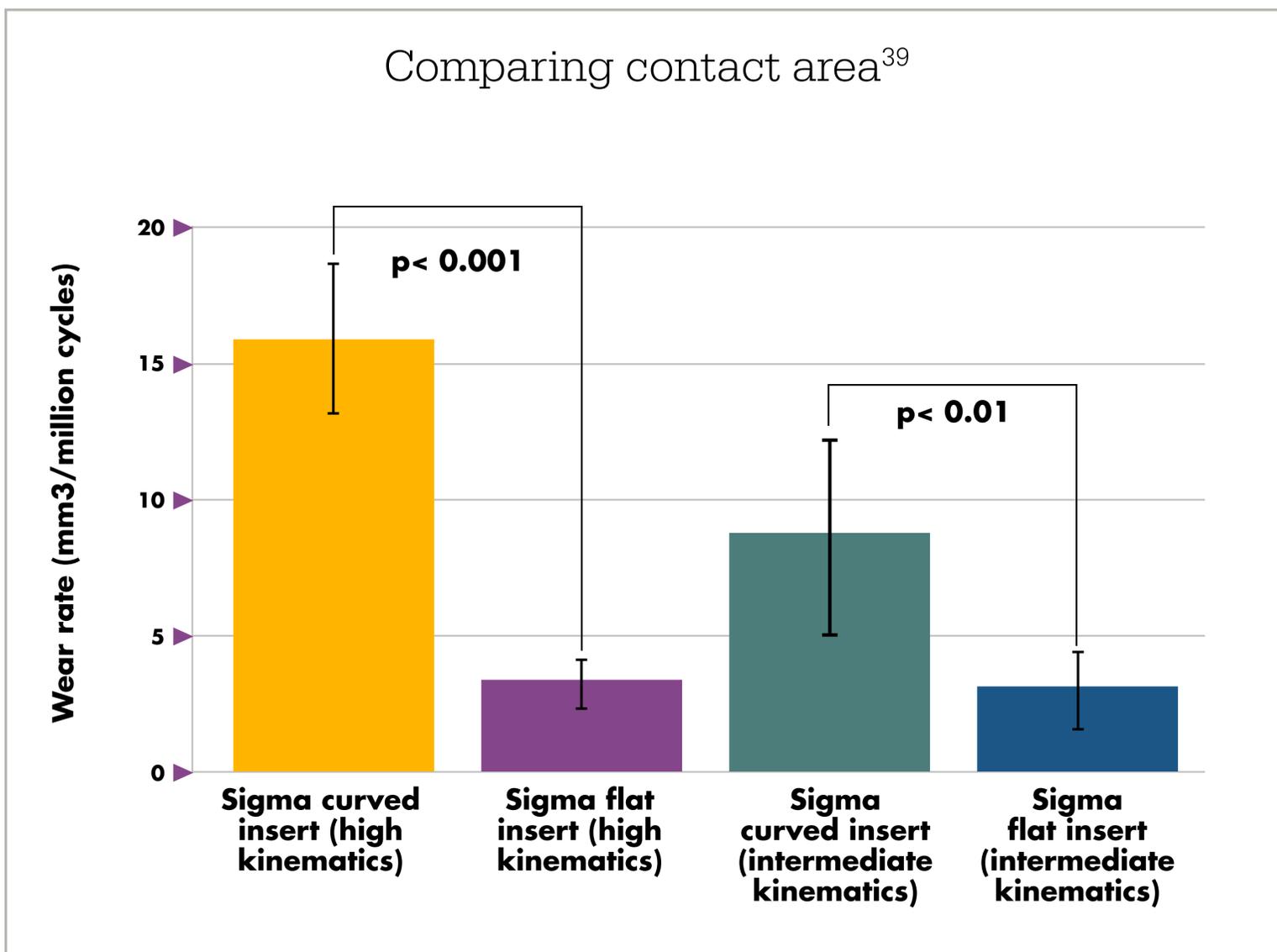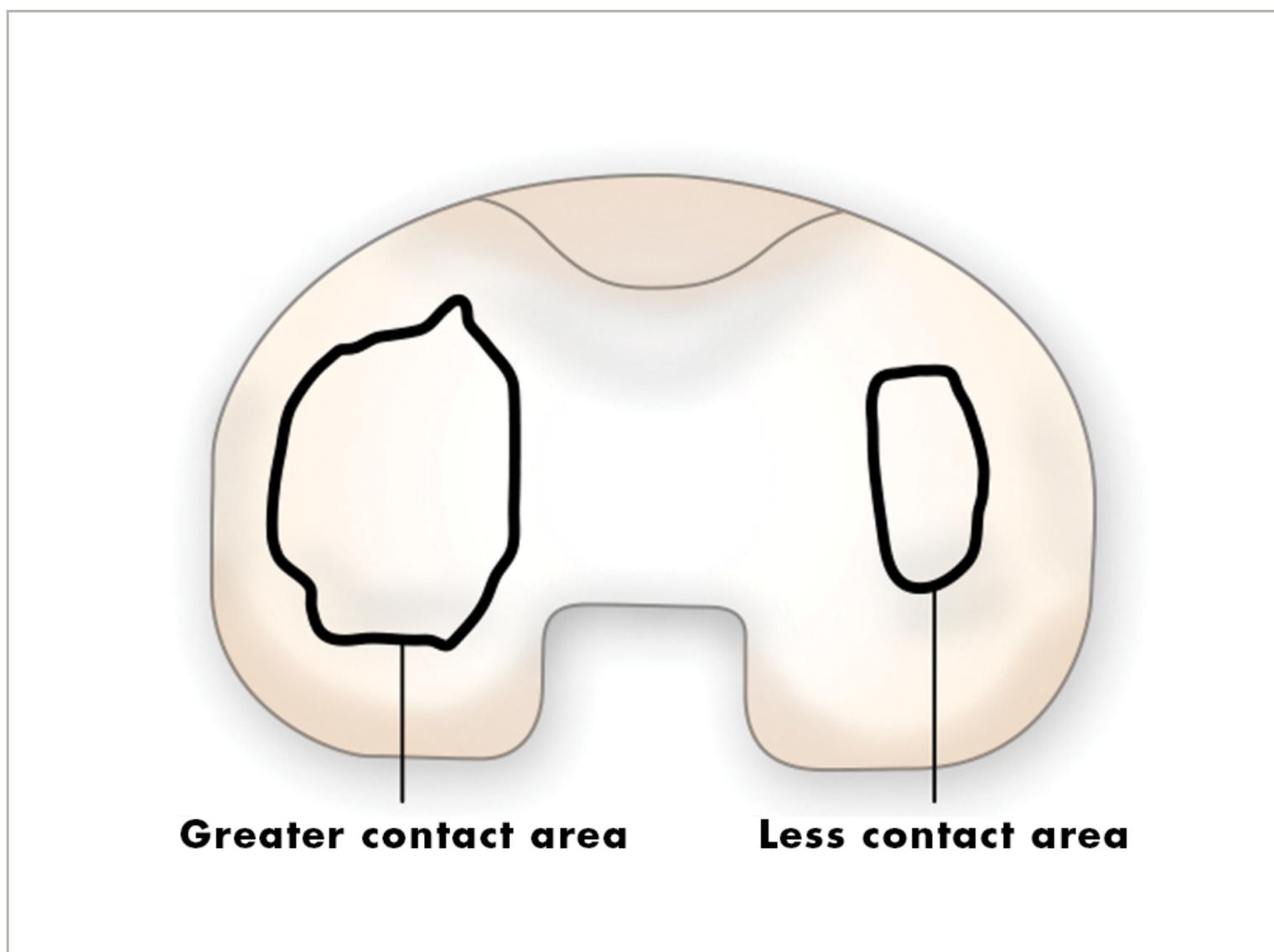



Micromotion and backside wear



Micromotion is a factor that contributes to backside wear.⁸⁵ Triathlon's full periphery locking rim, locking wire and anti-rotation island have demonstrated less micromotion than other designs.⁸⁴

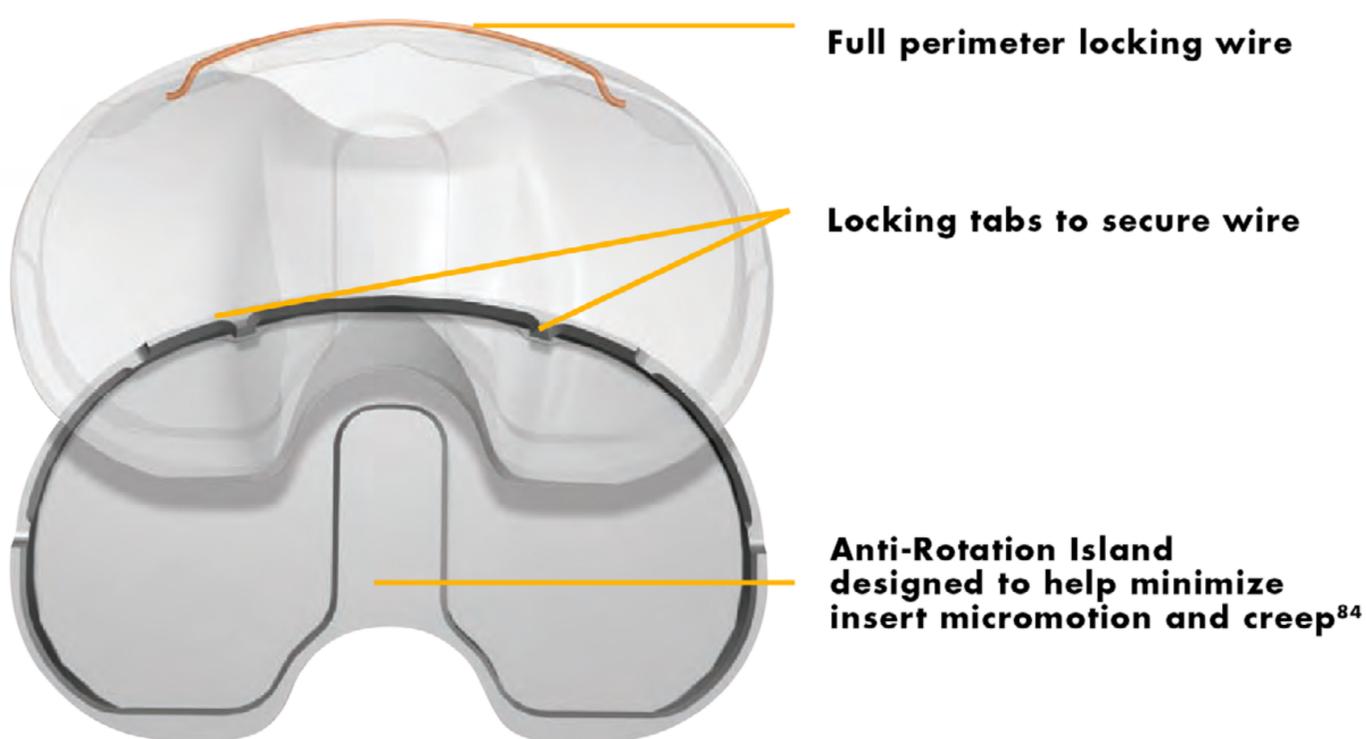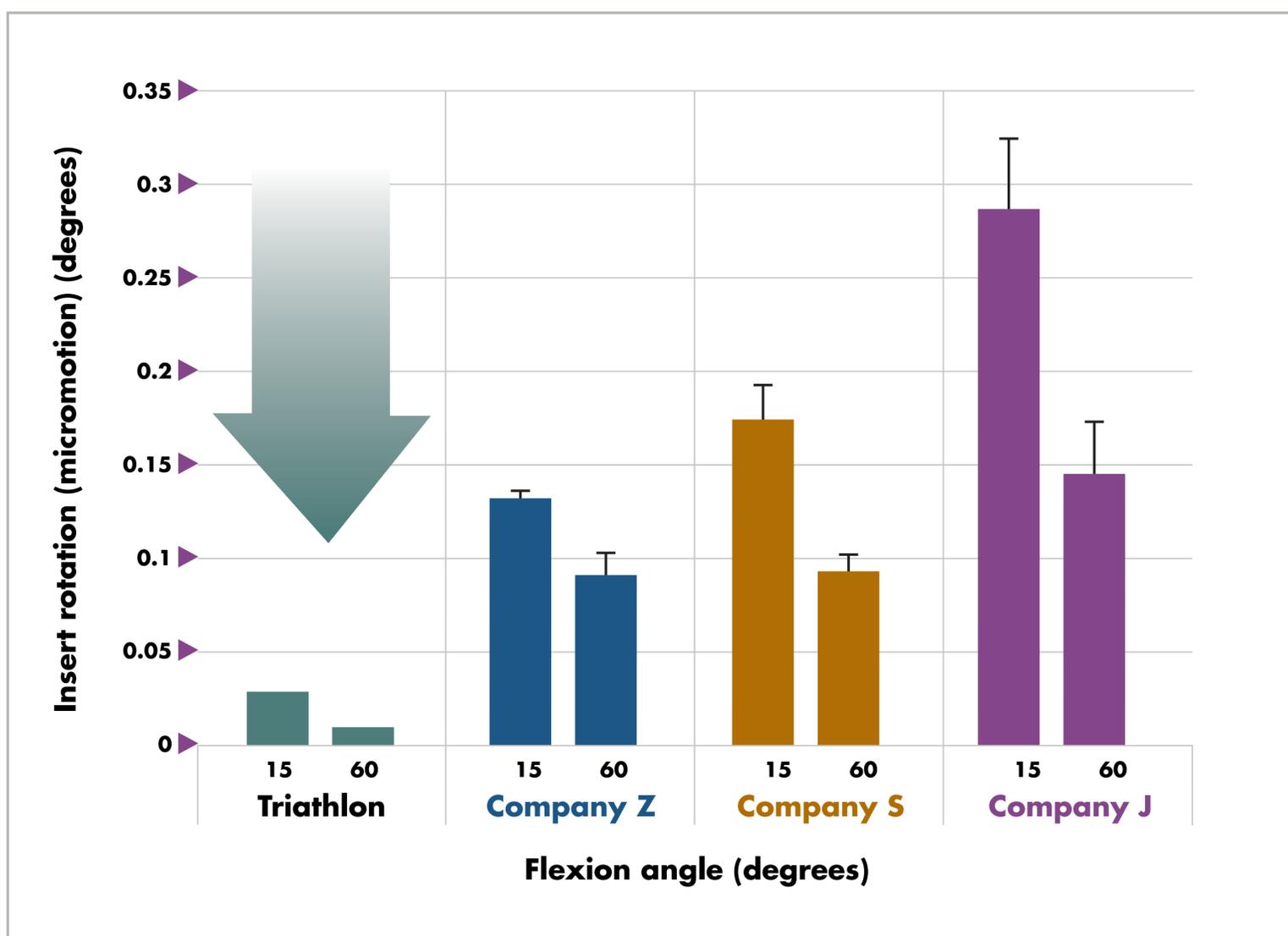



Mechanical strength

X3 is annealed, allowing it to maintain its mechanical strength.³³

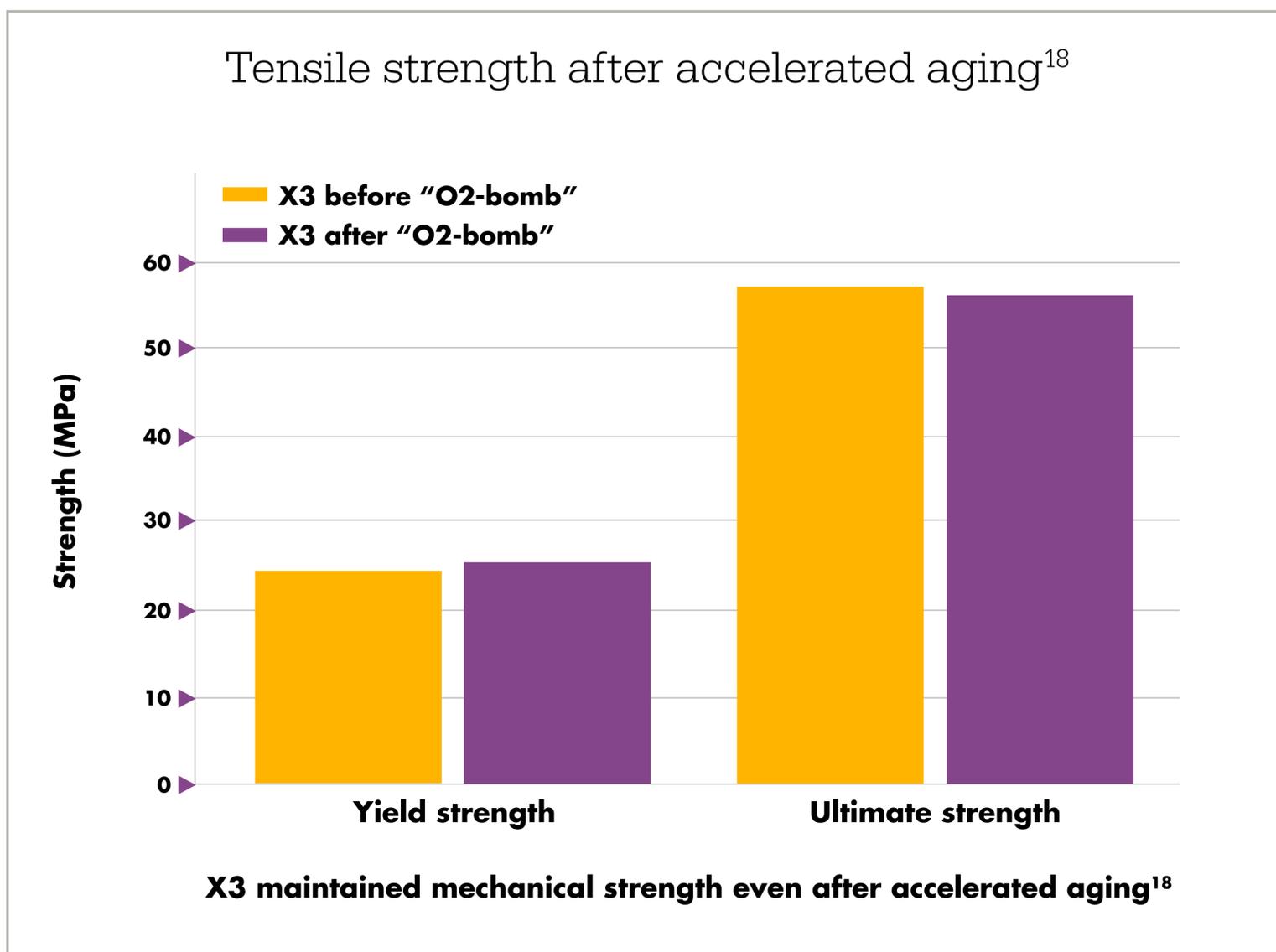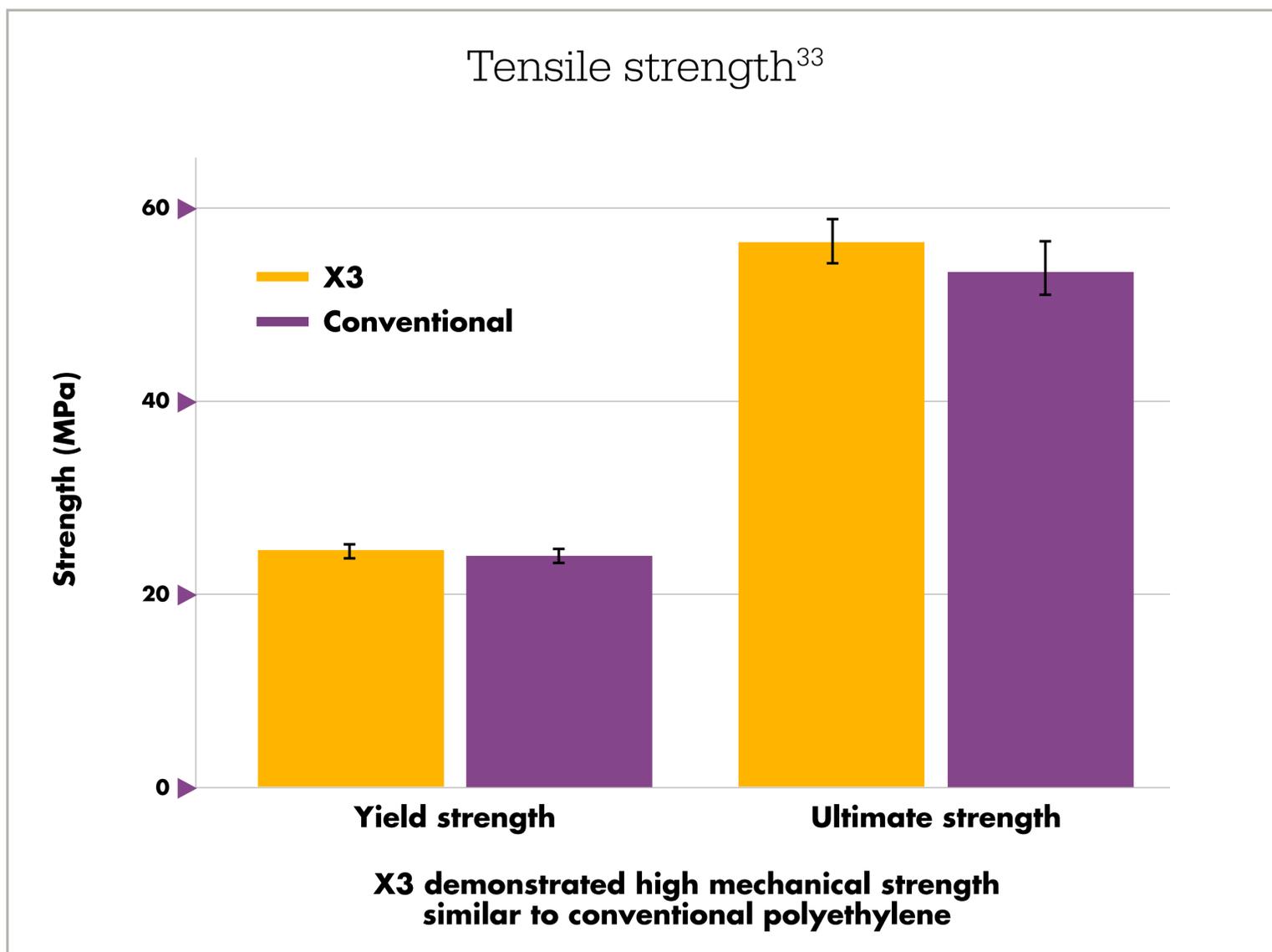




Impact strength

An accelerated aging simulation showed that wear and mechanical integrity of X3 PS inserts was unaffected by accelerated aging, even after a rigorous stair-climbing test.¹⁶ Furthermore, the fracture toughness of X3 is double the minimum requirement for TKA.¹⁸

Meneghini et al. demonstrated no mechanical failure or osteolysis in patients who received a Triathlon PS device with X3.⁸⁶ Minimum follow-up was 5 years.

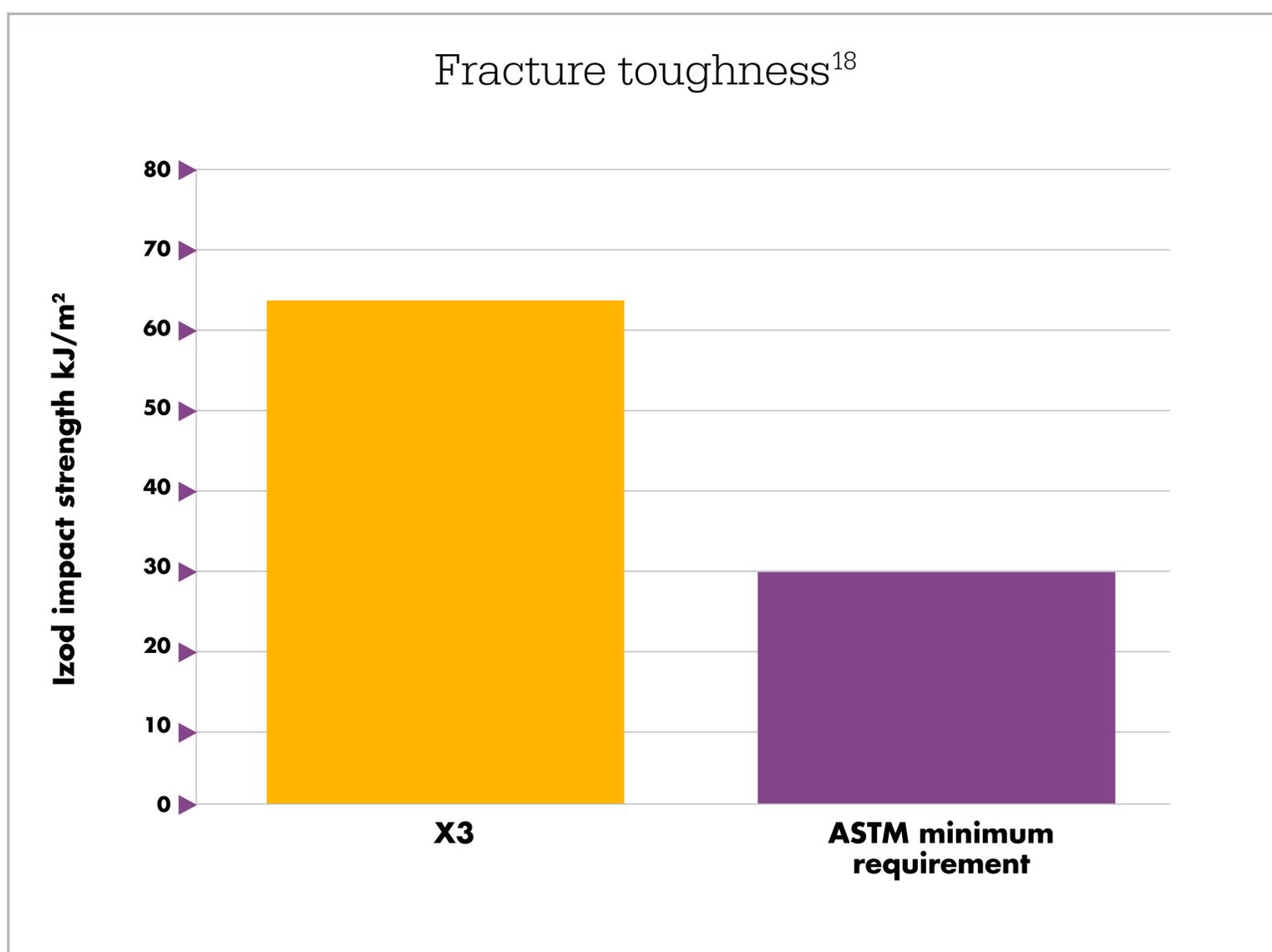

Reported fracture of antioxidant insert

A 2013 case study was published on a fractured tibial insert with an additive antioxidant polyethylene. Analysis of the retrieved insert suggested that fracture was probably a combination of clinical and component factors. Component factors included:⁸¹

- 5.1mm minimum poly thickness
- Gross macromotion related to the locking mechanism, which has no peripheral capture
- Diminished material properties


As with any new technology, additive polyethylene should be followed closely to understand any potential impact it may have.






Oxidation resistance

Multiple laboratory tests have confirmed that X3 virtually eliminates free radicals.^{19,33} X3 oxidation resistance is similar to that of virgin polyethylene.³³ Real-time shelf-life [five years] testing also demonstrated no change in free radical content over time.⁸²

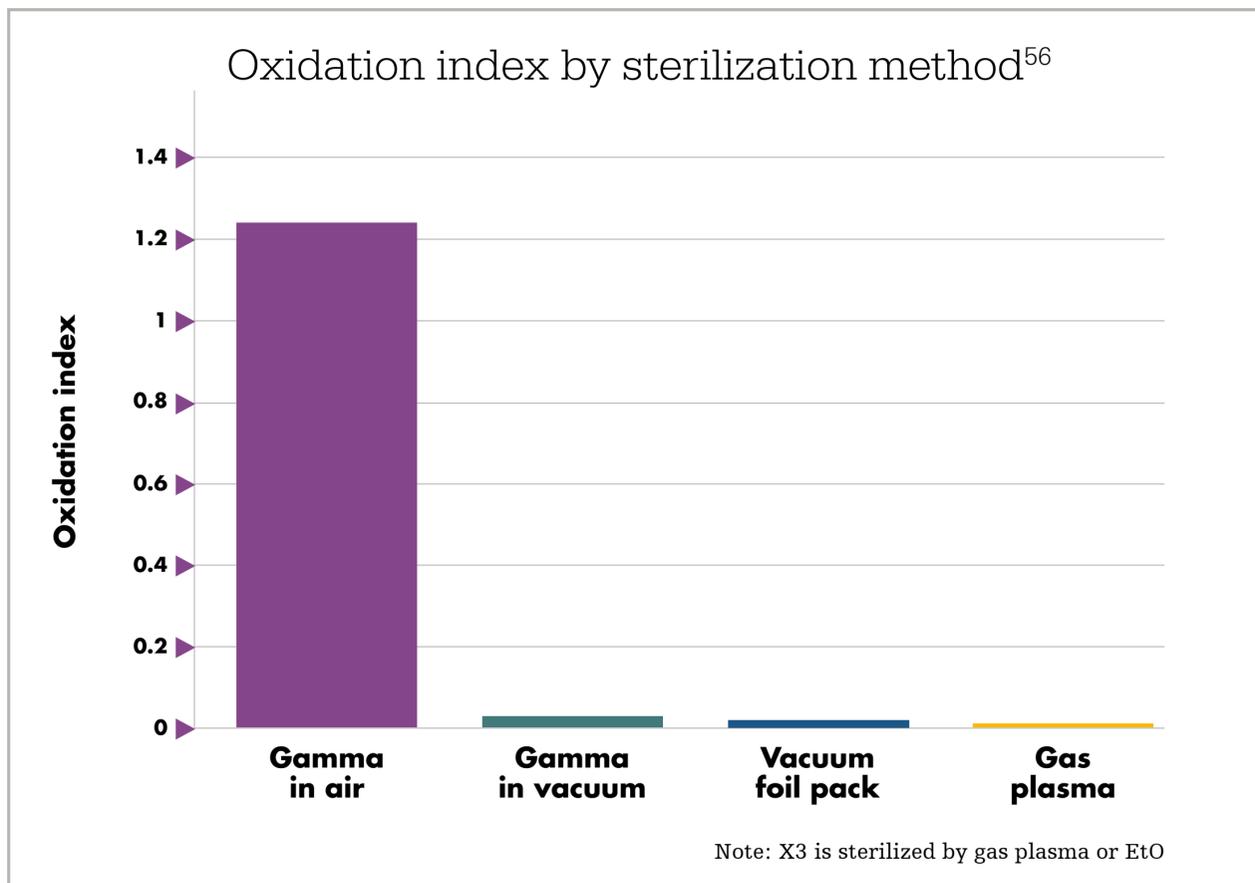

X3 is sterilized in either gas plasma or EtO. As depicted in the above graph, one study showed that polyethylene inserts sterilized in gas plasma yielded the lowest oxidation index in comparison to other current sterilization methods.⁵⁶ In a separate study of twenty retrieved knees that were implanted for 0.5-10 years, EtO-sterilized polyethylene inserts showed a low average oxidation index of 0.08 and stable oxidative properties.¹³⁵

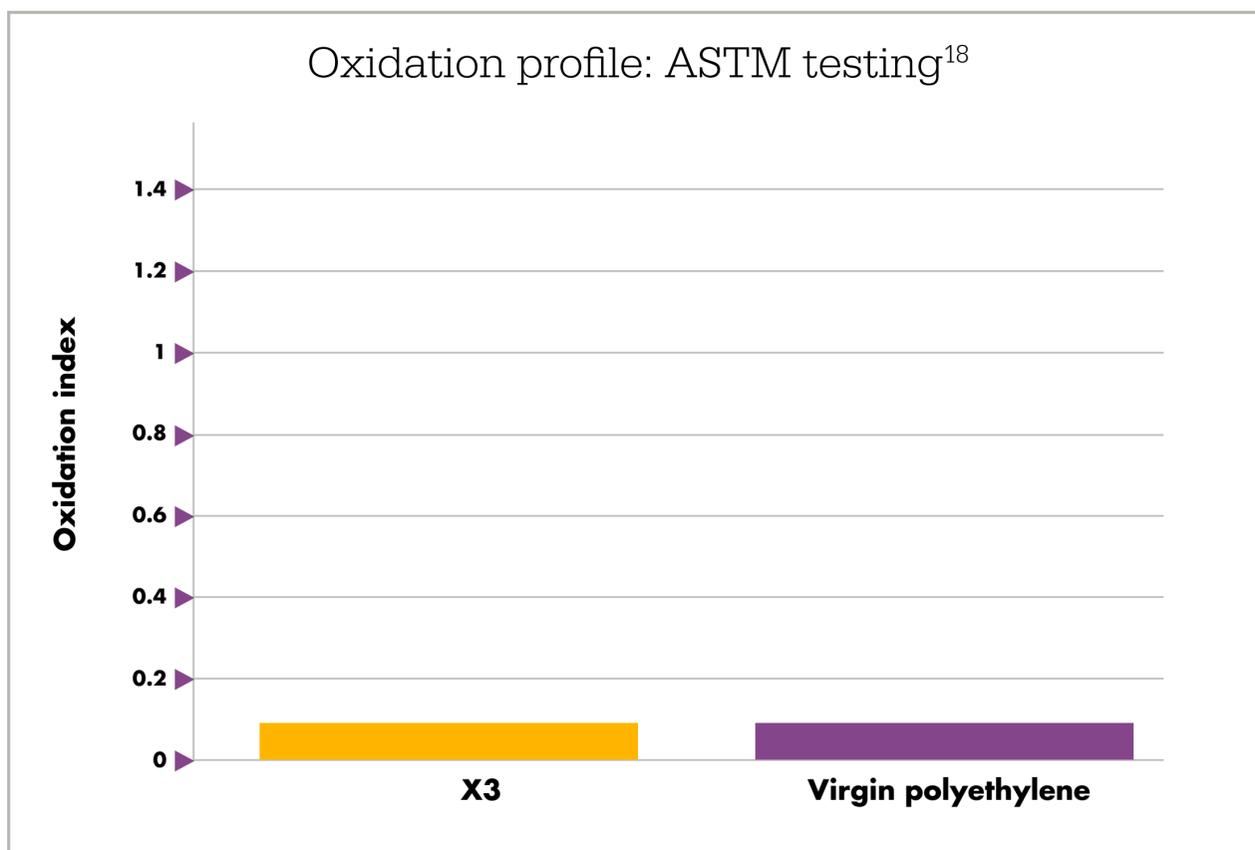



Kurtz et al. studied the largest cohort of X3 retrievals commissioned for any published study to date, showing that X3 exhibits a similar profile for oxidation in vivo to conventional polyethylene. None of the inserts examined in the study were subject to mechanical failures related to oxidative damage.¹³⁴

Discoloration


Investigations into polyethylene retrieval testing have helped to clarify the potential causation of discoloration, oxidation index and white bands. Discoloration seen on retrieved polyethylene may be attributed to residual absorbed body fluids. The image below depicts the results of a test demonstrating reduction in discoloration of a retrieved X3 insert over the course of three weeks in an environment of regular ambient light.⁶⁵






Oxidation index

Some researchers have published on the presence of oxidation on polyethylene retrievals.^{57,58} The test methods used by these retrieval centers have been called into question.⁴⁷ Data supports that the test methods used by institutions like Dartmouth⁵⁸ and Massachusetts General Hospital⁵⁷ can even induce oxidation when using extraction solutions such as heptane.⁴⁷

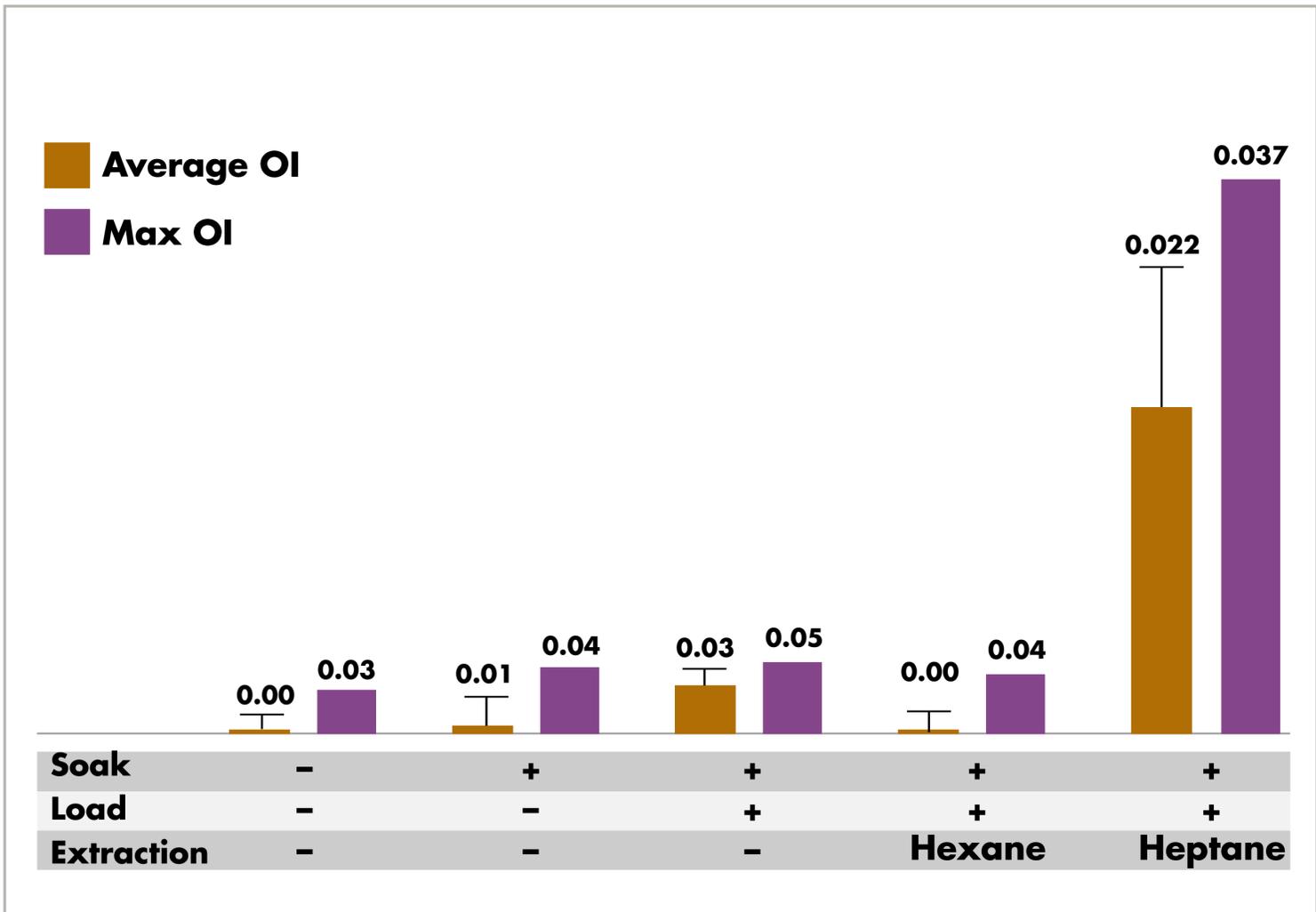

White bands

Evidence also supports that the way retrieval specimens are cross sectioned and processed can create the appearance of white bands.¹²⁶ The images below show how variables within different steps of the retrieval process can cause the appearance of white bands.

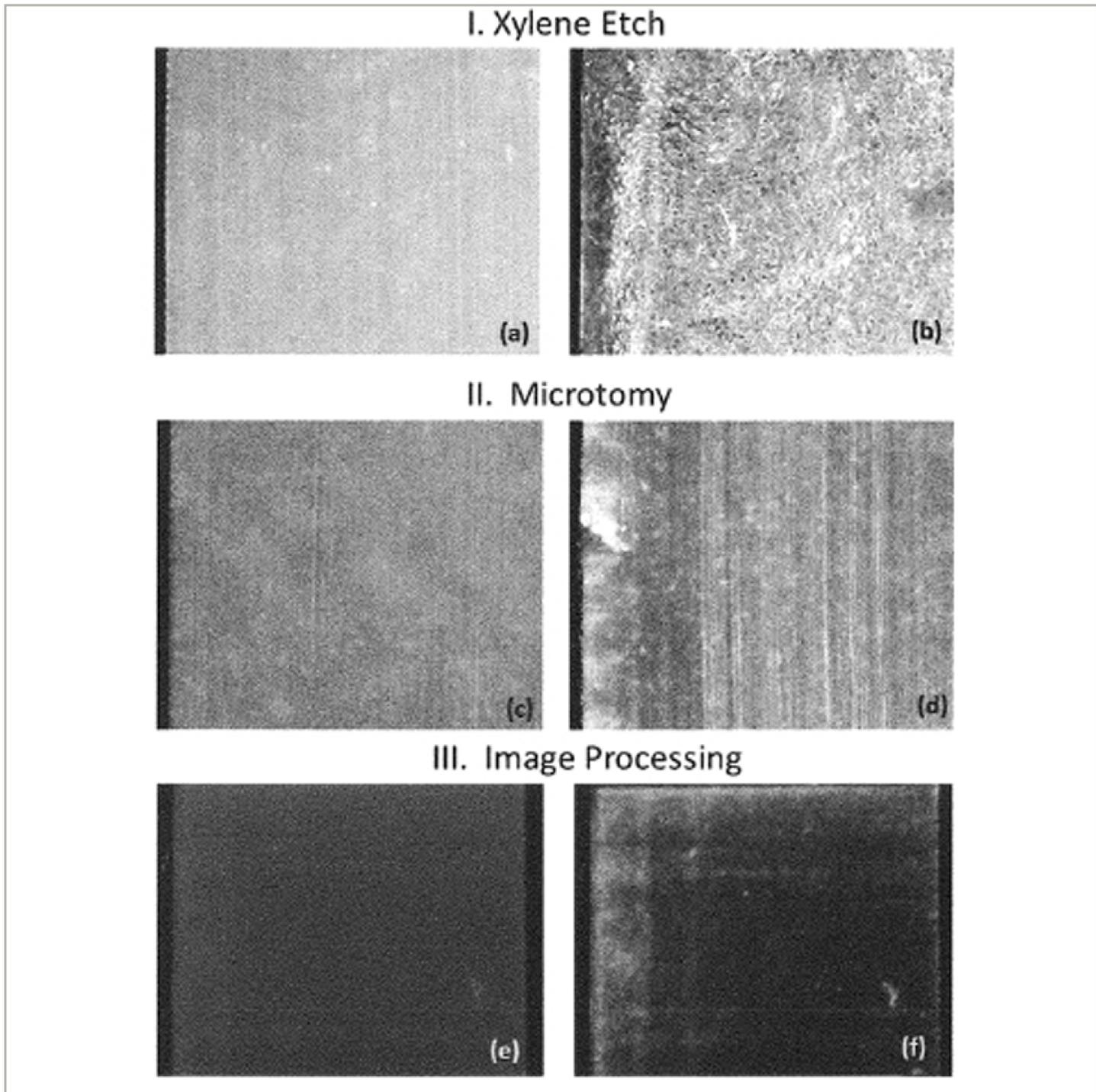



What issues are antioxidants trying to solve?

In an effort to reduce the residual free radicals that could lead to oxidation, many traditional polyethylenes were remelted. Remelting has been shown to improve oxidation resistance, however remelting has also been shown to decrease strength.³⁴

Today, some polyethylenes are manufactured with antioxidant additives.^{35,36} In doing so, these polyethylenes avoid the remelting process that reduces mechanical strength.³⁴

Antioxidant additives are used in an attempt to address concerns over the reduced mechanical strength of remelted polyethylenes.⁴⁶ There is no clinical evidence to support that new antioxidant additive polyethylene provides superior oxidation resistance.

Here are results of competitors' prior attempts to incorporate additives in polyethylene:


- Carbon fiber³⁸ – A carbon fiber additive was intended to improve wear and mechanical strength characteristics. It was later discovered that the additive was associated with polyethylene damage including fibers pulled from the surface, broken fibers and polyethylene removed from the surface fibers.
- Calcium stearate⁵¹ – One study showed that a calcium stearate additive in polyethylene induced bone resorption and promoted inflammation.
- Vitamin E³⁷ – In a lab test, vitamin E was confirmed to leach from inside of vitamin E-blended polyethylene.




Revision rates based on polyethylene type

The Australian Joint Registry data (shown below) shows there is potential long-term benefit in preventing loosening using HXLPE compared to non-HXLPE.⁸⁹ The registry also reports there is an increase in survivorship at 10 years with Triathlon PS and X3 compared to non-HXLPE, with a sample size of thousands of patients.⁸⁹

Furthermore, it has been demonstrated that X3 may be especially beneficial for younger and heavier patient populations. One study comparing X3 to conventional polyethylene showed that X3 had a significantly better survival rate among patients who were younger than 60 years old and/or had a BMI greater than 35.¹³⁸

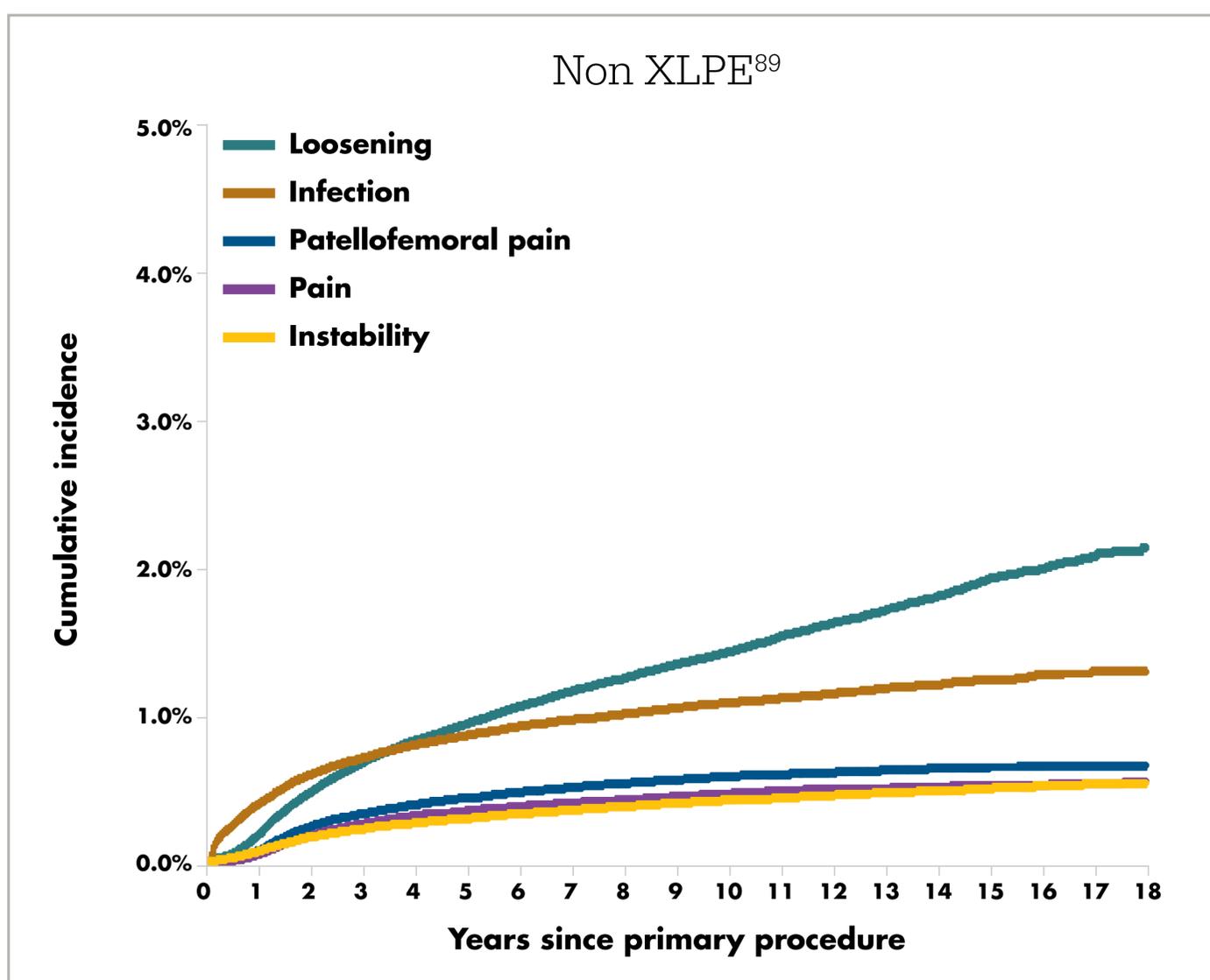


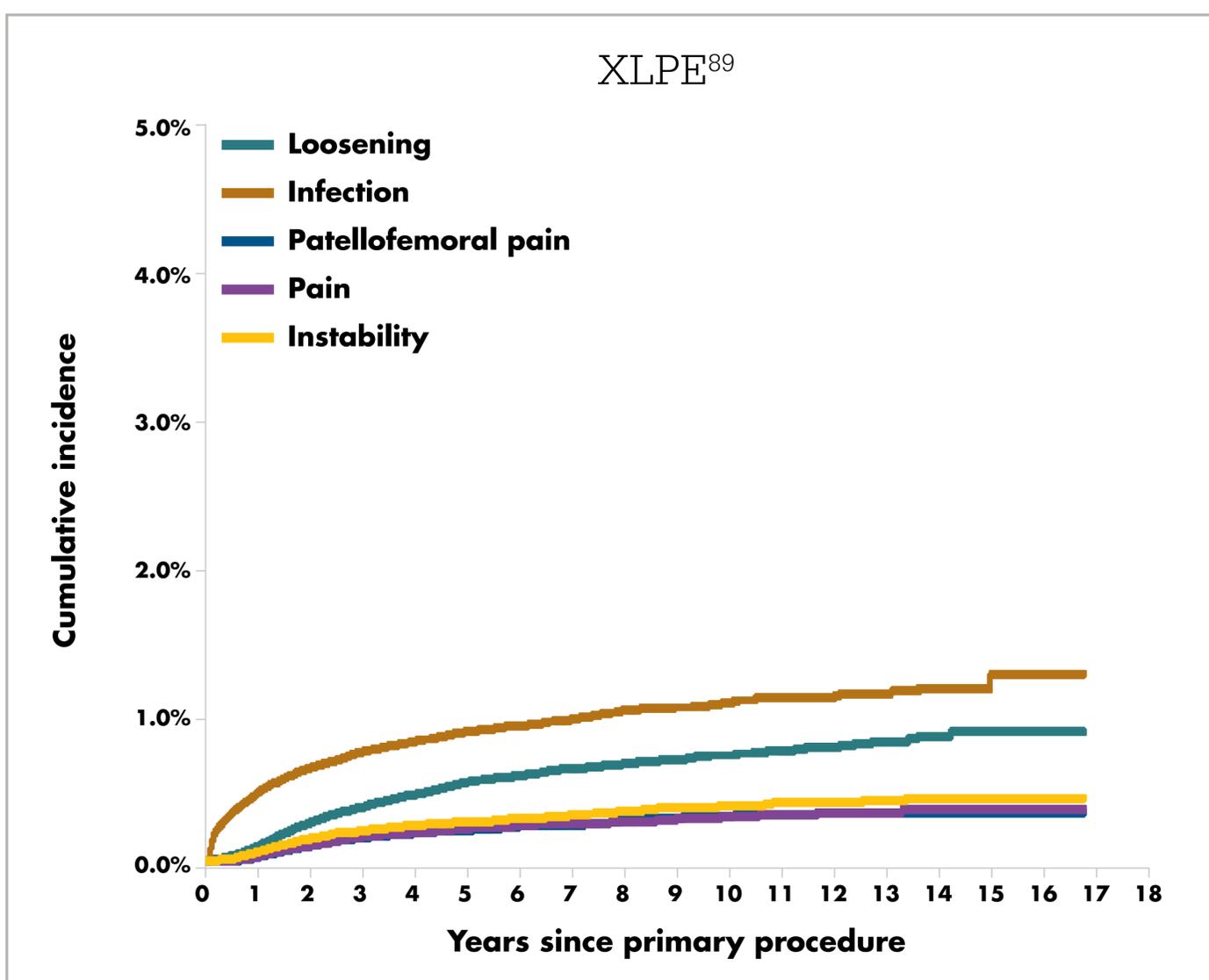



Gait


A 2015 gait investigation comparing patients with a single or multi-radius knee showed that patients with a single radius knee experienced gait patterns that more closely mimicked that of the non-diseased control group while the multi-radius knee group differed in important knee kinetic and kinematic properties.⁸²

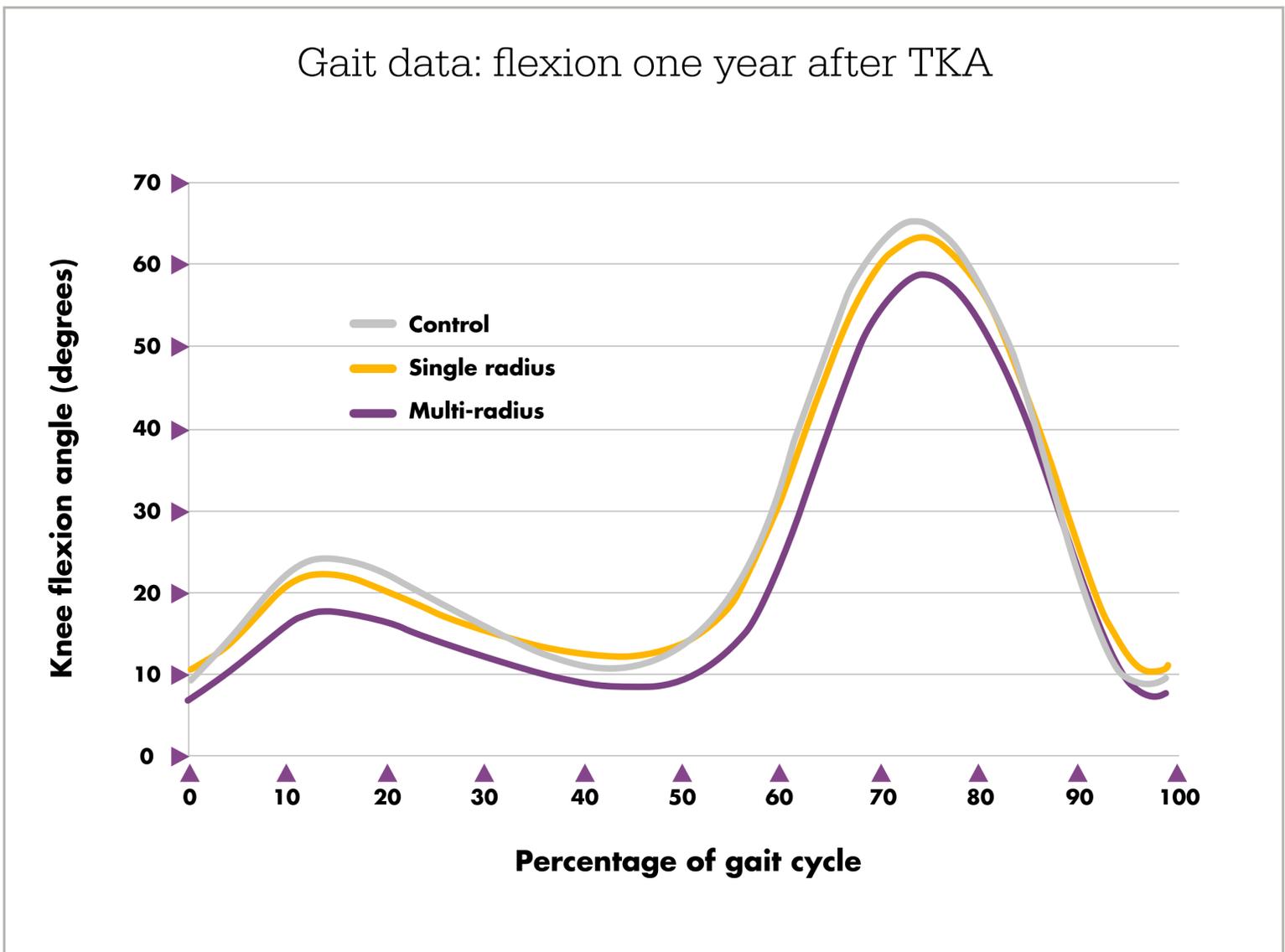



Pivot


Fluoroscopic data showed that Triathlon allows for medial pivot without features that drive a medial pivot.¹⁰⁷

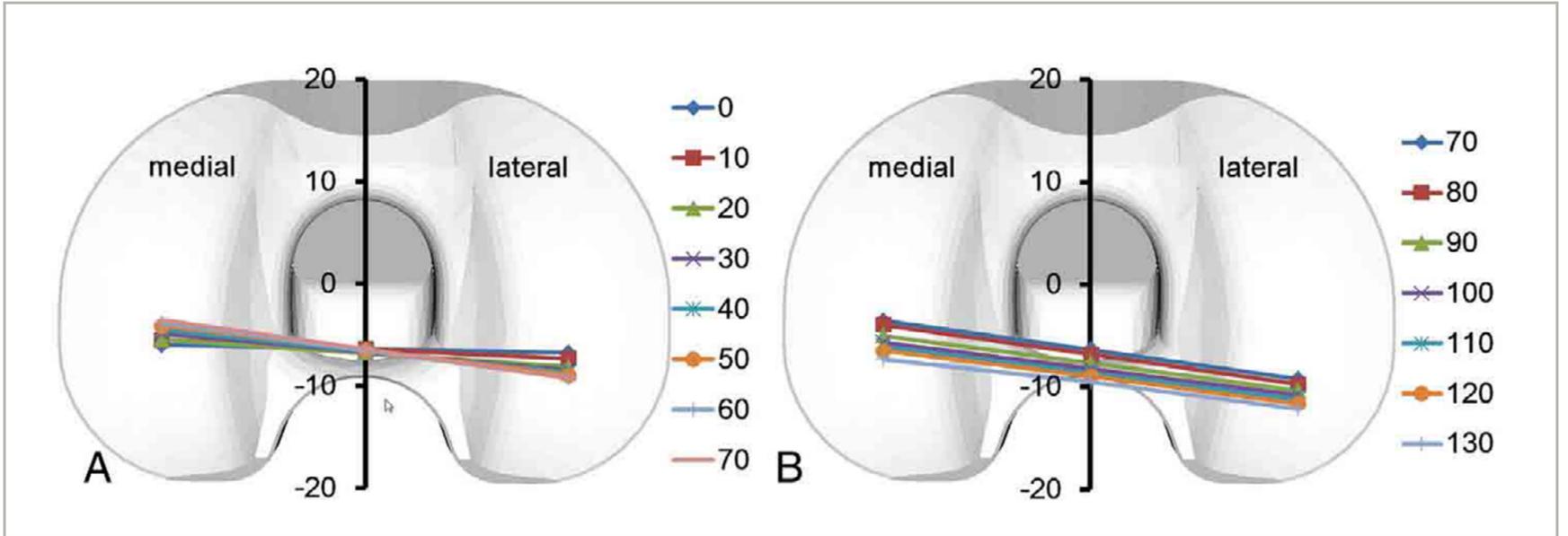



Deep flexion

Triathlon is designed to accommodate up to 150° of deep flexion.²⁸ This is achieved through the interaction of the shortened, flared posterior condyles and rotary arc. The flared posterior condyles⁷ and the rotary arc⁶ are designed to accommodate up to 20° of internal/external rotation. The shortened posterior condyles are designed to facilitate the relaxation of the soft tissues.⁶ The recommended tibial slope for Triathlon is no more than 3° .⁶⁸ A 2014 publication showed that in TKA revised for flexion instability, the average pre-revision slope was approximately 7° .¹²⁹

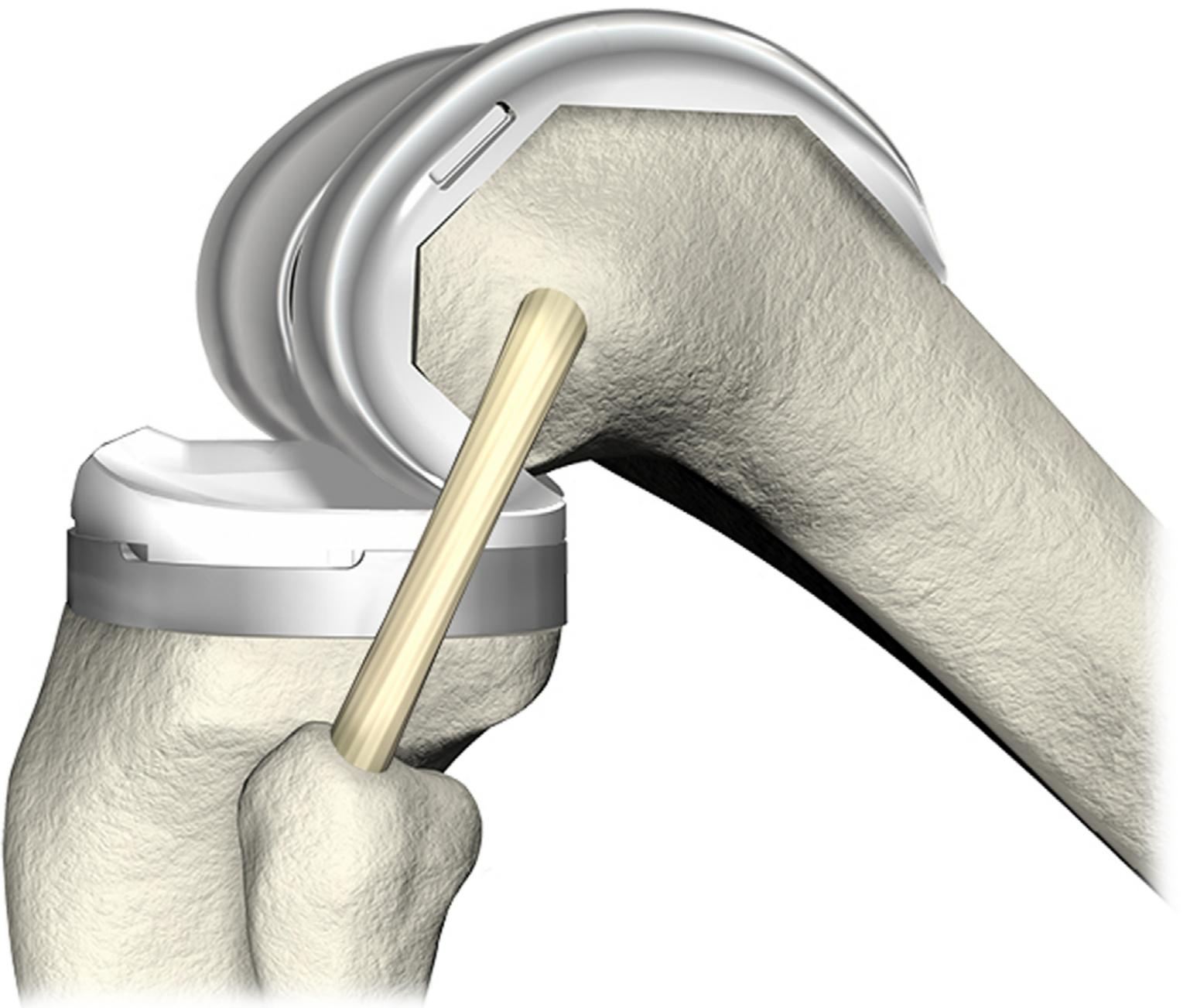



Hyperextension/extension


Triathlon is designed to allow for +/- 10° of rotation in hyperextension and extension.⁴⁴ In these early degrees of motion, Triathlon has demonstrated less post impingement and torque forces than other designs.⁴⁴





Pain scores

An investigation of Triathlon patients showed a high level of satisfaction, as evidenced by their patient-reported pain scores. This level was maintained at seven years.¹³²

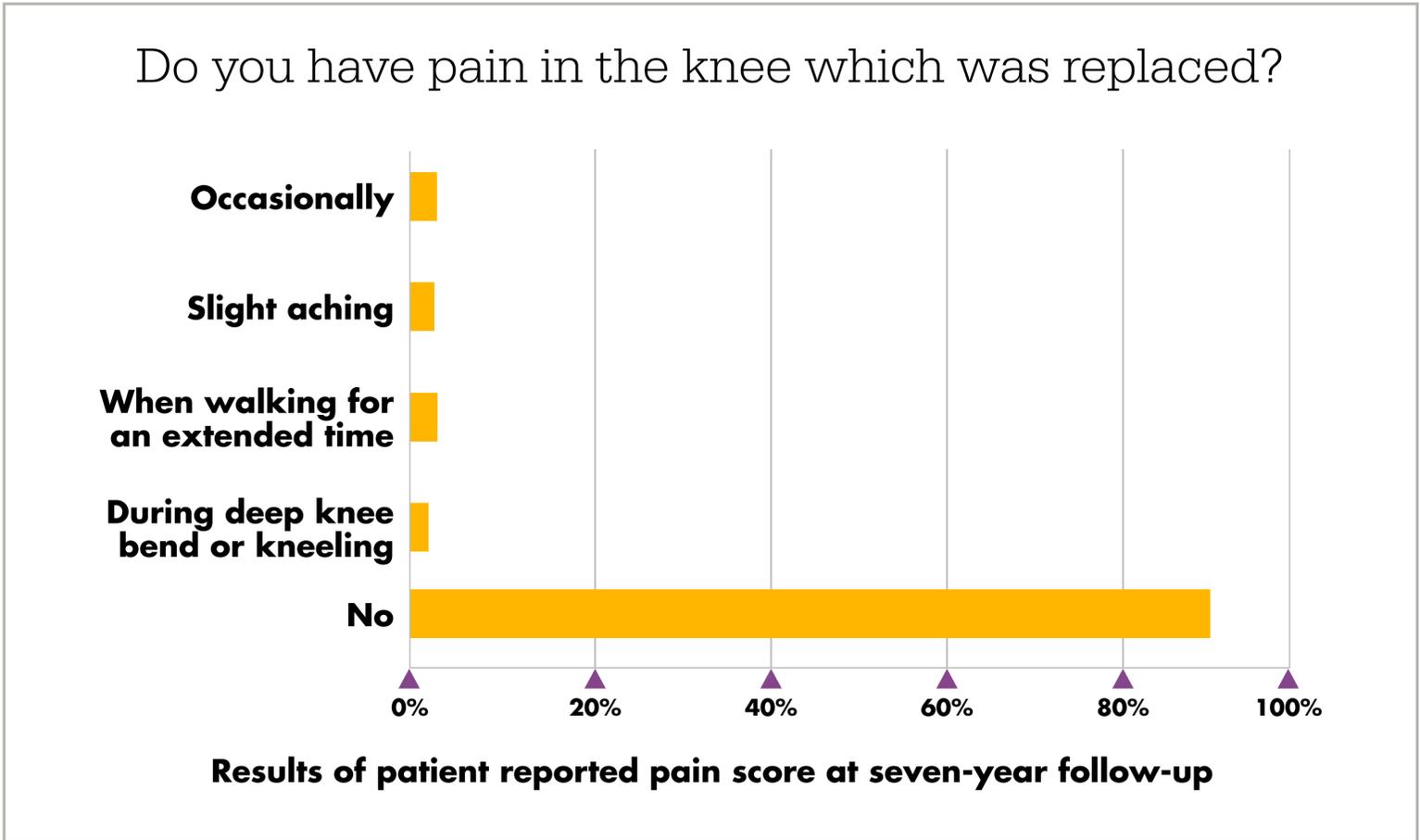





Recovery

Triathlon's single radius design allows for greater quadriceps efficiency in extension.⁴ Given the quadriceps' role in mobility, any undue strain may have an impact on recovery time. The single radius design may also allow for enhanced function by a reduction in the muscle work required to achieve the same level of activity.^{1,28}

Studies on Stryker's single radius knee systems have shown:

- Fewer physiotherapy sessions¹
- Less use of assistive devices^{1,26}
- Greater muscle function in mid-flexion¹²⁸
- Less need for muscle compensation in sit-to-stand activity³

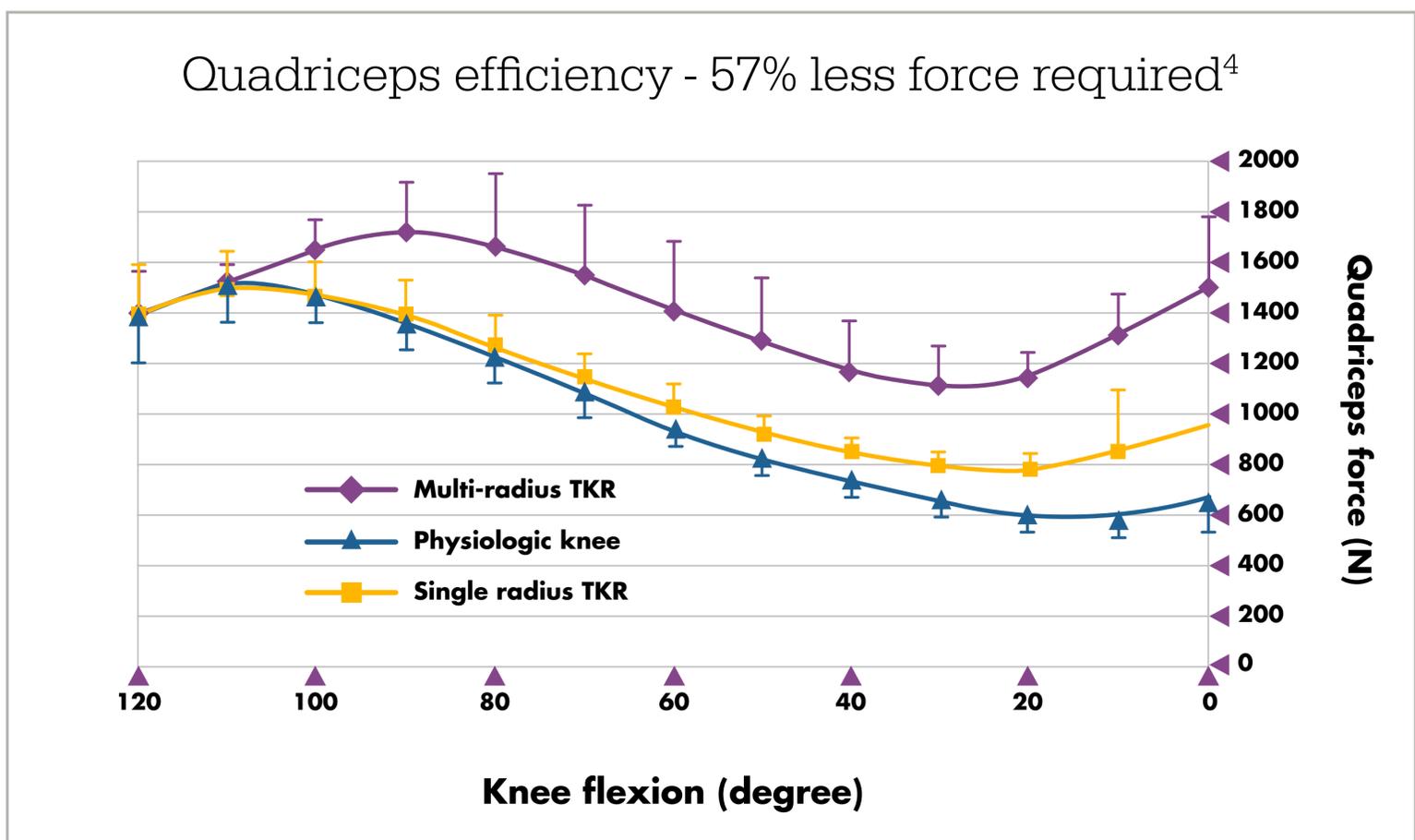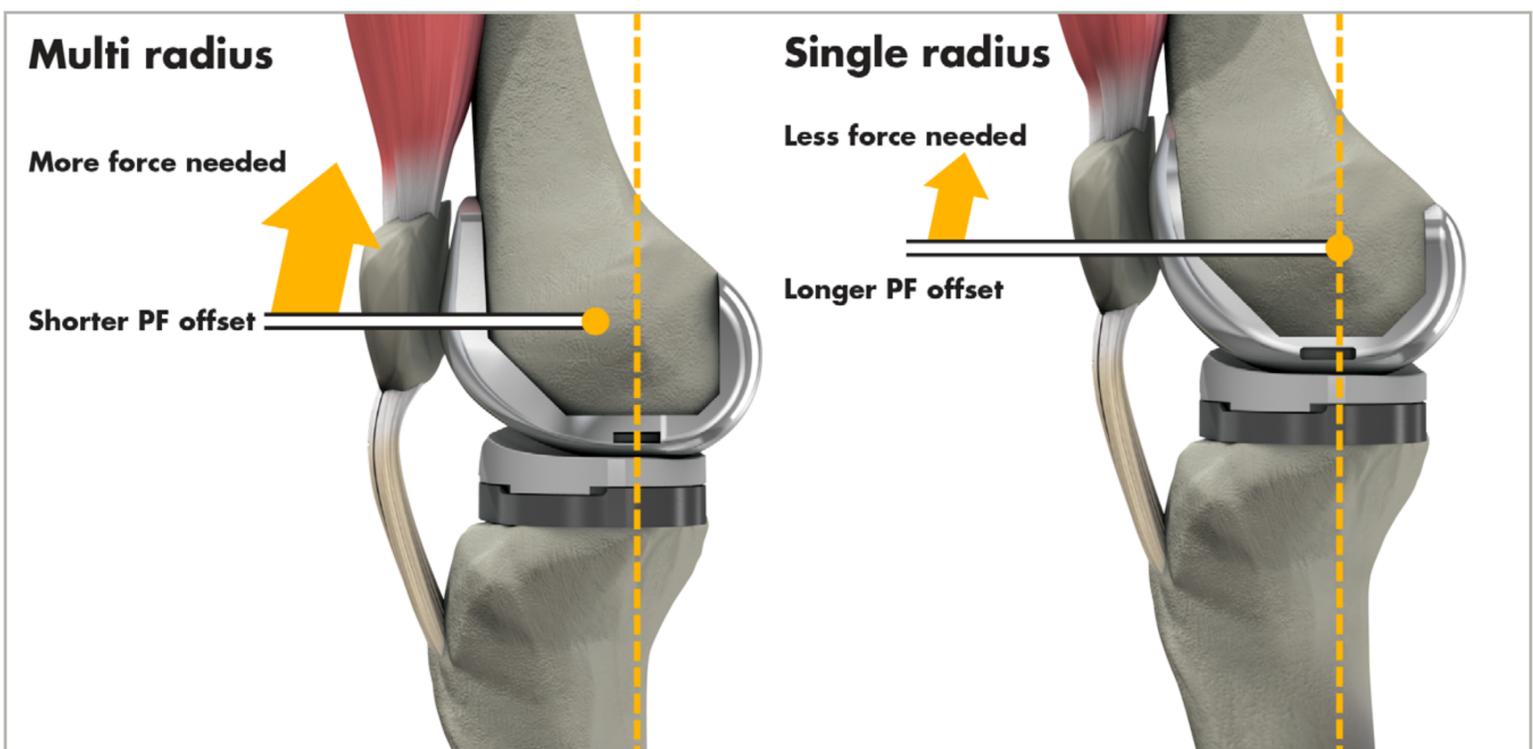



Studies and registries

Multiple studies^{80,111} and joint registries^{5,89} from around the world consistently show a high rate of survivorship with Triathlon.


Source	Survivorship
2019 Australian Orthopaedic Association National Joint Replacement Registry ⁸⁹	96.3% (CR), 94.7% (PS) at 10 years (including infection)
2019 National Joint registry (U.K./Wales) ⁵	96.4% at 13 years (including infection)
Mistry et al. ⁸⁰ 2016	99.0% at 10 years (including failure for all cause)
Scott et al. ¹¹¹ 2019	97.9% at 10 years




Femoral geometry

The single radius is designed to restore the knee's single center of rotation during active flexion.^{21,22} This allows for constant ligament tension and stability in flexion.^{2,26,82,107}

DePuy Synthes Attune⁹⁰

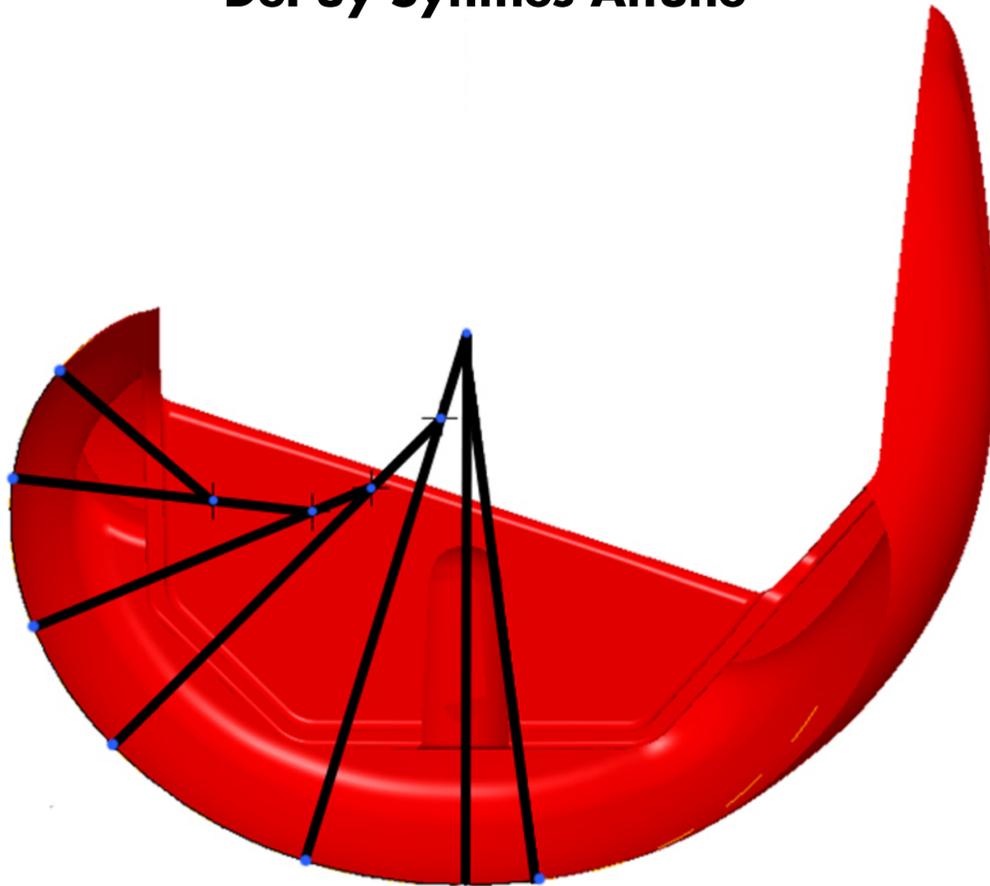

Stryker Triathlon

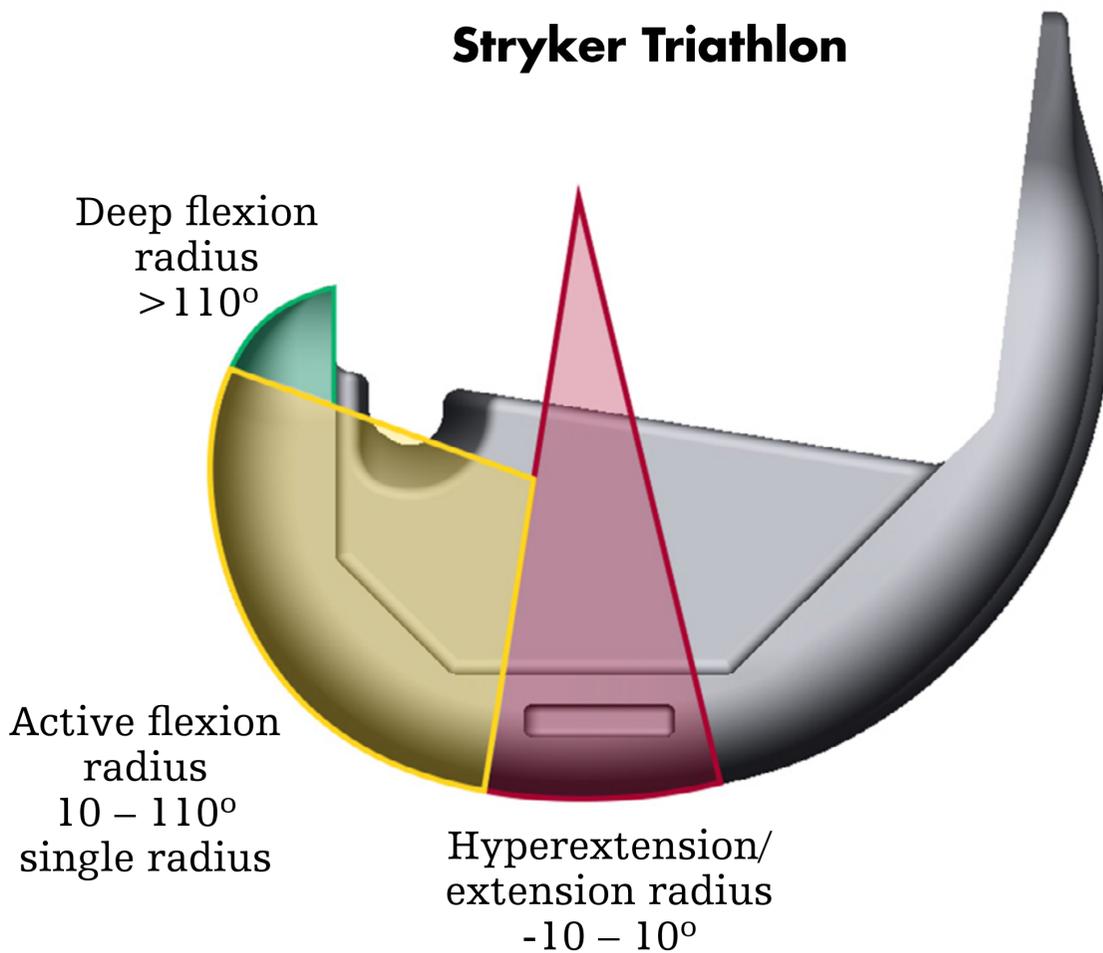

Deep flexion
radius
> 110°

Active flexion
radius
10 – 110°
single radius

Hyperextension/
extension radius
-10 – 10°




Potential contradictions in DePuy materials

Many of the past references that DePuy had used to draw comparisons to Triathlon were simulations and lab tests. Clinical studies on Triathlon and the single radius have been published for years and offer a more clinical perspective of Triathlon patient outcomes.

A closer look at past marketing materials that Depuy previously used reveals some contradictions that should raise some questions about their data.


Stability – which one is “correct”?

Claim 1: This chart from a past Attune brochure suggested that Triathlon allows for more rotation than Attune.⁹⁶

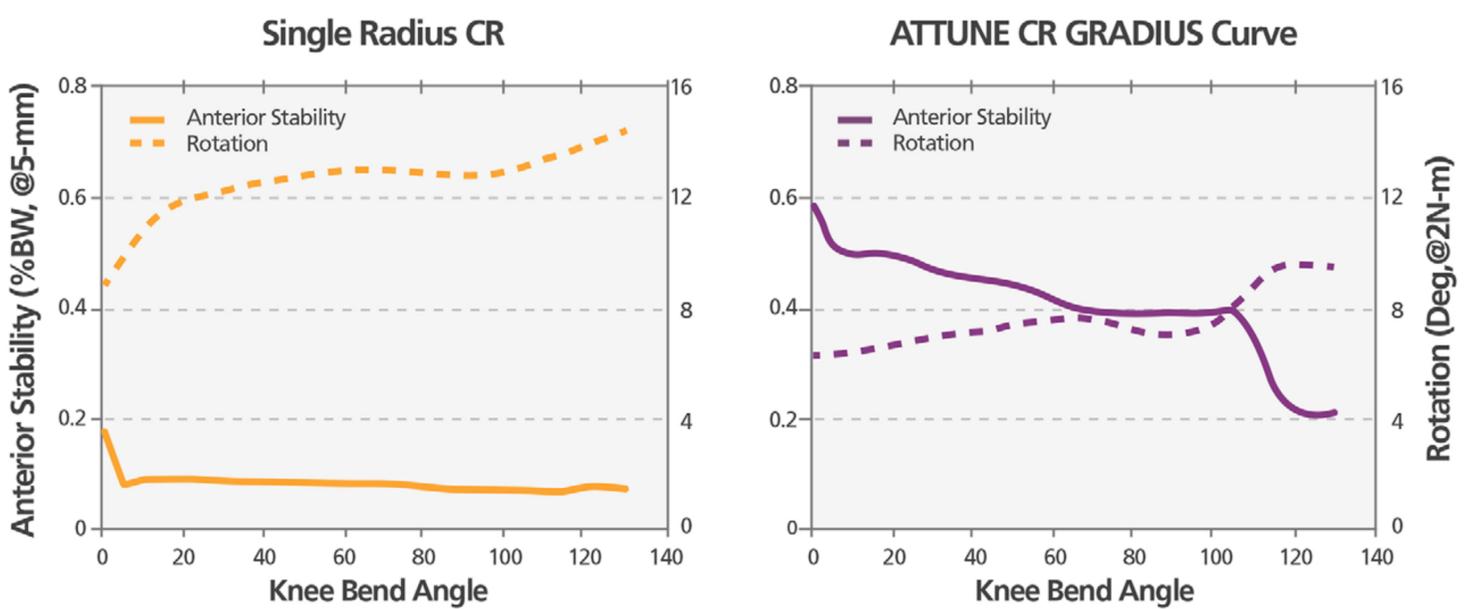

Claim 2: This chart from that same past Attune brochure suggested that Triathlon does not allow for femoral rotation and acts more like a “hinge.”⁹⁶

Allows rollback to mimic a hinge-like device which does not allow for natural femoral rotation

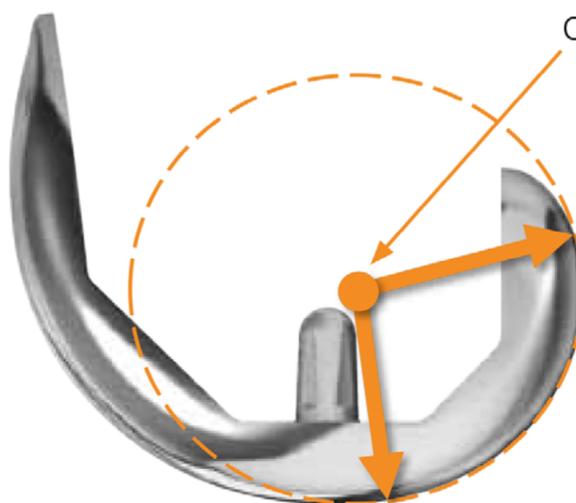

One center of axis


Creates no transition. Similar to a hinged knee




The below content from a past Attune brochure demonstrates inconsistencies in the results of a computer simulator that was used in an attempt to make comparisons to Triathlon.

Wear – which one is “correct”?

Claim 1: This caption suggested that Attune provides a 50% lower wear rate than Sigma.⁹⁶

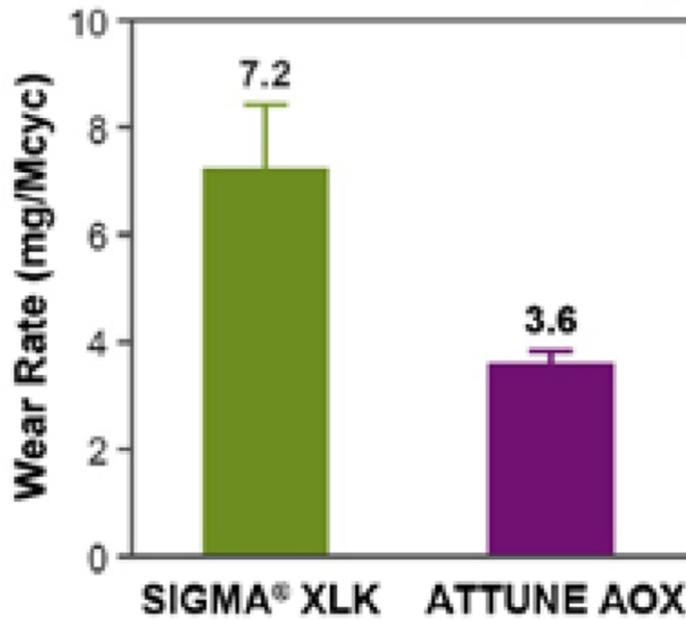

Claim 2: This chart from the same past Attune brochure suggested that Attune has higher wear than Sigma in certain activities.⁹⁶

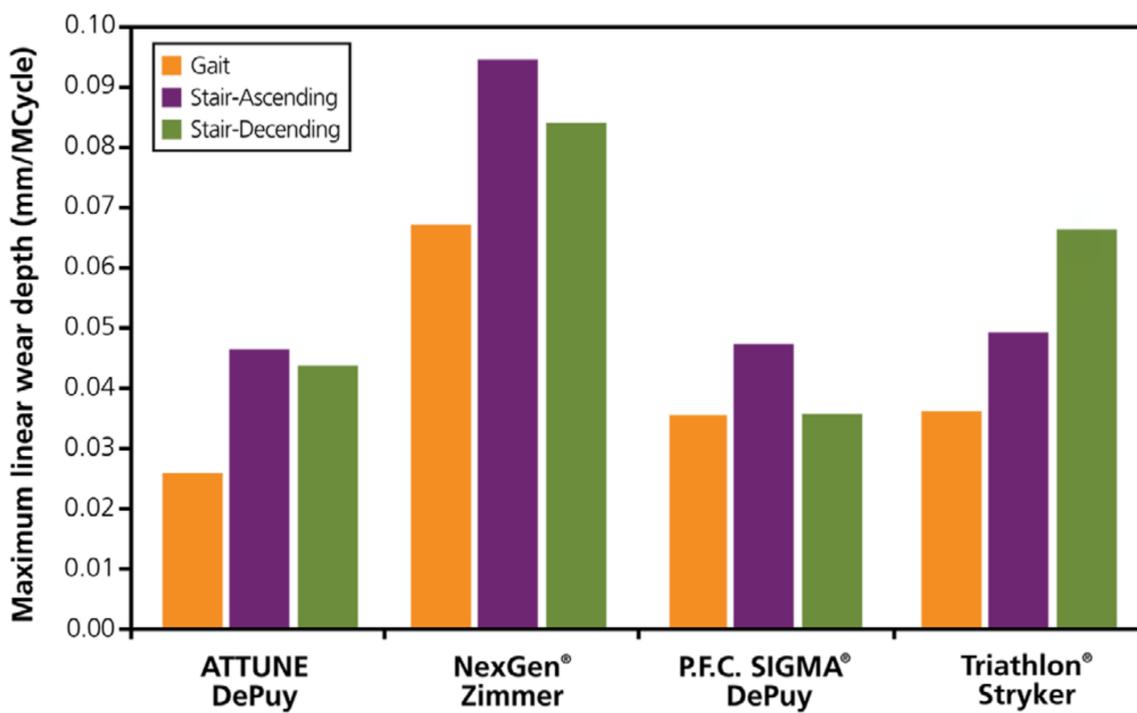



Cam/post – which is “correct”?

Claim 1: This chart from a past Attune brochure suggested that Attune PS cam and spine engage at 87° of flexion.⁶⁰

	DePuy Synthes ATTUNE PS	Zimmer NexGen LPS-Flex	Stryker Triathlon PS
Flexion at engagement (°)	87	90	53
Distance (mm)	4.8	15.1	5.2
Contact velocity (mm/°)	0.10	0.22	0.08

Claim 2: A separate past Attune brochure stated that Attune PS cam and spine engage at 70° of flexion.⁹⁷

The ATTUNE Cam and Spine are designed to engage at 70 degrees of flexion, or just outside of the gait cycle. Engagement may



Stability

- In a quantitative assessment comparing gait between sixteen patients who received a single radius and sixteen patients who received a multi-radius total knee, Triathlon single radius patients experienced gait patterns that mimicked those of healthy individuals.⁸²
- Triathlon's cam/post is designed to engage at approximately 45°, where natural PCL loading occurs.⁹ Attune cam-post engagement occurs after the gait cycle between 70° and 87°. ^{60,97} Note the difference in location of the femoral component between Triathlon and Attune when the cam and post engage.¹³⁰ In a lab test, the Attune component did not engage the post and slid anteriorly.¹³⁰





Deep flexion

- A clinical study showed that Triathlon is designed to accommodate up to 150° of flexion.²⁸ Triathlon's shorter, flared posterior condyles are designed to facilitate the relaxation of the soft tissues to enable deep flexion while maintaining contact area for stability in flexion.^{6,7}
- Triathlon achieves deep flexion with minimal tibial slope. Attune recommends up to 7° of tibial slope for the CR design.⁶² A recent publication showed that in TKA revised for flexion instability, the average pre-revision slope was approximately 7° .¹²⁹



Polyethylene

- A published study showed that wear and mechanical integrity of X3 PS inserts were unaffected by accelerated aging, even after a rigorous stair-climbing test.¹⁶ Furthermore, the fracture toughness of X3 is double the minimum ASTM F648 requirement.¹⁸
- Lab testing showed an 83% wear reduction with X3 compared to AOX.¹²¹ A recent case study was published on a fractured additive antioxidant polyethylene. Analysis of the retrieved insert suggested that fracture was probably a combination of clinical and component factors.⁸¹
- The polyethylene design in this study has some similarities to Attune/AOX.
 - 5.1mm minimum polyethylene thickness (Attune thickness = 5mm⁹⁵)
 - Gross micromotion related to the locking mechanism,⁸¹ which has no peripheral capture (Attune has an uncaptured peripheral locking mechanism⁶⁰)





Size charts

DePuy Attune CR – A/P and M/L dimensions ¹¹²			
Size	Overall A/P	Anterior cortex A/P	Overall M/L
1	47.7	43.7	54.1
2	50.8	46.7	57.2
3N	53.6	49.5	56.8
3	53.9	49.7	60.3
4N	56.6	52.5	59.9
4	57.1	52.7	63.4
5N	59.7	55.6	63.0
5	60.1	55.7	66.5
6N	62.8	58.6	66.1
6	63.4	58.8	69.6
7	66.5	61.8	72.7
8	69.6	64.9	75.8
9	72.9	68.0	78.9
10	76.1	71.1	82.0

DePuy Attune CR – Condyle dimensions ¹¹²			
Size	Posterior thickness	Distal thickness	Condyle length
1	8.0	9.0	30.8
2	8.0	9.0	32.2
3N	8.0	9.0	33.8
3	8.0	9.0	33.8
4N	8.0	9.0	35.3
4	8.0	9.0	35.3
5N	8.0	9.0	37.0
5	8.0	9.0	37.0
6N	8.0	9.0	38.7
6	8.0	9.0	38.7
7	8.0	9.0	40.6
8	8.0	9.0	42.5
9	8.0	9.0	44.5
10	8.0	9.0	46.6

**DePuy Attune PS – A/P and M/L dimensions¹¹²**

Size	Overall A/P	Anterior cortex A/P	Overall M/L
1	48.6	44.7	54.1
2	51.7	47.6	57.2
3N	54.5	50.5	56.8
3	54.8	50.6	60.3
4N	57.6	53.5	59.9
4	58.0	53.7	63.4
5N	61.0	56.5	63.0
5	61.0	56.6	66.5
6N	63.7	59.5	66.1
6	64.2	59.8	69.6
7	67.5	62.8	72.7
8	70.8	65.9	75.8
9	74.0	69.0	78.9
10	77.3	72.1	82.0

DePuy Attune PS – Condyle dimensions¹¹²


Size	Posterior thickness	Distal thickness	Condyle length
1	9.0	9.0	29.4
2	9.0	9.0	30.9
3N	9.0	9.0	32.3
3	9.0	9.0	32.4
4N	9.0	9.0	33.8
4	9.0	9.0	33.9
5N	9.0	9.0	35.5
5	9.0	9.0	35.5
6N	9.0	9.0	37.1
6	9.0	9.0	37.2
7	9.0	9.0	38.9
8	9.0	9.0	40.7
9	9.0	9.0	42.7
10	9.0	9.0	44.7




Femoral geometry

The single radius is designed to restore the knee's single center of rotation during active flexion.^{21,22} This allows for constant ligament tension and stability in flexion.^{2,26,82,107}

DePuy Synthes Sigma⁹⁰

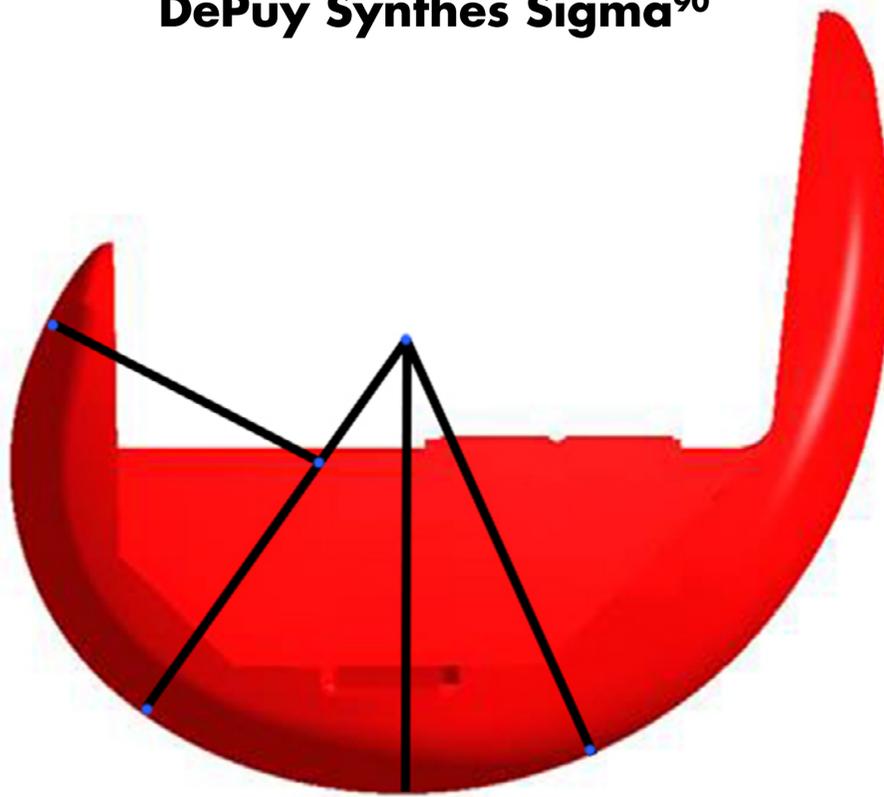

Stryker Triathlon

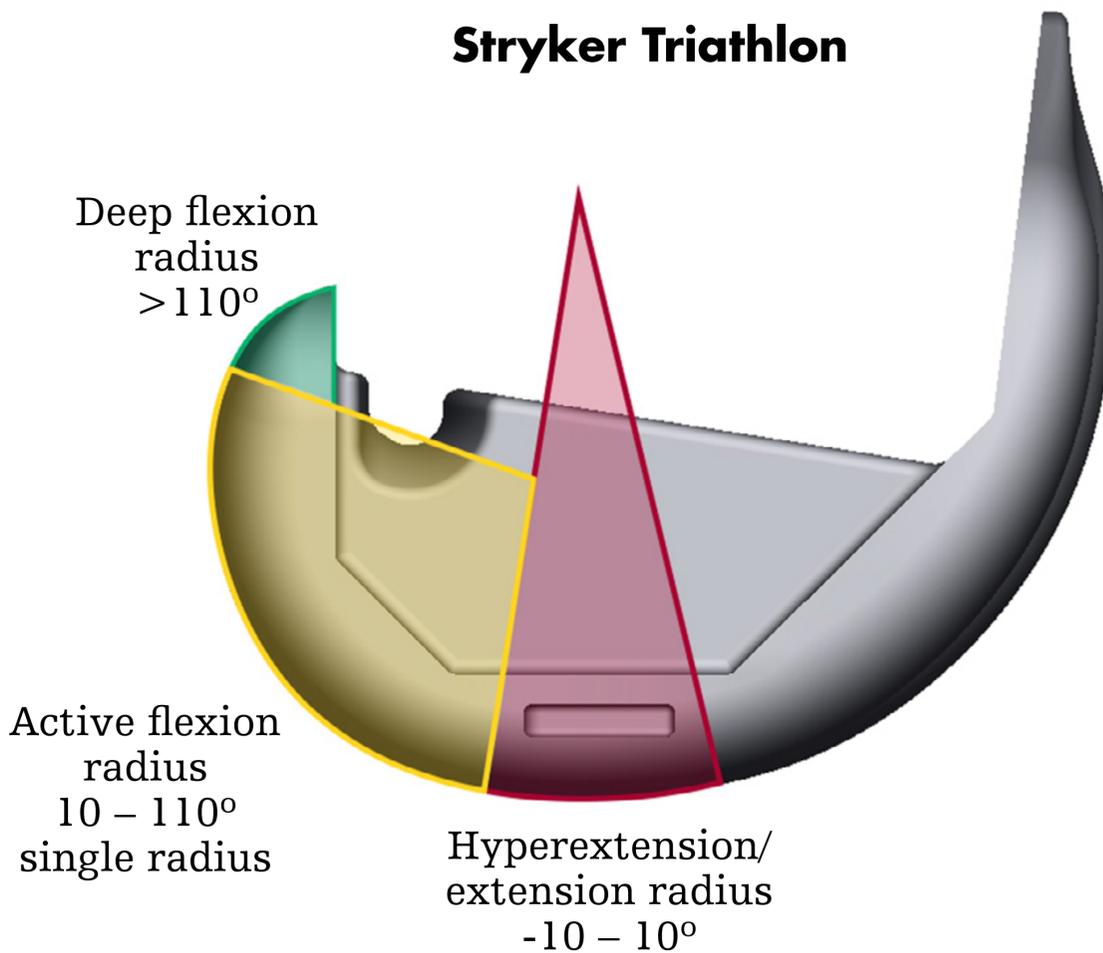




Rotating platform TKA

Triathlon is designed to accommodate up to 20° of internal and external rotation.^{6,7} Triathlon and X3 have also demonstrated 97% reduced wear versus mobile bearing designs.⁴⁸ The full periphery locking rim, locking wire and anti-rotation island have demonstrated less micromotion than other fixed-bearing designs.⁸⁴

Furthermore, a clinical study of twenty prostheses (nine mobile-bearing and eleven fixed-bearing) conducted in Europe showed no kinematic advantages of a Triathlon mobile-bearing versus Triathlon fixed-bearing implant.¹⁰³

Stability

- In a quantitative assessment comparing gait between sixteen patients who received a single radius and sixteen patients who received a multi-radius total knee, Triathlon single radius patients experienced gait patterns that mimicked those of healthy individuals.⁸²
- Triathlon's cam/post is designed to engage at approximately 45°, where natural PCL loading occurs.⁹³ Sigma cam/post engagement occurs after the gait cycle at 70°.⁹⁸
- Research published by DePuy suggests that mid-flexion instability with multi-radius designs may occur due to the change in radius that occurs in mid-flexion.⁹³

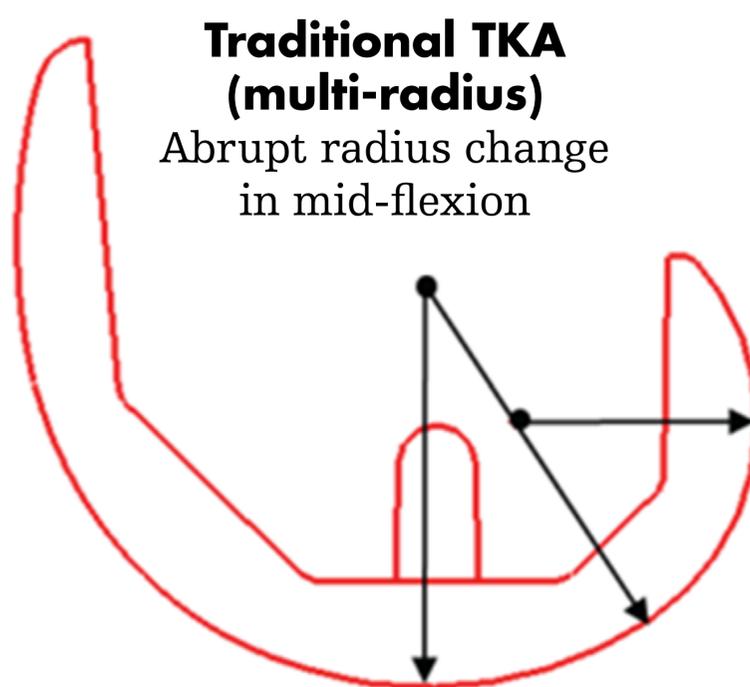


- Two studies have shown a relatively high incidence of patellar clunk with Sigma PS designs.^{99,100}
- Triathlon provides less rotational constraint than Sigma.⁴⁴



Deep flexion

- Triathlon's shorter, flared posterior condyles are designed to facilitate the relaxation of the soft tissues to enable deep flexion while maintaining contact area for stability in flexion.^{6,7} Sigma posterior condyles are longer than Triathlon which may create more tension in the posterior capsule.^{6,90}
- A closer look at Sigma's post design (below) shows how the post is used to lever the femur back as the cam travels up the post.¹⁰¹

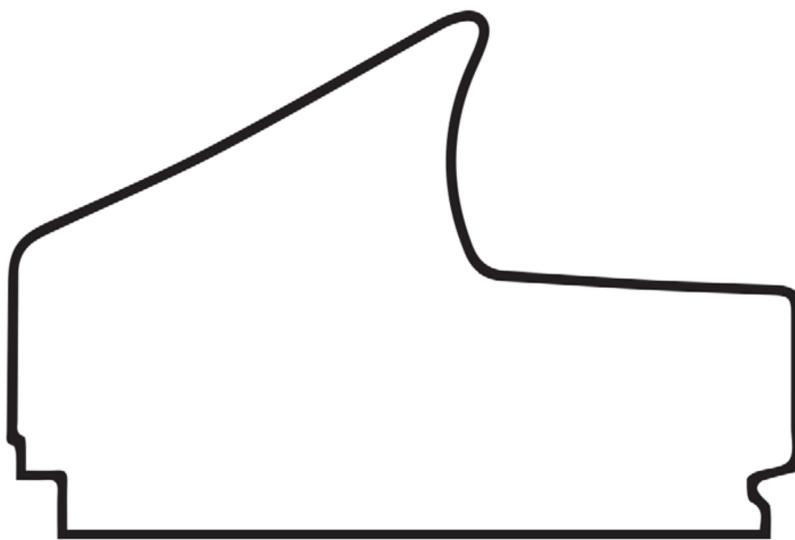



Polyethylene

- A published study showed that wear and mechanical integrity of X3 PS inserts were unaffected by accelerated aging, even after a rigorous stair-climbing test.¹⁶ Furthermore, the fracture toughness of X3 is double the minimum ASTM F648 requirement.¹⁸
- XLK, the polyethylene utilized with DePuy's Sigma system, is a remelted polyethylene. DePuy has claimed that with remelted polyethylene, "The consequence of the loss in crystallinity is a modest loss in mechanical strength and fatigue resistance."¹⁰²



Size charts

DePuy Sigma CR – A/P and M/L dimensions¹¹²

Size	Overall A/P	Anterior cortex A/P	Overall M/L
1.5	52.7	47.3	56.9
2	55.7	50.4	60.0
2.5	58.8	53.2	63.0
3	61.7	55.8	66.0
4N	65.6	60.0	66.0
4	65.0	60.0	71.0
5	69.8	64.0	73.0
6	75.4	69.7	76.5

DePuy Sigma CR – Condyle dimensions¹¹²

Size	Posterior thickness	Distal thickness	Condyle length
1.5	7.7	9.0	32.3
2	7.6	9.0	33.6
2.5	7.7	9.0	35.0
3	7.7	9.0	36.3
4N	7.7	9.0	38.1
4	7.7	9.0	37.3
5	7.6	9.0	40.0
6	8.5	10.0	42.0

DePuy Sigma PS – A/P and M/L dimensions¹¹²

Size	Overall A/P	Anterior cortex A/P	Overall M/L
1.5	52.7	47.3	56.9
2	55.7	50.4	60.0
2.5	58.8	53.2	63.0
3	61.7	55.8	66.0
4N	65.6	60.0	66.0
4	65.0	60.0	71.0
5	69.8	64.0	73.0
6	75.4	69.7	76.5



DePuy Sigma PS – Condyle dimensions¹¹²

Size	Posterior thickness	Distal thickness	Condyle length
1.5	7.7	9.0	32.9
2	7.6	9.0	34.3
2.5	7.7	9.0	35.4
3	7.7	9.0	38.2
4N	7.6	9.0	40.2
4	7.6	9.0	40.2
5	7.6	9.0	41.8
6	8.5	10.0	43.0

DePuy Sigma CR 150 – A/P and M/L dimensions¹¹²

Size	Overall A/P	Anterior cortex A/P	Overall M/L
1.5	52.7	47.3	56.9
2	55.7	50.4	60.0
2.5	58.8	53.2	63.0
3	61.7	55.8	66.0
4N	65.6	60.0	66.0
4	65.0	60.0	71.0
5	69.8	64.0	73.0
6	75.4	69.7	76.5


DePuy Sigma CR 150 – Condyle dimensions¹¹²

Size	Posterior thickness	Distal thickness	Condyle length
1.5	9.7	9.0	32.2
2	9.6	9.0	34.2
2.5	9.7	9.0	34.8
3	9.7	9.0	37.4
4N	9.6	9.0	39.8
4	9.6	9.0	39.8
5	9.6	9.0	41.2
6	10.5	10.0	42.4



Stability

- In a quantitative assessment comparing gait between sixteen patients who received a single radius and sixteen patients who received a multi-radius total knee, Triathlon single radius patients experienced gait patterns that mimicked those of healthy individuals.⁸²
- Fluoroscopic data of Triathlon patients showed that medial pivot is possible without features that force a medial pivot.¹⁰⁷




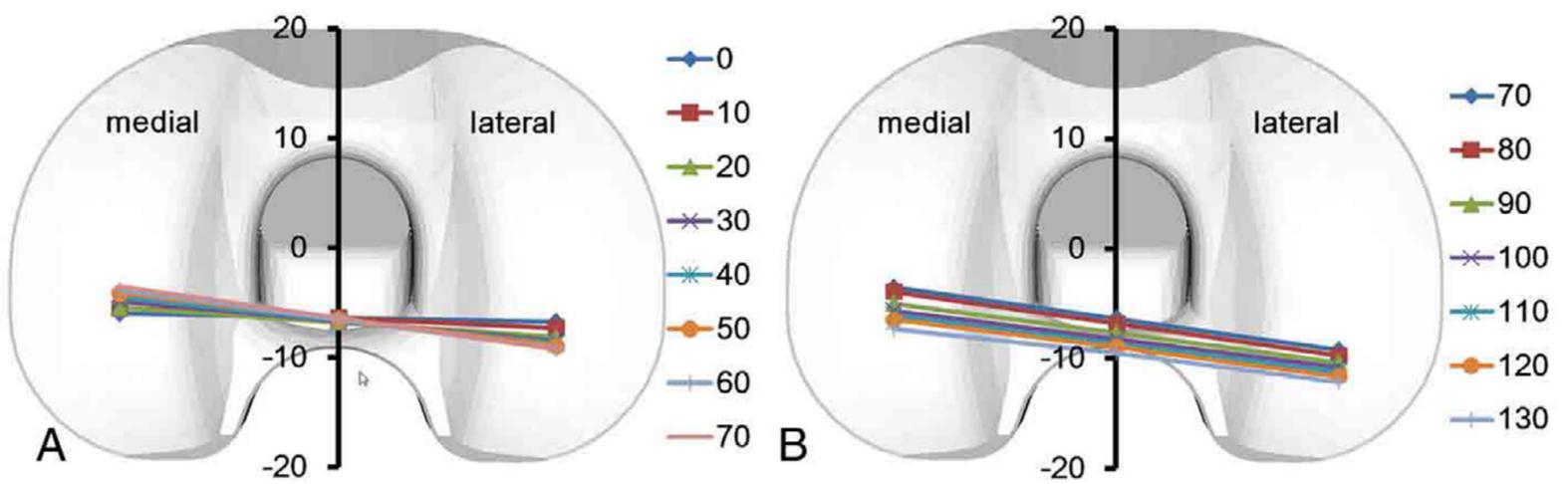




Deep flexion

A clinical study showed that Triathlon is designed to accommodate up to 150° of flexion.²⁸ Triathlon's shorter, flared posterior condyles are designed to facilitate the relaxation of the soft tissues to enable deep flexion while maintaining contact area for stability in flexion.^{6,7}

LifeMod, the simulator used to design Journey 2, showed approximately 15° less flexion with Journey 2 compared to original Journey.^{92,109}

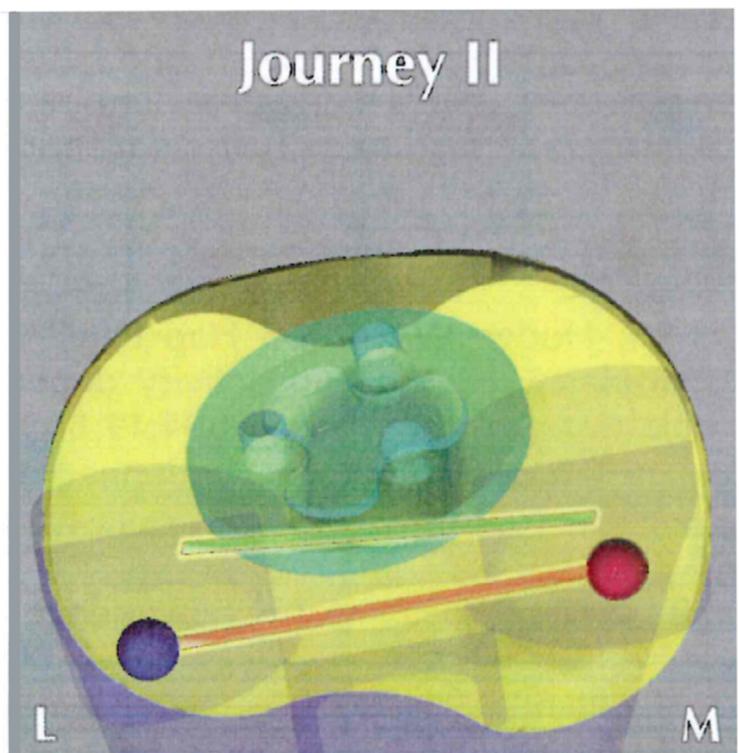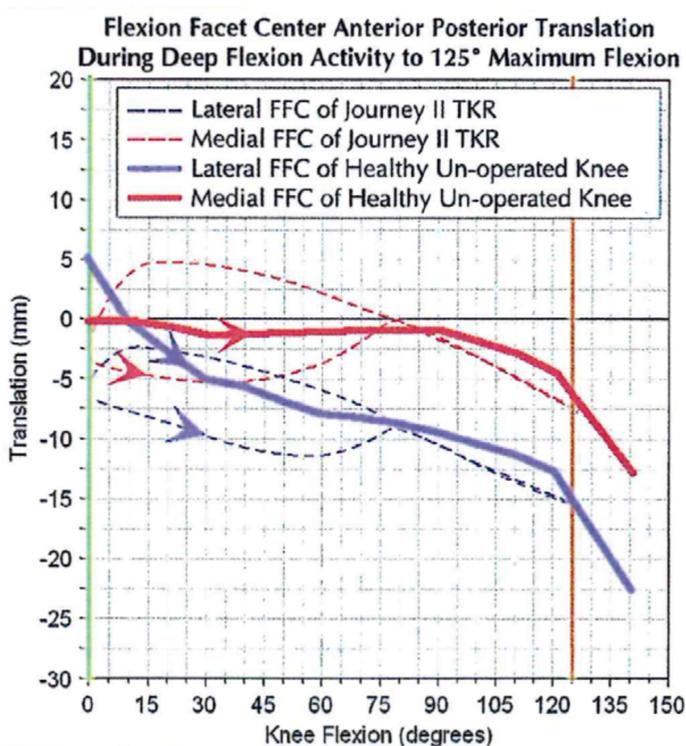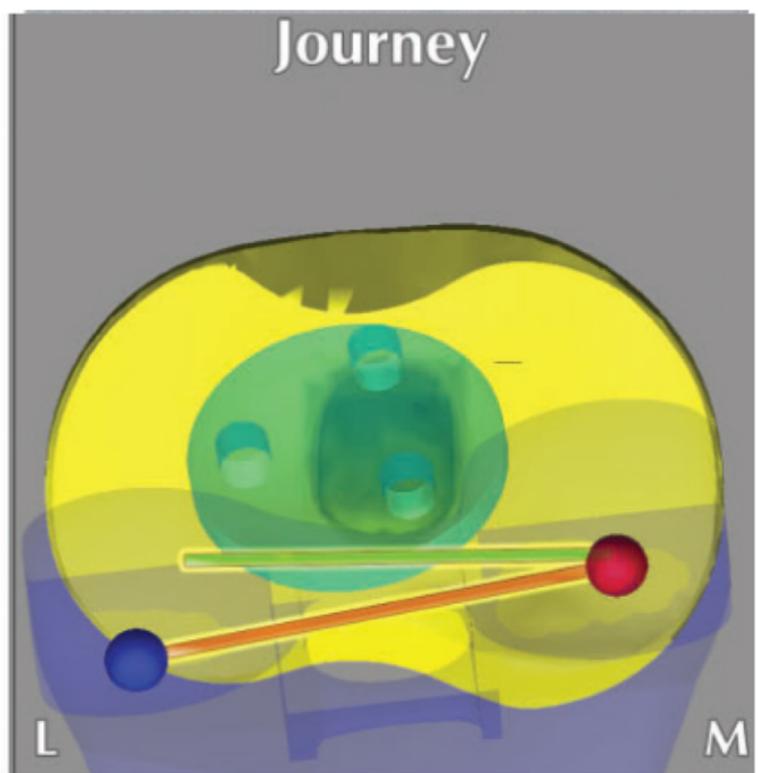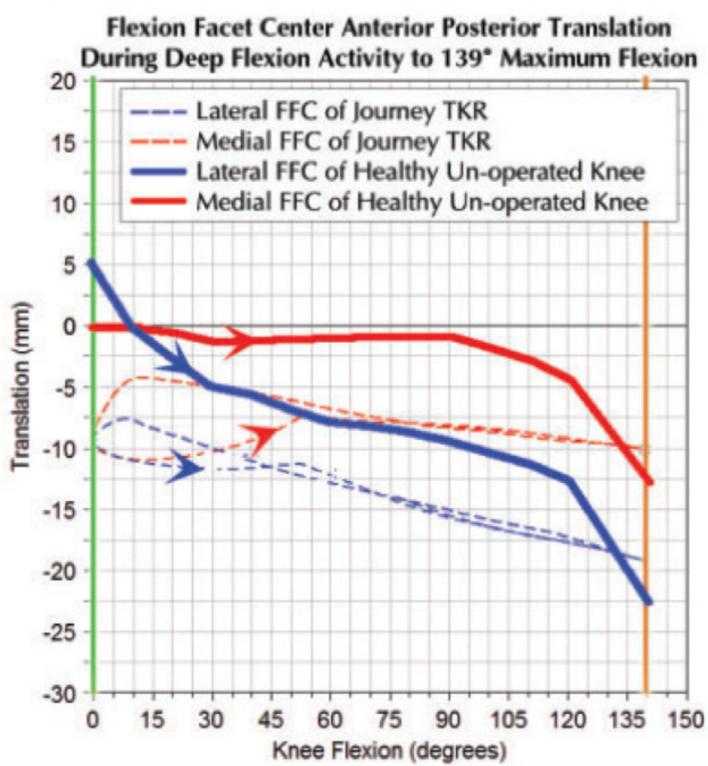



Polyethylene

A study showed that wear and mechanical integrity of X3 PS inserts were unaffected by accelerated aging, even after a rigorous stair-climbing test.¹⁶ Furthermore, the fracture toughness of X3 is double the minimum ASTM F648 requirement.¹⁸



Size charts

Smith & Nephew Journey 2 BCS – femoral component¹¹⁴

Size	M/L (mm)	A/P	Box width
1	59	51.7	16.5
2	60	53.7	16.5
3	61.5	56.7	16.5
4	64.5	59.7	16.5
5	67.5	62.7	16.5
6	70.5	65.7	16.5
7	73.5	68.8	16.5
8	76	71.8	16.5
9	80	75.8	16.5
10	82	79.8	16.5

Smith & Nephew Journey 2 BCS – tibial component¹¹⁴


Size	M/L (mm)	A/P
1	60	42
2	64	45
3	68	48
4	71	50
5	74	52
6	77	54
7	81	56
8	85	59




Femoral geometry

The single radius is designed to restore the knee's single center of rotation during active flexion.^{21,22} This allows for constant ligament tension and stability in flexion.^{2,26,82,107}

Smith & Nephew Gen II/Legion⁹⁰

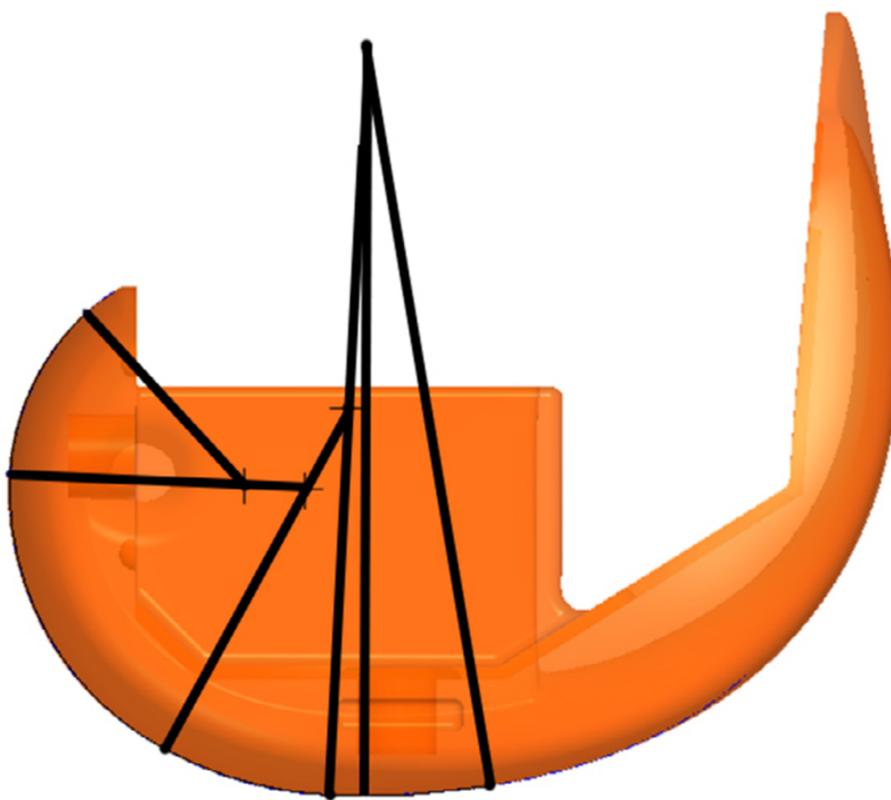


Stryker Triathlon

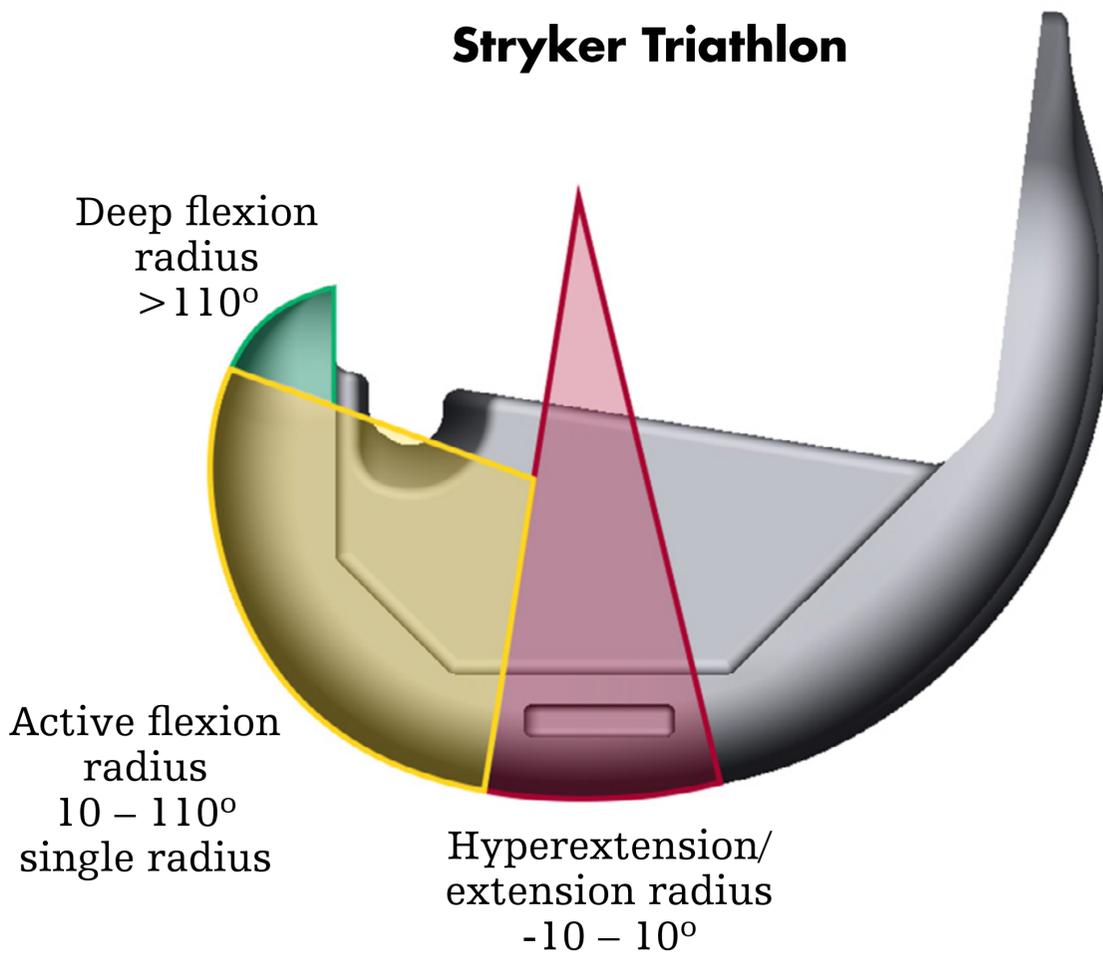



Stability

- In a quantitative assessment comparing gait between sixteen patients who received a single radius and sixteen patients who received a multi-radius total knee, Triathlon single radius patients experienced gait patterns that mimicked those of healthy individuals.⁸²
- Triathlon's cam/post is designed to engage at approximately 45°, where natural PCL loading occurs.⁹³ Gen II/Legion cam-post engagement occurs at 60-70°. ¹⁰⁴ A lab study showed that compared to Triathlon, Gen II demonstrated higher stress on the post when it engaged with the cam.¹⁰⁸ A retrieval study showed that the Gen II post demonstrated significant posterior wear damage.¹⁰⁵

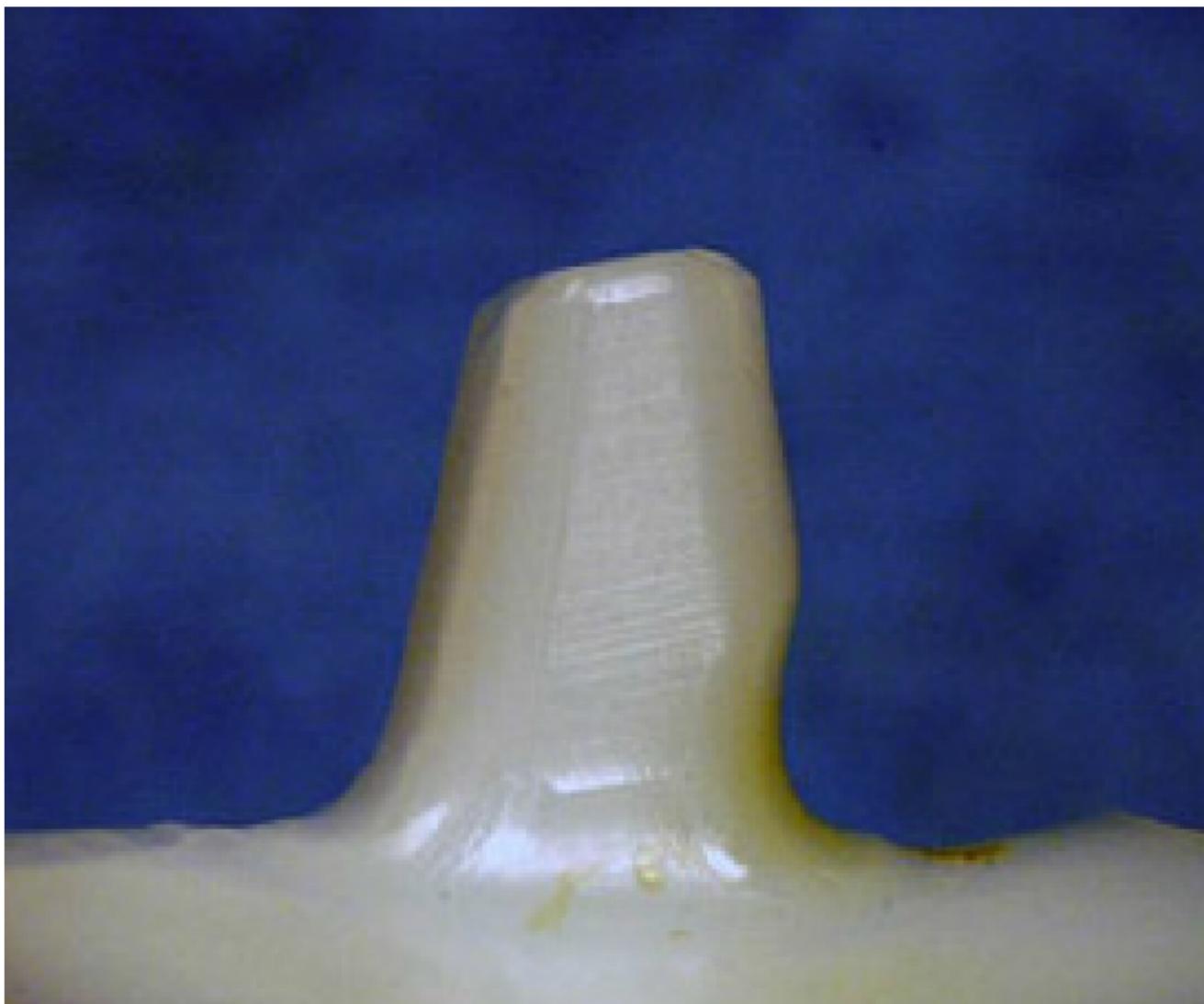



Deep flexion

A clinical study showed that Triathlon is designed to accommodate up to 150° of flexion.²⁸ Triathlon's shorter, flared posterior condyles are designed to facilitate the relaxation of the soft tissues to enable deep flexion while maintaining contact area for stability in flexion.^{6,7} Gen II/Legion posterior condyles are longer than Triathlon's, which may create more tension in the posterior capsule.^{6,90}

Triathlon can achieve deep flexion with minimal slope. Gen II/Legion recommends 7° slope.¹¹³ One publication showed that in TKA revised for flexion instability, the average pre-revision slope was approximately 7°.¹²⁹



Polyethylene

A published study showed that wear and mechanical integrity of X3 PS inserts were unaffected by accelerated aging, even after a rigorous stair-climbing test.¹⁶ Furthermore, the fracture toughness of X3 is double the minimum ASTM F648 requirement.¹⁸



Size charts

Smith & Nephew Genesis II – femoral component¹¹³

Size	M/L (mm)	A/P
1	55	47
2	58	50.5
3	62	54.5
4	66	58.5
5	70	62
6	73	65.5
7	77	69.5
8	80	75

Smith & Nephew Genesis II – tibial component¹¹³


Size	M/L (mm)	A/P
1	60	42
2	64	45
3	68	48
4	71	50
5	74	52
6	77	54
7	81	56
8	85	59




Femoral geometry

The single radius is designed to restore the knee's single center of rotation during active flexion.^{21,22} This allows for constant ligament tension and stability in flexion.^{2,26,82,107}

Zimmer Biomet Vanguard⁹⁰

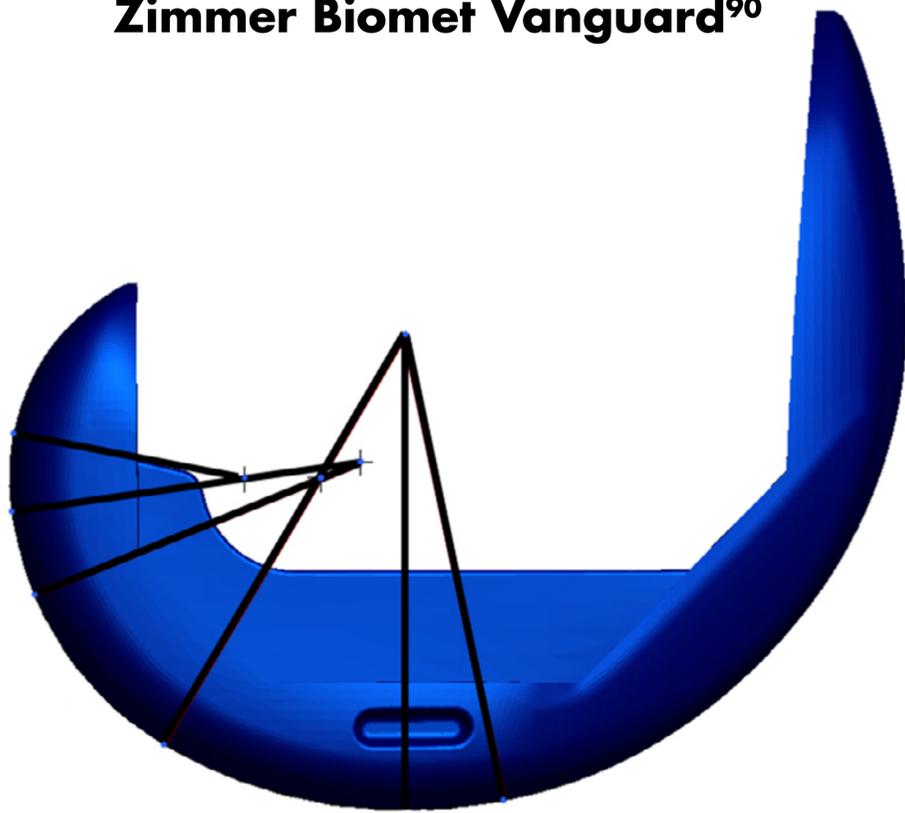


Stryker Triathlon

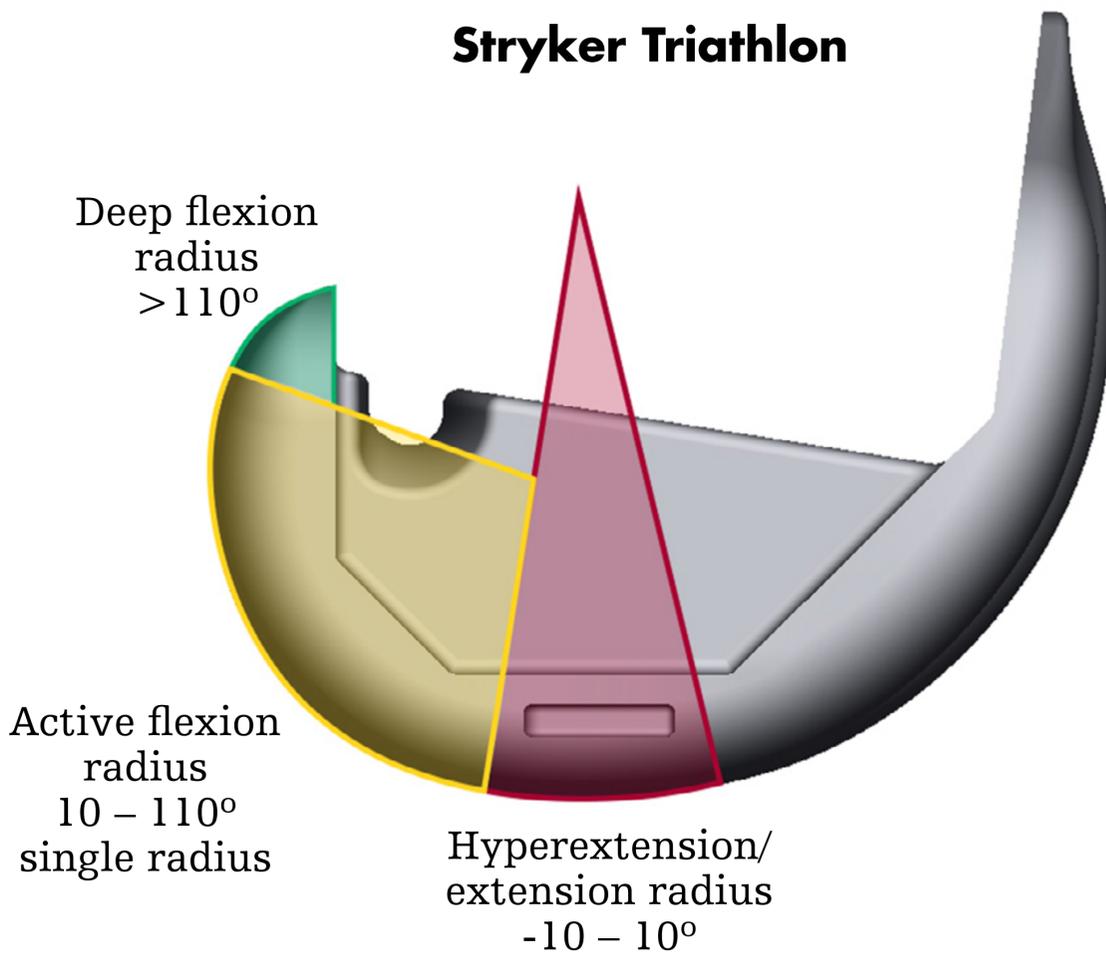



Stability

In a quantitative assessment comparing gait between sixteen patients who received a single radius and sixteen patients who received a multi-radius total knee, Triathlon single radius patients experienced more natural gait than patients with a Vanguard multi-radius design.⁸²

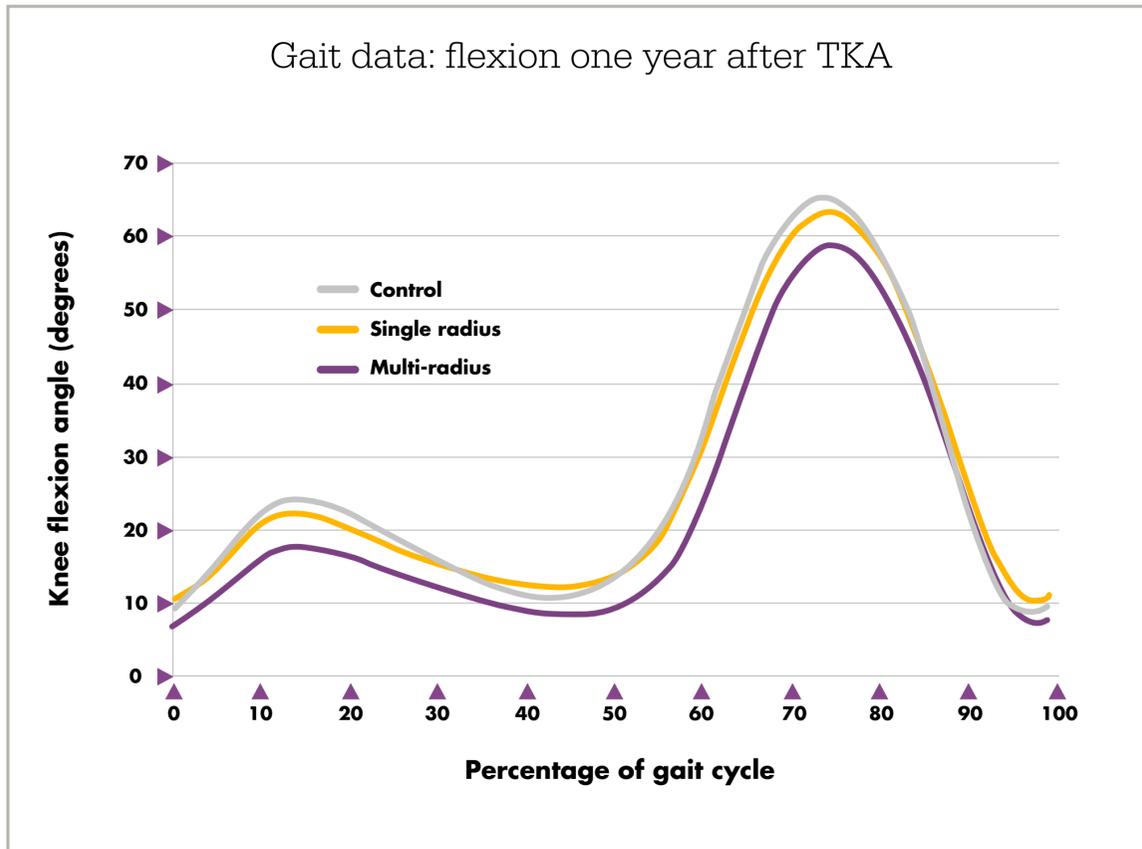

- In the Vanguard multi-radius design, there are abrupt changes in the femoral geometry during active flexion when gait occurs.^{82,90} A lab study showed that compared to Triathlon, Vanguard demonstrated higher stress on the post when it engaged with the cam.¹⁰⁸ Cam-post engagement was found to occur after 70° of flexion, later than Triathlon.¹⁰⁸
- Triathlon PS femurs articulate with constrained (TS) inserts which is an intraoperative option for increased stability in a primary TKA. Vanguard PS femurs articulate with their PS+ inserts but offer less internal/external rotation.⁹¹

PS femur/insert options	Triathlon	Vanguard ⁹¹
Standard PS	+/- 20° I/E rotation	+/- 15° I/E rotation
Constrained	+/- 7° I/E rotation	+/- 2° I/E rotation



Deep flexion

A clinical study showed that Triathlon is designed to accommodate up to 150° of flexion.²⁸ Triathlon's shorter, flared posterior condyles are designed to facilitate the relaxation of the soft tissues to enable deep flexion while maintaining contact area for stability in flexion.^{6,7} Vanguard posterior condyles are longer than Triathlon's, which may create more tension in the posterior capsule.^{6,90}

Polyethylene

- A published study showed that wear and mechanical integrity of X3 PS inserts were unaffected by accelerated aging, even after a rigorous stair-climbing test.¹⁶ Furthermore, the fracture toughness of X3 is double the minimum ASTM F648 requirement.¹⁸
- A recent case study was published on a fractured Biomet E1 insert with a Vitamin E additive.⁸¹ Fracture was probably a combination of clinical and component factors. Component factors included:
 - 5.1mm minimum polyethylene thickness
 - Gross micromotion related to the locking mechanism, which has no peripheral capture
 - Diminished material properties





Size charts

Biomet Vanguard – femoral component^{124,125}

Size	M/L (mm)	A/P
55	59	55
57.5	61	57
60	64	59
62.5	66	61
65	68	63
67.5	71	66
70	73	68
72.5	75	72
75	78	74
80	83	77

Biomet Vanguard – tibial component^{124,125}


Size	M/L (mm)	A/P
59	59	38
63	63	41
67	67	43
71	71	46
75	75	48
79	79	51
83	83	53
87	87	56
91	91	58




Femoral geometry

The single radius is designed to restore the knee's single center of rotation during active flexion.^{21,22} This allows for constant ligament tension and stability in flexion.^{2,26,82,107}

Zimmer Biomet NexGen LPS-Flex⁹⁰

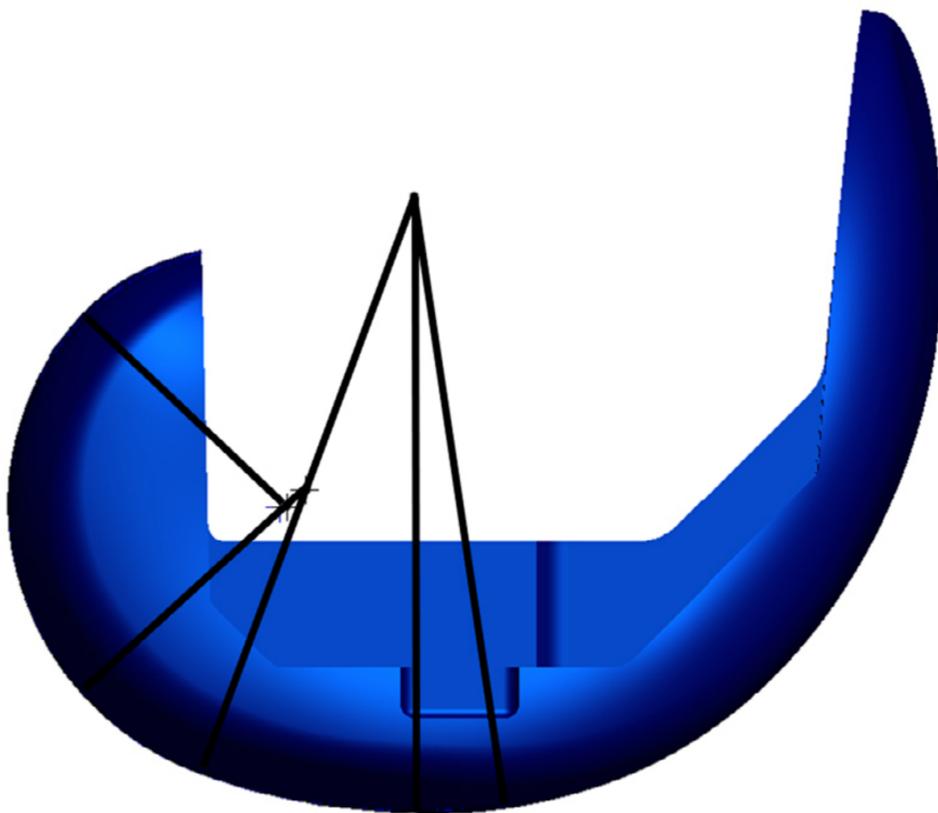

Stryker Triathlon

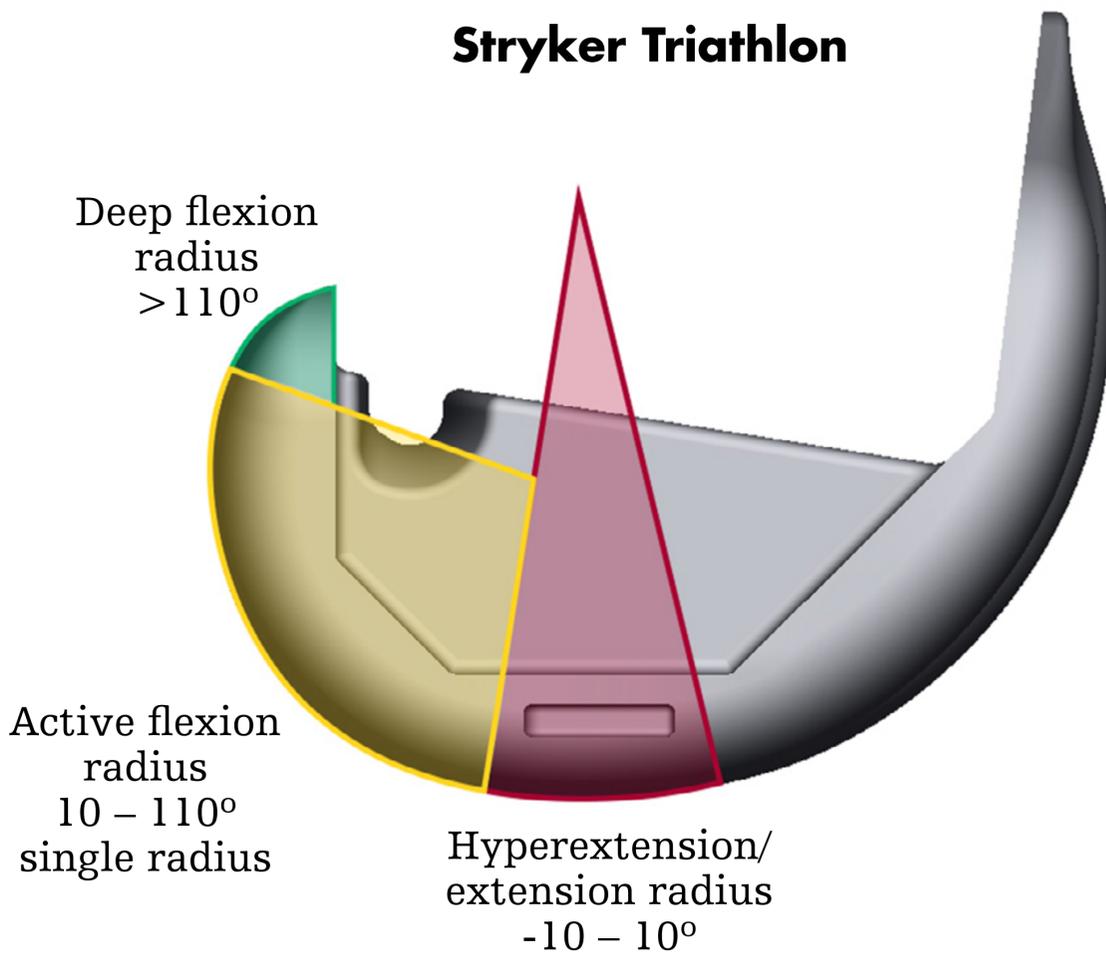



The value of Zimmer “premium” technologies

Triathlon is a modern knee backed by clinical data.^{1,3,9,17,26} Other orthopaedics companies have historically positioned products as premium technology, yet the clinical and/or lab data has not always shown an advantage compared to Triathlon. While some data has shown a potential advantage, the data below has shown either no difference or even decreased performance.⁷³⁻⁷⁵

“Premium” technology: Gender solutions

Promoted benefit: “This allows surgeons to address the female population with unprecedented accuracy.”⁷⁰

Published results: No statistical difference in range of motion, pain, or satisfaction between the gender-specific and non gender-specific designs.⁷⁵

“Premium” technology: “Flex” knee

Promoted benefit: Designed to accommodate resumption of high-flexion daily activities.⁷¹

Published results: The Zimmer high-flex design demonstrated higher femoral loosening than conventional designs.⁷³⁻⁷⁴



Single vs. multi-radius designs in mid-flexion

Zimmer Biomet NexGen is a multi-radius knee. Competitors such as Zimmer may cite a Stryker-funded study, “The Kinematics and Stability of Single-Radius Versus Multi-Radius Femoral Components Related to Mid-Range Instability after TKA”, which purportedly showed no difference between single and multi-radius designs in mid-flexion.

Response

Stryker funded a cadaver lab study to further examine the potential benefits of a single radius (Triathlon) compared to a multi-radius (Kinemax) TKA design. After the experiment was conducted, the authors acknowledged that a limitation of the study was that the loads applied were not typical of loads seen in mid-flexion activities like descending stairs. These clinically relevant loads could not be applied due to the frailty of the specimens.¹²⁷


Furthermore, a clinical study involving actual Triathlon and Kinemax patients did show a difference in muscle function during a mid-flexion activity.¹²⁸

Cadaver studies have limitations and should be compared to actual clinical results when possible. Numerous studies, both funded and non-funded by Stryker, have validated the potential benefits offered by Triathlon and the single radius design.^{1,3,26,82,111}



Stability

- In a quantitative assessment comparing gait between sixteen patients who received a single radius and sixteen patients who received a multi-radius total knee, Triathlon single radius patients experienced gait patterns that mimicked those of healthy individuals.⁸²
- Fluoroscopic data of Triathlon patients showed that medial pivot is possible without features that force a medial pivot.¹⁰⁷
- Triathlon's cam/post is designed to engage at approximately 45°, where natural PCL loading occurs.⁹ NexGen cam/post engagement occurs after the gait cycle at 90°. ⁶⁰
- In a quantitative study comparing the rotational constraint characteristics of four commercially available posterior-stabilized implants, Triathlon demonstrated less rotational constraint than NexGen.⁴⁴
- A retrieval study showed that the NexGen post demonstrated significant anterior post damage.¹⁰⁵ Note that the Zimmer recommends 7° slope with NexGen.¹³³ What impact could this have on the post in extension?

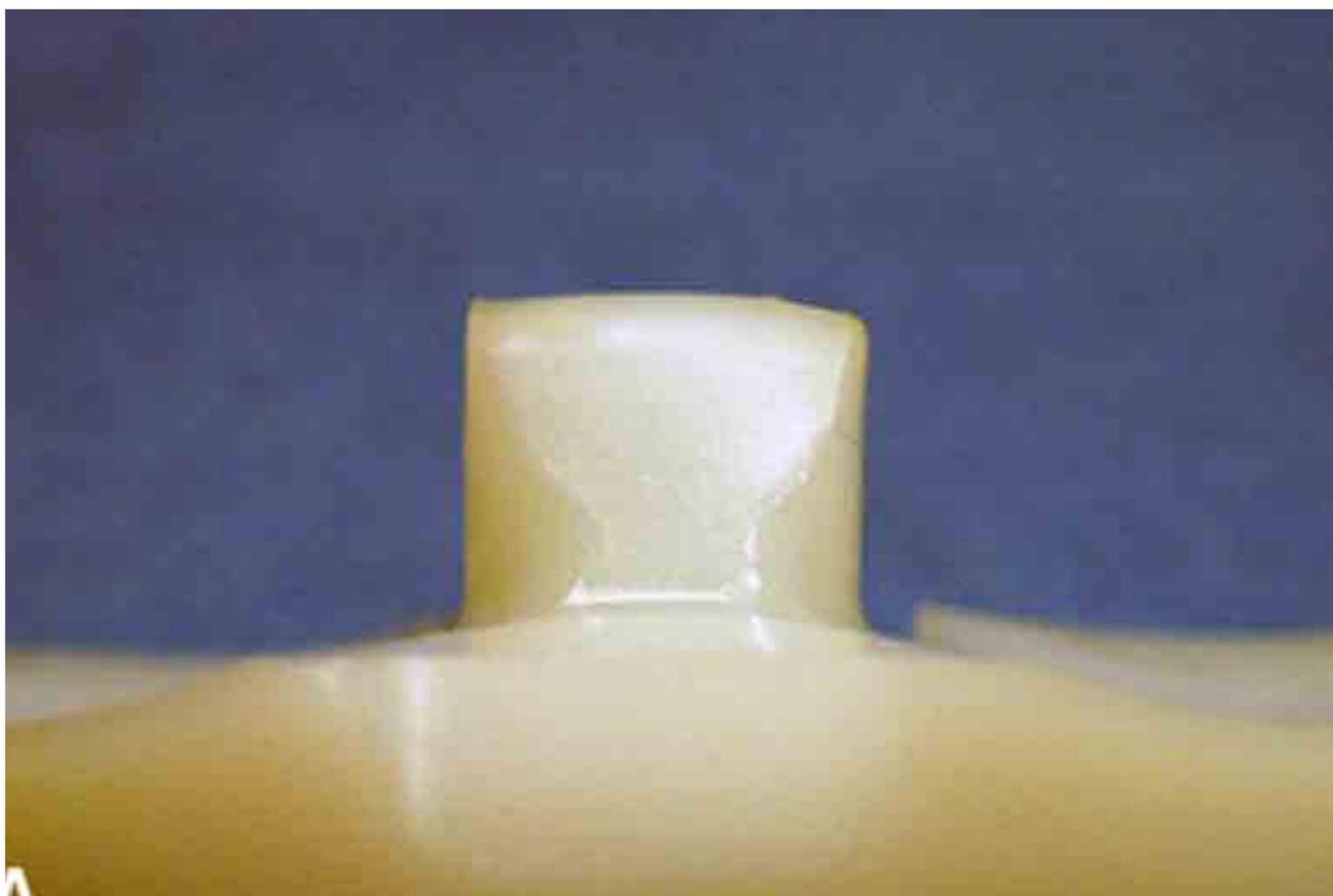




Deep flexion

Triathlon can achieve deep flexion without excessive tibial slope. NexGen calls for 7° of slope.¹³³ A publication showed that in TKA revised for flexion instability, the average pre-revision slope was approximately 7°.¹²⁹

A closer look at NexGen's post design shows how the post is used to lever the femur back as the cam travels up the post.

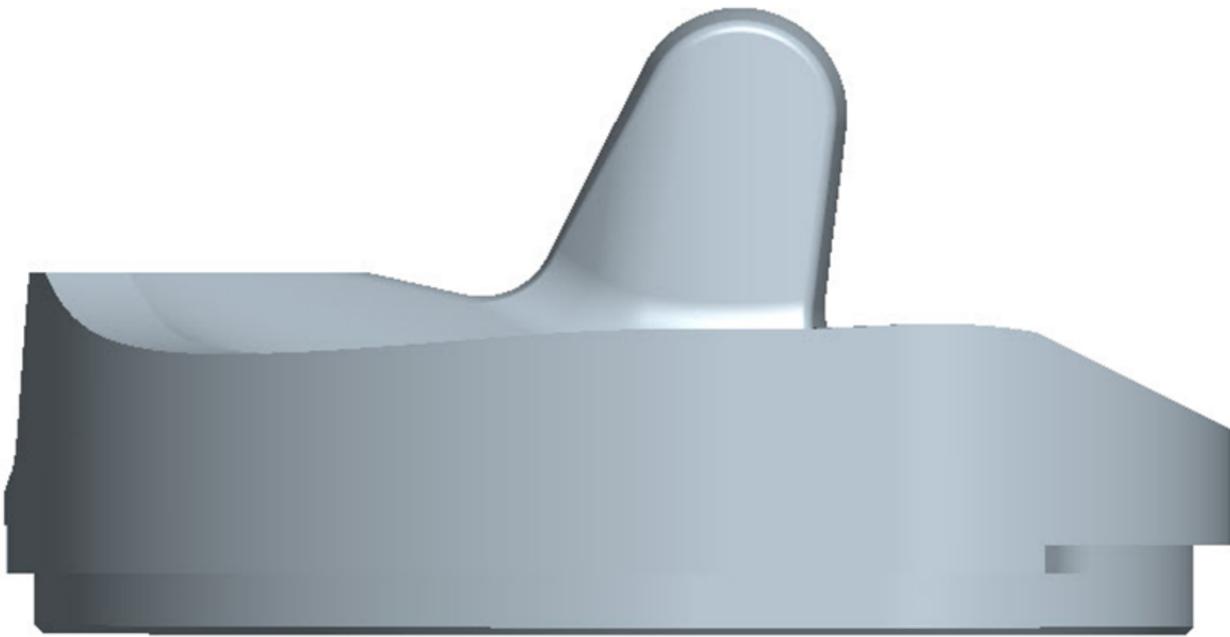



Polyethylene

- A published study showed that wear and mechanical integrity of X3 PS inserts were unaffected by accelerated aging, even after a rigorous stair-climbing test.¹⁶ Furthermore, the fracture toughness of X3 is double the minimum ASTM F648 requirement.¹⁸
- Prolong, the polyethylene used with the NexGen system, is a remelted polyethylene. Zimmer has claimed that the remelting process reduces strength.¹¹⁰



Size charts

Zimmer NexGen – CR Flex femoral component¹¹⁶

Size	M/L (mm)	A/P (standard)	A/P (minus size)
B	58	48.5	N/A
C	60	53.5	51.5
D	64	57.5	55.5
E	68	61.5	59.5
F	72	65.5	63.5
G	76.5	70.5	68.5

Zimmer NexGen – LPS Flex femoral component¹¹⁷


Size	M/L (mm)	A/P
A	54	46.5
B	58	50.3
C	60	54.5
D	64	58.6
E	68	62.5
F	72	66.5
G	76.5	71.6




Femoral geometry

The single radius is designed to restore the knee's single center of rotation during active flexion.^{21,22} This allows for constant ligament tension and stability in flexion.^{2,26,82,107}

Zimmer Biomet Persona⁹⁰

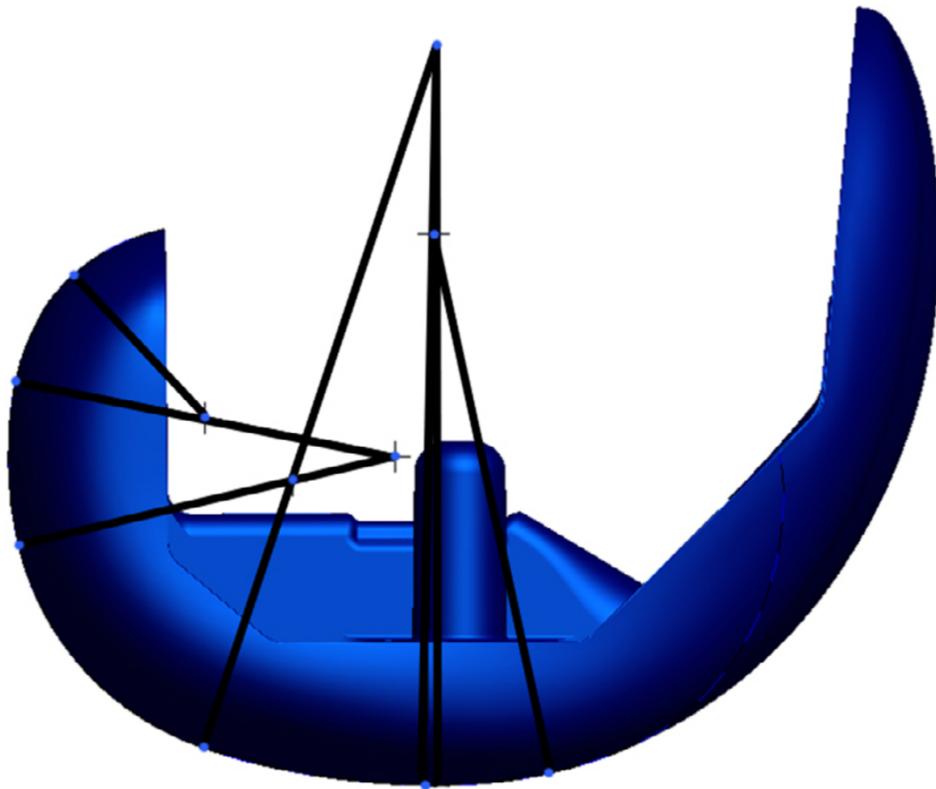


Stryker Triathlon

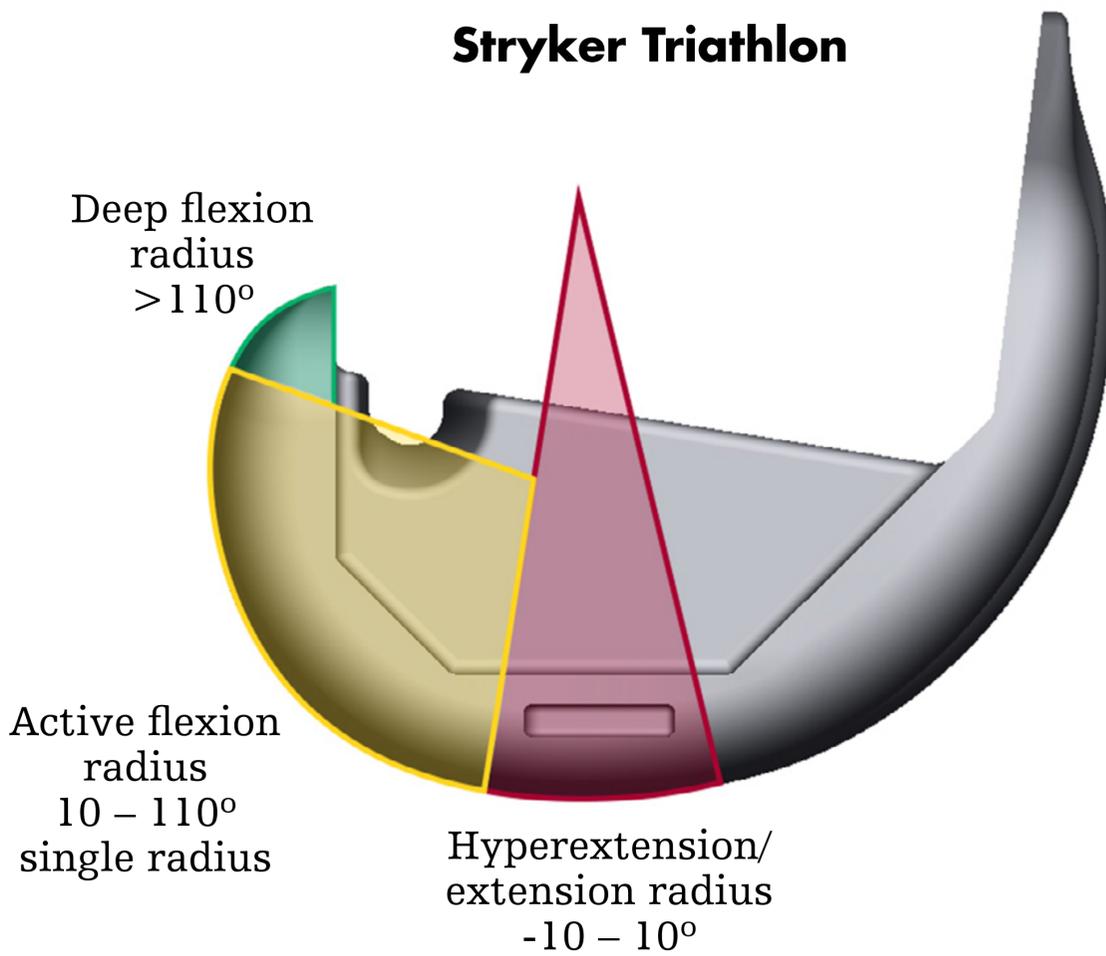



Stability

- In a quantitative assessment comparing gait between sixteen patients who received a single radius and sixteen patients who received a multi-radius total knee, Triathlon single radius patients experienced gait patterns that mimicked those of healthy individuals.⁸²
- Fluoroscopic data of Triathlon patients showed that medial pivot is possible without features that force a medial pivot.¹⁰⁷




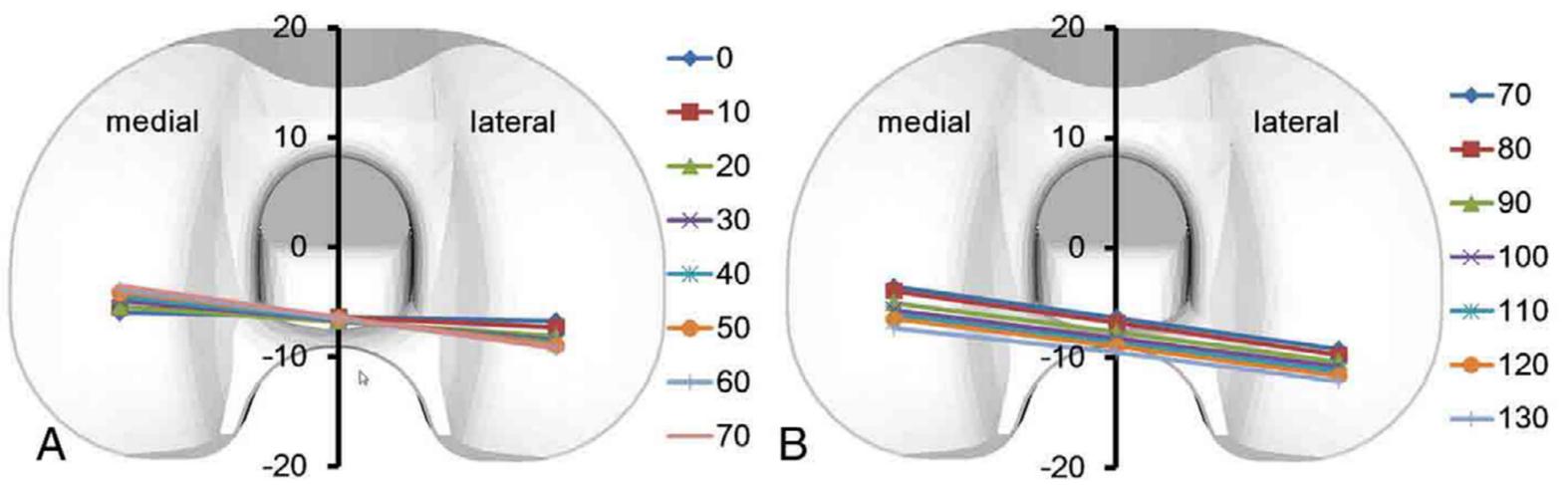




Deep flexion

Triathlon can achieve deep flexion without excessive tibial slope. The surgical technique for Persona calls for 7° of slope.¹³¹ One publication showed that in TKA revised for flexion instability, the average pre-revision slope was approximately 7°.¹²⁹

Zimmer uses LifeMod data to promote flexion with their implants.¹¹⁹ One LifeMod publication showed 22° less flexion with Persona compared to NexGen-LPS Flex.¹⁰⁹

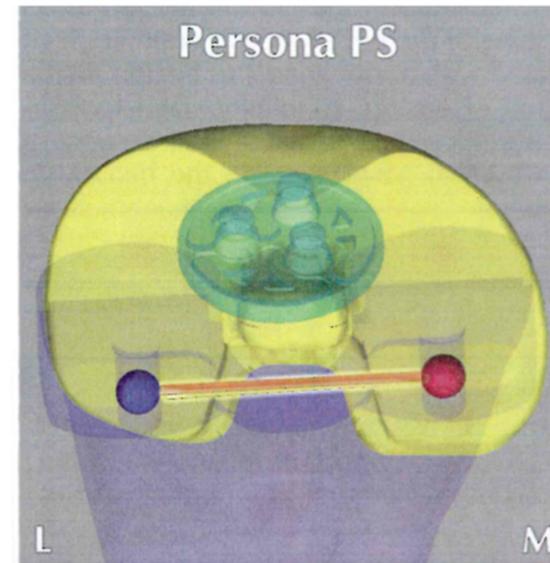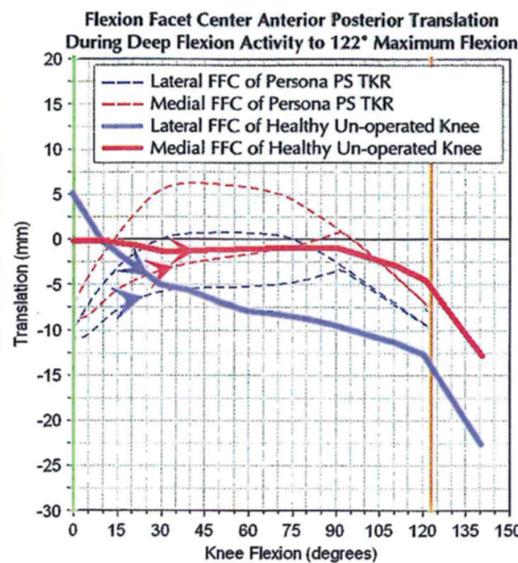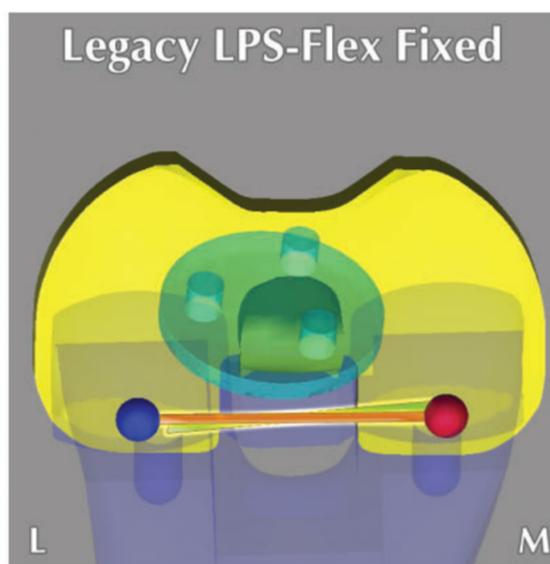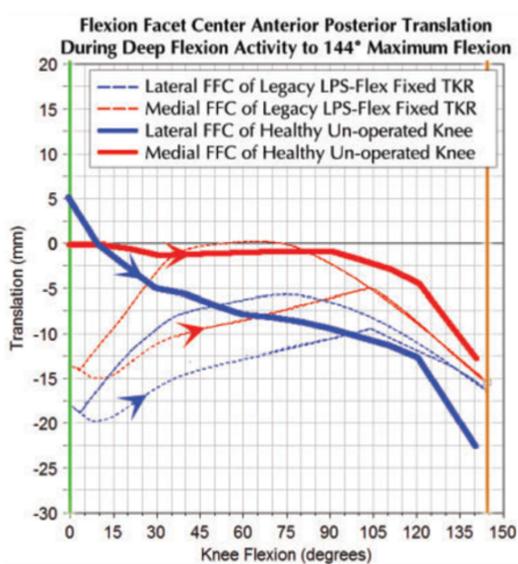

A closer look at Persona's post design shows how the post is used to lever the femur back as the cam travels up the post. Persona uses implant-driven motion to achieve deep flexion.





Polyethylene

- A published study showed that wear and mechanical integrity of X3 PS inserts were unaffected by accelerated aging, even after a rigorous stair-climbing test.¹⁶ Furthermore, the fracture toughness of X3 is double the minimum ASTM F648 requirement.¹⁸
- Zimmer Persona is available with Vitamin E polyethylene.⁶³ One case study reported early fracture of this Vitamin E polyethylene tibial insert following primary TKA.⁸¹





Size charts

Zimmer Persona CR – A/P and M/L dimensions ¹¹⁵					
Size	Functional A/P (mm)	M/L Narrow (mm)	M/L Std. (mm)	Overall A/P Narrow	Overall A/P Standard
1	41	55.5	N/A	48	N/A
2	43	57	N/A	50	N/A
3	45	58.5	62.5	52	53
4	47	60	64.25	54	55
5	49	61.5	66	56	57
6	51	63	67.75	58	59
7	53	64.5	69.5	60	61
8	55	66	71.25	62	63
9	57	67.5	73	64	65
10	59	69	74.75	66	67
11	61	70.5	76.5	70	71
12	65	N/A	77.5	N/A	78

Zimmer Persona PS – A/P and M/L dimensions ¹¹⁵					
Size	Functional A/P (mm)	M/L Narrow (mm)	M/L Std. (mm)	Overall A/P Narrow	Overall A/P Standard
1	42	55.5	N/A	48	N/A
2	44	57	N/A	50	N/A
3	46	58.5	62.5	52	53
4	48	60	64.25	54	55
5	50	61.5	66	56	57
6	52	63	67.75	58	59
7	54	64.5	69.5	60	61
8	56	66	71.25	62	63
9	58	67.5	73	64	65
10	60	69	74.75	66	67
11	62	70.5	76.5	70	71
12	66	N/A	77.5	N/A	78



Zimmer Persona PS – box dimensions¹¹⁵	
Box Width	Box Height
14.1	8
14.1	8
17	8
17	8
17	8
18.1	8
18.1	8
18.1	8
18.1	8
21.5	12
21.5	12
21.5	12

Zimmer Persona Stemmed Tibial – A/P and M/L dimensions¹¹⁵				
Size	Functional A/P (mm)	M/L Narrow (mm)	M/L Std. (mm)	Overall A/P Narrow
A	40.20	35.10	57.70	1
B	42.50	37.23	60.80	1
C	44.90	39.48	63.80	1
D	47.20	41.75	67.00	1.5
E	50.20	44.55	71.00	2
F	53.30	47.40	75.10	2.5
G	56.50	50.21	79.00	3
H	59.80	53.32	82.97	3.5
J	63.53	56.66	88.06	4



1. Gomez-Barrenna E, Fernandez –Garcia C. Functional Performance with a Single-Radius Femoral Design Total Knee Arthroplasty. *Clin Orthop Relat Res* (2010) 468:1214–1220
2. Designed to maintain collateral ligament stability throughout the range of motion. Stryker-Initiated Dynamic Computer Simulations of Passive ROM and Oxford Rig Test, Stephen Piazza, 2003.
3. Wang H, Simpson KJ, Ferrara MS, Chamnongkich S, Kinsey T, Mahoney, OM. Biomechanical differences exhibited during sit-to-stand between total knee arthroplasty designs of varying radii. *J Arthroplasty*. 2006;21(8):1193-1199.
4. Ostermeier S, Stukenborg-Colsman C. Quadriceps force after TKA with femoral single radius. An in vitro study. *Acta Orthopeda* 2011; 82 (3) 1-5.
5. National Joint Registry for England, Wales and Northern Ireland. 16th Annual Report 2019. Excerpt from table 3.29 Kaplan-Meier estimated cumulative percentage probability of first revision (95% CI) of a primary total knee replacement by main type of implant brand at the indicated number of years after primary operation.
6. Greene K. Range of Motion: Early Results from the Triathlon Knee System.
7. Stryker Test Data RD-03-041
8. Browne C et al. Patellofemoral forces after total knee arthroplasty: Effect of extensor moment arm. *Knee* 12:81, 2005.
9. Mahoney O et al. The Effect of total Knee Arthroplasty Design on Extensor Mechanism Function. *Journal of Arthroplasty* Vol. 17 No. 4 2002. 416-421
10. Hitt K, Shurman JR 2nd, Greene K, McCarthy J, Moskal J, Hoeman T, Mont MA. Anthropometric measurements of the human knee: correlation to the sizing of current knee arthroplasty systems. *J Bone Joint Surg Am*. 2003;85-A Suppl 4:115-22.
11. Mahoney O et al. Posterior Cruciate Function following total Knee Arthroplasty: A biomechanical study. *Journal of Arthroplasty* Vol. 9 No 6 1994, page 569
12. Kirby D. Hitt, Todd P. Pierce, Julio J. Jauregui, Jeffrey J. Cherian, Randa DK Elmallah, Evan Leibowitz, Scott Logan, & Michael A. Mont. Use of a Flexible Intramedullary Rod and its Influence on Patient Satisfaction and Femoral Size in Total Knee Arthroplasty. *Journal of Long-Term Effects of Medical Implants*, 25(3): 201–208 (2015).
13. S.J. Incavo, P.J. Ronchetti, J.G. Howe, J.P. Tranowski. Tibial plateau coverage in total knee arthroplasty. *Clin Orthop Relat Res*, 299 (1994), pp. 81–85
14. “Wear, oxidation and mechanical properties of a sequentially irradiated and annealed UHMWPE in total joint replacement”; A. Wang, H. Zeng, S-S Yau, A Essner, M Manely and J Dumbleton; *J. Phys. D: Appl. Phys.* 39 (2006) 3213–3219
15. Stryker Orthopaedics Technical Report #RD-05-076.
16. Wang A, Yau SS, Essner A, et al. A highly cross-linked UHMWPE for CR and PS total knee arthroplasties. *J Arthroplasty*. 2008 Jun; 23(4):559-566.
17. Hitt, K., Harwin, S.F., Greene, K.A. Early Experience with a New Total Knee Implant: Maximizing Range of Motion and Function with Gender-Specific Sizing Orthopedic Surgery, Surgical Technology International, XVI.
18. Stryker test report: RD-03-081 Rev 2. X3 UHMWPE virtually eliminates free radicals, as measured by Electron Spin Resonance (ESR). A very low (noise level, near instrument detection limit) concentration of residual free radicals was detected in the X3 UHMWPE. A 99% reduction of free radicals (14 2 versus 1550 32, 1014 spins/gram) was found when compared to N2Vac gamma sterilized UHMWPE.



19. J, Fischler, A, Colwell C, D'Lima D. The Effect of Oxidative Aging on the Wear Performance of Highly Crosslinked Polyethylene Knee Inserts under Conditions of Severe Malalignment. *Journal of Orthopaedic Research*. 1-6.
20. Frankel V, Burstein A, Brooks D. Biomechanics of Internal Derangement of the Knee. *J Bone Joint Surg Am*. 1971; 53:945-977
21. Iwaki H, Pinskerova V, Freeman MAR. Tibiofemoral movement 1: the shapes and relative movements of the femur and tibia in the unloaded cadaveric knee. *J Bone Joint Surg [Br]* 2000;82-B:1189-95.
22. Hollister, A., *The Axes of Rotation of the Knee, Clinical Orthopaedics and Related Research*. Number 290 pp. 259-268
23. Kapandji IA. Movements of the Femoral Condyles on the Tibial Plateau During Flexion and Extension. *The Physiology of the Joints*. Volume 2. Pg. 84.
24. Multicenter Determination of In Vivo Kinematics After Total Knee Arthroplasty, Dennis & Komistek. *Clinical Orthopaedics & Related Research* Number 416, pp. 37-57.
25. Sharkey et al. Why are Total knee arthroplasties failing today. *CORR*. 2002 Nov. 404; 7-13.
26. Cook L, Klika A, Szubski C, Rosneck J, Molloy R, Barsoum W. Functional Outcomes Used to Compare Single Radius and Multiradius of Curvature Designs in Total Knee Arthroplasty *J Knee Surg* 2012;25:249–254.
27. Bonutti P, Essner A, Herrera L, Longaray J, Kester M. Influence of Design and Bearing Materials on Wear: Comparing Two Different TKA Designs. *ISTA 2012 Sydney, Australia*. Poster #895
28. Harwin SF, Greene KA, Hitt K. Triathlon total knee arthroplasty: 4-year outcomes with a high-performance implant. *J Knee Surg*. 2008 Oct;21(4):320-6
29. Okamoto N, Breslauer L, Hedley A, Mizuta H, Banks S. In Vivo Knee Kinematics in Patients with Bilateral Total Knee Arthroplasty of 2 Designs. *Journal of Arthroplasty* Vol. 26 No. 6 2011
30. Zimmer NexGen Implant Options brochure. 97-5972-205.
31. Egol et al. Mismatch of Current Intramedullary Nails with Anterior Bow of the Femur. *J Orthop Trauma*; Volume 18, Number 7, August 2004. 410-415
32. McArthur J, Makrides P, Thangarajah T, Brooks S. Tibial component overhang in total knee replacement: incidence and functional outcomes. *Acta Orthop Belg*. 2012 Apr;78(2):199-202.
33. Yau S, Wang A, Essner A, Manley M, Dumbleton J. Sequential Irradiation and Annealing of Highly Crosslinked Polyethylenes Resist Oxidation Without Sacrificing Physical/Mechanical Properties. 51st Annual Meeting of the Orthopedic Research Society. Poster No 1670.
34. Oral E, Wannomae K, Hawkins N, Harris W, Muratoglu O. a-Tocopherol-doped irradiated UHMWPE for high fatigue resistance and low wear. *Biomaterials* 25 (2004) 5515–5522.
35. AOX Design Rationale. DePuy. 0612-59-508
36. E1 Antioxidant Infused Technology. Biomet. Form No. BOI0314.1; REV103109.
37. Okubo Y, Mori S, Yamamoto K, Tomita N. Confirmation of Pressure-Induced Leaching of Vitamin E from inside of Ultrahigh Molecular Weight Polyethylene. *Journal of Biomechanical Science and Engineering*. Vol 5, No 2, 2010. 154-162.
38. Peterson C, Hillberry B, Heck D. Component wear of total knee prostheses using Ti-6Al-4V, titanium nitride coated Ti-6Al-4V and cobalt chromium molybdenum femoral components. *J Biomedical Materials Research*, Vol 22, 887-903 (1988)



39. Fisher J, Jennings L, Galvin A, jim Z, Stone M, Ingham M. Polyethylene Wear in Total Knees. 2009 Knee Society Presidential Guest Lecture. Clin Orthop Relat Res (2010) 468:12–18
40. Stryker Test Report RD-13-005
41. Yong CK, Choon DSK, Soon HC. Midterm outcome of the duracon total knee arthroplasty. Journal of Orthopaedic Surgery 2008;16(2):197-200.
42. Mont M, Yoon T, Krackow K, Hungerford D. Eliminating Patellofemoral Complications in Total Knee Arthroplasty Clinical and Radiographic Results of 121 Consecutive Cases Using the Duracon System. Journal of Arthroplasty Vol. 14 No. 4 1999
43. Stryker Test Data RD-09-088
44. Bhimji S, Kester M, Schmalzried T. Rotational Constraint of Posterior-Stabilized Total Knee Prostheses. J Knee Surgery. 2008; 21: 315-319.
45. Shimizu N, Tomita T, Yamazaki K, Kurita M, Futai K, Kunugiza Y, Sugamoto K. In vivo movement of femoral flexion axis of a single-radius total knee arthroplasty. ORS 2012 annual Meeting. Poster# 1982.
46. Bracco P, Oral E. Vitamin E-stabilized UHMWPE for Total Joint Implants. Clin Orthop Relat Res (2011) 469:2286–2293.
47. Le, K-P; Song, L; Longaray, J; Carlos, A; Herrera, L; Yau, S-S; Essner, A. Can We Trust Oxidation Measurements from Retrieved UHMWPE? ORS 2013 POSTER No. 1235
48. Stryker Test Data RD-09094
49. US Patent 7,160,330
50. US Pat. 7,517,919
51. Dean D. Effect of polymer molecular weight and addition of calcium stearate on response of MG63 osteoblast-like cells to UHMWPE particles. Journal of Orthopaedic Research 19 (2001) 179—186
52. Zimmer MIS Multi-Reference 4-in-1 Femoral Instrumentation Surgical Technique.
53. Stryker Test Report MA-13-3551-TR Rev 0.
54. Gomaa S. Effect of Tibial Insert Geometry on Kinematics following Fixed-Bearing Cruciate-Retaining total Knee Arthroplasty. Orthopaedic Research Society. 55th Annual Meeting. Paper No. 191.
55. DePuy PFC Sigma RP-F brochure. 0612-00-503 (Rev. 1)
56. Bargmann LS, Bargmann BC, Collier JP, Currier BH, Mayor MB; Current Sterilization and Packaging Methods for Polyethylene; Clin Orthop Relat Res. 1999 Dec;(369):49-5
57. Rowell SL, Yi J, Micheli BR, Wannomae KK, Malchau H, Muratoglu OK. In Vivo Performance of Second Generation Highly Cross-linked UHMWPE. Poster No. 1033. ORS 2012 Annual Meeting
58. Reinitz, S D, Currier, B H, Franklin, K J, Tibbo, M E, Van Citters, D W, Collier, J P. Early Indications of Oxidative Degradation in Retrieved Annealed UHMWPE Bearings Resembles Gamma Sterilized Materials. Paper No. 0307. ORS 2012 Annual Meeting.
59. Bandi, M; Siggelkow, E; Klauser, W; Benazzo, F; Sauerberg. Assessing the Effect of Finer Increments of Tibial Inlay Thickness on Laxity and Kinematics in Total Knee Arthroplasty – An Experimental and Numerical Investigation. ORS 2013. Poster 0962.
60. Value Brief: Attune Knee System. DePuy Orthopaedics. 0612-39-513 0113
61. Bozic K, kinder J, Menegini M, Zurakowski D, Rosenberg A, Galante J. Implant Survivorship and Complication Rates after Total Knee Arthroplasty with a Third-generation Cemented System. CORR. Number 430, pp. 117-124.



62. Attune Surgical Protocol. DePuy Orthopaedics. 0612-09-512
63. Persona – The Personalized Knee System. Zimmer.
64. FDA 510K K101433
65. Stryker Test Report RD-12-170
66. D’Lima D, Patil S, Steklov N, Colwell C. The 2011 ABJS Nicolas Andry Award: ‘Lab’-in-a-Knee. In *Vivo Knee Forces, Kinematics and Contact Analysis*. CORR. Volume 469, Issue 10, pp. 2953-2970.
67. FDA 510(k) K120370
68. Triathlon Total Knee System Surgical Protocol. TRIATH-SP-3.
69. Wernecke G, Harris I, Houang M, Seeto B, Chen D, MacDessi S. Comparison of tibial bone coverage of 6 knee prostheses: a magnetic resonance imaging study with controlled rotation. *Journal of Orthopaedic Surgery* 2012;20(2):143-7.
70. Zimmer Gender solutions NexGen High-Flex Implants. Zimmer. 97-5764-001-00 Rev.2
71. NexGen Legacy Knee LPS-Flex. Zimmer. 97-5964-101.
72. PFC Sigma RPF. DePuy Orthopedics. 0612-00-503 (Rev. 1)
73. Han et al. High incidence of loosening of the femoral component in legacy posterior stabilised-flex total knee replacement. *J Bone Joint Surg [Br]* 2007;89-B:1457-61.
74. Bollars et al. Femoral component loosening in high-flexion total knee replacement. An In Vitro Comparison of High-Flexion Versus Conventional Designs. *J Bone Joint Surg Br* 2011;93-B:1355–61.
75. Thomsen MG, Husted H, Bencke J, Curtis D, Holm G, Troelsen A. Do we need a gender-specific total knee replacement?. *J Bone Joint Surg Br* 2012;94-B:787–92.
76. Lack of axial rotation in Mobile Bearing designs. Komistek, R., Wasielewski, R. et al. *CORR* 466:2622-2668.
77. Mobile-bearing Total Knee Arthroplasty: A Risk Factor for Early Aseptic Failure. Namba R, Slipchenko T, Paxton L, Funahashi, T, Fithian D. American Academy of Orthopaedic Surgeons Annual Meeting; February 26-28, 2009; Las Vegas, NV.
78. Yau et al. Real-Time Shelf Aging of Sequential Crosslinked and Annealed UHMWPE. Poster No. 450; 55th Annual Meeting of the Orthopaedic Research Society
79. Santiago et al. Cruciate Retaining vs Posterior Substituting Inserts in TKA. *J or Arthorplasty* Vol 28 No 2 2013.
80. Mistry J, Elmallah R, Chughtai M, Oktem M, Harwin S, Mont M. Long-Term Survivorship and Clinical Outcomes of a Single Radius Total Knee Arthroplasty. *International XXVIII*.
81. Barrack R. Retrieval Analysis of an Early Fracture of a Vitamin E-Stabilized tibial Liner in total Knee Arthroplasty. *JBJS Case Connect* 2013; 3(2): 1-6
82. Larsen et al. Quantitative, Comparative Assessment of Gait Between Single-Radius and Multi-Radius Total Knee Arthroplasty Designs *J Arthroplasty* (2015).
83. Stryker Test Report RD-13-062.
84. Bhimji S et al. Tibial Insert Micromotion of Various Total Knee Arthroplasty Devices. *Journal of Knee Surgery*. Volume 23. No 3. 153-162.
85. Conditt et al. Factors Affecting the Severity of Backside Wear of Modular Tibial Inserts. *J Bone Joint Surg Am*. 2004;24:305-311



86. Meneghini et al. Highly Crosslinked Versus Conventional Polyethylene in Posterior-Stabilized Total Knee Arthroplasty at a Minimum 5-Year Follow-up. International Society for Technology in Arthroplasty (ISTA) 26th Annual Congress. October 16-19, 2013. Poster #2577.
87. Scott DF, Smith RR, A Prospective, Randomized Comparison of Posterior Stabilized Versus Cruciate-Substituting Total Knee Arthroplasty: A Preliminary Report with Minimum 2-Year Results, *J Arthroplasty* (2014).
88. Stryker Test Report 06-013.
89. Australian Orthopaedic Association National Joint Replacement Registry. 2019 TY2 Cumulative Percent Revision of Primary Total Knee Replacement Combinations with 10 Year Data (Primary Diagnosis OA).
90. Stryker Technical Memo. Competitive Implant Geometric Comparisons. May 2014.
91. Vanguard Complete Knee System – Spectrum of Stability. Biomet. Form No. BOI0018.1; REV063010.
92. Morra et al The Influence of Contemporary TKA Knee Design on High Flexion: A Kinematic Comparison with The Normal Knee. Orthopaedic Research Laboratories. AAOS 2008.
93. Clary CW. Improving Dynamic mid-stance Stability: An Experimental and Finite Element Study. Poster No. 1044. ORS 2012 Annual Meeting
94. Attune Surgical Protocol. DePuy Orthopaedics. 0612-09-512
95. Attune 510(k). K101433.
96. Attune Value Dossier. DePuy Orthopaedics. Accessed May 2014.
97. Attune Technology Monograph. DePuy Orthopaedics. 0612-21-513
98. PFC Sigma Technical Monograph. DePuy Orthopaedics. 0611-29-050 (Rev. 1)
99. Dennis D. Control-Matched Evaluation of Painful Patellar Crepitus After Total Knee Arthroplasty. *Clin Orthop Relat Res* (2011) 469:10–17
100. Fukunaga K et al. The incidence of the patellar clunk syndrome in a recently designed mobile-bearing posteriorly stabilised total knee replacement. *J Bone Joint Surg [Br]* 2009;91-B:463-8.
101. Sigma Fixed Bearing Knees. DePuy Orthopaedics. 0612-65-508
102. Antioxidant Stabilized Orthopaedic Devices. DePuy Orthopaedics. 0612-33-509 (Rev 1)
103. Wolterbeek N et al. Kinematics and early migration in single-radius mobile- and fixed-bearing total knee prostheses. *Clinical Biomechanics* 27 (2012) 398–402
104. Genesis II Kinematics. Smith & Nephew. 7128-0436
105. Dolan M et al. Implant Design Influences Tibial Post Wear Damage in Posterior-stabilized Knees. *Clin Orthop Relat Res* (2011) 469:160–167
106. Legion 510(k). K093746
107. Shimizu N, et al, In Vivo Movement of Femoral Flexion Axis of a Single-Radius Total Knee Arthroplasty, *J Arthroplasty* (2014)
108. Fitzpatrick C et al. Mechanics of Post-Cam Engagement during Simulated Dynamic Activity. *Journal of Orthopaedic Research* September 2013. 1438-1446.
109. Morra et al. The Influence of Contemporary TKA Knee Design on High Flexion II: A Kinematic Comparison with The Healthy Intact Knee. Orthopaedic Research Laboratories. AAOS 2014.



110. Vivacit-E Vitamin E Highly Crosslinked Polyethylene. Long-term Performance for High Demand Patients. Zimmer Technical Memo. 97-7255-181-00 1209-H01
111. Scott C et al. Excellent 10-year patient-reported outcomes and survival in a single-radius, cruciate-retaining total knee arthroplasty. *Knee Surgery, Sports Traumatology, Arthroscopy* (2019). 27:1106-1115.
112. Attune dimensions. DePuy Orthopaedics.
113. System Versatility. Smith & Nephew. 7128-0435
114. Journey 2 Surgical Technique. Smith & Nephew. 00331 REVB
115. Persona Templates. Zimmer. 97-5026-050-00
116. NexGen CR Flex. Zimmer. 97-5952-02
117. NexGen LPS Flex. Zimmer.
118. Stryker Triathlon and the Single-Radius Myth. Smith & Nephew. 7128-1872 Rev 0.
119. The Truth About A Round Knee. Zimmer. Web link. Accessed 5/3/2014
120. Cherian et al. Differences in Readmission Rates for Two Total Knee Arthroplasty Prostheses. *J Knee Surg.* 2014
121. Stryker Test Report RD-13-109
122. Cates et al. In Vivo TKA Kinematics for Subjects Walking Down a Ramp. ICJR Pan Pacific Congress 2014 - JISRF Reconstructive Review; Vol. 4, Sup. 1, July 2014
123. Sharma et al. Comparison of Axial Rotation in Symmetrical Versus Asymmetrical PCR TKA. ICJR Pan Pacific Congress 2014 - JISRF Reconstructive Review; Vol. 4, Sup. 1, July 2014
124. Vanguard Complete Knee System. Biomet. The Facts About the Gender Specific Knee. Form No. BOI0452.0; REV071510
125. Vanguard Complete Knee System. Design Rationale. Biomet. Form No. BOI0465.0, REV051511
126. Stryker Technical Memo. White band in UHMWPE partially due to testing artifacts. July 2014.
127. J. Stoddard, D.J. Deehan, A.M.J. Bull, A.W. McCaskie, A.A. Amis. The Kinematics and Stability of Single-Radius Versus Multi-Radius Femoral Components Related to Mid-Range Instability after TKA. *Journal of Orthopaedic Research.* 2012. 1-6
128. Hamilton et al. Lengthening the Moment Arm of the Patella Confers Enhanced Extensor Mechanism Power Following Total Knee Arthroplasty. *Journal of Orthopaedic Research* August 2013. 1201-1207
129. Abdel et al. Stepwise Surgical Correction of Instability in Flexion After Total Knee Replacement. *Bone Joint J* 2014;96-B:1644–8.1644-1648
130. Stryker Test Report RD-14-073
131. Persona Surgical Technique. Zimmer. 97-5026-001-00 Rev. 6 1209-K03
132. Bhowmik-Stoker et al. Mid-Term Results of Patient Satisfaction Following Total Knee Arthroplasty. 2014 ICJR Transatlantic Orthopaedic Congress New York, NY October 3-5, 2014
133. NexGen Complete Knee Solution Intramedullary Instrumentation Surgical Technique. Zimmer. 97-5973-102 Rev. 2 Gomez-Barrenna E, Fernandez –Garcia C. Functional Performance with a Single-Radius Femoral Design Total Knee Arthroplasty. *Clin Orthop Relat Res* (2010) 468:1214–1220



134. MacDonald D et al. Oxidation, Damage Mechanisms and Reasons for Revision of Sequentially Annealed Highly Crosslinked Polyethylene in Total Knee Arthroplasty. *The Journal of Arthroplasty* 2018 Apr;33(4):1235-1241. doi: 10.1016/j.arth.2017.09.036. Epub 2017 Sep 25.
135. MacDonald D et al. In Vivo Oxidation and Surface Damage in Retrieved Ethylene Oxide-sterilized Total knee Arthroplasties. *Clinical Orthopaedics and Related Research* (2012). 470:1826-1833.
136. Hamilton D.F. et al. Implant design influences patient outcome after total knee arthroplasty. *Bone Joint J* (2015). 97-B:64-70.
137. Hamilton D.F. et al. Reduction in patient outcomes but implant-derived preservation of function following total knee arthroplasty: longitudinal follow-up of a randomized controlled trial. *Bone Joint J* (2020). 102-B(4):434-441.
138. Partridge T et al. Conventional Versus Highly Cross-Linked Polyethylene in Primary Total Knee Replacement. *J Bone Joint Surg Am* (2020). 102:119-127.



A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker's product offerings. A surgeon must always refer to the package insert, product label and/or instructions for use before using any of Stryker's products. The products depicted are CE marked according to the Medical Device Regulation 2017/745 or the Medical Device Directive 93/42/EEC. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your sales representative if you have questions about the availability of products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Duracon, Stryker, Stryker Orthopaedics, Triathlon, X3. All other trademarks are trademarks of their respective owners or holders.

TRIATH-BRO-12_Rev-1_25139

Copyright © 2020 Stryker