

Sage Prevalon

Heel Protector II

REF 7300-X




Features and benefits

Dermasuede fabric interior	<ul style="list-style-type: none"> Gently grips limb so it remains fully offloaded, even when patient is moving
Rip-stop nylon outer shell	<ul style="list-style-type: none"> Helps maintain patient's freedom of movement Slides easily over bed sheets
Open, floated heel design	<ul style="list-style-type: none"> Completely offloads heel Allows for easy monitoring between assessments
SCD compatible	<ul style="list-style-type: none"> Four ports sewn into boot design accommodate most SCD devices
Contracture Straps	<ul style="list-style-type: none"> Reduces plantar flexion contracture risk^{1,2}
Expandable Straps	<ul style="list-style-type: none"> Stretches to accommodate lower limb oedema No sharp edges or irritating surfaces

To offload the heel

1 Put the heel protector on
The heel protector is shipped **inside out**. Position heel over opening on grey side. Pull sides up around foot and toes.

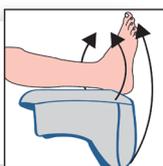

2 Tighten straps
Wrap straps around the heel protector and adjust. **Do not** over tighten.




3 Make sure the heel is floated



To use with sequential compression device:

Run tubing through the nearest available port. Ensure tubing is not kinked or contacting skin. Push tubing down into seams when using side ports. **Do not** use heel hole as a port.

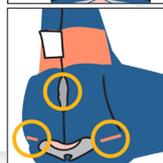

Packaging Information

Contents:

(1) Heel Protector

Intended Use:

For offloading the heel and reducing the risk of heel pressure ulcers, and for reducing the risk of plantar flexion contracture.^{1,2}

Caution:

For use on patient while in bed. Do not stand or walk while wearing due to fall risk. Periodically remove heel protector to assess skin according to protocol. Read enclosed instructions prior to using product.

Cleaning Instructions:

Wipe clean with a damp cloth.

Discard according to hospital protocol and local regulations. Not made with natural rubber latex. Single patient - multiple use.

Specifications

Product name	Sage Prevalon Heel Protector II
Product code	7300-X
Description	Offloads the heel. Reduces plantar flexion contracture risk. ^{1,2}
Manufacturer accreditation	ISO 13485:2016
Recommended for patients with calf circumference of:	25cm - 46cm (10in - 18in)

1. Meyers, T. (2017). Prevention of Heel Pressure Injuries and Plantar Flexion Contractures With Use of a Heel Protector in High Risk Neurotrauma, Medical, and Surgical Intensive Care Units: A Randomized Controlled Trial. Journal of Wound, Ostomy, and Continence Nursing: Official Publication of The Wound, Ostomy and Continence Nurses Society, 44(5), 429-433. 2. Meyers, T. (2010). Meyers T, Preventing Heel Pressure Ulcers and Plantar Flexion Contractures in High Risk Sedated Patients. J Wound Ostomy Continence Nurse. 2010 Jul-Aug;37(4):372-8

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it. The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area. Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders. For more information please contact your local Stryker sales representative.