

Sage

Heel Protector OR

REF 7330



Features and benefits

| | |
|------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Calf Cradles | <ul style="list-style-type: none"> • Elevates the heels • Distributes pressure over the lower leg • Sequential Compression Device (SCD) compatible |
| Side Rail Strap | <ul style="list-style-type: none"> • Helps prevent legs from migrating off the OR table, even when patient is in a lateral tilt position |
| Calf Cradle Connector Strap | <ul style="list-style-type: none"> • Helps prevent legs from migrating off the OR table, even when patient is in a lateral tilt position |
| Knee Cushion | <ul style="list-style-type: none"> • Helps maintain recommended knee flexion throughout surgery¹ • Replaces pillows |

Specifications

| | |
|----------------------------|---------------------------------------------------------------------|
| Product name | Sage Heel Protector OR |
| Product code | 7330 |
| Description | Protects heels and safely secures lower limbs in the operating room |
| Manufacturer accreditation | ISO 13485:2016 |

Packaging Information

Contents:

(1) Knee Cushion, (2) Calf Cradles

Caution:

For use on patient while on operating room table, bed, or stretcher. Do not allow patient to stand or walk while wearing due to fall risk.

Continue to follow your facility's protocol for skin assessment.

Non-sterile.
 Single use only.
 Consult instructions for use.
 Not made with natural rubber latex.

Sage

Heel Protector OR (7330)



Instructions for use


1 Remove the Knee Cushion and Calf Cradles from the bag

2 Place the Knee Cushion under the patient's knee(s)

Avoid hyperextension of the knee(s).

Caution: To avoid potential injury, do not use Calf Cradles without Knee Cushion.

Note: If using Sequential Compression Device (SCD), apply SCD sleeve(s) prior to placement of Calf Cradles.

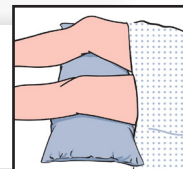

3 Open the spandex flaps of each Calf Cradle


With the blue sides facing down, slide Calf Cradles under patient's leg(s) until centered on the black strips.

Ensure the bottom of each Calf Cradle is under the patient's Achilles.

Ensure the top of each Calf Cradle is under the patient's calf.

If only one Calf Cradle is needed, disconnect the Calf Cradles at the Connecting Buckle.


Dispose of the unused Calf Cradle.

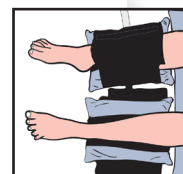

4 Close the spandex flaps around each leg to secure the Calf Cradle

Caution: Check tightness of Calf Cradles according to hospital protocol.

Caution: If using SCD, ensure tubing is not kinked or contacting skin.

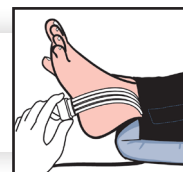

5 Patient's heels should not be in contact with the table surface

If heels contact table, move Calf Cradles toward feet until heels are floated.




6 Anchor Calf Cradles to OR table rails with black and white Calf Cradle Side Straps



7 Before transferring patient, release Side Straps

Release Connecting Buckle to separate Calf Cradles.

Transfer patient to OR table, bed, or stretcher.

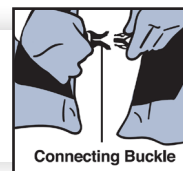

8 If product become soiled, dispose of product once surgery is completed

Otherwise, dispose of product after recovery phase.

Stryker Australia Pty Ltd

8 Herbert Street
St Leonards NSW 2065
61 2 9467 1000
www.stryker.com.au

Stryker New Zealand Limited

PO Box 17-136, Greenlane
Auckland 1546, New Zealand
64 9 573 1890
www.stryker.com

SG_3028_SAG

Copyright © 2021 Stryker

References:

1. Burlingame B. Guideline Implementation: Positioning the Patient. AORN Journal. 2017; 106(2). doi:10.1016/j.aorn.2017.07.010.

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it. The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area. Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders. For more information please contact your local Stryker sales representative.