

Sage Oral Care System Q4

REF 6964-X



Features and benefits

Suction Toothbrush

Ultra-soft nylon bristles	• Gently removes dental plaque, debris and oral secretions known to harbor respiratory pathogens
Over 2,600 bristles or “contact points”	• Cleans and removes plaque
Three suction ports	• Avoids clogging and keeps open path for debris
Connects directly to standard suction lines	• Helps provide maximum suction strength
Angled handles	• Provides cleansing of the back of the oral cavity

Suction Swab

Suction Swab	• Helps remove debris and secretions between brushings
Soft foam heads	• Gentle to fragile tissue

Specifications

Product name	Sage Oral Care System Q4
Product code	6964-X
Description	Daily kit to provide oral hygiene including tools for each 4-hour nursing shift
Manufacturer accreditation	ISO 13485:2016
Expiration date	24 months from date of manufacture
Length of Suction Toothbrush	18.4 cm
Length of each bristle	8 mm
Length of Suction Swab	18.4 cm
Length of Applicator Swab	15.9 cm
Amount of solution in each burst packet	7 mL
Case quantity	20

Packaging Information

Contents:

- (2) Packages of (1) Untreated Suction Toothbrush, (1) 7mL Sage Oral Solution, (1) Applicator Swab.
- (4) Packages of (1) Suction Swab with sodium bicarbonate, (1) 7mL Corinz™ Oral Rinse.

Indications of Use:

Suction Toothbrush: Aids in the removal of dental plaque, debris, and oral secretions.

Suction Swab: Aids in the removal of oral secretions and debris.

Instructions for Use:

Suction Toothbrush: Burst solution packet with thumbs. Peel package open. Attach Suction Toothbrush to suction line. Brush teeth and oral cavity for 1 minute. To suction, place thumb over port. Rinse to clear tubing after use. Discard according to hospital protocol and local regulations. Use Applicator Swab if additional cleansing is needed.

Sage Oral Solution: Do not rinse with water or other mouthwashes, brush teeth or eat immediately after using Sage Oral Solution. Do not swallow. To minimize discoloration of teeth, brush and floss daily. STORE ABOVE FREEZING (0°C or 32°F).

Suction Swab: Burst solution packet with thumbs. Peel package open. Attach Suction Swab to suction line. Clean mouth for 1 minute. To suction, place thumb over port. Rinse to clear tubing after use. Discard according to hospital protocol and local regulations.

Corinz Oral Rinse: Children under 3 years of age: Consult a dentist or doctor. Children under 12 years of age: Supervise use. Use under guidance of a healthcare professional. Keep out of reach of children.

Sage Oral Care System Q4

Caution:

Do not allow patient to bite down on oral care tool. Use a bite block if patient has altered levels of consciousness or cannot comprehend commands. Use caution with children and unresponsive individuals. Failure to follow these safety precautions may damage the device and present a choking/aspiration hazard. May not function as intended/potential risk of cross-contamination if device is reused.

Not made with natural rubber latex.

Suction Toothbrush and Suction Swab: MDD 93/42/EEC

Applicator Swab: MDR 2017/745

Corinz Oral Rinse and Sage Oral Solution: EC Cosmetic Regulation 1223/2009

Sodium bicarbonate ingredients

Water (aqua)
Sodium bicarbonate
Cellulose gum
Sodium lauryl sulfate
Flavour (aroma)
Sodium saccharin
Sodium benzoate

Corinz™ Oral Rinse ingredients

Water (aqua)
Glycerin
Xylitol
Flavour (aroma)
Hydroxyethylcellulose
Potassium sorbate
Cetylpyridinium chloride
Sodium saccharin
Citric acid
Polysorbate 20
Polysorbate 80
Menthol

Sage Oral Solution ingredients

Water (aqua)
Alcohol
Glycerin
PEG-40 sorbitan diisostearate
Flavor (aroma)
Chlorhexidine digluconate
Sodium saccharin
Blue 1 (C1 42090)

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders.

For more information please contact your local Stryker sales representative.

Stryker Australia Pty Ltd

8 Herbert Street
St Leonards NSW 2065
61 2 9467 1000
www.stryker.com.au

Stryker New Zealand Limited

PO Box 17-136, Greenlane
Auckland 1546, New Zealand
64 9 573 1890
www.stryker.com