

Sage Untreated Suction Toothbrush

With Alcohol-Free Mouthwash and Mouth Moisturiser

REF 6573-X



Features and benefits

Suction Toothbrush

Ultra-soft nylon bristles	• Gently removes dental plaque, debris and oral secretions known to harbor respiratory pathogens
Over 2,600 bristles or “contact points”	• Cleans and removes plaque
Three suction ports	• Avoids clogging and keeps open path for debris
Connects directly to standard suction lines	• Helps provide maximum suction strength
Angled handles	• Provide cleansing of the back of the oral cavity

Applicator Swab

Applicator Swab	• Helps remove debris and secretions between brushings
Soft foam heads	• Gentle to fragile tissue

Mouth Moisturiser

Mouth Moisturiser	• Moisturises and hydrates oral cavity
-------------------	--

Specifications

Product name	Sage Untreated Suction Toothbrush with Alcohol-Free Mouthwash and Mouth Moisturiser
Product code	6573-X
Description	Aids in the removal of dental plaque, debris and oral secretions
Manufacturer accreditation	ISO 13485:2016
Expiration date	24 months from date of manufacture
Length of Suction Toothbrush	18.4 cm
Length of each bristle	8 mm
Length of Applicator Swab	15.9 cm
Amount of solution in Alcohol-Free Mouthwash	7 mL
Amount of product in Mouth Moisturiser	2 g
Case quantity	100

Packaging Information

Contents:

(1) Untreated Suction Toothbrush, 7mL Alcohol-Free Mouthwash, 2g Mouth Moisturiser, (1) Applicator Swab

Indications of Use:

Aids in the removal of dental plaque, debris and oral secretions.

Instructions for Use:

Burst solution packet with thumbs. Peel package open. Attach Suction Toothbrush to suction line. Brush teeth and oral cavity for 1 minute. To suction, place thumb over port. Rinse to clear tubing after use. Discard. Dispense Mouth Moisturiser onto Applicator Swab and apply to lips and oral cavity as needed.

Sage Untreated Suction Toothbrush

With Alcohol-Free Mouthwash and Mouth Moisturiser

REF 6573-X

Caution:

Do not allow patient to bite down on oral care tool. Use a bite block if patient has altered levels of consciousness or cannot comprehend commands. Use caution with children and unresponsive individuals. Failure to follow these safety precautions may damage the device and present a choking/aspiration hazard. May not function as intended/potential risk of cross-contamination if device is reused.

Not made with natural rubber latex.

Suction Toothbrush: MDD 93/42/EEC

Applicator Swab: MDD 93/42/EEC

Moisturiser: EC Cosmetic Regulation 1223/2009

Alcohol-Free Mouthwash ingredients

Water (aqua)
Sorbitol
Flavor (aroma)
Potassium sorbate
Polysorbate 80
Polysorbate 20
Cetylpyridinium chloride
Citric acid
Blue 1 (CI 42090)

Mouth Moisturiser ingredients

Water (aqua)
Cocos nucifera (coconut) oil
Xylitol
Flavor (aroma)
Cellulose gum
Tocopheryl acetate
Mentha viridis (spearmint) leaf oil
Polysorbate 20
Polysorbate 80
Potassium sorbate
Carbomer
Cetylpyridinium chloride

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders.

For more information please contact your local Stryker sales representative.

Stryker Australia Pty Ltd

8 Herbert Street
St Leonards NSW 2065
61 2 9467 1000
www.stryker.com.au

Stryker New Zealand Limited

PO Box 17-136, Greenlane
Auckland 1546, New Zealand
64 9 573 1890
www.stryker.com