

Sage Suction System

with Corinz™ Oral Rinse

REF 6584-X



Features and benefits

Suction Swab

Suction Swab	• Helps remove debris and secretions between brushings
Soft foam heads	• Gentle to fragile tissue

Specifications

Product name	Sage Suction System with Corinz™ Oral Rinse
Product code	6584-X
Description	Aids in the removal of oral secretion and debris
Manufacturer accreditation	ISO 13485:2016
Expiration date	24 months from date of manufacture
Length of Suction Swab	18.4 cm
Amount of solution in Corinz Oral Rinse	7 mL
Case quantity	100

Sodium bicarbonate ingredients

Water (aqua)
Sodium bicarbonate
Cellulose gum
Sodium lauryl sulfate
Flavour (aroma)
Sodium saccharin
Sodium benzoate

Corinz™ Oral Rinse ingredients

Water (aqua)
Glycerin
Xylitol
Flavour (aroma)
Hydroxyethylcellulose
Potassium sorbate
Cetylpyridinium chloride
Sodium saccharin
Citric acid
Polysorbate 20
Polysorbate 80
Menthol

Packaging Information

Contents:

(2) Suction Swabs with sodium bicarbonate, (1) 7mL Corinz Oral Rinse.

Indications of Use:

Aids in the removal of oral secretions and debris.

Instructions for Use:

Burst solution packet with thumbs. Peel Package open. Attach Swab to suction line. Clean mouth for 1 minute. To suction, place thumb over port. Rinse to clear tubing after use. Discard according to hospital protocol and local regulations. Repeat as needed/directed.

Corinz™ Oral Rinse: Children under 3 years of age: Consult a dentist or doctor. Children under 12 years of age: Supervise use. Use under the guidance of a healthcare professional. Keep out of reach of children.

Caution:

Do not allow patient to bite down on oral care tool. Use a bite block if patient has altered levels of consciousness or cannot comprehend commands. Use caution with children and unresponsive individuals. Failure to follow these safety precautions may damage the device and present a choking/aspiration hazard. May not function as intended/potential risk of cross-contamination if device is reused.

Not made with natural rubber latex.

Suction Swab: MDD 93/42/EEC

Corinz Oral Rinse: EC Cosmetic Regulation 1223/2009

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders.

For more information please contact your local Stryker sales representative.