

Omega™ Double Double guide for reloading implant blister pack

It is best practice to load the implants with the reload blister pack flat on a table or mayo stand.



1 Orient the eyelet inserter handle so that is facing up and the arrow is pointed toward the eyelet. The inserter will be slightly angled, as pictured, for ease of insertion.



2 Insert the tip of the eyelet inserter into the eyelet and press to engage the eyelet with moderate pressure. A slight click may be felt at this time.

For reference, the proximal edge of the laser line will be flush with the edge of the reload blister pack.



3 Pivot the inserter handle upwards to disengage and remove the eyelet and handle from the reload blister pack.



4 Reinsert the pull tab kite through the center of the eyelet and attach to the shaft.



5 Insert the screwdriver into the screw until flush with the laser line and remove from the reload blister pack by pivoting the handle up similar to the eyelet.

Note: Orientation of the screwdriver is not relevant for reloading. Once loaded, ensure the screw is fully seated on the screwdriver.



Note: The eyelet, if properly loaded, should be seated on the inserter with the sharp tip of the inserter extending out of the tip of the eyelet and a small gap between the proximal edge of the eyelet and the inserter shaft.



Note: The screw, if properly loaded, should sit flush with the laser line.

Sports Medicine

A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A surgeon must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Omega, Stryker. All other trademarks are trademarks of their respective owners or holders.

1000903070

SC 5/19

Copyright © 2019 Stryker



Stryker Sports Medicine
5670 Greenwood Plaza Blvd, Ste 200
Greenwood Village, CO 80111 USA
t: 866 596 2022

www.stryker.com