# Philips ECG Leadset to OEM Trunk Cable Adapters

*Instructions for Use*

## 5-Lead Adapters

REF

989803174501 .....	GE AAMI/IEC
989803174511 .....	Spacelabs AAMI/IEC
989803174521 .....	Datascope AAMI
989803177881 .....	Datascope IEC
989803174531 .....	Mindray AAMI/IEC
989803182171 .....	Draeger/Fukuda-Denshi
989803182181 .....	Nihon-Kohden

# PHILIPS

---

# About This Edition

## Notice

### Copyright © 2012

Koninklijke Philips Electronics N.V.  
All rights are reserved.

## Printing History

### Current Edition/Revision:


Printed in China, September 2012  
Part Number 453564381741

### Previous Editions/Revisions:


November 2011  
Part Number 453564295191

## Contents

Instructions for Use.....	1	Návod na obsluhu.....	40
Manuel d'utilisation.....	4	Instrucțiuni de utilizare.....	43
Gebrauchsanweisung.....	7	Указания за употреба.....	46
Gebruiksaanwijzing.....	10	Kasutusjuhend.....	49
Instrucciones de uso.....	13	Naudojimo instrukcijos.....	52
Instruções de Uso.....	16	Lietošanas pamācība.....	55
Istruzioni d'uso.....	19	Инструкция по эксплуатации..	58
Bruksanvisning.....	22	Οδηγίες Χρήσης.....	61
Bruksanvisning.....	25	Kullanım Talimatları.....	64
Käyttöopas.....	28	Пайдалану нұсқаулары.....	67
Brugervejledning.....	31	ユーズーズガイド.....	70
Használati útmutató.....	34	使用説明.....	73
Návod k obsluze.....	37	使用说明书.....	76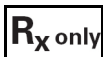

United States law restricts this device to sale by or on the order of a physician.

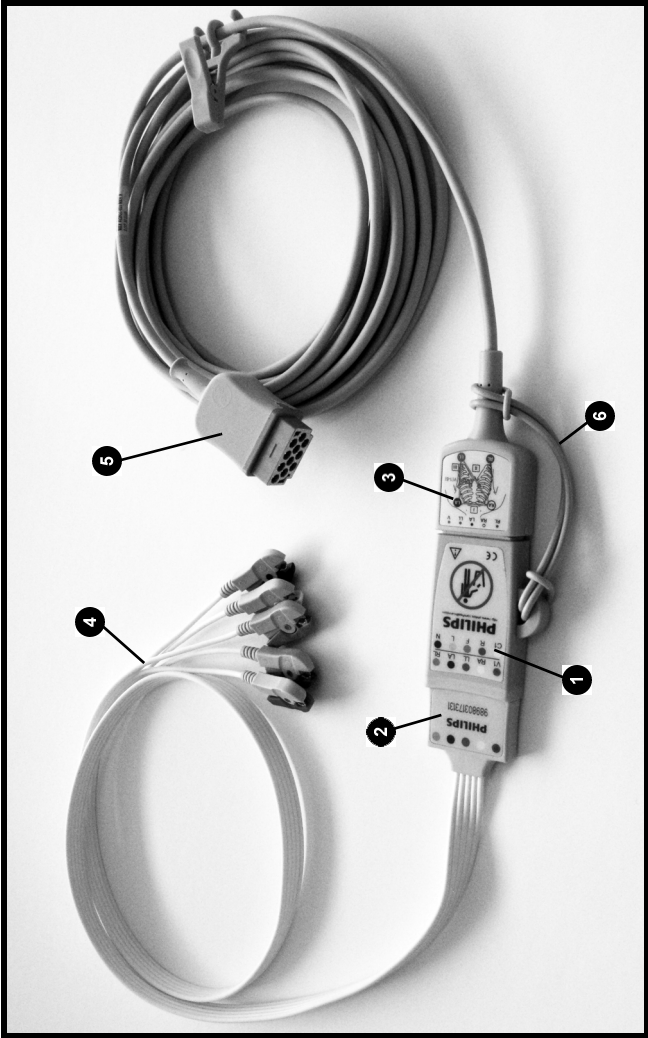# PHILIPS

989803173131  
989803174211

M1644A  
M1645A  
M1647A  
M1648A  
M1968A  
M1971A

989803174501

GE Multilink  
41-2931-xxx, 411910-0011

989803174511

Spacelabs 700-0008-06  
TruLink Shielded

989803174521 (AAMI)

Datascope  
0012-00-0620-02 (AAMI)  
or IEC Equivalent

989803177881 (IEC)

Mindray  
0012-00-1255-02

989803174531

Nihon Kohden  
JC-906P

989803182181

Draeger  
3368391, 5950196, 5191221

989803182171

Fukuda-Denshi  
CI-700D-5

---

# Product Overview

## Intended Use

Philips ECG Adapters allow care providers to connect Philips ECG leadsets to non-Philips ECG trunk cables. This enables a patient's ECG to be monitored via Philips electrodes/leads that are connected to OEM (Other Equipment Manufacturer) monitors.

Adapter use is limited only by:

- Compatibility with specific OEM trunk cables.
- Indications for use of the OEM monitors connected at the opposite end of the OEM trunk cable.

## Product Description/Application

The Philips series of ECG Adapter products offer compatibility with several OEM trunk cable manufacturers. Refer to *Compatibility* for specific Leadset-Adapter-Trunk connection options.

The Adapters are reusable and suitable for multi-patient use when cleaned and disinfected between patient applications.

A *typical* Leadset-Adapter-Trunk connection is shown on page i at the front of this document and described below:

- 1 The **Adapter** has a female end that accepts a compatible 5-lead Philips ECG leadset connector, and a male end that connects to a compatible OEM trunk cable connector.
- 2 The Philips ECG **Leadset Connector** connects to the leadset end of the Adapter.
- 3 The OEM **Trunk Cable Connector** connects to the trunk end of the Adapter.
- 4 Multiple **ECG Patient Leads** on the Philips leadset attach to ECG electrodes appropriately positioned on the patient.
- 5 The OEM Trunk Cable **Monitor Connector** connects to an OEM patient monitor.
- 6 The Adapter's **Tether Strap** secures the Adapter to the strain relief on the OEM trunk cable connector.

---

## Compatibility

The Adapters are compatible with the Philips ECG leadsets and OEM trunk cables listed on page ii at the front of this document.

**Note:** Be sure to verify compatibility before using an Adapter with any unlisted trunk cable.

---

## Warnings

Warnings listed below apply to Adapter application and patient safety. Be sure to also reference *Notes*, *Cautions*, and *Warnings* defined by the Instructions for Use (IFU) associated with each compatible Philips leadset, OEM trunk cable, and OEM monitor.

- Use the Adapter only with compatible Philips leadsets and OEM trunk cable combinations.
  - Visually inspect the Adapter for damage, visual/gross contamination, and cleanliness prior to use. If inspection shows any signs of cracks or damage, or if there appears to be any liquid, lint, or dust in the leadset or trunk connectors, do not use the Adapter.
  - Do not use the Adapter under massive influence of fluids (e.g. rain).
  - Be sure that the leadset and trunk cable connectors are properly and fully inserted into the Adapter connectors. All connections must be secure.
  - Carefully position the Adapter, leadset, and trunk cable to avoid patient entanglement, choking, and strangulation. Also verify that positioning does not cause entanglement of IV tubes or restrict IV flow.
  - Always clean and disinfect the Adapter prior to use on another patient (see *Cleaning and Low-Level Disinfection*).
- 

## Disposal

Any Adapter showing signs of physical or electrical deterioration or failure should be disinfected and disposed of according to local laws regarding disposal of hospital waste.

---

## Cleaning and Low-Level Disinfection

The Adapter must be cleaned and disinfected before being reused on another patient. Wipe the Adapter with a soft cloth saturated with mild soap and water to clean, wipe it with alcohol to disinfect, then wipe it dry being sure to remove all residue.

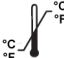
### Notes:

- Do not Sterilize the Adapter.
- Never immerse the Adapter.
- Use of cleaners other than mild soap, or disinfectants other than alcohol, may cause damage and reduce Adapter lifetime or cause safety risks to patients.

---

## Product Symbols

Symbols appearing on Adapter products and packaging are defined below:

 Quantity per pouch	<b>R<sub>X</sub> only</b> Prescription use only	 No latex	 Catalog number	 Caution consult instructions
 Reusable do not discard	 Storage temperature	 Storage humidity	 Reference Instruction Manual	

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For more information, please call the local Philips sales office listed in your telephone directory; or contact one of the regional sales offices listed below for the location of your nearest Philips sales office.

**United States:**


Philips Medical Systems  
North America Corporation  
3000 Minuteman Road  
Andover, MA 01810-1099  
+1 800 225-0230 toll free

**Canada:**

Philips Healthcare Canada  
281 Hillmount Road  
Markham, ON  
L6C 2S3  
(800) 291-6743

**Latin America Headquarters:**

Philips Healthcare Brazil  
Rua Verbo Divino 1400 - 7<sup>o</sup> Andar  
Chacara Santo Antonio  
04719-002 Sao Paulo, SP  
Brazil  
Tel: +55 112125-0744  
Fax: +55 11 2125-0761

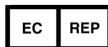**Europe, Middle East and Africa:**

Philips Medizin Systeme  
Böblingen GmbH  
Hewlett-Packard Str. 2  
71034 Böblingen, Germany  
Tel: (+49) 7031 463 2254  
Fax: (+49) 7031 463 1552

**Asia Pacific Headquarters:**

Philips Electronics Singapore Private  
Limited  
Philips Healthcare  
620A Lorong 1 Toa Payoh  
Singapore 319762  
Tel: 1800-PHILIPS  
Fax: +65 6255 4853

*<http://philips.com/healthcarestore>*

Made in China

Manufactured for  
Philips Medical Systems

**Manufacturing Address:**

Philips Medical Systems  
3000 Minuteman Road  
Andover, MA 01810-1099 USA

Contains no Latex

Subject to modification

© 2012

Koninklijke Philips Electronics N.V.  
Printed in China September 2012

Part No. 453564381741