

# Pangea<sup>TM</sup>

## Distal Posterior Tibia Plate


Design rationale



# Pangea Distal Posterior Tibia Plate

## Design rationale

**stryker**

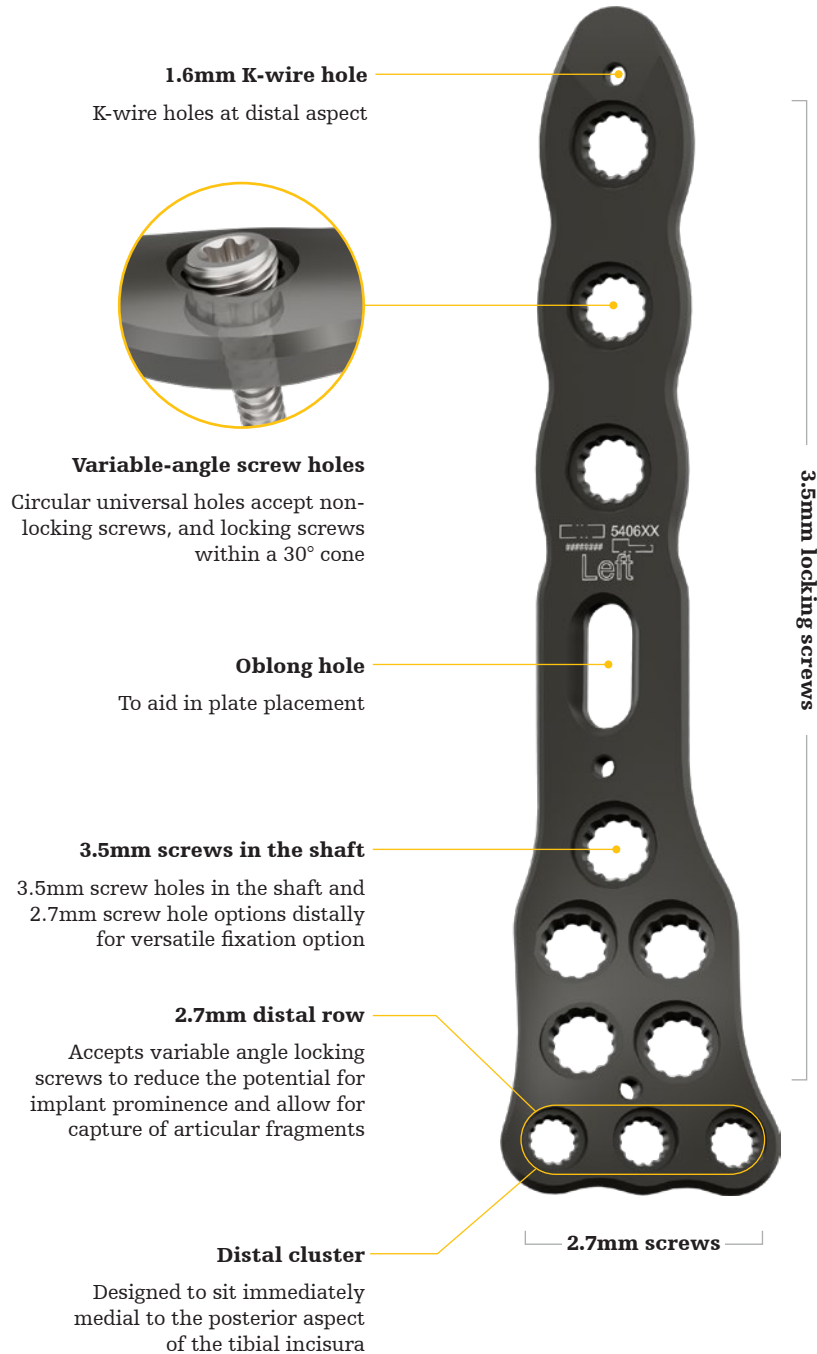

## Plate placement



- The plate is placed on the posterior surface of the distal tibia, specifically to sit on the lateral side of the posterior surface over the posterior malleolus.
- Proper position is achieved when the distal end of the plate is adjacent to the articular surface, allowing for the distal screws to support the plafond.
- Contour of the distal plate is designed to allow placement immediately proximal to the joint line for rafting while reducing plate prominence.



# Pangea Distal Posterior Tibia Plate

## Design rationale

**stryker**

### Fit


- Distal Posterior Tibia Plate is designed to act as a buttress of the posterior malleolus or posterior fixation of pilon fractures.
- The rotation of the plate is designed to allow for central placement on the tibial shaft while facilitating capture of the posterior malleolus.
- The fit and placement is designed to reduce the potential for irritation of the flexor hallucis longus (FHL tendon).
- Designed with the use of SOMA: Stryker Orthopedics Modeling and Analytics.
- SOMA analysis shows better anatomic plate fit of the Distal Posterior Tibia plate compared to competitive VA-LCP distal tibia plates.<sup>2</sup>

### Technical specification


- Standard plate lengths: 2-10 hole (75-187mm)
- Shaft thickness: 3.3mm
- Distal thickness: 2.3mm
- Left and right anatomic plate options
- **Drill bits for T8:**
  - Ø2.0mm x 135mm (542000(S))
  - Ø2.0mm x 175mm (542001(S))
- **Drill bits for T15:**
  - Ø2.5mm x 135mm (542020)
  - Ø2.5mm x 215mm (542021)

### Screw platform

#### T15 screw platform

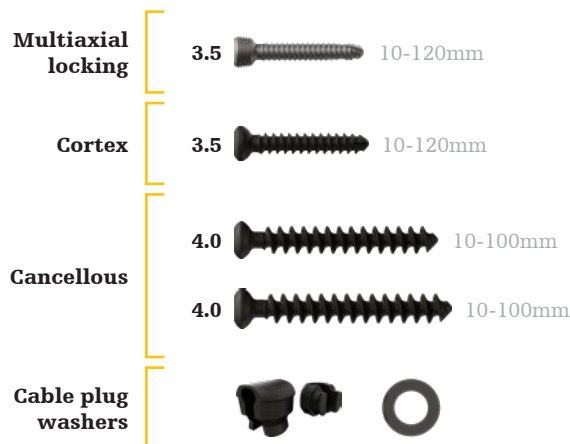

#### T8 screw platform

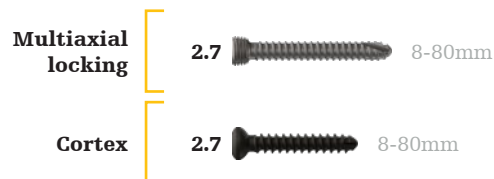

#### References:

1. Pangea Distal Tibia Plating Operative Technique. PGA-ST-5, 03-2023
2. Internal Report № D0000262573, Rev AA, Selzach, Switzerland

This document is intended solely for the use of healthcare professionals. A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker's product offerings. A surgeon must always refer to the package insert, product label and/ or instructions for use before using any of Stryker's products. Products may not be available in all markets because product availability is subject to the regulatory and/ or medical practices in individual markets. Please contact your sales representative if you have questions about the availability of products in your area. Not intended for promotional or marketing use outside of the US and Canada.

Stryker Corporation or its affiliates own, use, or have applied for the following trademarks or service marks: Pangea, Stryker. All other trademarks are trademarks of their respective owners or holders.

Content ID: PGA-BL-3\_Rev-1, 12-2024  
Copyright © 2023 Stryker



#### Manufacturer:

Stryker GmbH  
Bohnackerweg 1  
2545 Selzach, Switzerland

[stryker.com](https://www.stryker.com)