

COVID-19 update

Product supply

Flyte

Personal Protection System

Flyte and T4/T5 togas and hoods are components of Stryker’s personal protection system and are intended to protect the patient, healthcare personnel and operating room personnel against contamination, exposure of infectious bodily fluids and the transfer of microorganism and particulate material. *Our togas and hoods are sterile and single use only.

Stryker’s toga and hoods offer different AAMI level protection levels and protection zones. The AAMI standard defines critical and noncritical zones for surgical gowns. The AAMI level of a surgical gown is classified by the protection of the material in the critical zone. The critical zone of a surgical gown comprises the front area of the gown from chest to knees and the sleeves from the cuff to above the elbow.

The protection levels and areas in Stryker’s personal protection equipment (PPE) products are shown below.

Flyte Togas	<p>Zippered toga without peel away lens 0408-820-000, Large 0408-830-000, X Large 0408-840-000, 2X Large 0408-850-000, 3X Large</p> <p>Zippered toga with peel away lens 0408-820-100, Large 0408-830-100, X Large 0408-840-100, 2X Large 0408-850-100, 3X Large</p> <p>Pullover toga 0408-720-000, Large 0408-730-000, X Large 0408-740-000, 2X Large 0408-750-000, 3X Large</p>	<p>Sleeve material, sleeve seam and toga front material AAMI Level 4</p> <p>*Hood material AAMI Level 4</p> <p>Filter material ASTM F2100 Level 1</p>
-------------	---	--

COVID-19 update

Product supply

Flyte Hoods	<p>Hood without peel away lens 0408-800-000 0408-800-400</p> <p>Hood with peel away lens 0408-800-100 0408-800-500</p>	<p>*Hood material AAMI Level 4</p> <p>Filter material ASTM F2100 Level 1</p>
Flyte SurgiCool Togas	<p>Zippered toga 0408-821-000, Large 0408-831-000, X Large 0408-841-000, 2X Large 0408-851-000, 3X Large</p> <p>Zippered toga with peel away lens 0408-821-100, Large 0408-831-100, X Large 0408-841-100, 2X Large 0408-851-100, 3X Large</p> <p>Pullover Toga 0408-721-000, Large 0408-731-000, X Large 0408-741-000, 2X Large 0408-751-000, 3X Large</p>	<p>Sleeve material, sleeve seam and toga front material AAMI Level 4</p> <p>*Hood material AAMI Level 1 ASTM F2100 Level 1</p> <p>Filter material ASTM F2100 Level 1</p>
Flyte SurgiCool Hoods	<p>Hood without peel away lens 0408-801-400</p> <p>Hood with peel away lens 0408-801-500</p>	<p>*Hood material AAMI Level 1 ASTM F2100 Level 1</p> <p>Filter material ASTM F2100 Level 1</p>
T4 Togas	<p>Zippered togas 0400-810-000 0400-820-000 0400-830-000 0400-840-000 0400-850-000</p> <p>Pullover togas 0400-710-000 0400-720-000 0400-750-000 0400-760-000 0400-770-000</p>	<p>**Sleeve material and toga front material AAMI Level 4</p> <p>**Sleeve seam AAMI Level 1</p> <p>*Hood material AAMI Level 1 ASTM F2100 Level 1</p>

COVID-19 update

Product supply

T4 Hoods	Hood 0408-800-000	*Hood material AAMI Level 1 ASTM F2100 Level 1
T5 Togas	Zippered togas 0400-820-100 0400-850-100	**Sleeve material and toga front material AAMI Level 4 **Sleeve Seam AAMI Level 1 *Hood Material AAMI Level 1 ASTM F2100 Level 1
T5 Hoods	Hood 0400-800-100	*Hood material AAMI Level 1 ASTM F2100 Level 1

*Per AAMI PB70: Hood material is exempt from the standard, but the material meets the requirement of the listed AAMI Level

**T4/T5 Togas & Hoods are not cleared as meeting a barrier level of AAMI PB70 but the material meets the requirement of the listed AAMI Level.

Stryker’s PPE products are not cleared as respiratory protective products. CDC guidance for respiratory protection directs the use of an N95 or higher rated mask for staff performing Aerosol Generating Procedures, Coronavirus Disease 2019 (COVID-19) | CDC . In the context of PPE, respiratory protective products are devices that protect the wearer from inhalation of a hazardous atmosphere at a prescribed level. Stryker’s PPE products do not provide this level of respiratory protection. The hoods and togas provide resistance to fluid and viral penetration as described further above, whereas the N95 respirator provides respiratory protection. Healthcare personnel should adhere to Standard, Contact, and Airborne Precautions, including the use of eye protection (e.g., goggles).

N95 respirators can be used under any of our togas or hoods.

If you have any questions regarding our PPE products, please contact our customer service or your Orthopaedic Instruments sales rep:

Customer service
800 253 3210 toll free
inst.stryker.cs@stryker.com

COVID-19 update

Product supply

*A non-sterile version of our hoods and togas is being evaluated, additional information will be made available at a later date.