

# Tornier Perform<sup>®</sup> Reversed Augmented Glenoid

## Operative technique



This publication sets forth detailed recommended procedures for using Stryker devices and instruments. It offers guidance that you should heed, but, as with any such technical guide, each surgeon must consider the particular needs of each patient and make appropriate adjustments when and as required.

### Important

- The patient should be advised that the device cannot and does not replicate a normal healthy bone, that the device can break or become damaged as a result of strenuous activity or trauma and that the device has a finite expected service life.
- Removal or revision of the device may be required sometime in the future.
- Cleaning and sterilization information is provided in the applicable instructions for use.
- Non-sterile devices, including implants and instruments, must be cleaned and sterilized prior to use, in accordance with validated methods.
- Devices that are able to be disassembled should be disassembled prior to point-of-use processing. Additionally, devices with movable components that do not facilitate disassembly should be manually articulated during the point-of-use processing step in order to evacuate additional soils.
- Please remember that the compatibility of different product systems has not been tested unless specified otherwise in the product labeling.
- Consult Instructions for Use (<https://ifu.stryker.com>) for a complete list of potential adverse effects and adverse events, contraindications, warnings and precautions.
- The surgeon must advise patients of surgical risks, and make them aware of adverse effects and alternative treatments.
- An implant whose packaging is open or damaged or whose expiration date has passed must not be used. Every precaution must be taken to ensure sterility when opening the packaging of the implant and during implantation.

# Tornier Perform Reversed Augmented Glenoid

## Table of contents

<b>Overview</b> .....	<b>4</b>
• Tornier Perform Reversed Augmented Glenoid.....	4
• Indications/contraindications .....	4
• Pre-operative planning .....	5
• Glenoid exposure .....	5
<b>Surgical steps – half-wedge augment</b> .....	<b>6</b>
• Sizing the glenoid and pin placement .....	6
• Resurfacing the glenoid (paleo ream).....	8
• Augment reaming (neo ream) .....	8
• Augment trial .....	10
• Baseplate post and central screw drilling.....	10
• Sizing for central screw.....	11
• Central screw tap .....	11
• Baseplate assembly and insertion.....	12
• Final implantation .....	13
• Peripheral screw drilling, insertion and reaming .....	13
<b>Surgical steps – full-wedge augment</b> .....	<b>14</b>
• Sizing the glenoid and pin placement .....	14
• Pilot hole preparation .....	14
• Augment reaming (neo ream) .....	14
• Augment trial .....	15
• Baseplate post and central screw drilling.....	16
• Sizing for central screw.....	17
• Central screw tap .....	17
• Baseplate assembly and insertion.....	18
• Final implantation .....	18
• Peripheral screw drilling, insertion and reaming .....	20
<b>Press-fit post options</b> .....	<b>21</b>
• Drilling for press-fit short post .....	21
• Drilling for press-fit long post .....	21
• Baseplate assembly and insertion.....	22
<b>Appendix</b> .....	<b>23</b>
• Augment reamer assembly instructions.....	23
• Glenosphere and baseplate configuration chart.....	25
• Peripheral screw angulation .....	25
<b>Product specifications</b> .....	<b>26</b>

## Overview

### Tornier Perform Reversed Augmented Glenoid

The Tornier Perform Reversed Augmented Glenoid must be used in association with a compatible Stryker humeral component:\*

- humeral implants Tornier Perform Humeral System, Aequalis Flex Revive Shoulder System and Tornier Flex Shoulder System in reverse configuration
- or humeral implants Aequalis Reversed, Aequalis Reversed FX or Aequalis Adjustable Reversed Shoulder System,
- or humeral implants Aequalis Reversed FX2

Stryker's shoulder prostheses are intended for replacement of the shoulder joint to reduce pain and improve shoulder mobility for patients with designated indications.

### Indications

The Tornier Perform Reversed Augmented Glenoid is indicated for use as a replacement of shoulder joints for patients with a functional deltoid muscle and with massive and non-repairable rotator cuff-tear with pain disabled by non-inflammatory degenerative joint disease (i.e. osteoarthritis and avascular necrosis), rheumatoid arthritis, traumatic arthritis, fractures of the humeral head, revision of the devices if sufficient bone stock remains or correction of functional deformity.

### Contraindications

Absolute contraindications for shoulder arthroplasty are non-functional deltoid; paralysis of the axillary nerve; active local or systemic infection; sepsis and osteomyelitis; poor quality and insufficient quantity of glenoid bone stock; pre or peri-operative glenoid fracture; acromion fracture; elevation of sedimentation rate unexplained by other disease; elevation of WBC count, or marked shift in WBC differential count; significant injury to the upper brachial plexus.

Relative contraindications for shoulder arthroplasty are uncooperative patients or patients with neurologic disorders who are not capable of following directions; neuromuscular disease (e.g. joint neuropathy); osteoporosis; metabolic disorders which may impair bone formation; osteomalacia; distant foci of infections that may spread to the implant site; or rapid joint destruction, marked bone loss or bone resorption.

\*Not all glenoid and humeral components are available in all geographies.

## Pre-operative planning

Pre-operative planning is performed utilizing x-rays including a true anterior/posterior view of the glenohumeral joint or axillary views. The use of a CT scan or MRI is recommended to better determine the orientation of the glenoid, the quality of glenoid bone stock and to evaluate the integrity of the rotator cuff.

A careful analysis of X-rays and CT scan views is recommended before surgery to evaluate the following parameters: osteophytes, anterior, superior, posterior, and inferior wear of the glenoid, as well as the location, orientation and depth of the glenoid vault and presence of subcortical cysts.

## Glenoid exposure

Exposure of the glenoid is one of the more technically difficult aspects of shoulder arthroplasty. The size of the patient, soft tissue contractures, bony morphology, and the sequelae of previous surgeries are some of the potential challenges to adequate exposure. A thorough understanding of the neuroanatomy and techniques for protecting the axillary nerve, in particular, are routinely used to achieve successful exposure. In brief, a standard deltopectoral approach is typically used, with retraction of the deltoid laterally and pectoralis and conjoined tendon medially. A superior approach may also be utilized. Humeral exposure is performed per surgeon preference with appropriate subscapularis techniques and humeral head resection. The proximal humerus is then retracted posteriorly and access to the glenoid is gained. Residual labral tissue is excised, biceps tendon is released, and the capsule is released from the glenoid anteriorly, inferiorly, and posteriorly. Special attention is given for protection of the axillary nerve inferiorly. Appropriate glenoid retractors are then inserted and additional exposure techniques can then be used as needed. Please reference the Tornier Approach Shoulder Arthroplasty Technique (AP-015238) for additional details.

## Surgical steps – half-wedge augment

Tornier Perform Reversed Augmented Glenoid instrumentation allows for use of different surgical techniques to better suit the clinical situation and surgeon preference. The instruments have been designed to increase the safety of the procedure and to assist the surgeon in obtaining accurate and reproducible results. The instrumentation only allows for a cannulated preparation referencing a guide pin positioned at a chosen orientation. To minimize bone loss during an augmented procedure the surgeon can utilize the augmented trials prior to reaming to determine what type of augment will be most appropriate for the patient. The augment option that preserves the most native glenoid bone should be utilized.

**Note:** These devices are contraindicated in cases of significant bone loss where poor quality or insufficient quantity of glenoid bone stock exists.

### Sizing the glenoid and pin placement

Two types of pin guides are available (circular or anatomic). | **Figure 1** The circular guide has the same outer diameter as the glenoid baseplate in 25mm or 29mm diameters. The anatomical pin guides come in four sizes (S=small, M=medium, L=large, and XL=extra-large) that correspond to the varying patient anatomies. The anatomical pin guides have an inferior offset built in, which positions the pin 12mm from the bottom of the guide.<sup>1</sup> Two pin guide handles are offered in the instrument set, a 0° or 10° inferior tilt handle. The 0° pin guide handle can be used to prepare the baseplate perpendicular to the glenoid. The 10° pin guide handle can be used to place a 10° inferior tilt to the baseplate. The guides are assembled by rotating the distal end of the pin guide handle into the pin guide clockwise until it is fully seated. | **Figure 2**

According to surgeon preference, exposure, and surgical approach, the offset pin guide handle can be attached to the straight pin guide handles by sliding the offset handle down the shaft of the straight handle until it snaps in place. | **Figure 3** Use of the offset handle can provide better visualization as the guide pin is placed.

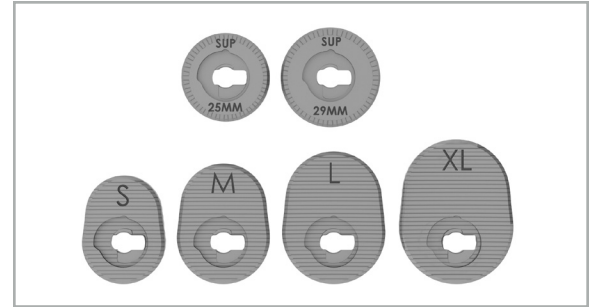


Figure 1

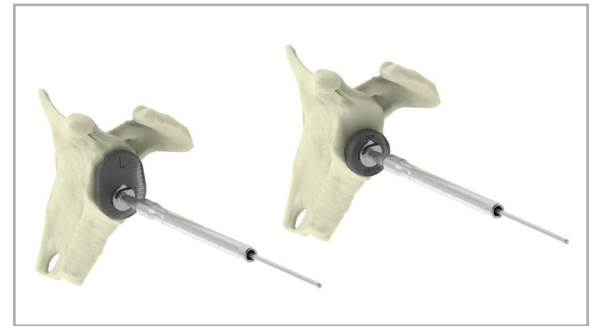



Figure 2



Figure 3


<sup>1</sup> Kelly, James D., C. Scott Humphrey, and Tom Norris. "Optimizing glenosphere position and fixation in reverse shoulder arthroplasty. Part One: The twelve-mm rule." *Journal of Shoulder and Elbow Surgery* 17.4 (2008): 589-594.

While referencing the face of the glenoid and appropriately seating the assembled pin guide on the inferior edge of the glenoid to reduce the risk of impingement, drill the 2.5mm guide pin through the guide pin handle until bicortical fixation is achieved. Care should be taken when placing the pin on a glenoid with abnormal erosion. | **Figure 4**

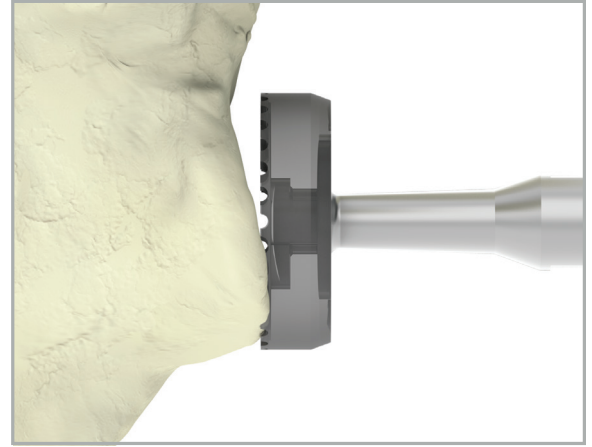

**Figure 4**

Once the 2.5mm guide pin is fixed in the glenoid with bicortical fixation, remove the drill and the pin guide assembly. Finally, before reaming, check to ensure the guide pin is accurately placed on the glenoid and no adjustments are needed. It is important to check the guide pin condition after every step of the glenoid preparation. If the guide pin is damaged or bent, a new guide pin should be inserted.

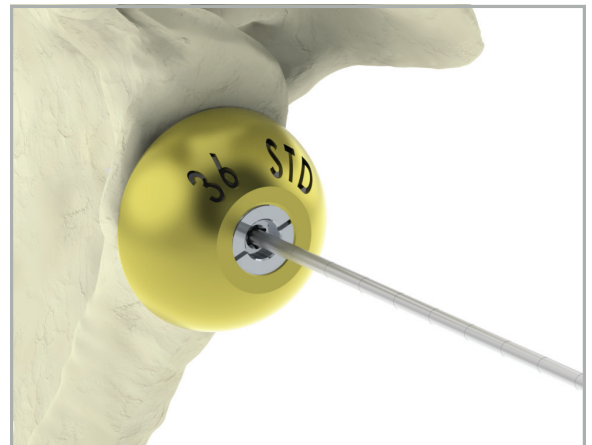

**Figure 5**

**Note:** An optional trialing step to estimate glenoid position can be performed at this point using the guide pin and the glenosphere trials. | **Figure 5**

## Resurfacing the glenoid (paleo ream)

To obtain complete seating and secure fixation of the glenoid baseplate, it is important to create a flat glenoid surface using the cannulated baseplate reamer of the same diameter of the baseplate that will be used. Half-moon reamers are provided standard in the Tornier Perform Reversed Glenoid instrument set. If preferred, full-moon reamers are available upon request.

Connect the appropriate reamer to power and select the reaming option on the drill. Slide the assembly onto the guide pin and ream.

Only the paleo glenoid, or surface that resembles the natural glenoid shape, should be reamed flat to best fit the implant. Care should be taken to not completely ream the entire glenoid surface. It is recommended to start the reamer before contacting the glenoid surface and ream until the paleo glenoid surface is flat. | **Figure 6**

If insertion of the reamer is difficult, remove or reposition retractors for greater exposure. A T-handle is provided in all of Stryker's humeral instrument sets if manual reaming is desired. Preserve as much bone as possible to support good primary fixation.



**Figure 6**

**Note:** Overly aggressive reaming should be avoided to minimize the risk of glenoid fracture.

## Augment reaming (neo ream)

After the paleo surface of the glenoid has been adequately prepared, ream the defected bone or the neo portion of the glenoid. When assembling the augment reamer (see instructions in the appendix) make sure the angle indicator is positioned at the 35° angle before starting to ream. Choose the appropriately sized neo reamer and neo reamer depth stop size based on whether a 25mm or 29mm sized baseplate will be used. The arrow on the depth stop indicates the position that is 180° opposite of the deepest point of the wedge. This arrow corresponds to marking on several other instruments that will follow. An anatomical mark on the rim of the glenoid made with a bovie can also be created to keep the location identified through the duration of the surgery.



Begin by placing the neo reamer assembly over the guide pin and slide the assembly down to the face of the glenoid. Make sure that the appropriate size neo reamer depth stop is aligned with the paleo reamed surface.

| **Figure 7A and 7B**

**Note:** It is imperative to maintain alignment with the guide pin and rotational position while neo reaming.


The neo reamer is intended to be power driven only. Begin reamer rotation prior to contacting the glenoid surface then apply light pressure. This will help reduce the risk of fracture. Take extra care to keep the neo reamer axially aligned with the guide pin so that the depth stop sits flush on the paleo surface when reaming is complete.

Depending on the defect, you will need to progressively ream until the depth stop is seated on the paleo surface. This will be determined in the next step with the augment trials. Care should be taken to keep the reamer steady to maintain alignment if progressive reaming is necessary.


The goal of neo reaming is to obtain a bony surface that matches the backside of the glenoid component while removing as little bone as possible.

**Note:** Over aggressive reaming should be avoided to prevent possible glenoid fracture and improper fit of the implant.


Once neo reaming is complete, remove the neo reamer assembly from the guide pin and out of the joint. | **Figure 8**

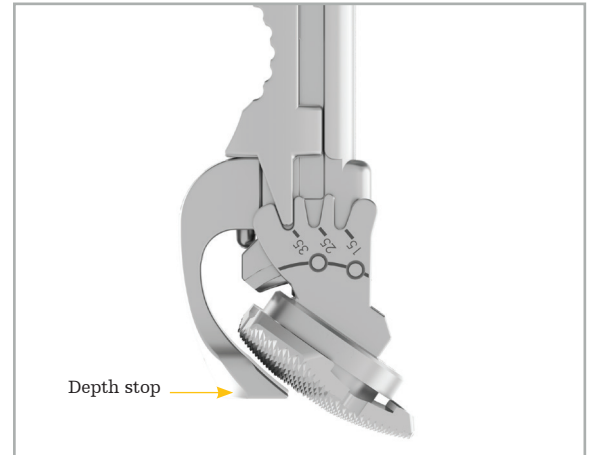

**Figure 7A**

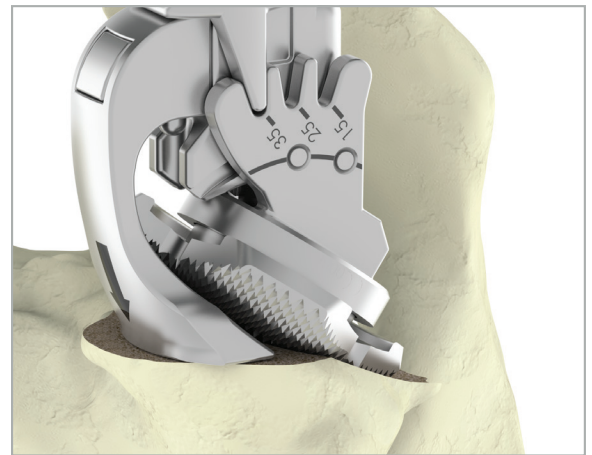

**Figure 7B**

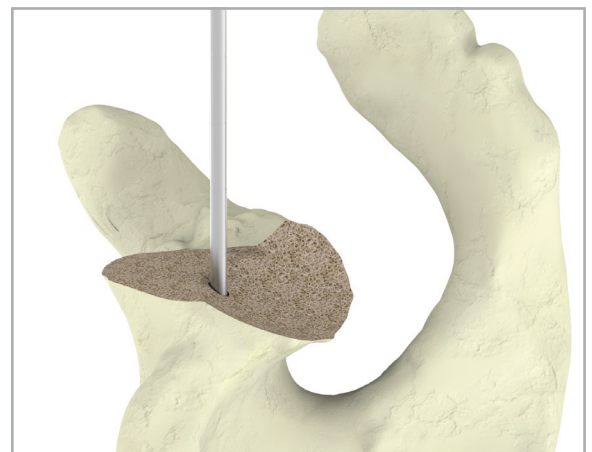

**Figure 8**

## Augment trial

With the neo surface prepared the corresponding augmented trial can be used to evaluate if proper neo reaming has been achieved. Make sure the trial being used to validate the fit is the appropriate size. The augmented trial should seat flush and stable on the reamed surface and match the location of the laser mark location of the depth stop above. If instability (rocking) or gaps are observed additional paleo and/or neo reaming may be required to correct. Recheck following any corrective reaming to ensure the final fit is flush and stable. | **Figure 9**



**Figure 9**

## Baseplate post and central screw drilling

The hole for the baseplate post is drilled over the guide pin using the cannulated 10mm diameter drill bit. A positive stop on the drill bit ensures that drilling will not go too deep and allows for press-fit fixation of the post. Be sure to stop the post drill at the paleo surface. | **Figure 10**

Remove the guide pin.

The surgeon determines the diameter of the central screw drill guide based on patient bone quality. It is recommended to start with a 6.5mm diameter drill bit as the hole can be expanded if necessary. 9.5mm diameter screws are recommended if inadequate fixation is achieved with 6.5mm diameter screw secondary to poor bone quality or for revision cases.

Place the corresponding half-wedge central screw drill guide into the hole in the glenoid face that was created using the baseplate post drill. Be certain that the wedge portion of the drill guide is placed in the neo reamed defect that was created to ensure proper stability while reaming.

| **Figure 11A and 11B** Align the laser marked arrow on the drill guide to the location identified in the steps above. The central screw hole is drilled using a 6.5mm or 9.5mm diameter drill bit. Laser marks can be used to help approximate the final implant length. The drilling is performed under power. Palpation of the drill bit tip can be performed to confirm the drill bit has exited the anterior cortex.



**Figure 10**



**Figure 11A**



**Figure 11B**

## Sizing for central screw

To determine the final central screw length, the central screw depth gauge is used. | **Figure 12** The gauge measures the recommended screw length. The actual prepared hole is approximately 3mm less to allow for bicortical fixation.

To ensure an accurate evaluation of the final screw length, make sure the flat end of the depth gauge is contacting the paleo surface of the glenoid.

The length of the central screw is matched with the color and number that appears on the depth gauge. If you fall on a line above a color, choose the length above the line. | **Figure 13**

**Note:** If the gauge measures between two lengths, pick screw length that is shown. (i.e. the depth gauge reads in the middle of the green/30mm band, choose the 30mm central screw).

## Central screw tap

Although the central screws are self-tapping, after measuring the central hole, the tap can be used to prepare the threads of the final implant and reduce the possibility of glenoid fracture in cases for hard bone.

**Note:** Tapping is recommended when using the 9.5mm central screw in order to prevent glenoid fracture.

Tapping should be done manually by connecting it to a T-handle (do not use with power). When tapping, it is important to maintain alignment to the axis of the previously drilled hole. There are laser markings on the tap to show depth. | **Figure 14** The tapping depth should be chosen similar to the depth of the drilled central hole. Using the measurements of the central screw length, stop at the level of the corresponding laser mark.



**Figure 12**



**Figure 13**



**Figure 14**

## Baseplate assembly and insertion

The final baseplate is chosen according to the reamed glenoid surface (25mm half-wedge or 29mm half-wedge). Additionally, the final central screw is chosen according to the measured length using the central screw depth gauge.

Ensure that the inner shaft of the baseplate inserter is backed out to the point where it moves freely within the outer sleeve yet is still contained. While lining up the pegs on the inserter with the peg holes on the baseplate, snap the inserter onto the baseplate. Screw the inner shaft down the sleeve to capture the baseplate onto the inserter. Care should be taken to ensure that the two pegs on the inserter seat properly into their respective holes on the baseplate. | **Figures 15A and 15B**

**Note:** Assemble the inserter with the arrow of the inserter opposite of the wedged portion of the baseplate.

There is a 6.5mm and 9.5mm slot corresponding to the screw diameter. The hex head portion of the screw is orientated in the up position.

| **Figure 16**

The baseplate inserter with baseplate attached is placed onto the screw and turned in a counterclockwise manner. Turn the baseplate until it is fully seated onto the screw. There will be a slight drop of the baseplate indicating that it has fully seated. The baseplate will spin independently from the screw once seated. | **Figures 17A and 17B** The baseplate/screw can be removed from the assembly tool.


It is important to continuously check the orientation of the baseplate relative to the prepared hole and reamed surface to ensure accurate implantation of the half-wedge baseplate. The baseplate inserter also has a laser marked arrow that can be used to align with corresponding arrows from previous instruments.



**Figure 15A**



**Figure 15B**

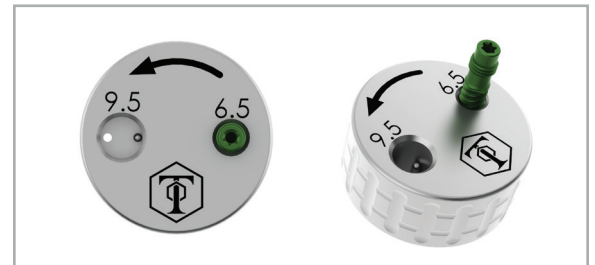

**Figure 16**



**Figure 17A**



**Figure 17B**

## Final implantation

Insert the baseplate inserter screwdriver down the shaft of the baseplate inserter and engage the head of the central screw. To insert the assembled baseplate, place the screw into the central screw drill hole making sure to align the laser mark on the baseplate inserter to the location identified above. | **Figure 18** Turn the central screw in a clockwise manner and screw the baseplate into the prepared glenoid until it has fully seated against the surface. There will be a slight audible clicking noise once the post begins to engage the prepared bone. This is normal and is due to the free-floating nature of the screw within the assembly.



**Figure 18**

**Note:** At the completion of glenoid component installation, the central locking screw of the glenosphere locks the central compression screw into the baseplate, creating a locked fixed angle implant.

Take care to ensure proper rotational orientation of the baseplate when screwing the baseplate down. | **Figure 19** Once the baseplate is seated flush on the glenoid surface, the baseplate inserter can be detached from the baseplate.



**Figure 19**

**Note:** The baseplate should be seated completely onto the prepared glenoid surface. Avoid over-tightening or excessive advancement of the baseplate into the subchondral bone. Gaps between the baseplate and glenoid surface should also be avoided.

**Note:** If the 6.5mm screw strips a 9.5mm screw can be used. This is accomplished by removing the baseplate and installing the 9.5mm screw in place of the 6.5mm screw.

## Peripheral screw drilling, insertion, and reaming


The half-wedge augmented baseplates contain one peripheral screw hole which is compression and located on the thick portion of the wedge. The other peripheral screw holes are multidirectional locking (see appendix for peripheral screw angulations).

The remaining steps for peripheral screw insertion, peripheral reaming and glenosphere attachment can be performed by following the standard procedure in the Tornier Perform Reversed Glenoid operative technique.

## Surgical steps – full-wedge augment

### Sizing the glenoid and pin placement


Using the same cannulated approach as described in steps 1-4 above, a single-use 2.5mm guide pin will be placed using a combination of pin guides and pin guide handles. Place the pin guide onto the glenoid surface making sure that its bottom surface is seated on the bone. Care should be taken when placing the pin in the glenoid that contains a defect. Make sure that the pin guide is placed onto the glenoid where the best possible position can be achieved. To limit any risk of impingement, it is important to properly position the pin guide referencing the inferior glenoid edge. | **Figure 20**

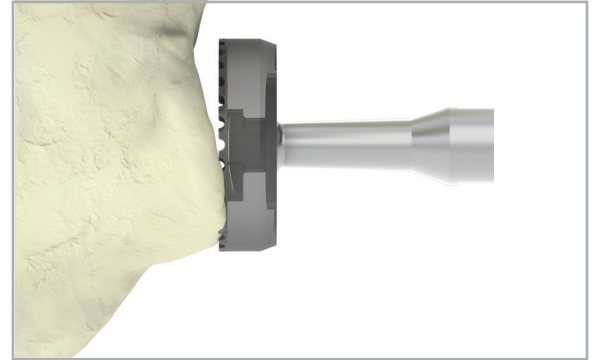

**Figure 20**

### Pilot hole preparation

A pilot hole must be drilled in order for the full-wedge augment reamer assembly to properly ream the neo surface of the glenoid. Drill until the positive stop has been reached. | **Figure 21A and 21B**

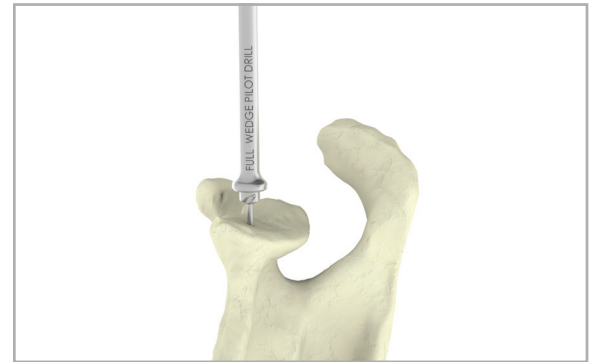

**Figure 21A**



**Figure 21B**

### Augment reaming (neo ream)

When assembling the augment reamer, refer to the assembly instructions in the appendix. Choose the appropriate neo reamer to match the baseplate diameter to be used, 25mm or 29mm.

Make sure the angle indicator is positioned at the 15° angle before first starting to ream. | **Figure 22** The neo reamer is intended to be power driven only.

**Note:** The depth stop is not used for the full-wedge preparation. Alignment can be determined by positioning the depth stop connection point opposite to the deepest portion of the defect.



**Figure 22**

Place the neo reamer assembly over the guide pin and slide the assembly down to the face of the glenoid. Begin reamer rotation prior to contacting the glenoid surface then apply light pressure (let the cutter do the work).

| **Figure 23**


**Note:** It is imperative to maintain alignment with the guide pin and rotational alignment to the glenoid while neo reaming.

The goal of neo reaming is to obtain a bony surface that matches the backside of the glenoid component while removing as little bone as possible.

Once neo reaming is complete, remove the neo reamer assembly from the guide pin and out of the joint. | **Figure 24**

### Augment trial

With the neo surface prepared, the corresponding augmented trial can be used to evaluate if proper reaming has been achieved. Make sure the trial being used to validate the fit is the appropriate size. The augmented trial should seat flush and stable on the reamed surface and match the location of the laser mark location of the depth stop above. If instability (rocking) or gaps are observed additional neo reaming may be required to correct. Recheck following any corrective reaming to ensure the final fit is flush and stable. | **Figure 25**

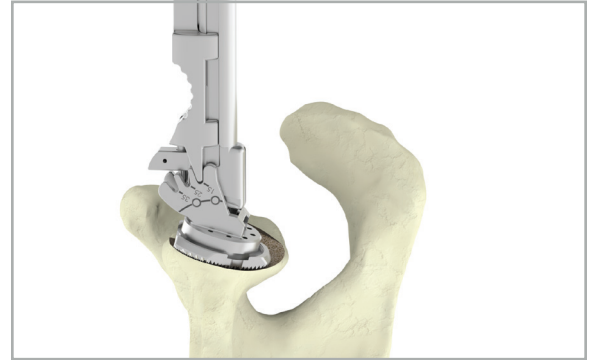

**Figure 23**



**Figure 24**



**Figure 25**

## Baseplate post and central screw drilling

The hole for the baseplate post is drilled over the guide wire using the cannulated 10mm diameter drill bit. A positive stop on the drill bit maintains drilling will not go too deep and ensures a press-fit fixation for the post. **Be sure to fully seat the post drill on the neo reamed surface of the glenoid.** | **Figure 26**

Remove the guide pin.

The surgeon determines the diameter of the central screw drill bit based on patient bone quality. It is recommended to start with the 6.5mm diameter drill bit as the hole can be expanded if necessary. 9.5mm diameter screws are recommended if inadequate fixation is achieved with a 6.5mm diameter screw or for revision cases.

Place the corresponding full-wedge central screw drill guide into the hole in the glenoid face that was made by the baseplate post drill. Align the laser mark on the drill guide to the location identified in the steps above. Be certain that the wedge portion of the drill guide is placed in the neo reamed defect that was created to ensure proper stability while reaming. The central screw hole is drilled using a 6.5mm or 9.5mm diameter drill bit. Laser marks can be used to approximate the final implant length. | **Figure 27** The drilling is performed under power over the guide wire. Palpation of the drill bit tip can be performed to confirm the drill bit has exited the cortex.

After the drill bit has penetrated the anterior cortex, the laser markings on the drill bit can be used in addition with the depth gauge as a guide to determine the size of the central screw to be used when assembling the baseplate.



**Figure 26**



**Figure 27**



## Sizing for central screw

To determine the final central screw length, the central screw depth gauge is used. | **Figures 28A and 28B** The gauge measures the recommended screw length. The actual prepared hole is approximately 3mm less to allow for bicortical fixation.

To ensure an accurate evaluation of the final screw length, make sure the flat end of the depth gauge is contacting the neo reamed surface of the glenoid.

The length of the central screw is matched with the color and number that appears on the depth gauge. If you fall on a line above a color, choose the length above the line.

**Note:** If the gauge measures between two lengths, pick screw length that is shown. (i.e. The depth gauge reads in the middle of the green/30mm band, choose the 30mm central screw).

## Central screw tap

Although the central screws are self-tapping, after measuring the depth of the central hole, the tap can be used to prepare the threads of the final implant and reduce the possibility of glenoid fracture in cases for hard bone. Tapping is recommended when using the 9.5mm central screw in order to prevent glenoid fracture.

Tapping should be done manually by connecting it to a T-handle (do not use with power). When tapping, it is important to maintain alignment to the axis of the previously drilled hole. There are laser markings on the tap to show depth. | **Figure 29** The tapping depth should be chosen similar to the depth of the drilled central hole. Using the measurements of the central screw length, stop at the level of the corresponding laser mark.



Figure 28A



Figure 28B



Figure 29

## Baseplate assembly and insertion

The final baseplate is chosen according to the reamed glenoid surface (25mm full-wedge or 29mm full-wedge). Additionally, the final central screw is chosen according to the measured length using the central screw depth gauge.

Ensure that the inner shaft of the baseplate inserter is backed out to the point where it moves freely within the outer sleeve yet is still contained. While lining up the pegs on the inserter with the peg holes on the baseplate, snap the inserter onto the baseplate. Screw the inner shaft down the sleeve to capture the baseplate onto the inserter. Care should be taken to ensure that the two pegs on the inserter seat properly into their respective holes on the baseplate. | **See Figures 15A and 15B**


There is a 6.5mm and 9.5mm slot corresponding to the screw diameter. The hex head portion of the screw is orientated in the up position. | **See Figure 16**

The baseplate inserter with baseplate attached is placed onto the screw and turned in a counterclockwise manner. Turn the baseplate until it is fully seated onto the screw. There will be a slight drop of the baseplate indicating that it has fully seated. The baseplate will spin independently from the screw once seated. | **See Figures 17A and 17B** The baseplate/screw can be removed from assembly tool.

**Note:** It is important to continuously check the orientation of the baseplate relative to the prepared hole and reamed surface to ensure accurate implantation of the full-wedge baseplate to fit properly.

## Final implantation

Insert the baseplate inserter screwdriver down the shaft of the baseplate inserter and engage the head of the central screw. To insert the assembled baseplate, place the screw into the central screw drill hole and turn the central screw in a clockwise manner. | **Figure 30** Screw the baseplate into the prepared glenoid until it has fully seated against the surface. There will be a slight audible clicking noise once the post begins to engage the prepared bone. This is normal and is due to the free-floating nature of the screw within the assembly.

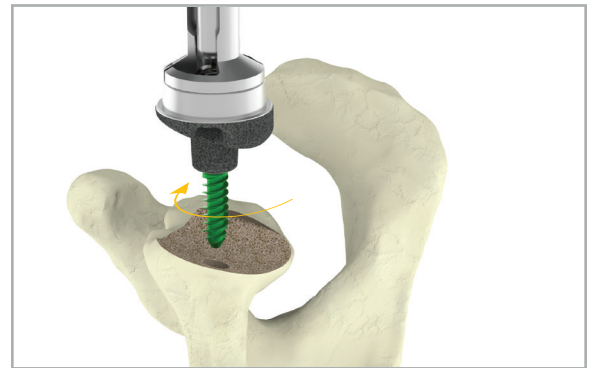

**Figure 30**

**Note:** At the completion of glenoid component installation, the central locking screw of the glenosphere locks the central compression screw into the baseplate, creating a locked fixed angle implant.

Take care to ensure proper rotational orientation of the baseplate when screwing the baseplate down. Once the baseplate is seated flush on the glenoid surface, the baseplate inserter can be detached from the baseplate. | **Figure 31**

**Note:** The baseplate should be seated completely onto the prepared glenoid surface. Avoid over-tightening or excessive advancement of the baseplate into the subchondral bone. Gaps between the baseplate and glenoid surface should also be avoided.

**Note:** If the 6.5mm screw strips a 9.5mm screw can be used. This is accomplished by removing the baseplate and installing the 9.5mm screw in place of the 6.5mm screw.

**Note:** Longer peripheral screws are required to account for the augmented offset from the bone. A minimum peripheral screw length of 26mm should be used for the superior and inferior screw holes to ensure sufficient bone purchase.



**Figure 31**

**Note:** Intra-operative removal of a central screw from an implanted Tornier Perform Reversed Glenoid Baseplate is not advised, as it may be difficult to unscrew from the baseplate. Introducing a central screw into a baseplate creates a compressive engagement force, which—upon implantation—may make attempts at central screw removal problematic. It is advised to confirm proper screw length **prior** to implantation.


If a central screw change needs to be made, please follow the steps below to properly excise/extract the implant. If the central screw does not disengage from the baseplate, a new baseplate with the correct length central screw will need to be implanted.

**Step 1:** Ensure the baseplate inserter handle [MWJ118] is attached to the baseplate.

- **Do not use the central screw assembly tool [MWJ163]**

**Step 2:** Apply downward pressure with the baseplate inserter screwdriver [MWJ123] and turn the baseplate inserter screwdriver **counter-clockwise** until the screw falls out. | **Figure 32**

- You may encounter some resistance, however continue turning until the screw threads out of the baseplate.

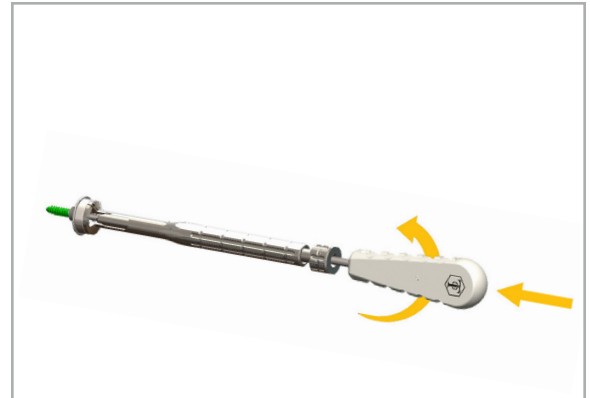

**Figure 32**

## Peripheral screw drilling, insertion, and reaming

The full-wedge augmented baseplates contain one peripheral screw hole which is compression and is located on the thick portion of the wedge. The other peripheral screw holes are multidirectional locking (See appendix for peripheral screw angulations).

The remaining steps for peripheral screw drilling and insertion, peripheral reaming and glenosphere attachment can be performed by following the standard procedure in the Tornier Perform Reversed Glenoid operative technique.


## Press-fit post options

The initial glenoid preparation is the same for the press-fit post option. Please refer to pages 6-8 for cannulated technique. After these steps, perform the following:

### Drilling for press-fit short post

Final drilling of the glenoid central hole is performed under power using the press-fit short post drill to enable a press-fit when impacting the final glenoid baseplate (the baseplate post has a diameter of 11mm).

Attach the press-fit short post drill to power and drill over the guide pin to prepare for the baseplate. Drill until the depth stop contacts the surface of the glenoid bone. | **Figure 33** The press-fit short post drill is designed to drill the hole for the baseplate post and 7mm press-fit post in a single step. A positive stop on the drill bit ensures that drilling will not go too deep and allows for press-fit fixation of the baseplate post. Remove the drill bit.

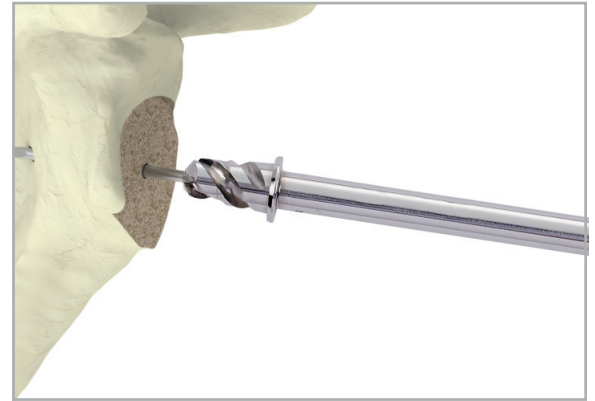


**Figure 33**

### Drilling for press-fit long post

Final drilling of the glenoid central hole is performed under power using the 8mm diameter press-fit post drill to enable a press-fit when impacting the final glenoid baseplate (the baseplate post has a diameter of 11mm).

Attach the 8mm diameter press-fit post drill to power and drill into the prepared hole in the glenoid. Drill until the depth stop contacts the surface of the glenoid bone. | **Figure 33A** The hole for the baseplate post is drilled over the guide pin using the cannulated 10mm diameter drill bit. A positive stop on the drill bit ensures that drilling will not go too deep and allows for press-fit fixation of the baseplate post. Remove the drill bit.

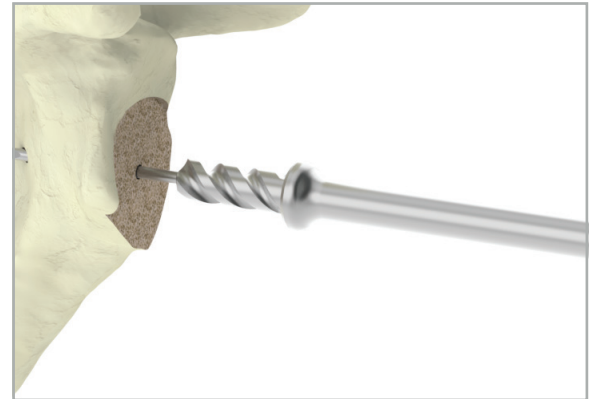

**Figure 33A**

## Baseplate assembly and insertion

The final baseplate is chosen according to the reamed glenoid surface (25mm or 29mm).

The baseplate is then attached to the baseplate inserter in the same manner as on page 12 from above.

The central post is attached by hand to the baseplate by screwing it onto the baseplate in a counterclockwise motion.

| **Figure 34** The post must be securely screwed onto the baseplate. To achieve a secure attachment, insert the baseplate inserter screwdriver down the shaft of the baseplate inserter and engage the head of the post. In a clockwise motion, tighten the post to the baseplate. | **Figure 35**

To insert the assembled baseplate, place the post into the prepared hole and using a mallet gently impact the baseplate into the glenoid until it has fully seated against the surface. Once the baseplate inserter is seated flush on the glenoid surface, the baseplate holder can be detached from the baseplate. The baseplate should be seated completely onto the prepared glenoid surface. Gaps between the baseplate and glenoid surface should be avoided.

The remaining steps for peripheral screw drilling and insertion, peripheral reaming and glenosphere attachment can be performed by following the standard procedure in the Tornier Perform Reversed Glenoid operative technique.



Figure 34


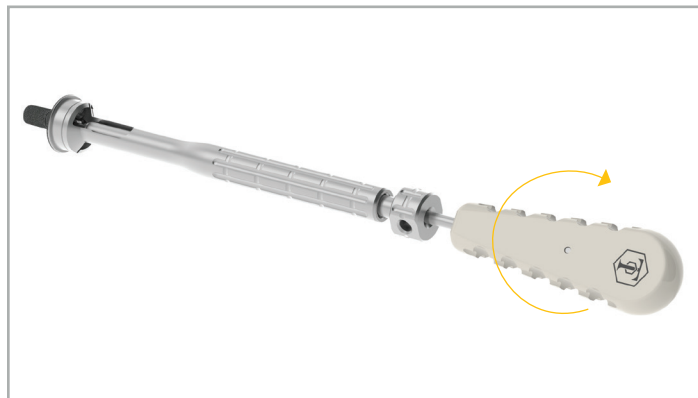


Figure 35

## Appendix

### Augment reamer assembly instructions:

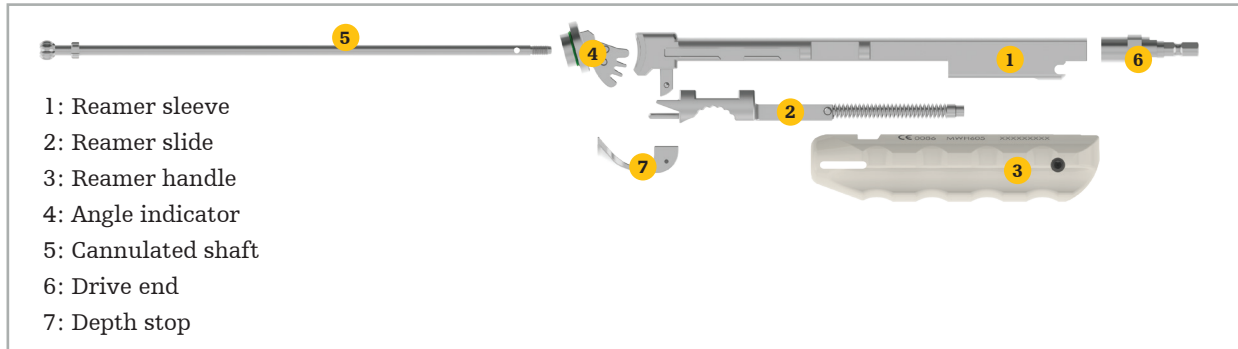

#### Step 1

Connect the reamer sleeve and the reamer slide of the neo reamer making sure the knob on the reamer slide fits into the cut-out section of the reamer sleeve. Once this connection is made, you will pull back on the trigger of the reamer slide until it is attached to the reamer sleeve and the knob of the reamer slide is through the reamer sleeve.

| Figures 36A and 36B

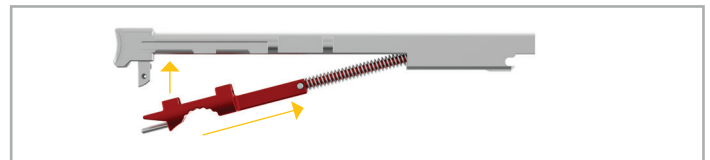


Figure 36A

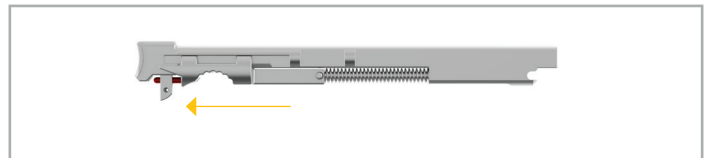


Figure 36B

#### Step 2

Position the pin of the reamer handle into the "U" shaped slot of the reamer sleeve. Fully retract the reamer slide and pivot the reamer handle until it snaps onto the reamer sleeve.

| Figures 37A and 37B

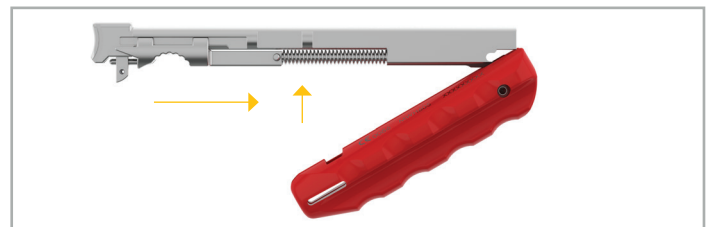


Figure 37A



Figure 37B

#### Step 3

Retract the reamer slide and place the angle indicator into the slot of the reamer sleeve. Set the angle to 15°.

| Figure 38

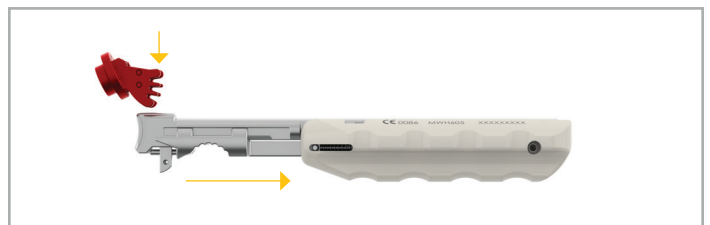


Figure 38

**Step 4**

The cannulated shaft is now placed through the top of the angle indicator and down the reamer sleeve of the assembly. Make sure before progressing to the next step of assembly that the cannulated shaft is flush at the entry point of the angle indicator. Once fully seated, pivot the angle indicator past the 15° position to a neutral position. This will lock the device for the next step. | **Figures 39A and 39B**

**Step 5**

Thread the drive end onto the reamer sleeve ensuring a tight connection. | **Figures 40A and 40B**

**Step 6**

With the angle indicator still in the lock position, thread the reamer head onto the reamer sleeve. Much like the drive end in step five, make sure this connection is tight. | **Figure 41**

**Note:** When using the augment reamer for the full-wedge baseplate the assembly is complete. For the half-wedge baseplate proceed to Step 7.

**Step 7**


Pulling back once again on the trigger noted in previous steps, attach the depth stop to the reamer end of the instrument.

Once these steps are complete, the instrument is assembled properly and can be used for surgery. | **Figures 42A, 42B and 42C**

**Note:** Disassembly instructions are the reverse of the assembly instructions.



**Figure 39A**

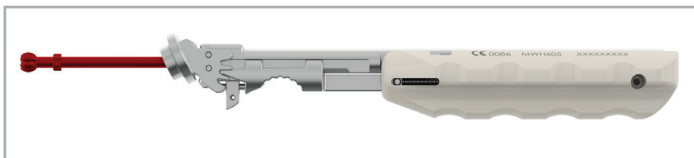

**Figure 39B**



**Figure 40A**



**Figure 40B**



**Figure 41**



**Figure 42A**



**Figure 42B**



**Figure 42C**



## Tornier Perform Reversed Glenoid Glenosphere and Baseplate configuration chart

The Tornier Perform Reversed Glenoid Baseplates have been designed to be compatible with the Tornier Perform Reversed Glenoid Glenospheres. With the addition of the Adaptis Integrated Porous Metal on the backside of the baseplate, certain combinations may have the potential to create an impingement with the humeral insert. For more information on the cleared combinations, refer to the configuration chart below. The boxes highlighted in green indicate that there should be no impingement of the poly insert on the humeral side with the porous titanium on the baseplate.


Glenosphere		Baseplate									
		Standard		Lateralized				Wedge augment			
		25mm	29mm	25mm (+3)	29mm (+3)	25mm (+6)	29mm (+6)	25mm half-wedge	29mm half-wedge	25mm full-wedge	29mm full-wedge
Standard	36mm										
	39mm										
	42mm										
Eccentric	36mm +2 ECC										
	39mm +3 ECC										
	42mm +4 ECC										
Lateralized	33mm +3 LAT										
	36mm +3 LAT										
	39mm +3 LAT										
	42mm +3 LAT										

 Cleared combination  Not cleared combination

## Tornier Perform Reversed Glenoid Peripheral Screw angulation


Baseplate	Multidirectional locking screws		Compression screws	
	Superior - inferior	Transverse	Superior - Inferior	Transverse
Half-wedge baseplates	0-25°	±10°	0°	3°
Full-wedge baseplates	0-25°	±6°	0°	3°

## Product specifications

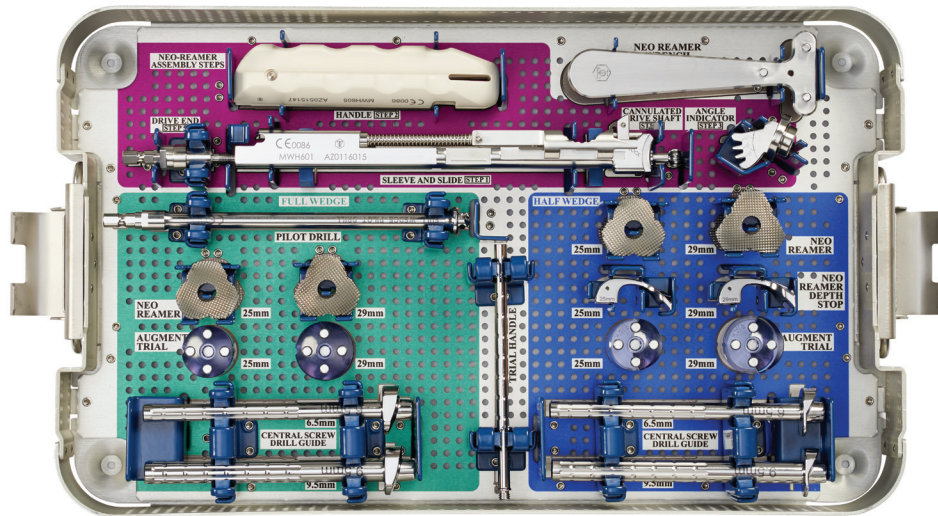


### Tornier Perform Reversed Augmented Glenoid instrument tray (Ref. YKAD264)

Reference	Description	Reference	Description
MWH601	Neo reamer sleeve	MWJ168	Neo reamer depth stop half-wedge, 25mm
MWH602	Neo reamer cannulated drive shaft	MWJ169	Neo reamer depth stop half-wedge, 29mm
MWH603	Neo reamer angle indicator	MWJ145	Augment half-wedge trial, 25mm
MWH604	Neo reamer drive end	MWJ146	Augment half-wedge trial, 29mm
MWH605	Neo reamer handle	MWJ147	Augment full-wedge trial, 25mm
MWH606	Neo reamer slide	MWJ148	Augment full-wedge trial, 29mm
MWH607	Neo reamer wrench	MWJ151	Central screw drill guide half-wedge, 6.5mm
MWH630	Trial handle	MWJ152	Central screw drill guide half-wedge, 9.5mm
MWJ170	Neo reamer, 25mm full-wedge	MWJ153	Central screw drill guide full-wedge, 6.5mm
MWJ171	Neo reamer, 29mm full-wedge	MWJ154	Central screw drill guide full-wedge, 9.5mm
MWJ172	Neo reamer, 25mm half-wedge	MWJ129	Full-wedge pilot drill, 8mm
MWJ173	Neo reamer, 29mm half-wedge		

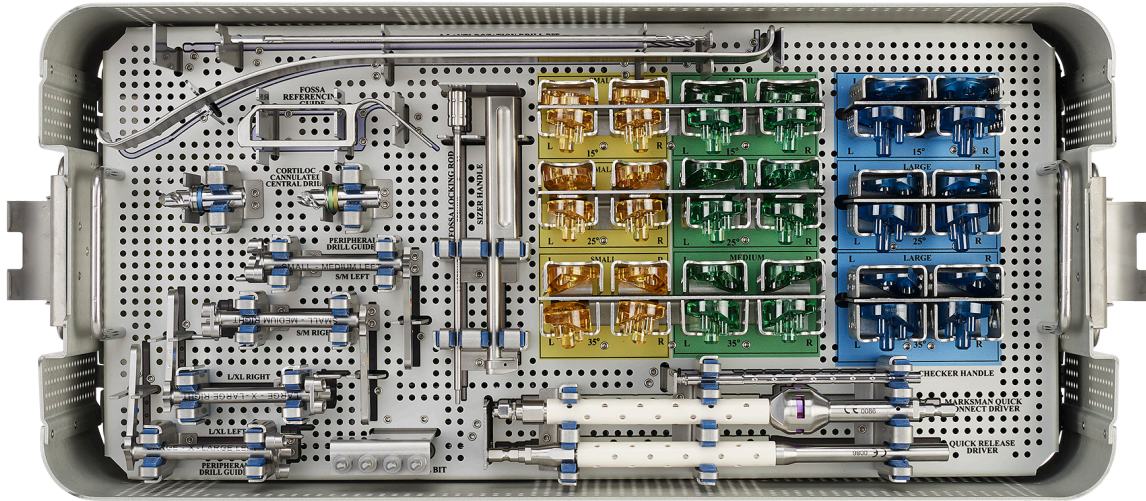

**Tornier Perform Anatomic Augmented Glenoid – Express tray – upper level YKAD10518**

Reference	Description	Reference	Description
MWH631	2.5mm anti-rotation drill bit	MWH643	Medium Cortiloc/Pegged checker LA35
MWH623	Posterior retractor	MWH644	Large Cortiloc/Pegged checker RA15
MWH629S	Fossa referencing guide	MWH645	Large Cortiloc/Pegged checker LA15
MWE202	Cortiloc cannulated central drill bit – small/medium	MWH646	Large Cortiloc/Pegged checker RA25
MWE203	Cortiloc cannulated central drill bit – large/extra large	MWH647	Large Cortiloc/Pegged checker LA25
MWH680S	Cortiloc fasttrack peripheral drill guide S/M left	MWH648	Large Cortiloc/Pegged checker RA35
MWH681S	Cortiloc fasttrack peripheral drill guide S/M right	MWH649	Large Cortiloc/Pegged checker LA35
MWH682S	Cortiloc fasttrack peripheral drill guide L/XL left	MWH691S	Small Cortiloc glenoid trial RA15
MWH683S	Cortiloc fasttrack peripheral drill guide L/XL right	MWH692S	Small Cortiloc glenoid trial LA15
MWH685	Cortiloc/pegged quick release drill bit	MWH693S	Small Cortiloc glenoid trial RA25
MWH622	Fossa locking rod	MWH694S	Small Cortiloc glenoid trial LA25
MWH628S	Sizer handle	MWH695S	Small Cortiloc glenoid trial RA35
MWH620	Marksman quick connect driver	MWH696S	Small Cortiloc glenoid trial LA35
MWH686	Quick release driver	MWH697S	Medium Cortiloc glenoid trial RA15
MWH630	Checker handle	MWH698S	Medium Cortiloc glenoid trial LA15
MWH632	Small Cortiloc/Pegged checker RA15	MWH699S	Medium Cortiloc glenoid trial RA25
MWH633	Small Cortiloc/Pegged checker LA15	MWH700S	Medium Cortiloc glenoid trial LA25
MWH634	Small Cortiloc/Pegged checker RA25	MWH701S	Medium Cortiloc glenoid trial RA35
MWH635	Small Cortiloc/Pegged checker LA25	MWH702S	Medium Cortiloc glenoid trial LA35
MWH636	Small Cortiloc/Pegged checker RA35	MWH703S	Large Cortiloc glenoid trial RA15
MWH637	Small Cortiloc/Pegged checker LA35	MWH704S	Large Cortiloc glenoid trial LA15
MWH638	Medium Cortiloc/Pegged checker RA15	MWH705S	Large Cortiloc glenoid trial RA25
MWH639	Medium Cortiloc/Pegged checker LA15	MWH706S	Large Cortiloc glenoid trial LA25
MWH640	Medium Cortiloc/Pegged checker RA25	MWH707S	Large Cortiloc glenoid trial RA35
MWH641	Medium Cortiloc/Pegged checker LA25	MWH708S	Large Cortiloc glenoid trial LA35
MWH642	Medium Cortiloc/Pegged checker RA35		

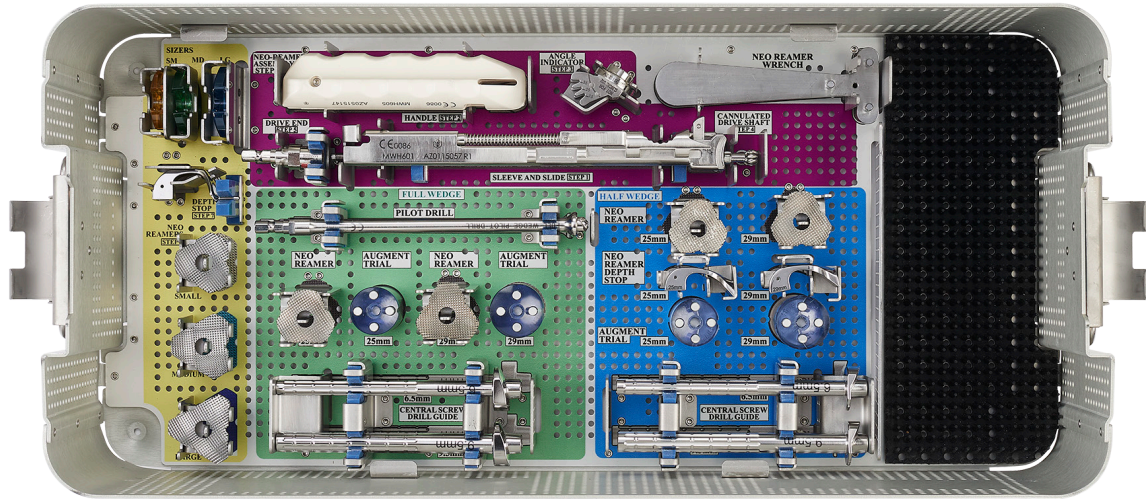

**Tornier Perform Anatomic Augmented Glenoid – Express tray – lower level YKAD10518**

Reference	Description	Reference	Description
MWH624S	Small sizer	MWJ170	Neo reamer, 25mm full-wedge
MWH625S	Medium sizer	MWJ171	Neo reamer, 29mm full-wedge
MWH626S	Large sizer	MWJ147	Augment full-wedge trial, 25mm
MWH608	Neo reamer datum foot	MWJ148	Augment full-wedge trial, 29mm
MWH610	Small neo reamer	MWJ153	Central screw drill guide full-wedge, 6.5mm
MWH611	Medium neo reamer	MWJ154	Central screw drill guide full-wedge, 9.5mm
MWH612	Large neo reamer	MWJ172	Neo reamer, 25mm half-wedge
MWH605	Neo reamer handle	MWJ173	Neo reamer, 29mm half-wedge
MWH603	Neo reamer angle indicator	MWJ145	Augment half-wedge trial, 25mm
MWH607	Neo reamer wrench	MWJ146	Augment half-wedge trial, 29mm
MWH604	Neo reamer drive end	MWJ151	Central screw drill guide half-wedge, 6.5mm
MWH601	Neo reamer sleeve	MWJ152	Central screw drill guide half-wedge, 9.5mm
MWH606	Neo reamer slide	MWJ168	Neo reamer depth stop half-wedge, 25mm
MWH602	Neo reamer cannulated drive shaft	MWJ169	Neo reamer depth stop half-wedge, 29mm
MWJ129	Full-wedge pilot drill, 8mm		

**Wedged augmented baseplates**

Reference	Description
DWJ514	Half-wedge augment baseplate (35°), 29mm
DWJ504	Half-wedge augment baseplate (35°), 25mm
DWJ515	Full-wedge augment baseplate (15°), 29mm
DWJ505	Full-wedge augment baseplate (15°), 25mm



**Press-fit posts**

Reference	Description
DWJ002	Press-fit long post, 15mm
DWJ001	Press-fit short post, 7mm







This document is intended solely for the use of healthcare professionals. A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate a Stryker product. A surgeon must always refer to the package insert, product label and/or instructions for use, including the instructions for cleaning and sterilization (if applicable), before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

The instructions for use, operative techniques, cleaning instructions, patient information leaflets and other associated labeling may be requested online at [ifu.stryker.com](http://ifu.stryker.com) or [wright.com](http://wright.com). If saving the instructions for use, operative techniques, cleaning instructions from the above mentioned websites, please make sure you always have the most up to date version prior to use.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Aequalis, Perform, Stryker, Tornier. All other trademarks are trademarks of their respective owners or holders.

Content ID: AP-010186E 23-Nov-2021

Copyright © 2021 Stryker

**Manufacturer:**

Tornier, Inc.  
10801 Nesbitt Avenue South  
Bloomington, MN 55437  
t: 888 867 6437  
t: 952 426 7600

[stryker.com](http://stryker.com)